



VistA Blood Establishment Computer Software (VBECS) - ProVue Interface Configuration and Setup Guide, Version 6.0

December 2022

Department of Veterans Affairs
Enterprise Project Management Office

Table of Contents

TABLE OF CONTENTS	2
INTRODUCTION	3
RELATED MANUALS AND REFERENCE MATERIALS	4
SET UP AUTOMATED INSTRUMENT	5
1 Network Connectivity Setup	5
2 Testing Template Setup	5
3 User ID Setup	5
SET UP INSTRUMENT MANAGER	7
1 Instrument Manager Version	7
2 Instrument Manager to Automated Instrument Connectivity	7
3 Installing Instrument Driver	8
4 Set Up Instrument Manager Configuration	8
4.1 Download Instrument Configuration Files	9
4.2 Import Instrument Side Configuration	11
4.3 Modify Test Code Mapping for Instrument Side Configuration (only for sites that perform 2 Cell Antibody Screen)	14
4.4 Verify Test Code Mapping for Instrument Side Configuration	16
4.5 Verify Rules	16
4.6 Configure Rules	17
4.7 Import VBECS (HL7) Side Configuration	20
4.8 Modify Test Code Mapping for VBECS Side Configuration (only for sites that perform 2 Cell Antibody Screen)	23
4.9 Verify Test Code Mapping for VBECS Side Configuration	25
5 Set Up HL7 Connection to VBECS TEST	26
6 Set Up Instrument Connection	28
7 Test New Connections	30
8 Validate Instrument Connectivity to VBECS TEST	31
9 Set up HL7 Connection to VBECS PROD	31
GLOSSARY	35
APPENDICES	36
APPENDIX A: INSTRUMENT SIDE MAPPING	36
APPENDIX B: HL7 (VBECS) SIDE MAPPING	37
APPENDIX C: RULES	38
APPENDIX D: TESTING TEMPLATES	40
APPENDIX E: VALIDATION PLANNING AND EXAMPLE TEST SCENARIOS	42
TEST GROUP ONE: AI INTERFACE DISABLED	43
TEST GROUP TWO: VERIFY AI INDIVIDUAL TEST(S)	44
TEST GROUP 2 SCENARIO 1: VERIFY AI TAS TEST	45
TEST GROUP 2 SCENARIO 2: VERIFY AI SEROLOGIC CROSSMATCH TEST	46
TEST GROUP 2 SCENARIO 3: VERIFY AI PATIENT DIAGNOSTIC TESTS	47
TEST GROUP 2 SCENARIO 4: VERIFY AI BLOOD UNIT TESTS	48
REVISION HISTORY	49

Introduction

VBECS is a Blood Bank application that facilitates ongoing compliance with Food and Drug Administration (FDA) standards for medical devices and enhances the VA's ability to produce high-quality blood products and services to veterans. The system follows blood bank standards, standards of national accrediting agencies, FDA regulations and VA policies.

VBECS 2.2.0 introduced a new interface for blood bank testing performed by blood bank instrumentation to VBECS. The implementation of the interface and its associated validation are described in this guide.



Unauthorized access or misuse of this system and/or its data is a federal crime. Use of all data must be in accordance with VA security and privacy policies.



The U.S. FDA classifies this software as a medical device. Unauthorized modifications will render this device an adulterated medical device under Section 501 of the Medical Device Amendments to the Federal Food, Drug, and Cosmetic Act. Acquiring and implementing this software through the Freedom of Information Act requires the implementer to assume total responsibility for the software and become a registered manufacturer of a medical device, subject to FDA regulations. Adding to or updating VBECS software without permission is prohibited.




Instructions in this Setup Guide must be followed for the interface to deliver information to VBECS. Local validation is required to confirm proper operation before use. Validation and verification is required to ensure connectivity to VBECS.

This guide is provided to assist you with the multi-faceted required setup of your local blood bank testing instrument(s), Data Innovations Instrument Manager (DI IM) and VBECS to electronically transmit instrument test results to VBECS for use in the transfusion service.

This guide includes download and install of a set of configuration files (.gsb files) as a starting point for your instrument setup. These configuration files are not created and validated by the VBECS team but are shared by other VBECS sites already using this instrument and are made available for installation from the VBECS SharePoint site. As noted above, local validation is required to confirm proper operations with the configurations implemented by your site.

There are specific setup requirements to test and transmit those testing results to VBECS for review using DI IM (Figure 1).

Figure 1: Hardware and Interface Configuration

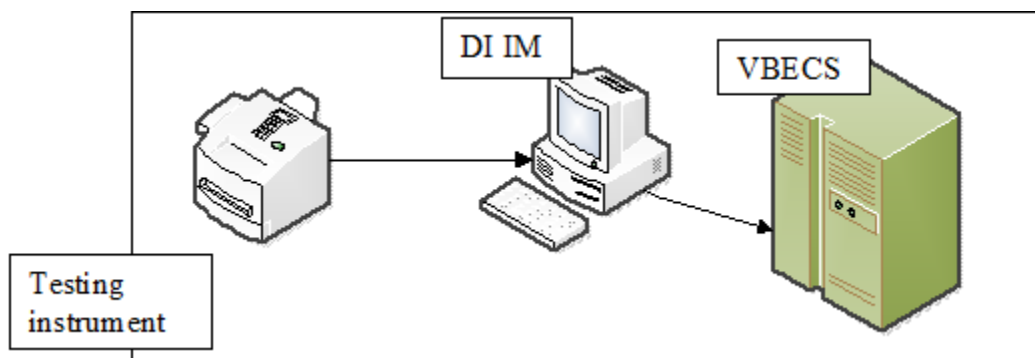


Your local testing instrument(s) communicates with DI IM via an instrument specific driver provided by DI that must be downloaded from DI and installed locally.

DI IM communicates directly with VBECS via a generic HL7 interface driver that must be downloaded from DI and installed, locally. This driver is customized for VBECS by downloading and installing the driver configuration file.

VBECS has an interface that must be configured in VBECS Administrator to receive messages from DI IM (Figure 2).

Figure 2: Setup Path

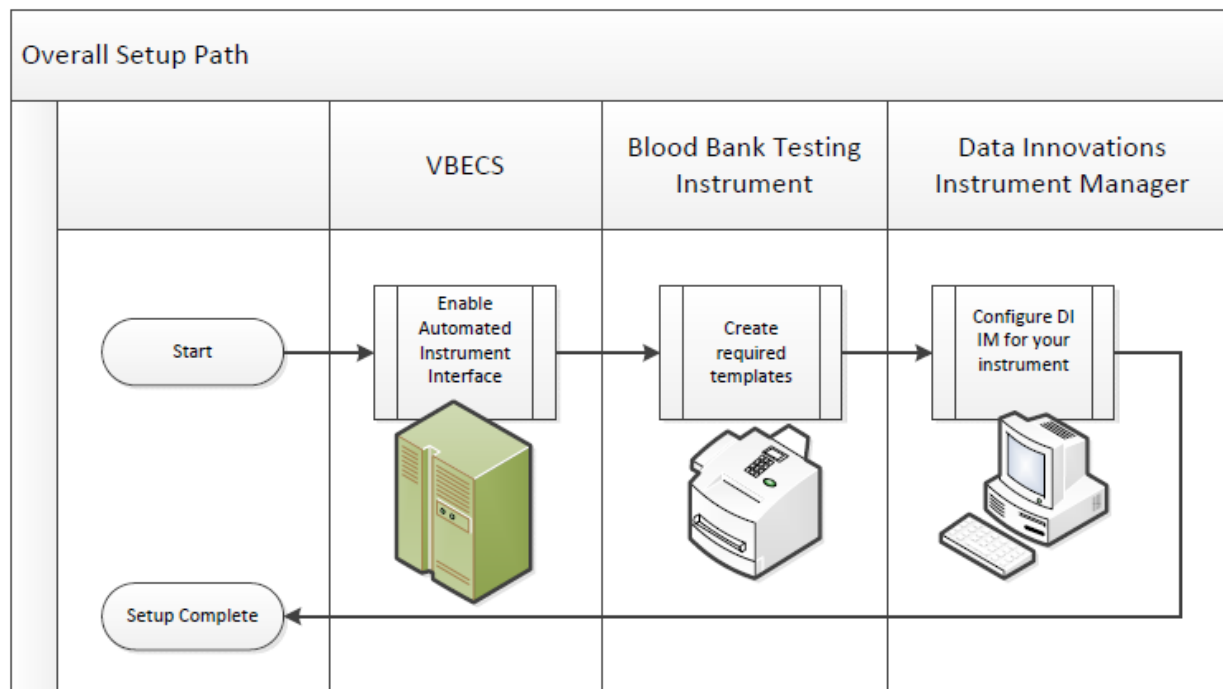

Related *Manuals and Reference Materials*

- *VBECS Technical Manual-Security Guide*
- *Data Innovations Instrument Manager Manual*
- *Blood Bank Analyzer User's Guide (Instrument Manual)*

Set Up Automated Instrument

1 Network Connectivity Setup

In order to ensure proper functioning of an interface between an Automated Instrument and Instrument Manager, the Instrument needs to be connected to the VA network. The static IP Address and Port number has to be assigned to the Instrument (further referred in this document as **<Instrument IP>** and **<Instrument Port>**). Please refer to the Instrument Manual or contact your vendor for the instructions about how to perform this setup. Check with Local IT staff to establish the connection to Instrument Manager.

2 Testing Template Setup



If testing templates are not configured properly it will be impossible for instrument results to be accepted in VBECS.

Please refer to **Appendix D: Testing Templates** for the list of instrument templates. These test templates must be set up on your Instrument before any results can be sent to VBECS. Always use them for results that are sent to VBECS. Any results sent that do not follow the above templates will be rejected. Please refer to the Instrument Manual or contact your vendor for the instructions about how to perform template setup.



*Please note that **Appendix D: Testing Templates** contains definition of two Crossmatch templates:*

VBECS Crossmatch.pln - template for blood banks that perform IS and IAT crossmatch together

VBECS Crossmatch IAT.pln- template for blood banks that perform IAT only crossmatch (no Immediate Spin)

These two templates are mutually exclusive so please select only one that is appropriate for your site.



AABB Standards require a serological XM to detect ABO incompatibility (5.16.1) and a local policy must be in place if the site is not performing an IS AHG as part of their serologic crossmatch test, manual or using an instrument.

3 User ID Setup



Failure to set up proper user IDs on an instrument will prevent instrument results from being accepted in VBECS.

In order for VBECS to properly recognize the person who performed testing on an Instrument, all users accessing the instrument must have their user IDs set up to match their network user IDs (e.g., **REDACTED**).

Due to the constraints of the ProVue messaging system, only 10 characters of the user name can be transferred with test results to VBECS. To ensure the uniqueness of all user names, please strip the front characters of network user ID to ensure that it is no longer than 10 characters long. For user IDs that are already 10 characters or shorter you can use full user ID in ProVue. For example:

Length of User ID	Example of User ID	ProVue ID
12	VHASFCYOUNGS	SFCYOUNGS
11	VHASFCTRANK	HASFCTRANK
10 or less	VHASFCTSTU	VHASFCTSTU

Please refer to the Instrument Manual or contact your vendor for the instructions about how to perform setup of user IDs.

Set Up Instrument Manager

1 Instrument Manager Version

Please verify that you are using Instrument Manager Version 8.13 or greater (Figure 3). In Instrument Manager navigate to **Help -> About Instrument Manager**.

Figure 3: Instrument Manager Version Screen



*If your version of Instrument Manager is older than 8.13 **STOP** executing this guide and update the software first. **Do not proceed until the issue is resolved.***



If your Instrument Manager is greater than 8.13 you may proceed. The user must execute the instructions and validate functionality on newer versions. Discrepancies in the instructions must be reported as a service desk ticket.



*Instrument Manager must have the **Specimen Management Module** licensed and activated. **Do not proceed until this issue is resolved.***

2 Instrument Manager to Automated Instrument Connectivity

Please contact your local network administrative staff and ensure that your local network allows two-way TCP/IP connectivity between **<Instrument Manager IP>** address and **<Instrument IP>** address on **<Instrument Port>**.

3 Installing Instrument Driver

Two drivers are required for the correct operation of the Automated Instrument interface to VBECS (Figure 4).

Figure 4: Required Drivers

Driver	Name	Version	Description
Diamed Diana	diadiani	8.00.0009	Instrument to DI IM driver
Data Innovations LLC, Configurable HL7	diihl7ml	8.00.0044	DI IM to VBECS (HL7) driver

Please refer to the user's manual for Instrument Manager or contact Data Innovations for the instructions on how to install drivers for Instrument Manager.

After successful installation of drivers go to **Report -> Available Drivers** menu option in Instrument Manager and verify that the drivers listed in Figure 4 are present.

4 Set Up Instrument Manager Configuration



Execute instructions in this section for each instrument that will be connecting to VBECS.



Modifying rules or test code mappings as imported using this Instrument Manager configuration Setup Guide may lead to malfunction of the Automated Instrument to VBECS interface.

Prerequisites for the Instrument Manager Configuration files download:

- You must be an administrator on <***Instrument Manager Server***>.
- Once the above prerequisite is met you may proceed.


4.1 Download Instrument Configuration Files



Use local procedures for copying the instrument configuration files to the Instrument Manager server.


1. Navigate to **REDACTED** (Figure 5).

Figure 5: Connecting to the VBECS SharePoint

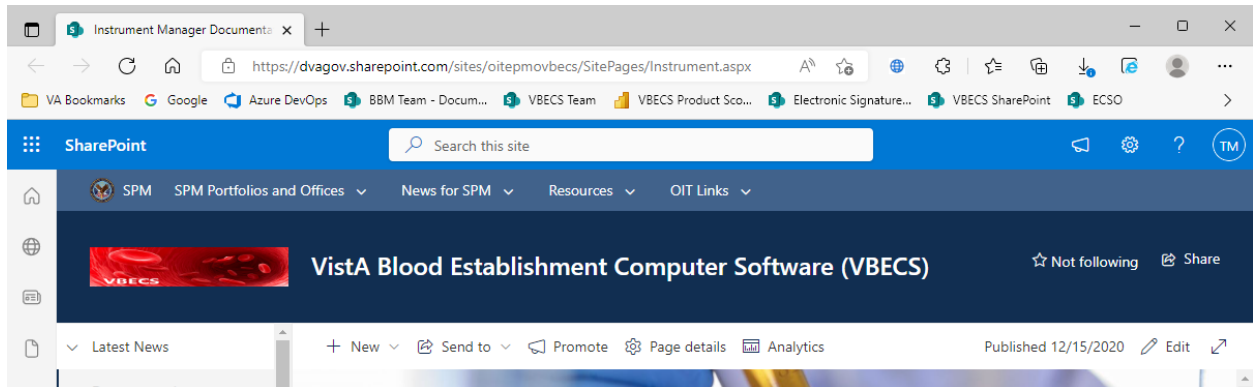


2. To download a file from SharePoint, click the ellipses (...) and select **Download** (Figure 6).

Figure 6: Example of SharePoint Download

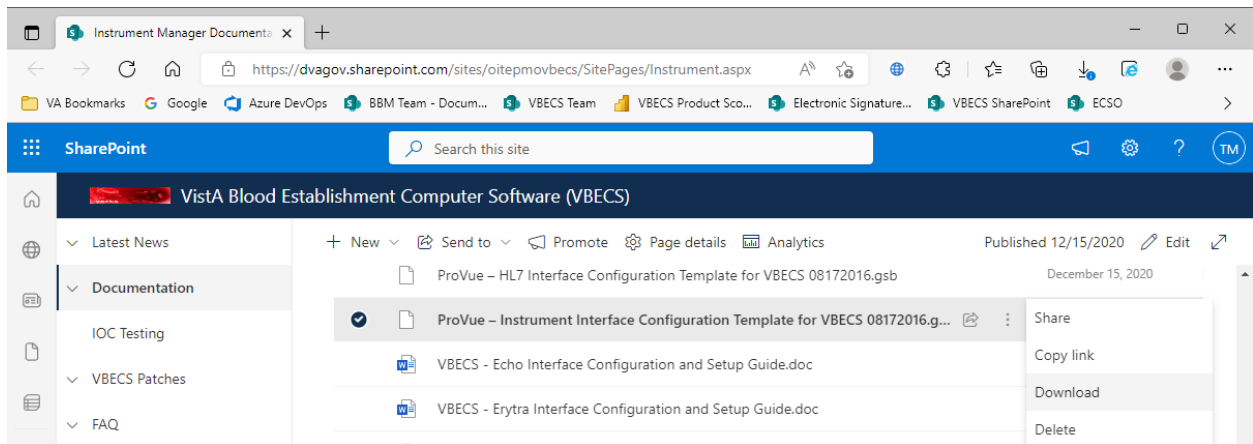


3. In the Downloads screen, select **Save as** (Figure 7).

Figure 7: Example of Downloads

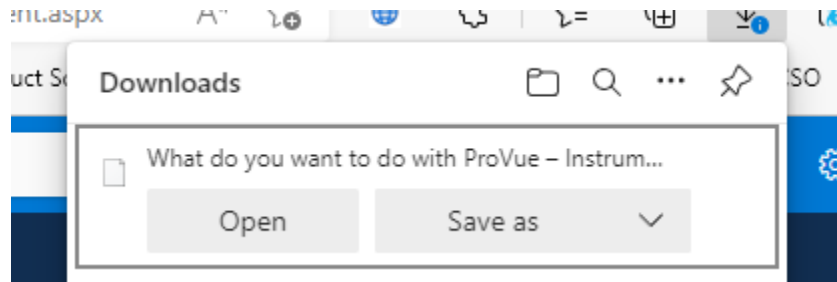

4. In the next screen, specify the directory to save (Figure 8).

Figure 8: Example of Save As

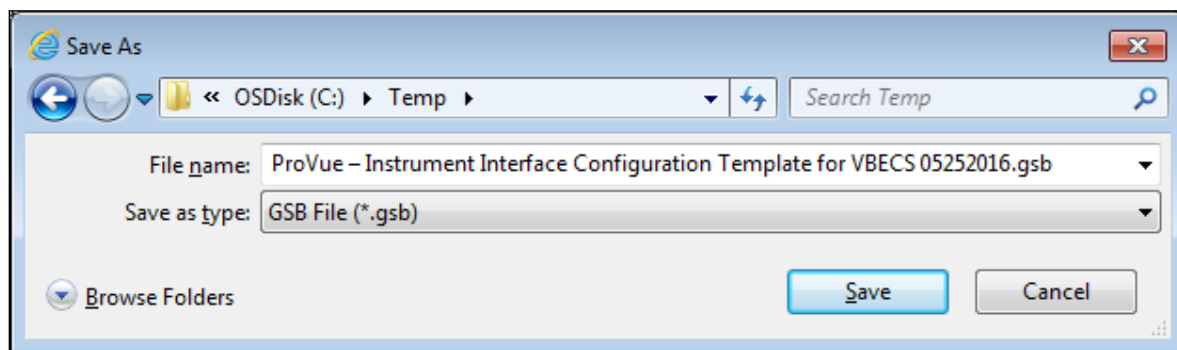

5. Save both the **ProVue - HL7 Interface Configuration Template for VBECS 08172016.gsb** and the **ProVue - Instrument Interface Configuration Template for VBECS 08172016.gsb** files to the **C:\temp** directory.
6. Per local procedures copy both files to C:\temp on the Instrument Manager server
7. On the Instrument Manager server; Click **Start**, and in the “Search programs and files” box type **Run** and hit enter. Type **powershell** and click **OK** to launch PowerShell
8. Copy and paste or type the following commands to generate checksums for configuration files:

```
certutil -hashfile "C:\Temp\ProVue - HL7 Interface Configuration Template for VBECS 08172016.gsb" MD5 <press Enter>
```

```
certutil -hashfile "C:\Temp\ProVue - Instrument Interface Configuration Template for VBECS 08172016.gsb" MD5 <press Enter>
```




To copy, highlight the lines in grey and enter CTRL-C.

To insert the copied line into a PowerShell window, right click in the PowerShell window and select “Paste”.

9. Verify that checksums for both files match those shown in **Figure 9**.

Figure 9: Instrument Manager Configuration File Checksums

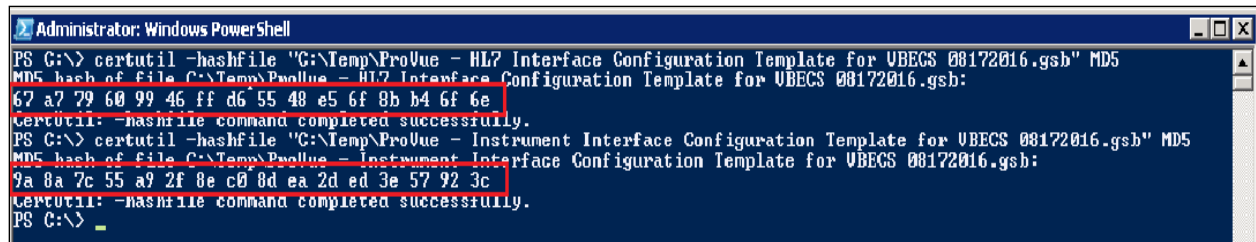

```
PS C:\> certutil -hashfile "C:\Temp\ProVue - HL7 Interface Configuration Template for VBECS 00172016.gsb" MD5
MD5 hash of file C:\Temp\ProVue - HL7 Interface Configuration Template for VBECS 00172016.gsb:
67 a7 79 60 99 46 ff d6 55 48 e5 6f 8b b4 6f 6e
certutil: -hashfile command completed successfully.
PS C:\> certutil -hashfile "C:\Temp\ProVue - Instrument Interface Configuration Template for VBECS 00172016.gsb" MD5
MD5 hash of file C:\Temp\ProVue - Instrument Interface Configuration Template for VBECS 00172016.gsb:
9a 8a 7c 55 a9 2f 8e c0 0d ea 2d ed 3e 57 92 3c
certutil: -hashfile command completed successfully.
PS C:\>
```




If the checksums do not match, stop and file a service desk ticket to coordinate assistance with installation. Do not proceed until the issue is resolved.

10. Close the **PowerShell** window.

4.2 Import Instrument Side Configuration


1. After downloading and verifying configuration files, open Instrument Manager and navigate to **Configuration -> Configuration Editor**.
2. Click on the **Import** button (Figure 10).

Figure 10: Example of Configuration Editor

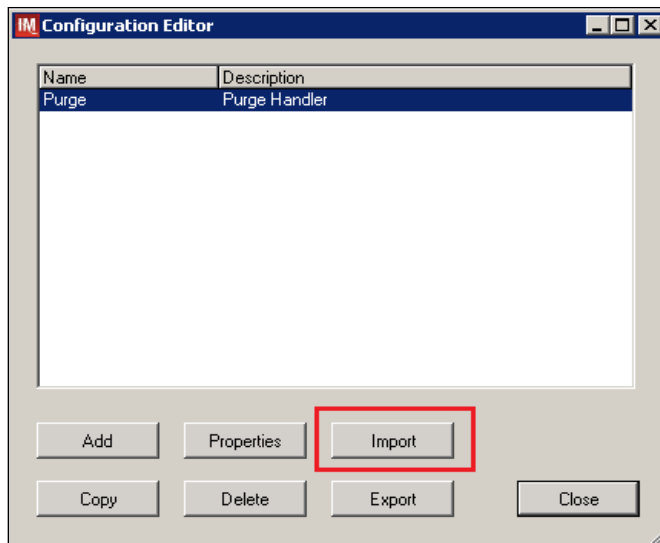

3. Once the import screen opens, click the **Browse** button and select **C:\Temp** folder (Figure 11).
4. Click OK.

Figure 11: Example of Configuration Import Screen

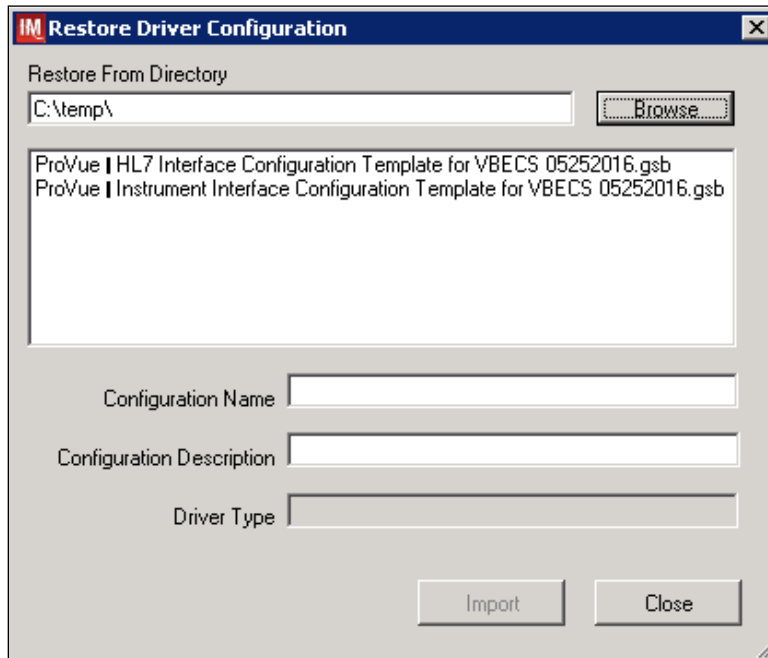


5. Select **ProVue - Instrument Interface Configuration Template for VBECS 08172016.gsb** file from the list.
6. Enter **Configuration Name** that contains 3 letter location code of the instrument (e.g., **HIN** for Hines VAMC), word **ProVue** and sequence number (**1** for the first instrument, **2** for the second etc.). Example Configuration Name for instrument configuration located at Hines would be:

HIN_ProVue_1

This configuration name will be further referred in this document as *<Instrument Side Configuration>*.

7. Enter **Configuration Description** and click **Import** button. Verify that the following confirmation window displays (Figure 12).


Figure 12: Example of Successful Configuration Import

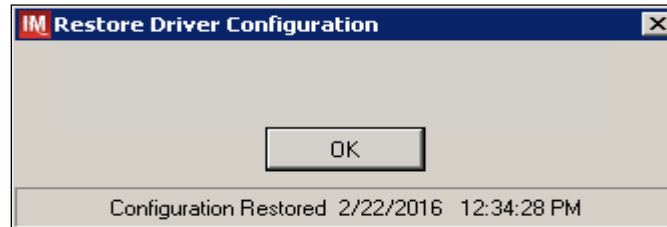

If you are using a newer version of the driver than the one mentioned in section 3, the Instrument Manager will warn you about the discrepancy in driver versions. Please acknowledge this warning and continue.

8. Click **OK** and then close the **Restore Driver Configuration** window.
9. Verify that **Configuration Editor** shows the new configuration on the list (Figure 13).

Figure 13: Example of Configuration Editor Window Showing Newly Imported Configuration

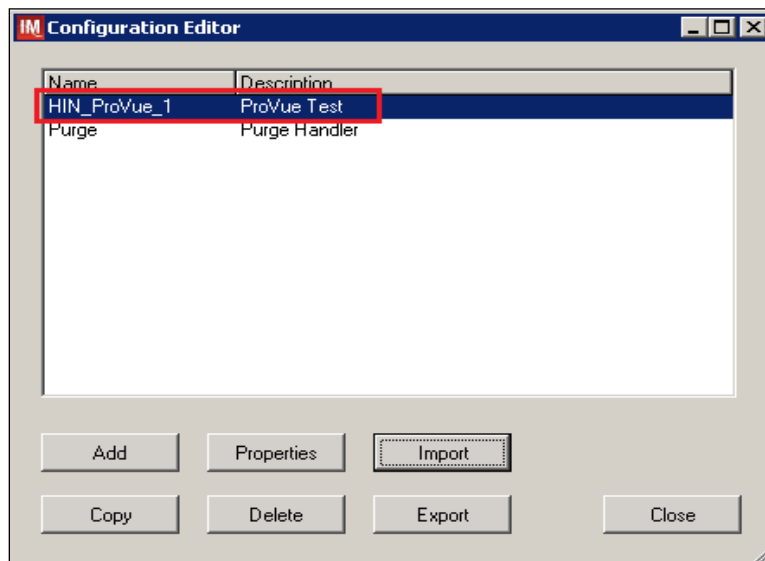

10. **Close** the Configuration Editor Window


4.3 Modify Test Code Mapping for Instrument Side Configuration (only for sites that perform 2 Cell Antibody Screen)



Execute instructions in this section only if you perform Antibody Screen with 2 Screening Cells instead of 3.


1. In Instrument Manager navigate to **Configuration -> Configuration Editor**.
2. Click on the **Properties** button (Figure 14)

Figure 14: Example of Configuration Editor Window

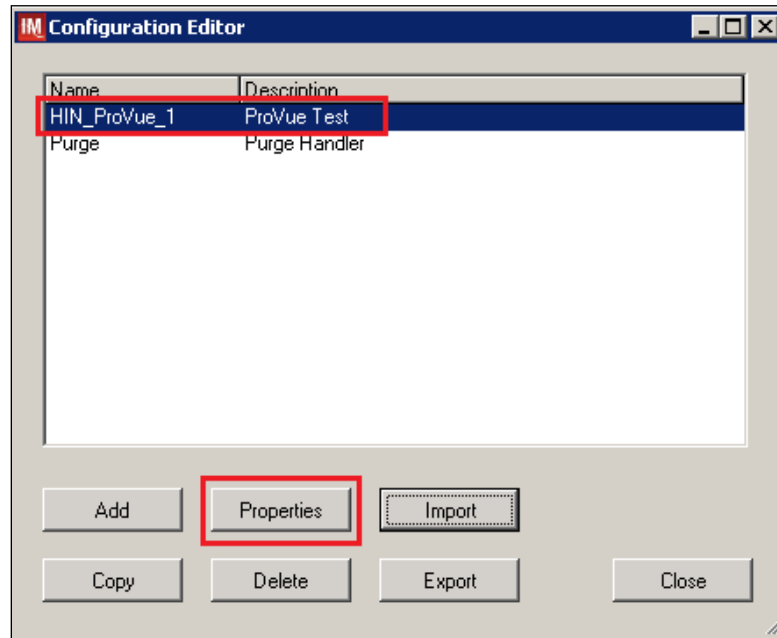


3. On the **Configuration Properties** window click on **Test Code Map** (Figure 15)

Figure 15: Example of Configuration Properties Window

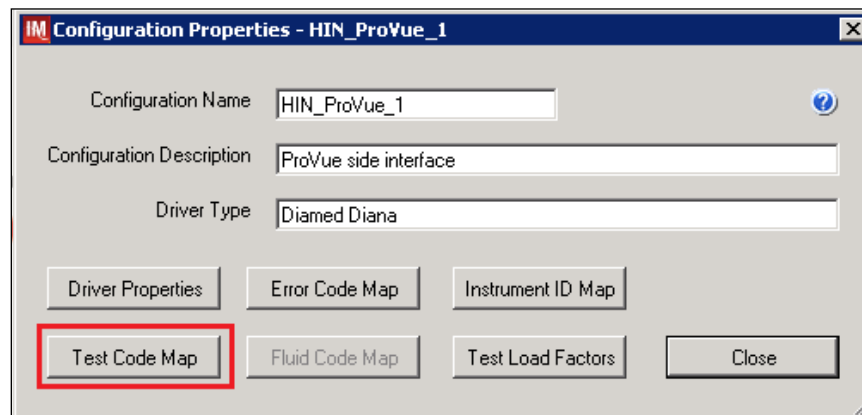


- Using the dropdown in the upper left corner of the **Test Code Mapping** window select test code **^VBECS ABS.pln^3 (SC3)** and click **Delete** button (Figure 16)

Figure 16: Example of deletion of SC3 test code for Antibody Screen

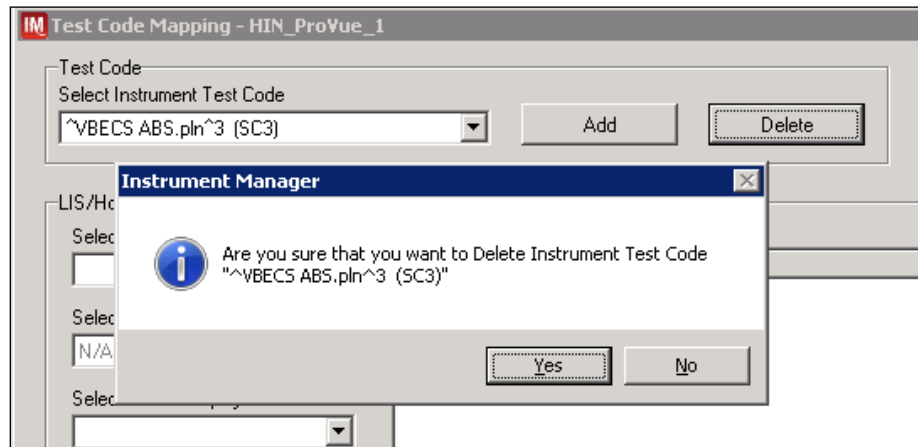

- Click **Yes** to confirm deletion
- Using the dropdown in the upper left corner of the **Test Code Mapping** window select test code **^VBECS TAS.pln^11 (SC3)** and click **Delete** button (Figure 17)

Figure 17: Example of deletion of SC3 test code for Type and Screen

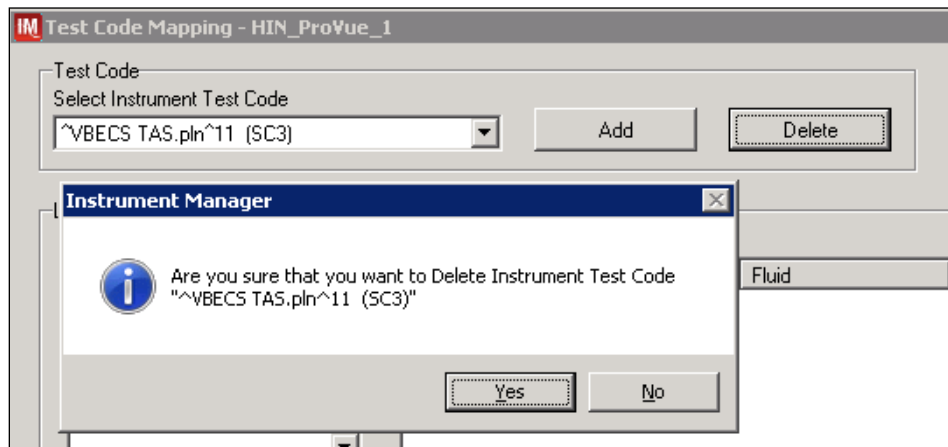

- Click **Yes** to confirm deletion
- Click **Close** on **Test Code Mapping** window
- Click **Close** on **Configuration Properties** window
- Click **Close** on **Configuration Editor**

4.4 Verify Test Code Mapping for Instrument Side Configuration

1. Navigate to **Reports -> Configuration Options and Mappings**.
2. Select the **<Instrument Side Configuration Name>** from the pull down menu (Figure 18)
3. Select the **Results Test Code Mapping Tab** (Figure 18).
4. Verify the **Results Test Code Mapping Report Tab** matches the list in *Appendix A: Instrument Side Mapping*.

Figure 18: Example of Results Test Code Mapping Tab

REDACTED



If mismatches in Test Codes names, or missing or extra Test Codes are encountered, file a service desk ticket to coordinate assistance with installation. Do not proceed until the issue is resolved.

5. Close **Configuration Options and Mappings** window.

4.5 Verify Rules

1. Next navigate to **Reports -> Configuration Options and Mappings**.
2. Select the **<Instrument Side Configuration Name>** from the drop-down in the upper left corner. (Figure 19).
3. Select the **Rules Tab** (Figure 18).
4. Verify the **Rules Tab** matches all rules listed in *Appendix C: Rules*.

Figure 19: Example of Rules Tab

REDACTED

5. Close the **Configuration Options and Mappings** window.




If mismatches in rules are encountered, file a service desk ticket to coordinate assistance with installation. Do not proceed until the issue is resolved.


4.6 Configure Rules

In this section you will establish the name of the instrument associated with test results for VBECS reports. *If you are setting up multiple instruments, make sure that each has a unique name.*

1. Navigate to Configuration -> Specimen Management Configuration -> Rules Processing.
2. Select **<Instrument Side Configuration Name>** from the pull down menu (Figure 20).
3. Locate rule **SetInstrumentID**.
4. Click on the **Then** line in the rule.
5. Modify the rule by typing **<Instrument Name>** between the quotation marks in the lower box (Figure 20).

Figure 20: Example of Instrument Name Setup

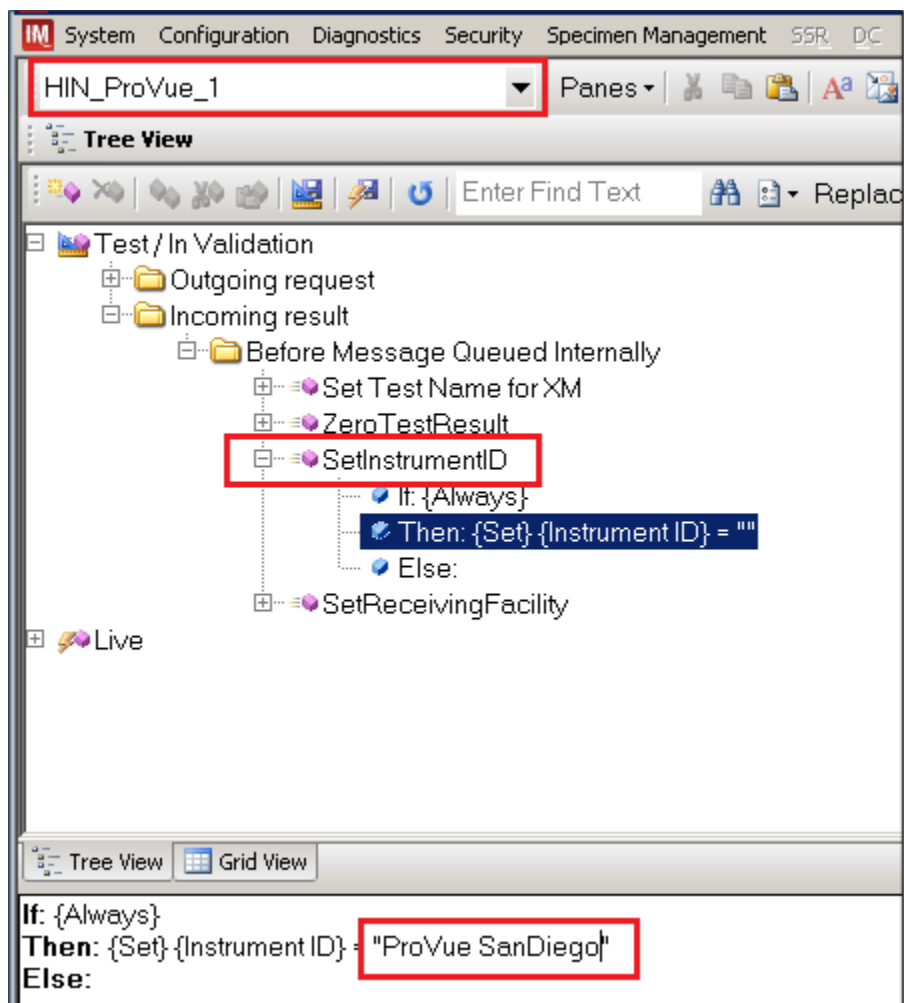

-  **<Division Code>** also known as Station Number in VistA is a unique alphanumeric code that is associated with each hospital (e.g. **589** for VA Heartland West VAMC). This code must match the division code configured in **VBECS Administrator** application for a given site.


The screenshot shows the HIN_ProVue_1 configuration window. The Tree View on the left displays the test logic structure:

- Test/ In Validation
 - Outgoing request
 - Incoming result
 - Before Message Queued Internally
 - Set Test Name for XM
 - ZeroTestResult
 - SetInstrumentID
 - If: {Always}
 - Then: {Set} {Instrument ID} = "ProVue SanDiego"
 - Else:
 - SetReceivingFacility (highlighted with a red box)
 - If: {Always}
 - Then: {Set} {Receiving Facility} = ""
 - Else:

The bottom pane shows the configuration for the selected step, SetReceivingFacility:


```
If: {Always}
Then: {Set} {Receiving Facility} = "589"
Else:
```

- Figure 22: Example of Save Rules Button**

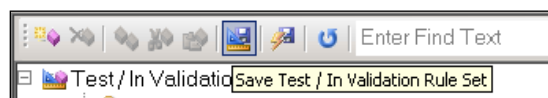

9. Verify that the Test / In Validation Rule Set Saved message is received (Figure 23).

Figure 23: Example of Successful Rule Save

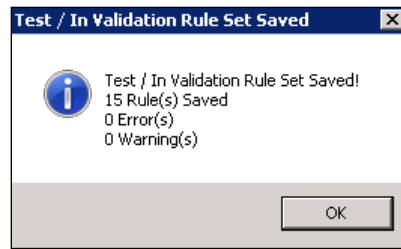

STOP *If errors while saving rules are encountered, file a service desk ticket to coordinate assistance with installation. Do not proceed until the issue is resolved.*


10. Click on the **Save Live Rule Set** button (Figure 24) located in the toolbar and click **Yes** to confirm.

Figure 24: Example of Save Rules in Live Set Button



11. Expand the **Live Rule Set**, and verify that rule text matches *Appendix C: Rules* and includes changes made to **SetInstrumentID** and **SetReceivingFacility** (Figure 25).

Figure 25: Example of Live Rules Set View

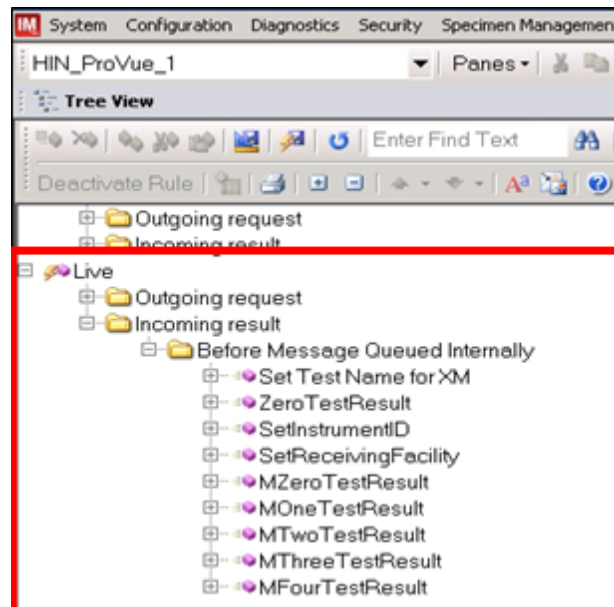



*If differences in rules are encountered, **file a service desk ticket to coordinate assistance with installation.** Do not proceed until the issue is resolved.*

12. Close the **Rule Setup** window, returning to the main menu.

4.7 Import VBECS (HL7) Side Configuration




Only one VBECS side configuration is needed even if you use multiple ProVue instruments. All ProVues will share the same configuration.



Modifying rules or test code mappings in the Instrument Manager configuration outside of this Setup Guide is not allowed and may lead to malfunction of the Automated Instrument to VBECS interface.

1. Navigate to **Configuration -> Configuration Editor** (Figure 26).
2. Click on the **Import** button.

Figure 26: Example of Configuration Editor Window

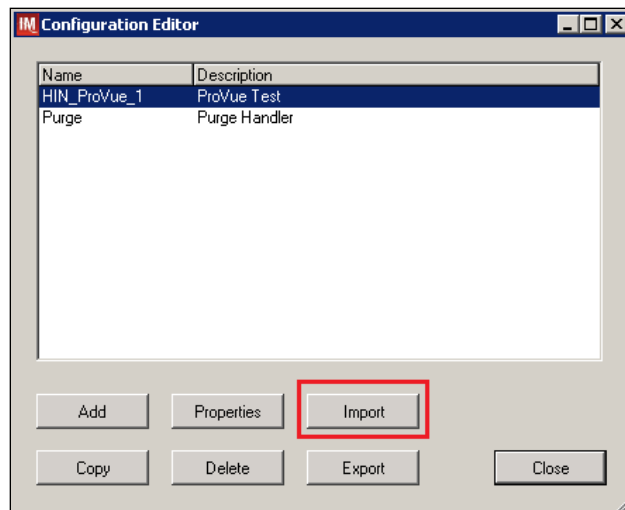


- Once the **Restore Driver Configuration** window opens, click the **Browse** button and select **C:\Temp** folder (Figure 27).
- Select **ProVue - HL7 Interface Configuration Template for VBECS 08172016.gsb** file from the list.
- Enter a **Configuration Name** that contains 3 letter location code of the instrument (e.g. **HIN** for Hines VAMC), word **VBECS** and sequence number (**1** for the first configuration, **2** for the second etc.). Example Configuration Name for VBECS side configuration located at Hines would be:

HIN_VBECS_1


This configuration name will be further referred in this document as *<HL7 Side Configuration>*.

Figure 27: Example of Configuration Import Window

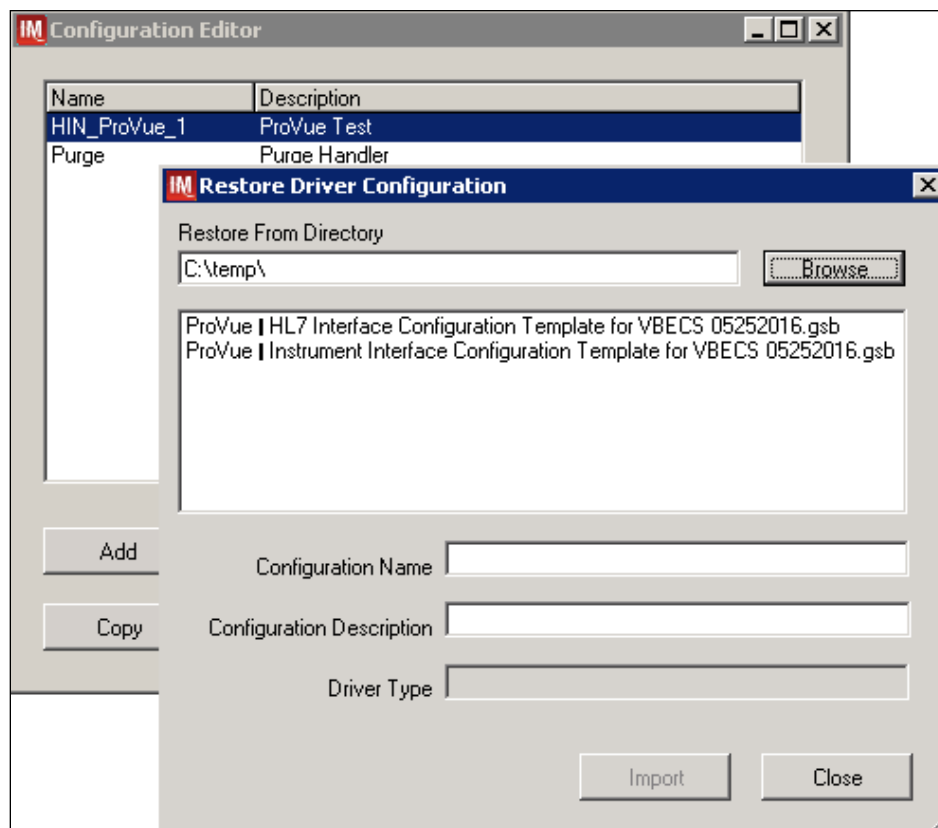

- Enter **Configuration Description** and click **Import** button. Verify that the following confirmation window (Figure 28) displays.

Figure 28: Example of Successful Configuration Import

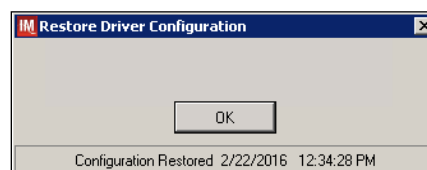




If you are using a newer version of the driver than the one mentioned in section 3, the Instrument Manager will warn you about the discrepancy in driver versions. Please acknowledge this warning and continue.

7. Click **OK** and close the **Restore Driver Configuration** window.
8. Verify that **Configuration Editor** shows the new configuration on the list (Figure 29).

Figure 29: Example of Newly Imported HL7 Configuration

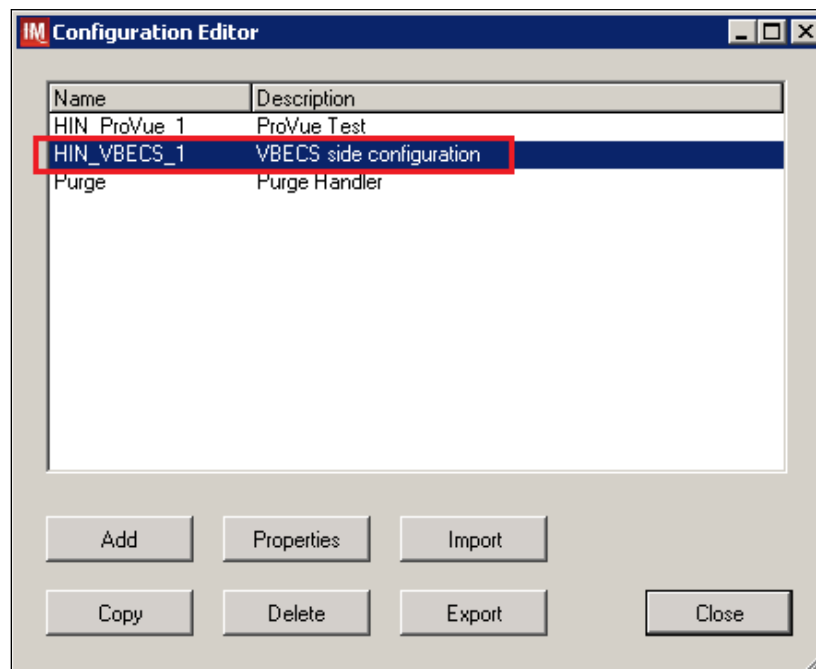


4.8 Modify Test Code Mapping for VBECS Side Configuration (only for sites that perform 2 Cell Antibody Screen)



Execute instructions in this section only if you perform Antibody Screen with 2 Screening Cells instead of 3.


1. Select configuration imported in previous section and click on the **Properties** button (Figure 30)

Figure 30: Example of Configuration Editor Window

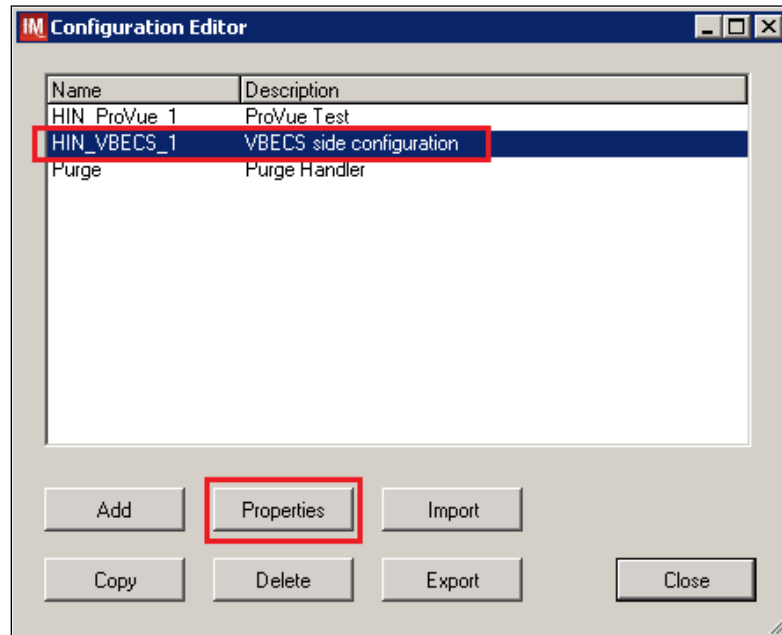


2. On the **Configuration Properties** window click on **Test Code Map** (Figure 31)

Figure 31: Example of Configuration Properties Window

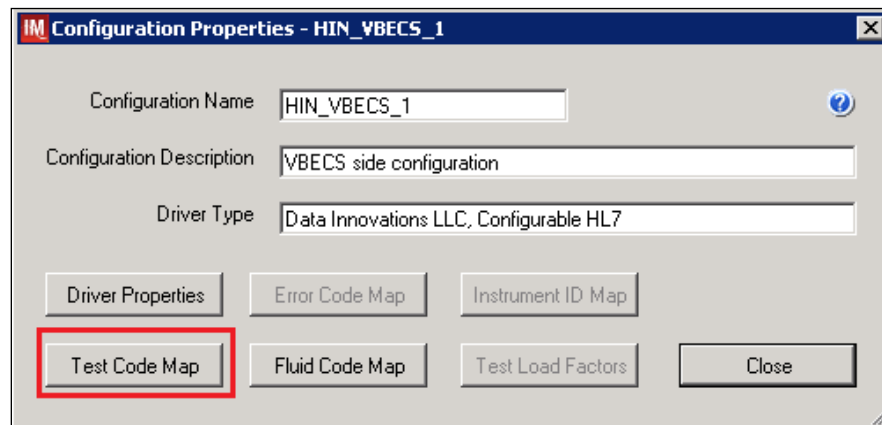


- Using the dropdown in the upper left corner of the **Test Code Mapping** window select test code **^VBECS ABS.pln^3 (SC3)** and click **Delete** button (Figure 32)

Figure 32: Example of deletion of SC3 test code for Antibody Screen

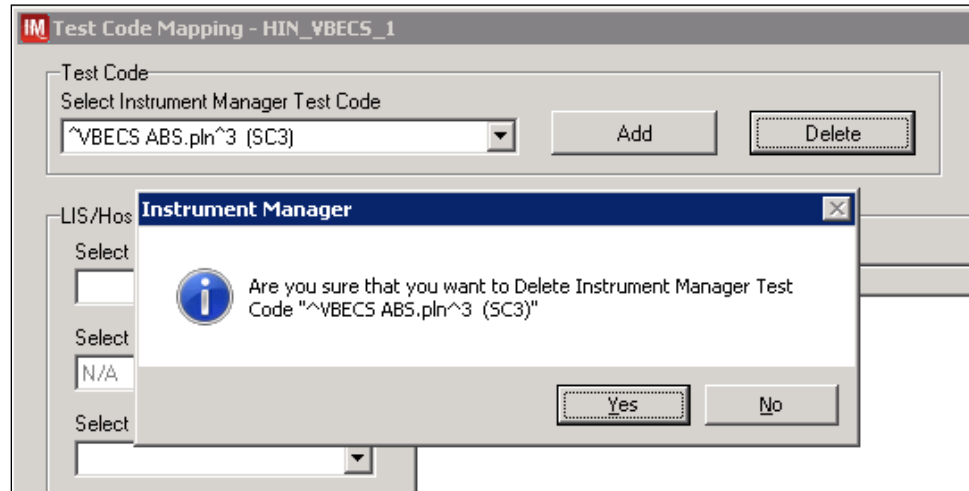

- Click **Yes** to confirm deletion
- Using the dropdown in the upper left corner of the **Test Code Mapping** window select test code **^VBECS TAS.pln^11 (SC3)** and click **Delete** button (Figure 33)

Figure 33: Example of deletion of SC3 test code for Type and Screen

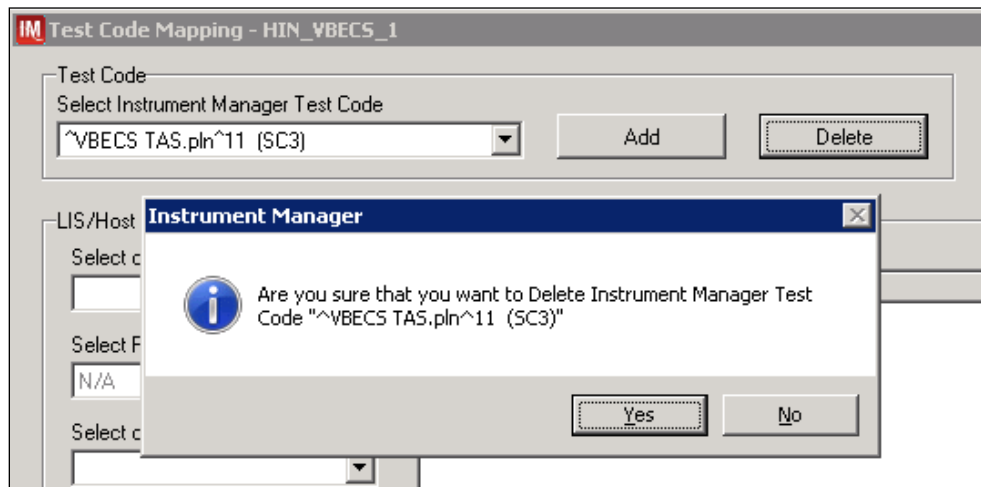

- Click **Yes** to confirm deletion
- Click **Close** on **Test Code Mapping** window
- Click **Close** on **Configuration Properties** window
- Click **Close** on **Configuration Editor**

4.9 Verify Test Code Mapping for VBECS Side Configuration

1. Navigate to **Reports ->Configuration Options and Mappings**.
2. Select the **<HL7 Side Configuration Name>** from the drop-down in the upper left corner (Figure 34).
3. Select the **Results Test Code Mapping Tab** (Figure 34).
4. Verify that the **Results Test Code Mapping Tab** matches all rules listed in *Appendix B: HL7 (VBECS) Side Mapping*.

Figure 34: Example of HL7 Configuration Report Window

REDACTED

5. Close **Configuration Options and Mappings** window.



If mismatches in Test Codes names, missing or extra Test Codes are encountered, file a service desk ticket to coordinate assistance with installation. Do not proceed until the issue is resolved.

5 Set Up HL7 Connection to VBECS TEST



Modifying rules or test code mappings as imported using this Instrument Manager configuration Setup Guide may lead to malfunction of the Automated Instrument to VBECS interface.

1. Navigate to **Configuration -> Connection Assignment**.
2. On the **Connection Assignment** window click **Add** button.
3. Select **Configuration Name** <*HL7 Side Configuration*> (Figure 35).
4. Enter the **Connection Name** that matches the <*HL7 Side Configuration*> and word **Connection**. For example:

HIN_VBECS_1_Connection

5. Check **Include in Specimen Management** check box (Figure 35).
6. Select **TCP/IP connection** in Device (Figure 35).

Figure 35: Example of Connection Properties Window

Connection Name: HIN_VBECS_1_Connection

Configuration Name: HIN_VBECS_1

Site:

Location:

☒ Start on System Start

Destination Line(s):

Number of Days to Keep:

- Incoming Messages: 3
- Outgoing Messages: 3
- Communications Trace: 3
- Error Messages: 3
- Driver Data: 3

Advanced Options:

- ☐ Explode Coded Entries for this Connection
- ☒ Include in Specimen Management
- Default userid:
- ☐ Update Specimen Management on Status Messages
- ☐ Include in Specimen Storage and Retrieval

Device:

- ☐ NULL
- ☐ COM
- ☒ TCP/IP


Device Parameters ...

Close

7. Click on **Device Parameters** button.

8. Enter **TCP/IP Address** and **TCP/IP Port Number** that matches **VBECS TEST Application IP Address and IP Port Number** configured in **VBECS TEST Administrator** application for **Auto Instrument Interface** (Figure 36). (Please refer to *VBECS Technical Manual Security Guide* for instruction on how to configure interfaces for VBECS.)


Figure 36: Example of TCP/IP Configuration Window

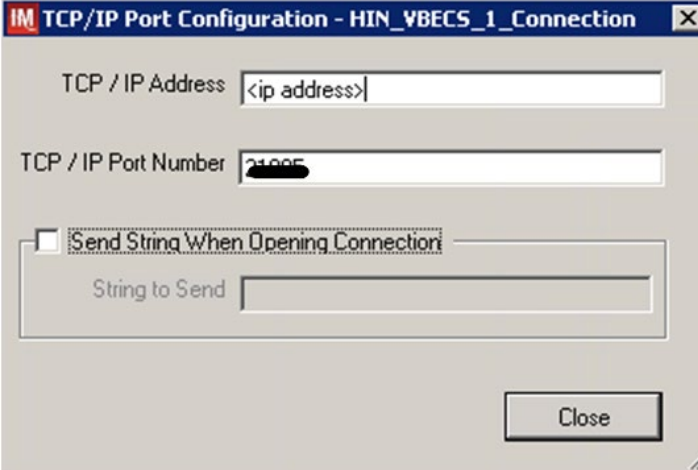

The screenshot shows a window titled "IM TCP/IP Port Configuration - HIN_VBECS_1_Connection". It contains two text input fields: "TCP / IP Address" with the placeholder text "<ip address>" and "TCP / IP Port Number" with the value "21005". Below these is a checkbox labeled "Send String When Opening Connection" which is currently unchecked. Underneath the checkbox is a text input field labeled "String to Send". A "Close" button is located at the bottom right of the window.

9. Close the **TCP/IP Port Configuration** window and click **Yes** to save.
10. Close the **Connection Properties** window and click **Yes** to save.
11. Verify that the newly created connection shows on the **Connection Assignment** window (Figure 37).

Figure 37: Example of Newly Created Connection

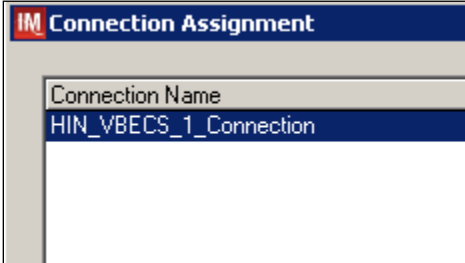

The screenshot shows a window titled "IM Connection Assignment". It features a table with a single row. The first column is labeled "Connection Name" and the value in this row is "HIN_VBECS_1_Connection".

Connection Name
HIN_VBECS_1_Connection

6 Set Up Instrument Connection



Execute instructions in this section for each instrument that will be connecting to VBECS.




Modifying connection settings in the Instrument Manager outside of this Setup Guide is not allowed and may lead to malfunction of the Automated Instrument to VBECS interface.

1. On the **Connection Assignment** window, click **Add** button.
2. On the **Connection Properties** window, enter the **Connection Name**. Enter a name that contains *<Instrument Side Configuration>* and word **Connection**. For example:

HIN_ProVue_1_Connection


3. Select *<Instrument Side Configuration>* as **Configuration Name**.
4. Check **Include in Specimen Management** check box.
5. Select **TCP/IP** connection.
6. On the **Destination Lines** list, check the box next to connection configured in previous section (Figure 38).

Figure 38: Example of Connection Properties Window

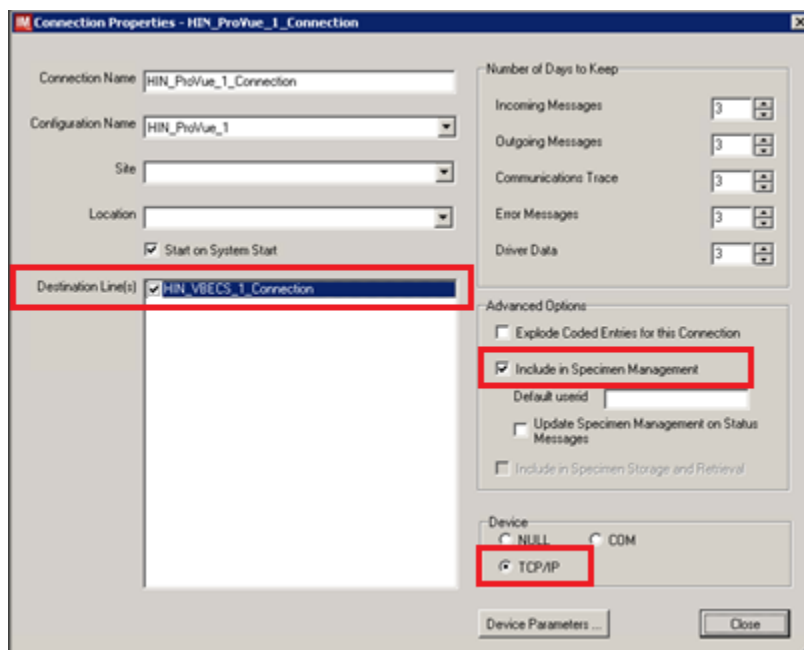


7. Click on **Device Parameters** button.
8. Enter **TCP/IP Address** and **TCP/IP Port Number** that matches *<Instrument IP>* and *<Instrument Port>* discussed in Section **Network Connectivity Setup**.
9. Uncheck **Send String When Opening Connection** check box (Figure 39).

Figure 39: Example of TCP/IP Configuration Window

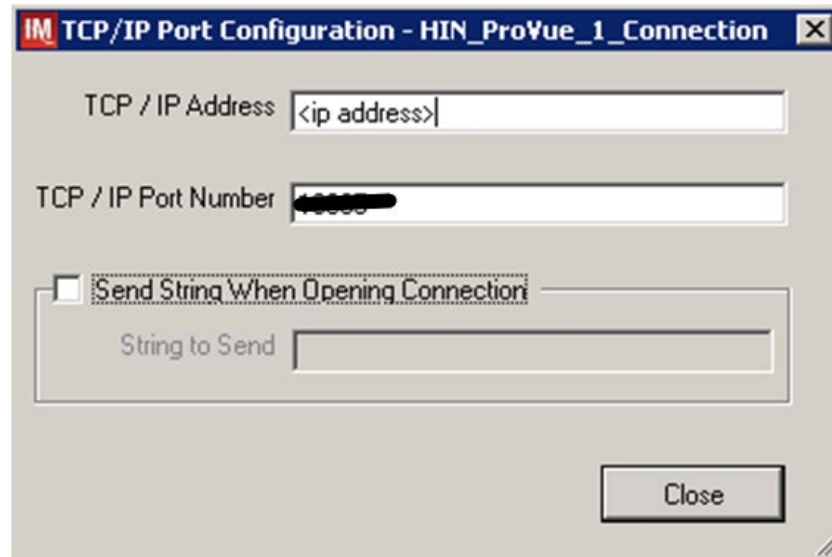

10. Click close on the **TCP/IP Port Configuration** window and click **Yes** to save.
11. Click close on the **Connection Properties** window and click **Yes** to save.
12. Verify the newly created connection shows on **Connection Assignment** window (Figure 40).
13. Close the **Connection Assignment** window.


Figure 40: Example of Newly Created Connection

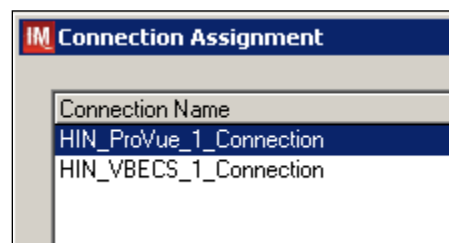

7 Test New Connections


1. Navigate to **System -> Status**.
2. Right-click on each newly created connection and choose option to **Start Selected Connections** (Figure 41).

Figure 41: Example of Connection Status Window

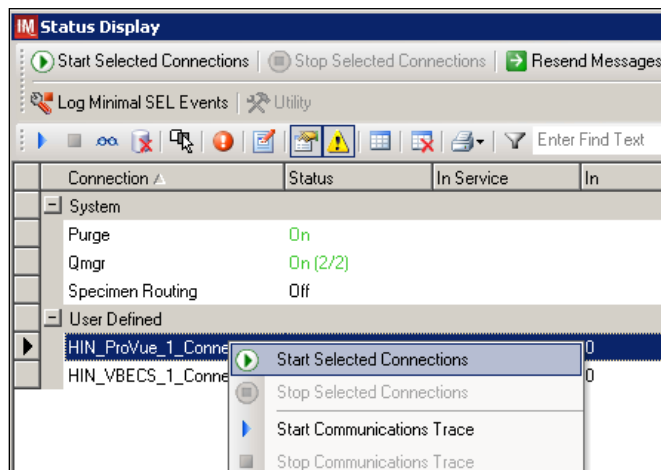

3. Verify that all newly created connections are showing **Status** of **On** after about a minute or so. (Figure 42).

Figure 42: Example of Successful Connection Test

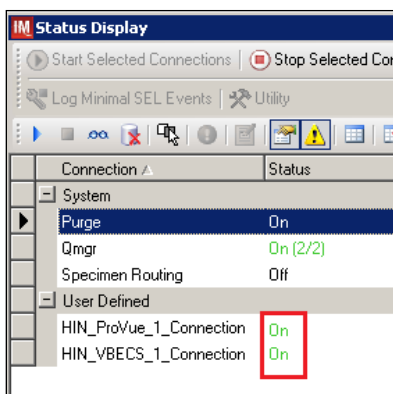

If one or more connections fail to start, file a service desk ticket to coordinate assistance with installation. Do not proceed until the issue is resolved.

4. Close the **Status Display**.

8 Validate Instrument Connectivity to VBECS TEST


Execute validation instructions from *Appendix E: Validation Planning and Example Test Scenarios* to verify that Instrument is properly interfacing with VBECS.

STOP *If one or more validation scenarios fail, file a service desk ticket to coordinate assistance with installation. Do not proceed until the issue is resolved.*

9 Set up HL7 Connection to VBECS PROD


1. Navigate to **System -> Status**.
2. Right-click <VBECS_Connection> and choose option to **Stop Selected Connections** (Figure 43).

Figure 43: Example of Stop Selected Connections

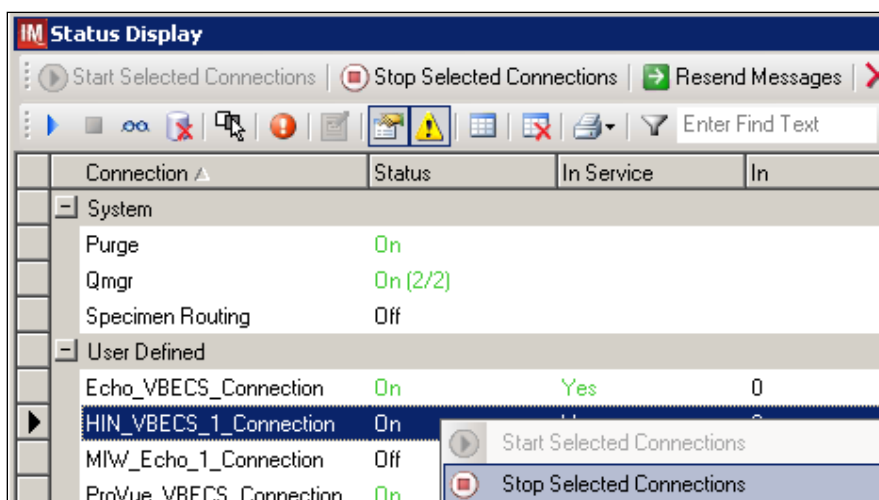

3. Wait until connection status changes to **Off**. Navigate to **Configuration -> Connection Assignment**.
4. Select <VBECS_Connection> and click **Properties** (Figure 44).

Figure 44: Example of Connection Assignment

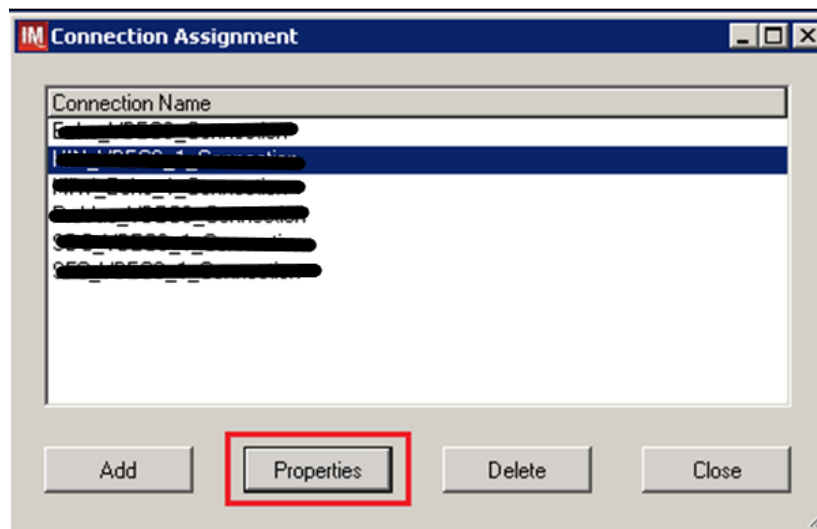

5. On Connection Properties window click **Device Parameters** (Figure 45).

Figure 45: Example of Connection Properties

Connection Properties - HIN_VBECS_1_Connection

Connection Name: [Text Field]

Configuration Name: [Dropdown Menu]

Site: [Dropdown Menu]

Location: [Dropdown Menu]

☒ Start on System Start

Destination Line(s): [List Box]

Number of Days to Keep:

- Incoming Messages: [3]
- Outgoing Messages: [3]
- Communications Trace: [3]
- Error Messages: [3]
- Driver Data: [3]

Advanced Options:

- ☐ Explode Coded Entries for this Connection
- ☐ Include in Specimen Management
 - Default userid: [Text Field]
 - ☐ Update Specimen Management on Status Messages
- ☐ Include in Specimen Storage and Retrieval


Device:

- ☐ NULL
- ☐ COM
- ☒ TCP/IP

Device Parameters ... [Close]


6. Enter **TCP/IP Address** and **TCP/IP Port Number** that matches **VBECS PROD Application IP Address and IP Port Number** configured in **VBECS PROD Administrator** application for **Auto Instrument Interface** (Figure 46). Please refer to *VBECS Technical Manual Security Guide* for instruction on how to configure interfaces for VBECS.

Figure 46: Example of TCP/IP Configuration Window

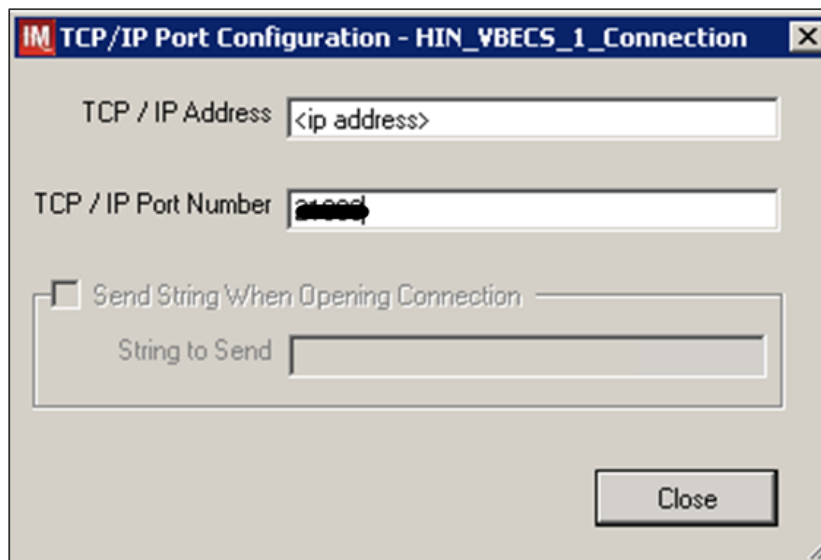


7. Close the **TCP/IP Port Configuration** and click **Yes** to confirm changes.
8. Close the **Connection Properties window** and click **Yes** to confirm changes.
9. Close the **Connection Assignment** window.
10. Navigate to **System -> Status**.
11. Right-click **<VBECS_Connection>** and choose option to **Start Selected Connections** (Figure 47).


Figure 47: Example of Connection Status Window

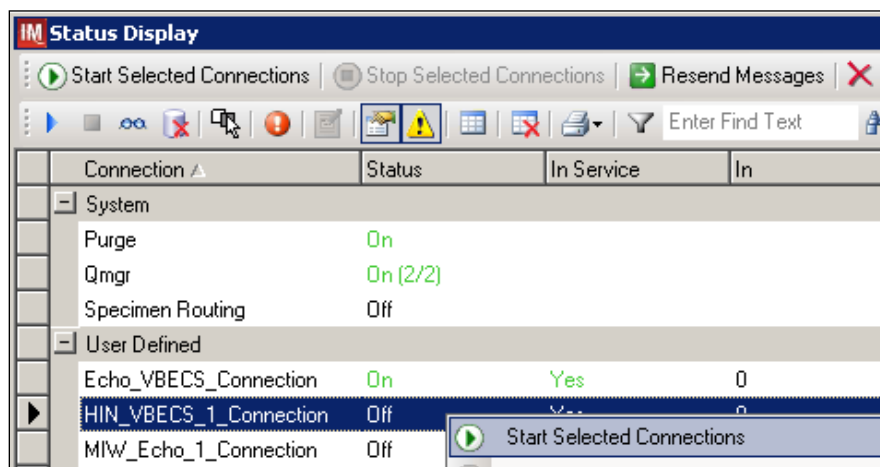

12. Verify that <VBECS_Connection> is showing Status of **On** after about a minute or so. (Figure 48).

Figure 48: Example of Successful Connection Test

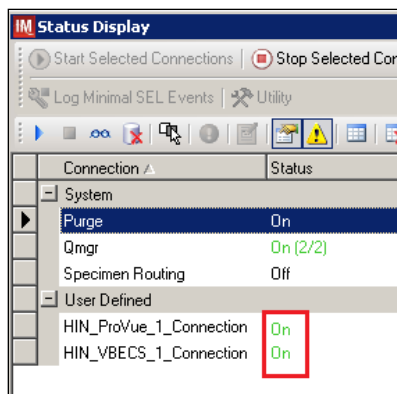

 ***If connection fails to start, file a service desk ticket to coordinate assistance with installation. Do not proceed until the issue is resolved.***

13. Close the **Status Display** and Log Off the system.

Glossary

Acronym, Term	Definition
Automated Instrument	Blood Bank Analyzer that performs blood testing.
Division Code	Also known as Station Number in VistA is the unique alphanumeric code that is associated with each hospital (e.g. 589 for VA Heartland West).
Instrument Manager (IM)	Software created by Data Innovations that serves as a middleware between Automated Instrument and VBECS. It translates messages containing test results sent from an instrument into HL7 messages that are then parsed into VBECS.
VA	Department of Veterans Affairs.
VBECS	VistA Blood Establishment Computer Software.
VistA	Veterans Health Information Systems and Technology Architecture. VistA software, developed by the VA, is used to support clinical and administrative functions at VA Medical Centers nationwide. VistA is composed of packages that undergo a verification process to ensure conformity with name spacing and other VistA standards and conventions.

Appendices

Appendix A: Instrument Side Mapping

Table 1: Instrument Side Mapping

REDACTED

Appendix B: HL7 (VBECS) Side Mapping

Table 2: HL7 (VBECS) Side Mapping

REDACTED

Appendix C: Rules

Figure 48: Rules

--- Test / In Validation - Incoming result - Before Message Queued Internally ---

Rule # 1

Desc - Set Test Name for XM
If - {Test Code} {On Any Test} {Contains} "Crossmatch"
Then - {Set} {Test Code Sub ID} {On That Test} = "XM"

Rule # 2

Desc - Set Test Name for TAS
Child of Rule # 1 - Else
If - {Test Code} {On Any Test} {Contains} "TAS"
Then - {Set} {Test Code Sub ID} {On That Test} = "TAS"

Rule # 3

Desc - Set Test Name for DAT
Child of Rule # 2 - Else
If - {Test Code} {On Any Test} {Contains} "DAT"
Then - {Set} {Test Code Sub ID} {On That Test} = "DAT"

Rule # 4

Desc - Set Test Name for Patient ABORh
Child of Rule # 3 - Else
If - {Test Code} {On Any Test} {Contains} "ABDRev"
Then - {Set} {Test Code Sub ID} {On That Test} = "Patient ABORh"

Rule # 5

Desc - Set Test Name for ABS
Child of Rule # 4 - Else
If - {Test Code} {On Any Test} {Contains} "ABS"
Then - {Set} {Test Code Sub ID} {On That Test} = "ABS"

Rule # 6

Desc - Set Test Name for Unit ABORh
Child of Rule # 5 - Else
If - {Test Code} {On Any Test} {Contains} "Confirm"
Then - {Set} {Test Code Sub ID} {On That Test} = "Unit ABORh"

Rule # 7

Desc - Set Test Name for Antigen Typing
Child of Rule # 6 - Else
If - {Test Code} {On Any Test} {Contains} "Phenotype"
Then - {Set} {Test Code Sub ID} {On That Test} = "Antigen Typing"

Rule # 8

Desc - ZeroTestResult
If - {Result} {On Any Test} = "-"
Then - {Set} {Result} {On That Test} = "0"

Rule # 9

Desc - SetInstrumentID
If - {Always}
Then - {Set} {Instrument ID} = ""

Rule # 10

Desc - SetReceivingFacility
If - {Always}
Then - {Set} {Receiving Facility} = ""

Rule # 11

Desc - MZeroTestResult

If - {Result} {On Any Test} = "M-"

Then - {Set} {Result} {On That Test} = "0"

Rule # 12

Desc - MOneTestResult

If - {Result} {On Any Test} = "M1+"

Then - {Set} {Result} {On That Test} = "1+"

Rule # 13

Desc - MTwoTestResult

If - {Result} {On Any Test} = "M2+"

Then - {Set} {Result} {On That Test} = "2+"

Rule # 14

Desc - MThreeTestResult

If - {Result} {On Any Test} = "M3+"

Then - {Set} {Result} {On That Test} = "3+"

Rule # 15

Desc - MFourTestResult

If - {Result} {On Any Test} = "M4+"

Then - {Set} {Result} {On That Test} = "4+"

Appendix D: Testing Templates

Table 3: Testing Templates

Template Name	Selected Fields	VBECS Test	Test Component
VBECS ABDRev.pln	A B A1 Cell B Cell D Ctl Group Rh Remarks	Patient ABO/Rh	Anti-A Anti-B A1 Cells B Cells Anti-D D Control ABO Interp Rh Interp Comments
VBECS ABS.pln	Cell1 Cell2 Cell3 AbScr Remarks	Antibody Screen	Screening Cells1 Screening Cells2 Screening Cells 3 ABS Interp Comments
VBECS TAS.pln	A B A1 Cell B Cell D Ctl Group Rh Cell1 Cell2 Cell3 AbScr Remarks	Type and Screen	Anti-A Anti-B A1 Cells B Cells Anti-D D Control ABO Interp Rh Interp Screening Cells1 Screening Cells2 Screening Cells 3 ABS Interp Comments
VBECS DAT.pln	IgGC3 DAT Remarks	DAT	AHG DAT Interp Comment
VBECS DAT IgG.pln	IgG DAT IgG Remarks	DAT IgG	IgG DAT IgG Interp Comment
VBECS Crossmatch IAT.pln**	XIAT XM IAT Remarks	Crossmatch	AHG XM Interp Comments
VBECS Crossmatch.pln**	XIS XIAT XM IS XM IAT Remarks	Crossmatch	IS AHG IS Interp AHG Interp Comments
VBECS Confirm.pln	A B AB D Fwd Gr Rh Remarks	Unit ABO/Rh Confirmation	Anti-A Anti-B Anti-A,B Anti-D ABO Interp Rh Interp Comments


Template Name	Selected Fields	VBECS Test	Test Component
VBECS Phenotype.pln	D Phen* C E K sm c sm e Ctl Phen* Rh Phen* Remarks	Patient and Unit Antigen Typing	N/A Anti-C Anti-E Anti-K Anti-c Anti-e N/A N/A Comment

*Test results not used in VBECS. They are on the template to allow users to print them if necessary.

** VBECS Crossmatch IAT.pln and VBECS Crossmatch.pln are mutually exclusive. Please setup only one that is appropriate for your site

Appendix E: Validation Planning and Example Test Scenarios

The following is a flowchart to help assess any one change and plan accordingly.

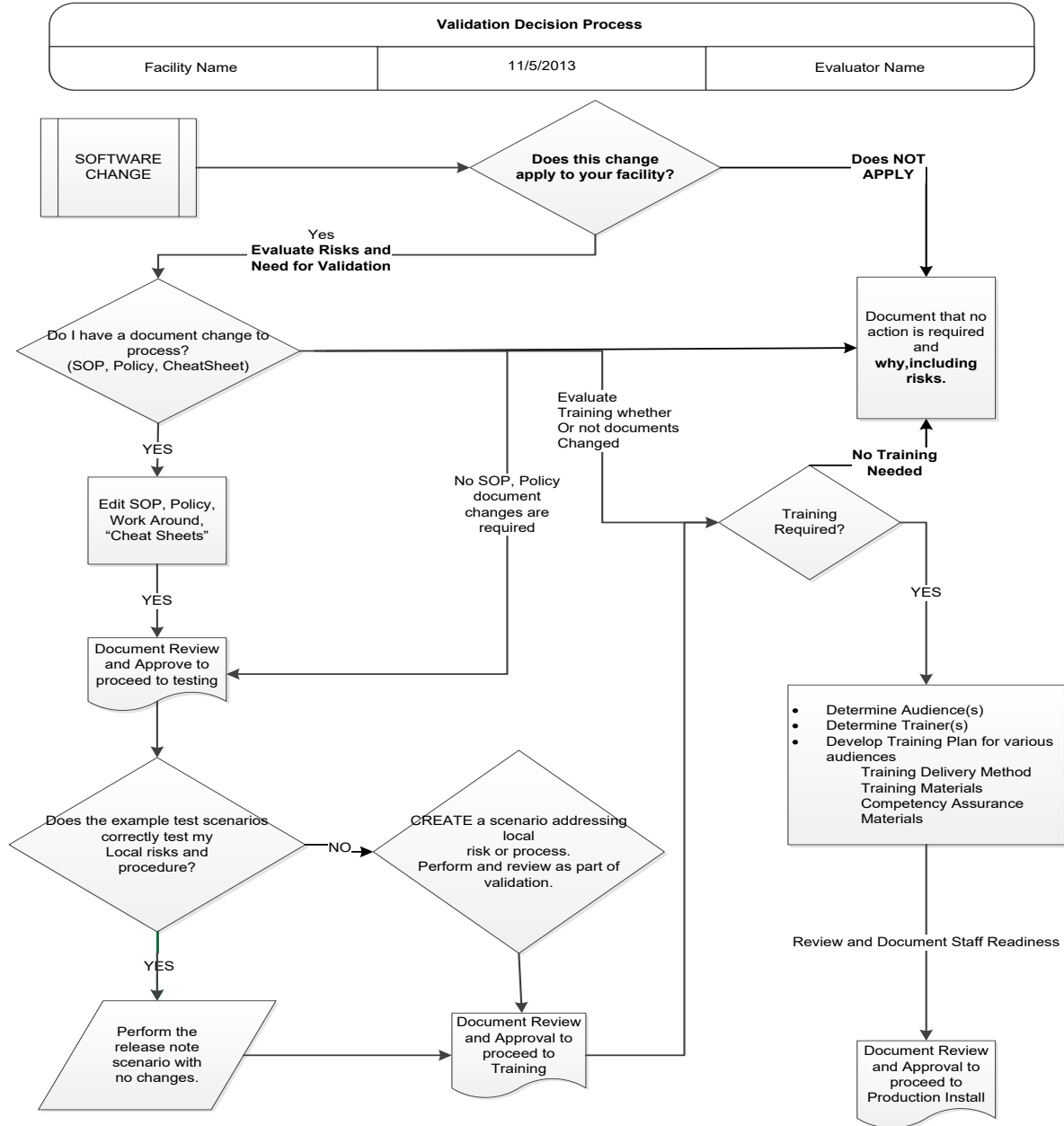



These are examples of test scenarios. Each site is responsible for evaluating changes for their intended use, assess risk, and for establishing additional validation test scenarios.

All test scenarios have a suggested user role, this may require adjustment to align with the patient or unit data selected to execute the scenario. Process any overrides as well, based on patient or unit selection.

**The Expected Outcome numbering corresponds to the Step number where the change verification appears.*

Test Group One: AI interface disabled

Test Objective: Demonstrate that the system will reject test results sent from automated instrument if the Automated Instrument interface is disabled.

Option: VBECS Administrator, Automated Instrument Interface

At least one instrument available in any division and configured for connection with VBECS via Instrument Manager.

Note: This may be executed at only one division if VBECS is used in a multidivisional configuration as the interface is enabled/disabled for all or none.

Data	<p>Before beginning, verify current configuration, activity, and status of the existing interfaces, VistALink, and CPRS in your test account.</p> <p>Make sure that the VBECS-OERR HL7 link in VistA is NOT shutdown.</p> <ol style="list-style-type: none">1. Create RBC and TAS orders in CPRS and accept them in VBECS.
User	<p>VBECS Administrator access is required to configure the AUTOMATED TESTING interface.</p> <p>No specific user role is required to process TAS and RBC orders in VBECS.</p>
Steps	<ol style="list-style-type: none">1. Log into VBECS Administrator:2. Disable the AUTOMATED TESTING interface using Configure Interfaces option.3. A message appears (You are about to disable Auto Instrument Interface. It will cause VBECS to stop sending and receiving messages via that interface. Continue?), Click Yes.4. Close the window as all fields are disabled.5. Perform TAS test on the instrument and send results of testing to VBECS
Expected Outcome	<ol style="list-style-type: none">5. Verify that after sending test results from an instrument fails it is possible to complete TAS test using manual testing grids and subsequently issue and transfuse blood unit to a patient.
Reports:	<ol style="list-style-type: none">6. Review the Audit Trail report for changes to the interface.

Test Group Two: Verify AI individual test(s)

Test Objective: Demonstrate that the system will allow the selected user role to perform normal workflow activities using your local blood bank testing instrument.

Note: Validate all tests that are performed at your site using an automated instrument.



Prior to initiating any testing of the automated instrument interface, see the VistA Blood Establishment Computer Software (VBECS) 2.2.0 Technical Manual-Security Guide for instructions for configuring your local instrument and Data Innovations Instrument Manager.




Configure your Test UID to clearly differentiate from a Production UID during your validation, for example, production is 1234567890, with 123 being your consistent prefix, change 123 or 12 to TS or TST or 999. Please change the Numeric Identifier for the Blood Bank Accession Area in your site Test Account.

Enable Automated Instrument Interface in VBECS Admin if it was disabled while performing testing in Test Group One.

Verify your VBECS processes with the Automated Instrument interface.

Your local test plan will demonstrate that the system will perform normal daily work per your local policies, procedures and local validation plan that may include:

Blood Units: Automated Instrument or via the short cut key 

- ABO-Rh confirmation
- Unit Antigen Typing or Repeat

Patients: Automated Instrument or via the short cut key 

- Perform a Type and Screen test (ABS and ABS with Auto-control only)
- Perform crossmatch tests: Serologic (*Selected in Blood Units: Select Units before testing.**)
- Patient Tests:
 - ABO/Rh or Repeat
 - Antibody Screen or Repeat
 - Direct Antiglobulin Test or Repeat
 - Antigen Typing or Repeat
- Overrides, (ABO/Rh Discrepancy, Crossmatch incompatible: Give with Medical Director Approval)
- Report data from these actions is available for retrieval.
 - Testing Worklist Report
 - Patient History Report
 - Unit History Report
 - Exception Report (ABO/Rh Discrepancy, Give with Medical Director Approval)

*Remember to click NO to proceeding to the serologic crossmatch when selecting units for automated instrument testing.

Test Group 2 Scenario 1: Verify AI TAS test

Note: the Type and Screen (TAS) is a combination of two tests and may be saved as individual tests without completing the TAS as a whole.

Data	<p>VBECS: Select a patient.</p> <p>CPRS: Place a Type and Screen (TAS) order for the patient</p> <p>LAB: Accession the order</p> <p>VBECS: Accept the order. (Orders, Accept Orders)</p>
User	No specific user role is required to process TAS order in VBECS.
Steps	<ol style="list-style-type: none"> 1. User checks the Patient testing list to make sure the order is accepted in VBECS and appears on the Pending Task List (PTL). (Patient, Patient Testing, Diagnostic Tests). 2. Close the PTL. 3. Process the specimen on the instrument using the recommended TAS template for that instrument. Complete all work needed to transmit the information to VBECS. 4. Select Patients, Automated Instrument to review TAS results. 5. Select the specimen UID, scanning the UID is preferred. 6. Review TAS test results. 7. Accept only the ABS or ABO/Rh test. Close the window. 8. Open the PTL. 9. Try to select TAS on PTL and complete testing on it manually. 10. Open the Automated instrument window and accept the second part of TAS. 11. Close the automated instrument window. 12. Check Reports
Expected Outcome	<ol style="list-style-type: none"> 5. Verify that the specimen UID is selectable by scanning, entry or patient selection 6. Verify that the correct test results appear on the Automated Instrument review list. 8. Verify that the TAS appears on the PTL with a status of "Instrument Results Pending Review". 9. Verify that system prevents user from completing TAS since they are still pending results from an instrument for it. 12. Verify that the results and comments appear as expected on the reports: <ul style="list-style-type: none"> • Testing Worklist Report • Patient History Report • Exception Report

Test Group 2 Scenario 2: Verify AI serologic crossmatch test

Data	<p>VBECS: Select a patient.</p> <p>CPRS: Place a Type and Screen (TAS) order and Red Blood Cell order for the patient</p> <p>LAB: Accession the orders</p> <p>VBECS: Accept the order. (Orders, Accept Orders)</p> <p>Process the TAS to completion.</p> <p>Select a blood unit for a selected patient*:</p> <ul style="list-style-type: none"> • Previously entered into the division's inventory (Blood Units, Incoming Shipment) • ABO compatible • May be available or selected for another patient (available, selected, crossmatched) status. • May or may not trigger selection overrides <p>*Remember to click NO to proceeding to the serologic crossmatch when selecting units for automated instrument testing.</p>
User	No specific user role is required to process crossmatch test in VBECS.
Steps	<ol style="list-style-type: none"> 1. Process the component unit's specimen on the instrument using the recommended template for that instrument. Complete all work needed to transmit the information to VBECS. 2. Select Patients, Automated Instrument to review crossmatch results. 3. Select the specimen UID, scanning the UID is preferred. 4. Review results, select compatibility and accept crossmatch result. 5. Print or do not print tag as desired. 6. Accept the test and close the automated instrument window. 7. Check Reports.
Expected Outcome	<ol style="list-style-type: none"> 4. Verify that crossmatch results sent from an instrument show correctly on the screen. 7. Verify that the results and comments appear as expected on the reports: <ul style="list-style-type: none"> • Testing Worklist Report • Patient History Report (interpretations only). • Exception Report

Test Group 2 Scenario 3: Verify AI patient diagnostic tests

(ABO/Rh, Antibody Screen, Direct Antiglobulin Test, Patient Antigen Typing, and the reflex test)

Data	VBECS: Select a patient. CPRS: Place a diagnostic test order for the patient LAB: Accession the order VBECS: Accept the order. (Orders, Accept Orders)
User	No specific user role is required to process diagnostic tests in VBECS.
Steps	<ol style="list-style-type: none">1. User checks the Patient testing list to make sure the order is accepted in VBECS and appears on the Pending Task List (PTL). (Patients, Patient Testing, Diagnostic Tests).2. Close the PTL.3. Process the specimen on the instrument using the recommended testing template for that instrument. Complete all work needed to transmit the information to VBECS.4. Select Patients, Automated Instrument to review test results.5. Select the specimen UID, scanning the UID is preferred.6. Review and accept test results7. Close the automated instrument window.8. Check reports.
Expected Outcome	<ol style="list-style-type: none">5. Verify that the specimen UID is selectable by scanning, entry or patient selection.6. Verify that patient test results sent from an instrument show correctly on the screen.8. Verify that the results and comments appear as expected on the reports:<ul style="list-style-type: none">• Testing Worklist Report• Patient History Report• Exception Report

Test Group 2 Scenario 4: Verify AI blood unit tests

Data	<p>VBECS: Select a blood unit previously entered into the division's inventory (Blood Units, Incoming Shipment).</p> <p>For ABO/Rh Confirmation testing on the instrument, unit should be in a Limited status.</p> <p>For Unit Antigen Typing, the unit may or may not have been confirmed.</p>
User	<p>No specific user role is required to process blood unit tests in VBECS.</p>
Steps	<ol style="list-style-type: none">1. Process the component unit's specimen on the instrument using the recommended template for that instrument. Complete all work needed to transmit the information to VBECS.2. Select Blood Units, Automated Instrument to review test results.3. Select the blood component unit's Donor Identification Number (DIN), scanning the DIN is preferred.4. Select the product code (if there are multiple blood units with the same product code)5. Review the transmitted blood unit test.6. Accept the test and close the automated instrument window.7. Check Reports
Expected Outcome	<ol style="list-style-type: none">5. Verify that blood unit test results show correctly on the screen.7. Verify that the results and comments appear as expected on the reports:<ul style="list-style-type: none">• Testing Worklist Report• Unit History Report (interpretations only).• Exception Report

Revision History

Date	Revision	Description	Author
06-03-16	1.0	Initial version.	BBM team
		Defect 338501 <ul style="list-style-type: none"> Figure 4 Corrected driver name Section 4.1 <ul style="list-style-type: none"> Added information bar about getting files to the Instrument Manager server Removed Step 1 that had the user getting the files from the Instrument Manager server Added step 5 to copy files to the Instrument Manager Server Updated step 6 to say on the Instrument Manager Server Highlighted checksum command Added an information bar about copying and pasting into a PowerShell window Updated Figure 8 with new checksums Section 4.3 <ul style="list-style-type: none"> Changed Verify step to use a report instead of screen captures Figure 13 Updated to show Results Test Code Mapping Report Window Section 4.4 <ul style="list-style-type: none"> Changed Verify step to use a report instead of screen captures Removed “and what is listed in Figure 11” from what is now step 4 Figure 14: Updated to show Rules Report Window Section 4.7 <ul style="list-style-type: none"> Changed Verify step to use a report instead of screen captures Figure 25: Updated to show Results Test Code Mapping Report Window Figure 18: Updated to show 15 rules Figure 20: Updated to show additional rules Section 5 <ul style="list-style-type: none"> Changed steps 9 and 10 to include save steps Removed step 12 Appendix A: Reordered alphabetically by Instrument Test Code Appendix B: Reordered alphabetically by Instrument Test Code Appendix C: Changed to use the report instead of the screen captures Appendix D: Corrected spelling of VBECS Appendix F: Updated validation scenarios Updated Footer to include the Electronic Document Control System. Moved Footer notes to Header, Updated Document to Version to 2.0	
06-28-16	2.0		BBM team

Date	Revision	Description	Author
08-19-16	3.0	Defect 352802: <ul style="list-style-type: none"> Updated section Testing Template Setup to describe selection of only one crossmatch template Updated section User ID Setup to accommodate users with short user IDs Updated names and checksums of configuration files in section 4 Added section 4.3 with instructions on how to remove test code mapping for Screening Cell 3 for instrument side configuration Added section 4.8 with instructions on how to remove test code mapping for Screening Cell 3 for VBECS side configuration Updated appendices A and B with new test codes for IAT only crossmatch Updated appendix D with new template for IAT only crossmatch Defect 355473: <ul style="list-style-type: none"> Added warning about discrepancy in driver versions in sections 4.2 and 4.7 Added information about proper configuration of test Specimen UIDs to Test Group Two warning flag section Removed information about Event Log from Test Group One and replaced it with checking Audit Trail Report 	BBM team
09-07-16	4.0	Set Up Automated Instrument, Testing Template Setup section: Added informational message that refers to AABB standards for crossmatch. (Defect 372140)	BBM team
08-03-17	5.0	Added information about requirement for Specimen Management Module being active (Defect 465496)	BBM team
12-29-22	6.0	Header and Footer: Updated. Throughout: Updated references from CA SDM ticket to service desk ticket and removed Appendix E CA SDM Support Ticket Template. Introduction: Added fourth paragraph about configuration files. Related Manuals and Reference Materials: Updated. Section 4.1: Updated Steps 1 and 2, and added Step 3. Updated Figures 4 and 6, added Figure 7. All figures: Added alternate text for 508 compliance. (Task 111163)	BBM team

This is the last page of *VBECS - ProVue Interface Configuration and Setup Guide*.