

VistA Imaging Exchange (VIX) Enhancements

MAG*3.0*348

Enhanced Image Viewer User Guide



September 2023

Version 23.1

Department of Veterans Affairs (VA)

Office of Information and Technology (OIT)

Revision History

Date	Revision	Description	Author
09/01/2023	23.1	Updated for changes to MAG*3.0*348 Print and Export.	VA IT VistA Imaging Technical team
07/03/2023	23.0	Updated for MAG*3.0*348. Added ECG.	VA IT VistA Imaging Technical team
05/01/2023	22.0	Updated for MAG*3.0*329. Added multi-page TIFF imaging.	VA IT VistA Imaging Technical team
11/01/2022	21.0	Updated for MAG*3.0*303	VA IT VistA Imaging Technical team
04/05/2022	20.2	Updates for MAG*3.0*269 T4	VA IT VistA Imaging Technical team
02/18/2022	20.1	Updates for MAG*3.0*269 name	VA IT VistA Imaging Technical team
10/18/2021	20.0	Updated for MAG*3.0*269	VA IT VistA Imaging Technical team
05/21/2021	19.0	Updated for MAG*3.0*284	REDACTED
07/27/2020	18.0	Updated for MAG*3.0*254	VA IT VistA Imaging Technical team
06/19/2019	17.1	Updated title page for MAG*3*230 remaining content is the same	REDACTED
05/24/2019	17.0	Updated screenshots with captions	REDACTED
11/15/2018	16.0	Additional MAG*3.0*221 Updates	REDACTED
10/03/2018	15.0	Updated for MAG*3.0*221	REDACTED
08/22/2018	14.0	Updated for MAG*3.0*201	REDACTED
07/11/2018	13.0	Additional MAG*3.0*197 Updates	REDACTED
03/26/2018	12.0	Updated for MAG*3.0*197	REDACTED
11/06/2017	11.0	Additional MAG*3.0*185 Updates	REDACTED
10/12/2017	10.0	Initial MAG*3.0*185 Updates and Incorporated comments from reviewers.	REDACTED
10/04/2017	9.0	Removed ECG references. Date Updates	REDACTED
09/06/2017	8.0	Date Updates	REDACTED
06/27/2017	7.0	Updated for MAG*3.0*177	REDACTED
04/18/2017	6.0	Formatting updates	REDACTED
04/13/2017	5.0	Formatting and screenshot updates	REDACTED

Date	Revision	Description	Author
04/06/2017	4.0	Updated for MAG*3.0*170	REDACTED
03/24/2017	3.0	Date Updates	REDACTED
02/14/17	2.0	Formatting updates and added reference to Enhanced Image Viewer. Removed Radiology Viewer Section	REDACTED
09/29/16	1.0	Initial Draft	REDACTED

Table of Contents

1. Introduction	9
1.1. Purpose.....	9
1.2. Document Orientation.....	9
1.2.1. Organization of the Manual	9
1.2.2. Assumptions	10
1.2.3. Coordination.....	10
1.2.4. Disclaimers	10
1.2.4.1. Software Disclaimer	11
1.2.4.2. Documentation Disclaimer	11
1.2.5. Documentation Conventions	11
1.2.6. References and Resources.....	11
1.3. Enterprise Service Desk and Organizational Contacts.....	12
2. System Summary	13
2.1. System Configuration	13
2.2. Data Flows	13
2.3. User Access Levels	13
2.4. Continuity of Operation	13
3. Getting Started	14
3.1. Logging On	14
3.2. Exit System.....	14
3.3. Caveats and Exceptions	14
4. Using the Software	15
4.1. Launching the Viewer	15
4.2. The Main Viewer Page	16
4.2.1. The Main Toolbar	17
4.2.2. Thumbnails and Image Loading	19
4.2.3. The Image Navigation Toolbars	21
4.3. Working with Images	22
4.3.1. Manipulating View Layout	22
4.3.2. Viewing DICOM Images.....	23
4.3.2.1. Viewing Stacked Images	23
4.3.3. Viewing an Electrocardiogram (ECG/EKG)	23
4.3.4. Viewing TIFF Images	25
4.3.5. Viewing Normal Web Images.....	25
4.3.6. Viewing Documents	25
4.3.7. Printing and Exporting Images	26
4.3.7.1. Printing an Image	26
4.3.7.2. Exporting an Image	27
4.3.8. External Links	28

4.3.8.1. Release of Information (ROI) Print Queue Status and ROI Submission	29
4.3.9. Viewing Information in an Image	30
4.3.10. Changing the Settings of an Image	31
4.3.11. Changing the Study Layout	36
4.3.12. Presentation State	36
4.3.13. Using the Measurements Tools	37
4.3.14. Using the RGB Tool	39
4.3.15. Conditions for Image Viewer Functions	39
4.4. Image Quality Assurance (QA)	40
4.4.1. Image Information	42
4.4.2. Image QA	43
4.4.3. Viewing and Setting Image Status	48
4.4.4. Image QA Report	49
4.5. Closing the Viewer	51
5. Troubleshooting	52
5.1. Special Instructions for Error Correction	52
6. Acronyms and Abbreviations	53

Table of Figures

Figure 1: Image Viewer System Summary	13
Figure 2: Loading Message	15
Figure 3: Images Not Loading	15
Figure 4: Main Viewer Page	16
Figure 5: Main Toolbar	17
Figure 6: Sample Disabled Toolbar Button.....	17
Figure 7: Main Viewer Page with Thumbnail Panel.....	20
Figure 8: Sample Image Navigation Toolbars	21
Figure 9: Frame Rate	22
Figure 10: Main Viewer Page with Images Loaded	23
Figure 11: DICOM ECG with Context Menu	24
Figure 12: ECG Preferences	24
Figure 13: PDF is Not Ready Status in the Lower Right Corner.....	25
Figure 14: PDF is Ready Status in the Lower Right Corner	25
Figure 15: Main Viewer Page with a Document Loaded.....	25
Figure 16: Print Menu.....	27
Figure 17: Print Menu - Device Selection	27
Figure 18: Exporting an Image	28
Figure 19: External Links Button in the Top Right Corner	28
Figure 20: External Links Menu.....	28
Figure 21: ROI Print Queue Status	29
Figure 22: Print Queue	29
Figure 23: DICOM Attributes	30
Figure 24: Image Metadata	30
Figure 25: Settings Menu	31
Figure 26: Annotations Preferences.....	31
Figure 27: Measurement Preferences	32
Figure 28: Ellipse and Rectangle Preferences	32
Figure 29: Label Preferences	33
Figure 30: Annotation Preferences.....	33
Figure 31: Cine Preferences	34
Figure 32: Layout Preferences	34
Figure 33: Layout Preference Choices	35
Figure 34: Copy Attributes Preferences	35
Figure 35: Layout Attribute Preference Choices.....	36
Figure 36: Presentation State Button	36
Figure 37: Presentation State Menu.....	36
Figure 38: Measurement Tool View 1.....	37
Figure 39: Measurement Tool View 2.....	37

Figure 40: Measurement Tool View 3.....	37
Figure 41: 2D Length Calibration	38
Figure 42: Hounsfield Measurement Tools.....	38
Figure 43: RGB Tool Button	39
Figure 44: Main Viewer Page with Image Loaded	40
Figure 45: External Links Menu on Main Viewer Page.....	41
Figure 46: Image Information	42
Figure 47: Image Editor	43
Figure 48: QA Review Page	43
Figure 49: QA Filters and Date Range	44
Figure 50: Search Filter Details	44
Figure 51: QA Status Choices.....	45
Figure 52: Percentage of Images to Display	45
Figure 53: Saved Filters	46
Figure 54: QA Filter Results	47
Figure 55: QA Filter Results and Image Information	47
Figure 56: QA Thumbnail	48
Figure 57: QA Review Status Update Menus.....	48
Figure 58: QA Statistics Reports Page.....	49
Figure 59: Run QA Report.....	50
Figure 60: View QA Report	50
Figure 61: Viewer Close Button.....	51
Figure 62: Viewer Version Number.....	52
Figure 63: Viewer Information Dialog	52

Table of Tables

Table 1: Toolbar Icons - Names, Functions, and Descriptions	17
Table 2: Thumbnail Elements, Names, Functions, and Descriptions	20
Table 3: Functions and Uses of the Image Navigation Toolbar Buttons	21
Table 4: Viewing Images and Manipulating View Layout	22
Table 5: Image Print/Export Behavior.....	26
Table 6: Conditional Image Viewer Functions	39

1. Introduction

1.1. Purpose

This manual describes the "Viewer" web application which is also known as the:

- Central VistA Imaging Exchange (CVIX) Viewer
- Enhanced Image Viewer
- Image Viewer
- VistA Imaging Exchange (VIX) Viewer
- VistA Imaging Viewer

The Viewer displays and provides tools for clinical and medical media in web applications like the Joint Legacy Viewer (JLV). The media can be:

- Audios
- Data
- Documents – Typically medical reports with extensions of DOC, Portable Document Format (PDF), Rich Text Format (RTF), and Text format (TXT)
- Images
 - Digital Imaging and Communications in Medicine (DICOM)
 - Non-DICOM, such as Joint Photographic Experts Group (JPEG) and Tag Image File Format (TIFF)
- Videos

To keep things simple, henceforth in this document, we use the term images to identify all media types unless a specific type is applicable. We also use the term VIX to mean CVIX or VIX.

This manual assumes you are familiar with web applications. This manual also assumes you are using the Viewer with the latest release level. The release level appears on this document's cover page and at the top of its Revision History page. Section 5 describes how to see the complete release number on the main Viewer page.

1.2. Document Orientation

1.2.1. Organization of the Manual

This document contains the following sections to assist you in using the Viewer:

- **Section 1 – Introduction**
Describes the document and helps you to follow the document.
- **Section 2 – System Summary**
Describes a summary of the Viewer.
- **Section 3 – Getting Started**
Describes a quick overview to start using the Viewer.

- **Section 4 – Using the Software**

Describes in-depth instructions how to use the Viewer. References the main and image navigation toolbars and buttons and explains how to close the Viewer. Describes how to use the Quality Assurance features of the Viewer.

- **Section 5 – Troubleshooting**

Describes how to access some of the information contained in the Viewer. The information assists you in troubleshooting some issues that might occur when using the Viewer.

- **Section 6 - Acronyms**

Defines the Acronyms and Terms used and referenced in this document.

1.2.2. Assumptions

The user has been provided the appropriate active roles, menus, and security keys required for the VIX Viewer. Also, the user has basic knowledge of the use of a web browser, menu options, and navigation tools.

1.2.3. Coordination

Use of the Viewer is subject to the following provisions:



Caution: Federal law restricts this device to use by or on the order of either a licensed practitioner or persons lawfully engaged in the manufacture or distribution of the product.



The Viewer is not intended for the primary interpretation of radiology exams. When the Viewer installs on approved and properly maintained hardware, primary interpretation of other image types is permissible by licensed practitioners at their discretion.



The Food and Drug Administration (FDA) classifies VistA Imaging and the Viewer (as a component of VistA Imaging) as a medical device. Unauthorized modification to VistA Imaging, including the Viewer, such as installing unapproved software, adulterates the medical device. The use of an adulterated medical device violates US federal law (21CFR820).

1.2.4. Disclaimers

This is a controlled document. No changes to this document may be made without the express written consent of the VistA Imaging Office of Enterprise Development group.

While every effort has been made to assure the accuracy of the information provided, this document might include technical inaccuracies and/or typographical errors. Changes are periodically made to the information herein and incorporated into new editions of this document. Product names mentioned in this document might be

trademarks or registered trademarks of their respective companies and are hereby acknowledged.

- VistA Imaging Office of Enterprise Development Department of Veterans Affairs
- Internet: <http://www.va.gov/imaging>
- Imaging VistA Imaging SharePoint **REDACTED**

1.2.4.1. Software Disclaimer


This software was developed at the Department of Veterans Affairs by employees of the Federal Government in their official duties. Pursuant to Title 17 Section 105 of the United States Code, this software is not subject to copyright protection and is in the public domain. VA assumes no responsibility whatsoever for its use by other parties and makes no guarantees, expressed or implied, about its quality, reliability, or any other characteristic. We would appreciate acknowledgment if the software is used. This software can be redistributed and/or modified freely if any derivative works bear some notice that they are derived from it, and any modified versions bear some notice that they have been modified.

1.2.4.2. Documentation Disclaimer

The appearance of external hyperlink references in this manual does not constitute an endorsement by the VA of the website or the information, products, or services contained therein. The VA does not exercise any editorial control over the information you may find at these locations. Such links are provided and are consistent with the stated purpose of the VA.

1.2.5. Documentation Conventions

This document uses the following conventions:

- Button names, menu names, and menu option names are shown in **Bold**.
- A vertical bar is used to separate menu choices. For example: "Click **File | Open**" means: "Click the **File** menu, then click the **Open** option."
- Keyboard shortcuts are shown like **Ctrl+C** which means to press the control (Ctrl) key on your keyboard, keep it pressed, and press the C key.
- When this document is used online, hyperlinks are indicated by [blue text](#).
- Useful or supplementary information is shown in a **Tip**. Important or required information is shown in a **NOTE**.
- Critical information is indicated by: 

1.2.6. References and Resources

Additional information about VistA Imaging can be found at:

- VistA Imaging: <http://www.va.gov/vdl/application.asp?appid=105>

1.3. Enterprise Service Desk and Organizational Contacts


If you encounter any problems using the Viewer, contact your local Imaging Coordinator or support staff. If the problem cannot be resolved locally, use YourIT Service Desk Manager to place a service request, or contact the Enterprise Service Desk at **REDACTED**.

2. System Summary

Vista Imaging provides formal authorization to a web application to embed, or host, the Image Viewer. The web-hosting application authorizes users to use its own functions as well as the functions in the Image Viewer.

Web-hosting applications, such as the JLV, embed the Image Viewer (Figure 1) in the host application's web page. The Image Viewer communicates with the CVIX/VIX application. The host application might also directly communicate with the CVIX/VIX application. The Image Viewer operates inside the VA network.

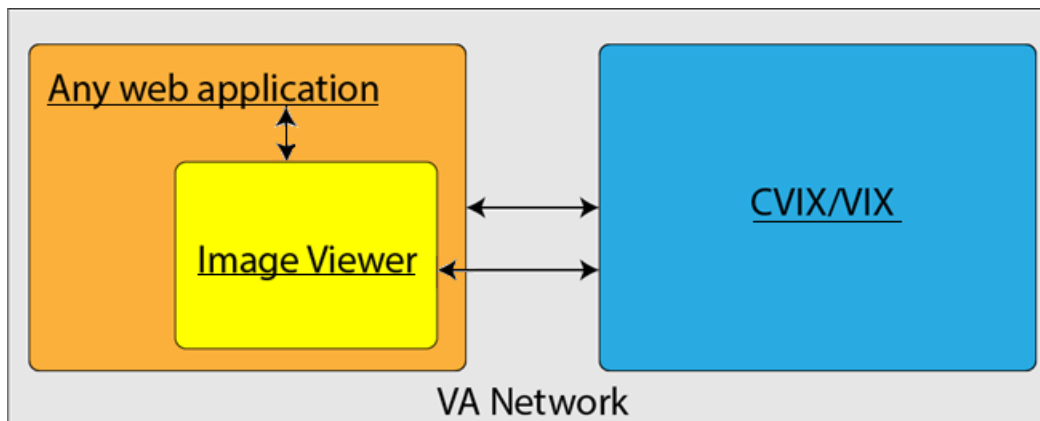Figure 1: Image Viewer System Summary



2.1. System Configuration

The Viewer adds image display capabilities to web applications in a similar way as the Clinical Display desktop application adds image display capabilities to the Computerized Patient Record System (CPRS).

The Viewer is a web-based application that provides a subset of features of Clinical Display. This means it does not supply all features that Clinical Display offers.

2.2. Data Flows

The Viewer is a web application that enables you to access patient images using VA approved versions of Chrome or Edge.

The Viewer allows users to locate and display any patient images available to the VA from Vista Imaging, including Department of Defense (DoD) images.

2.3. User Access Levels

Users use a service account in Vista with the DICOM VISA [MAG DICOM VISA] and CPRSChart [OR CPRS GUI CHART] secondary menu options, at a minimum, to access the Image Viewer through the web application.

2.4. Continuity of Operation

Since the Viewer is embedded in another application, the Viewer relies on the other application's continuity of operation.

3. Getting Started

After logging into the web-hosting application and selecting a patient study, the hosting application launches the Viewer (Section 4.1) to display the study on its main page (Section 4.2).

3.1. Logging On

To access the Viewer and authenticate, the user logs on through the web-hosting application. The hosting application logs in users to authenticate their credentials by getting a security token. This token is used throughout the Viewer's web pages.

3.2. Exit System

To exit the system, close the Viewer's web pages that are open in the web browser tab(s).

3.3. Caveats and Exceptions

If encountering issues, clear the cache in the web browser for the Viewer, and refresh the page.

4. Using the Software

This section describes how to use the Viewer web application.

4.1. Launching the Viewer

Upon launching the Viewer, the following loading message (Figure 2) displays while the Viewer loads the study's images:

Figure 2: Loading Message




NOTE: The VA's and your network's bandwidth dictate the amount of time it takes to transfer the images. Some images can be very large which usually means they take a while to load, sometimes even minutes. The faster the bandwidth, the faster the computer can transfer data.

The first time a study launches, the Viewer pre-processes the study's images and loads them into a cache on the VIX server. Caching images reduces the load time for later subsequent views.

If there is trouble loading one or more images, the following message displays (Figure 3) with the option to either proceed or empty the cache and try again. If you select proceed, the study loads without the problem images. If you choose to empty the cache and retry, the Viewer attempts to re-process all the images for the study.

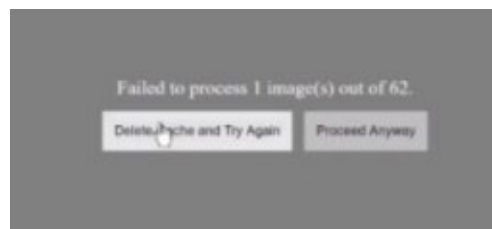
Figure 3: Images Not Loading



4.2. The Main Viewer Page

When the main Viewer page opens, it shows the patient information, main toolbar, external links button, thumbnail panel(s), viewport(s), image navigation toolbar (Section 4.2.2), status, and Viewer version number (Figure 4). The Viewer displays thumbnail placeholders until the thumbnails are retrieved, then displays the images in the viewport(s). Section 4.3.15 describes the components that appear and are enabled according to the active viewport's image as indicated by a yellow outline.

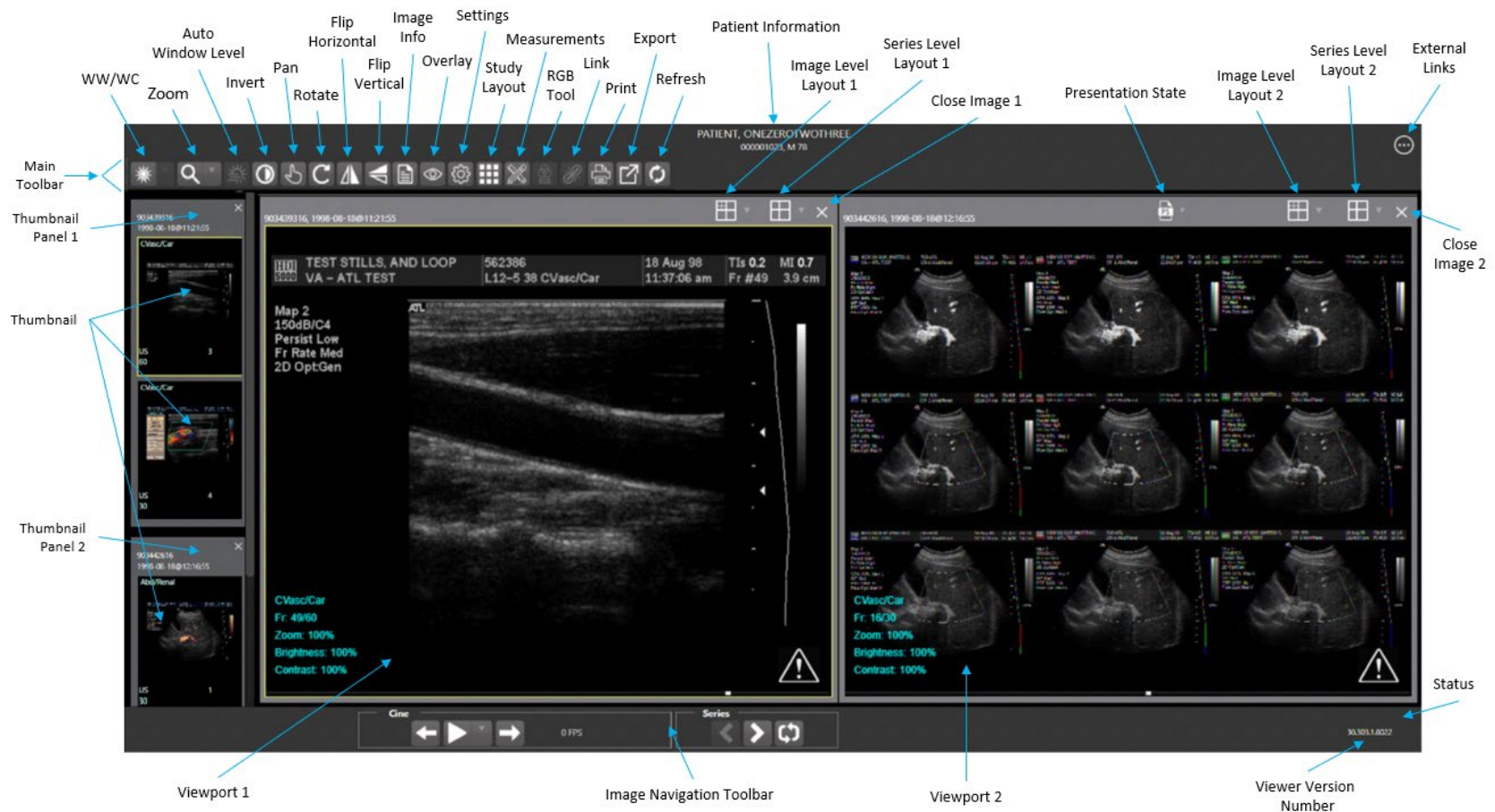Figure 4: Main Viewer Page



4.2.1. The Main Toolbar

The main toolbar (Figure 5) provides buttons to access the Viewer functions while viewing an image. The Viewer enables (brightens) and disables (grays out, Figure 6) the buttons depending on the image type in the active viewport. See Figure 5 for an image of the toolbar.

Figure 5: Main Toolbar

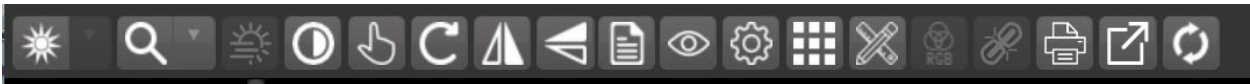











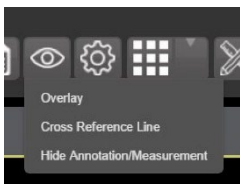


Figure 6: Sample Disabled Toolbar Button









Table 1 describes the functions and uses of each button in the toolbar.

Table 1: Toolbar Icons - Names, Functions, and Descriptions

Button Icon	Button Name and Function	Description
	WW/WC (Window Level)	Manually click and drag an image to make the Viewer alter the grayscale value of the image. For Computerized Tomography (CT) images, a preset dropdown menu displays with the following options: Default (the image as initially loaded and displayed), Abdomen, Bone, Brain, Head/Neck, Lung, and Customize.
	Zoom	Click and drag on an image to zoom in and out. A preset dropdown menu provides the options to resize the image to 100%, or Fit to Window, or Fit Width to Window, or Fit Height to Window, or to Customize the sizing.
	Auto Window Level	Click and drag a rectangular area on an image to adjust the window level based on the image's minimum pixel value.

Button Icon	Button Name and Function	Description
	Invert	Invert an image so that dark is light, and light is dark.
	Pan	Click and drag an image to move its viewing area.
	Rotate	Rotate an image 90 degrees clockwise.
	Flip Horizontal	Flip an image horizontally.
	Flip Vertical	Flip an image vertically.
	View Image Info	Display an image's DICOM attributes and metadata.
	Overlay	Toggle the overlay on and off. The Cross Reference Line option toggles the scout lines on and off. The Hide Annotation/Measurement option toggles the annotations and measurements on the image. The 6000 Overlay option appears when the image contains a 6000 Overlay and toggles its display on and off.
	Settings	Set the Electrocardiogram (ECG), Annotation, Cine, and Layout Preferences. For more information on the settings, see Sections 4.3.3 and 4.3.10.
	Study Level Layout	Set the layout for displaying the study's viewports to 1x1, 1x2, 1x3, 2x1, 2x2, 2x3, 3x1, 3x2, or 3x3 tiles.

Button Icon	Button Name and Function	Description
	Measurements	Measure the 2D Length, the Angle Tool, 2D Point, 2D Length Calibration, or Delete All according to Section 4.3.13.
	RGB Tool	Access the Red Green Blue (RGB) Tool as described in Section 4.3.14.
	Link	Select two image stacks and scroll.
	Print	Access the Print Tool as described in Section 4.3.7.
	Export	Access the Export Tool as described in Section 4.3.7.2.
	Refresh	Refresh the page to the original settings without reloading the images from the server.

4.2.2. Thumbnails and Image Loading

The following describes the steps that occur when the main Viewer page displays:

1. The thumbnail panel(s) display on the left side of the page.
2. The Viewer selects the first thumbnail, indicated by a yellow outline.

3. The Viewer displays the loading progress (Figure 7).
4. The Viewer displays the loaded image associated with the thumbnail in the viewport.
5. After the above steps are complete, you can click the thumbnail for your desired image if it isn't already selected, and its image displays in the viewport. Section 4.3 describes how to work with images.

Figure 7: Main Viewer Page with Thumbnail Panel

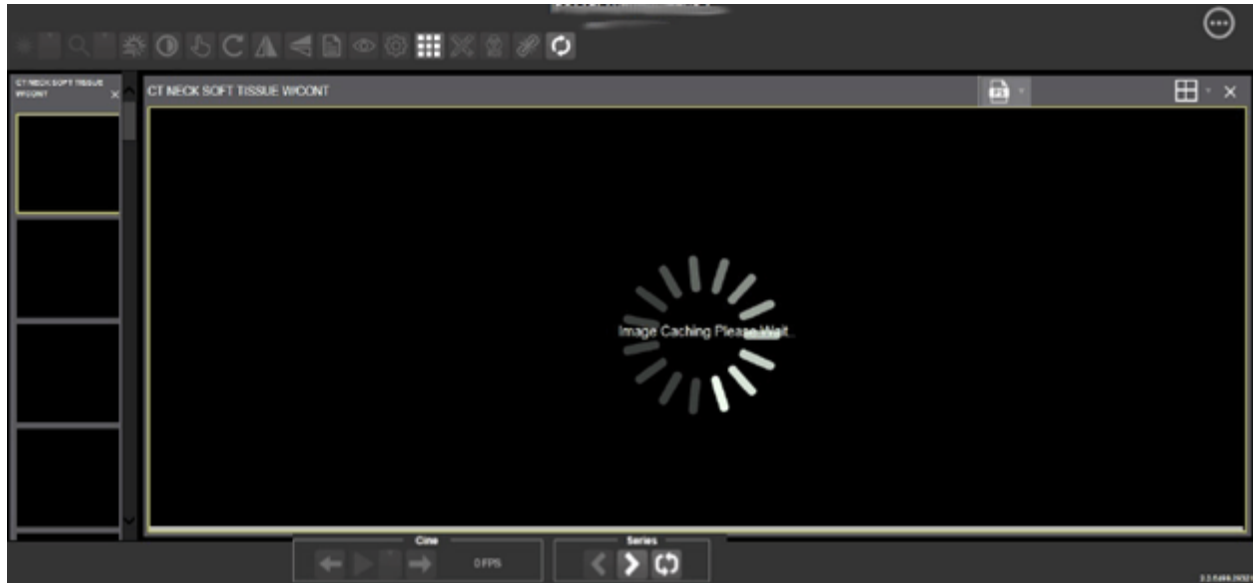
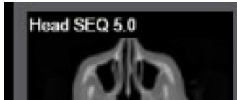



Table 2 describes the thumbnail elements.

Table 2: Thumbnail Elements, Names, Functions, and Descriptions

Web Page Element	Element Name and Function	Description
	Thumbnail	The title of the series appears at the top left of the thumbnail. The image type appears at the bottom left, with the number of images in the series directly below that. The series number displays on the bottom right of the thumbnail. For DICOM images, the modality code appears on the bottom left.

Web Page Element	Element Name and Function	Description
	Thumbnail Panel	To adjust the size of the thumbnail panel, click and drag the right edge of the panel to the left or right. This affects how many thumbnails are shown per row within the panel but does not alter the size of the thumbnails. When hovering over the panel with stacked images, if you have a mouse wheel, you can use it to scroll between the different series thumbnails. Use drag-and-drop to drag thumbnails to the viewport. A yellow outline appears around the thumbnail that currently shows in the viewport.

4.2.3. The Image Navigation Toolbars

An image navigation toolbar appears automatically when the Viewer determines it applies to the image displayed in the active viewport. Figure 8 shows samples of these toolbars.

Figure 8: Sample Image Navigation Toolbars

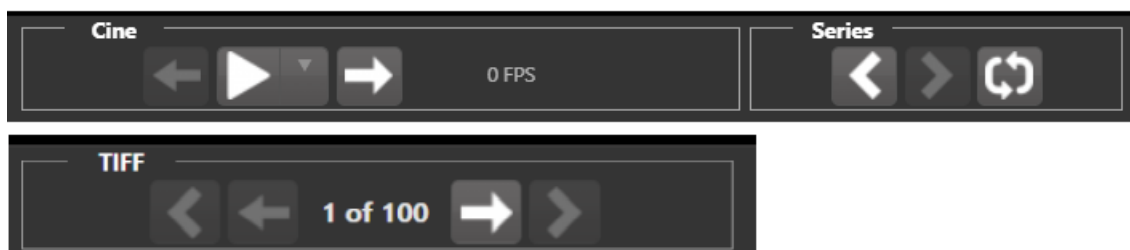






Table 3 describes the functions and uses of each button in the image navigation toolbar.

Table 3: Functions and Uses of the Image Navigation Toolbar Buttons

Button Icon	Button Name and Function	Description
	Previous Series and Next Series	Navigate through the study's series. Alternatively, you can click the thumbnail for the series to view.
	Previous Image and Next Image	Navigate through the active series' or TIFF's images. If you have a mouse wheel, you can also use it to scroll through the frames when you hover over the viewport.

Button Icon	Button Name and Function	Description
	Play and Stop	Begin and end playing through a cine image scene. The down arrow changes forward or backward direction.
	Repeat	Select to repeat the series. Deselect to play through the series before repeating the series.

While a series plays, a number (Figure 9) appears in the navigation toolbar. The number is the frame rate the series is currently playing.

Figure 9: Frame Rate



- **NOTE:** The first time a series plays, it might play slower while caching the images. Once the images are cached, the series plays faster.

4.3. Working with Images

If the image in the active viewport is not the one you desire, you can:




- Click another viewport, if any.
- Click the thumbnail for your desired image, and its image displays in the viewport.

The remainder of this section explains how to work with images.

4.3.1. Manipulating View Layout

Table 4 describes the buttons and functions to change how the images display in the viewport.

Table 4: Viewing Images and Manipulating View Layout

Button Icon	Button Name and Function	Description
	Image Level Layout	Manipulates the number and position of the series' images that can display on the viewport.
	Series Level Layout	Manipulates the number and position of the study's series that can display on the viewport.
	Close Image	Closes the currently selected image from the viewport.

4.3.2. Viewing DICOM Images


The Viewer displays DICOM images by simulating Picture Archiving and Communication System (PACS) functionality. For example, when a CT scan displays, you can manipulate its window level, zoom, annotate, and so on as described in Sections 4.2.1 and 4.2.3. You can work with other modalities in a similar fashion, but they do not support window leveling.

4.3.2.1. Viewing Stacked Images

Stacked images (such as for the CT, MR, and US modalities) represent a series, each with its own thumbnail in the thumbnail panel. When viewing a stacked image series and you have a mouse wheel, you can use it to scroll between the images in the stack when hovering over the thumbnail panel. You can also scroll through the stack by clicking the **Series** buttons described in Section 4.2.2.

Upon scrolling, if multiple series display in the viewport, scout lines appear on the images of the other series in a different plane to indicate what part of the body the series displays as shown in Figure 10 below.

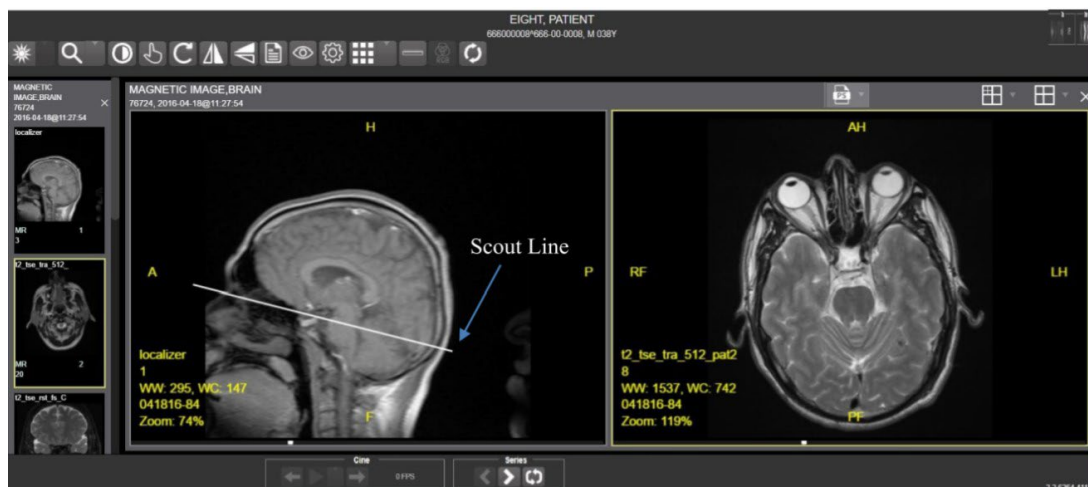Figure 10: Main Viewer Page with Images Loaded



4.3.3. Viewing an Electrocardiogram (ECG/EKG)


Electrocardiograms, typically referred to as ECGs in the United States and EKGs in Europe, measure the electrical activity of the heartbeat. Sometimes people refer to ECGs as 12-Lead ECGs, because the standard ECG involves medical staff placing 10 electrodes in precise locations on the patient's body to obtain 12 different perspectives (leads). Each lead is a graphical representation of the electricity in one or more electrodes.

An electrical impulse (or “wave”) travels through the heart with each beat. This electrical wave causes the muscle to squeeze and pump blood from the heart. A normal heartbeat on an ECG shows the rate and rhythm of the contractions in the upper and lower chambers.

The Viewer displays three types of ECG files: bitmap (such as JPEG), PDF, and DICOM. The bitmap and PDF renditions are simply snapshots of the ECG reading and offer little functionality compared to what you can do with DICOM files.


With a DICOM ECG, you can right-click on the grid to display a context menu as shown in Figure 11 below. The context menu lets you modify the following characteristics: Grid Type, Grid Color, Lead Format, Gain, Signal Thickness, Hide Caliper, Print, and Export.

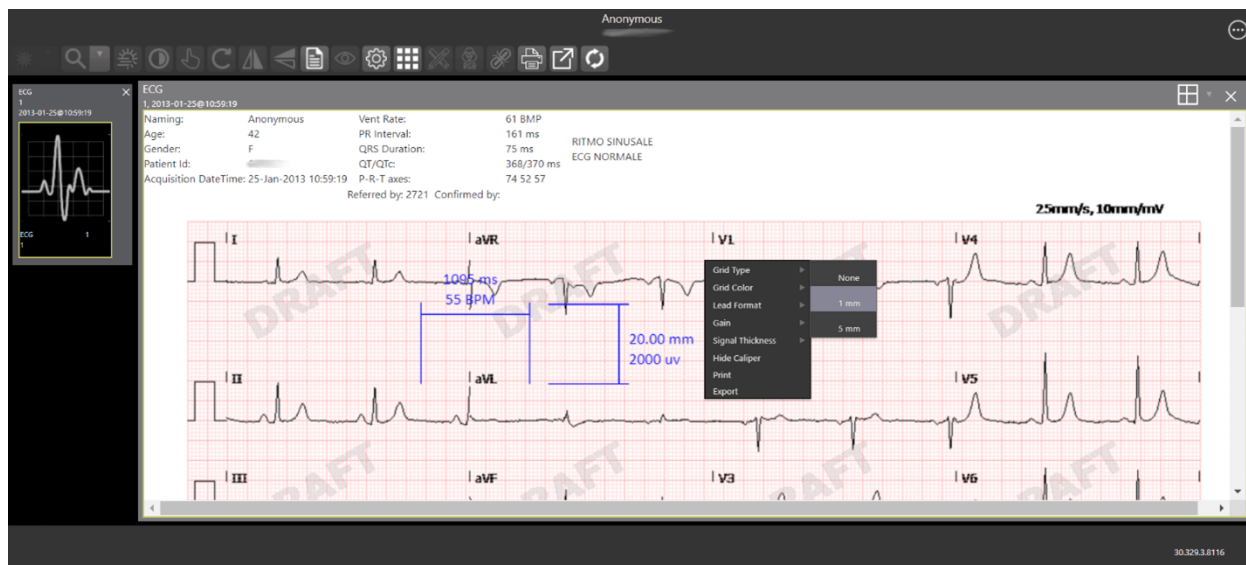Figure 11: DICOM ECG with Context Menu



Click the **Settings** main toolbar icon to change the ECG Preferences (Figure 12). After changing the settings, click the **Apply** button, and click the **Cancel** button after the update message leaves.

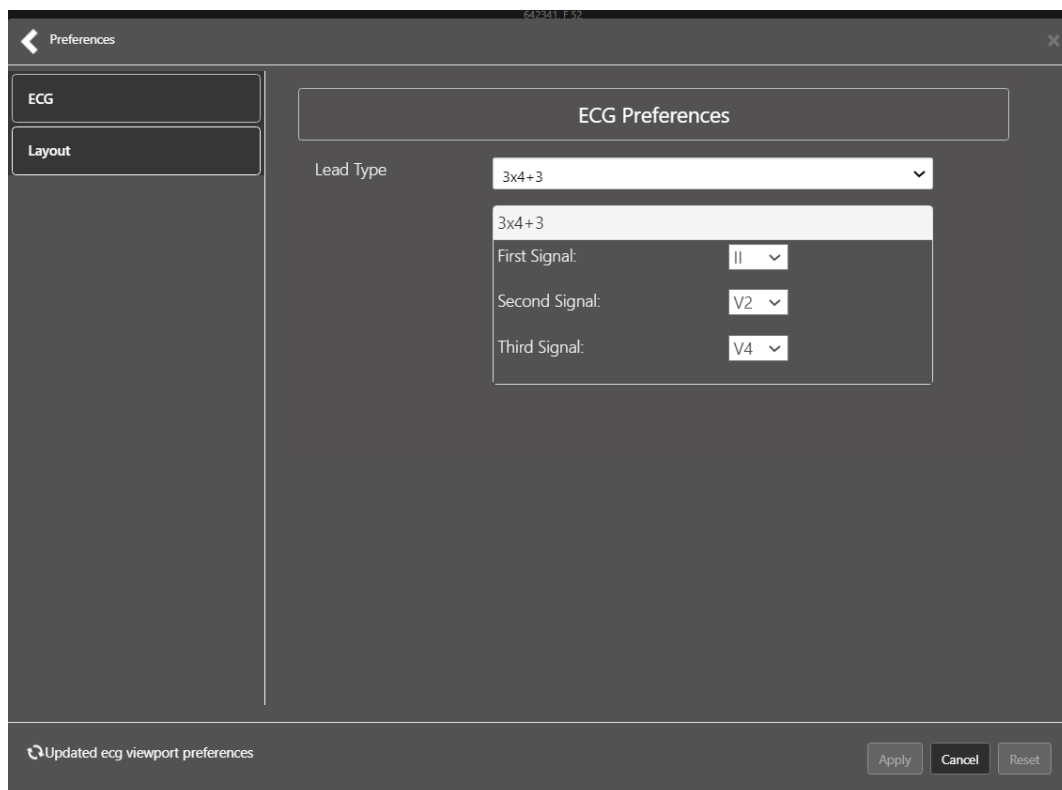
Figure 12: ECG Preferences



4.3.4. Viewing TIFF Images


When an image is in the TIFF format (one or multiple pages), the Viewer converts each page to the PNG format to display it easily and quickly in the browser. Meanwhile, behind the scenes, the Viewer converts the entire TIFF file to PDF, sets the status to indicate the PDF is not yet ready (Figure 13), and disables the **Print** and **Export** main toolbar icons.

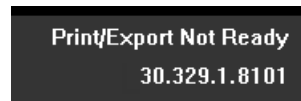Figure 13: PDF is Not Ready Status in the Lower Right Corner



When the PDF is ready, the Viewer updates the status (Figure 14), enables the **Print** and **Export** main toolbar icons, and displays **Print** and **Export** options on the context menu as described in Sections 4.3.7 and 4.3.7.2.

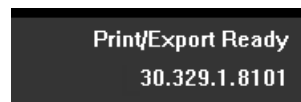Figure 14: PDF is Ready Status in the Lower Right Corner



You can use the **Next Image** and **Previous Image** navigation toolbar icons to move the displayed images forward and backward as described in Section 4.2.3.


4.3.5. Viewing Normal Web Images

The Viewer enables most main toolbar icons when displaying normal web images such as JPEG and single-page TIFFs. Window leveling, however, does not apply.

4.3.6. Viewing Documents

When viewing a document, the document viewport displays (Figure 15), and the Viewer disables most of the main toolbar icons. The display might look different than shown depending on the browser.

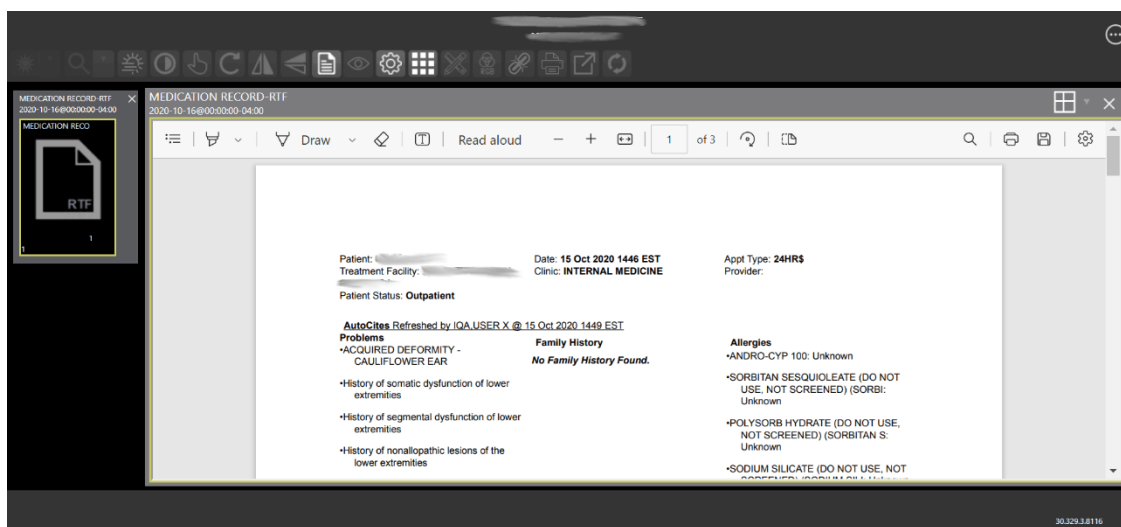Figure 15: Main Viewer Page with a Document Loaded



4.3.7. Printing and Exporting Images

Table 5 shows when you can print and export images. Use either the **Print** and **Export** main toolbar icons, context menu items, or the PDF Viewer plugin to print or export images.

Table 5: Image Print/Export Behavior

Media Type	Print/Export Main Toolbar Icon	Print/Export Context Menu Items	Note
Audio and Video	Disabled	Not shown	Such as Audio Video Interleave (AVI), Compact Disc Audio (CDA), QuickTime Movie Format (MOV), Media Picture Experts Group 4 (MP4).
DICOM (not with embedded PDF)	Enabled	Shown	N/A
Documents (DOC, PDF, Rich RTF, TXT)	Disabled	Not shown	Use the PDF Viewer plugin.
Embedded PDFs in DICOM	Disabled	Not shown	Use the PDF Viewer plugin.
Non-DICOM and Non-TIFF Images	Enabled	Shown	Such as Bitmap (BMP), JPEG, Graphics Interchange Format (GIF), and Portable Network Graphics (PNG).
TIFF	Enabled only when its converted PDF is ready.	Shown only when its converted PDF is ready.	See Section 4.3.4.


4.3.7.1. Printing an Image

To print an image, do one of the following:

- Click the **Print** main toolbar icon.

- Right-click on the image to display a context menu. See Figure 16 below for an example.

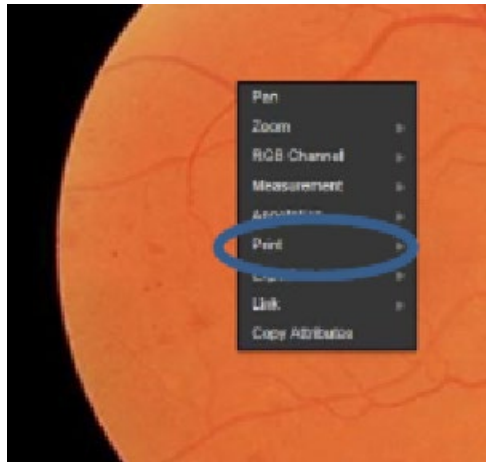Figure 16: Print Menu



After selecting to print, you might be prompted for a print reason and your digital signature. You can select print options. After selecting the print options, the image opens in a new tab to preview and select the device. See Figure 17 below for an example.

Figure 17: Print Menu - Device Selection


4.3.7.2. Exporting an Image

To export an image, do one of the following:

- Click the **Export** main toolbar icon.

- Right-click on the image to display a context menu. See Figure 18 for an example.

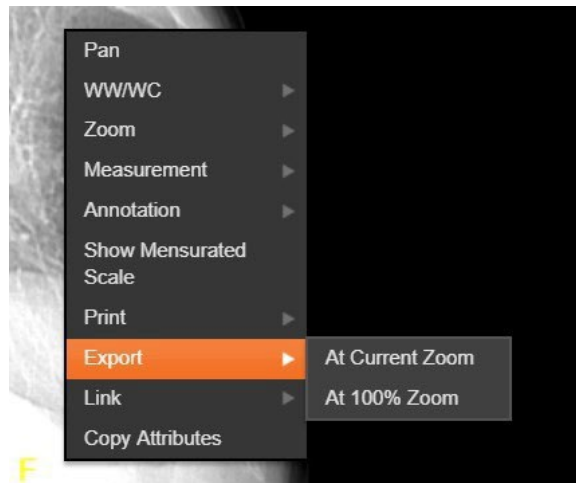Figure 18: Exporting an Image



After selecting the desired zoom, you might be prompted for your digital signature before exporting the image using the browser.


4.3.8. External Links

Click the button in the top right corner (Figure 19) of the Viewer page to view the **External Links** menu (Figure 20).

Figure 19: External Links Button in the Top Right Corner



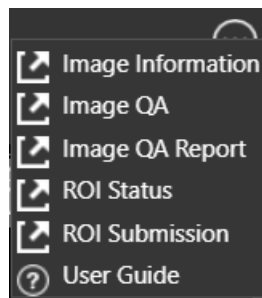Figure 20: External Links Menu



4.3.8.1. Release of Information (ROI) Print Queue Status and ROI Submission

To view the ROI Print Queue Status, click **ROI Status** on the **External Links** menu, and the following screen displays (Figure 21).

Figure 21: ROI Print Queue Status

Patient Name	SSN4	Patient ID	Status	Last Updated	DICOM Routing
			EXPORT_QUEUE	06/23/2022@16:06:52	ROI TEST - CD BURNER, 500
			EXPORT_QUEUE	06/17/2022@15:33:39	ROI CD WRITER, 994
			EXPORT_QUEUE	06/22/2022@10:09:24	ROI TEST - CD BURNER, 500
			EXPORT_QUEUE	06/28/2022@14:52:49	ROI TEST - CD BURNER, 500
			EXPORT_QUEUE	07/09/2022@16:47:35	ROI TEST - CD BURNER, 500
			EXPORT_QUEUE	07/21/2022@12:11:34	ROI TEST - CD BURNER, 500
			EXPORT_QUEUE	12/08/2022@16:07:11	ROI CD WRITER, 994
			EXPORT_QUEUE	01/22/2019@02:15:33	ROI TEST - CD BURNER, 500
			EXPORT_QUEUE	01/22/2019@02:16:16	ROI CD WRITER, 994
			EXPORT_QUEUE	01/22/2019@02:17:38	

To perform ROI Submissions, click **ROI Submission** on the **External Links** menu and the following screen displays (Figure 22).

Figure 22: Print Queue

Procedure	Date	Location	Release
RAD CT	2018-08-28T16:44:00-04:00	VA	<input type="checkbox"/>
	2018-08-28T16:44:00-04:00	VA	<input type="checkbox"/>
	2018-08-28T16:44:00-04:00	VA	<input type="checkbox"/>
CLIN	2018-06-25T00:01:00-04:00	VA	<input type="checkbox"/>
RAD PR	2017-10-11T13:02:00-04:00	VA	<input type="checkbox"/>
XRAY	2017-10-11T13:02:00-04:00	VA	<input type="checkbox"/>
CLIN	2017-09-26T00:01:00-04:00	VA	<input type="checkbox"/>
CON/PROC	2017-09-20T10:44:00-04:00	VA	<input type="checkbox"/>
CON/PROC	2017-09-20T10:44:00-04:00	VA	<input type="checkbox"/>
RAD CT	2017-09-12T18:43:00-04:00	VA	<input type="checkbox"/>


To print:

- Select one or more items and click **Submit ROI CD Request**, where CD stands for Compact Disc.

4.3.10. Changing the Settings of an Image


Select the **Settings** main toolbar icon to view or update the settings options (Figure 25).

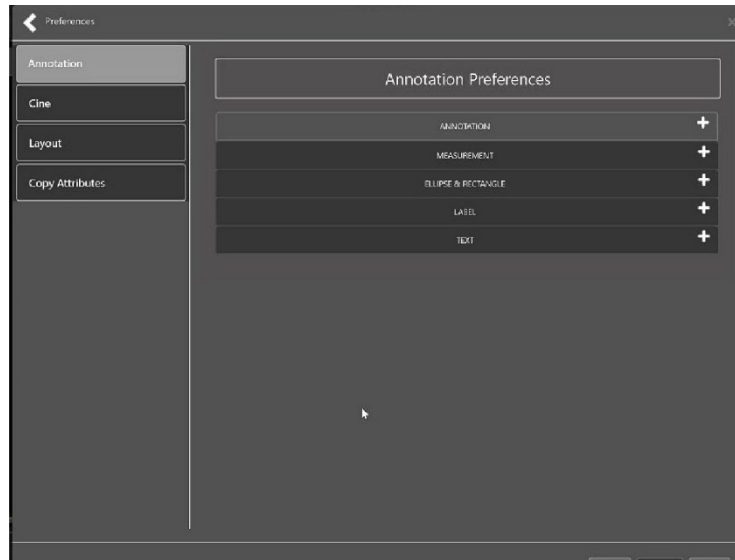Figure 25: Settings Menu




Click **Annotation**, and the following screen displays (Figure 26).

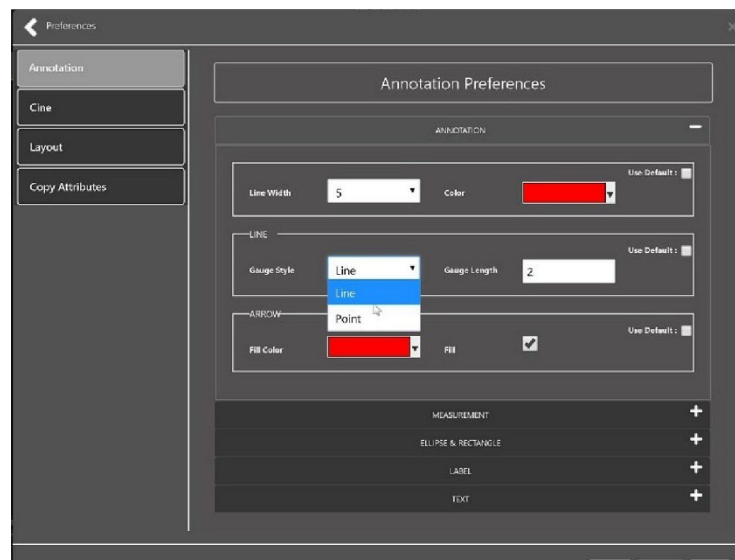Figure 26: Annotations Preferences



Click **Measurement**, and the following screen displays (Figure 27).

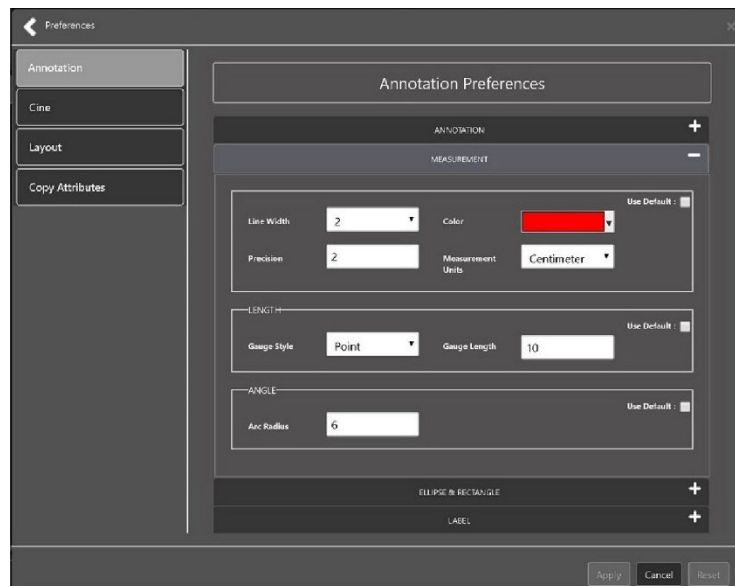
Figure 27: Measurement Preferences



The 'Measurement Preferences' dialog box is shown with the 'MEASUREMENT' section expanded. The left sidebar contains 'Annotation', 'Cine', 'Layout', and 'Copy Attributes'. The main area has sections for 'ANNOTATION', 'MEASUREMENT', 'LENGTH', 'ANGLE', 'ELLIPSE & RECTANGLE', and 'LABEL'. The 'MEASUREMENT' section includes 'Line Width' (2), 'Precision' (2), 'Color' (red), and 'Measurement Units' (Centimeter). The 'LENGTH' section includes 'Gauge Style' (Point) and 'Gauge Length' (10). The 'ANGLE' section includes 'Arc Radius' (6). At the bottom are 'Apply', 'Cancel', and 'Reset' buttons.

Click **Ellipse and Rectangle**, and the following screen displays (Figure 28).

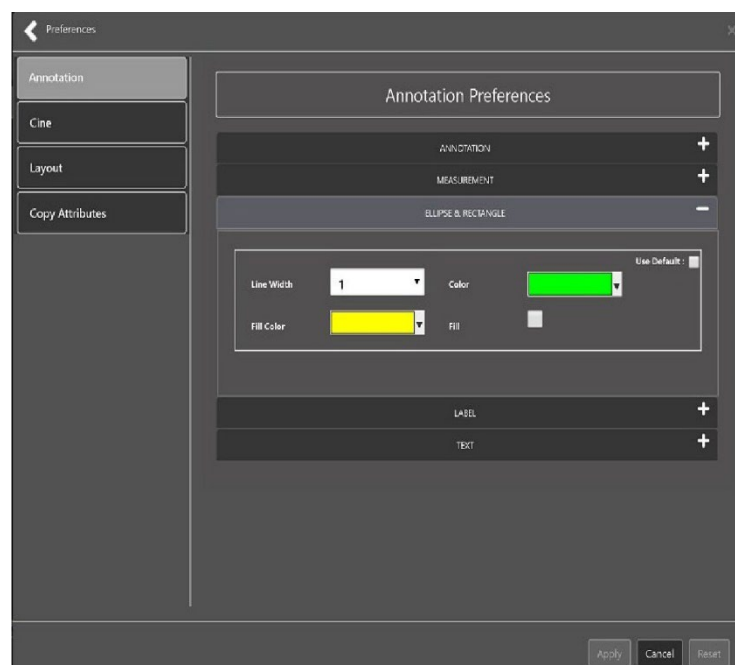
Figure 28: Ellipse and Rectangle Preferences



The 'Measurement Preferences' dialog box is shown with the 'ELLIPSE & RECTANGLE' section expanded. The left sidebar is the same. The main area shows 'ANNOTATION', 'MEASUREMENT', 'ELLIPSE & RECTANGLE', 'LABEL', and 'TEXT'. The 'ELLIPSE & RECTANGLE' section includes 'Line Width' (1), 'Color' (green), 'Fill Color' (yellow), and 'Fill' (checkbox). At the bottom are 'Apply', 'Cancel', and 'Reset' buttons.


Click **Label**, and the following screen displays (Figure 29).

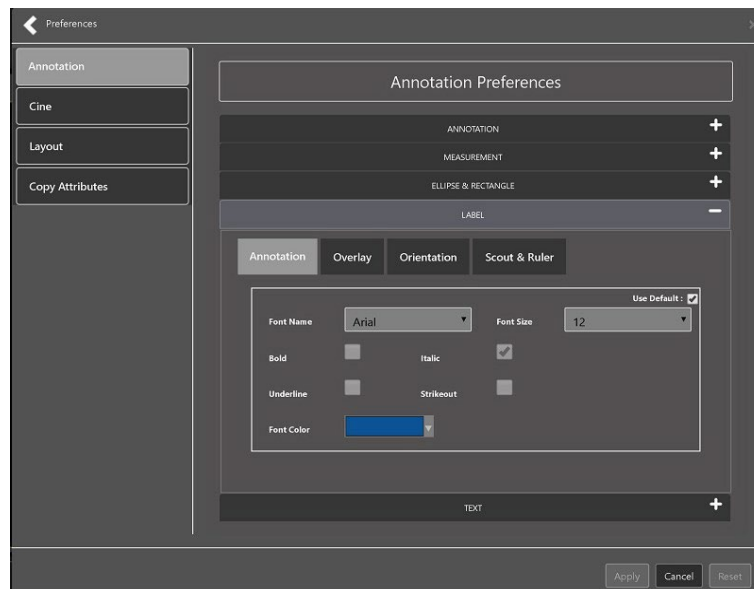Figure 29: Label Preferences



Under Label preferences, choose Annotation, Overlay, Orientation, or Scout & Ruler. Click **Text**, and the following screen displays (Figure 30).

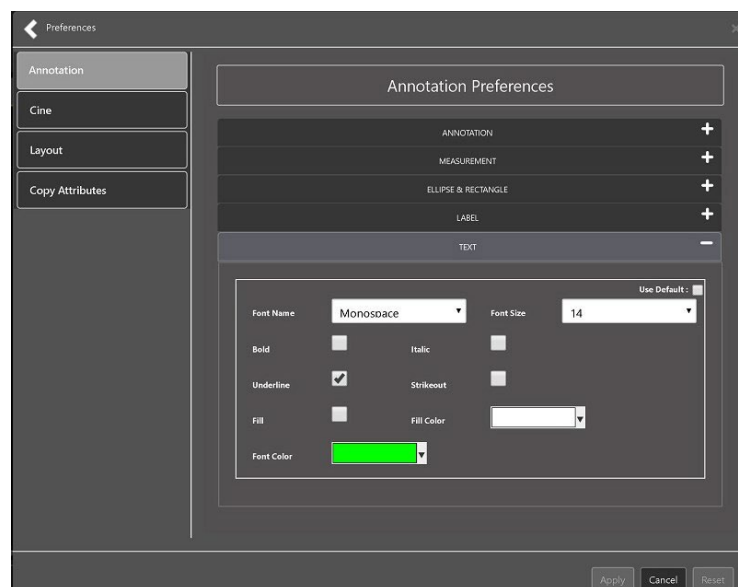Figure 30: Annotation Preferences



To view or update the Cine Settings, select the **Cine Preference** and the following screen displays (Figure 31).

Figure 31: Cine Preferences

The screenshot shows the 'Cine Preferences' dialog box. On the left, a sidebar contains four buttons: 'Annotation', 'Cine' (which is highlighted), 'Layout', and 'Copy Attributes'. The main panel is titled 'Cine Preferences' and contains the following settings:

- Cine mode:** A dropdown menu set to 'Stack'.
- Repeat cine run:** A text input field containing the number '3', with '(times)' written below it.
- when the number of frames in a cine loop are less than or equal to:** A text input field containing the number '9'.
- When looping through a study, pause single frame images for:** A text input field containing the number '3', with '(seconds)' written below it.

At the bottom right of the main panel is a checkbox labeled 'Use Default'. At the bottom of the dialog box are three buttons: 'Apply', 'Cancel', and 'Reset'.

To view the Layout settings, select the **Layout Preference** to display the following screen (Figure 32).

Figure 32: Layout Preferences


The screenshot shows the 'Layout Preferences' dialog box. On the left, a sidebar contains four buttons: 'Annotation', 'Cine', 'Layout' (which is highlighted), and 'Copy Attributes'. The main panel is titled 'Layout Preferences' and contains a table with the following data:

Modality	Rows	Columns	Zoom Mode	Actions
MR	1	1	100%	<input checked="" type="checkbox"/>

At the bottom right of the main panel is a checkbox labeled 'Use Default'. At the bottom of the dialog box are three buttons: 'Add', 'Apply', and 'Cancel'.

Click **Add** in the bottom right of the page to add a new layout preference (Figure 33).

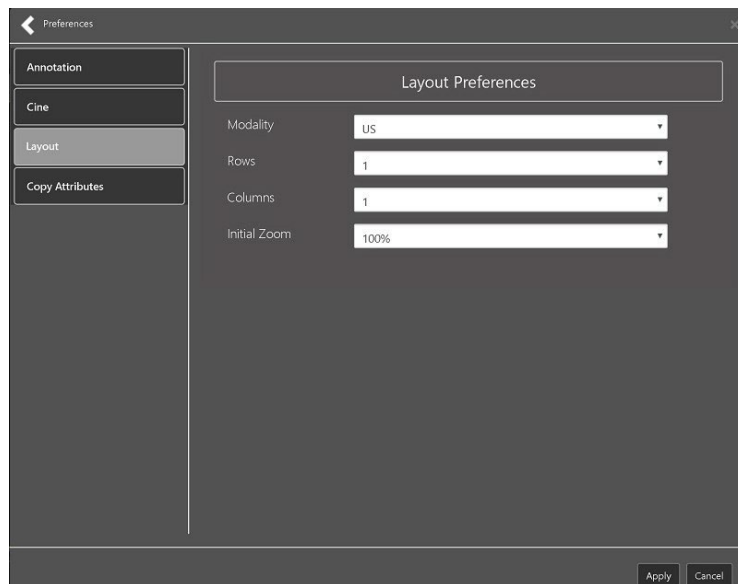
Figure 33: Layout Preference Choices



The screenshot shows a 'Preferences' window with a sidebar on the left containing four items: 'Annotation', 'Cine', 'Layout', and 'Copy Attributes'. The 'Layout' item is selected and highlighted. The main area of the window is titled 'Layout Preferences' and contains four settings, each with a dropdown menu: 'Modality' is set to 'US', 'Rows' is set to '1', 'Columns' is set to '1', and 'Initial Zoom' is set to '100%'. At the bottom right of the window are 'Apply' and 'Cancel' buttons.

To view the Copy Attributes settings, select the **Copy Attributes Preference**, and the following screen displays (Figure 34).

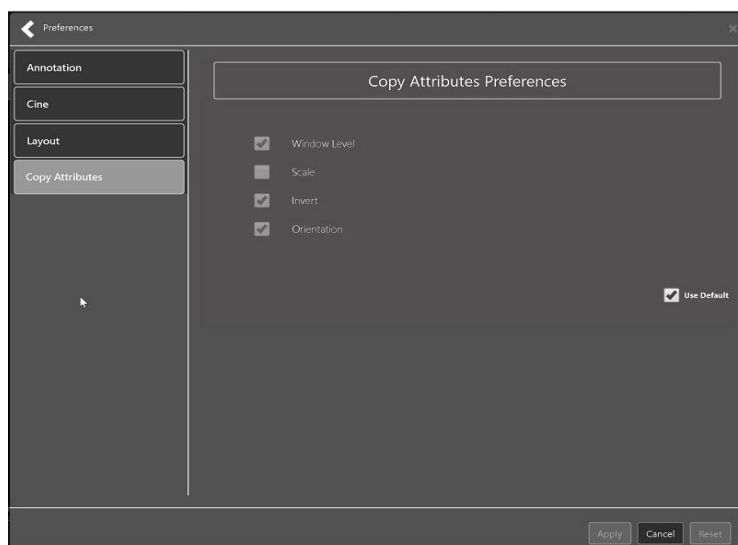Figure 34: Copy Attributes Preferences




The screenshot shows the same 'Preferences' window, but now the 'Copy Attributes' item in the sidebar is selected and highlighted. The main area is titled 'Copy Attributes Preferences' and contains four checked checkboxes: 'Window Level', 'Scale', 'Invert', and 'Orientation'. There is also an unchecked checkbox labeled 'Use Default' at the bottom right. At the bottom of the window are 'Apply', 'Cancel', and 'Reset' buttons.

4.3.11. Changing the Study Layout

To change the study layout, select the **Study Layout Button**, and the following screen displays (Figure 35).

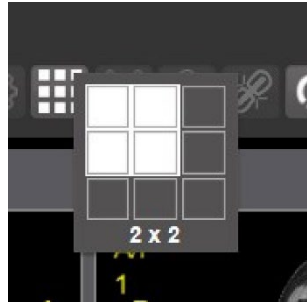Figure 35: Layout Attribute Preference Choices



Select the **study level** that best suits your purpose.


4.3.12. Presentation State

To view presentation state options, click the **PS Icon** (Figure 36), and the following menu appears (Figure 37).

Figure 36: Presentation State Button



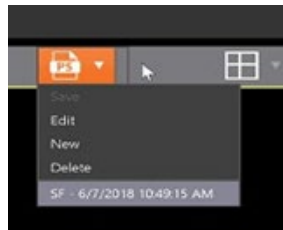Figure 37: Presentation State Menu



The presentation state icon is available only if there are presentation states associated with the study and there is an annotation drawn on the image. The bottom of the context menu displays all the saved presentation states associated with the study. To view additional information about a specific presentation state, hover over it in the context menu.

4.3.13. Using the Measurements Tools

To use the various measurement tools, select the **Measurements** button, and the following screens display depending on the option you choose (Figure 38, Figure 39, and Figure 40).

Figure 38: Measurement Tool View 1



Figure 39: Measurement Tool View 2


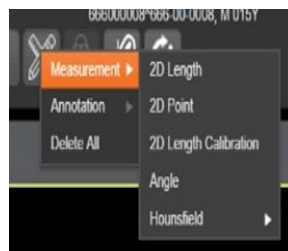


Figure 40: Measurement Tool View 3

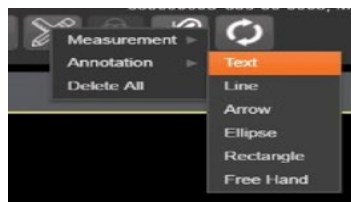


The **2D Length** option draws the necessary line(s) to obtain the desired measurements.

The **Angle Tool** draws the necessary line(s) to measure the desired angle.

The **2D Point** tool places the necessary dots to obtain the desired measurements. Use this option for ultrasound studies to measure velocity.


The **2D Length Calibration** option calibrates measurements to ensure accurate data. Images with DICOM info use the DICOM info to inform measurement calibration, but images without this data need calibration to show accurate data. See Figure 41 below.

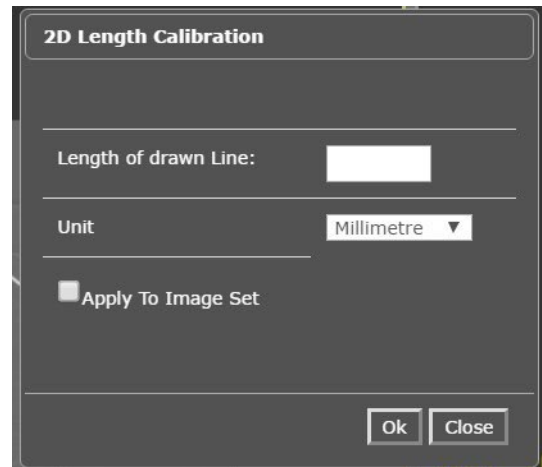Figure 41: 2D Length Calibration



The **Hounsfield** tool takes circular and rectangular density measurements. See Figure 42 below.

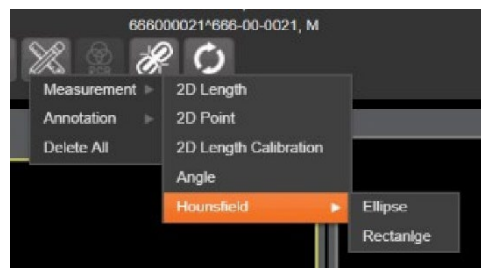Figure 42: Hounsfield Measurement Tools



The **Text** tool creates text boxes on the image.

The **Line** tool draws straight lines on the image.

The **Arrow** tool draws straight arrows on the image.

The **Ellipse** tool draws ellipses on the image.

The **Rectangle** tool draws rectangles on the image.

The **Free Hand** tool connects points that you have specified to create any desired shape.

To delete all the measurements and markings you made on the current image, select the **Delete All** option.

4.3.14. Using the RGB Tool

The **RGB Tool** button filters the image for certain color channels to allow for various observations. When selected, a context menu (Figure 43) appears that allows you to select the desired color channel.

Figure 43: RGB Tool Button



4.3.15. Conditions for Image Viewer Functions

Table 6 describes the Viewer functions dependent on certain conditions.


Table 6: Conditional Image Viewer Functions

Function	Section Reference	MAG Keys
Delete an image	Not applicable at this time	MAG DELETE or MAG SYSTEM
Edit an image	4.4.3	MAG EDIT, or MAG QA REVIEW, or MAG SYSTEM
Image navigation toolbar	4.2.2	Not applicable
Main toolbar buttons	4.2.1	Not applicable
Print and Export an image	4.3.7 and 4.3.7.2	Not applicable
Status	4.3.4	Not applicable

4.4. Image Quality Assurance (QA)


To use the image quality assurance features, load an image in the Viewer (Figure 44).

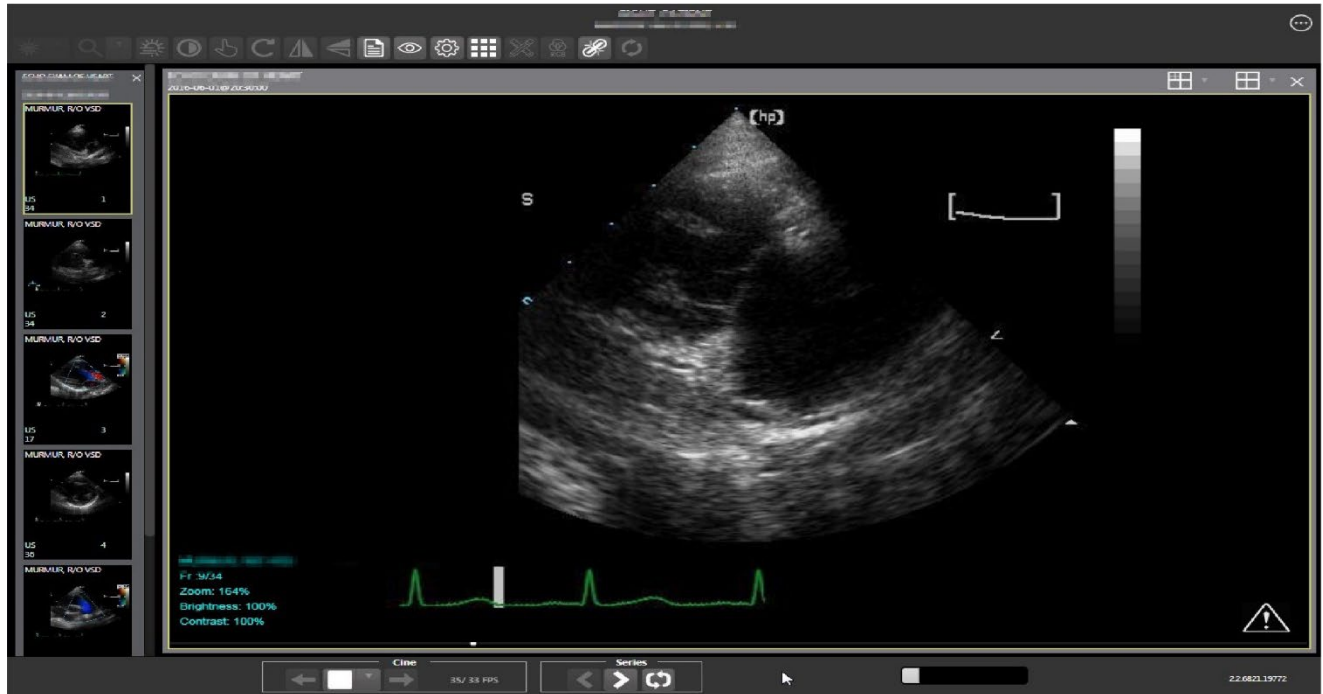Figure 44: Main Viewer Page with Image Loaded



Click the **External Links** button on the Viewer page and a drop-down menu (Figure 45) appears.

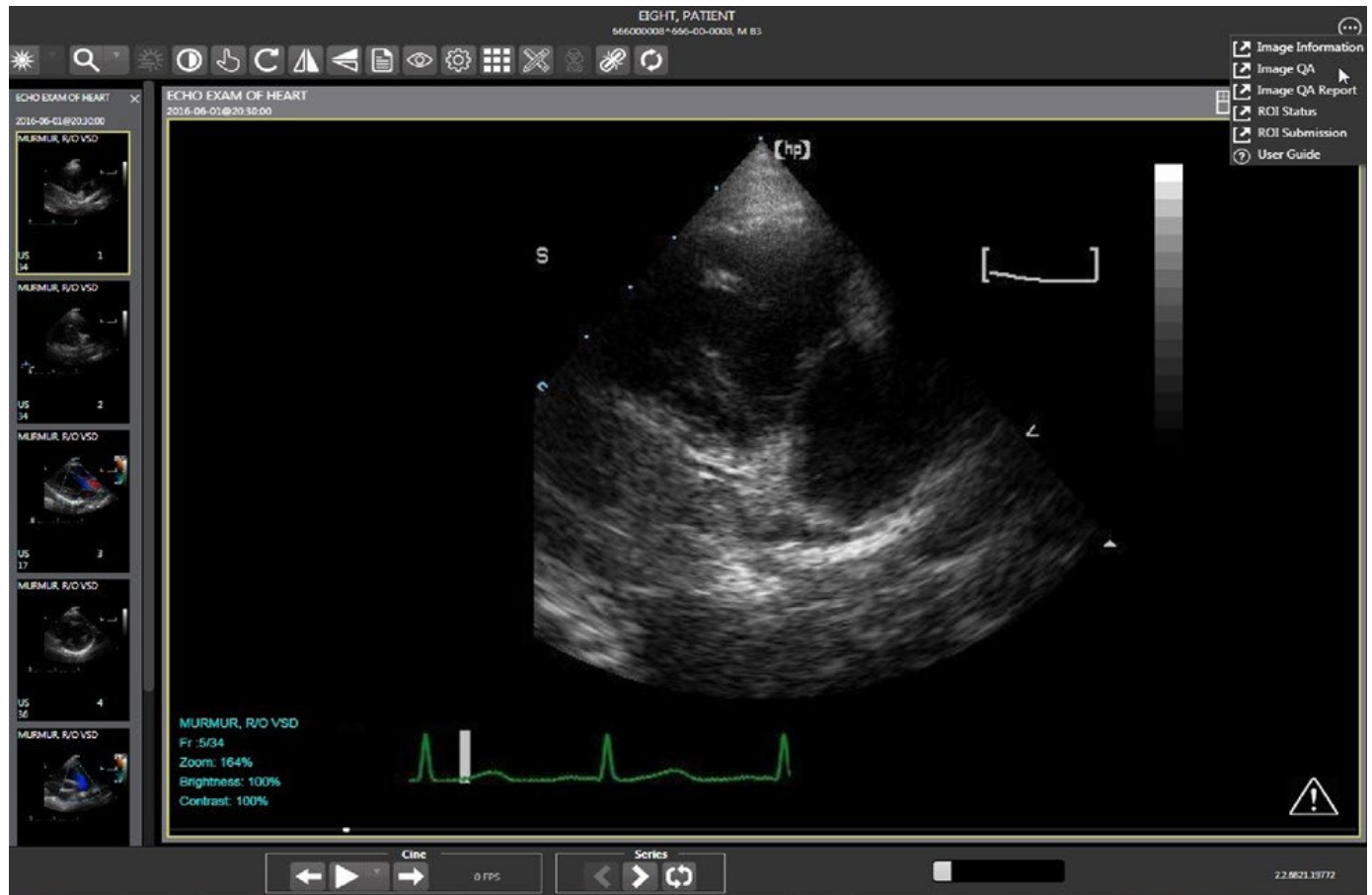Figure 45: External Links Menu on Main Viewer Page



Select the **Image Information** link from the menu. The image information page (Figure 46) displays.

ECHO EXAM OF HEART	
1	
1/6	

ECHO EXAM OF H...	ECHO EXAM OF H...	ECHO EXAM OF H...	ECHO EXAM OF H...
1	2	3	4


ECHO EXAM OF H...	ECHO EXAM OF H...
5	6


```
***** Finish for Image ID# 1666 *****
ACQUISITION DEVICE      = (150)    = GE MEDICAL SYSTEMS (VERSION# 50 AED_31.3.2)
ACQUISITION SITE        = (100)    = CANS MANTIS
CAPTURE APPLICATION     = (10)     = DICOM Gateway
CLASS INDEX             = (1)       = CLIN
CONTROLLED DATE         = (3160615.201602) = JUN 15, 2016@20:16:13
DATE/TIME IMAGE SEQU    = (3160615.201602) = JUN 15, 2016@20:16:13
DICOM SOP CLASS         = (101)     = 1.2.840.10006.5.1.4.1.1.3.1
VIEW & VOLUME, ANGLE    = (12)     = RASLN
DISE & VOLUME, MAGNIFIC = (12)     = MAGLN
DOCUMENT DATE           = (3000223.154702) = FEB 23, 2000@15:47:02
FILEEXT                 = (PAPIDOC001166).JPG = PAPIDOC001166.JPG
GROUP NAME              = (1666)     = ECHOTESTEMP 66A-DD-DGEM ECHO EXAM OF HEART
LAST ACQSD DATE         = (3103114.094155) = NOV 14, 2016@09:41:55
NEW SOP INSTANCE UID     = 1.2.840.133754.1.7.500.0.20160613.202335.0
METHOD                  = 1666
OBJECT NAME             = ECHOTESTEMP 66A-DD-DGEM ECHO EXAM OF HEART
OBJCT PYR              = (100)     = DICOM IMAGE
PACKAGE CHECK           = RAD
PROC PROCEDURE          = (730)     = ECHO EXAM OF HEART
PROC UID                = 1.2.840.133754.1.7.500.2.20160613.201613.0
PATIENT DATA FILE      = (741)     = RADIOLOGY
PATENT GLOBAL ROOT ID   = 722
PATIENT                 = (3)       = ECHOTESTEMP
PROC/EVENT CHECK        = (100)     = ULTRASOUND
PROCEDURE               = US
PROCEDURE/EXAM DATE/TIME = (3160602.203) = JUN 01, 2016@20:30
RADIOLOGY REFERENCE     = (722)     = 040116-69
STILLID UID             = 1.2.840.133754.1.5.500.48329390.7949.1.40116-06.1
SHORT DESCRIPTION        = ECHO EXAM OF HEART [E]
SRVCD/SRVPRC INDEX     = (149)    = RADIOLOGY
STATUS                  = (1)       = Viewable
START DATE              = (3160615.201602) = JUN 15, 2016@20:16:13
TIME INDEX              = (75)      = IMAGE
```

VIX Enhancements MAG*3.0*348

User Guide

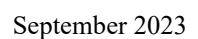43



Select **Image QA** from the **External Links** menu to open the QA review page (Figure 48).

QA Review

ViewerImages

Search Filters

Select Date Range

Today

Select

Captured By

Has Status of

Select image status

Person ID: team

1

Or

Select Filter

Search

Image Information

Pattern

<pattern>

SSN

<SSN>

Short Desc

<short_desc>

Type

<type>

Specialty

<specialty>

Procedure

<procedure>

Status

<status>

Revision

<revision>

QA Reviewed

PrevNext

Needs Review


Manage Study

Search Results

Item	Patient	Img	Image ID	Action
------	---------	-----	----------	--------

Click the **Select** button from the **Search Filters** group to open the **Select Date Range** pop-up (Figure 49). Select a date range.

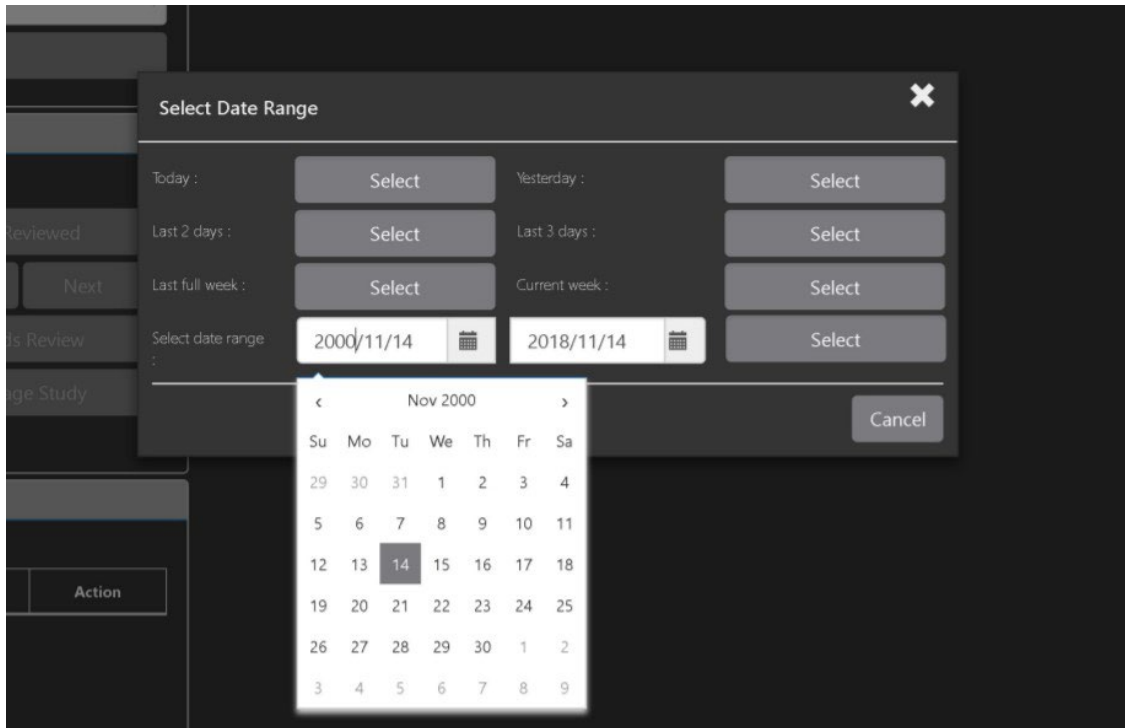Figure 49: QA Filters and Date Range



The **Captured By** dropdown populates with a list of valid users for the date range (Figure 50).

Figure 50: Search Filter Details

Click the **Has Status of** dropdown, and choose one or more images to choose the status (Figure 51).

Figure 51: QA Status Choices

The screenshot shows the 'Search Filters' panel. The 'Has Status of' dropdown is open, displaying four options with checkboxes: 'Viewable', 'QA Reviewed', 'Needs Review', and 'Image Never Existed'. Other filters visible include 'Select Date Range' with a 'Date range' input and 'Select' button, 'Captured By' with a dropdown showing 'Radtech,Fiftytwo', and 'Percen (%) return'.

Restrict the study count in the search results using the % **return** dropdowns (Figure 52). The left dropdown is in the percentage of study items to display. The right dropdown is the maximum number of items to display.

Figure 52: Percentage of Images to Display

The screenshot shows the 'Search Filters' panel. The 'Percen (%) return' dropdown is open, displaying two columns of options. The left column contains percentages: 1, 3, 5, 10, 20, 30, and 50. The right column contains maximum item counts: 200, 300, 500, 750, and 1000. Other filters visible include 'Select Date Range' with a 'Date range' input and 'Select' button, 'Captured By' with a dropdown showing 'Radtech,Fiftytwo', and 'Has Status of' with a dropdown showing '4 selected'.


Choose a filter using the **Select Filter** dropdown (Figure 53).

Figure 53: Saved Filters

The screenshot shows a software interface with a 'Search Filters' section. It contains several filter options: 'Select Date Range' with a 'Date range' input and a 'Select' button; 'Captured By' with a dropdown showing 'Radtech,Fiftytwo'; 'Has Status of' with a dropdown showing '4 selected'; 'Percen (%) return' with two dropdowns showing '1' and 'Or'; and 'Select Filter' with a dropdown showing '15'. The 'Select Filter' dropdown is open, displaying a list of filters: '15' (highlighted in blue), 'TEST FILTER', 'TEST FILTER 2', 'NST ORIGIN', 'Clinical All', and 'Admin All'. Below the 'Search Filters' section is an 'Image Information' section with fields for Patient, SSN, Short Desc, Type, and Specialty, each followed by a colon and a placeholder value. A 'Needs Review' button is visible at the bottom right of the 'Image Information' section.


Click **Search** to search for studies. The first study in the list of studies found displays automatically on the right. Click on the **Viewer** icon (circled in red, Figure 54) to display the study. Click on the **Images** icon (circled in yellow, Figure 54) to display the images tab for the study.

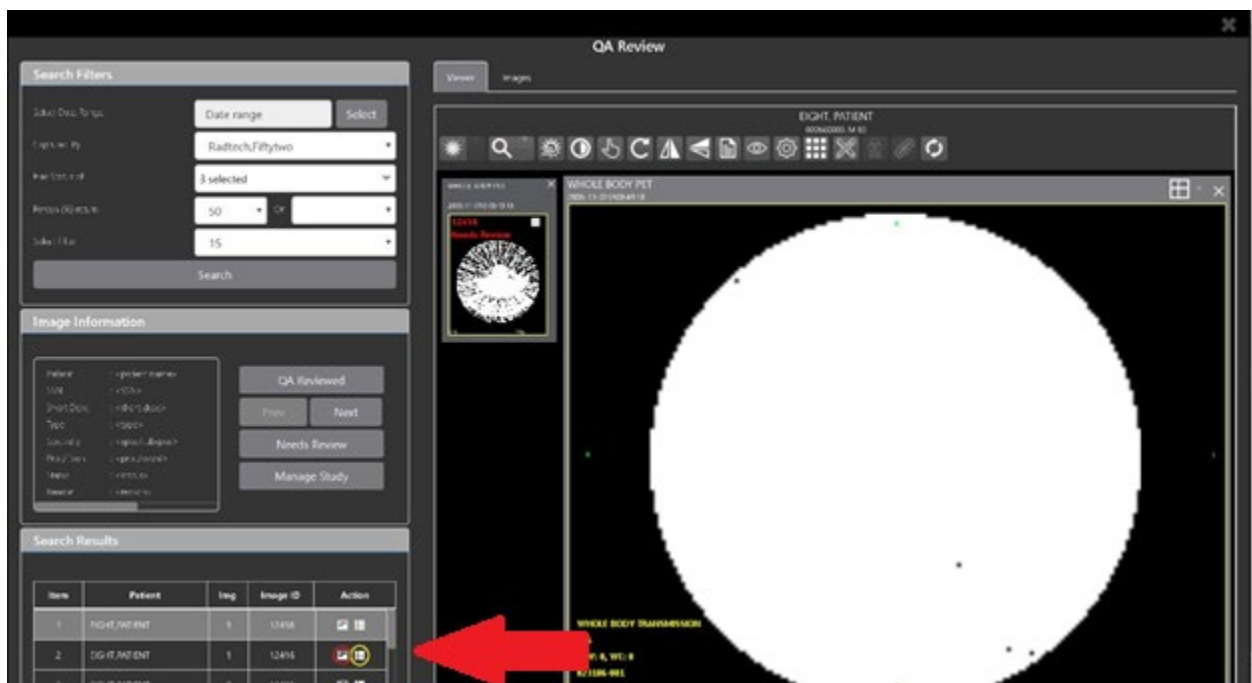Figure 54: QA Filter Results



You can click on the **Images** tab at the top after displaying the study (Figure 55).

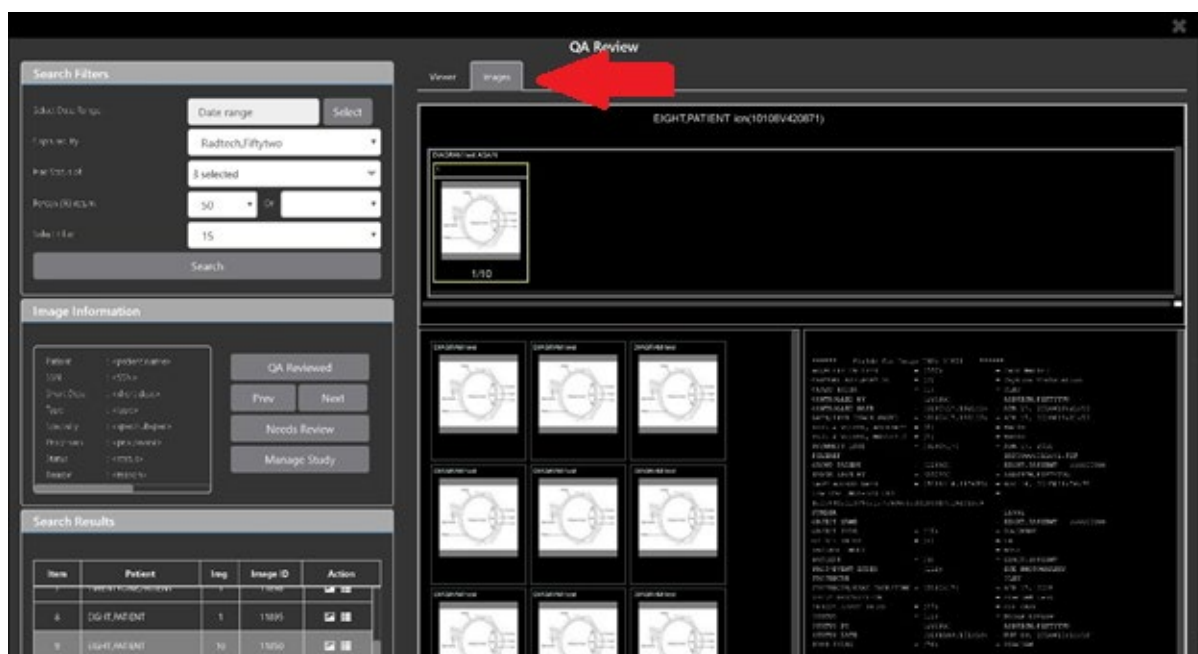
Figure 55: QA Filter Results and Image Information



4.4.3. Viewing and Setting Image Status

Click on a thumbnail. Its image information, including the view status, displays in a panel on the left (highlighted in blue, Figure 56).

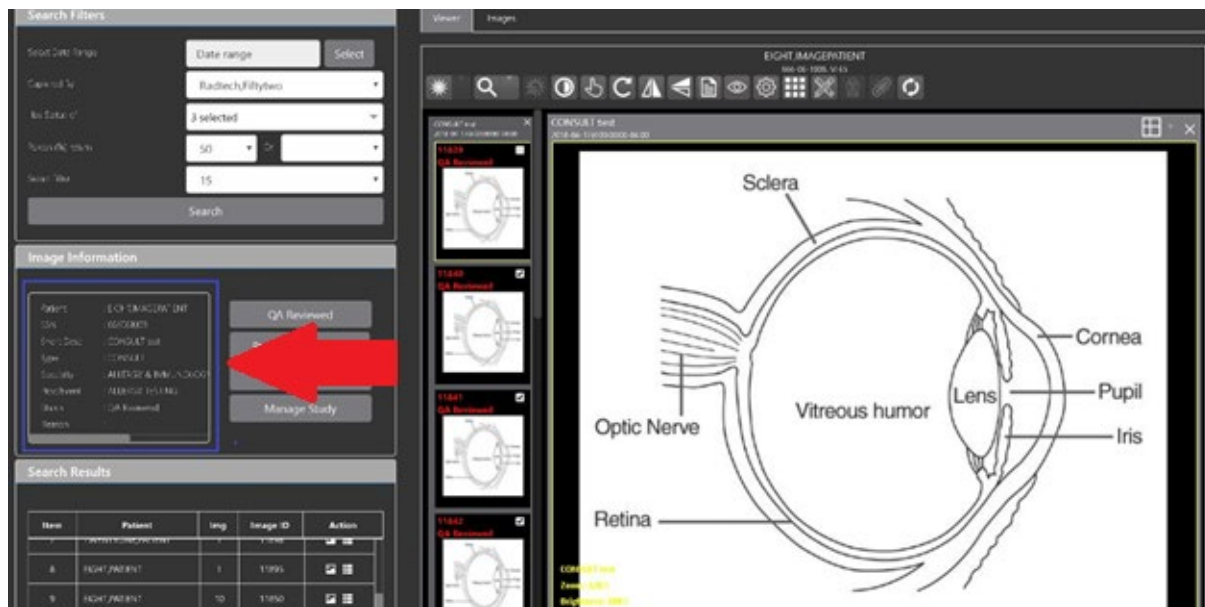Figure 56: QA Thumbnail




Click on an **image thumbnail** to display the corresponding image. More than one thumbnail can be selected using the shift key.

NOTE: The user's VistA account requires the MAG EDIT or MAG QA REVIEW keys to edit/set this information.

Click the **QA Reviewed** or **Needs Review** button to set the corresponding image status. You can also right-click on the image or the image thumbnail to set the image status (Figure 57).

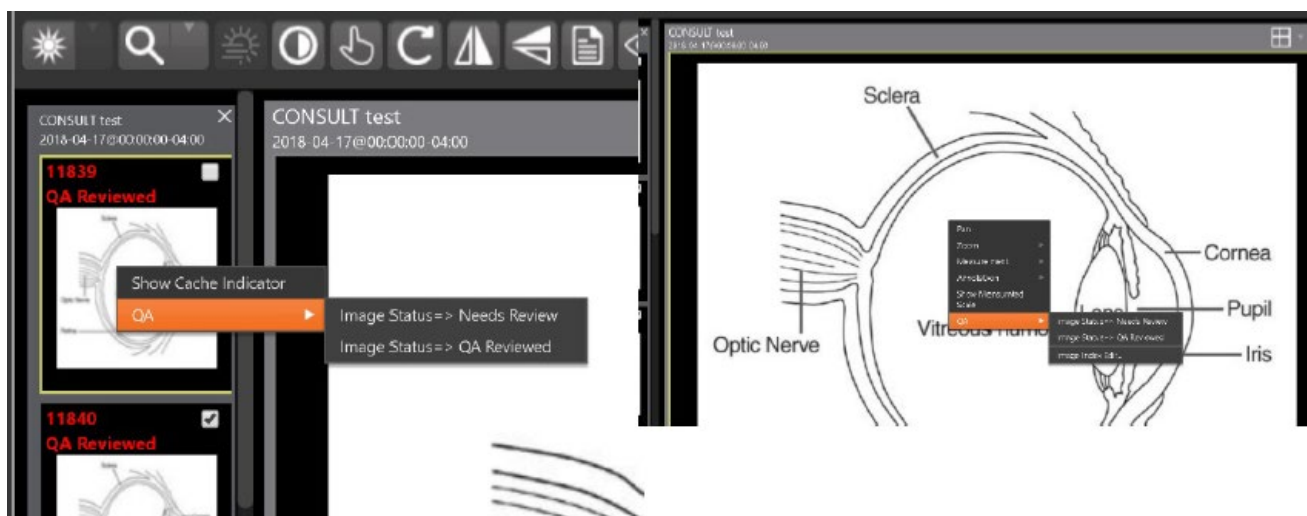Figure 57: QA Review Status Update Menus



You can invoke the Image Index Editor page by clicking on the **Image Index Edit** menu item.

Use the **Prev** and **Next** buttons to traverse the search results.

Click the **Manage Study** button to display the **Images** tab for the study.

4.4.4. Image QA Report

Select the **Image QA report** from the Viewer page's **External Links** menu to open the QA Statistics Reports page (Figure 58).

Figure 58: QA Statistics Reports Page

The screenshot displays the 'QA Statistics Reports' interface. On the left, the 'New Report Parameters' panel allows users to configure the report. It includes a 'Date Range' section with 'From' and 'To' date pickers both set to '2023/05/20', and a 'Range: Today' button. Below this is a section for including or excluding deleted images, with 'Include Existing Images' checked. The 'Return Image Counts By' section has two radio button options: 'Grouped by Status' (selected) and 'Grouped by Users and Status'. A 'Run Report' button is located at the bottom of this panel. The main area on the right, titled 'QA Statistics Reports', shows 'Report requests for user: TEST,VIXVIEWER'. It contains a table with columns: 'Report Type', 'Status', 'From', 'To', 'Started At', and 'Ended At'. The table is currently empty. At the bottom right of this section are 'Refresh List' and 'View Report' buttons.

Fill in the report parameters fields and click the **Run Report** button (Figure 59).

Figure 59: Run QA Report

New Report Parameters

Date Range

From :

2013/05/20

To :

2023/05/20

Range : 10 year(s) 2 day(s)

☒ Include Deleted Images

☒ Include Existing Images

Return Image Counts By

☐ Grouped by Status

☐ Grouped by Users and Status

Run Report

QA Statistics Reports

Report requests for user : TEST,VIXVIEWER

Report Type	Status	From	To	Started At	Ended At
TBD	TBD	05/15/2023	05/15/2023	05/15/2023 16:23:24	05/15/2023 16:23:24
TBD	TBD	01/16/2021 01:01:33	02/16/2021 02:02:33	03/16/2021 03:03:33	04/16/2021 04:04:33
TBD	TBD	05/16/2021 01:01:33	06/16/2021 02:02:33	07/16/2021 03:03:33	08/16/2021 04:04:33
TBD	TBD	09/16/2021 13:52:33	10/16/2021 13:52:33	11/16/2021 13:52:33	12/16/2021 13:52:33

Refresh List

View Report

Select any item, and click **View Report** button, then the selected image report statistics display (Figure 60).

Figure 60: View QA Report

New Report Parameters

Date Range

From :

2013/05/20

To :

2023/05/20

Range : 10 year(s) 2 day(s)

☒ Include Deleted Images

☒ Include Existing Images

Return Image Counts By

☐ Grouped by Status

☐ Grouped by Users and Status

Run Report

QA Statistics Reports

Report requests for user : TEST,VIXVIEWER

Report Type	Status	From	To	Started At	Ended At
TBD	TBD	05/15/2023	05/15/2023	05/15/2023 16:23:24	05/15/2023 16:23:24
TBD	TBD	01/16/2021 01:01:33	02/16/2021 02:02:33	03/16/2021 03:03:33	04/16/2021 04:04:33
TBD	TBD	05/16/2021 01:01:33	06/16/2021 02:02:33	07/16/2021 03:03:33	08/16/2021 04:04:33
TBD	TBD	09/16/2021 13:52:33	10/16/2021 13:52:33	11/16/2021 13:52:33	12/16/2021 13:52:33

Refresh List

View Report

Image Report Statistic

This Report Was Started At: 05/15/2023 16:23:24

For Date Range: 05/15/2023 thru 05/15/2023

Range : Today

Report Flags :

Include existing images

Return image counts grouped by users and status

Export


User	Status	Entities	Pages	QA%
Akiona,Kari	Viewable	29	29	
Akiona,Kari		29	29	0
Alaghari,Vydehi	Viewable	4	4	
Alaghari,Vydehi	Needs Review	1	1	
Alaghari,Vydehi		5	5	0
Atteburg,Cedric 8 OD	Viewable	62	62	

Click **Export** to generate a .csv file using the results.

4.5. Closing the Viewer

To close the Viewer page, click the "X" on the browser tab. See Figure 61 below.

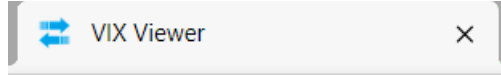Figure 61: Viewer Close Button




NOTE: Depending on the hosting application, the Viewer page might automatically close when you select a different patient.

5. Troubleshooting

The **View Image Info** main toolbar button can offer data to help you troubleshoot. See Section 4.3.9 for more information.

To display version information and message history about the Viewer, first click the **Viewer Version Number** located in the lower right corner of the Viewer main page (Figure 62).

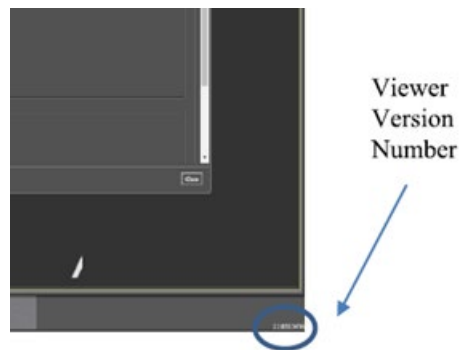Figure 62: Viewer Version Number



In the Viewer Information dialog (Figure 63) that opens, click on **Version History** to display version release information or click on **Message History** to display message history items.

Figure 63: Viewer Information Dialog



If encountering issues, clear the cache in the web browser for the Viewer, and refresh the page.

5.1. Special Instructions for Error Correction

Please refer to the VIX Overview - View Image Viewer – Troubleshooting Section in [MAG3_0P348_VIX_ADMIN_GUIDE.PDF](#).

6. Acronyms and Abbreviations

Acronym/Term	Definition
AVI	Audio Video Interleave
BMP	Bitmap
Cache	A temporary storage place allowing faster access to data.
CD	Compact Disc
CDA	Compact Disc Audio
CPRS	Computerized Patient Record System
CT	Computerized Tomography
CVIX	Central Vista Imaging Exchange
DICOM	Digital Imaging and Communications in Medicine
DoD	Department of Defense
ECG / EKG	Electrocardiogram
FDA	Food and Drug Administration
GIF	Graphics Interchange Format
JLV	Joint Legacy Viewer
JPEG	Joint Photographic Experts Group
MOV	QuickTime Movie Format
MP4	Media Picture Experts Group 4
OIT	Office of Information and Technology
PACS	Picture Archiving and Communication System
PDF	Portable Document Format
PNG	Portable Network Graphics
QA	Quality Assurance
RGB	Red Green Blue
ROI	Release of Information
RTF	Rich Text Format
TIFF	Tag Image File Format
TXT	Text
VA	Department of Veteran Affairs
VistA	Veterans Health Information System and Technology Architecture
Viewer	Enhanced Image Viewer
VIX	VistA Imaging Exchange Services
WW/WC	Window Level