

# AutoPulse Individual Operational Competencies Checklist

**Provider Name:** \_\_\_\_\_ **Date:** \_\_\_\_\_

Competency	Behavioral Criteria	Pass	Retest
<b>Describe the Appropriate Indications for Use</b>	Patients in cardiac arrest that need CPR		
	Patients 18 years of age or older		
	Not indicated for trauma patients		
<b>Perform a Daily Check Out</b>	Remove AutoPulse from the case		
	Rotate batteries		
	Power on AutoPulse and check for green LED		
<b>Apply and Operate the AutoPulse</b>	Describe purpose of yellow line indicator		
	Align the mannequin on the board		
	Apply LifeBand and fully extend it to "home" position		
	Start the AutoPulse systems		
	Attach Shoulder Restraints with the system operating		
	Describe how to address User Alerts		
	Fully extend LifeBand to "home" position and restart		
	Stop AutoPulse and Power off.		
<b>Readying the AutoPulse for the Next Use</b>	Fully extend LifeBand		
	Remove LifeBand and install a new one		
	Replace the battery		
	Remove and replace the battery.		
	Store the AutoPulse in the carry case		

**Scoring criteria:** Participant must successfully demonstrate all Behavioral Criteria

**Pass** \_\_\_\_\_ **Retest** \_\_\_\_\_

**Comments:** \_\_\_\_\_

\_\_\_\_\_

**Evaluator's Signature:** \_\_\_\_\_ **Date:** \_\_\_\_\_