



# Vista Blood Establishment Computer Software (VBECS) Version 2.3.1

User Guide






June 2019



Department of Veterans Affairs  
Enterprise Project Management Office

This page intentionally left blank.

# Table of Contents

<b>INTRODUCTION.....</b>	<b>1</b>
FREWARE DISCLAIMER.....	2
RELATED MANUALS AND MATERIALS .....	2
<b>ORIENTATION.....</b>	<b>3</b>
HOW THIS USER GUIDE IS ORGANIZED.....	3
Terms.....	3
Background and Basic Knowledge .....	3
Figures and Tables .....	3
Options .....	3
Supplemental Information.....	4
Appendices .....	4
SECURITY.....	4
APPLICATION ARCHITECTURE .....	6
HARDWARE AND INFRASTRUCTURE ARCHITECTURE.....	7
Options That Require VistALink .....	8
VBECS Patching.....	10
<b>CUSTOMER SUPPORT.....</b>	<b>11</b>
PROBLEMS?.....	11
<b>USING THE SOFTWARE.....</b>	<b>13</b>
SOFTWARE BASICS .....	13
COMMONLY USED SYSTEM RULES.....	15
Date and Time .....	15
Locking and Time-Outs.....	15
Working with Data .....	15
Searching the Database .....	16
Testing.....	17
Creating and Viewing Reports .....	21
REPORT TOOLBAR .....	18
Navigation Arrows and Report Pages .....	18
Report Layout.....	18
Report Zoom-in/Zoom-out .....	19
Report Search .....	19
Report Export .....	20
Copying an Exported Report.....	23
USING VBECS .....	25
USING ONLINE HELP.....	25
Searching Using the Table of Contents.....	28
Viewing the VBECS Version Number.....	28
VISTA RECORDS IN VBECS .....	31
ACCESSING THE SYSTEM .....	32

Log into VBECS and VistA .....	32
Exit VBECS .....	39
Screen Settings .....	40
Screen Display.....	44
Sound.....	45
Volume .....	51
Keyboard Shortcuts for Windows .....	52
Drop-Down Menus.....	53
Tool Tips .....	53
Icons and Buttons .....	53
Toolbars.....	53
<b>PATIENT INFORMATION TOOLBAR .....</b>	<b>54</b>
Patient Blood Availability .....	55
 Medication Profile .....	56
 View Special Instructions and Transfusion Requirements .....	57
View Recent Orders, Recent Transfusions/Issued Units, and Transfusion Reaction History .....	59
 Patient Information Toolbar: Recent Orders.....	59
 Patient Information Toolbar: Recent Transfusions/Issued Units .....	60
 Patient Information Toolbar: Transfusion Reaction History .....	60
<b>OTHER VBECS FUNCTIONS .....</b>	<b>62</b>
Collecting Workload .....	62
<b>CONFIGURING SITE PARAMETERS .....</b>	<b>64</b>
UPDATE USER ROLES .....	64
COMPONENT CLASSES .....	67
CONFIGURE DAILY QC .....	70
CONFIGURE DIVISION .....	73
Configure Division.....	73
Configure Testing.....	75
Product Modifications .....	77
Order Alerts.....	79
Login Message .....	81
<b>LOCAL FACILITIES .....</b>	<b>83</b>
Blood Products .....	86
Antibodies .....	89
Canned Comments .....	91
<b>SETTING TRANSFUSION PARAMETERS.....</b>	<b>93</b>
Transfusion Complications.....	93
Transfusion Effectiveness .....	95
MSBOS .....	97
Workload Codes.....	99
<b>REAGENTS AND SUPPLIES.....</b>	<b>103</b>
Enter Daily QC Results .....	103
<b>REAGENTS .....</b>	<b>111</b>
Supplies .....	120
Equipment .....	125

<b>COMPONENT PROCESSING .....</b>	<b>130</b>
SHIPMENTS .....	130
Incoming Shipment .....	130
Edit Invoice Text .....	151
UNIT SEARCH SCREEN .....	152
TEST UNITS .....	154
ABO/Rh Confirmation .....	154
Unit Antigen Typing .....	158
MODIFY COMPONENTS .....	169
Modify Units .....	169
Modify Units: Split a Unit .....	181
Modify Units: Pool Units .....	184
Add and/or Remove Units from a Pool .....	189
MAINTAIN UNIT RECORDS .....	192
Edit Unit Information .....	192
Edit Financial Data .....	198
Free Directed Unit For Crossover .....	200
Discard or Quarantine .....	202
SELECT UNITS .....	208
ISSUE BLOOD COMPONENTS .....	220
RETURN ISSUED UNITS TO BLOOD BANK .....	233
RELEASE UNITS FROM PATIENT ASSIGNMENT .....	237
PRINT UNIT CAUTION TAG & TRANSFUSION RECORD FORM .....	240
INVALIDATE TEST RESULTS .....	250
REMOVE FINAL STATUS .....	254
<b>PROCESSING ORDERS .....</b>	<b>258</b>
ACCEPT ORDERS: ACCEPT AN ORDER .....	258
Accept Orders: Pending Order List .....	264
 Display Patient Update Alerts .....	268
 Display Patient Merge Alerts .....	270
EMERGENCY TRANSFUSION ORDER PROCESSING .....	271
MAINTAIN SPECIMEN .....	274
PATIENT TESTING: CANCEL AN ACTIVE ORDER .....	278
PATIENT TESTING: PENDING TASK LIST .....	280
Accept Orders: Cancel a Pending Order .....	282
ORDER REFLEX TESTS .....	284
<b>PATIENT SERVICES .....</b>	<b>288</b>
PATIENT SEARCH SCREEN .....	288
Patient Merge .....	290
Updated Patients and Deceased Patients .....	294
Patient Testing: General Instructions .....	296
Patient Testing: Record a Patient ABO/Rh .....	302
Patient Testing: Record a Patient Antibody Screen .....	305
Patient Testing: Record a Direct Antiglobulin Test .....	307
Patient Testing: Record a Crossmatch .....	310

Patient Testing: Record a Patient Antigen Typing .....	316
Patient Testing: Enter Antibody Identification Results.....	322
Patient Testing: Record a Transfusion Reaction Workup .....	325
Finalize/Print TRW .....	333
MAINTAIN PATIENT RECORDS.....	338
Special Instructions & Transfusion Requirements: Enter a Transfusion Requirement.....	338
Special Instructions & Transfusion Requirements: Enter and Remove Special Instructions.....	343
POST-TRANSFUSION INFORMATION .....	346
DOCUMENT ABO INCOMPATIBLE TRANSFUSIONS.....	352
Justify ABO/Rh Change.....	354
<b>AUTOMATED TESTING .....</b>	<b>358</b>
AUTOMATED TESTING INTERFACE.....	358
AUTOMATED TESTING REVIEW FOR BLOOD UNITS .....	362
AUTOMATED TESTING REVIEW FOR PATIENTS .....	366
<b>REPORTS.....</b>	<b>372</b>
ADMINISTRATIVE DATA REPORT .....	372
AUDIT TRAIL .....	375
BLOOD AVAILABILITY.....	377
COST ACCOUNTING REPORT .....	379
C:T RATIO REPORT.....	381
DIVISION TRANSFUSION REPORT .....	383
DIVISION WORKLOAD REPORT.....	385
EXCEPTION REPORT.....	387
FINALIZE/PRINT TRW (SEE PATIENT SERVICES, FINALIZE/PRINT TRW).....	389
INAPPROPRIATE TRANSFUSION REQUEST REPORT .....	389
ISSUED/RETURNED UNIT REPORT .....	390
MEDICATION PROFILE (SEE PATIENT INFORMATION TOOLBAR).....	392
ORDER HISTORY REPORT .....	392
PATIENT HISTORY REPORT.....	395
PROLONGED TRANSFUSION TIME REPORT .....	397
PATIENT TESTING WORKLIST AND TESTING WORKLIST REPORTS .....	400
TRANSFUSION COMPLICATIONS REPORT.....	403
TRANSFUSION EFFECTIVENESS REPORT.....	405
TRANSFUSION REACTION COUNT REPORT .....	407
TRANSFUSION REQUIREMENTS REPORT.....	409
UNIT HISTORY REPORT .....	411
<b>REFERENCES.....</b>	<b>414</b>
<b>GLOSSARY.....</b>	<b>418</b>
<b>APPENDICES.....</b>	<b>428</b>
APPENDIX A: DOWNTIME FORMS AND INSTRUCTIONS .....	428
Patient Testing Form .....	430
Unit ABO/Rh Confirmation Form .....	433
Unit Issue and Inspection Log.....	434
Unit Modification Form .....	435

APPENDIX B: DATABASE TABLE INFORMATION .....	438
Antibody and Antigen Tables.....	438
Antibody and Antigen Table: Irregular Antibodies.....	438
Antibody and Antigen Table: Database Conversion.....	441
Antibody Screen Test Interpretation .....	442
Canned Comment Category Types and Text.....	443
CPRS Orderable Blood Components (Component Classes) Mapped to ICCBBA Component Classes .....	448
Details in Audit Trail Report.....	448
Details in Exception Report .....	453
Allowable Product Modifications by Original Product Type.....	456
Order Status Flowchart.....	462
Rules for Electronic and Serologic Crossmatch.....	463
Unit Status Flowchart.....	464
VBECS Maximum Surgical Blood Order Schedule .....	466
BCE COTS Message Updates.....	467
Workload Process to Use Case Mapping Table .....	471
APPENDIX C: VBECS WORK FLOW .....	484
APPENDIX D: LIMITATIONS AND RESTRICTIONS .....	488
APPENDIX E: KNOWN DEFECTS AND ANOMALIES .....	508
APPENDIX F: SYSTEM VALIDATION.....	510
Introduction .....	510
Validation Plan.....	510
Prerequisites .....	511
Record System Settings.....	512
Configure Division.....	512
Configure Testing: Full Service .....	512
Product Modifications: Full Service.....	513
Order Alerts.....	514
Configure Daily QC: Full Service.....	515
Maintain Minimum Levels: Full Service .....	516
Log In Reagents: Full Service .....	517
Blood Products .....	518
Local Facilities .....	519
Component Classes .....	520
Transfusion Complications.....	521
Transfusion Effectiveness .....	522
Workload Codes.....	523
Canned Comments .....	523
MSBOS .....	523
Edit Invoice Text.....	523
Antibodies .....	524
Update User Roles.....	525
Testing.....	525
VBECS Forms and Reports.....	526
APPENDIX G: WORK PROCESS FLOWCHARTS .....	528
APPENDIX H: FREQUENTLY ASKED QUESTIONS .....	538
APPENDIX I: COMMON ERROR CORRECTIONS.....	540

Correct Patient Test Results .....	540
Invalidate Test Results .....	540
UN-Pooling a Pooled Unit (in the computer only).....	541
UN-Modifying a Unit.....	541
APPENDIX J: RECOMMENDED REPORT USAGE .....	542
APPENDIX K: BARCODE SCANNER CONFIGURATION AND TROUBLESHOOTING .....	544
APPENDIX L: SYSTEM RESPONSES TO ACTIVE TRANSFUSION REQUIREMENTS IN SELECT UNIT AND ISSUE BLOOD COMPONENT.....	546
APPENDIX M: WORKLOAD PROCESS MAPPING TO APPLICATION OPTION TABLE .....	556
<b>INDEX.....</b>	<b>564</b>
<b>REVISION HISTORY .....</b>	<b>570</b>



# Introduction

VistA Blood Establishment Computer Software (VBECS) is a computer software application that manages transfusion service activities. The purpose of VBECS is to automate the documentation and record keeping of selected daily activities of the transfusion service.

VBECS accommodates:

- The processing of blood component shipments.
- The modification of blood units.
- Electronic receipt of orders from the Department of Veterans Affairs VistA computer system.
- The recording of patient compatibility testing, transfusions, transfusion processing requirements, and special instructions.

VBECS does not support the collection of blood products (donor module).

VBECS is an improved Blood Bank application that facilitates ongoing compliance with Food and Drug Administration (FDA) standards for medical devices and enhances the Department of Veterans Affairs (VAs) ability to produce high-quality blood products and services to veterans. The system follows blood bank standards, standards of national accrediting agencies, FDA regulations, and VA policies.

## Limitations and Restrictions

**VBECS does not support:**

- Blood donor collection activities (donor module)
- Direct data entry of antibody identification
- Direct data entry of transfusion reaction workups
- Patient testing, transfusions, and blood components recorded in Legacy VistA are not available in VBECS
- Pediatric blood product preparation
- Printing of blood product labels
- User defined customized queries or reports
- User editing blood product, antigen, antibody, or transfusion reaction tables
- Validation record storage

Limitations and Restrictions are listed in [Appendix D: Limitations and Restrictions](#)

by option as well as throughout this document as part of the detailed information associated with the functionality.



*Unauthorized access or misuse of this system and/or its data is a federal crime. Use of all data, printed or electronic, must be in accordance with VA policy on security and privacy.*



*The U.S. Food and Drug Administration classifies this software as a medical device. Unauthorized modifications will render this device an adulterated medical device under Section 501 of the Medical Device Amendments to the Federal Food, Drug, and Cosmetic Act. Acquiring and implementing this software through the Freedom of Information Act require the implementer to assume total responsibility for the software and become a registered manufacturer of a medical device, subject to FDA regulations.*

## **Freeware Disclaimer**

Requests for VBECS may be submitted through the Freedom of Information Act (FOIA). Effective on September 19, 2011, the Department of Veterans Affairs will accept FOIA requests electronically. Most VA Central Office (VACO) Offices are in the process of establishing an electronic mailbox. In the meantime, the FOIA Service will accept electronic FOIA requests for any VACO FOIA Office that does not yet have a mailbox in place, and will forward your request to the appropriate office for processing.

Submit requests to [http://www.foia.va.gov/FOIA\\_Offices.asp](http://www.foia.va.gov/FOIA_Offices.asp)

## **Related Manuals and Materials**

- *VistA Blood Establishment Computer Software (VBECS) 2.3.1 Technical Manual-Security Guide*
- *VistA Blood Establishment Computer Software (VBECS) 2.3.1 Administrator User Guide*
- *VistA Blood Establishment Computer Software (VBECS) 2.3.1 Release Notes*

Copies of *VistA Blood Establishment Computer Software (VBECS) User Guide* may be obtained from the VDL: VistA Documentation Library (VDL), Clinical Section, Laboratory: VistA Blood Establishment Computer Software (VBECS) Web site.

# Orientation

This section covers:

- How This User Guide Is Organized
- Security: who can use VBECS
- Application Architecture: the architectural requirements for VBECS
- Hardware and Infrastructure Architecture: the system environment for installation and operation of VBECS
- VistA and VBECS: how VBECS compares to VistA
- Customer Support: how to get help
- VBECS Patching

## *How This User Guide Is Organized*

### Terms

For consistency and space considerations, the pronouns “he,” “him,” and “his” are used as pronouns of indeterminate gender equally applicable to males and females.

In many instances, a user may scan a barcode or enter data manually (by typing). The term “enter” is used throughout this guide to mean “enter manually.”

See the Glossary for definitions of other terms and acronyms used in this guide.

### Background and Basic Knowledge

The Introduction, Orientation, and Using the Software sections provide the background and basic knowledge VBECS users need.

### Figures and Tables

If you refer to figures and tables from the user guide in your local policy and procedure documents, you may wish to use their titles only, without figure or table numbers: as the user guide is updated, those numbers may change.


### Options

The Configuring Site Parameters, Component Processing, Processing Orders, Patient Services, and Reports sections cover VBECS options.


Each option section starts with an introduction, a short paragraph that explains what a function allows a user to do, followed by:

- Assumptions: tasks that must be completed before performing a function.
- Outcome: results of performing a function.
- Limitations and Restrictions: features of the system that are restricted by and exist within this function.
- Additional Information: information that helps the user perform a function.
- User Roles with Access to This Option: which users may perform this option in whole or in part.
- A detailed description of what the function allows each user to do. Two or more arrows (➤) indicate functions limited to a specific role.
- Table of steps for the basic path (user and VBECS actions).
- Tables of steps for alternate paths (user and VBECS actions).

- Figures (examples of reports, screen shots, etc.).
- Table of alerts that may occur during an option.

*A  (bullhorn) indicates that VBECS emits an audible alert. Keep the sound on to hear the alerts (see Volume for instructions on adjusting the volume).*

- Outlined, italicized text highlights tips, warnings, and cautions, for example:

*A  signals potential hazards throughout this guide as they occur.*

## Supplemental Information

The third group of sections (References, Glossary) provides knowledge that may be needed beyond the standard, daily use of VBECS.


## Appendices

The appendices contain truth tables, diagrams, and other materials for reference.

Click a section name or page number in the table of contents to move to a section. The index does not incorporate this feature.

## Security

*User roles* identify who can access which functions. Not every VBECS user has access to all VBECS options or functions within options. Two or more arrows (➤) indicate functions limited to a specific role.

* It is critical that each site evaluate its staff and determine appropriate security levels based on training and qualifications.*

User roles also govern which options are available and which warnings can be overridden. VBECS displays options that are not available to a user in gray text.

Each of the six user roles is associated with a security level (see Table 1). Privileges accumulate as the security level increases. For example, a Lead Technologist's privileges include those of a Blood Bank Technologist and Enhanced Technologist.

**Table 1: User Roles and Functions**

Security Levels and User Roles	Accessible Functions	Who Should Fill These Roles?
Level 1: Blood Bank Technologist (all users)	<ul style="list-style-type: none"> <li>• Standard access:               <ul style="list-style-type: none"> <li>○ patient and donor testing</li> <li>○ accepting and canceling orders</li> <li>○ modifying units</li> <li>○ processing shipments</li> <li>○ selecting and issuing blood that does not require higher level overrides</li> <li>○ processing transfusion reaction workups</li> <li>○ add patient's transfusion requirements</li> <li>○ accessing report functions</li> <li>○ enter daily QC results</li> <li>○ discard/quarantine blood units</li> <li>○ release units from patient assignment</li> <li>○ return issued units to blood bank</li> </ul> </li> <li>• Activate/edit some shipper information.</li> <li>• Activate/edit some blood product information.</li> </ul>	Rotating technologists, new employees, and students trained to perform associated tasks and overrides.
<ul style="list-style-type: none"> <li>▶▶ Level 2: Enhanced Technologist</li> </ul>	<ul style="list-style-type: none"> <li>• Edit unit cost.</li> <li>• Modify/issue expired blood products.</li> <li>• Release blood products from quarantine.</li> <li>• Edit unit information prior to defining patient associations.</li> <li>• Edit verified unit confirmation testing.</li> <li>• Edit patient record/verified data and test results (except the ABO/Rh for historic record and the antibody field, which results in an antigen negative requirement).</li> <li>• Release directed units to the main blood supply.</li> </ul>	Experienced technologists trained to process associated overrides and options.
<ul style="list-style-type: none"> <li>▶▶▶ Level 3: Lead Technologist</li> </ul>	<ul style="list-style-type: none"> <li>• Select/issue ABO incompatible blood.</li> <li>• Select/issue antigen positive or untyped red blood cells (for the clinically significant antibodies in the table).</li> <li>• Maintain login message.</li> </ul>	Experienced technologists trained to process associated overrides and options (e.g., evening- or night-shift supervisors).
<ul style="list-style-type: none"> <li>▶▶▶▶ Level 4: Traditional Supervisor</li> </ul>	<ul style="list-style-type: none"> <li>• Maintain reagent minimum levels.</li> <li>• Edit permanent record of the patient's historic ABO/Rh.</li> <li>• Edit Transfusion Requirements (TRs) and Special Instructions (SIs).</li> <li>• Edit patient record red cell antibody permanent fields (match antigens in the antigen table).</li> <li>• Edit a blood unit's record (verified unit data when unit has any previous patient associations).</li> <li>• Edit a Patient's Transfusion Record.</li> <li>• Remove a Blood Unit's Final Status.</li> </ul>	Experienced blood bank supervisors trained to process associated overrides and options

Security Levels and User Roles	Accessible Functions	Who Should Fill These Roles?
>>>>> Level 5: Enhanced Supervisor	<ul style="list-style-type: none"> <li>• Edit antigen frequency table to reflect local population.</li> <li>• Add new local shipper, not already on the table issued with VBECS.</li> <li>• Edit and create canned comments for the division.</li> <li>• Edit and create consultative reports, templates, MSBOS, reagents, and equipment maintenance logs for the division.</li> </ul>	Experienced blood bank supervisors or VBECS administrators trained to process associated overrides and options
>>>>>> Level 6: Administrator/Supervisor	<ul style="list-style-type: none"> <li>• Define VBECS users' security settings.</li> <li>• Define the division as full service or transfusion only.</li> <li>• Enable electronic crossmatch at the division.</li> <li>• Populate International Council for Commonality in Blood Banking Automation, Inc. (ICCBBA) number for the division.</li> <li>• Define valid unit modifications for the division.</li> <li>• Assign workload codes for the division.</li> <li>• Configure testing.</li> <li>• Configure division.</li> </ul>	VBECS administrators trained to process all overrides and options not designed to change frequently. This role may be assigned temporarily to a blood bank supervisor at setup to configure the division.

The VA Wide Area Network (WAN) is protected from unauthorized access. The hardware running the VBECS application and its database will be installed at the Austin Information Technology Center [AITC (primary site)] and the Philadelphia Information Technology Center [PITC (disaster recovery site)]. Both sites are secure data centers where they are protected against unauthorized modifications and accidental damage.

## Application Architecture

VBECS operates as a discrete application and interfaces with other VA systems using Health Level 7 (HL7) and VistALink.

VBECS is a Windows-based application hosted on a server. Users access VBECS through a graphical user interface and other VA systems interface with VBECS through Health Level 7 (HL7) and VistALink.

The major goals of the application are:

- Improve the safety of blood/blood component transfusions by decreasing the risk of errors through effective use of current technology such as barcode scanning, retrieval of previous records to be used for comparison, and detection of inconsistencies in data input
- Improve the quality of patient care through evaluation of transfusion appropriateness and follow-up on the effectiveness of transfusions
- Provide comprehensive reporting capabilities for quality monitoring within the transfusion service and for clinicians

VBECS operates in accordance with FDA regulations regarding change to a medical device. Areas of operation that are part of the VistA Blood Bank v5.2 software medical device (except for the donor module) are contained in this encapsulated VBECS package. Patient orders originate and are received from the Computerized Patient Record System (CPRS) and Laboratory applications.

## ***Hardware and Infrastructure Architecture***


The VBECS application requires that hardware and system software serve five users in a standard configuration and up to twenty-five users in an integrated Veterans Integrated Service Network (VISN) environment. Figure 1: System Schematic describes the major system components: a Windows 2008 Server system (the execution environment for the VBECS application) and Windows workstations for access to VBECS through Windows Remote Desktop Connection. The VBECS server communicates with and exchanges information with VistA applications through Extensible Markup Language (XML) and Health Level 7 (HL7) messages over Transmission Control Protocol/Internet Protocol (TCP/IP) networking.

VBECS is installed in a virtualized network using the vSphere platform as the virtualization platform.

SQL Servers exist in Windows Server Failover Clustering with AlwaysOn cluster which consists of three nodes. The Primary and High Availability server reside at the primary site while a Disaster Recovery server resides at an alternate location:


- Primary server: This server fields all requests. Its data are replicated to the High Availability and Disaster Recovery servers.
- High Availability (HA) SQL server: This server provides database backup services through synchronous replication. Its data are guaranteed to be consistent with the Primary. It becomes the Primary should the original Primary server fail or become unreachable. Failover to this server is automatic.
- Disaster Recovery (DR) SQL server: This server resides at a remote site and provides backup database services through asynchronous replication. It becomes the Primary server should both the Primary and HA server fail or become unreachable. Failover to this server is a manual process.

**Figure 1: System Schematic**

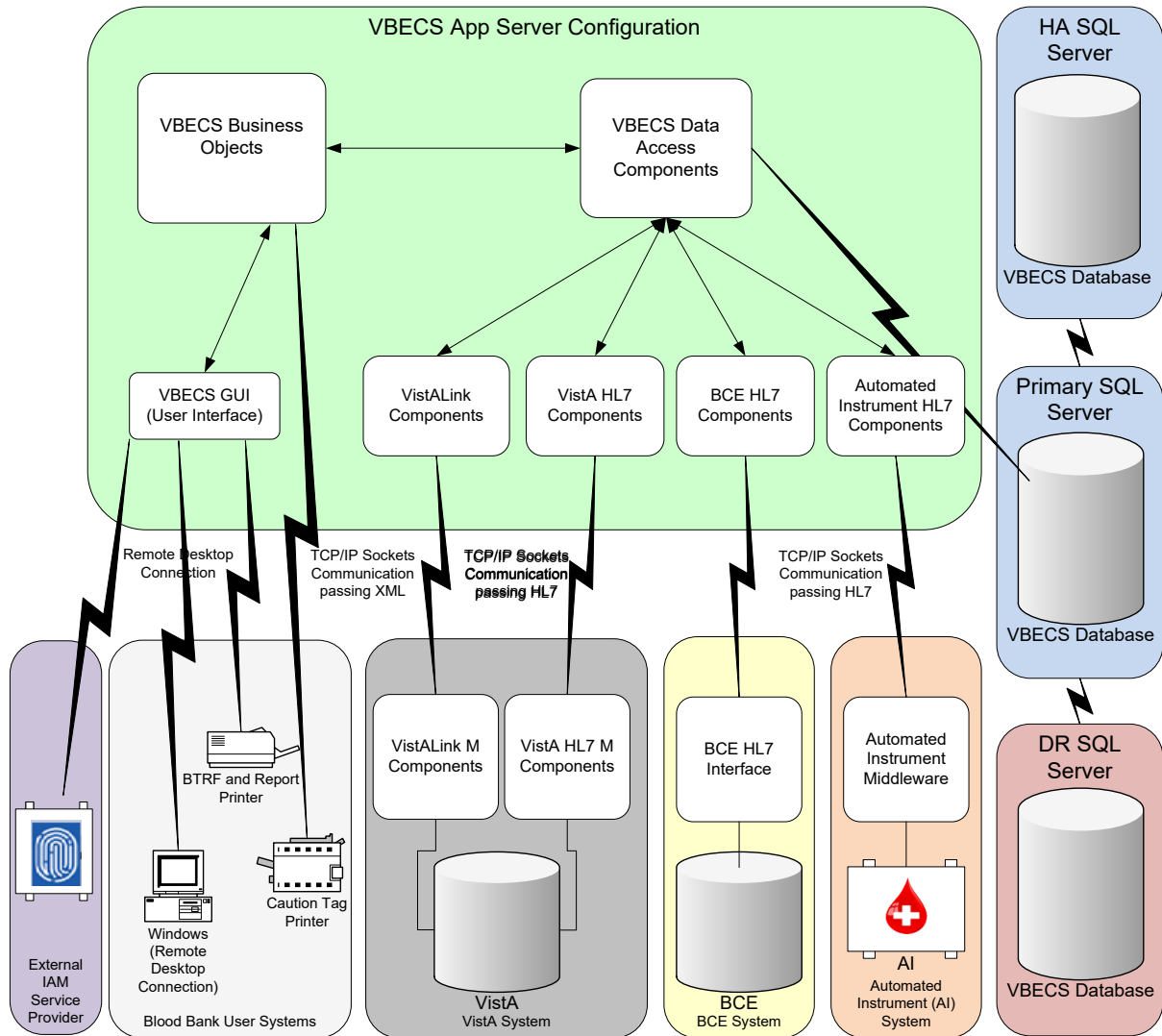

### Options That Require VistALink

These options require a connection through VistALink. When VistALink is not active and VBECS requires access to VistA data, VBECS prompts the user to activate VistALink. When VistALink is not available, VBECS warns the user and asks him to try again later.

 *Contact your local support staff immediately when VistALink is not available. VBECS, VistA and CPRS all depend on VistALink to exchange information.*

Main Menu: Orders

- Accept Orders: Pending Order List

Main Menu: Blood Units

- Edit Unit Information (used when editing patient restrictions)



- Return Issued Units To Blood Bank

Main Menu: Patients

- Post-Transfusion Information
- Return Issued Units To Blood Bank
- Issue Blood Components (to include a hospital location)

Main Menu: Shipments

- Incoming Shipment (used when restricting units to patients)

Main Menu: Tools

- Blood Products
- Component Classes
- Transfusion Complications
- Transfusion Effectiveness
- Workload Codes

Main Menu: Supervisor

- Document ABO Incompatible Transfusion

Main Menu: Reports

- Transfusion Complications Report
- Transfusion Effectiveness Report
- Transfusion Requirements Report
- Medication Profile (Reports or from the Patient Tool Bar icon)


Main menu window, icon in the bottom of the window:

- Display Order Alerts

## VBECs Patching

During the time a VBECs patch/upgrade is being applied, users connected to a VBECs server will see the VBECs shortcuts for that environment change to stoplight icons. If the user clicks on either shortcut, the message shown in Figure 2 is displayed. This message can be closed by clicking **Close**, or it will automatically disappear after 30 seconds. The shortcuts will revert back to VBECs shortcuts once the patch completes successfully. The environment is not ready for use until the VBECs shortcuts have been restored on the desktop.

**Figure 2: Example of VBECs Patching**

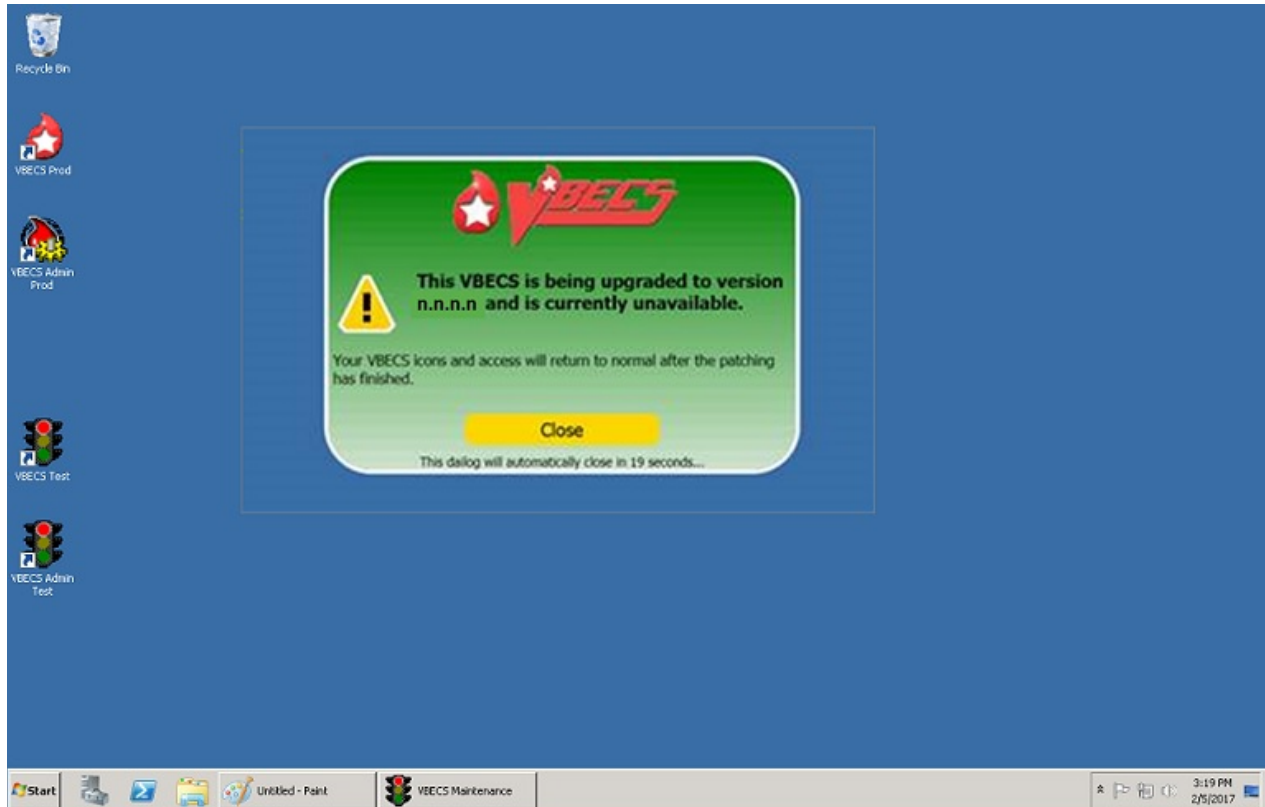

# Customer Support

## ***Problems?***

Contact your **Local** Office of Information Technology (OIT) or Laboratory Information Manager (LIM) if you encounter VistA or CPRS connection problems and for training support **before** contacting the Service Desk (SD).



**Please ensure local VistA Support contact information is available at all times. National Support will engage Data Center (DC) personnel as needed.**

**Problems with connectivity to VistA and CPRS may require personnel from DC with VBECS server administrator access and VistA IT support access.**

**If you experience an FDA reportable adverse event (patient death or serious injury) that VBECS may have caused or contributed to, contact the Service Desk directly to enter a ticket for Blood Bank software support.**

### **Service Desk Contact**

See **Customer Support** section of the *VBECS 2.3.1 Release Notes*.

For troubleshooting error messages in VBECS that read "Contact Your System Administrator", contact the Service Desk for VBECS Server support.

This page intentionally left blank.


# Using the Software

Using the Software explains:


- How to access VBECS and Vista
- What users will see on the screen
- What the icons and other symbols mean
- Which conventions are used in this guide
- How to exit VBECS

## Software Basics

- When a message window requires a user response and VBECS displays it behind the active window, the user must minimize the active window or click and drag dialog boxes to expose the message window.
- Press the **F1** key to view screen-specific online help.
- A user may click **Cancel**, **Close**, or  in the upper right corner at any time to exit without saving.
- All operations depend on the user being logged into the appropriate division.
- A user may click a column heading to resort the items in that column (Figure 3 and Figure 4).
- Reaction results entries are allowed for only those reagents defined as part of the reagent rack.
- Select a date from the calendar in the drop-down list to edit a date, or click each part of the date to edit the date and time (Figure 5 and Figure 6).
- Labels shown in figures are visual aids only and do not represent actual labels.
- VBECS does not offer a spell-check feature. (The user may write text in Microsoft Word, spell check the text, and then copy and paste the text into VBECS.)

 Although the copy-and-paste function is enabled, do not use it to enter patient identification information. It is highly recommended that the user scan this information when possible; otherwise, enter it.

**Figure 3: Click to Sort by Patient Name**

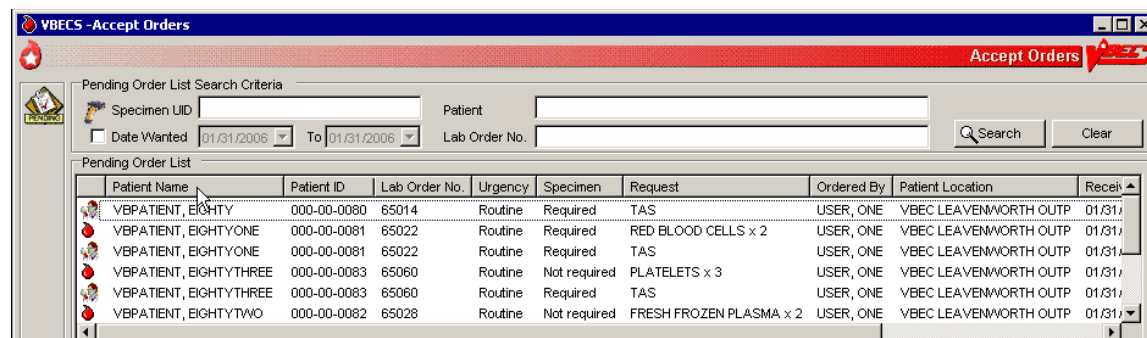

The screenshot shows the VBECS - Accept Orders window. At the top, there is a search criteria section with fields for Specimen UID, Patient, Date Wanted (01/31/2006), and Lab Order No. Below this is a table titled 'Pending Order List'. The table has columns for Patient Name, Patient ID, Lab Order No., Urgency, Specimen, Request, Ordered By, Patient Location, and Receipt Date. The 'Patient Name' column is highlighted, indicating it is selected for sorting. The table contains five rows of data for various patients, including 'VBPATIENT, EIGHTY', 'VBPATIENT, EIGHTYONE', 'VBPATIENT, EIGHTYTWO', 'VBPATIENT, EIGHTYTHREE', and 'VBPATIENT, EIGHTYTHREE'.

Patient Name	Patient ID	Lab Order No.	Urgency	Specimen	Request	Ordered By	Patient Location	Receipt Date
VBPATIENT, EIGHTY	000-00-0080	65014	Routine	Required	TAS	USER, ONE	VBEC LEAVENWORTH OUTP	01/31/2006
VBPATIENT, EIGHTYONE	000-00-0081	65022	Routine	Required	RED BLOOD CELLS x 2	USER, ONE	VBEC LEAVENWORTH OUTP	01/31/2006
VBPATIENT, EIGHTYTWO	000-00-0081	65022	Routine	Required	TAS	USER, ONE	VBEC LEAVENWORTH OUTP	01/31/2006
VBPATIENT, EIGHTYTHREE	000-00-0083	65060	Routine	Not required	PLATELETS x 3	USER, ONE	VBEC LEAVENWORTH OUTP	01/31/2006
VBPATIENT, EIGHTYTHREE	000-00-0083	65060	Routine	Required	TAS	USER, ONE	VBEC LEAVENWORTH OUTP	01/31/2006
VBPATIENT, EIGHTYTWO	000-00-0082	65028	Routine	Not required	FRESH FROZEN PLASMA x 2	USER, ONE	VBEC LEAVENWORTH OUTP	01/31/2006

Figure 4: Click to Sort by Request



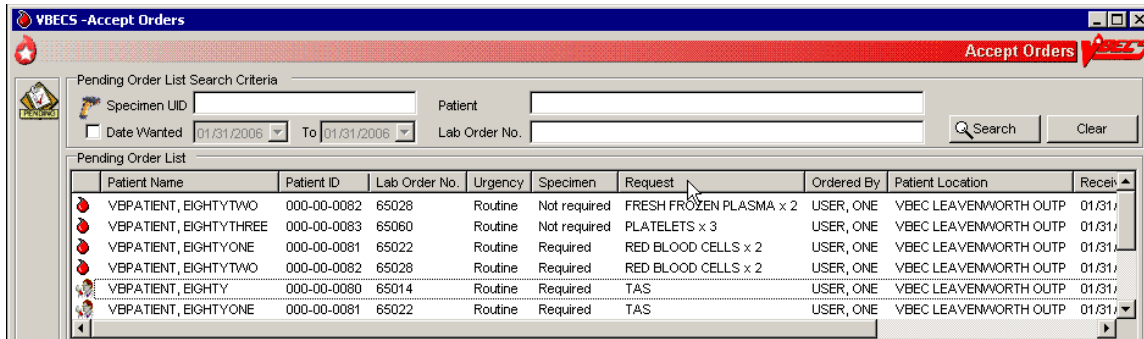


Figure 5: Select a Date from the Calendar

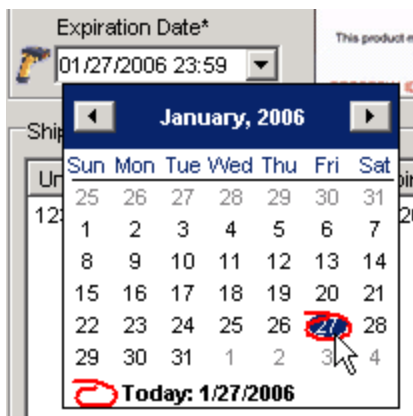

**Figure 6: Edit the Date and Time**

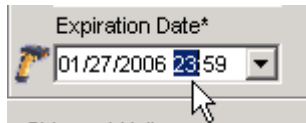

## **Commonly Used System Rules**

This section includes system rules that apply to several or all options.

### **Date and Time**

- Users must contact the Service Desk when the date or time is incorrect.
- The date, time, and time zone in VBECS is defined where the blood bank is located. Any requests for data from a particular blood bank database will reflect the time zone where the data was created and not the time zone where the report is generated.
- Refer to *Vista Blood Establishment Computer Software (VBECS) 2.3.1 Technical Manual-Security Guide* to change the time zone.

### **Locking and Time-Outs**

- In many cases, VBECS's locking feature prevents more than one user from accessing data. When a user attempts to access locked data, VBECS displays a message indicating which user has the locked data.
- A local configurable setting (Lock Inactivity Timeout field) specifies how long a user can be idle and in control of data being edited. VBECS warns the user 60 seconds before the lock inactivity period expires that he will lose priority for the data. When he responds within 60 seconds, VBECS clears the warning and resets the lock activity timer. Otherwise, VBECS informs him that his lock was released and he must reenter his changes.

### **Working with Data**

- VBECS will emit an audible alert when a user exceeds the maximum character limit for a field.
- The user may enter interpretations and reactions in full-service facilities. The user may enter only interpretations in transfusion-only facilities.
- Information displayed on an active window is updated with information from other options only when it is refreshed (i.e., closed and reopened) in the active window.
- When VBECS displays a scanner icon next to a data entry field, a user may enter or scan the data. The Caps Lock key must not be engaged.
- Entry of Codabar labeled blood units is disabled.
- VBECS captures all technologist IDs and the name of the technologist who displays and/or prints testing information on reports associated with entering test results on a worksheet.
- With each valid keystroke entry in the antibody screen and crossmatch test grids, VBECS moves the cursor from top to bottom in the far-left column within the cells in which results are being entered. The cursor returns to the top cell in the next column to the right and so on across the worksheet grid. The cursor stays in the reaction results cells by default (the technologist may navigate independently) until the last entry is made in the reaction result portion of the grid, then the cursor moves to the first (top) interpretation cell.
- To clear an entry in a test grid cell, press the **Backspace** key.

- VBECS allows selection of deceased patients without a warning message presented when VBECS has not received a patient death update message from VistA.
- VBECS displays the data that a user enters in a session. The user may edit the data and save them. VBECS warns the user that if he chooses to cancel using the cancel or “X” button or by leaving/refreshing the current page, VBECS will not save any data changes entered during the session. VBECS closes the form and returns the user to the main menu screen that may include unrelated open windows.
- VBECS displays and enforces active Transfusion Requirements (TRs) to all divisions in a multidivisional database.
- The user may not edit the title column of the data grid that specifies the reagent used in the test with a patient specimen.
- Unless otherwise indicated, data entry and retrieval are restricted to the division logged into.
- To enter data after VBECS has been unavailable, a user may enter a past date, time, and the technologist who performed the work.
- VBECS displays changes in reverse chronological order.
- VBECS calculates and displays the days (hours, if less than one day) remaining until a unit’s expiration based on the date and time of unit selection entered during modification.
- A scanned ISBT 128 expiration date barcode is identified by the presence of ‘“=>’ or ‘&>’ as the hidden identifier. Once the hidden identifier is removed, there are two possible formats of the barcode, one involving the date only and another involving date and time. If no time in the barcode, then 23:59 is assumed.
- Data are never deleted: they are saved indefinitely.
- Although a user may appear to select and delete data in gray fields, VBECS does not save such edits.
- A column’s width may be adjusted or hidden for a session. This view returns to the default presentation when the view is opened again.
- The ABO/Rh confirmation worklist may be sorted by Unit ID, ABO/Rh, Product Type, Component Type, or Date/Time Received. When displaying the Date/Time Received, seconds are not displayed, but are used to sort the worklist when selecting Date/Time Received for the worklist sort criteria.

## Searching the Database

- A user may search for a patient by:
  - Patient’s last name (symbols such as an apostrophe and/or dash are not allowed)
  - Patient’s ID (omit hyphens) (“Full”)
  - Patient’s last initial and last four digits of the ID (“Last 4”)
  - Patient’s last four digits of the ID (“Last 4”)
- For example, for a patient whose last name is TEST with an ID of 000-11-2222:
  - Patient's last name: user enters "TEST"
  - Patient's ID: user enters "000112222" (no dashes or spaces)
  - Patient's last initial and last four digits of the patient ID: user enters "T2222" (no dashes or spaces)
  - Patient's last four digits of the patient ID: user enters "2222"

The system displays all patients that fit the search criteria, allowing the user to select a patient from a pick list.



**Note:** Searching by last four digits on the patient ID in VBECS against the VistA database will return matches on the last four digits or the first four digits. The following search formats in VistA are not allowed in VBECS:

- Last name, first name
- Various patient ID lengths other than 4 or 9 digits
- VistA Patient ID
- A user must scan or enter a unit ID and a product code to select a unit from inventory.
- When VBECS does not find a unit record, it displays an error message stating that the unit was not found and instructing the user to reenter unit information. A user may close this message, click **Clear**, and enter another unit ID and product code.
- VBECS lists all or part of a division's blood component inventory, based on system rules. A user may enter a varied combination of unit information to create a customized list of selectable units, including:
  - o Unit ID
  - o Product type
  - o ABO/Rh
  - o Special testing (CMV Neg, SC Neg, or equal to or greater than (= >) 2 mL RBC Contamination)
  - o RBC antigens (available only for the RED BLOOD CELLS component class)
  - o Received date range
  - o Expiration date range

VBECS displays this unit information in the selection list whether full inventory, subset, or filtered subset:

- o Unit ID
- o Unit ABO/Rh
- o Product short name
- o Product code
- o Unit status
- o Unit expiration date
- o Donation type
- o Restricted for patient name, when available
- o Date and time received (standard format)


## Testing


- >>>>>> Administrator/Supervisors may configure reagent racks to use a tube, solid phase, or gel method.
  - o VBECS allows a user to perform QC testing at AHG and later perform patient testing at all phases using a rack configured for AHG only.
  - o VBECS does not verify that the phase of testing matches the reagents used for testing.
  - o Users working with reagent racks containing different reagents must do so in accordance with local policy and procedures.
- Reaction result cells in the data grid contain only symbols. A legend of the symbols and their meanings, specific to the data grid and its truth tables, is located in the bottom left corner of the form and is available to the user throughout data entry.

- Entry of a symbol (not a number or letter) into a test grid reaction result cell or interpretation field will not elicit a system response. The entry of the symbol is not allowed and does not overwrite the previous entry, if any. See Table 2 for valid observed test result entries.
- After entering a canned or free-text comment in the testing comment field of the testing grid, the user must press **Enter** to store the comment.
- The user may enter canned comments in comment cells by selecting from a pick list, or enter free-text comments by selecting “Other.”

**Table 2: Valid Observed Test Result Entries**

Entry	VBECS Translation	
H	Hemolysis	Positive
W	Weak	Positive
F	Mixed Field	Positive
M	Microscopic	Positive
1	1+	Positive
2	2+	Positive
3	3+	Positive
4	4+	Positive
X	Not Tested	Not Tested
0	No Agglutination	Negative
R	Rouleaux, No Agglutination	Negative

- When recording test results:
  - In the Test Details window, click a check box to select a rack.
  - In the Date Tested field, select the default date and time (current date and time) for testing performed, which the user may edit to a past date and time.
  - In the Tested By field, select the default identification (current user) for the testing technologist or another user from the drop-down menu.
  - Click the appropriate radio button to determine the grid configuration for antibody screen and crossmatch grids [All phases or Anti-Human Globulin (AHG) only (pre-checked default varies with local configuration)].
  - When an automated instrument is used, click the **Using Automated Instrument** check box, enter the instrument name in the Instrument Name field, and click the **Instrument QC'd?** check box, as appropriate.
- VBECS requires units in ICCBBA component classes Red Blood Cells, Whole Blood, Granulocytes and Leukocytes to have ABO confirmation testing (forward typing), and Rh confirmation testing for units labeled as Rh negative.
- The user may not edit VBECS-populated grid worksheet fields that contain unit or patient data.
- The user may sort worksheet entries only before beginning data entry, preferably during unit selection. When the user wishes to sort a worksheet after entering the first reaction result, he must exit without saving the work and begin again.
-  When a user enters an invalid symbol in a cell, VBECS emits an audible alert and prevents the cursor from moving to the next cell.
- VBECS moves the cursor to the comment cell automatically, when applicable; the user may also use the mouse to move to another cell.
- VBECS displays **E** (error) at the beginning of invalid rows and **V** (valid reagent) or **W** (reagent will expire within 24 hours) at the beginning of valid rows. These warning icons change as the user updates information. When the user holds the mouse over an icon, VBECS displays the message associated with the truth table interpretation (as with a tool tip).
- VBECS compares user-entered interpretations with truth table data.
- The user performs tests and enters data simultaneously.
- Throughout this option, the user may view the patient's VBECS demographics, including the information entered in the patient Special Instructions (SIs), Transfusion Requirements (TRs), and other clinical lab results related to the transfusion through CPRS.
- The historical ABO/Rh display includes the date and division of creation or indicates that the record is from the VistA database conversion.

- The test record captures the entering and testing technologists' ID and the display/print technologists' name, the division, date, and time of the save, the content (details of the verified testing), QC rack ID (if not by automated instrument), test method, automated test instrument information, if indicated, data entries, interpretations, and comments. (When the configuration changes in Configure Testing after the test data are entered, the record is retrievable as originally configured.)
-  When a site is defined as "full service" and daily QC was not performed on the selected rack or the Not in Use Today check box was checked for the current 24-hour period (beginning with the time set for the alert defined in Configure Daily QC), VBECS emits an audible alert, warns the user, and asks whether he wishes to continue. **Yes** requires a comment and captures details for inclusion in the Exception Report (exception type: QC not performed on rack used for testing). **No** allows the user to select a different rack or cancel and return to the main menu.
- Repeat tests are not billable. Repeat tests are not available to the patient chart unless they are part of a corrected report test.
- When reagents or antisera are within 24 hours of expiration (23:59 on the expiration date), VBECS warns the user when the reagent will expire and asks whether the user wishes to continue. **Yes** allows the user to continue testing. **No** moves the cursor to a cleared lot number cell, which is ready for the user to enter a new lot number.
- The unit record will capture the entering tech ID, and display and/or print the entering tech name; the selecting tech ID, and display and/or print the selecting tech name; division, date, and time of the save; and the content (details of the unit selection), if indicated, data entries, interpretations, and comments.
- VBECS allows a user to save partially completed tests and allows the same or different user to complete the tests later. A first or second user may enter only new results and may not edit prior entries. VBECS allows the user to save partially completed testing. The user may return to complete the pending tasks, but may not edit saved interpretations.

## Creating and Viewing Reports

- Click  in the Include Report Sections area to select all report sections (Figure 7).
- Click  in the Include Report Sections area to clear all report sections (Figure 8).

Figure 7: Click to Select All Report Sections

The screenshot shows the 'VBECS - Division Transfusion Report' dialog box. The 'Report Date Range' section has 'Start Date\*' set to 01/26/2006 and 'End Date\*' set to 01/26/2006. The 'Report Type\*' section has 'Treating Specialty Report' selected. The 'Include Treating Specialties\*' section has a checkmark in the top right corner, and all specialty checkboxes are checked: ACUTE PSYCHIATRY (<=45 DAYS), ALCOHOL DEPENDENCE TRMT, ALLERGY, BLIND REHAB, BLIND REHAB OBSERVATION, CARDIAC-STEP DOWN U, CARDIOLOGY, DERMATOLOGY, DOD BEDS IN VA FACIL, and DOMICILIARY. The 'Report Compilation\*' section has 'Create Report Now' selected. The 'Print Date\*' is 01/27/2006 14:21 and 'Printer Name\*' is empty. 'OK' and 'Cancel' buttons are at the bottom.

Figure 8: Click to Clear All Report Sections

The screenshot shows the 'VBECS - Division Transfusion Report' dialog box. The 'Report Date Range' section has 'Start Date\*' set to 01/26/2006 and 'End Date\*' set to 01/26/2006. The 'Report Type\*' section has 'Treating Specialty Report' selected. The 'Include Treating Specialties\*' section has a checkmark in the top right corner, but all specialty checkboxes are unchecked: ACUTE PSYCHIATRY (<=45 DAYS), ALCOHOL DEPENDENCE TRMT, ALLERGY, BLIND REHAB, BLIND REHAB OBSERVATION, CARDIAC-STEP DOWN U, CARDIOLOGY, DERMATOLOGY, DOD BEDS IN VA FACIL, and DOMICILIARY. The 'Report Compilation\*' section has 'Create Report Now' selected. The 'Print Date\*' is 01/27/2006 14:21 and 'Printer Name\*' is empty. 'OK' and 'Cancel' buttons are at the bottom.

- In the Report Compilation area, click the **Create Report Now** radio button and **OK** to view the report, or click the **Schedule Print Job** radio button and **OK**, then enter or select a print date and printer name, then click **OK** to view and print the report.
- Reports are printed only in their entirety.
- A single-division report includes information from one division.
- When a site is a single division or a user does not have multidivisional access and a report has a multiple-division option, the user may not select a multidivisional report.
- VBECS maintains a cumulative history of all transactions, including testing and status changes, for every blood unit entered in the database (Unit History Report).
- VBECS captures changes to verified data for inclusion in an audit trail report (Audit Trail Report, Unit History Report, Patient History Report, and Order History Report).
- VBECS displays patient ABO/Rh results in reverse chronological order, regardless of division, then from the VistA database conversion. When no ABO/Rh records exist, VBECS displays "NR" (no records). When there are discrepant ABO/Rh results from the same universal ID (UID) or from two specimen UIDs, VBECS displays the results recorded just prior to the latest result that created the discrepancy and warns the user.
- When a search will return a large number of items on the report, VBECS notifies the user that the report will be longer than 50 pages and asks whether he wants to display and perhaps print the report. **Yes**

notifies the user that the process will take a few minutes. **No** clears the screen and returns to the option's main menu.


- When no information that fulfills search criteria is available, VBECS notifies the user that no information is available, per entered search criteria.
- The user may change the default start and end dates (the previous day). The end date and time must be later than or the same as the start date and time. Report activity begins at and includes midnight on the start date, and ends at and includes midnight on the end date. When the date range includes the current date and time, VBECS displays "Preliminary" in the header on each page of the report.
- VBECS displays headers on every page of a report; it displays search and sort criteria and date ranges at the end of the report. VBECS identifies whether reports are generated from a test environment or from a production environment.
- A user may schedule a report to run and print in the future. When the activity date range includes a date that precedes the date and time of printing, VBECS processes the data on the date and time the report is scheduled to print. When a date range is greater than 30 days, VBECS warns the user and asks whether he wishes to continue.
- A report requested to print in the future prints per Central Time (CT).
- When printing reports exceeding 20 pages, allow the report to load in the preview window before selecting "print". This prevents the VBECS session from freezing, and causing issues with printing future documents.
- Local record retention policy instructs users to print, review, document, and save the Audit Trail, Exception, and Testing Worklist Reports. The reports contain only data entries generated in the user's login division. Each entry includes an area for supervisor acknowledgement (signature or initials and date of review) and comment.
- The search functionality (binoculars icon) may not work in a displayed report.
- VBECS displays the patient historical ABO/Rh, antibodies identified, antigen negative requirements, and transfusion reaction history (date, division of testing, whether from the VistA database conversion), and whether the historical ABO/Rh was changed.
- When a patient has a history of an ABO/Rh change justified through Justify ABO/Rh Change, VBECS displays "History of justified ABO/Rh change" next to the historical ABO/Rh information.

Reports in VBECS are created using a Microsoft technology called SQL Server Reporting Services (SSRS). Reports are rendered on screen using ReportViewer control. Reports are not deployed on the reporting server but as a part of the main application. All reports can be exported. The user has the option to export reports in three different formats (PDF, Excel and/or Word).

## Report Toolbar



Each report in VBECS will have a toolbar on top that allows the user to interact with the report (Figure 9). The following sub-sections elaborate on the controls found on the Report Toolbar. It is suggested that users create a local directory for exporting their reports (see the *Vista Blood Establishment Computer Software (VBECS) 2.3.1 Technical Manual-Security Guide*, Configure the Report Share and the Configure a Shortcut to the Report Share sections for additional information).

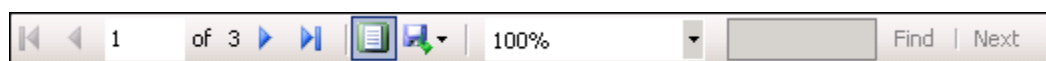**Figure 9: Report Toolbar**



## Navigation Arrows and Report Pages

Navigation arrows allow the user to move to the next or previous page of the report. If the user is on the first page of the report, the arrows pointing to the left will be disabled. If the user is on the last page of the report, the arrows pointing to the right will be disabled.


The first arrow button  moves the report by one page forward (or backward for left facing arrow). The second  moves the report to either the last or first page (in case of the left facing arrow). The number X of Y tells user which page is currently being viewed (X) and what is the total number of pages (Y).

Using arrow keys may cause erroneous “Invalid Key Pressed” messages or swap values while entering control results. Use the mouse to move between cells instead of using the arrow keys. Verify the control cell entries are correct before saving.

## Report Layout

There are two ways (layouts) in which reports can be displayed. By default, reports are displayed in Print Layout. It allows reports to appear on the screen exactly how they will look when printed. However, when reports are displayed in Print Layout mode, the Report Search function (see Report Search) is disabled. In order to be able to use Report Search, the user has to turn off Print Layout mode by pressing this icon:

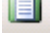


. When Print Layout is disabled, the icon will look like this: . It can be pressed again to return to Print Layout mode.

Page size of reports not displayed in Print Layout mode differs from the regular letter size (8.5in. x 11in.) and that is why the total number of pages for the same report displayed in different layouts may differ.

## Report Zoom-in/Zoom-out

Clicking the drop-down menu showing 100% will give the user a choice of the report display size (Figure 10).

**Figure 10: Report Zoom-in/Zoom-out Control**

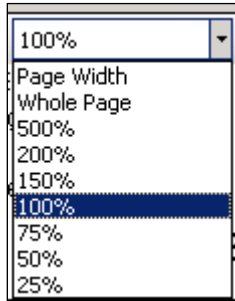

The default value is 100%. The option Page Width will resize the report to fit the screen horizontal size. The option Whole Page will display the entire page on the screen. Selecting an option less than 100% will zoom out the report making the font smaller and allowing more data to fit on the screen. Selecting an option higher than 100% will zoom in the report making the font bigger. If the report does not fit on the screen, the user will be able to use scroll bars to scroll up or down and left or right to view the entire report.

## Report Search


This option allows the user to search for specific text in a report and is only enabled if Print Layout is disabled (see Report Layout). By default, the **Find** and **Next** buttons are disabled. Once the user enters text in the search box, the **Find** button becomes enabled (Figure 11).

**Figure 11: Report Search Control**



Clicking the **Find** button will search the entire report. If the searched text is found, the report will display the page where it was found. The **Next** button will then become enabled allowing the user to search for subsequent instances of the search string. If the searched text cannot be found, the user will be presented with the message shown in Figure 12.

**Figure 12: Report Search Warning Message**

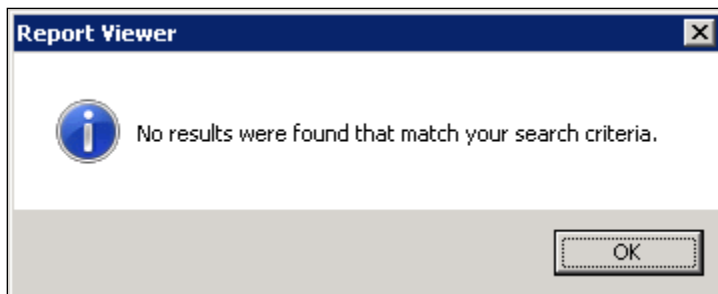



## Report Export

For all sites, reports will be saved in subfolders named after Division Code. VBECS will check to see if the subfolder exists. If the subfolder does not exist, it will attempt to create it. If the creation fails, it will attempt to save the report to the default report directory (Figure 14).


The user has an option to export the report in three different formats: PDF, Excel and Word. The export to PDF creates a report that is Read-Only (may not be modified) and will match the report that users can see directly in VBECS in terms of data and format. The export to Excel and Word creates a report that may be modified. **VBECS reports exported to Excel or Word can be altered by a user. Only copies printed directly from VBECS should be shared with clinicians to ensure data integrity.**

**The data output from VBECS is not controlled by the medical device and must be strictly regulated by local facility personnel. Many of the reports contain patient sensitive information and must be treated as such. Use of the exported reports, including alteration of data, is the responsibility of the user.**


Export of the reports to Excel and Word comes with known problems. Reports exported to Excel or Word contain the same data as reports that can be seen directly in VBECS; however, their formatting may differ from the original report. Not all reports can be exported to all the identified formats. Table 3 lists the reports and their exportable formats.

**Table 3: Recommended Exportable Formats for Reports**


<b>Report Name</b>	<b>PDF</b>	<b>Word</b>	<b>Excel</b>
Maintain Reagent Inventory	Yes	Yes	No
Testing Worklist Report	Yes	Yes	No
Blood Availability Report	Yes	Yes	No
Division Transfusion Report	Yes	Yes	No
Blood Bank Supplies Report	Yes	Yes	No
Unit History Report	Yes	Yes	No
Patient History Report	Yes	Yes	No
Other VBECS reports	Yes	Yes	Yes

By clicking on , the user has the option to select and export to an Excel, PDF or Word file (Figure 13). Choosing any of these options will open the dialog shown in Figure 14.

**Figure 13: Report Export Toolbar**

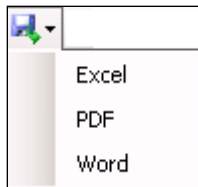

**Figure 14: Example of Report Export Dialog for Blood Bank with Division Code 589**

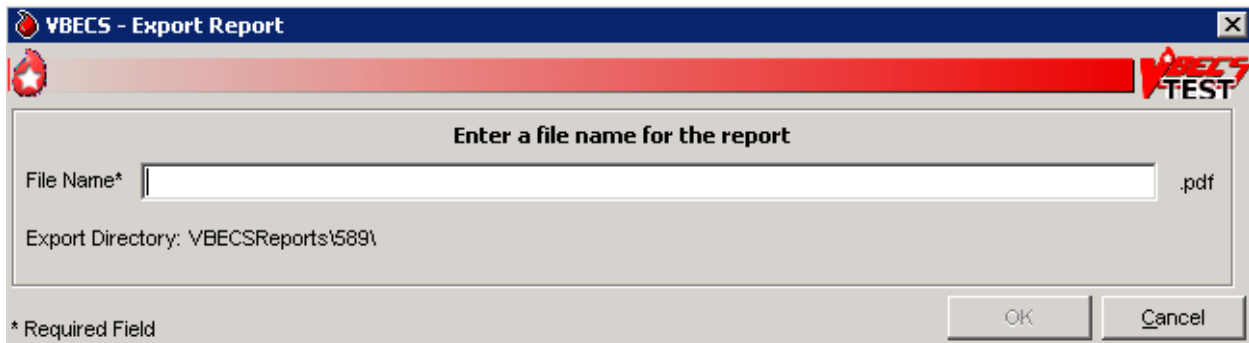


This dialog allows the user to specify the name of the file to which the report will be exported. The extension of the file is pre-filled for the user and depends on which export option was selected (.xls for Excel, .pdf for PDF or .doc for Word). The file name length is limited to 200 characters and will not accept special characters as part of the name (e.g., \ / ? % \* | : ' " < >).

If the user attempts to use a special character in the file name, it will be ignored and not appear on the screen.


Once the user enters a proper file name, the **OK** button will become enabled. Clicking the **OK** button will present the user with the message shown in Figure 15.

**Figure 15: Report Exporting in Progress**

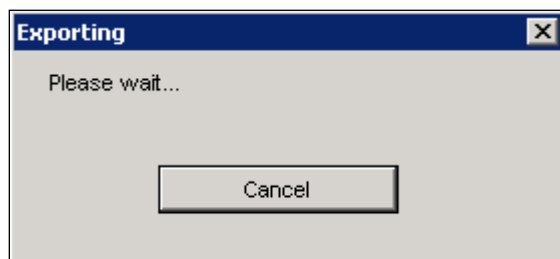

Clicking the **Cancel** button will stop the export and the user will be reverted back to viewing the report. If the export is successful, the user will be presented with the message shown in Figure 16.


**Figure 16: Example of Report Exporting Success**

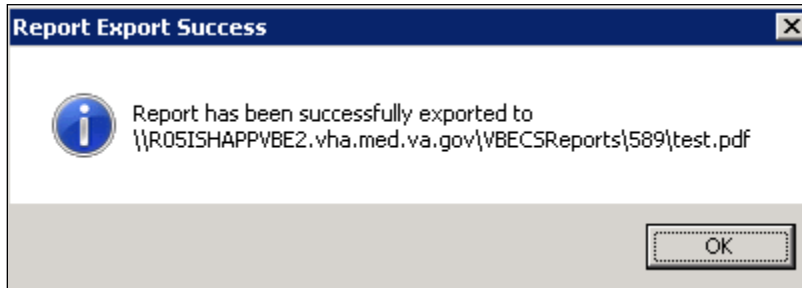

This dialog shows the full path to the report and its name allowing the user to find it quickly using Windows Explorer.

If the user enters the name of a report that already exists, the system will present the message shown in Figure 17.

**Figure 17: Report File Already Exists**

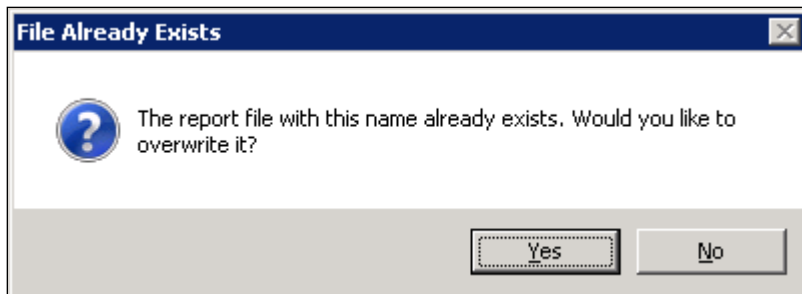

Clicking **Yes** will overwrite the existing file. Clicking **No** will allow the user to specify a new name.

## Copying an Exported Report


Here are some tips for copying a file or folder of files from one location to another.

### Before you begin:

- Verify that you or the person copying the file has permissions to access both the folder you are copying files from and the folder you are copying the files to.
- Determine the type of file (i.e., PDF, Word or Excel) you want to use for your local document before you export. Do not change the file type during this process.

You may have a shortcut defined on your desktop that opens the report export folder on the server (Figure 18). This allows you to open files on the server on your local PC (see the *VBECS 2.3.1 Technical Manual-Security Guide* for additional information).

**Figure 18: Example of D: Drive**

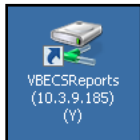

**Option One:** Drag and drop a copy of the file in “MY LOCAL FOLDER”.

This works well when you do not need to change the file name when placing it in your local folder.


- 1) When in ‘Windows Explorer’ (not Internet Explorer), left-click on the file and while holding down the mouse button, drag the file to the destination folder.
- 2) Release the left mouse button when hovering over “MY LOCAL FOLDER”. The file may copy without any notification or may have a copying window appear if it is a large file.
- 3) Skip to Step 3 of Option Two.

**Option Two:** Open the document and save a copy to another file.

- 1) Open the document from the server folder.
- 2) Select the **Save As** option and then the local folder, which in this example is called MY LOCAL FOLDER.


You may edit the file name at this point. The name you use for the file in your local folder should be something that allows you to manage those files as you use them. You can name the exported file accordingly or edit the file name in your local folder. A generic file name allows you to save and refresh the information. It may be appropriate for something like the Cumulative Transfusion Requirements report (Figure 19).

**Figure 19: Example of File Name**

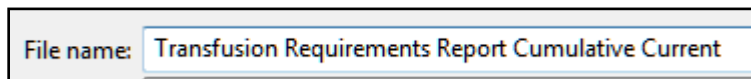


You may also opt to save with a date in the file name to allow a snapshot report for historic purposes (Figure 20).

**Figure 20: Example of Dated File Name**

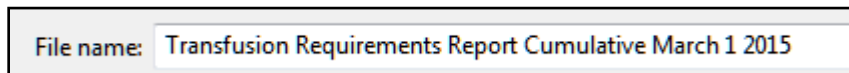

- 3) Verify that "MY LOCAL FOLDER" contains the saved file. You can edit and save to this folder with those changes (Figure 21). Unlike the server file, you may also delete this file.

**Figure 21: Example of Saved File**

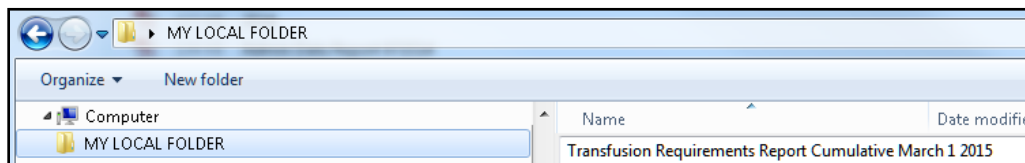

**Note:** The copy of the file you save to your local folder is separate and does not update the file in the server folder.

## Using VBECS

VBECS is a Windows-based application.

One VBECS system is implemented for each VistA database.

Subject matter experts (SMEs) at user sites will help implement VBECS and train blood bank staff and other authorized users.

The VBECS SharePoint site contains training, Frequently Asked Questions (FAQs), Technical Bulletins, ListServ messages and sign up, etc. to support the implementation and use of VBECS REDACTED. Users may access links to VBECS training modules and frequently asked questions, and join a ListServ (an email distribution list to manage discussion groups) for announcements and updates on the VBECS SharePoint site. Contact information is provided for comments and questions.


## Documentation

Users may retrieve VistA documentation from the VistA Documentation Library (VDL) Web page REDACTED


Using Online Help

Select **VBECS Help** from the Help menu (Figure 22) and a PDF file of the VBECS 2.3.1 User Guide opens (Figure 23).

**Figure 22: VBECS Help**

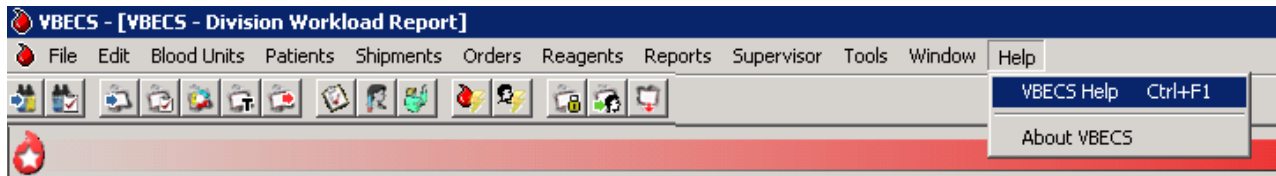


**Figure 23: Example of Opening VBECS User Guide PDF File**

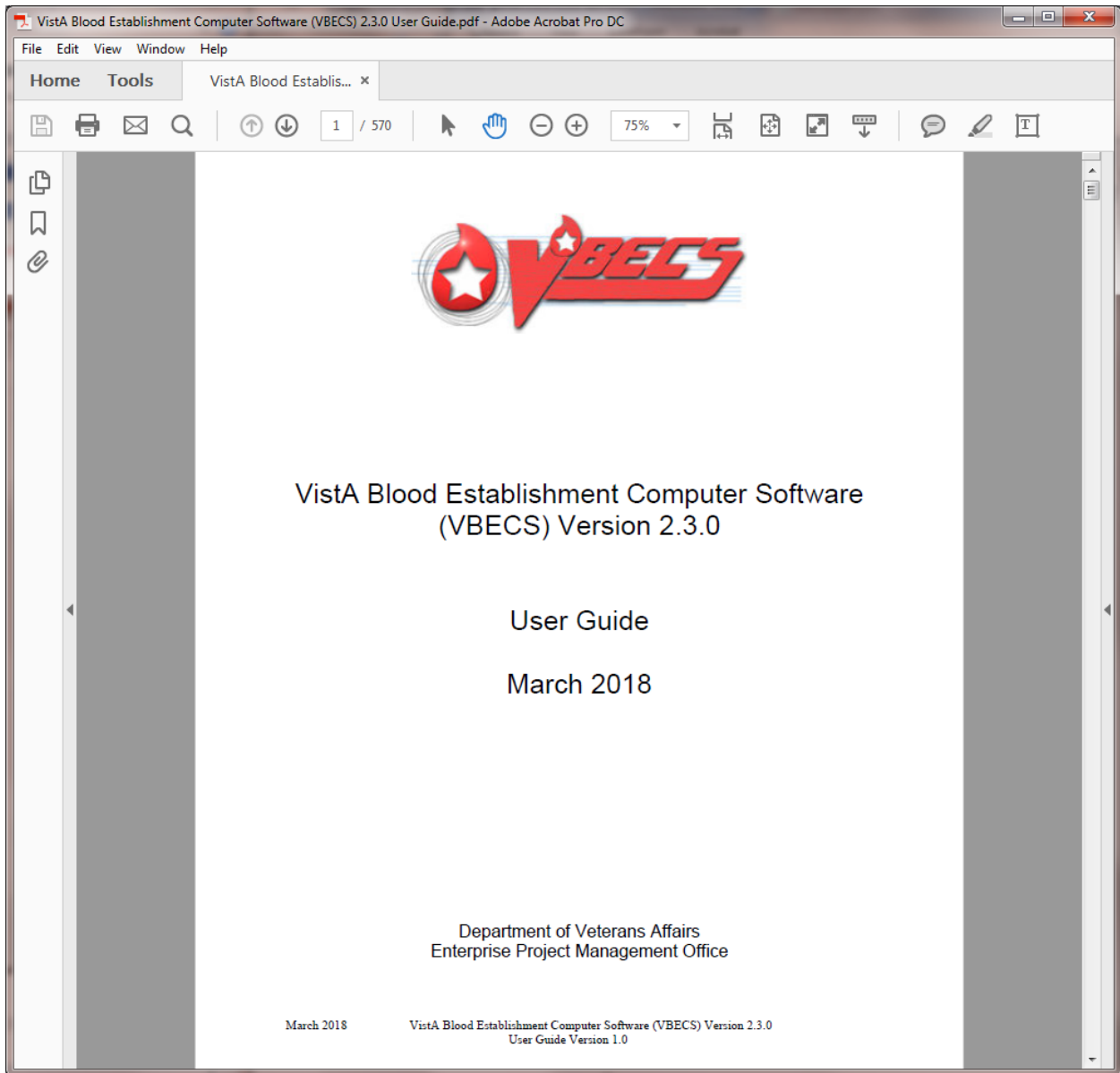

Press the **F1** key to open help file (Figure 24).

**Figure 24: Example of Pressing F1 Key From A Specific Option**

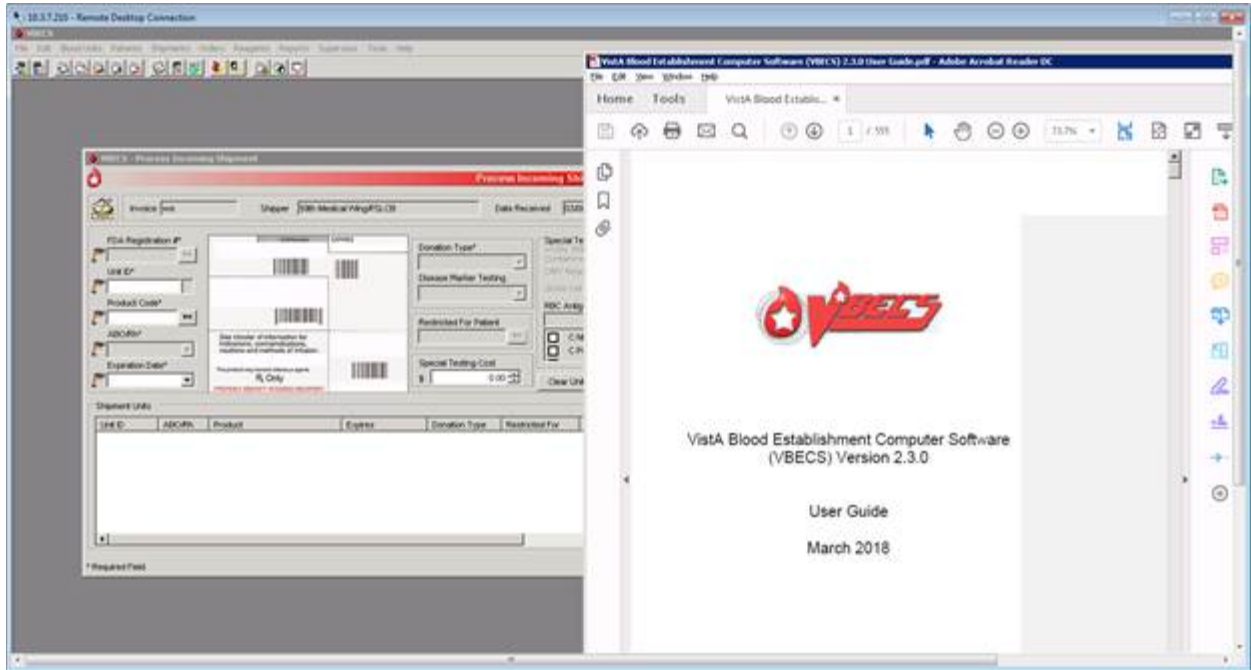



## Searching Using the Table of Contents

Scroll through the Table of Contents and press **Control** and click any Table of Contents topic to go to the desired page. Alternatively, click **Ctrl + F** and search for desired content.

## Viewing the VBECS Version Number

In earlier VBECS patch releases, the user documentation referred to the VBECS version in a 4-digit format (e.g., 2.1.0.2 – where 2.1.0 represents the patch version and the last digit (2) is the patch build number). The build number is used by VBECS Product Support for diagnostic and troubleshooting purposes.

The VBECS build number was changed starting with the VBECS 2.2.0 patch release. The new patch release number is now composed of four sets of numbers (a.k.a., elements), separated by periods in this format: “majorminor.patchcontent.days.time” (e.g., 0202.0001.5746.31276). The elements are defined in this manner:

- The *major* element is a two-digit number that is incremented when the functionality is significantly altered.
- The *minor* element is a two-digit number that is incremented for all other fixes or maintenance.
- The *patch* element is a two-digit number that is for emergency cases of a high priority that may impact the application’s production environment execution or processing capabilities. These changes are usually time sensitive and cannot be postponed to the next planned release.
- The *content* element is a two-digit number used to identify the need for a new build (e.g., scope increases, defect discovered, etc.).
- The *days* element is a four-digit number that is incremented daily and represents the number of days since January 1st 2000.
- The *time* element is a five-digit number that is calculated by taking the number of seconds since midnight local time (without taking into account time zone adjustments for daylight saving time), divided by 2.

The first two numbers [*<MajorMinor>* and *<PatchContent>*] tell us the content/scope that we are building to (e.g., 0202.0001 which can be also referred to as 2.2.0.1).

The last two numbers *<days>* and *<time>* are automatically generated by the software used to perform the code builds and are unique for each build. These last 2 numbers along with the first two numbers formally identify the latest official build (e.g., 0202.0001.5746.31276).

When it comes time to assign a new version number, the first four elements (major, minor, patch, content) are dependent on each other which means that when any of these four elements is incremented, all subsequent elements are reset as follows:

- At a major release start, the major number is incremented by one, the minor and patch numbers return to zero, the content number always returns to one, and the *days* and *time* numbers are regenerated using the build time (e.g., 0203.0303.*days.time* → 0300.0001.*days.time*).
- At a minor release start, the major number remains the same, the minor number is incremented by one, the patch number returns to zero and the content number returns to one, and the *days* and *time* numbers are regenerated using the build time (e.g., 0203.002.*days.time* → 0204.0001.*days.time*).

- At a patch release start, the major and minor numbers remain the same, the patch number is incremented by one, the content number returns to one, and the *days* and *time* numbers are regenerated using the build time (e.g., 0204.0003.*days.time* →0204.0101.*days.time*).
- When new content/scope is being added, the content number is incremented by one (e.g., 0204.0003.*days.time* →0204.0004.*days.time*).

This patch release number methodology is specifically used for software identification while VBECS document titles use various elements of the software version number.


The revision letter tracks database-only updates (e.g., blood product table and canned comment updates). The revision letter is normally a single alpha character (e.g., C), but can be two characters (e.g., AA, AB, AC) in the unlikely event that more than 25 database updates are made before a code change is implemented. The revision letter starts at A with each new code change and is incremented to B when the first database-only update is made. The revision letter is then updated by one alpha character for every successive database-only update until a new code change is implemented at which time the revision letter reverts back to A. The version submitted for system testing will be revision A, but the version customers receive can be revision A, B or higher.

The VBECS **Help, About VBECS** window (Figure 25 and Figure 26) display the release number (e.g., 2.3.1), the formal build number (e.g., 0203.0101.6956.2414) and the revision letter (e.g., A).

**Figure 25: About VBECS**



Figure 26: Example of VBECS Version and Build Numbers

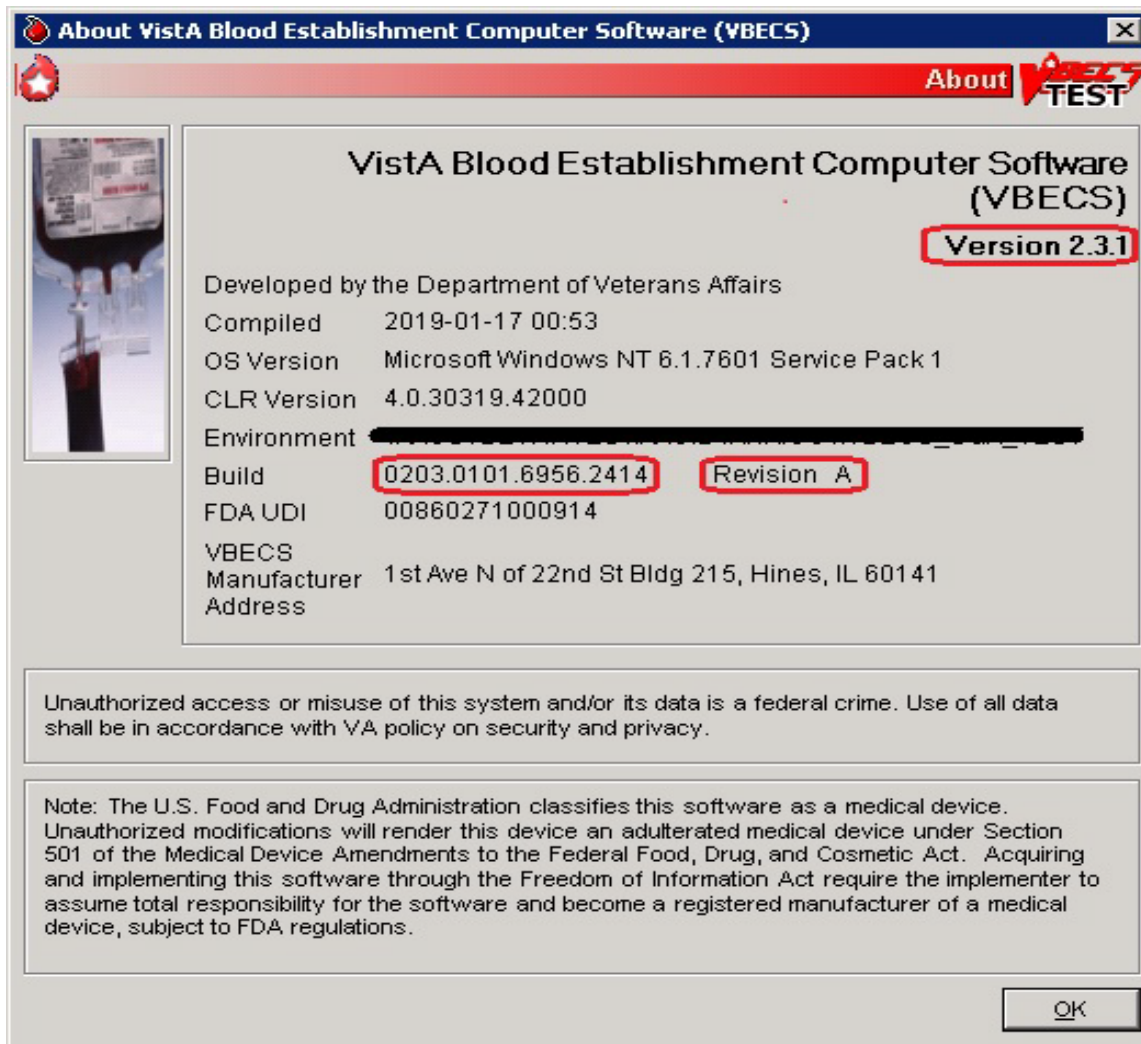

## VistA Records in VBECS

Historic VistA blood bank records were transferred to VBECS and validated during the database conversion. VBECS handles these records in accordance with system rules:



*Patient names in VistA that exceed the database standard for File #2 (30 characters) or do not have a last name do not convert to VBECS.*

- VBECS may display patient information differently from VistA. For example, VBECS displays patient names as “LAST NAME, FIRST NAME, MIDDLE NAME or INITIAL, SUFFIX.”
- A patient name will be truncated in the following screens when one of the name fields (Last, First, or Middle) exceeds 22 characters. The full name is displayed elsewhere on the screens identified or it is provided as read-only information and is not used to make decisions for the selected patient.
  - Blood Unit Details tab
  - Patient Specimen Selector control
  - Patient Order Selector control
  - Patient Order Selector for Report control
- When only a few non-specific characters are entered in the patient search field, no matches are found and the option times out without displaying a message to the user. Enter the full last name, or the full patient ID, last name, last name initial, and last four digits of the patient ID (SSN), or the full patient ID. Entering the standard patient search criteria (full SSN) avoids the potential for confusion.
- VBECS will only display the first 25 characters of the name.
- VBECS displays historic ABO/Rh types from the database conversion, but doesn't use them for issuing blood or for comparing test results.
- VBECS truncates Special Instructions (SIs) text from the VistA database conversion.
- SIs from database conversion entries are labeled as such.
- VistA SIs from the database conversion may include component or antigen negative requirements. The VistA SIs remain SIs and are not enforced unless they are entered in the Transfusion Requirements tab in Special Instructions & Transfusion Requirements.
- Patient antibody identifications from VistA were included in the database conversion.
- Antigen negative requirements properly set in VistA were transferred to antigen negative requirements in VBECS.
- VBECS informs the user when the displayed historical ABO/Rh, transfusion reaction history, and/or antigen typing result from the database conversion.
- Transfusion reaction types without their associated units were transferred to VBECS.

## **Accessing the System**

### **Log into VBECS and VistA**

A user may work in only one division at a time. After login<sup>1</sup>, a user with access privileges in several divisions may change divisions. For users with access to a single division, an option to change divisions will not be available. VBECS allows access only to valid divisions associated with the user.

#### **Assumptions**

- There is not an active VistALink connection in VBECS/Administrator.
- VistA is set up, operating normally and accessible via the network.
- The user is defined in VistA, has a DUZ, and connectivity to VistA can be established.
- The user must be defined in the VistA new person file and have valid Access and Verify Codes.
- The user's division is configured in both VistA and VBECS. (See Update User Roles)

#### **Outcome**

- The user is logged into the selected division.
- A VistALink connection is established or restored and the user is logged onto VistA.
- VBECS is available for use whether or not VistALink is connected, depending on the option.

#### **Limitations and Restrictions**

- VBECS automatically logs a user with access to only one division into that division.
- Connectivity by IAM services access approval (PIV Card) is not possible when IAM services are not available.
- Connectivity by IAM services access approval (PIV Card) is not required when a site has a PIV exemption or when IAM services are not available.
- An Access and Verify code is required to connect to VistA when the IAM services are not available.
- Connectivity is not possible when VistA is not available.
- The user may log into multiple terminals simultaneously, but accessing VBECS in multiple sessions is no longer possible due to the way the PIV card works.
- The Windows Security screen may be hidden when a user attempts to restore a VistALink connection. For the Windows Security screen to appear, the user must drag it from behind another window in order to select a certificate and log in.

#### **Additional Information**

- The initial entry into VBECS will require a VistA Access and Verify Code in order to initiate the IAM PIV sign on process. This is only required for the first use of the system for each user.
- Users will see several certificates listed during the sign on process. User should select the last certificate with their name from the list of certificates.

---

<sup>1</sup> There is a slight difference in terminology between VistA and VBECS: VistA uses "log on" and "logon," and VBECS uses "log in" and "login." Therefore, both terms are used throughout this guide. "Log in" and "login" are used generically when referring to both systems at one time.

- VBECS provides an interface (VistALink) with VistA to allow the user to log onto VistA (not required for most options; see the list of options that use VistALink in Hardware and Infrastructure Architecture).
- The system attempts to reuse the security credentials initially entered in the user's session to perform VistA logoff and logon without user interaction.
- The VistA security credentials may be saved for the user's session and may not be stored for future sessions.
- If the user is required to enter a VistA Access and Verify Code, VBECS checks the characters in the Access and Verify Codes and warns when there are fewer than six characters or when a semicolon (;), colon (:), or caret (^) is entered. VBECS sends the entered Access and Verify Codes to VistA. VistA checks the codes and responds to VBECS with a message stating whether the entered Access and Verify Codes are invalid or valid for the user attempting to log on. VBECS warns when VistA does not accept the Access and Verify Codes entered.
- After a number of unsuccessful attempts to log onto VistA (defined in VistA server settings), the VistA server terminates the logon session and displays a message prompting the user to log onto VistA again.
- After 15 minutes of inactivity, VBECS disconnects the user from the server. When the user logs back in, he will see the last screen he worked on. After 30 minutes of inactivity, work done after the last save will be lost. The user must log in and start over.
- Removing PIV card is similar to locking your computer in that it does not close active programs. The VBECS session will remain active but disconnected as long as you log back in within 15 minutes.
- The user may log into multiple terminals simultaneously, but accessing VBECS in multiple sessions is no longer possible due to the way the PIV card works<sup>2</sup>.
- A user with multidivisional access privileges may change divisions during a VBECS session.
- Infrequently, when a user cancels the VistA Logon - Authorization window after logging into VistA and selecting a division, the window closes and then automatically re-opens, requiring the user to cancel again. In this case, the user must cancel the VistA Logon - Authorization window four times before the window ceases to automatically re-open.
- VBECS will attempt to resend queued DSS information when VistALink is available after any user logs in.

## User Roles with Access to This Option

All users

### Log into VBECS

User Action	VBECS
1. To log into VBECS, double-click the <b>Remote Desktop Connection icon</b> on the Windows desktop.	<ul style="list-style-type: none"> <li>• Displays the Remote Desktop Connection screen.</li> </ul>
2. Enter or select the name of the VBECS server on the <b>Remote Desktop Connection</b> (Figure 27) screen and click <b>Connect</b> .	<ul style="list-style-type: none"> <li>• Displays Security Warning screen.</li> </ul>
3. Click <b>OK</b> on the Security Warning screen.	<ul style="list-style-type: none"> <li>• Displays Log On to Windows screen.</li> </ul>
4. If site is not PIV exempt, use PIV and PIN to enter the Remote	<ul style="list-style-type: none"> <li>• Displays VBECS server desktop with VBECS Prod, VBECS Test, VBECS Admin Prod and VBECS Admin Test icons.</li> </ul>

<sup>2</sup> See FAQ Proper Use of PIV Card with VBECS

User Action	VBECS
<p>Desktop Session (Figure 28 and Figure 29).</p>	<p><b>NOTES</b> _____</p> <p>Users with VBECS Administrator access can select to start VBECS administrator or VBECS.</p>
<p>5. Click on the <b>VBECS Prod</b> or <b>VBECS Test</b> icon to access VBECS.</p>	<ul style="list-style-type: none"> <li>• Displays the Select Division screen when the user has access privileges to multiple divisions in a single VBECS installation.</li> </ul> <p><b>NOTES</b> _____</p> <p>A user may exit without logging into a division.</p>
<p>6. Multidivision users will be prompted to select a division from the <b>Select Division</b> list and click <b>OK</b>, or double-click the division (Figure 30).</p>	<ul style="list-style-type: none"> <li>• Displays an error message to the user when logging into VBECS, or switching divisions, if the default printer for the division was removed.</li> <li>• Displays a division login message.</li> </ul> <p><b>NOTES</b> _____</p> <p>When VBECS is started, the user may have access to more than one division.</p> <p>The division login message allows the display of data as each user logs into his division. When no login message text is entered, VBECS does not display a login message.</p>
<p>7. If site is not PIV exempt, use PIV and PIN to connect to VistALink (Figure 28 and Figure 29).</p> <p>If the IAM PIV sign on process succeeds, continue at Step 9.</p> <p>If the IAM PIV sign on process fails, continue at next step.</p>	<ul style="list-style-type: none"> <li>• Directs user to the IAM service interface to select PIV certificate and possibly to re-enter PIN.</li> <li>• If the IAM PIV sign on process fails, VBECS continues to the next step and prompts user for VistA Access and Verify Code. The user may log onto VistA or continue and log on as needed.</li> </ul> <p><b>NOTES</b> _____</p> <p>The system prompts the user for their access and verify code when there is no active IAM service certification.</p> <p>When a user logs into VBECS, the connection to VistA is established and the data in the cache tables are out of date. VBECS retrieves the current data from VistA and saves them. This update process may delay user login.</p>

User Action	VBECS
<p>8. Enter the VistA Access and Verify Codes (Figure 31).</p>	<ul style="list-style-type: none"> <li>• Allows a user to log on by entering VistA Access and Verify Codes, separated by a semicolon (;), in the Access Code data entry field.</li> <li>• Verifies that the user credentials for the VBECS and VistA Access and Verify Codes belong to the same user.</li> <li>• Emits a tone to verify that the workstation's sound works.</li> </ul> <p><b>NOTES</b> _____</p> <p>The VBECS user must have his own VistA Access and Verify Codes when accessing VistA. The system will not allow the user to log onto VistA if his DUZ in the VBECS database does not match the user's DUZ in VistA.</p> <p>The user will log onto VistA by entering VistA Access and Verify Codes if the system is reconfigured during this session (for example: VistALink dropout and recovery, division change) and subsequently requires a logoff or logon to VistA.</p> <p>The system will prompt user to enter current Verify Code, the new Verify Code, and a confirmation for a new Verify Code.</p> <p>When a Verify Code is changed, the system must check that the new code is different from the currently used Verify Code.</p>
<p>9. Continue working in VBECS in the selected division.</p>	<ul style="list-style-type: none"> <li>• Displays the main menu.</li> </ul>
<p>10. Users with multidivisional access: to change division, select <b>File</b>.</p> <p>Select <b>Change Division</b>.</p> <p>Continue at Step 6.</p>	<ul style="list-style-type: none"> <li>• Ensures that there are no incomplete activities for the user's session.</li> <li>• Displays a division login message.</li> <li>• Opens the selected division.</li> </ul> <p><b>NOTES</b> _____</p> <p>A user may log into or change divisions without being logged onto VistA.</p> <p>VBECS displays a message advising the user to close dialogs and transactions with pending data changes, and closes open dialogs with no pending data changes before the user changes division.</p> <p>VBECS automatically logs onto VistA for the selected division.</p>



Figure 27: Remote Desktop Connection Options

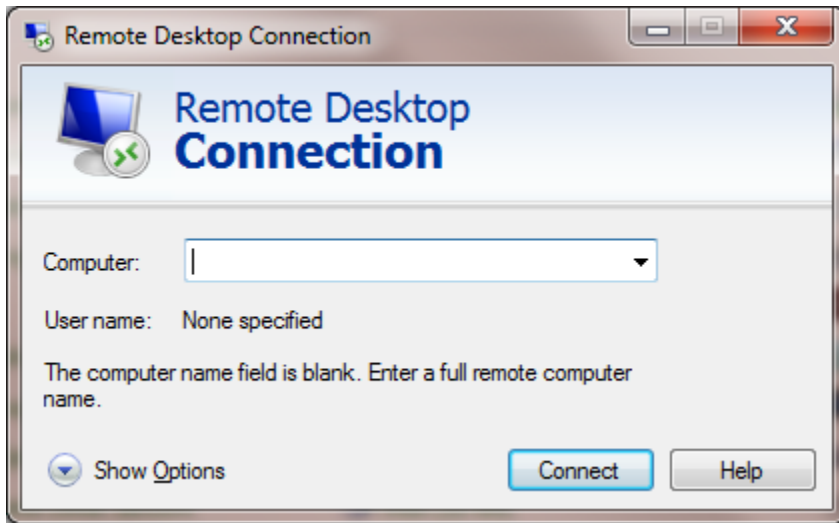


Figure 28: IAM – PIV Certificate

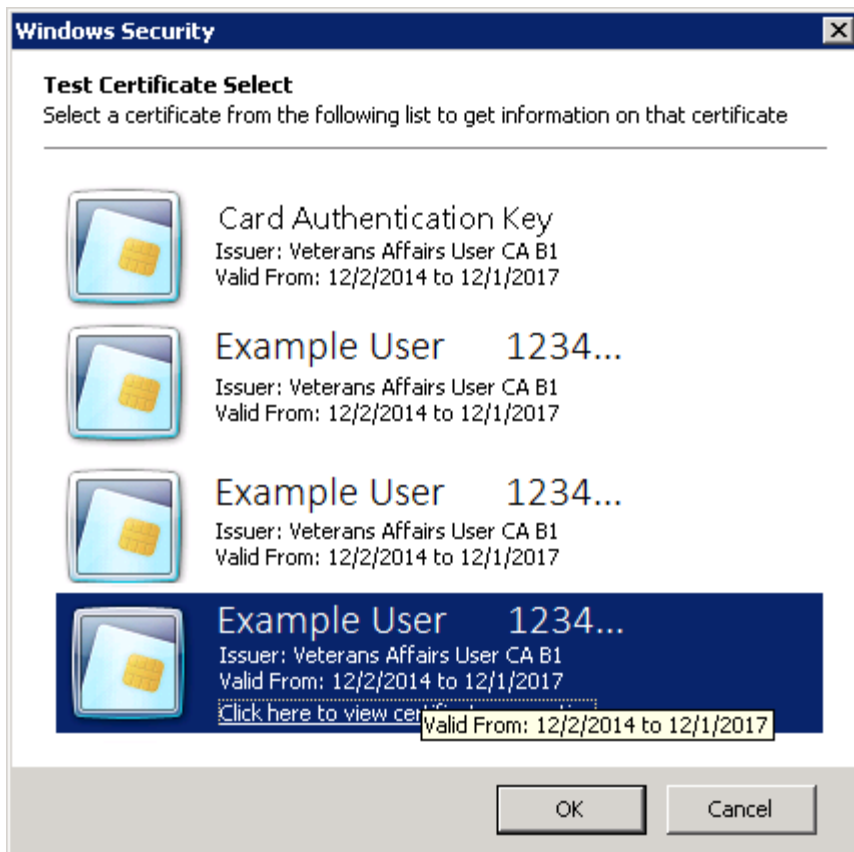


Figure 29: ActivClient Login

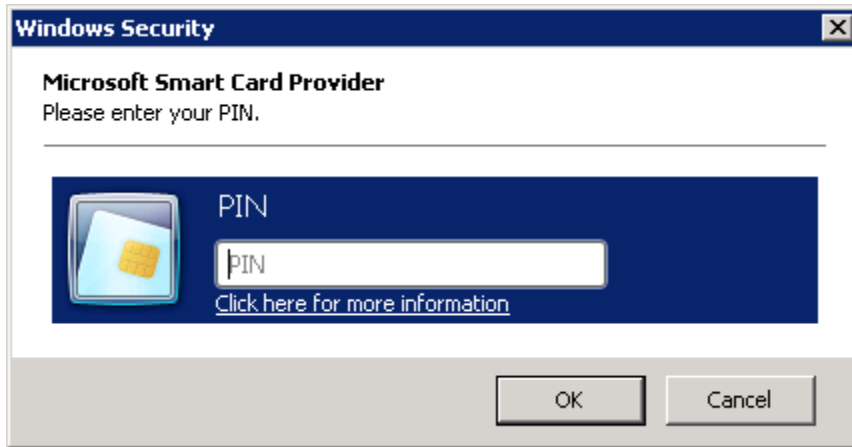


Figure 30: Division Logon


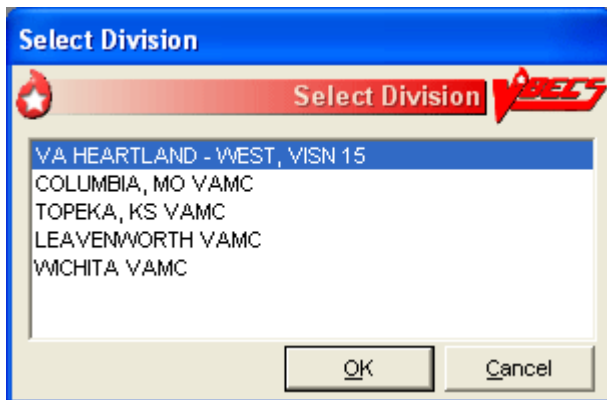


Figure 31: VistA Access and Verify Code Logon

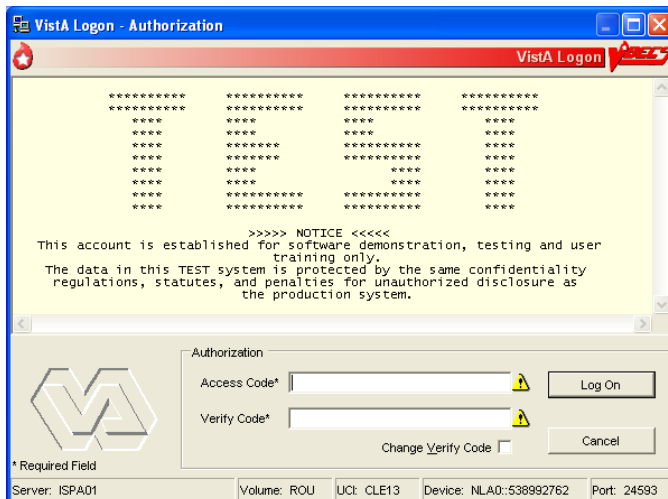


## Restore VistALink Connection

Before any option requiring VistALink is started, the system validates VistALink availability. When the server does not respond or there is no active connection, the system brings up the dialog informing the user of the VistALink connection's unavailability and offering the option to restore the connection.

## Restore VistALink Connection

User Action	VB ECS
<p>1. The <b>Restore</b> option is presented. Choose to restore or cancel (Figure 32).</p>	<ul style="list-style-type: none"> <li>Attempts to connect using the user's previously entered PIV/PIN information when logon information is entered.</li> <li>When VB ECS/Administrator is up and running and user attempts to access a VistALink dependent option when VistALink is not connected, the system will prompt a reconnection dialog where user will have two options: to reconnect or to cancel. When the reconnect process fails or when the user cancels, the system shows an error dialog.</li> <li>Closes the pop-up when the user clicks <b>Cancel</b>.</li> </ul>
<p>2. If site is not PIV exempt, use PIV and PIN to connect to VistALink (Figure 28 and Figure 29).</p> <p>If the IAM PIV sign on process succeeds, continue at Step 4.</p> <p>If the IAM PIV sign on process fails, continue to next step.</p>	<ul style="list-style-type: none"> <li>Directs user to the IAM service interface to select PIV certificate and possibly to re-enter PIN</li> <li>If the IAM PIV sign on process fails, VB ECS continues to the next step and prompts user for VistA Access and Verify Code. The user may log onto VistA or continue and log on as needed.</li> </ul> <p><b>NOTES</b></p> <p>The system prompts the user for their access and verify code when there is no active IAM service certification.</p> <p>When a user logs into VB ECS, the connection to VistA is established and the data in the cache tables are out-of-date. VB ECS retrieves the current data from VistA and saves them. This update process may delay user login.</p>
<p>3. Enter the VistA Access and Verify Codes (Figure 31).</p>	<ul style="list-style-type: none"> <li>The system will reuse the security credentials initially entered in this user's session (VistA Access and Verify Codes) to perform VistA logoff and logon without user interaction.</li> <li>Allows a user to log on by entering VistA Access and Verify Codes, separated by a semicolon (;), in the Access Code data entry field.</li> <li>Verifies that the user credentials for the VB ECS and VistA Access and Verify Codes belong to the same user.</li> <li>Emits a tone to verify that the workstation's sound works.</li> </ul>
<p>4. Continue working in the VistALink dependent option.</p>	<ul style="list-style-type: none"> <li>Displays the VistALink dependent option.</li> </ul>

**Figure 32: Restore VistA Connection**

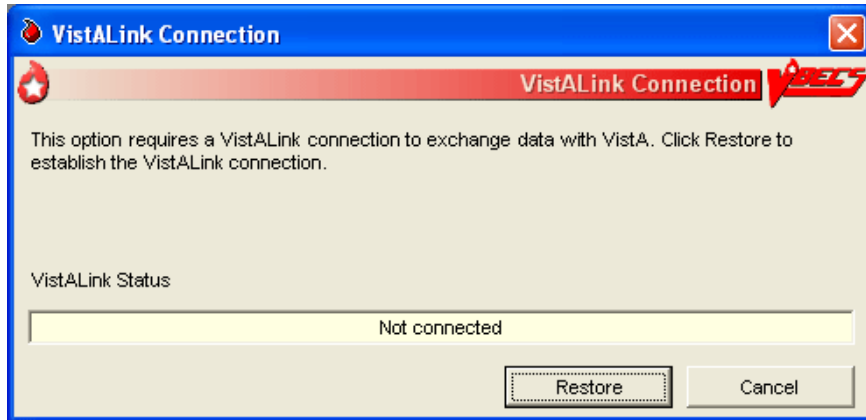


## Exit VBECS

A user may log off the VBECS session.


## Log Off VBECS

User Action	VBECS
1. To log off VBECS, select <b>File, Exit VBECS</b> .  Click <b>Yes</b> at the prompt.	<ul style="list-style-type: none"><li>• Exposes the server desktop.</li></ul>
2. Select <b>Start, Log Off</b> .  At the prompt, select <b>Log Off</b> or <b>Cancel</b> .	<ul style="list-style-type: none"><li>• Logs off the session.</li><li>• Terminates open applications.</li></ul>

## Screen Settings

 *VBECS was designed to operate with the settings described in this section to display complete screens. Failure to maintain these settings may result in partial display of safety-critical information.*

### Local Machine Screen Resolution


 *Set the local machine's desktop resolution to 1024 by 768 pixels or higher for optimal display.*

To adjust the resolution of the screens that you view:

- 1) Click Start, Control Panel.
- 2) Click Display.

- 3) Click Adjust resolution (Figure 33).
- 4) Click, hold, and slide the pointer to a screen resolution of at least 1024 by 768 pixels.
- 5) Click Apply.

**Figure 33: Local Machine Screen Resolution**

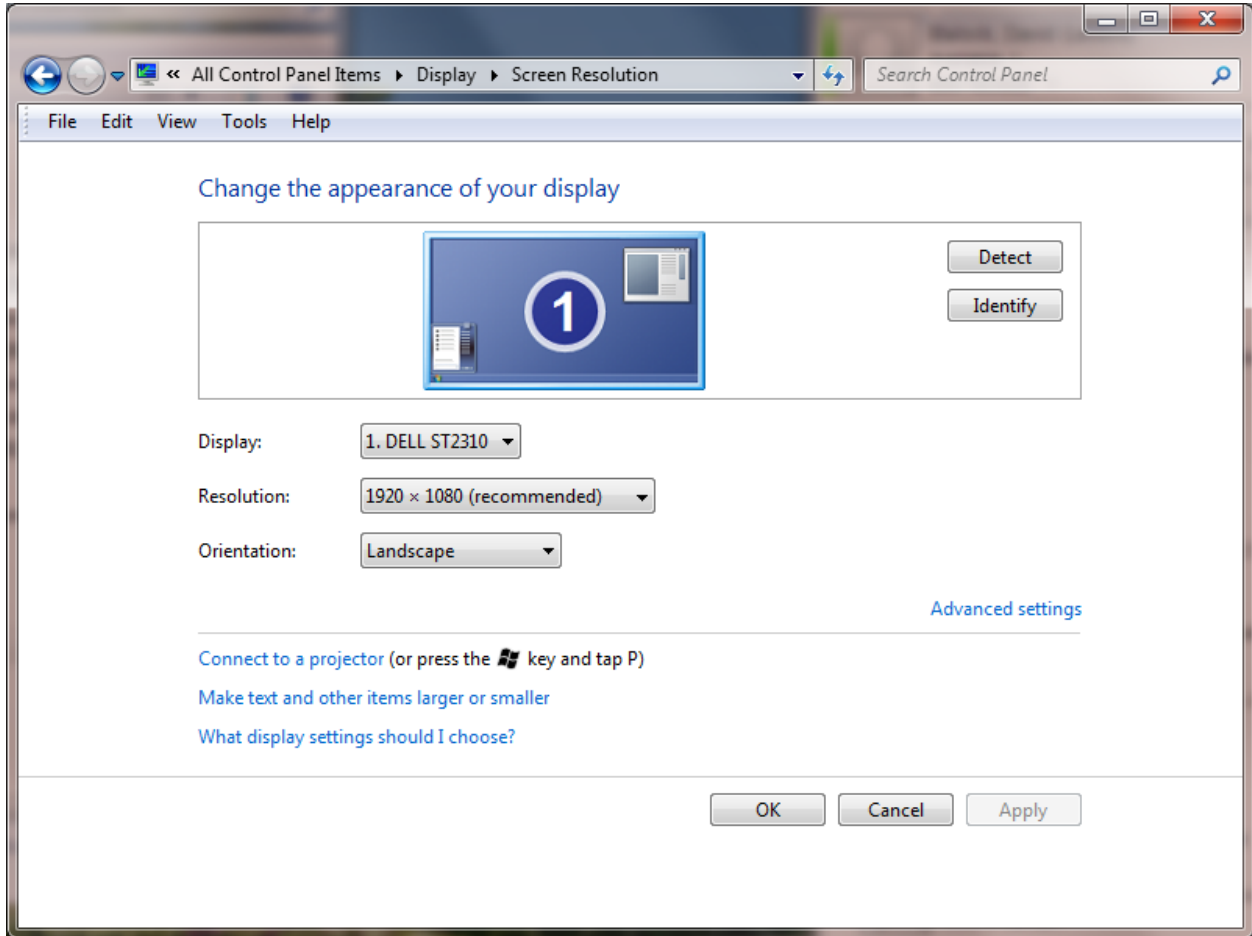

## Remote Desktop Connection Screen Size




*The Remote Desktop Connection desktop size must be set to “Full Screen.”*

To adjust the dimension of the screens that you view:


- 1) Double-click the VBECS (**Remote Desktop Connection icon**) located on the local workstation desktop.
- 2) Click Show Options (Figure 34).

**Figure 34: Remote Desktop Connection Options**

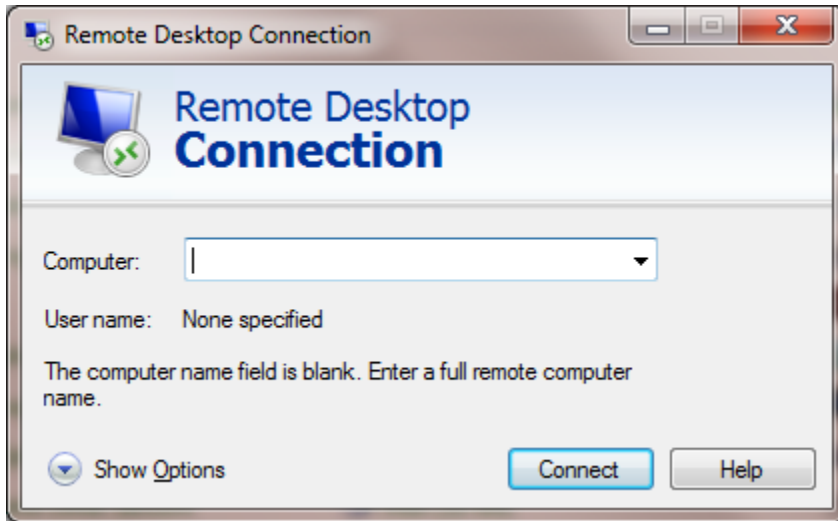

3) Click the Display tab (Figure 35).

**Figure 35: Remote Desktop Connection Remote Desktop Size**

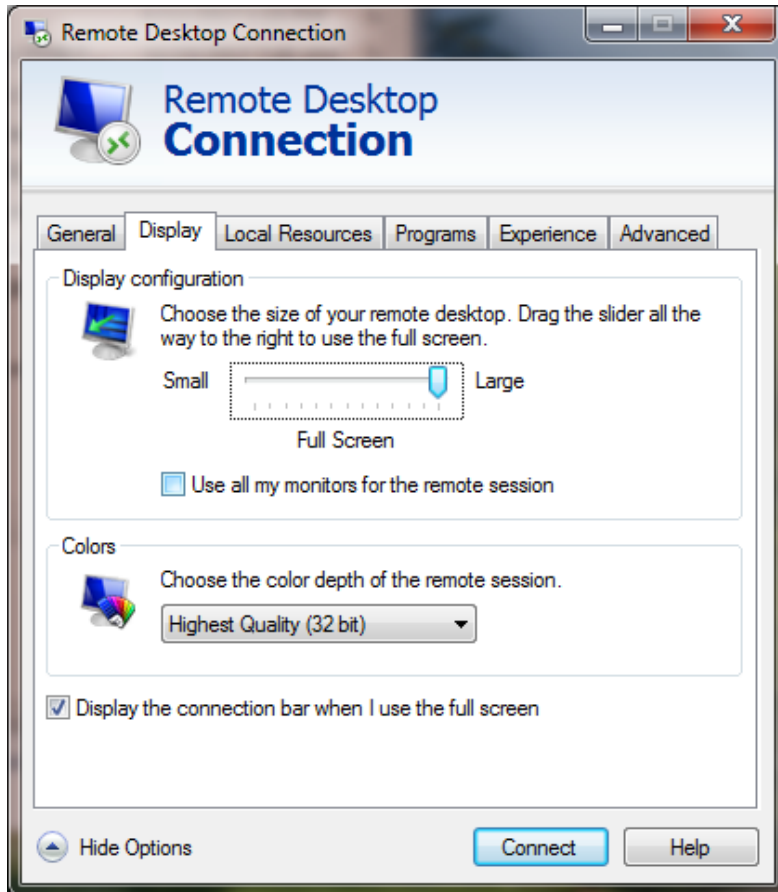

- 4) Click, hold, and slide the pointer to the right until “Full Screen” appears.
- 5) Click on the General tab.
- 6) Click SAVE to save the setting.



## Screen Display

A *window* is the section of the screen that displays the active application or file.

Click the control buttons in the upper-right corner (Figure 36) of an open file or application window to:

- Minimize: reduce the window to appear as a button (rectangle) at the bottom of the screen. Click the button to enlarge the window to its original size.
- Maximize: enlarge a window to fill the entire screen
- Restore: return a window to a smaller size that can be adjusted (Figure 38)
- Close: shut down a document or application quickly



*Maximize option windows to display safety-critical information in full.*

**Figure 36: Resize or Close a Window**

**Figure 37: Restore a Window**



*When a window is partially displayed or freezes; or when you move from one window to another, enter information, return to the original window, and the entered information is not available, minimize then maximize the window to refresh it.*

To resize an open window:

- Make sure the Maximize symbol is visible.
- Change the width: move the mouse cursor to the left or right window border until the pointer changes into a horizontal double-headed arrow, then drag the border to the right or left.
- Change the height: move the mouse cursor to the top or bottom window border until the pointer changes into a vertical double-headed arrow, then drag the border up or down.
- Change the height and width together: move the mouse cursor to any window corner until the pointer changes into a diagonal double-headed arrow, and then drag the border in any direction.

*A maximized window cannot be resized or moved.*

To move a window, click the dark blue bar at the top of the window. While depressing the mouse button, drag the window to the desired position.


Select **Help**, **VBECS Help**, or press F1 at any time for screen-specific online help.

The first time a user opens a form [such as the Blood Transfusion Record Form (BTRF)], VBECS displays it in cascade style. When a user opens the form again, VBECS displays it in the size and position last selected.


*Forms can be resized and moved in the same way that windows are resized and moved.*

## Sound

To enable sound on Remote Desktop Connection:


- 1) Double-click  (the **Remote Desktop Connection** icon).
- 2) Click **Show Options** (Figure 38).

**Figure 38: Remote Desktop Connection Options**

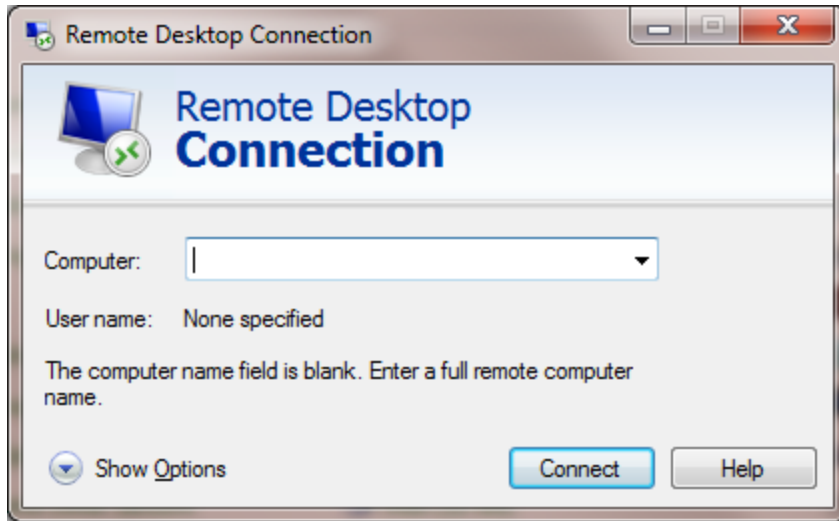


3) Click the **Local Resources** tab (Figure 39).

**Figure 39: Remote Desktop Connection Remote Computer Sound**

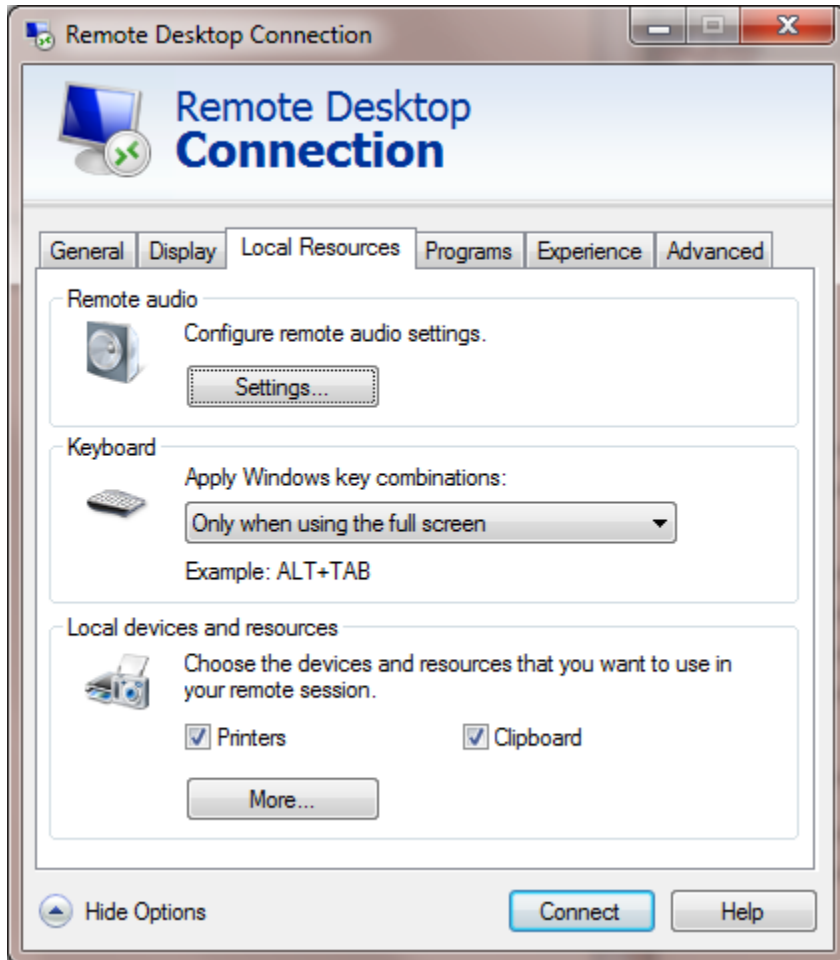


4) Click **Settings** (Figure 40).

**Figure 40: Remote Desktop Connection Settings**

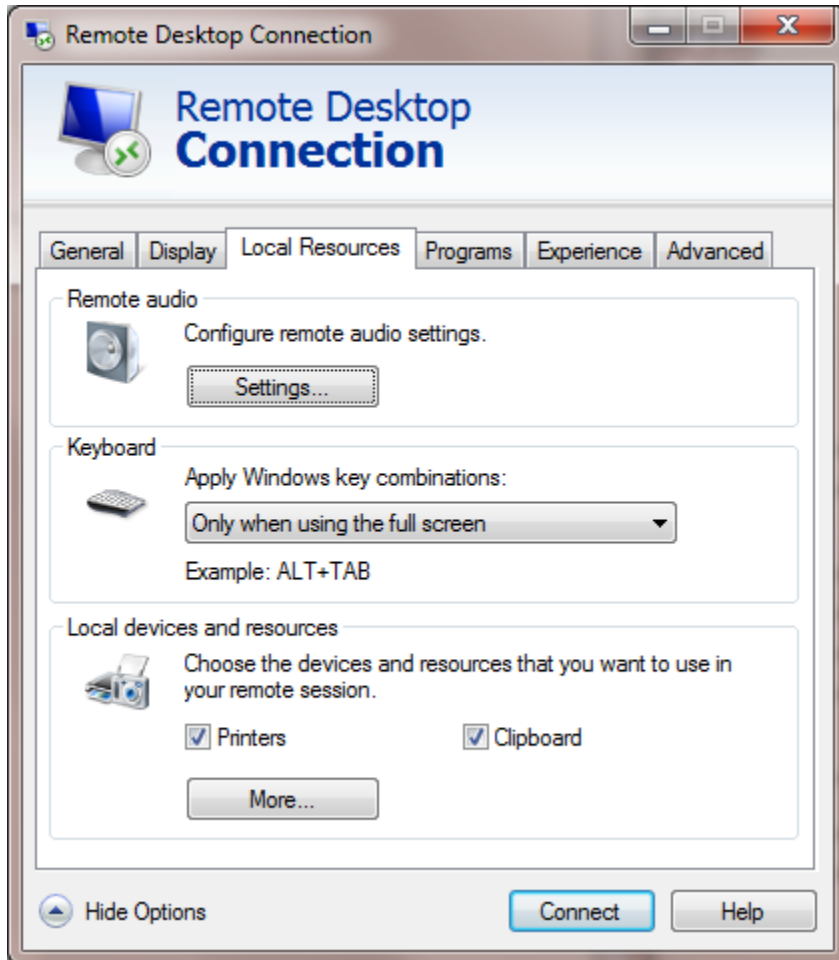

5) Select **Play on this computer** from the Remote computer sound drop-down list (Figure 41).


**Figure 41: Remote Desktop Connection Play on this computer**

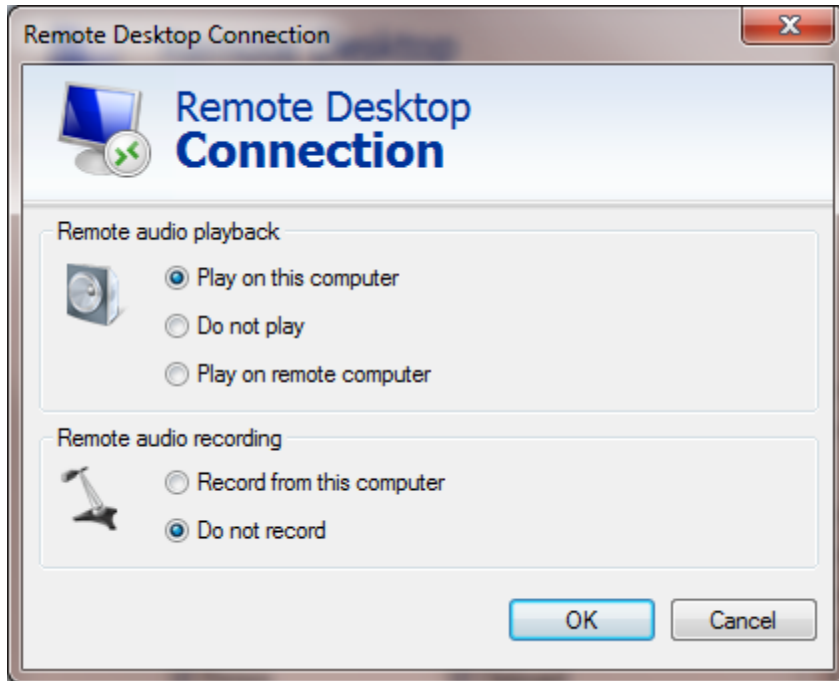

6) Click **OK**.


7) Click the **Experience** tab (Figure 42).

**Figure 42: Remote Desktop Connection Choose Connection Speed**

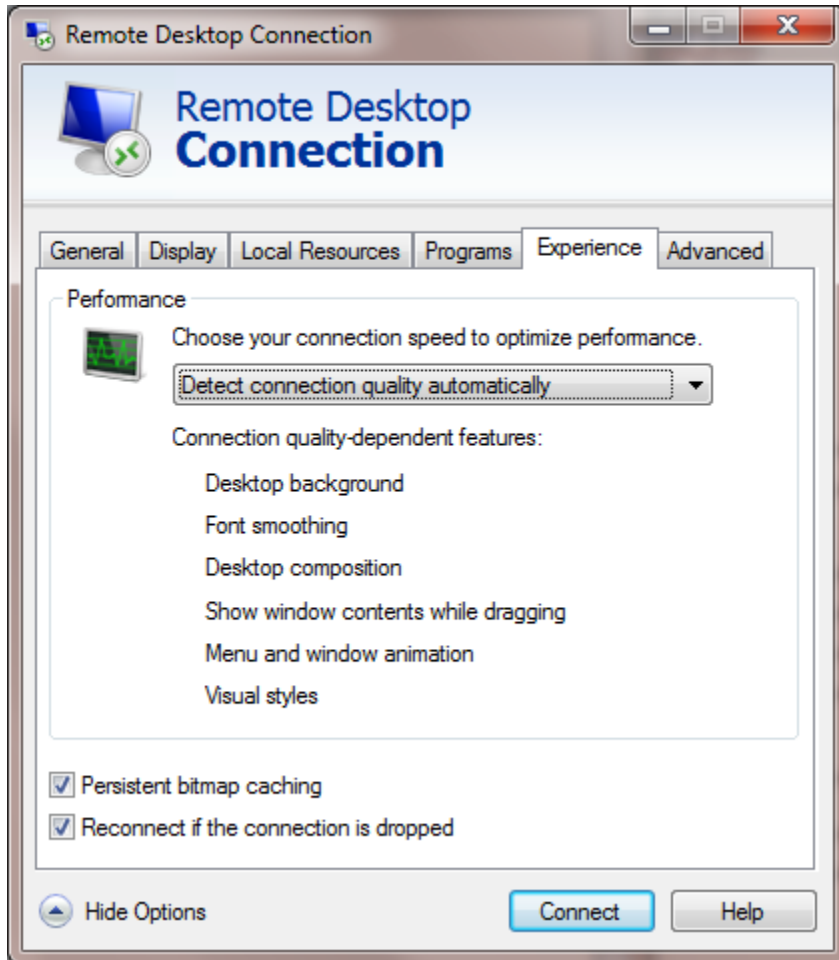

- 8) Select **LAN** from the **Choose your connection speed to optimize performance** drop-down list (Figure 43).

**Figure 43: Remote Desktop Connection Choose Connection Speed**

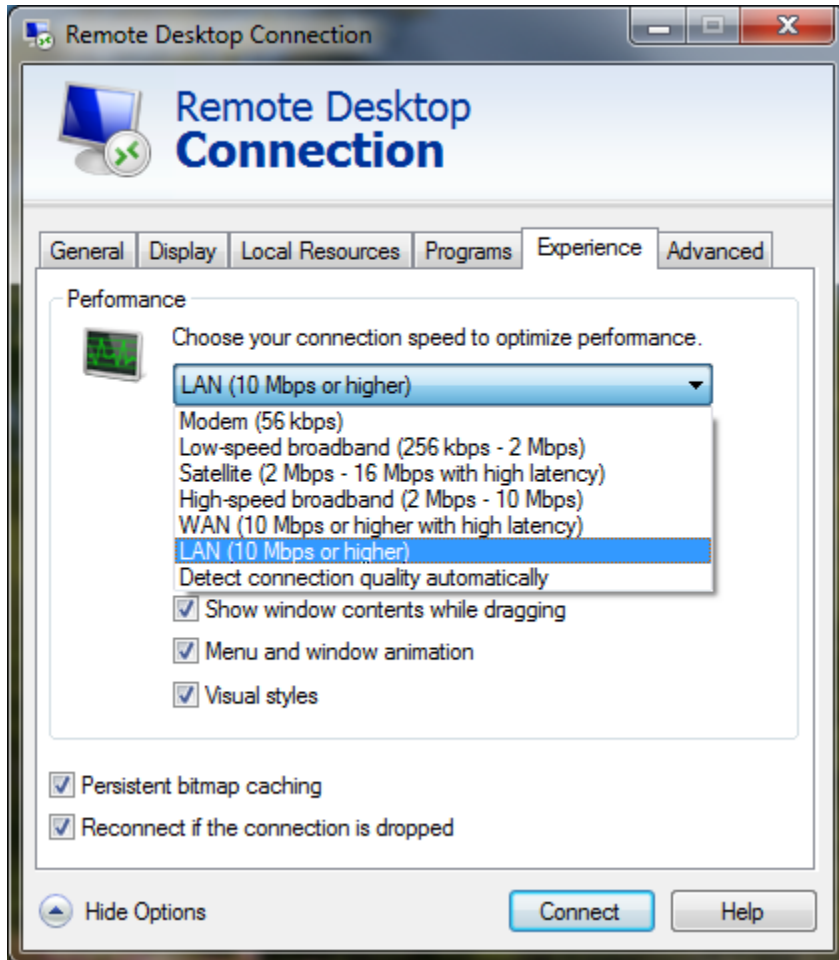

- 9) Click the **General** tab and click **Save**.  
10) Click **Connect** (Figure 43).

## Volume




*Do not change volume settings unless alerts are inaudible (muted) or low.*

VBECS uses sound as one way to alert users. To restore the volume setting on your workstation so that alerts are audible:

- 1) Click **Start, Control Panel**.
- 2) Click **Sound**.
- 3) Click **Properties**.
- 4) Select the **Levels** tab (Figure 44).
- 5) Click, hold, and slide the pointer to adjust the volume.
- 6) Click **OK** to save the setting.

**Figure 44: Sounds and Audio Devices: Volume**

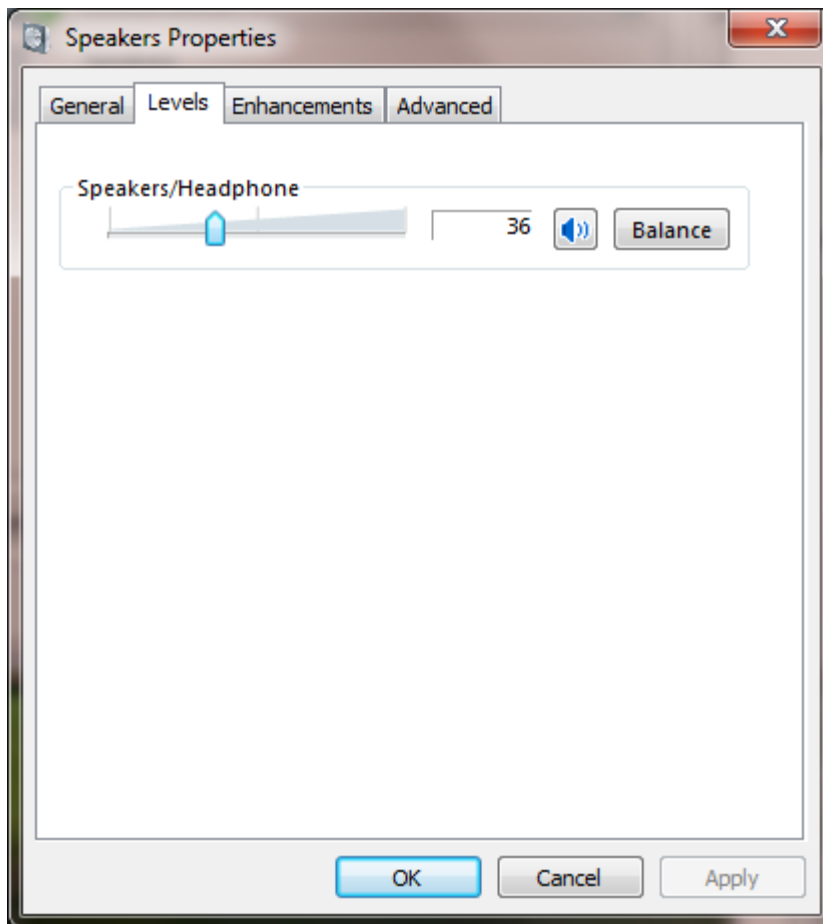



## Keyboard Shortcuts for Windows


Using a mouse to navigate VBECS requires fewer steps than using the keyboard. A user who prefers using the keyboard may refer to these keyboard shortcuts. Keys preceded and/or followed by “+” indicate that they must be pressed simultaneously:

*Keyboard navigation of VBECS does not accommodate the presentation of error providers or tool tips. A mouse is required to reveal this information as needed.*

### CPRS and VBECS

- Toggle between **VBECS** and **CPRS** or another application: **Alt + Tab**

### Menus

- Display the **Start** menu (use the arrow keys to select an item): **Alt + Home**
- Start VBECS online help: **F1**
- Activate a menu bar option: **F10**
- Open a menu: **Alt** + underscored letter
- Move to and select a menu item: Arrow key + **Enter**
- Open a submenu item: **Shift** + letter (case insensitive)
- Open a drop-down menu: **Alt** + down arrow key
- Close a drop-down menu: **Alt** + up arrow key
- Display a shortcut menu for the selected item: Application key ()

### Buttons, Check Boxes, and Fields

- Move to and select fields and buttons: **Tab** + **Enter**
- Select or clear a check box: **Tab**, arrow key (or **Alt** + underscored letter), spacebar
- Select or clear a radio button: **Tab**, arrow key (or **Alt** + underscored letter)

### Windows

- Display the **Control** window to restore, move, resize, minimize, maximize, or close the window: **Alt** + **Delete**
- Switch between a window and a full screen: **Ctrl** + **Alt** + **Break**
- Close the current window: **Alt** + **F4**
- Display the **Windows Security** dialog box to lock the computer, log off, shut down, change a password, and access **Task Manager**: **Ctrl** + **Alt** + **End**

### Screen Prints

- Place a snapshot of the desktop on the local clipboard and paste it in another application: **Ctrl** + **Alt** + **Plus** (+ on the numeric keypad; **Ctrl** + **V** to paste)
- Place a snapshot of the active window on the local clipboard and paste it in another application: **Ctrl** + **Alt** + **Minus** (– on the numeric keypad; **Ctrl** + **V** to paste)

## Drop-Down Menus

A *drop-down menu* may appear when a button is clicked or text is selected. Click or hold the mouse pointer over an item in a drop-down menu to reveal other items and submenus. Press an underscored letter (mnemonic) to select a menu item. To view underscored letters, press the **Alt** key once.

*A ► (submenu arrow) next to a menu item indicates that a submenu is available. An option that is not available to a user is visible but grayed out.*

## Tool Tips

A *tool tip* is a description or explanation that appears when the mouse pointer is held over (not selected or highlighted) a tool, button, or other object (without clicking a mouse button).

*When a tool tip does not appear, wait a few moments, then reposition the mouse pointer over the tool, button, or other object. If necessary, exit and reenter the window.*

Tool tips appear with:

- Some control buttons
- Icons in the Patient Information Toolbar

*Text enclosed in angle brackets (< >) in tool tips and messages will change to reflect the current activity.*

*The user may click column headings in some data displays, such as the Pending Task List, Pending Order List, and Select Units, to sort the information in those columns.*

## Icons and Buttons

VBECS uses several types of images to help you move through the system:

- *Icons* represent objects (information used in an application). The rectangular area in the lower right corner of the screen (system tray) contains icons of programs that always run in the background, such as the clock.
- *Buttons* are icons that represent tools or commands.
- *Radio buttons* are small circles preceding text. Related radio buttons are grouped together. Only one button in a group may be selected: clicking a button turns the other radio buttons off.

## Toolbars

A *tool* is a shortcut button, usually shown on a bar near the top of a window. Tools provide quick access to commonly used functions. A *toolbar* displays groups of tools. Toolbars may contain buttons, menus, or combinations of both. (See also Patient Information Toolbar.)















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
*A toolbar with tools for several options (Main Toolbar) appears near the top of the screen.*

Click a tool to open a function quickly.

*A function may also be opened by pressing **Alt** and the **key for the underscored letter** in the tool name.*

## **Main Toolbar**

The *Main Toolbar* appears at the top of the screen. Click the icons in the Main Toolbar to access<sup>3</sup>:



-  Log In Reagents
-  Enter Daily QC Results
-  Incoming Shipment
-  ABO/Rh Confirmation
-  Modify Units
-  Unit Antigen Typing
-  Outgoing Shipment
-  Accept Orders: Accept an Order
-  Maintain Specimen
-  Patient Testing
-  Automated Instrument Pending Blood Unit Tests
-  Automated Instrument Pending Patient Tests
-  Select Units
-  Issue Blood Components
-  Post-Transfusion Information

*Clicking an icon in the Main Toolbar saves a step: it's the same as clicking the main menu item, and then on the option.*

VBECs uses tree views to displays list items. Click a plus sign (+) to expand a list. Click a minus sign (-) to condense a list.


## **Patient Information Toolbar**


The *Patient Information Toolbar* appears in the active window when a patient record is displayed. VBECs displays only the icons appropriate to the option. Click the icons in the Patient Information Toolbar to display:

-  a Blood Availability Report (Blood Availability)
-  a patient's current medication profile (Medication Profile)


---

<sup>3</sup> Larger versions of these icons appear throughout VBECs in the corresponding option windows. They cannot be used to access options.

 a patient's recent orders (Recent Orders)

 a patient's Special Instructions and/or Transfusion Requirements (View Special Instructions and Transfusion Requirements)

 a patient's recent transfusion history (Recent Transfusions/Issued Units)

 a patient's transfusion reaction history (Transfusion Reaction History)

### **Patient Blood Availability**

The user views blood products available to a patient. The Blood Availability Report does not include unassigned, restricted blood products. To view unassigned, restricted blood products and other custom reports, see Blood Availability.

## Medication Profile

The user views a patient's current medication profile.

### Assumptions

- The connection to VistA is active.

### Outcome

- VBECS displays a patient's current medications.

### Limitations and Restrictions

- None

### Additional Information


- The user may access CPRS for additional information on a patient medication order.
- These data are accessible to the user through the Patient Information Toolbar on all patient-specific screens.

### User Roles with Access to This Option

All users

### Medication Profile

At the user's request, VBECS queries VistA to search for active prescriptions within a user-specified time frame or a default time frame of six months, regardless of division, in a multidivisional database.

User Action	VBECS
1. Select <b>Reports</b> from the main menu.  Select <b>Medication Profile</b> .  Select a patient, or  Select an option that displays the Patient Information Toolbar.  Click  to view a patient's current medication list.	<ul style="list-style-type: none"> <li>• Lists report names.</li> <li>• Displays the report date range, sections, and compilation criteria.</li> <li>• Displays options for processing patient-related functions.</li> <li>• Compiles the active patient identification data.</li> </ul> <p><b>NOTES</b> _____</p> <p>The user may edit the default start date (six months before the current date) for the patient medication search.</p>
2. Accept or edit the default start and end dates.	<ul style="list-style-type: none"> <li>• Displays each medication name status, order number, and issue/start date returned from the VistA query.</li> <li>• When a query does not return any medications, notifies the user that he may repeat the query with a different date range.</li> </ul>
3. View the display. When desired, go to Step 1 to repeat the query,	<ul style="list-style-type: none"> <li>• Allows the user to print the displayed patient medication profile.</li> </ul>
4. Print the profile or click <b>Cancel</b> to exit.	<ul style="list-style-type: none"> <li>• Displays and prints the medication information profile.</li> </ul>


## View Special Instructions and Transfusion Requirements

The user views patient-specific Special Instructions (SIs) and Transfusion Requirements (TRs).


### Assumptions

- SI entries were created through Special Instructions & Transfusion Requirements: Enter a Special Instruction or from database conversion.
- TR entries were entered through Special Instructions & Transfusion Requirements: Enter a Transfusion Requirement or Patient Testing: Enter Antibody Identification Results.

### Outcome

- VBECS displays a patient's SIs and TRs.
- This icon  will be disabled if the icon is displayed within the patient search window, because special instructions and transfusion requirements are already displayed automatically. This icon will also be disabled if the icon is displayed within the patient's SIs and TRs dialog. This icon will always be disabled if the selected patient does not have SI's or TR's to display.

### Limitations and Restrictions

- Changing a patient's antigen negative requirement or antibody identified after a unit is crossmatched does not provide an alert at Issue Unit regarding the type of crossmatch (electronic or serologic) performed on a unit available for issue.
- This icon  will be disabled when the icon is displayed:
  - within the patient search window because special instructions and transfusion requirements are already displayed automatically within this option.
  - within this option, the selected patient does not have SIs or TRs to display.

### Additional Information

- VBECS displays a patient's known antibodies in all divisions in a multidivisional database. VBECS does not restrict viewing SI and TR entries by division; they are linked to the patient's record and not to an order, a specimen, or a unit.
- VBECS does not enforce SIs; VBECS enforces TRs and persistent antigen negative requirements throughout the database.

### User Roles with Access to This Option



All users

### View Special Instructions and Transfusion Requirements

The user views a patient record. When SIs and TRs are available for the patient, VBECS activates the Patient Information Toolbar icons. VBECS alerts the user when a patient's record includes SIs and TRs.



*It is highly recommended that the user click every active icon when reviewing patient data.*

User Action	VBECS
<p>1. Select <b>Patients</b> from the main menu.</p> <p>Select <b>Special Instructions &amp; Transfusion Requirements</b>, or</p> <p>Click  in the Patient Information Toolbar, when it is active.</p>	<ul style="list-style-type: none"> <li>• Displays options for processing patient-related functions.</li> <li>• Displays the option to select a patient, when one is not already selected.</li> </ul> <p><b>NOTES</b> _____</p> <p> When a patient's VBECS record includes an active TR, SI, or antigen negative requirement, VBECS emits an audible alert and notifies the user by enabling the icon.</p>
<p>2. Select a patient and click <b>OK</b>.</p> <p>View the data.</p> <p>Click <b>OK</b> to continue.</p>	<ul style="list-style-type: none"> <li>• Displays the data.</li> </ul> <p><b>NOTES</b> _____</p> <p>VBECS displays active TRs to all divisions within a multidivisional database.</p> <p>VBECS alerts the user when an active TR, SI, or antigen negative requirement is in a patient's record.</p> <p>VBECS displays component requirements to all divisions in a multidivisional database. No division allows issue until requirements are met.</p> <p>When VBECS displays the Patient Information Toolbar, it calculates the overall compatibility percentage of units to be screened to find compatible units.</p> <p>The compatibility percentage displayed is based on the following calculation:</p> <p>Type Specific ABO % x Antigen compatibility % as defined in Antibodies equals % compatible.</p> <p>For example, if a patient is A Positive and has both Anti-E (compatibility percentage = 75%) and Anti-K (compatibility percentage = 91%) antigen negative requirements, the percentage of the population considered compatible for this patient would be <math>.45 \times .75 \times .91 = .30</math> (rounded).</p> <p>Rh type of the patient is not used in the calculation when the patient does not have an Anti-D as Rh Pos or Rh Neg units may be screened.</p> <p>The blood type percentages in VBECS are as follows and are general percentages for the American population: A is 45%, B is 15%, AB is 6% and O is 45%.</p>
<p>3. Select another patient and return to Step 2, if desired.</p>	
<p>4. Click <b>OK</b> to exit.</p>	

## View Recent Orders, Recent Transfusions/Issued Units, and Transfusion Reaction History

The user views portions of a patient’s data previously recorded in VBECS in three formats: recent orders, recent transfusions/issued units, and transfusion reaction history.

### Assumptions

- A VBECS record must be established for a patient in this database in at least one division, if multidivisional.

### Outcome

- The user views:
  - A patient’s active order data (recent orders).
  - A patient’s transfusions and currently issued units (recent transfusion history).
  - A snapshot of the patient’s VBECS clinical record (transfusion reaction history).

### Limitations and Restrictions

- Recent Orders is showing Order Status values like “Not Started” and “Filled”, which are actually Task Statuses.

### Additional Information



- None

### User Roles with Access to This Option

All users

### Patient Information Toolbar: Recent Orders



The user views current (pending and active) order data (including reflex tests), which include clinical data from all divisions within a multidivisional database.

User Action	VBECS
1. In the Patient Information Toolbar, click  to view the patient’s recent orders.	<ul style="list-style-type: none"> <li>• Allows the user to click an icon to view a patient’s recent orders.</li> </ul> <p><b>NOTES</b> _____</p> <p>When the user selects , VBECS displays all orders associated with the selected patient, including open orders and orders placed and processed within the last 14 days.</p> <p>Diagnostic Test orders sort and display by the date the order was placed in CPRS. Component orders sort and display by the date the order was accepted in VBECS.</p> <p>Orders with a status of Canceled, Completed, Expired or Filled are not displayed in this view. See Order History Report.</p>
2. View the data and click <b>Close</b> to exit.	





## Patient Information Toolbar: Recent Transfusions/Issued Units

The user views transfused and currently issued units for a patient.

User Action	VBECs
1. In the Patient Information Toolbar, click  to view the recent transfusion history.	<ul style="list-style-type: none"><li>Allows the user to click an icon to view a patient's recent transfusion history.</li></ul> <p><b>NOTES</b> _____</p> <p>When the user selects , VBECs displays blood units issued and transfused within the last seven days in reverse chronological order. The date and time issued or the location issued to does not display.</p>
2. View the data and exit.	





## Patient Information Toolbar: Transfusion Reaction History

The user views a snapshot of the patient's transfusion reaction record.









User Action	VBECs
1. In the Patient Information Toolbar, click  to view the transfusion reaction history.	<ul style="list-style-type: none"><li>Allows the user to click an icon to view a patient's transfusion reaction history.</li></ul> <p><b>NOTES</b> _____</p> <p>When the user selects , VBECs displays all finalized transfusion reaction entries in the database.</p>
2. View the data and exit.	

## Main Status Bar

The following icons are displayed in the main status bar:

-  The Order Alert icon blinks three times and remains visible when orders are added to the system. Hold the mouse pointer over the icon to see order information. Click the associated button to open the Pending Order List.
-  The Patient Alert icon flashes three times and remains displayed when patient information is updated. Hold the mouse pointer over the icon to see what type of patient change occurred. Click the Patient Alert Icon to view the menu which allows navigation to Updated Patients, Deceased Patients, Patient Merge. These menu options enable when alerts exist in each category.
-  The VistALink icon indicates that the connection to VistA is active.
-  The VistALink Failure icon indicates that the connection to VistA is severed.

## Throughout VBECS

-  The Delete/Invalidate icon appears on buttons that cause entries to be deleted or invalidated in the database.
- Click to select all items in a list.
- Click to clear all items in list.
-  Click the **find** button to display a list of items for selection.
-  Click the **Print** icon's associated button to print a document.
-  Click the **Save** icon's associated button to update the database.
-  The Scanner icon indicates which screen prompts permit scanned entries.
-  Click the **Search** icon's associated button to search for entries meeting search criteria entered.
-  Click the **Order Details** icon's associated button to display the full details of a selected order.
-  The Input Error icon indicates that a user made an invalid entry at a screen prompt. The icon appears and flashes next to the invalid entry. Hold the mouse pointer over the icon to see a message explaining the problem. Enter valid information to make the icon disappear.

## ***Other VBECS Functions***

**Collecting Workload** In some options, VBECS collects workload (statistical data used to calculate productivity) when the user saves data:

- ABO/Rh Confirmation
- Automated Testing
- Discard or Quarantine
- Document ABO Incompatible Transfusion
- Edit Unit Information
- Enter Daily QC Results
- Finalize/Print TRW
- Free Directed Unit For Crossover
- Invalidate Test Results
- Issue Blood Components
- Modify Units
- Modify Units: Pool Units
- Modify Units: Split a Unit
- Outgoing Shipment
- Patient Testing: Enter Antibody Identification Results
- Patient Testing: Record a Patient Antigen Typing
- Patient Testing: Record a Transfusion Reaction Workup
- Patient Testing: Record Patient Test Results
- Post-Transfusion Information
- Processing Incoming Shipment
- Release Units From Patient Assignment
- Remove Final Status
- Return Issued Units To Blood Bank
- Select Units
- Unit Antigen Typing

This page intentionally left blank.

# Configuring Site Parameters

Configuration must be done as part of the VBECS installation.

## Update User Roles

The user reviews and changes the user roles and associated security levels that determine which options and functions are available to a user in the division.

### Assumptions

- The user is logged into VBECS in the division of the users to be edited.
- A VBECS Application Administrator defined the users to the VBECS database and assigned a division-specific role to each user.

### Outcome

- A user's role code changed.

### Limitations and Restrictions

- One role at a time may be assigned to a user at a division. The role may be reset.
- VBECS does not allow users to edit or delete the sole Administrator/Supervisor.

### Additional Information

- An undefined user remains in the VBECS database.
- There must be at least one Administrator/Supervisor in the division to maintain users.
- A VBECS user ID is the user's Windows Logon ID.
- One or more users may be selected for batch update.
- A VBECS Application Administrator defines and undefines VBECS users.
- A user may be defined at multiple divisions within a multidivisional database. The settings for a user may differ from division to division and are mutually exclusive.
- See *VistA Blood Establishment Computer Software (VBECS) 2.3.1 Technical Manual-Security Guide*, VBECS Maintenance Operations, Edit Users section for Active Directory tool installation.
- See *VistA Blood Establishment Computer Software (VBECS) 2.3.1 Technical Manual-Security Guide* to add a user to a mail group that receives messages when VistALink errors occur.
- See *VistA Blood Establishment Computer Software (VBECS) 2.3.1 Technical Manual-Security Guide* for administrator activities.

### User Roles with Access to This Option

- Administrator/Supervisor
- Lead Technologist

### Update User Roles

➤➤➤➤➤ An Administrator/Supervisor maintains assigned user roles, including associated levels of system access and the ability to process system overrides. These roles determine the available options and the ability to override certain warnings.

*It is recommended that more than one staff member be assigned the role of Administrator/Supervisor: when more than one Administrator/Supervisor updates roles simultaneously, changes made by one user are not visible to the other.*


One of six user roles may be assigned to users, by division:

- Level 1: Technologist ►
- Level 2: Enhanced Technologist ►►
- Level 3: Lead Technologist ►►►
- Level 4: Traditional Supervisor ►►►►
- Level 5: Enhanced Supervisor ►►►►►
- Level 6: Administrator/Supervisor ►►►►►►

The functions associated with each user role are described in Table 1: User Roles and Functions.

User Action	VBECS
1. Select <b>Tools</b> from the main menu.  Select <b>Update User Roles</b> (Figure 45).	<ul style="list-style-type: none"> <li>• Displays options for processing administrative functions.</li> <li>• Lists active users and associated information, including user name, ID, role, initials and email address.</li> <li>• Displays an option to print a VBECS User Report.</li> </ul> <p><b>NOTES</b> _____</p> <p>Click <b>Print</b> at any time in this process to print a VBECS User Report.</p>
2. Click one or more check boxes to select users.	<ul style="list-style-type: none"> <li>• Allows the assignment of a security level to one or more users at a time.</li> </ul> <p><b>NOTES</b> _____</p> <p>The role that a user selects is assigned to all selected users.</p>
Select a user role from the drop-down list in the Use Role field.	<ul style="list-style-type: none"> <li>• Requests confirmation of the names of users to be changed and the role to be assigned.</li> </ul>
3. Click <b>OK</b> .	
4. Click <b>Yes</b> to confirm the save or <b>No</b> to change the users selected.	<ul style="list-style-type: none"> <li>• Creates an Audit Trail Report entry.</li> </ul> <p><b>NOTES</b> _____</p> <p>See the sample VBECS User Report.</p>
5. Repeat Steps 2–4 to change the roles of additional users.	
6. Click <b>Cancel</b> to exit.	

**Figure 45: Update User Roles**

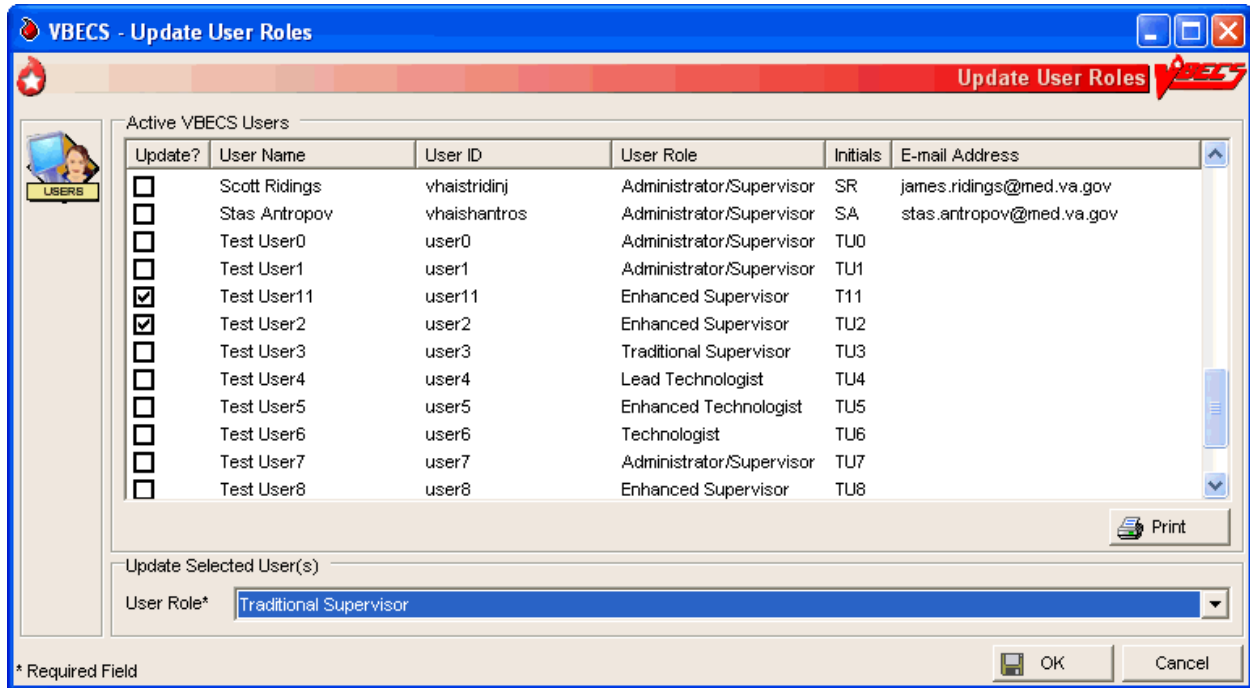

### VBECS User Report

The user may generate a division report that includes the names, VBECS user IDs, and roles of active users. >>> A Lead Technologist may generate a report of current VBECS users, by division, that includes the users' name, user ID, division, and role.

## Component Classes

The user configures the component class parameters for the selected division. For more information, see Appendix F: Work Process Flowcharts.

### Assumptions

- The connection to VistA is active.
- The WHOLE BLOOD and RED BLOOD CELLS component classes are deployed as requiring current specimen ABO/Rh results before issue and may not be edited.
- The FFP, PLT, CRYO and OTHER component classes do not have a specimen requirement defined at deployment.

### Outcome

- Divisions have local settings for all component classes.

### Limitations and Restrictions

- The user may not change the specimen requirement for a component class when there are incomplete orders for that component class.
- There is no retrospective data entry or user selection for this option.

### Additional Information

- None

### User Roles with Access to This Option

➤➤➤➤➤ Administrator/Supervisor

## Component Classes

The user configures the parameters for the division's component classes. These parameters are applied to all blood units that descend from this class. Component classes may be reset when business practices affecting the use of component classes change at a division.

User Action	VBECS
1. Select <b>Tools</b> from the main menu.  Select <b>Component Classes</b> (Figure 46).	<ul style="list-style-type: none"> <li>• Displays options for processing administrative functions.</li> <li>• Lists component classes.</li> <li>• Displays options for configuring division component classes:               <ul style="list-style-type: none"> <li>◦ Require Current Specimen ABO/Rh Results?</li> <li>◦ Maximum Transfusion Time.</li> </ul> </li> <li>• Displays options for defining inappropriate request indicators.</li> </ul> <p><b>NOTES</b> _____</p> <p>See Appendix B: Table 19: CPRS Orderable Blood Components (Component Classes) Mapped to ICCBBA Component Classes.</p>



User Action	VBECS
<p>2. Select a class from the Select Component Class list.</p> <p>Click <b>OK</b> to accept the list, or click the <b>Require Current Specimen ABO/Rh Results?</b> check box, when appropriate.</p>	<ul style="list-style-type: none"> <li>• Requires current specimen ABO/Rh results for issue.</li> <li>• Defines the maximum transfusion times.</li> <li>• Defines inappropriate request indicators.</li> <li>• Allows the user to edit this parameter for component classes that do not contain only red blood cells.</li> <li>• Saves the component to the division list.</li> </ul> <p><b>NOTES</b> _____</p> <p>Component class configuration is required. Component classes: RED BLOOD CELLS, FFP, CRYO, PLT, WHOLE BLOOD, and OTHER.</p>
<p>3. Edit the maximum transfusion time.</p>	<p><b>NOTES</b> _____</p> <p>VBECS records the maximum transfusion time from the transfusion start to the transfusion end time in minutes with a default of 270 minutes. The user may edit the maximum transfusion time to a minimum time of 5 minutes and a maximum time of 999 minutes.</p> <p>VBECS uses the maximum transfusion times in minutes to determine whether the time used to transfuse a unit of blood is inappropriate for a class.</p>
<p>4. Define the inappropriate request indicators.</p> <p>Click the <b>find</b> button to search for a new laboratory test name.</p> <p>Enter a partial laboratory test name and click <b>Search</b>.</p> <p>Select a laboratory test name and click <b>OK</b> to add it.</p> <p>Click the <b>Show Inactive Inappropriate Request Indicators</b> check box, if desired.</p>	<ul style="list-style-type: none"> <li>• Facilitates data entry of one or more inappropriate request indicators: <ul style="list-style-type: none"> <li>o Laboratory Test Name</li> <li>o Specimen Type</li> <li>o Threshold Result</li> </ul> </li> <li>• Uses VistALink to retrieve the laboratory test name and specimen type indicators. The user enters the threshold result indicator.</li> <li>• Displays the entry and allows the user to edit it.</li> <li>• Displays inactive tests and allows the user to reactivate them.</li> </ul> <p><b>NOTES</b> _____</p> <p>VBECS uses the laboratory test name as the only input parameter to search for all possible matches and return the Laboratory Test Name, Specimen Type(s), and VistA Lab Test Internal Entry Number (IEN).</p> <p>A numeric threshold result must include "&lt;" or "&gt;" and a numerical result, e.g., "&gt;8.0."</p> <p>The settings here indicate predetermined levels used to determine the appropriateness of a component order. Settings are different for each component class. The example of a "&gt;8.0" hemoglobin level is set according to local policy.</p> <p>An example of an inappropriate request indicator:</p> <ul style="list-style-type: none"> <li>• The test name is "HGB."</li> <li>• The specimen is entered as "Blood."</li> <li>• The threshold result is set to "&gt;8.0."</li> </ul>
<p>5. Click <b>Add</b> to add the VBECS component class site parameters to the database.</p>	<ul style="list-style-type: none"> <li>• Displays the component class parameters and allows the user to edit them before saving.</li> </ul>

User Action	VBECS
6. Repeat Steps 2–5 for each component class to be edited.	
7. Click <b>OK</b> to save.  Click <b>Yes</b> to confirm changes and exit.	<ul style="list-style-type: none"> <li>Updates the database and captures changes to previously defined parameters for inclusion in an Audit Trail Report.</li> </ul>

**Figure 46: Component Classes**

**VBECS - Configure Component Classes**

**Configure Component Classes**

Select Component Class

- RED BLOOD CELLS**
- FRESH FROZEN PLASMA
- PLATELETS
- CRYOPRECIPITATE
- WHOLE BLOOD
- OTHER

Selected Component Class Details

Require Current Specimen ABO/Rh Results?

**Warning: Setting cannot be configured for RED BLOOD CELLS.**

Maximum Transfusion Time\* 270 Minutes

Inappropriate Request Indicators

Laboratory Test Name  Find

Specimen Type

Threshold Result\*

Add Clear

Active	Lab Test Name	IEN	Specimen Type	Threshold Result

Show Inactive Inappropriate Request Indicators

\* Required Field

OK Cancel

## **Configure Daily QC**

The user sets up the routine reagent racks and QC template for the division before performing daily reagent QC.

### **Assumptions**

- The division is “full service.”
- In the first use of this option, there are no defaults or data in place until the user selects the type of QC template (commercial or non-commercial).

### **Outcome**

- A testing template is available for Enter Daily QC Results; reagent racks and identifiers were created.
- Minimum reaction strengths for each reagent type are defined and used by Enter Daily QC Results.
- A historical record for each rack will be maintained.
- Rack names created are available for use by other options.

### **Limitations and Restrictions**

- This option pertains only to routine reagents.
- When there are partially or currently QC'd racks in the system, the user may not change the QC template.

### **Additional Information**

- This option is disabled for transfusion-only facilities.
- The naming convention for the active template is not editable. When fewer racks are indicated, the system will inactivate existing rack names to keep the list in order without skipping a number or letter. Rack names are reusable. It is recommended for sites comprising a multidivision to assign different reagent rack names to prevent overlap (See FAQ Multidivisional QC Rack Workaround). The daily use field for QC Rack template entries exported as REQUIRED for Daily Use are not editable by the user. The daily use field for entries exported with the daily use as OPTIONAL may be changed to REQUIRED and remain editable by the user at any time.
- If Screening Cell 3 (SC3) and/or (SC4) are set to “required” as part of the configuration of the antibody screening cell set they are enabled on the antibody screen data grid.
- The user must identify an enhancement media reagent type as the primary. In addition, the user must identify the Anti-Human Globulin serum used during daily quality control testing of the screening cells and primary enhancement media.
- VBECS does not evaluate maximum strength of reaction for daily QC, as required by CAP. Local policy must define maximum strength of reaction.

### **User Roles with Access to This Option**

➤➤➤➤➤ Enhanced Supervisor


## **Configure Daily QC**

The user sets up the daily reagent QC template for reagent rack testing for the division. In the first use of this option, all fields default to blank with no template preferred. Additional tests are available and may be included in the daily QC testing template in accordance with local practice. When a user changes the

active template, there is no carryover of field settings to the new template. All are set to the default and configuration must be completed.

User Action	VBECS
1. Select <b>Reagents</b> from the main menu.  Select <b>Configure Daily QC</b> .	<ul style="list-style-type: none"> <li>• Displays options for processing reagents.</li> <li>• Displays an option to configure online Daily Reagent Rack Quality Control.</li> </ul>
2. Click the <b>Commercial</b> or <b>Non-Commercial</b> radio button (Figure 47).	<ul style="list-style-type: none"> <li>• Lists the two system-supplied Daily QC templates.               <ul style="list-style-type: none"> <li>o Commercial Daily QC</li> <li>o Non-commercial Daily QC</li> </ul> </li> <li>• When there are partially or currently QC'd racks in the system, warns the user to wait until QC expires (passes the daily alert time) and then make the changes.</li> </ul>
3. Set the daily alert time.	<ul style="list-style-type: none"> <li>• Asks the user to set the time the daily alert to all users at the site will begin.</li> </ul> <p><b>NOTES</b> _____</p> <p>The user indicates the local time that defines the start of the 24-hour period (the QC day) during which a rack is considered QC'd. VBECS alerts users when a rack was not performed after this local time passed.</p> <p>The user may test a rack no earlier than the alert time for the next 24 hours.</p>
4. Enter the number of racks at the site.  Select a naming convention.	<ul style="list-style-type: none"> <li>• Asks the user to set the number of racks to be QC'd daily at the division.</li> <li>• Asks the user to set the naming convention for racks.</li> </ul> <p><b>NOTES</b> _____</p> <p>A user may indicate 1–26 racks per site and names them according to the convention selected (A–Z or 1–26).</p>
5. Accept or edit the default settings of the template.  Select primary and secondary enhancement medium options from the drop-down menus.	<ul style="list-style-type: none"> <li>• Lists the default reagent types for inclusion in the daily reagent rack quality control and expected minimum reactions for each test type.</li> <li>• Displays changes and edits for review before saving.</li> </ul> <p><b>NOTES</b> _____</p> <p>One setting covers all racks at the site.</p> <p>Reagent types selected display on the rack lot number template and the reaction result entry worksheet for each rack in Enter Daily QC Results with some exceptions: gel kit, selected AHG, and automated testing lots numbers.</p> <p>When previously entered parameters for daily QC rack definitions are changed, VBECS requires a comment before the user saves the update.</p>
6. Click <b>OK</b> to save.	

**Figure 47: Commercial and Non-Commercial Radio Buttons**

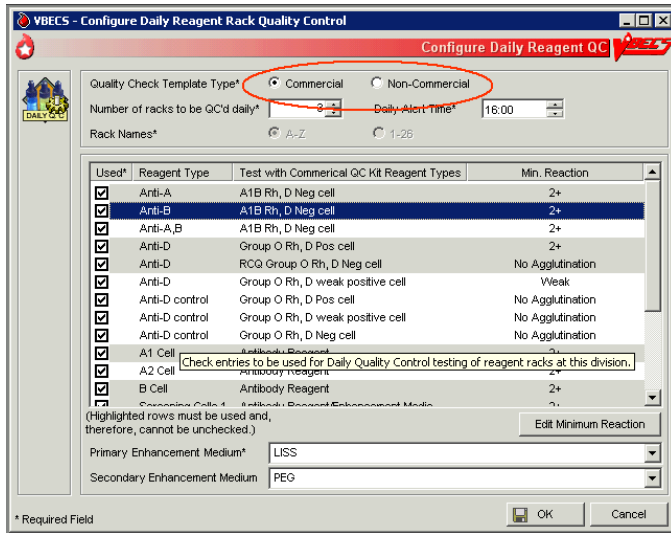

## Configure Division

### Configure Division

A user defines site parameters that affect how VBECS behaves at a specific division. VBECS may be configured to function differently at each division within a consolidated database. Site parameters may be redefined. For additional information, see Appendix F: Work Process Flowcharts.

#### Assumptions

- The site administrator defines the VBECS configuration, adds users, and assigns privileges for each division so that they mirror the associated VistA database.

#### Outcome

- Functional parameters are set for divisions that use the VistA database.

#### Limitations and Restrictions

- Changes made to the division setting are not applied to a user until they re-connect to VBECS after the change is made. It is highly recommended that all users log off VBECS prior to any configuration setting change.

*It is recommended that users configure a division before configuring testing (click the **Configure Testing** tab).*

#### Additional Information

- To incorporate changes in division codes and names from VistA to VBECS, and to change time zones, see *VistA Blood Establishment Computer Software (VBECS) 2.3.1 Technical Manual-Security Guide*.
- VBECS records configuration changes (such as time zone) on the Audit Trail Report.

#### User Roles with Access to This Option

➤➤➤➤➤ Administrator/Supervisor


### Configure Division

Once VBECS is configured to identify all of the divisions, the user sets up critical fields required for routine use of VBECS.

User Action	VBECS
1. Select <b>Tools</b> from the main menu.  Select <b>Configure Division</b> (Figure 48).	<ul style="list-style-type: none"><li>• Displays options for processing administrative functions.</li><li>• Displays the Configure Division tab with fields for entering division-specific blood product information.</li></ul>

User Action	VBECS
<p>2. Enter the ICCBBA Registration Number, the medical director's name (or select it from the drop-down list), and the maximum number of days until the specimen expires in their respective fields.</p>	<p><b>NOTES</b></p> <p>An ICCBBA Registration Number contains an alphabetical character followed by four numeric characters.</p> <p>A user may enter the name of the blood bank medical director or acting medical director to appear on reports. The user may edit the name and title but may select only one name at a time for use. VBECS saves these edits for retrieval.</p> <p>The user may edit the default setting for the maximum number of specimen expiration days from "3" to any value from "3" to "93." This setting allows the user to extend the expiration date of any one specimen to the maximum date indicated. This does not change the normal specimen expiration date of all specimens received.</p>
<p>3. Click <b>OK</b> to save and click <b>OK</b> again to confirm the save and exit, or click another tab.</p>	<ul style="list-style-type: none"> <li>• Displays the information entered.</li> <li>• Requests confirmation to update the database.</li> <li>• Updates the database.</li> </ul>

**Figure 48: Configure Division Tabs**

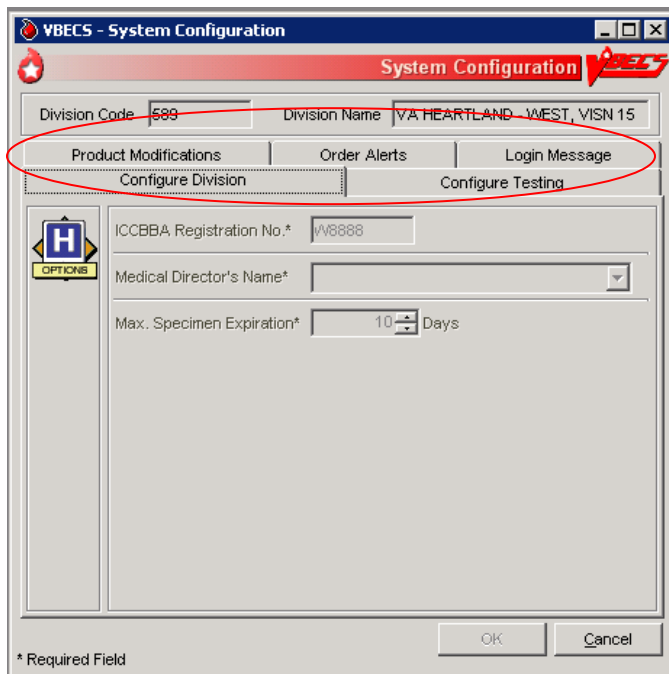

## Configure Testing

The user sets testing preferences according to a division's medical policies.

### Assumptions

- A division's worksheets affected by the option are not open or incomplete at the time of configuration.

### Outcome

- Division testing preferences are configured.

### Limitations and Restrictions

- None

*It is recommended that users configure testing before modifying products (click the **Product Modifications** tab).*

### Additional Information

- None

### User Roles with Access to This Option

➤➤➤➤➤ Administrator/Supervisor

## Configure Testing

The user sets the system to display testing grids to the users in their division. The user determines whether an autocontrol is used and whether a Direct Antiglobulin Test (DAT) is added as part of the routine Antibody Screen Test (ABS) described in Patient Testing: Record a Patient Antibody Screen.

User Action	VB ECS
1. Select <b>Tools</b> from the main menu.  Select <b>Configure Division</b> .	<ul style="list-style-type: none"> <li>• Displays options for processing administrative functions.</li> <li>• Displays the Configure Division tab with fields for entering division-specific testing information.</li> </ul>
2. Click the <b>Configure Testing</b> tab (Figure 49).	<ul style="list-style-type: none"> <li>• Displays the Configure Testing tab with fields for entering division-specific blood product information.</li> <li>• Displays options to set standard operating procedures for a division's laboratory testing.</li> </ul>
3. Click the appropriate radio buttons to:  <b>Enable</b> Electronic Crossmatch  Determine antibody screen (display antibody screen only, or with a DAT or Autocontrol)  Determine the default test grid displayed for ABS and XM Testing Phases	<ul style="list-style-type: none"> <li>• Displays default configuration settings of:               <ul style="list-style-type: none"> <li>o Electronic Crossmatch: Disabled</li> <li>o Antibody Screen: Antibody Screen Only</li> <li>o ABS and XM Testing Phases: All Phases</li> </ul> </li> <li>• Changes to the configuration settings are captured for the Audit Trail Report.</li> </ul>
4. Click <b>OK</b> to save and click <b>OK</b> again to confirm the save and exit, or click another tab.	<ul style="list-style-type: none"> <li>• Displays the appropriate testing grids for subsequent patient orders.</li> </ul>



Figure 49: Configure Testing Tab

The screenshot shows the 'VBECS - System Configuration' dialog box with the 'Configure Testing' tab selected. The 'Division Code' is 589 and the 'Division Name' is VA HEARTLAND - WEST, VISN 15. The 'Service Type' is set to 'Full-Service Facility'. Under 'Electronic Crossmatch\*', 'Disabled' is selected. Under 'Antibody Screen\*', 'Antibody Screen Only' is selected. Under 'ABS and XM Testing Phases\*', 'All Phases' is selected. The dialog includes 'OK' and 'Cancel' buttons and a '\* Required Field' note.

Division Code	589	Division Name	VA HEARTLAND - WEST, VISN 15
Configure Division		Product Modifications	
Order Alerts		Configure Testing	
Login Message			
Service Type	Full-Service Facility		
Electronic Crossmatch*	<input checked="" type="radio"/> Disabled <input type="radio"/> Enabled		
Antibody Screen*	<input checked="" type="radio"/> Antibody Screen Only <input type="radio"/> Antibody Screen with DAT <input type="radio"/> Antibody Screen with Autocontrol		
ABS and XM Testing Phases*	<input checked="" type="radio"/> All Phases <input type="radio"/> AHG Only		

\* Required Field

## Product Modifications

The user enables the types of modifications made at a division. For additional information, see Appendix F: Work Process Flowcharts.

### Assumptions

- An ICCBBA number was entered for the division.
- VBECS locks a user out when the user attempts to change the configuration and there are open or partially completed worksheets or processes in the division.

### Outcome

- This option dictates the available modification types displayed in Modify Units: Pool Units, Modify Units: Split a Unit, and Modify Units.

### Limitations and Restrictions

- None

### Additional Information

- None

### User Roles with Access to This Option

➤➤➤➤➤ Administrator/Supervisor


## Product Modifications

The user enables specific modification type procedures performed in the blood bank in the hospital division. The user may enable one, all, or some of the modification processes. Once the types of modifications are enabled, individual blood unit modification is controlled by the rules of modification truth tables. VBECS is exported with all modification types disabled. VBECS allows for full customization upon deployment as well as changes to the enabled modifications when a division's capabilities and procedures change.

User Action	VBECS
1. Select <b>Tools</b> from the main menu.  Select <b>Configure Division</b> .	<ul style="list-style-type: none"> <li>• Displays options for processing administrative functions.</li> <li>• Displays the Configure Division tab with fields for entering division-specific testing information.</li> </ul>
2. Click the <b>Product Modifications</b> tab (Figure 50).	<ul style="list-style-type: none"> <li>• Lists valid modification procedures.</li> </ul> <p><b>NOTES</b> _____</p> <p>A user may enable one, more, or all modification types.</p>

User Action	VBECS
<p>3. Click one or more check boxes to enable modification types.</p> <p>Enter the processing cost associated with each modification procedure.</p>	<ul style="list-style-type: none"> <li>Lists modification procedures.</li> </ul> <p><b>NOTES</b></p> <hr/> <p>Modification procedures:</p> <ul style="list-style-type: none"> <li>Thaw</li> <li>Pool</li> <li>Thaw/Pool Cryo</li> <li>Split/Divide</li> <li>Irradiate</li> <li>Leukoreduce</li> <li>Volume reduce</li> <li>Wash</li> <li>Rejuvenate</li> <li>Freeze</li> <li>Deglycerolize</li> </ul> <p>The modification process fee is added to the base unit cost of the first target unit and special testing costs, creating a patient bill for a transfused blood product, not including on-site antigen typing performed. The modification process fee may be U.S. \$0.00 to \$9,999.99. VBECS will generate an audit trail report entry when this field is filled or changed.</p>
<p>4. Click <b>OK</b> to save and click <b>OK</b> again to confirm the save and exit, or click another tab.</p>	<ul style="list-style-type: none"> <li>Updates the database and captures changes to any previously defined parameters for inclusion in the Audit Trail Report.</li> </ul>

**Figure 50: Product Modifications Tab**

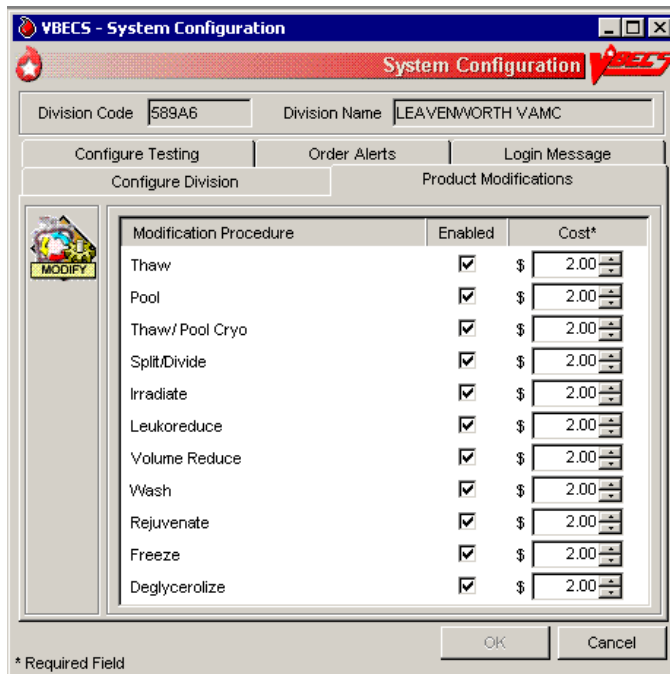

## Order Alerts

The user configures VistA alerts that appear at login. For additional information, see Appendix F: Work Process Flowcharts.

### Assumptions

- None

### Outcome

- User alerts of physician orders for blood bank diagnostic tests and blood components are configured.

### Limitations and Restrictions

- This is an alert only and contains limited information from the CPRS order.
- This option does not create a mechanism to accept or reject orders or specimens or define reports for viewing orders.
- The user may select the option to display printer alerts only when the designated printer is configured and available.
- The printer must be configured by the System Administrator.

### Additional Information

- A user may review current user alert configurations.
- VBECS uses other alerts (patient update alerts, VistALink status alerts) that do not require configuration.
- The user may set VBECS to automatically print an order alert on the designated printer when VBECS acknowledges the order. The CPRS can be configured to print a copy of the order on a printer in the blood bank.
- Printed order alerts are not queued.

### User Roles with Access to This Option

➤➤➤➤➤ Administrator/Supervisor


### Order Alerts

The user configures diagnostic test or component order alerts, and turns them on or off. The alerts notify users when new orders are entered, updated, or canceled in VistA.

User Action	VBECS
1. Select <b>Tools</b> from the main menu.  <b>Select Configure Division.</b>	<ul style="list-style-type: none"><li>• Displays options for processing administrative functions.</li><li>• Displays configuration tabs.</li></ul>

User Action	VB ECS
<p>2. Click the <b>Order Alerts</b> tab (Figure 51).</p> <p>Click the <b>Printer Alerts On</b> and/or the <b>Icon Alerts On</b> check box.</p> <p>Select the order alert type from the drop-down menus in the Order Alert Type fields.</p>	<ul style="list-style-type: none"> <li>• Displays the existing order alert configuration and allows the user to edit it.</li> <li>• Requires the user to select a printer name and to define which orders to print.</li> <li>• Displays the selected printer, the selected order alert type, and the number of hours (default: 48 hours) entered for review.</li> </ul> <p><b>NOTES</b></p> <p>VB ECS lists application server printers.</p> <p>The user must select an order alert type:</p> <ul style="list-style-type: none"> <li>• All orders (orders received by VB ECS)</li> <li>• Hours limit to fill order (orders received by VB ECS needed in the next <i>n</i> hours)</li> <li>• Specimen OK or N/A (orders where no specimen is required)</li> </ul>
<p>3. Enter or select a refresh rate (the interval between searches for new orders) for the order alert in the Refresh Rate field.</p>	<ul style="list-style-type: none"> <li>• Requires the user to enter a refresh rate (default: 60 seconds) (Icon Alerts On).</li> <li>• Displays the updated refresh rate.</li> <li>• Displays the selected order alert type.</li> </ul>
<p>4. Click <b>OK</b> to save</p>	<ul style="list-style-type: none"> <li>• Saves the user alert configuration settings.</li> </ul>
<p>5. Click <b>Yes</b> to confirm the changes and exit.</p>	

**Figure 51: Order Alerts Tab**

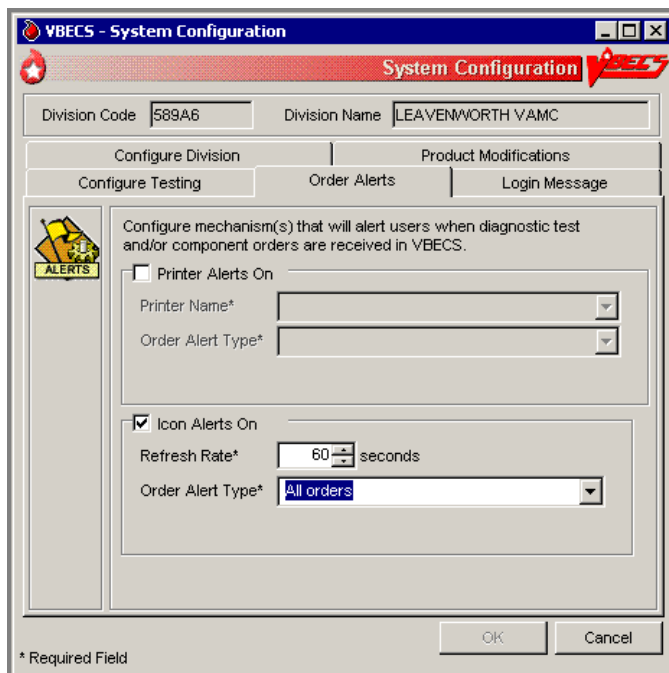

## Login Message

The user creates a division-wide VBECS message that appears at login.

### Assumptions

- None

### Outcome

- A message was created for display when users log into a VBECS division.

### Limitations and Restrictions

- None

### Additional Information

- None

### User Roles with Access to This Option


- Lead Technologist

## Login Message

The user may add, update, or delete a message that VBECS displays to users logging into the division.

User Action	VBECS
1. Select <b>Tools</b> from the main menu.  Select <b>Configure Division</b> .	<ul style="list-style-type: none"><li>• Displays options for processing administrative functions.</li><li>• Displays configuration tabs.</li></ul>
2. Click the <b>Login Message</b> tab (Figure 52).  Enter a message in the message field to display at login.	<ul style="list-style-type: none"><li>• Displays the existing login message and allows the user to edit or delete it.</li><li>• Displays the updated login message.</li></ul> <p><b>NOTES</b> _____</p> <p>VBECS does not display a blank login message.</p> <p>The user may click <b>Clear</b> to clear the message field and enter a new login message.</p>
3. Click <b>OK</b> to save.	<ul style="list-style-type: none"><li>• Saves the login message changes.</li></ul>
4. Click <b>Yes</b> to confirm the changes and exit.	

Figure 52: Login Message Tab

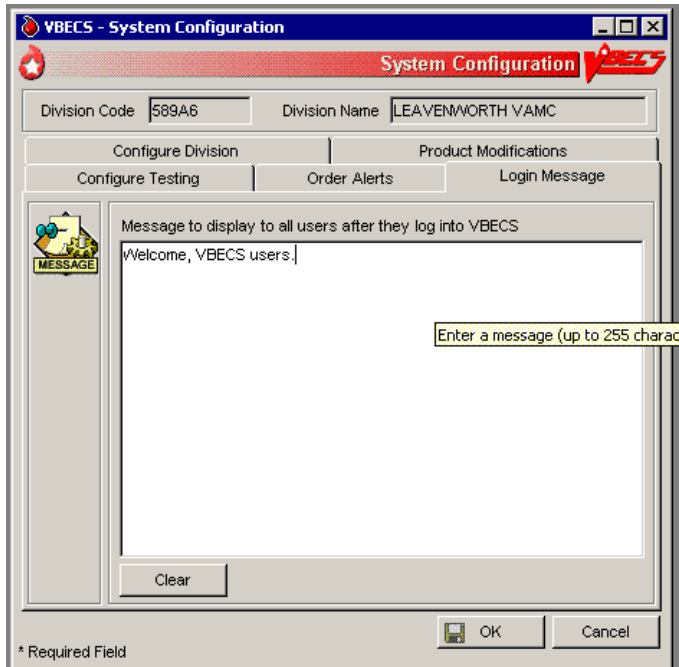

## Local Facilities

The user customizes collection facility information. (A “collection facility” is not a facility for blood collection: it is for blood distribution only.)

### Assumptions

- A table of known FDA-registered facilities accompanies VBECS.

### Outcome

- The supplier is configured for a division.

### Limitations and Restrictions

- A user may:
  - Activate and edit collection facility records distributed with VBECS.
  - Add and locally edit active collection facilities.
  - Activate and/or deactivate local collection facilities.
  - Edit a division name, which may result in a mismatch between VistA and VBECS division names.
  - The Local Facility Address maximum length is 45 characters including spaces to allow the address to display on VBECS reports. The third line of the facility address is not displayed on VBECS reports.

### Additional Information

- A facility must be activated and have at least one blood product defined to be a shipper.
- A user may identify his site as a collection facility through this option.
- An ICCBBA Registration Number has one letter, indicating the country of origin (“W” for the U.S.) and four numbers. The field is five characters long.
- The user may change facility addresses that appear on reports.

### User Roles with Access to This Option

All users

### Local Facilities

This option allows a user to activate an entry from the national collection facility table, thereby adding it to a division’s local collection facility table, and add or update the facility’s information (except for the FDA Registration Number). A user may select an active collection facility to associate blood products (which makes the facility a shipper).

This option also allows a user to create a collection facility.

*Verify that site information entered during installation is correct prior to adding local facilities.*

User Action	VBECS
1. Select <b>Tools</b> from the main menu. Select <b>Local Facilities</b> .	<ul style="list-style-type: none"><li>• Displays options for processing administrative functions.</li><li>• Displays fields for entering facility information.</li><li>• Allows a user to access an existing facility record.</li></ul>



User Action	VBECS
<p>2. To find a collection facility, scan or enter an FDA Registration Number in the FDA Reg. No. field and go to Step 4 (or click the <b>find</b> button to open the Facility Search window).</p>	<ul style="list-style-type: none"> <li>• Displays the collection facilities that match the search criteria.</li> </ul>
<p>3. In the Facility Search window, enter a full or partial facility name in the Partial Facility Name field (or enter a full or partial FDA Registration Number in the FDA Reg. No. field), and click <b>Search</b> to view a list of active facilities.</p> <p>Select a facility name from the list and click <b>OK</b>.</p>	<ul style="list-style-type: none"> <li>• Displays properties of the selected collection facility.</li> </ul> <p><b>NOTES</b> _____</p> <p>The collection facility is tied to the user's current login division.</p> <p>A user may activate a facility. By default, VBECS marks active facilities as collection facilities.</p> <p>➤➤➤➤ The Enhanced Supervisor may enter search criteria that do not exist in the national or the local collection facility tables to add a collection facility to the division's local list. By default, VBECS marks such entries as a collection facility.</p> <p>When a user clears the Active Facility check box for a collection facility in a division, warns the user, and asks whether he wishes to inactivate this facility and continue.</p> <p><b>Yes</b> allows the action to continue. <b>No</b> leaves the collection facility marked as active for the division.</p> <p>In order to inactivate a facility that has been previously configured as a product shipper, you must first inactivate all product shipper entries for that facility. To deactivate product shippers, refer to the Blood Products section. Contact the Service Desk if you would like a list of products linked to the facility you want to inactivate.</p>
<p>4. Edit or enter information in the collection facility information fields (for example, address, telephone number, whether the facility is a testing and/or collection facility) (Figure 53) and click <b>OK</b> to save the updated information.</p>	<ul style="list-style-type: none"> <li>• Displays entries for review and acceptance.</li> <li>• Prompts to save.</li> </ul> <p><b>NOTES</b> _____</p> <p><b>OK</b> remains disabled until the user edits or enters information or simply clicks in a field or check box.</p> <p>The eye-readable prefix (when used) is always a 2-character numeric field.</p>
<p>5. Click <b>Yes</b> to confirm the save.</p>	<ul style="list-style-type: none"> <li>• Saves the active collection facility record and prompts to add or edit another facility.</li> </ul> <p><b>NOTES</b> _____</p> <p>VBECS saves an active collection facility only when it is identified as a collection facility.</p> <p>A facility must be activated and have at least one blood product defined to be a shipper. Refer to Blood Products section for additional information.</p>

User Action	VBECS
6. Repeat these steps to process another facility.	
7. Click <b>Cancel</b> in the Maintain Facility screen to exit.	

**Figure 53: Maintain Facility**

**VBECS - Maintain Facility**

**Maintain Facility**

**FACILITY**

FDA Reg. No.\* 1474346 Find Clear

Facility Name\* VAMC Hines, IL

ICCBBA Reg. No. W0384

Facility Address 5th Avenue and Roosevelt Road

Address Line 2

Address Line 3

City\* Hines

State\* IL Phone ( ) -

Zip Code 60141-0000 Fax ( ) -

Codabar Unit Format

Unit ID contains alpha characters?

2-digit eye-readable prefix (if applicable)

Facility Type\*

Collection Facility?

Testing Facility?

Active Facility?

OK Cancel

\* Required Field

## Blood Products

The user activates, deactivates, and edits a limited number of variables for an existing blood product site parameter. VBECS provides an ISBT 128 Blood Product reference table to facilitate these and other actions.

### Assumptions

- The connection to VistA is active.
- The user understands Healthcare Common Procedure Coding System (HCPCS) codes. (See Additional Information.)
- A shipper must be marked as an active collection facility for a division before a user may select it as a valid shipper for a blood product for that division. This includes marking the division facility itself as an active collection facility to accommodate the creation of new blood products through modification.

### Outcome

- Blood product codes are activated in the user's division.
- Each product's HCPCS code and its activation and deactivation dates are maintained in a permanent historical record.

### Limitations and Restrictions

- Blood product records distributed through VBECS are national standards.
- The division must have activated the valid target blood products for modification.

### Additional Information

- HCPCS codes are required for medical and surgical services and supplies for the outpatient setting under the final code set standards for the Health Insurance Portability and Accountability Act of 1996 (HIPAA). See your local Current Procedural Terminology (CPT) coordinator for additional information.
- At least one valid shipper and product cost must be defined for each active product code in use in the division; multiple shippers may be associated with a blood product code.

### User Roles with Access to This Option

All users

## Blood Products

Before processing a blood unit in an incoming shipment, or activating or creating a blood unit in-house, the user identifies the shipper(s), cost, and return credit percentage for each shipper. The user also tags each product as active for use within the division and assigns an HCPCS code to each. These are the only fields the user may edit when maintaining an existing blood product code.

A "shipper" is the collection facility that delivers and bills for the blood product, regardless of the facility named on the unit's label. The collection facility that delivers the product to the shipper is not considered a shipper nor does it have a product cost associated with it, but it needs to be activated in the division for VBECS to interpret the unit ID. When components are modified within the division, the shipper is defined as the division. The cost may be \$0.00 or other U.S. dollar value assigned by the shipper.

User Action	VBECS
<p>1. Select <b>Tools</b> from the main menu.</p> <p>Select <b>Blood Products</b>.</p>	<ul style="list-style-type: none"> <li>• Displays options for processing administrative functions.</li> <li>• Displays fields for entering blood product information.</li> </ul>
<p>2. To identify the product, scan or enter a. ISBT product code.</p> <p>Click the <b>find</b> button to search for a product.</p> <p>Select a product type from the drop-down Product Type list or enter a partial product name in the Partial Product Name field.</p> <p>Click <b>Search</b>.</p>	<ul style="list-style-type: none"> <li>• Displays current blood product information.</li> </ul> <p><b>NOTES</b> _____</p> <p>When a user enters a product code that VBECS does not recognize, VBECS warns that the blood product was not found and that it does not support entry of new blood products.</p> <p><b>OK</b> returns the user to a blank screen to enter a different product. VBECS clears the screen. The user may enter a different product code.</p>
<p>3. Select a product from the Search Results list and click <b>OK</b> (Figure 54).</p> <p>Select an HCPCS code from the drop-down HCPCS Code list or enter an HCPCS code.</p>	<ul style="list-style-type: none"> <li>• Allows a user to enter the HCPCS code.</li> </ul> <p><b>NOTES</b></p> <p>A blood product may have only one HCPCS code at a time.</p> <p>A user may also add a HCPCS code after the blood product code is activated.</p>
<p>4. Select or enter product shipper information: shipper name, FDA Registration Number, cost, and return credit percentage.</p>	<p><b>NOTES</b> _____</p> <p>Site parameters determine which products are active for a division. Products marked as active can be received in an incoming shipment or created in-house through modification.</p> <p>A user with no increased security can maintain an existing blood product for use only within the user's current division.</p> <p>The return credit default is "100%."</p> <p>In order to inactivate a product shipper, all units via that shipper must be in a final status (or inactivated). Contact the Service Desk if you would like a list of units linked to the product shipper you want to inactivate.</p>
<p>5. Repeat Steps 3 and 4 for each selected product's shipper and for other products.</p>	<ul style="list-style-type: none"> <li>• Displays the information entered.</li> <li>• Requests confirmation to update the database.</li> <li>• Updates the database or warns the user with no override for discrepancies.</li> </ul>
<p>6. Click <b>OK</b> to exit.</p>	

Figure 54: Maintain Blood Products

**Blood Product**

Product Code\* 04210 Find Clear

Short Product Name RBC CPD>AS1

Full Product Name AS-1 RED BLOOD CELLS

Product Type RED BLOOD CELLS

Volume 350 mL

Maximum Storage 42 Days 0 Hours

**HCPCS**

HCPCS Code\* P9023

HCPCS Text\* FROZEN PLASMA, POOLED, SD

**Product Shippers**

Shipper Name\*

FDA Reg. #\*

Cost (\$)\* 0.00

Return Credit %\* 100 Add

Active	FDA Reg #	Shipper	Cost	R
<input checked="" type="checkbox"/>	1474396	VAMC North Chicago, IL	\$1.00	1
<input checked="" type="checkbox"/>	1472204	LifeSource - Glenview	\$1.00	1

\* Required Field

OK Cancel

## Antibodies

The user edits certain fields in the supplied antibody table.

### Assumptions

- VBECS is configured to reflect antibody reference parameters. Only some of these parameters may be changed locally.
- The default settings are enforced without user interaction.

### Outcome

- Each division has a populated antibody table whether or not the user makes any edits.

### Limitations and Restrictions

- None

### Additional Information

- The antibody table defines an antigen negative requirement for all instances of the antibody for all patients in the division. When an antibody specificity does not generally require antigen negative blood but a patient has a unique appearance of that antibody that requires antigen negative blood, the user may enter the antigen negative requirement for that patient in Special Instructions & Transfusion Requirements: Enter a Transfusion Requirement.
- Anti-A,B is not selectable as a patient antibody. Users can enter the Anti-A,B antibody in the patient's Special Instructions, if necessary.

### User Roles with Access to This Option

➤➤➤➤➤ Enhanced Supervisor


## Antibodies

The user chooses to edit the system-supplied blood antibody table. The Compatibility Percentage field is fully editable (except for ABO antibodies and anti-D) to allow facilities to reflect local populations. The user may edit only the Clinically Significant entries initially exported as "No."

User Action	VBECS
1. Select <b>Tools</b> from the main menu.  Select <b>Antibodies</b> .	<ul style="list-style-type: none"> <li>• Displays options for processing administrative functions.</li> <li>• Lists available antibodies based on the system-supplied table.</li> </ul> <p><b>NOTES</b> _____</p> <p>See Appendix B: Table 14: Antibody and Antigen Table: Irregular Antibodies.</p> <p>The table's presentation order is predefined at implementation and may not be sorted.</p>

User Action	VBECS
2. Select the antibody, edit each field, and click <b>Update</b> (Figure 55).	<ul style="list-style-type: none"> <li>Displays data and allows the user to edit them.</li> <li>Accommodates data entry of whole numbers (0–100) and edit of the Antigen Negative Compatibility Percentage and Higher-Level Override fields. The user may not edit fields displayed on a gray background.</li> </ul> <p><b>NOTES</b></p> <p>When Table 14: Antibody and Antigen Table: Irregular Antibodies “antibody sets an antigen negative requirement” field is defined as REQ (required) at installation, it is not editable. When the default settings is OPT (optional), the user may change the name to REQ at any time and it will remain editable.</p>
3. Repeat for each antibody to be edited.	
4. Click <b>OK</b> and <b>Yes</b> to confirm changes and exit.	

**Figure 55: Maintain Antibodies**

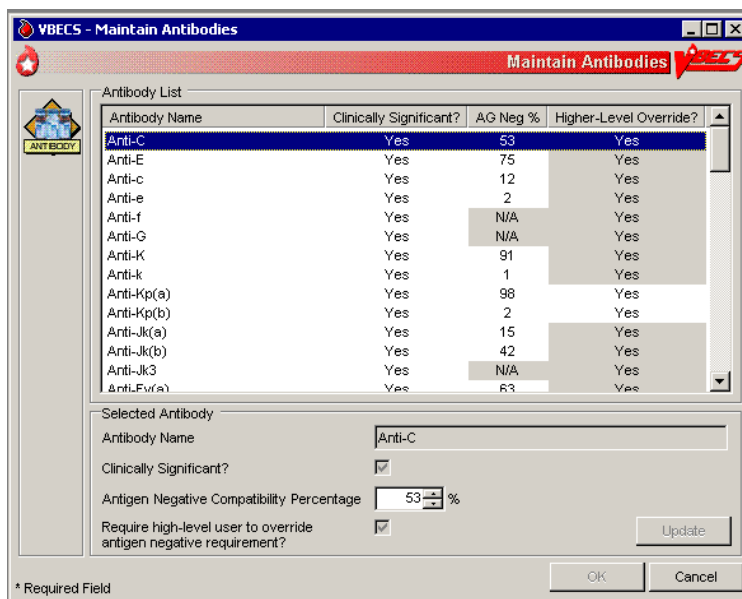

## Canned Comments

The user defines canned comments for a division.

### Assumptions

- None

### Outcome

- Canned comments for a division are available for selection.

### Limitations and Restrictions

- The user may not edit comments provided with VBECS but may create additional comments. Although comments can also be removed, there should remain at least one comment per category available for selection, (e.g., “Other”).
- Site personnel must enter comments in accordance with local policy and procedures.
- If you receive a system error when entering canned comments, the user who configured the division must be added to the users list in VBECS. File a service request if the user is not known.

### Additional Information

- Canned comments are specific to each division. See Appendix B: Table 17: Canned Comment Category Types and Message Text defines the category types and comments.
- Currently, VBECS does not identify the testing laboratory and include its address on results reports in CPRS. While this will be fixed in a later version, a workaround for the issue is to create Canned Comments for the missing information and apply them to blood bank results.

User Roles with Access to This Option

➤➤➤➤➤ Enhanced Supervisor

## Canned Comments

It is recommended that the user review the canned comments provided with VBECS for each category type and inactivate those that will not be used.

The user creates canned comments to use in lieu of entering text. Canned comments are restricted for selection based on the category type of the comment field.


The user may select a comment and has the option to enter explanatory text in the details field.

User Action	VBECS
1. Select <b>Tools</b> from the main menu.  Select <b>Canned Comments</b> .	<ul style="list-style-type: none"><li>• Displays options for processing administrative functions.</li></ul>
2. Select a comment category type from the drop-down list (Figure 56).	<ul style="list-style-type: none"><li>• Lists the category types for canned comments.</li><li>• Allows the user to create a comment or inactivate an existing one.</li></ul>



User Action	VBECS
<p>3. Enter text in the Comment field and click <b>Add</b> to add a comment.</p> <p>Click one or more check boxes to inactivate existing comments.</p>	<ul style="list-style-type: none"> <li>Displays entered data and allows the user to edit them before saving.</li> </ul> <p><b>NOTES</b> _____</p> <p>A user may:</p> <ul style="list-style-type: none"> <li>Add canned comments.</li> <li>Inactivate canned comments.</li> <li>Reactivate inactive comments.</li> <li>View inactive comments and dates of inactivation.</li> <li>Control the order in which VBECS displays comments in a context.</li> </ul>
<p>4. Click the <b>Show inactive Comments</b> check box to view inactive comments.</p> <p>Click one or more check boxes to reactivate inactive comments.</p>	<ul style="list-style-type: none"> <li>Displays inactive comments and allows the user to reactivate them.</li> </ul>
<p>5. Repeat Steps 2–4, as needed.</p> <p>Click <b>OK</b> and <b>Yes</b> to save and exit, or <b>Cancel</b> to exit without saving.</p>	<ul style="list-style-type: none"> <li>Updates the database.</li> </ul>

**Figure 56: Maintain Comments**

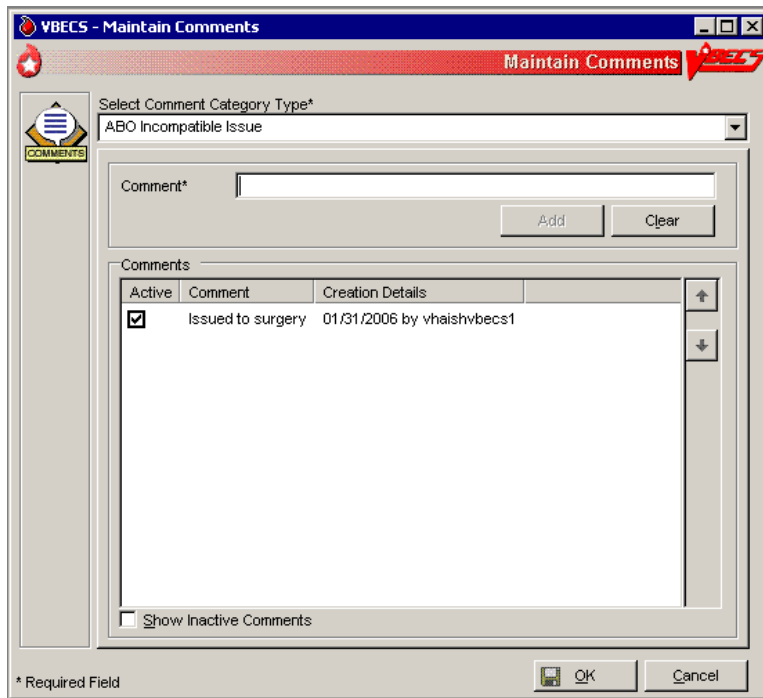

## Setting Transfusion Parameters

### Transfusion Complications

The user defines Laboratory tests and threshold values used to generate a report listing patients who were transfused within a specified period and who may have developed a transfusion-associated disease. For additional information, see Appendix F: Work Process Flowcharts.

#### Assumptions

- VistA tests are defined.
- The connection to VistA is active.

#### Outcome

- Parameters are defined for future Transfusion Complication Reports.

#### Limitations and Restrictions

- None

#### Additional Information

- The report searches for evidence of a transfusion in VBECS only. If the user enters a start date that requires a search of VistA records, the report includes a message indicating that only transfusions documented in VBECS are included in the report and that the user needs to run a supplementary report in VistA.

#### User Roles with Access to This Option

➤➤➤➤➤ Enhanced Supervisor

### Transfusion Complications

The user defines Laboratory tests for use in the transfusion complication table.

User Action	VBECS
1. Select <b>Tools</b> from the main menu.  Select <b>Transfusion Complications</b> .	<ul style="list-style-type: none"><li>• Displays options for processing administrative functions.</li><li>• Displays fields for entering transfusion complication threshold data.</li></ul>
2. Click the <b>find</b> button to search for a Laboratory test.  Select a test.  Click <b>OK</b> .	<ul style="list-style-type: none"><li>• Allows the user to edit or add previously defined tests for use in a Transfusion Complication Report.</li></ul> <p><b>NOTES</b> _____</p> <p>VBECS displays the entered laboratory test names in all uppercase letters. The retrieval of lab tests occurs in VistA and results for the Transfusion Complications Report are not case sensitive. There is a limit of 25 active entries. (If the user enters all of the available results for the test, he can get results that are not filtered by VBECS.)</p>

User Action	VBECS
<p>3. Click the <b>Numeric Threshold</b> radio button and enter or edit test threshold information (Figure 57).</p> <p>Click <b>Add</b> to add the test to the list, or</p> <p>Click <b>Update</b> to save changes to a previously saved test.</p> <p>Click <b>OK</b> to save and <b>Yes</b> to confirm the save.</p>	<ul style="list-style-type: none"> <li>Allows the user to enter one or more tests.</li> <li>Uses VistALink to retrieve test data.</li> </ul> <p><b>NOTES</b></p> <p>When defining a test reported as a text value, VBECS accommodates the input of specific text (1–255 characters) that will be used to search for matching values.</p>
<p>4. Click <b>OK</b> to save and <b>Yes</b> to confirm the save and exit.</p>	<ul style="list-style-type: none"> <li>Updates the database.</li> </ul>
<p>5. Repeat Steps 2–4 for each transfusion complication test to be defined or click <b>Cancel</b> to exit.</p>	

**Figure 57: Transfusion Complication Thresholds**

## Transfusion Effectiveness

The user defines Laboratory tests to monitor the effectiveness of transfusions.

### Assumptions

- VistA lab tests are defined.
- The connection to VistA is active.

### Outcome

- Parameters are defined for future reports.

### Limitations and Restrictions

- None

### Additional Information

- None

### User Roles with Access to This Option

- Enhanced Supervisor

## Transfusion Effectiveness

The user enters the tests to be included in the Transfusion Effectiveness Report.

User Action	VBECS
1. Select <b>Tools</b> from the main menu.  Select <b>Transfusion Effectiveness</b> .	<ul style="list-style-type: none"> <li>• Displays options for processing administrative functions.</li> <li>• Allows the user to edit a previously defined Transfusion Effectiveness Tests or add a test.</li> </ul>
2. Click the <b>find</b> button to search for a Transfusion Effectiveness Test.  Enter part or all of a laboratory test name.  Click <b>Search</b> .  Click one to four check boxes to select tests.  Click <b>OK</b> .  Click <b>Add</b> to add the tests (Figure 58), or  Click the <b>Show Inactive Transfusion Effectiveness Report Tests</b> check box, if desired.  Click <b>Update</b> to edit the tests.	<ul style="list-style-type: none"> <li>• Allows the user to enter one or more Transfusion Effectiveness Tests.</li> <li>• Uses VistALink to retrieve Laboratory test data for each of these indicators:               <ul style="list-style-type: none"> <li>○ Laboratory test name</li> <li>○ VistA Laboratory test Internal Entry Number (IEN)</li> <li>○ Specimen type</li> </ul> </li> </ul>

User Action	VBECS
3. Repeat Step 2 for each Transfusion Effectiveness Test to be defined, or click <b>Cancel</b> to exit.	
4. Click <b>OK</b> to save and <b>Yes</b> to confirm the save.	<ul style="list-style-type: none"> <li>Updates the database, including the date and time and the technologist ID. Saves changes to previously defined tests for inclusion in an Audit Trail Report.</li> </ul>

**Figure 58: Transfusion Effectiveness Report Tests**

Define Transfusion Effectiveness Report Test

Laboratory Test Name\*  Find

Specimen Type\*

Add Clear

Transfusion Effectiveness Report Tests

Active	Lab Test Name	IEN	Specimen Type

Show Inactive Transfusion Effectiveness Report Tests.

\* Required Field

OK Cancel

## MSBOS

The user defines parameters for comparing blood component orders placed by a clinician with the Maximum Surgical Blood Ordering Schedule (MSBOS).

### Assumptions

- A table of generic surgery names is available for MSBOS activation.

### Outcome

- CPRS users will see only active surgery names when placing a blood bank order.
- MSBOS will be available to VBECS users to evaluate the appropriateness of pre-op orders.

### Limitations and Restrictions

- None

### Additional Information

- The maximum number of blood products serves as a guideline. Orders in excess of the MSBOS are not prohibited.
- The VBECS user may view active and inactive surgery names.

### User Roles with Access to This Option

➤➤➤➤➤ Enhanced Supervisor


## MSBOS

The user selects a surgery name from the generic surgery list or creates a new surgery name, then defines the associated MSBOS recommendation. MSBOS is used when VBECS acknowledges orders from VistA that are written for preoperative patients.

User Action	VBECS
1. Select <b>Tools</b> from the main menu. Select <b>MSBOS</b> .	<ul style="list-style-type: none"><li>• Displays options for processing administrative functions.</li><li>• Displays the schedule and component recommendation options.</li></ul>

User Action	VBECS
<p>2. To activate an existing surgery name, click the <b>Show Inactive MSBOS entries</b> check box (Figure 59).</p> <p>Select a surgery name.</p> <p>Click the <b>Active MSBOS Entry?</b> check box, or</p> <p>To add and activate a surgery name, click <b>New Surgery</b> and enter a surgery name in the Surgery Name field. Click <b>Add</b>, or</p> <p>To edit and/or inactivate an existing surgery name, click the appropriate options in the MSBOS Component Recommendation area of the screen edit. Click <b>Update</b>.</p>	<ul style="list-style-type: none"> <li>• Lists active surgery names or allows the user to create an entry.</li> <li>• Activate, add, or edit a MSBOS entry and its associated data.</li> <li>• Displays surgery names available for activation.</li> <li>• Displays entries for review and acceptance.</li> </ul> <p><b>NOTES</b></p> <p>MSBOS is specific to the division.</p> <p>See Appendix B: Table 24: VBECS Maximum Surgical Blood Order Schedule (MSBOS).</p> <p>Each surgery name has a Type &amp; Screen (TAS) (no blood order recommended) or one or more component classes defined as recommended orders.</p> <p>The user may select one or more component classes for MSBOS recommendation. Each defined component class must have an associated maximum number recommended. The user may inactivate previously activated component classes for an MSBOS recommendation. The OTHER component class is not available for selection.</p>
<p>3. Repeat Step 2 until all new entries are created or existing entries are edited.</p> <p>Click <b>OK</b> to save the updated data.</p>	<p><b>NOTES</b></p> <p>VBECS does not save changes to the database when the user clicks <b>Cancel</b> without clicking <b>OK</b>.</p>
<p>4. Click <b>Cancel</b> to exit.</p>	

**Figure 59: Maintain MSBOS**

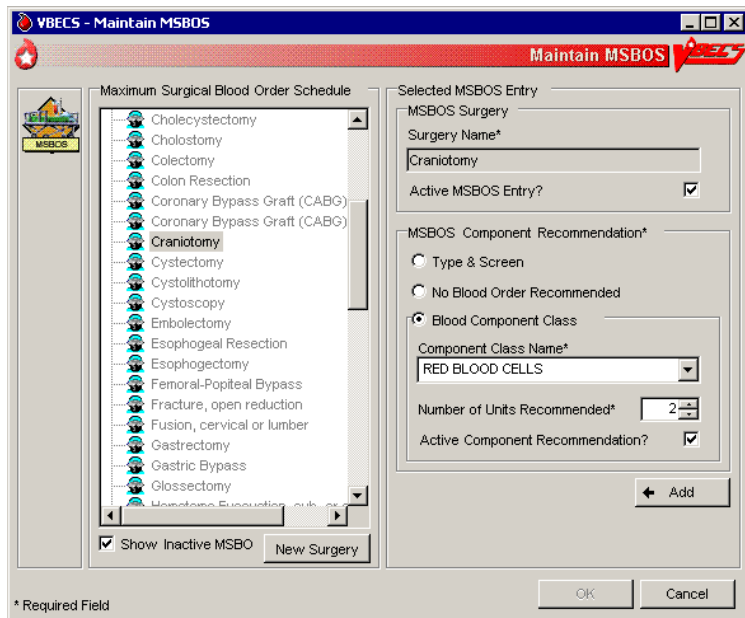

## Workload Codes

The user assigns workload Laboratory Management Index Program/National Laboratory Test (LMIP/NLT) tests to processes performed through VBECS. VBECS test names are associated with VistA workload codes, which VBECS uses when compiling and storing data to provide workload-related reports to VistA. For additional information, see Appendix F: Work Process Flowcharts.

### Assumptions

- The VistA database was updated to include all required workload codes for the VBECS process.
- The VistA database was updated to associate CPT codes and costs with the workload codes to be used by VBECS.
- The connection to VistA is active.

### Outcome

- Data are available to the VistA workload package.

### Limitations and Restrictions

- Transfusion-only sites must not map workload codes to processes, such as ABO/Rh testing, not performed on site.
- Retrieved LMIP/NLT codes from VistA may have multiple CPT codes. Inactivating any of the associated CPT codes in VBECS will inactivate the LMIP/NLT code. Editing of the LMIP/NLT code associations must be done in VistA.
- Invalidating split units will result in extra workload equal to the number of splits created applied to the total number of units logged in. This occurs when split units are incorrectly processed in VBECS.

### Additional Information

- VistALink is required to query VistA files 64 (NLT/LMIP) and 81 (CPT) for appropriate entries.
- The division provides CPT codes and costs associated with the workload codes when they need updating.
- Each site is responsible for notifying users when incorrect CPT codes are used.
- Workload events are placed in a queue for transmission via VistALink.

### User Roles with Access to This Option

- Administrator/Supervisor

## Workload Codes


As part of record-keeping activity, VBECS allows the user to account for workload units associated with each activity and to assign costs incurred by the blood bank due to these activities. Data gathered because of this workload collection are available to VistA, as needed, for various reports.

User Action	VBECS
1. Select <b>Tools</b> from the main menu.  Select <b>Workload Codes</b> .	<ul style="list-style-type: none"><li>• Displays options for processing administrative functions.</li><li>• Displays fields for entering workload-related data.</li><li>• Lists and retrieves processes that may involve workload.</li></ul>



User Action	VBECS
<p>2. Select a process for which to define workload codes.</p>	<ul style="list-style-type: none"> <li>Allows the user to edit existing data or add data.</li> </ul> <p><b>NOTES</b> _____</p> <p>The user may assign multiple LMIP and CPT workload codes to a process.</p> <p>VBECS checks for active and inactivated workload associations and does not allow the user to enter a duplicate association. VBECS allows the user to reactivate previously inactivate entries and add workload associations.</p>
<p>3. Select an LMIP/NLT code.</p> <p>Click <b>Add</b> to add the code, or click <b>Update</b> to select a current association to edit.</p>	<ul style="list-style-type: none"> <li>Lists previously assigned LMIP/NLT codes for the selected process.</li> <li>Allows the user to select an LIMP/NLT code.</li> </ul> <p><b>NOTES</b> _____</p> <p>VBECS uses VistALink to query the VistA WKLD CODE File (#64) for valid LMIP/NLT codes.</p> <p>VistA uses the LMIP/NLT code to create workload reports as well as reports to patient care encounter (PCE) and the decision support system (DSS) database.</p>
<p>4. Enter the start or stop date (whichever is appropriate) of the workload code association with the VBECS process (Figure 60).</p>	<ul style="list-style-type: none"> <li>Allows the user to enter a date.</li> </ul> <p><b>NOTES</b> _____</p> <p>The user may enter an inactivation date only when an activation date was previously entered (default start date and time: current date and time).</p> <p>The inactivation date must be later than or the same as the activation date. When the user attempts to enter an inactivation date that is earlier than the activation date, VBECS warns the user. There is no override.</p>
<p>5. Repeat Steps 3–4 for each LMIP/NLT code to be associated with the selected process.</p>	
<p>6. Add the selected parameters to the selected process.</p>	<ul style="list-style-type: none"> <li>Allows the user to add the selected parameters.</li> </ul>
<p>7. Repeat Steps 1–6 for to define workload parameters for another process, or exit.</p>	

**Figure 60: Assign Workload Codes**

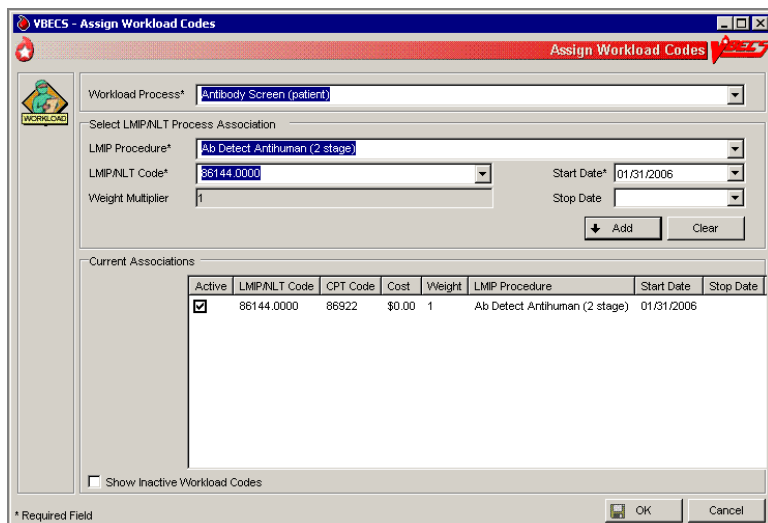

### Remapping Workload Codes

When workload codes or a workload code multiplier is changed, VBECS must be updated. The active workload process to LMIP/NLT Code association must be inactivated and remapped for VBECS to track workload correctly.

User Action	VBECS
1. Select <b>Tools</b> from the main menu.  Select <b>Workload Codes</b> .	<ul style="list-style-type: none"> <li>Displays options for processing administrative functions.</li> <li>Displays fields for entering workload-related data.</li> <li>Lists and retrieves processes that may involve workload.</li> </ul>
2. Select a process to edit workload codes.	<ul style="list-style-type: none"> <li>Allows the user to edit existing data or add data.</li> </ul>
3. Click one or more check boxes to inactivate LMIP/NLT Code.  Click <b>OK</b> and <b>Yes</b> to save and exit.	<ul style="list-style-type: none"> <li>Removes the code from the Current Associations window.</li> </ul> <p><b>NOTES</b> _____</p> <p>Inactive codes can be viewed by clicking the Show Inactive Workload Codes checkbox.</p>
4. Click <b>OK</b> and <b>Yes</b> to save and exit.	<ul style="list-style-type: none"> <li>Saves the changes to the workload code mapping.</li> </ul>
5. Select <b>Tools</b> from the main menu.  Select <b>Workload Codes</b>	<ul style="list-style-type: none"> <li>Displays options for processing administrative functions.</li> <li>Displays fields for entering workload-related data.</li> <li>Lists and retrieves processes that may involve workload.</li> </ul>
6. Select a process to edit workload codes.	<ul style="list-style-type: none"> <li>Allows the user to edit existing data or add data.</li> </ul>

User Action	VBECS
<p>7. Select an LMIP/NLT code.</p> <p>Click <b>Add</b> to add the code.</p>	<ul style="list-style-type: none"> <li>• Allows the user to select an LIMP/NLT code.</li> </ul> <p><b>NOTES</b> _____</p> <p>LMIP/NLT weight multipliers values are set in VistA and that value is imported into VBECS.</p> <p>Changes made to VistA files 64 (NLT/LMIP) and 81 (CPT) are updated in VBECS on a rolling 24-hour clock. The first user to log in 24 hours after the last update will trigger the next update.</p>
<p>8. Repeat Steps 1–7 for to edit workload parameters for another process.</p>	

## ***Reagents and Supplies***

### **Enter Daily QC Results**

The user enters observed serological reactions for all phases of testing and interpretations of the various routine reagents.

#### **Assumptions**

- The division is “full service.”
- Site parameters regarding reagent quality control are set up.
- Minimum inventory levels for reagent types are defined.
- Inventory control functionality is in use and reagents are in the inventory of the user’s division.
- Daily QC testing does not include reagents used in special antigen typing.

#### **Outcome**

- Routine reagents are available for use, shutting down the alert for the day associated with the rack.
- The user may enter and save reaction results below the minimum expected reaction; an Exception Report may be generated for supervisory review; the rack may or may not be used by the site for routine testing throughout that day.

#### **Limitations and Restrictions**

- Additional user-defined reagent types not available in Configure Daily QC or Enter Daily QC Results are not part of a reagent rack’s daily QC testing.

#### **Additional Information**

- VBECS allows the user to save a partially filled worksheet for a specific QC rack ID and allows the same or a different user to redisplay the form for the entry of remaining test results at a later time. The user, first or second, may enter new results but may not edit prior entries. VBECS records the ID of each user who enters test observations on a worksheet.
- VBECS issues warnings when they are defined in Maintain Minimum Levels.
- VistALink does not need to be active when a user enters daily quality control results.
- A testing template is generated for daily quality control testing of routine reagents for each rack in a division. (Multiple templates may be used on a given day.)
- System rules determine the validity of the test interpretations and determine if all phases of testing were recorded for both scenarios.
- A user must enter and may view one day’s QC data at a time. The supervisor or administrator must review these entries and the reagent information daily.
- VBECS compares the current rack QC results with the prior day’s QC results.
- Multiple technologists may use the same rack.
- Rack number and reagents used to test a specimen are linked to the lot number, manufacturer, and expiration date of the reagent type as well as the rack identifier.
- A user may use more than one testing method in accordance with local policies and procedures.
- If a rack has never been used with reagents prior to using the QC Documentation Offline option and the rack is not so checked when the user attempts to use this untested rack, the message appears that this

rack has never been tested and cannot be used. The user may not proceed. To use this rack on this date, the QC Offline check needs to be indicated or testing performed.

- A rack can be set as partially tested and marked as "QC'ed Offline" at the same time. The rack will be listed as "Tested = No", Selecting it will prompt for the "Rack QC not performed" override, as the partially tested rack setting is enforced.
- The antiglobulin testing grids for QC have a title of IAT. The same grid for patient testing is titled AHG.
- If two instances of Enter Daily QC are opened, the second instance will not recognize testing performed in the first instance, and will accept retesting of the rack. See FAQ QC Setup and FAQ Retesting QC for additional information.

## User Roles with Access to This Option

All users


## Enter Daily QC Results



The American Association of Blood Banks' *Standards for Blood Banks and Transfusion Services* requires that commercial reagents in the blood bank—antisera and routinely-used red cell reagents—must be tested for performance “on the day of use.” As routine reagents are always used, they are tested daily. Records of these reagents, their lot numbers, and their testing results must be stored for reference. VBECS verifies and indicates whether the reagent is satisfactory for use.

This option facilitates the direct entry of reaction results and interpretations, constituting daily reagent quality control testing for a rack of reagents.

VBECS allows the user to change individual reagents without repeating the QC for the full rack during the day.

User Action	VBECS
1. Select <b>Reagents</b> from the main menu.  Select <b>Enter Daily QC Results</b> .	<ul style="list-style-type: none"> <li>• Displays options to perform daily reagent rack QC.</li> </ul>
2. Verify the default date tested and tester, and edit them, if needed.	<ul style="list-style-type: none"> <li>• Displays the date and time the testing was performed; the default is the current date and time, which are editable to allow retrospective entry.</li> <li>• Displays the testing technologist's identification; the default is the current user. Lists valid division users for selection.</li> <li>• Displays the option to indicate the rack number of the reagents to test, a list of racks to be tested for the current date, and a retest option for the racks with testing completed for the date.</li> </ul> <p><b>NOTES</b> _____</p> <p>Daily QC testing does not include reagents used in special antigen typing.</p>

User Action	VBECS
<p>3. Select the <b>Test Rack(s), Partially Tested Rack(s),</b> or <b>Retest Rack(s)</b> tab (Figure 61).</p> <p>Click one or more check boxes to select rack numbers.</p> <p>Click <b>OK</b>.</p>	<ul style="list-style-type: none"> <li>• Displays the lot number verification template for each rack selected.</li> </ul> <p><b>NOTES</b> _____</p> <p>A reagent rack is displayed on one of three tabs based on its testing status and the daily alert time setting:</p> <ul style="list-style-type: none"> <li>• Test Rack(s): the current alert time has passed and is/are considered “untested”</li> <li>• Partially tested: racks that have incomplete test results for the current 24-hour time clock and are considered “untested”</li> <li>• Retest Racks: racks that are fully tested within the current alert time and are available for new lot number selection and its testing.</li> </ul> <p>The user may test a rack no earlier than the alert time for the next 24 hours.</p> <p>On occasion, a user may save a partially completed QC worksheet and finish it later. The same user or a different user may complete the testing.</p> <p>The user may retest a rack when a reagent lot number changes after initial testing. VBECS does not allow the user to retest saved valid results. VBECS requires the user to indicate the rack identifier to provide a testing template for each daily reagent rack QC.</p> <p>Site settings determine reagent types and lot numbers present. VBECS displays only reagent types previously entered for the division using the reagent inventory function. VBECS allows entry of lot numbers only. When VBECS displays lot numbers for selection, it checks the inventory system for the reagent name and lot numbers in use in a rack. VBECS does not offer a reagent with an unsatisfactory value in the “Inspection” field or with a quantity of zero as a selectable reagent. The user cannot force the entry of a lot number that VBECS disallowed on the selection list. The user must choose a different reagent.</p> <p> VBECS compares the expiration date of each reagent on the selected rack to the testing date, emits an audible alert, and warns when the reagent is expired.</p> <p><b>Yes</b> requires a comment and VBECS captures details for inclusion in an Exception Report (exception type: expired reagent QC'd).</p> <p><b>No</b> removes the unsatisfactory reagent from the daily QC rack. The user must select a new one. VBECS does not display unsatisfactory lot numbers.</p> <p>The user may enter the lot numbers of the reagents used for the daily QC rack and the lot numbers of the enhancement media reagents defined in Configure Daily QC. VBECS does not display these reagents on the reaction results grid for a rack.</p>

User Action	VBECS
<p>4. Select a worksheet to complete the testing (Figure 62), or</p> <p>Click the <b>Rack not in use today</b> check box to indicate that testing of a rack will not be performed.</p> <p>Click the <b>QC Testing Documentation Offline</b> check box to acknowledge QC is not performed in VBECS but has been properly documented. Click <b>Yes</b> to save changes. Close the window.</p>	<ul style="list-style-type: none"> <li>• Displays the selected worksheet.</li> <li>• Displays the selected partially completed worksheet with the option to cancel the worksheet or continue to enter more testing information.</li> </ul> <p><b>NOTES</b> _____</p> <p>When the QC Testing Documentation Offline check box is checked for a reagent rack, VBECS allows the rack to be used for patient testing until the next daily alert time.</p> <p>The primary lot number can be tested and changed and the results for both will display on the Testing Worklist Report.</p> <p> A user may cancel or invalidate a partially completed QC rack worksheet when it is available. VBECS emits an audible alert and warns that invalidating this worksheet will necessitate starting over and asks whether the user wishes to continue.</p> <p><b>No</b> returns the user to the partially partially-completed worksheet. <b>Yes</b> invalidates the worksheet. VBECS does not delete reaction results from the database.</p> <p>An Exception Report comment documents the invalidation of the rack's worksheet. VBECS does not record workload calculation for invalidated testing. The invalidated reaction results and associated testing information remain in the database for future retrieval (exception type: deletion of partially completed QC).</p> <p> VBECS compares the results entered to those of the previous day for the same rack number and alerts the user when there is a change in reactivity of two or more or when the result is changed from positive to negative for each reagent type by lot number. VBECS emits an audible alert and warns that there is a decrease in reagent reactivity of two or more and asks whether the user wishes to continue to use this vial.</p> <p><b>Yes</b> allows the user to continue and requires a comment. VBECS captures details for inclusion in an Exception Report and allows this reagent to be used with the change (within acceptable limits) (exception type: QC decrease reagent reactivity <math>\geq 2</math>).</p> <p><b>No</b> allows the user to return to the lot number entry for this reagent type and approve this lot number (different vial) or enter a new lot number. VBECS returns to the testing worksheet where the user left off. (An individual vial of the same lot number may replace the one losing its reactivity.)</p> <p>The screening cell and reverse typing reagents are delivered in sets and share the same lot number; therefore, VBECS displays only one reagent type on the lot number template.</p>

User Action	VBECS
<p>5. Verify or select from the drop-down menu the reagent type lot number and test method for each rack chosen for processing.</p> <p>Click <b>OK</b>.</p>	<ul style="list-style-type: none"> <li>• Displays the lot numbers and expiration dates for verification, if any.</li> <li>• Displays each rack in its own tab.</li> </ul> <p><b>NOTES</b> _____</p> <p>VBECS allows for one lot number entry per reagent type per rack template. When an entry was made, VBECS fills in the reagent type lot number, the expiration date of that lot number, and the manufacturer. The user may edit the lot number. The user must verify the lot numbers of racks in the selected batch before VBECS allows the user to move to the reaction results grids.</p> <p>Each reagent rack has a separate reaction results grid to collect the user's direct observations.</p> <p>VBECS displays reagent types (except for the gel and the instrument lot numbers) listed as REQUIRED (by VBECS or by the user) in the daily use field on the rack lot number template and the reaction result entry worksheet for each rack in Enter Daily QC Results.</p> <p>When a rack is tested, VBECS displays a retest option at the daily reagent QC prompt with the untested rack numbers. When a user requests to retest a rack on a given date, VBECS displays the option to reenter a lot number for reagent types for all or part of the daily QC with a comment. All other reagent type results previously entered are locked and may not be edited. VBECS saves the new information and the original data, then generates a report entry.</p> <p>When a user changes a reagent lot number, VBECS clears the testing template for that reagent type and allows the user to enter new results.</p>



User Action	VBECS
<p>6. Enter the observed serologic reactions (Figure 63).</p> <p>Click <b>OK</b> and <b>Yes</b> to save and exit.</p> <p>Repeat for each rack selected in Step 3.</p>	<ul style="list-style-type: none"> <li>• Displays the testing template reaction results grid for each rack selected, when no lot number changes were made.</li> <li>• Determines whether the entered results are acceptable and indicates acceptability of the lot number for the reagent type.</li> <li>• Verifies (by type and lot number) that the reagent is satisfactory or unsatisfactory for use.</li> <li>• Saves the partially or fully completed data.</li> </ul> <p><b>NOTES</b> _____</p> <p>The user may enter results on multiple reagent racks. VBECS displays each rack in its own tab.</p> <p>VBECS verifies satisfactory and unsatisfactory QC testing based on site parameters and user data entry. If the user-entered test results indicate a satisfactory interpretation, VBECS displays "Satisfactory." VBECS saves results that are satisfactory only when the testing for a row is complete and the worksheet is saved. VBECS clears results associated with an unsatisfactory interpretation from the screen/worksheet and the testing for the affected QC rack is considered incomplete until the rack is retested with satisfactory results for the affected row. (Warnings generated by other options when QC testing is incomplete apply).</p> <p>The daily reagent QC for a date requires a valid entry for each reagent displayed on the worksheet. VBECS considers a rack ready and available for patient testing on that date when all reagents on a rack have valid test results.</p>

Figure 61: Enter Daily QC Results

**VBECS - Enter Daily QC Results**

Enter Daily QC Results

**DAILY QC**

Date Tested\* 02/02/2006 11:58

Tested By\* vhaishvbecs1

Select Rack(s)\*

Test Rack(s) | Partially Tested Rack(s) | Retest Rack(s)

	Rack
<input checked="" type="checkbox"/>	A

OK Cancel

\* Required Field

**Figure 62: Record Lot Numbers for Daily QC Testing**

**VBECS - Enter Daily QC Results**

Enter Daily QC Results

Record Lot Numbers for Daily QC Testing    QC Testing Documentation Offline    Rack not in use today

Reagent Type	Lot Number*	Manufacturer	Expiration Date	Test Method*
✓ Anti-A	RA000001	Immucor	07/25/2012 11:15	Tube
✓ Anti-B	RA000003	Immucor	07/25/2012 11:16	Tube
✓ Anti-D	RA000007	Immucor	07/25/2012 11:17	Tube
✓ Screening Cells	RA000015	Immucor	07/25/2012 11:20	Tube
✓ Reverse ABO Typing Cells	RA000007	Immucor	07/25/2012 11:14	Tube
✓ QC kit	RA000099	Immucor	07/25/2012 11:22	Tube
✓ LISS	RA000028	Immucor	07/25/2012 11:21	Tube
✓ AHG, PS	RA000101	Immucor	07/25/2012 11:23	Tube
✓ CC	RA000019	Immucor	07/25/2012 11:20	Tube

Key

- ✓ Valid Reagent
- W Reagent will expire within 24 hours.
- E Expired Reagent

\* Required Field    OK    Cancel

**Figure 63: Enter Daily QC Results with Serologic Reactions**

**VBECS - Enter Daily QC Results**

Daily QC - Enter Results

Reagent Type	Test With	Min Result	Phase	IS*	IAT*	CC*
S Anti-A	A1B Rh, D Neg cell	2	IS	2		
S Anti-B	A1B Rh, D Neg cell	2	IS	2		
S Anti-D	Group O Rh, D Pos cell	2	IS	2		
S Anti-D	RCQ Group O Rh, D Neg cell	0	IS/MAT	0	0	2
S Screening Cells 1	Antibody Reagent/Enhancement Media	2	IAT		2	
S Screening Cells 2	Antibody Reagent/Enhancement Media	2	IAT		2	
S A1 Cell 1	Antibody Reagent	2	IS	2		
S B Cell 2	Antibody Reagent	2	IS	2		

Key

- S Satisfactory
- W Warning
- U Unsatisfactory

Valid Entries    0 = No Agglutination    M = Microscopic    W = Weak    1 = 1+    2 = 2+    3 = 3+    4 = 4+

\* Required Field    Invalidate Test Results    OK    Cancel

## **Reagents**

The options related to reagent inventory control record the receipt of specific reagents by reagent type for each division.

### **Log In Reagents, Update Inventory, View/Print Inventory, Maintain Minimum Levels**

The reagent log is maintained by individual division even in a multidivisional database. Update Inventory allows the user to update reagent inventory by changing the number of vials available. The user may view inventory online using various search criteria and print a copy of the search results.

### **Assumptions**

- The division is “full service.”
- Reagents are in the inventory of the user’s division.

### **Outcome**

- A user may select previously logged-in reagents through Enter Daily QC Results, Patient Testing: Record a Patient Antigen Typing, and Unit Antigen Typing.
- Reagents entered are accessible only within the division in which they were entered.

### **Limitations and Restrictions**

- VBECS does not display a reagent report by invoice number.
- The unsatisfactory portion of a shipment must be recorded individually.
- There is no retrospective data entry or user selection for these options.
- Each site must set a standard for recording quantity: it may use the number of cases or the number of units within the cases.
- When a user enters a lot number in the Lot # field and clicks the **magnifying glass**, VBECS displays only the first instance of a duplicate lot number.
- Free text details can be added only when “Other” is selected as the canned comment.
- When reagents with the same Manufacturer Reagent Name and Lot Number are entered from multiple invoices, only one of the reagents is visible for update.
- Reagents that have had their Number of Vials Available set to zero (0) are no longer visible on the Update Reagent Inventory screen or the Reagent Inventory Report. Print a copy of the Reagent Inventory Report prior to setting the Number of Vials Available to zero (0).

### **Additional Information**

- This option is disabled for transfusion-only facilities.
- VBECS displays alerts, as appropriate, during execution of Enter Daily QC Results to notify the user when reagent inventory levels fall below minimum inventory levels, as defined in Maintain Minimum Levels.
- Local policies and procedures must define the use of patient antisera as reagents.

### **User Roles with Access to This Option**

All users

## Log in Reagents

A user records the receipt of specific reagents by type. The reagent record includes the date the shipment was received, reagent name, lot number, expiration date, manufacturer, and number of vials received in a shipment.

User Action	VBECs
1. Select <b>Reagents</b> from the main menu.	<ul style="list-style-type: none"> <li>Displays options for Quality Control functions.</li> <li>Displays options for processing reagents.</li> </ul>
2. Select <b>Log In Reagents</b> .	<ul style="list-style-type: none"> <li>Displays fields for entering invoice and reagent information.</li> </ul>
3. Invoice Information:  Select or edit the date and time in the Date Received field.  Enter the invoice number in the Invoice Number field.  Enter the name of the manufacturer in the Manufacturer field, or select a name from the drop-down list.	<b>NOTES</b> _____  More than one technologist at a time may process a large shipment of reagents; only one technologist at a time may process individual reagents in the shipment.  The default date received is the current date and time. The user may edit it to a past date and time only.  When the reagent type and lot number were already entered for this invoice, VBECs warns that the user is trying to enter a reagent that was already logged into the system for this invoice. <b>Yes</b> takes the user to the "Update Reagent Inventory" screen where VBECs displays the reagent type and lot number for update. <b>No</b> clears all fields on the screen, allowing the user to log in another reagent.
4. Reagent Information:  Enter the reagent name and reagent type in the Manufacturer Reagent Name and Reagent Type fields, or select from the drop-down lists.  Enter the lot number in the Lot Number field.  Select or edit the date and time in the Lot Expiration Date field.  Enter (or select) a quantity (1–999) in the Vials Received per Lot Number field.	<b>NOTES</b> _____  The default lot expiration date is one minute before midnight (23:59) of the current date.
5. Visual Inspection Information:  Click the <b>Satisfactory</b> radio button, or click the <b>Unsatisfactory</b> radio button and select a comment from the drop-down list in the Comment field.  Click <b>Add</b> to add the reagents to the inventory list.	<b>NOTES</b> _____  Satisfactory reagents are available for later selection, unsatisfactory reagents are not.  When a user selects Other from the drop-down list in the Comment field, entry of additional text in the Details field is required.


User Action	VBECS
6. Repeat Steps 3–5 to log in more supplies, as desired.	<p><b>NOTES</b> _____</p> <p>Click <b>Clear</b> to clear all but the Invoice Number, Date Received, Manufacturer, and Lot Expiration Date fields.</p>
7. Click <b>OK</b> to save.  Click <b>Yes</b> to confirm the save and exit.	<ul style="list-style-type: none"> <li>• Requests confirmation to update the database.</li> </ul>

## Update Reagent Inventory

The user changes the number of available items in the reagent inventory.

User Action	VBECS
1. Select <b>Reagents</b> from the main menu.	<ul style="list-style-type: none"> <li>Displays options for Quality Control functions.</li> <li>Displays options for processing reagents.</li> </ul>
2. Select <b>Update Inventory</b> .	<ul style="list-style-type: none"> <li>Lists existing satisfactory reagents by default.</li> </ul>
3. Select a reagent to edit.  To view reagents with a specific lot number, enter the lot number in the Lot # field and click the <b>magnifying glass</b> .	<p><b>NOTES</b> _____</p> <p>Click a reagent type to reveal manufacturer names. Click a manufacturer name to reveal lot numbers.</p>
4. Click a lot number to enable the Number of Vials Available field.  Enter (or select) a quantity that is less than or equal to the current entry in the Quantity Available field.  Select or edit the date and time in the Date Changed field (Figure 64).	<p><b>NOTES</b> _____</p> <p>The default change date is the current date and time. The user may edit it to a past date and time only.</p> <p>The list of selectable comments in Reason for Change field is not associated with a canned comment category. When a user selects Other from the drop-down list in the Reason for Change field, he may enter additional text in the Comments field. To avoid a system error, do not remove the reason for change before saving.</p>
5. Repeat Steps 3 and 4 to edit more reagents, as desired.	
6. Click <b>OK</b> to save.  Click <b>Yes</b> to confirm the save and view the updated list.	<ul style="list-style-type: none"> <li>Prompts the user to confirm the save.</li> <li>Saves the record and redisplay current inventory, as in Step 3.</li> <li>Displays an option to exit.</li> <li>Click the Clear button to continue if a warning message displays after selecting "No" to the entry confirmation message.</li> </ul>
7. Click <b>Close</b> to exit.	

Figure 64: Update Reagent Inventory

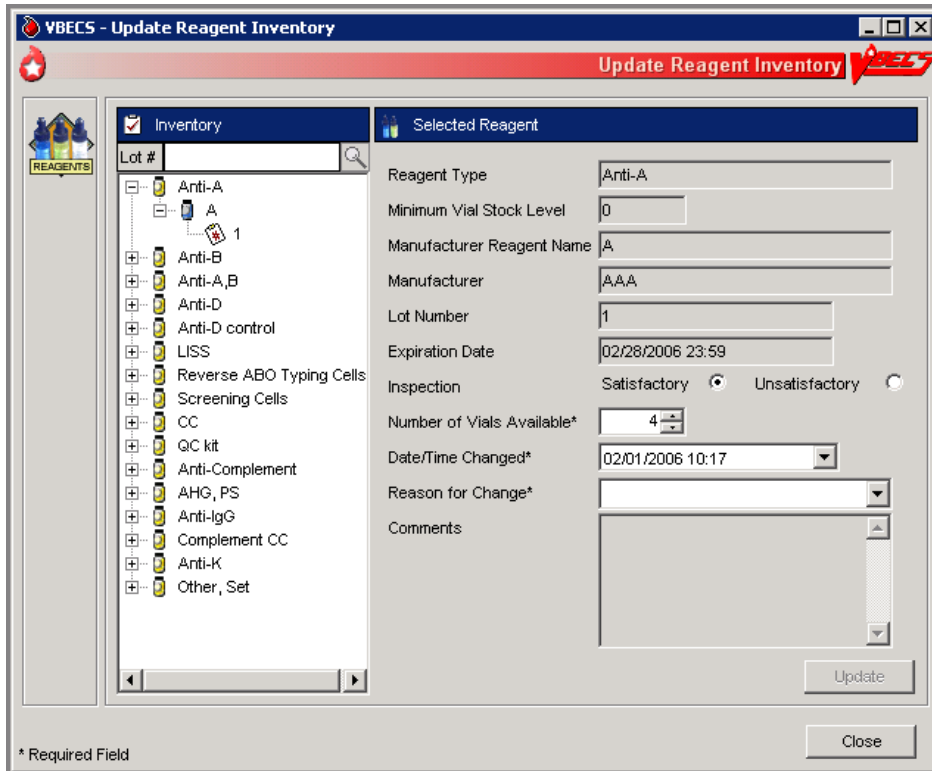




## View/Print Reagent Inventory

The user views inventory online based on various search criteria and prints the search results.

User Action	VBECS
1. Select <b>Reagents</b> from the main menu.	<ul style="list-style-type: none"> <li>• Displays options for Quality Control functions.</li> <li>• Displays options for processing reagents.</li> </ul>
2. Select <b>View/Print Inventory</b> .	<ul style="list-style-type: none"> <li>• Lists reagent types.</li> </ul>
3. Click one or more check boxes in the Reagent Type list to select reagent types.	<ul style="list-style-type: none"> <li>• Displays or allows the user to print a report of all current in-date, unsatisfactory, expired reagent inventory and minimum stock levels as a read-only display based on the selection criteria entered:               <ul style="list-style-type: none"> <li>o Reagent type</li> <li>o In-date (active) reagents</li> <li>o Unsatisfactory reagents</li> <li>o Expired (inactive) reagents</li> <li>o Expiration date range (the user enters a date to begin and end the search)</li> <li>o Date received (the user enters a date to begin and end the search)</li> </ul> </li> </ul> <p><b>NOTES</b> _____</p> <p>VBECS displays only reagents selected based on search criteria.</p> <p>In a multidivisional environment, the report for a division will print one line-item for each division that has set minimum levels for that reagent type.</p> <p>Multiple shipments of the same lot number of a reagent type can cause a discrepancy in the total number available displayed on the report. Always refer to the most recent entry of a reagent for the number of vials available when the shipment was received.</p>
4. Click <b>OK</b> to accept the displayed selection criteria (Figure 65).	<ul style="list-style-type: none"> <li>• Compiles the report and displays an option to print or exit the report.</li> </ul>
5. Click <b>Print</b> to print the list of reagents.	<ul style="list-style-type: none"> <li>• Sends the report to print on the printer selected by the user.</li> </ul>
6. Print another report or click <b>Close</b> to exit.	

Figure 65: Reagent Search

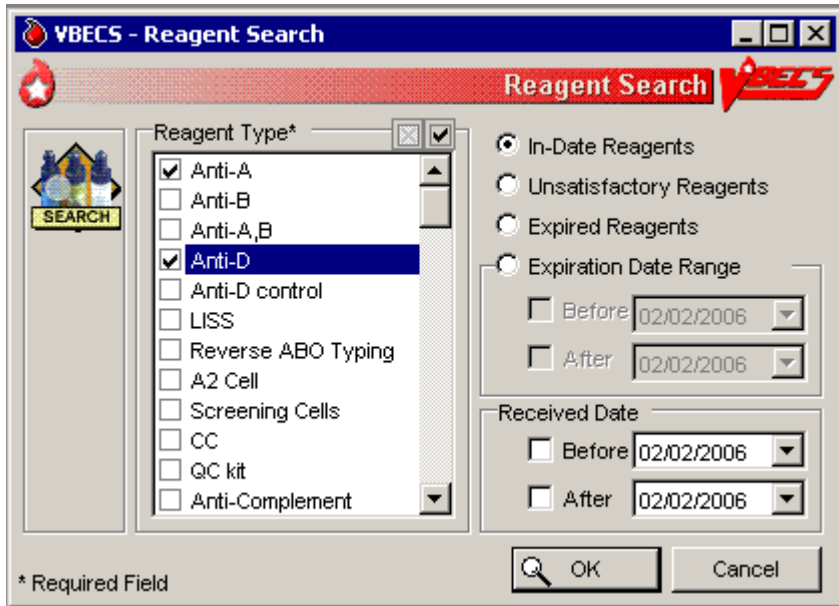

## Maintain Minimum Levels

The user sets minimum stock levels for reagent types used in the division.

User Action	VBECS
1. Select <b>Reagents</b> from the main menu.	<ul style="list-style-type: none"> <li>Displays options for processing reagents.</li> </ul>
2. Select <b>Maintain Minimum Levels</b> .	<ul style="list-style-type: none"> <li>Lists available reagent types and their associated minimum stock levels.</li> </ul>
3. Select a <b>Reagent Type</b> to edit the minimum level threshold.	<ul style="list-style-type: none"> <li>Displays the reagent type selected and allows the user to edit the Minimum Stock Level field or select another reagent.</li> </ul> <p><b>NOTES</b> _____</p> <p>A minimum inventory level (a whole number from 0 to 999) is set for each reagent type. This is related to the reagent type, independent of manufacturer name and lot numbers.</p>
4. Enter (or select) a minimum number of vials in the <b>Minimum Stock Level</b> field.	<ul style="list-style-type: none"> <li>Displays the changes and allows the user to select another reagent to edit.</li> </ul> <p><b>NOTES</b> _____</p> <p>When the number of vials in stock is at or is less than the stock minimum, VBECS notifies the user and instructs him to check the inventory.</p> <p>Inventory levels are tied to the reagent type, not to the manufacturer reagent name or the lot number. A particular type may have multiple manufacturer reagent names and multiple lot numbers associated with that manufacturer reagent name, all of which are summed to create the inventory of that reagent type. VBECS tracks the total number of vials in the system for a particular reagent type.</p> <p>Each reagent type has one minimum inventory level established regardless of manufacturer name and lot numbers. Valid entries are whole numbers from 0 to 999.</p>
5. Review the displayed information and click <b>Update</b> (Figure 66).	<ul style="list-style-type: none"> <li>Displays the changes and allows the user to select another reagent to edit.</li> <li>Prompts the user to confirm the save.</li> </ul>
6. Repeat Steps 3–5 until all minimum reagent levels are set.	
7. Click <b>OK</b> to save, and <b>Yes</b> to confirm the save.	<ul style="list-style-type: none"> <li>Saves the record and exits.</li> </ul>

Figure 66: Maintain Minimum Reagent Levels

The screenshot shows a software window titled "VBECS - Maintain Minimum Reagent Levels". The window contains a table with two columns: "Reagent Type" and "Minimum Stock Level". The table lists various reagent types and their corresponding minimum stock levels. The "Anti-A,B" reagent is highlighted in blue. Below the table, there is a form with a "Reagent Type" field containing "Anti-A,B" and a "Minimum Stock Level" field containing "0". An "Update" button is located to the right of the "Minimum Stock Level" field. At the bottom of the window, there are "OK" and "Cancel" buttons. A small icon with the text "MIN LEVELS" is visible in the top left corner of the window. A note at the bottom left of the window states "\* Required Field".

Reagent Type	Minimum Stock Level
Anti-A	0
Anti-B	2
Anti-A,B	0
Anti-D	0
Anti-D control	0
LISS	0
Reverse ABO Typing Cells	0
A2 Cell	0
Screening Cells	0
CC	0
QC kit	0
Anti-Complement	0
Albumin	0
AHG, PS	0

Reagent Type: Anti-A,B  
Minimum Stock Level: 0

\* Required Field

## Supplies

The user documents the receipt of supplies, and updates and views the supply inventory.

### Assumptions

- A user may maintain supply inventory only in the division logged into.

### Outcome

- A supply inventory that is accessible only within the division of entry was established.

### Limitations and Restrictions

- The division's modification processes may allow or require the user to select previously entered supplies for association with a blood unit's modification. An inventory of containers (transfer bags) and sterile connection device (SCD) wafers is *required* for various modification processes.
- There is no retrospective data entry or user selection for this option.
- The unsatisfactory portion of a shipment must be recorded individually.
- Each site must set a standard for recording quantity: it may use the number of cases or the number of units within the cases.

### Additional Information

- Maintaining Blood Bank Equipment (Equipment: Log In Equipment, Equipment: Maintain Equipment) and Reagents (Log In Reagents, Update Inventory, View/Print Inventory, Maintain Minimum Levels) accommodate the maintenance of reagents and equipment, respectively.
- VBECS saves the lot number, manufacturer name, and expiration date of supplies used in a unit's modification process as part of a unit's modification record when associated during the modification transaction.
- VBECS creates a list of manufacturers as they are entered and makes the list available for selection. Users may not edit entries.

### User Roles with Access to This Option

All users

### Log in Supplies

The user records the receipt of specific supplies by type. The supply record includes the date the shipment was received, supply name, lot number, expiration date, manufacturer, and number of items received in a shipment.

User Action	VBECS
1. Select <b>Tools</b> from the main menu. Select <b>Supplies</b> .	<ul style="list-style-type: none"><li>• Displays options for processing administrative functions.</li><li>• Displays options for processing supplies.</li></ul>
2. Select <b>Log In Supplies</b> .	<ul style="list-style-type: none"><li>• Displays fields for entering invoice and supply information.</li></ul>

User Action	VBECS
<p>3. Invoice Information:</p> <p>Enter the invoice number in the Invoice Number field.</p> <p>Select or edit the date and time in the Date Received field.</p> <p>Enter the name of the manufacturer in the Manufacturer field, or select a name from the drop-down list.</p>	<p><b>NOTES</b> _____</p> <p>More than one technologist at a time may process a large shipment of supplies; only one technologist at a time may process individual supplies in the shipment.</p> <p>The default date received is the current date and time. The user may edit it to a past date and time only.</p> <p>When the supply type and lot number were already entered for this invoice, VBECS warns that the user is trying to enter a supply that was already logged into the system for this invoice. <b>Yes</b> takes the user to the "Update Supply Inventory" screen where VBECS displays the supply type and lot number for update. <b>No</b> clears all fields on the screen, allowing the user to log in another supply.</p>
<p>4. Supply Information:</p> <p>Enter the name of the manufacturer and supply type in the Manufacturer and Supply Type fields, or select from the drop-down lists.</p> <p>Enter the lot number in the Lot Number field.</p> <p>Select or edit the date and time in the Lot Expiration Date field.</p> <p>Enter (or select) a quantity (1–999) in the Quantity Received field.</p>	<p><b>NOTES</b> _____</p> <p>The default lot expiration date is one minute before midnight (23:59) of the current date.</p>
<p>5. Visual Inspection Information:</p> <p>Click the <b>Satisfactory</b> radio button, or click the <b>Unsatisfactory</b> radio button and select a comment from the drop-down list in the Comment field.</p> <p>Click <b>Add</b> to add the supplies to the inventory list (Figure 67).</p>	<p><b>NOTES</b> _____</p> <p>Satisfactory inventory is available for later selection, unsatisfactory inventory is not.</p> <p>When a user selects Other from the drop-down list in the Comment field, entry of additional text in the Details field is required.</p>
<p>6. Repeat Steps 3–5 to log in more supplies, as desired.</p>	<p><b>NOTES</b> _____</p> <p>Click <b>Clear</b> to clear all but the Invoice Number, Date Received, Manufacturer, and Lot Expiration Date fields.</p>
<p>7. Click <b>OK</b> to save.</p> <p>Click <b>Yes</b> to confirm the save and exit.</p>	<ul style="list-style-type: none"> <li>• Requests confirmation to update the database.</li> </ul>

**Figure 67: Log In Supplies**

**VBECs - Log In Supplies**

Log In Supplies

Invoice

Received Date\* 02/01/2006 10:53

Invoice Number\* 10241

Manufacturer\* Super Supplier

Supply

Manufacturer Supply Name\*

Supply Type\*

Lot Number\*

Lot Expiration Date\* 02/01/2006 23:59

Quantity Received\* 1

Visual Inspection\*

Satisfactory

Unsatisfactory

Comment\*

Details

Add Clear

	Supply Name	Lot Number	Expires	Qty	Invoice No.	Manufacturer	Supply Type	Inspection
<input checked="" type="checkbox"/>	Gorgeous Gloves	042205	02/01/2007 23:59	200	10241	Super Supplier	Gloves	Satisfactory
<input checked="" type="checkbox"/>	Perfect Paper	052154	02/01/2010 23:59	50	10241	Super Supplier	Printer Paper (generic)	Unsatisfactory

\* Required Field

OK Cancel


### Update Supply Inventory

The user changes the number of available items in the supply inventory.

User Action	VBECs
1. Select <b>Tools</b> from the main menu.  Select <b>Supplies</b> .	<ul style="list-style-type: none"> <li>Displays options for processing administrative functions.</li> <li>Displays options for processing supplies.</li> </ul>
2. Select <b>Update Supply Inventory</b> .  Click the <b>Include Expired Supplies</b> check box to include expired supplies in the list.	<ul style="list-style-type: none"> <li>Lists existing satisfactory supplies by default.</li> </ul>
3. Select a supply to edit.  To view supplies with a specific lot number, enter the lot number in the Lot # field and click the <b>magnifying glass</b> .	<p><b>NOTES</b> _____</p> <p>Click a supply type to reveal manufacturer supply names. Click a manufacturer supply name to reveal lot numbers.</p>

User Action	VBECs
<p>4. Click a lot number to enable the Quantity Available field.</p> <p>Enter (or select) a quantity that is less than or equal to the current entry in the Quantity Available field.</p> <p>Select or edit the date and time in the Date Changed field (Figure 68).</p>	<p><b>NOTES</b> _____</p> <p>The default change date is the current date and time. The user may edit it to a past date and time only.</p> <p>When a user selects Other from the drop-down list in the Reason for Change field, he may enter additional text in the Comments field.</p>
<p>5. Repeat Steps 3 and 4 to edit more supplies, as desired.</p>	
<p>6. Click <b>OK</b> to save.</p> <p>Click <b>Yes</b> to confirm the save and view the updated list.</p>	
<p>7. Click <b>Print Supply Inventory</b> to print the list of supplies.</p>	<ul style="list-style-type: none"> <li>Displays the Supply Inventory window (see View/Print Supply Inventory).</li> </ul>
<p>8. Click <b>Cancel</b> to exit.</p>	

**Figure 68: Update Supplies**

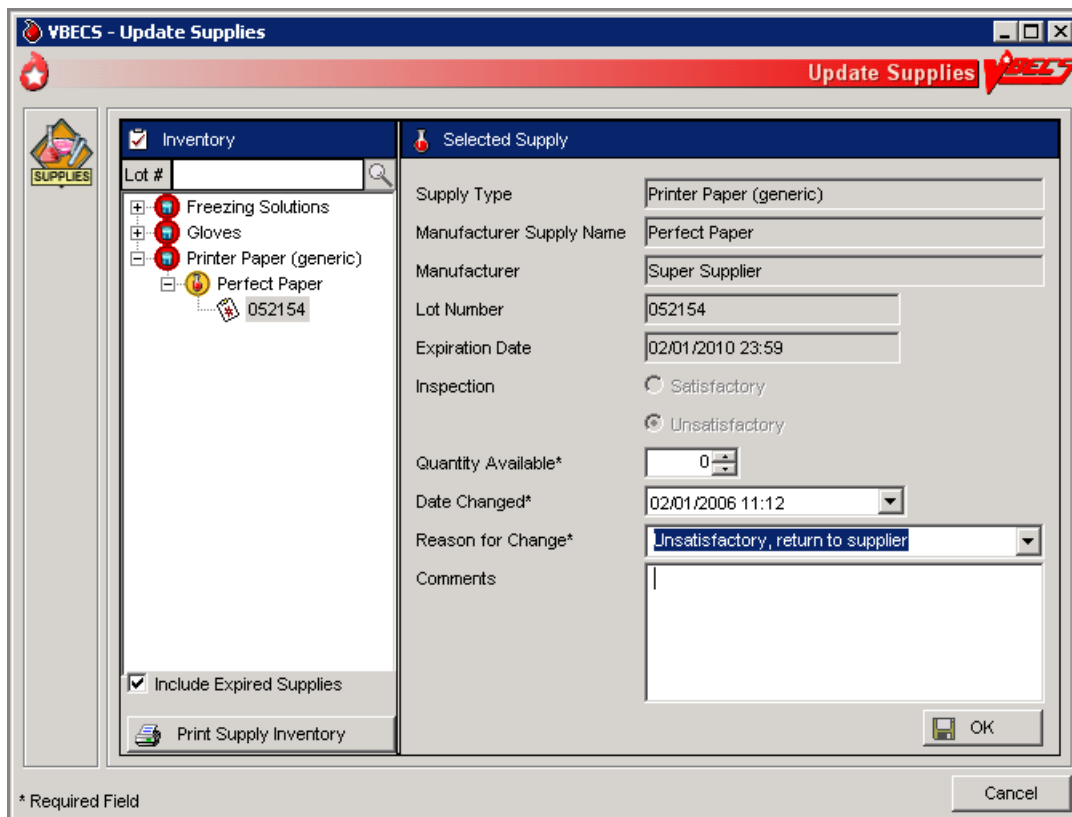



## View/Print Supply Inventory

The user views inventory online based on various search criteria and prints the search results.

User Action	VBECs
1. Select <b>Tools</b> from the main menu.  Select <b>Supplies</b> .	<ul style="list-style-type: none"> <li>• Displays options for processing administrative functions.</li> <li>• Displays options for processing supplies.</li> </ul>
2. Select <b>View/Print Supply Inventory</b> .	<ul style="list-style-type: none"> <li>• Lists supply types.</li> </ul>
3. Click one or more check boxes in the Supply Type list to select supply types.	<ul style="list-style-type: none"> <li>• Displays or allows the user to print a report of all current in-date, expired, and unsatisfactory supply inventory and minimum stock levels as a read-only display based on the selection criteria entered:               <ul style="list-style-type: none"> <li>o Date range (the user enters a date to begin and end the search)</li> <li>o Supply type</li> <li>o In-date (active) Supplies</li> <li>o Expired (inactive) Supplies</li> <li>o Expiration date of the supply</li> <li>o Date received</li> </ul> </li> </ul> <p><b>NOTES</b> _____</p> <p>A user may view the supply inventory by one, some, or all of the selection criteria.</p>
4. Click <b>OK</b> to accept the displayed selection criteria.	<ul style="list-style-type: none"> <li>• Compiles the report and displays an option to print or exit the report.</li> </ul>
5. Click <b>Print Supply Inventory</b> to print the list of supplies.	<ul style="list-style-type: none"> <li>• Sends the report to print on the printer selected by the user.</li> </ul> <p><b>NOTES</b> _____</p> <p>VBECs displays and prints the report in the supply type order : in-date by lot number in reverse chronological order of expiration date, followed by expired and unsatisfactory supplies. Details included in the report:</p> <ul style="list-style-type: none"> <li>• Supply Type (Unit of Measure)</li> <li>• Manufacturer</li> <li>• Manufacturer's Supply Name</li> <li>• Lot Number</li> <li>• Expiration Date</li> <li>• Number Available</li> <li>• Inspection upon receipt</li> <li>• Invoice Number</li> <li>• Received Date/Time</li> <li>• Received by Tech Name</li> <li>• Comments</li> </ul>
6. Print another report or click <b>Cancel</b> to exit.	

## Equipment

The user registers blood bank instruments and other equipment (refrigerators, freezers, thawing bath, incubators, centrifuges, etc.) and records associated maintenance activities.

### Assumptions

- A user may maintain equipment inventory only in the division currently logged into.

### Outcome

- An equipment maintenance record, accessible within the division of entry, was established.

### Limitations and Restrictions

- VBECS does not check for duplicate entries of equipment names and identifiers.
- There is no retrospective data entry or user selection for this option.

### Additional Information

- VBECS does not calculate inventory levels for supplies or equipment.

### User Roles with Access to This Option

All users

### Equipment: Log In Equipment

A user adds new equipment or maintains existing equipment. After equipment is registered, the user must record periodic maintenance. The equipment records are updated and retained according to local procedure and policy. Once an equipment record is established, it may be selected as equipment used during a blood product modification.

User Action	VBECS
1. To add new equipment, select <b>Tools</b> from the main menu.  Select <b>Equipment</b> .	<ul style="list-style-type: none"><li>• Displays options to maintain equipment maintenance.</li></ul>
2. Select <b>Log In Equipment</b> to create an equipment record or <b>Maintain Equipment</b> to record maintenance (see Equipment: Maintain Equipment).	<ul style="list-style-type: none"><li>• Displays an empty equipment property template.</li><li>• Displays all in-service equipment items for the blood bank associated with the user's division and offers the option to display out-of-service equipment.</li></ul>

User Action	VBECS
<p>3. Equipment Info area: Enter the equipment information in the appropriate fields. Enter or select an equipment type from the drop-down menu.</p> <p>Installation Info area: Select or edit the date and time in the Date Received/Installed field.</p> <p>Click the <b>Expiration/Renewal Date</b> check box when there's an expiration or renewal date, and edit the date as above.</p> <p>Manufacturer Info area: Enter the manufacturer name or select a manufacturer from the drop-down list.</p> <p>Enter the manufacturer's contact information in the appropriate fields or click the <b>find</b> button to select the manufacturer's address.</p> <p>Click <b>Add</b> to add the equipment to inventory (Figure 69).</p>	<ul style="list-style-type: none"> <li>• Displays fields for entering equipment name, ID, serial number, and type in the Equipment Info area.</li> <li>• Displays fields for entering the date the equipment was received or installed, warranty or service contract number, and expiration or renewal service contract date in the Installation Info area.</li> <li>• Displays fields for entering the manufacturer name, address, and phone in the Manufacturer Info area.</li> </ul>
<p>4. Repeat Step 3 to add new items to the equipment list.</p> <p>Click <b>OK</b> to save.</p>	<ul style="list-style-type: none"> <li>• VBECS maintains a historical record for each piece of equipment entered.</li> </ul>
<p>5. Click <b>Yes</b> to confirm and exit.</p>	

Figure 69: Log In Equipment

**VBECS - Log In Equipment**

**Log In Equipment**

**Equipment Info**

Equipment Name\*

Local Equipment ID

Serial Number\*

Equipment Type\*

**Installation Info**

Date Received/Installed\*

Warranty/Service Contract No.

Expiration/Renewal Date

**Manufacturer Info**

Manufacturer Name\*

Address Line 1

Address Line 2

City

State

Zip

Phone

**Equipment Logged In**

Equipment Name	Serial No.	Type	Manufacturer	Received	Contract No.

\* Required Field

OK Cancel

## Equipment: Maintain Equipment

The user maintains an online equipment record to record periodic maintenance. Equipment records are updated and retained according to local procedure and policy.

User Action	VBECS
<p>1. To maintain equipment, select <b>Tools</b> from the main menu.</p> <p>Select <b>Equipment</b>.</p>	<ul style="list-style-type: none"> <li>Displays options to maintain equipment maintenance.</li> </ul>
<p>2. Select <b>Maintain Equipment</b> to record maintenance or <b>Log In Equipment</b> to create an equipment record (see Equipment: Log In Equipment).</p>	<ul style="list-style-type: none"> <li>Displays entered in-service equipment items for the blood bank associated with the user's division and offers the option to display out-of-service equipment.</li> </ul>
<p>3. In the Equipment tab, enter the equipment serial number (or enter part of the serial number and click the <b>magnifying glass</b> to view serial numbers), or click <input type="checkbox"/> in front of each equipment type and name to view equipment items.</p> <p>Select the serial number of the item whose record will be maintained.</p> <p>Click the <b>Include Out of Service Equipment</b> check box, if desired.</p>	<ul style="list-style-type: none"> <li>Displays fields for entering the type of maintenance event performed, the date the maintenance was performed, the results of the event, the next scheduled event, the next scheduled maintenance date, and pending maintenance issues in the Maintenance tab, and allows the user to select a maintenance event type and event details.</li> <li>Displays entered information.</li> </ul>
<p>4. Review and edit the equipment data in the Equipment tab.</p>	<ul style="list-style-type: none"> <li>Displays entered information.</li> </ul>
<p>5. Select the <b>Maintenance tab</b>.</p> <p>To add a maintenance event, enter the maintenance type or select it from the drop-down list.</p> <p>Select or edit the date and time in the Maintenance Date field.</p> <p>Enter details in the Maintenance Results and Pending Issues fields.</p> <p>Click <b>Add</b> to add the maintenance event to the Maintenance Events list (Figure 70).</p> <p>Click <b>Print</b> to print the record.</p>	<ul style="list-style-type: none"> <li>Displays recorded maintenance information and allows the user to update information.</li> <li>Displays an option to print an Equipment Report.</li> </ul>
<p>6. Click the <b>Out of Service Equipment</b> check box to remove equipment.</p>	<ul style="list-style-type: none"> <li>Removes the equipment from the list of currently maintained equipment.</li> </ul>
<p>7. Repeat Steps 3–6, as desired.</p>	

User Action	VBECS
<p>8. Click <b>OK</b> and <b>Yes</b> to save and continue, or <b>No</b> to return to the screen and continue.</p> <p>Click <b>Cancel</b> to exit without saving.</p>	<ul style="list-style-type: none"> <li>• Prompts the user to confirm the save.</li> <li>• Saves the updates to the equipment record.</li> </ul>

**Figure 70: Maintain Equipment**

**VBECS - Maintain Equipment**

**Inventory**

Serial #

- Cell Washer
  - Deluxe Cell Washer
    - CW-09-24-5019

Include Out of Service Equipment

**Selected Equipment**

Equipment Maintenance

New Maintenance Event

Maintenance Type\* [Dropdown]

Maintenance Date\* 02/01/2006 12:28 [Dropdown]

Maintenance Results\* [Text Area]

Pending Issues [Text Area]

[Add] [Clear]

Date	Type	Results	Pending Issues
02/01/2006 12:26	Filter replacement	Faster processing	None

Next Scheduled Maintenance Event [Text Area]

Out of Service Equipment

[Print] [OK] [Cancel]

\* Required Field

# Component Processing

## Shipments

### Incoming Shipment

A shipment handled in this option contains one or more containers of human blood products.

The user may use a barcode scanner to read information from a unit blood bag (recommended) or enter this information. The user reviews and confirms the information before adding a unit to the database.

### Assumptions

- The division is configured.
- The shipper is active.
- Units are inspected and deemed satisfactory for receipt before they are entered in VBECS.

### Outcome

- A shipment or partial shipment of blood units was processed and added to the blood bank inventory.
- The initial status for each unit was set in accordance with component class.

### Limitations and Restrictions

- An ISBT 128 Product Code's first characters may be E or X.
- VBECS does not check whether blood products with the same unit ID and different product codes are the same blood type.
- VBECS does not compare the ABO/Rh of an autologous unit or display a message when the ABO/Rh is incompatible with the restricted for patient.

*When a product type, code, or local code is not in VBECS or a blood supplier creates a product code, contact the Service Desk. See Customer Support for contact information.*

- The user may not save the unit record until the donation type and its requirements are satisfied.
- No longer supports the entry of blood products labeled in Codabar label type to new or existing invoices.
- The unit expiration [Maximum Storage Time (MST)] is set to the longest possible storage time based on product type, core condition, attributes, and available references as a guide. The MST is set at a higher level than an individual collection product code. VBECS does not know the specific collection details. It is the responsibility of each user to accurately enter and process any one blood unit based on its expiration date/time as labeled by the collection facility.
- When blood products intended for quarantine are scanned, VBECS does not automatically quarantine them: the user must do so.
- VBECS does not recognize scanned labels for para-Bombay or Bombay ISBT units.
- VBECS does not allow duplicate IDs for the same product code.
- When a user scans a product code or expiration date barcode in the Unit ID field, VBECS does not display the entered information.

- When a user scans an ISBT blood product code ID with “0” as the donation type (sixth character), VBECS indicates that the donation type is not specified. When the user selects a donation type from the drop-down menu, VBECS replaces “0” with a recognizable character in the product code text box and stores it as such. When the user rescans the product ID, he must manually replace “0” or search using the unit ID alone, or the unit ID and the first five characters of the product ID. When scanning a blood product derivative, this does not apply.
- Blood product code entries in the blood product table include those from ICCBBA for ISBT 128. Product codes created after the release of VBECS cannot be entered until the updated tables are incorporated through patches to VBECS. Refuse acceptance of the unit, if possible. Users may not edit the blood product table.
- ISBT 128 tables include blood product codes labeled as not for manufacturing or transfusion. VBECS does not restrict such products from being entered, issued, or transferred (local policy controls their use), but does not accommodate blood modification for these products.
- No entry of an ICCBBA Product Description Code starting with A, B, C, or D aka National or Local Codes A0000-D9999. *If use of such a code is necessary for patient care, contact Service Desk and file a request for the code to be added to VBECS.*

### **Additional Information**

- A unique blood unit record is defined by the following data elements:
  - VBECS Division
  - Unit ID
  - Product code
    - ISBT 128 (8 alphanumeric characters).
- VBECS stores scanned ISBT 128 product codes and displays them as eight characters:
  - Characters 1–5: product type
  - Character 6: donation type
  - Characters 7 and 8: division code (split)
- When ISBT 128 units received during downtime must be issued, it is recommended that the user record the check character. When a blood unit is to be logged in after downtime and the unit is unavailable for scanning, the user must enter the ISBT unit ID and the case-sensitive check character (the character in the square following the unit ID).
- To calculate a check character, see the Quick K Calculator on the ICCBBA Web page under ISBT 128, Other Tools. Follow the instructions to download the application.
- Each division may have only one blood unit record of the same product type, product code, and unit ID. A unit shipped out (transferred) and subsequently shipped to (received by) a division in the same database will maintain individual records of the unit ID and product code for each division.
- Several data elements, including the unit ID and product code, define a blood unit record as unique. The unit record also includes expiration date and time, blood type, and donation type. These data elements are not editable.
- Some suppliers relabel blood products and do not affix their own FDA Registration Number to those units. It is recommended that the user obtain FDA Registration Number barcodes from their suppliers and use them for scanning the shipper.

### **User Roles with Access to This Option**

All users



## Incoming Shipment

In a “transfusion-only” facility, units must be restricted for use by a specific patient while an incoming shipment is processed. VBECS identifies the original collection facility to determine the presence of an eye-readable prefix. A user may enter unique unit information regarding special testing and antigen phenotyping. The user confirms that all entered units from the shipment were added to the database in the blood bank.

User Action	VBECS
1. Select <b>Shipments</b> from the main menu.  Select <b>Incoming Shipment</b> .	<ul style="list-style-type: none"><li>• Displays options for processing blood product shipments.</li><li>• Displays invoice information fields for entering shipment information.</li></ul> <p><b>NOTES</b> _____</p> <p>The system is set as a default to ISBT 128 label type.</p>

2. Scan the invoice number barcode or enter the invoice number.

To search for an existing invoice, click the **find** button. Enter an invoice number and/or select a shipper from the drop-down list, then click **Search**. Click an invoice number to select it and click **OK**.

- When an identified invoice does not exist, asks the user whether he wants to create an invoice and add units to it.
- Displays the option to search for existing invoices by invoice number and/or by active shipper. When no match is found, notifies the user.

#### NOTES

---

When a user chooses to select an existing invoice to add units, VBECS ensures that the shipper is marked as active. VBECS displays and allows the user to edit existing invoices associated with active shippers.

To correct an invoice associated with an inactive shipper, the user must reactivate the shipper.

Do not select the blank row in the shipper list box. If the blank row is selected, click it, close the window and start a new search.

Multiple users may process different units from a shipment. A large shipment of blood can be physically processed by more than one tech simultaneously, however, each individual unit in the shipment can only be processed by one tech.

When more than one user intends to work on an invoice, one user must create the invoice. The other users must select that invoice and add units to it.

When using a scanner to read barcodes on blood product bags, the system must display the input to the user in the eye-readable form that appears on the blood bag.

When processing a shipment, the system should provide an indication to the user which specific barcode the system is expecting. Once the scan is complete, the system automatically 'moves' the indication to the next logical input. The user can override this progression by manually selecting a different input field. It is assumed that the scanners will be configured to append the proper command at the completion of each scan.

If the user entered shipment properties for an existing invoice, VBECS displays a message: Do you want to continue and add units to an existing invoice? (Select "No" to use another invoice.) If the user responds "No," the shipment property fields are cleared, allowing the user to enter data for another invoice; otherwise, VBECS continues to the next step.


If the user entered shipment properties for a new invoice, VBECS displays a message: Do you want to continue and add units to the new invoice? (Select "No" to use another invoice.) If the user responds "No," the shipment property fields are cleared, allowing the user to enter data for another invoice; otherwise, the system continues to the next step.

The combination of invoice number, shipper, and date/time received uniquely identifies the shipment for data retrieval.

User Action	VBECS
<p>3. Select a source from the Source (Shipper) drop-down list. If needed, edit the date received in the Date Received field.</p>	<p><b>NOTES</b> _____</p> <p>After a user enters or selects an invoice number, the user may enter information in the shipper, date received, and processing technologist fields.</p> <p>During retrospective entry, the user must enter the shipment date and time before entering any other unit transaction.</p> <p>Shipment received date/time cannot be a future date/time.</p>
<p>4. Click <b>OK</b> again to continue.</p>	<ul style="list-style-type: none"> <li>• When the barcode is scanned, determines the labeling type (autodiscriminates).</li> </ul> <p><b>NOTES</b> _____</p> <p>Entry of Codabar labeled blood units is disabled.</p> <p>A scanned ISBT 128 Unit ID barcode is identified by the presence of the '=' as the hidden identifier. Once the hidden identifier is removed, the Unit ID is interpreted as the first 13 characters, the first character being a letter, most likely 'w'. The Unit record is flagged as scanned during initial receipt.</p>
<p>5. Click <b>Yes</b> to add units to the invoice or click <b>No</b> to select another invoice.</p>	<ul style="list-style-type: none"> <li>• When a user selects an existing invoice, displays the shipment information and blood unit details associated with the invoice.</li> </ul>
<p>6. Select a unit to add. For an ISBT 128-labeled unit, scan the label, or enter the unit ID and check character (in that order).</p>	<ul style="list-style-type: none"> <li>• Disables the FDA Registration Number field.</li> <li>• Displays information corresponding to the unit ID barcode scanned. When a user enters an ISBT 128-labeled unit ID, VBECS prompts the user to enter the check character from the blood bag. <ul style="list-style-type: none"> <li>o When a user scans ISBT 128 labels units and the cursor is in an inappropriate field, VBECS warns that the incorrect barcode was scanned.</li> </ul> </li> </ul> <p><b>NOTES</b> _____</p> <p>For ISBT 128-labeled units: When the user-entered check character from the label does not match the VBECS-generated check character, VBECS warns the user.</p> <p>When VBECS does not find a matching entry for the ICCBBA Registration Number on the local supplier table, the user must activate or add a local supplier before continuing.</p>
<p>7. Scan the product code barcode or enter the product code.</p>	<ul style="list-style-type: none"> <li>• When a user scans a product code barcode that is not activated in the user's division, prompts the user to activate the code. (See Blood Products.)</li> <li>• Displays the corresponding product code, product name, and (for ISBT) donation type (see Step 10).</li> <li>• When a unit ID of the same product type and code exists in a division, the expiration date is in the future, and there is no record of inactivation or transfer of that unit ID and product code in the division's record: <ul style="list-style-type: none"> <li>o Warns that the blood unit already exists in the division.</li> </ul> </li> </ul>

User Action	VBECS
	<ul style="list-style-type: none"> <li>o Clears the data entry fields. The user may enter a different unit ID and product code.</li> <li>• When a unit ID of the same product type and code exists in a division and the unit in the division has an expiration date in the past, warns that the blood unit already exists in the division.</li> <li>• When a unit ID of the same product type and code exists in a division, the unit status is “Transferred,” and the ABO/Rh and unit expiration date are identical to the existing unit of record: <ul style="list-style-type: none"> <li>o Makes the existing unit record available.</li> <li>o Warns that the user is attempting to reenter a unit in inventory. <b>No</b> clears the unit. <b>Yes</b> and saves the unit.</li> <li>o Updates the unit status to “Limited” or “Available.”</li> <li>o Updates the unit record to reflect the reentry of the unit in the division’s inventory. The user may add new information about this unit through Edit Unit Information.</li> </ul> </li> <li>• System determines barcode type(s) scanned based on hidden identifiers. System interprets and displays product name and donation type, as appropriate.</li> <li>• System determines if the facility is full service or transfusion only.</li> <li>• System assigns the donation type to the unit or requires user input, as appropriate.</li> </ul> <p><b>NOTES</b> _____</p> <p>Scanned Product Codes</p> <p>When a unit is transferred between facilities in a consolidated database or is reentered after transfer to an unrelated facility, VBECS records all unit activity.</p> <p><b>OK</b> returns the user to a blank screen to enter a different product. The user may enter a different product code.</p> <p>VBECS requires the full 8-digit ISBT 128 product code. The user should enter the 8-digit ISBT 128 product code as it appears on the blood unit label of the actual blood unit being entered into VBECS. When a user does not enter the sixth through eighth characters, VBECS automatically completes the required information with defaults, V and 00, respectively. This may be changed by clearing the unit information and entering the complete 8-digit ISBT 128-unit product code.</p> <p>VBECS assigns a volume to the unit based on the average volume field from the Blood Product Table entry corresponding to the unit’s blood product code of the unit.</p> <p>For each product type in a shipment, site parameters must indicate that the product is marked as active for the division and that a cost is associated with the shipper. If not, VBECS prompts the user to mark the component as active. Blood product data entry cannot continue until the product is marked as active and a base unit cost is associated with the shipper.</p> <p>When the Product Code’s first characters are E or X, the 7th character is alphanumeric {A–Z; 0–9}, and that the 8th character is alphanumeric {a–z; 0–9}.</p>

User Action	VBECS
	<p>For ISBT 128-labeled units:  VBECS stores scanned ISBT 128 product codes and displays them as eight characters:</p> <ul style="list-style-type: none"> <li>• Characters 1–5 are used to search the ISBT Blood Product Table to identify the product type.</li> <li>• Character 6 indicates donation type.</li> <li>• Character 7 indicates a division at the time of collection.</li> <li>• Character 8 indicates an additional aliquot.</li> </ul> <p>When the user does not enter a donation type and unit division codes when manually entering a unit, the donation type is defaulted to "V" and defaults to "00" for the division characters. These are not editable after the unit is saved to the invoice.</p> <p>Allow the entry of individual products associated with the active product categories.</p> <p>When a blood product code in the RED BLOOD CELLS or WHOLE BLOOD component class is entered, VBECS enables data entry of sickle cell negative and antigen typing.</p> <p>The cytomegalovirus (CMV) negative data entry option is always enabled.</p> <p>The system presents an indicator for RBC contamination of equal to or greater than (<math>\geq</math>) 2 mL. RBC, WB and plasma products are set and not editable. PLATELET and OTHER products require evaluation and entry by the user.</p> <p>The RBC contamination indicator setting as saved is recorded as part of the unit record. The indicator displays with the Incoming Shipment entry on the Unit History Report.</p> <p>When the site parameter indicates that the site is a 'Transfusion Only' facility when units are added to the database, they are given a status of Available regardless of the required need for ABO/Rh confirmation testing as it is not required in this configuration.</p> <p>When processing units labeled using ISBT 128, all eight (8) characters are required when either:</p> <ul style="list-style-type: none"> <li>• The site parameter indicates the site is a 'transfusion only' facility</li> <li>• The donation type requires restriction. The blood unit processed must be restricted with a specific patient.</li> </ul> <p>An ISBT 128 ABO/Rh barcode is identified by the presence of "=%" as the hidden identifier. Once the hidden identifier is removed, the ABO/Rh is a four-character code.</p>

User Action	VBECS
<p>8. For ISBT 128-labeled units, scan the ABO/Rh number barcode, enter the ABO/Rh, or select the ABO/Rh from the drop-down list.</p>	<ul style="list-style-type: none"> <li>• Displays the corresponding ABO/Rh.</li> <li>• Allows the selection of only ABO/Rh types acceptable for ICCBBA component classes.</li> <li>• Enables the ABO/Rh field when a valid product code is entered.</li> </ul>
<p>9. Scan the expiration date barcode or enter the expiration date and time when an expiration time is required.</p>	<p>• Displays the corresponding expiration date and time.</p> <p><b>NOTES</b></p> <hr/> <p>A scanned ISBT 128 expiration date barcode is identified by the presence of '=' &gt;' or '&amp;&gt;' as the hidden identifier. Once the hidden identifier is removed, there are two possible formats of the barcode, one involving the date only and another involving date and time. If no time in the barcode, the default expiration time is 23:59.</p> <p>VBECS compares the entered blood product's expiration date to the maximum storage date and time limit for the blood product. The date and time of shipment receipt are used as the collection date and time. When the expiration date entered exceeds the defined maximum storage time, VBECS displays an error message and clears the field. The user may reenter an expiration date and time for the product.</p> <p>Products with a maximum expiration date and time of <math>\leq 24</math> hours default to an expiration time <math>\leq 24</math> hours from the time of the incoming shipment. Blood products with a maximum expiration date and time of <math>\geq 24</math> hours have a default expiration time of 23:59.</p> <p>Expiration date entries can be corrected before saving, as the date is presented for review before final acceptance. To correct a saved, invalid Expiration Date, inactivate and reenter the unit.</p> <p> When a unit expiration date precedes the incoming shipment date and time, VBECS emits an audible alert, warns that the user is attempting to enter an expired unit, and asks whether the user wishes to continue.</p> <p><b>No</b> clears the unit expiration date field. The user must enter a new, acceptable expiration date. <b>Yes</b> requires the user to enter a comment.</p> <p>VBECS captures details of this override in an Exception Report.</p>

User Action	VBECS
<p>10. For ISBT 128-labeled units, accept the donation type based on character 6 of the product code (cannot be edited) or, when character 6 is "0," indicate a donation type.</p>	<ul style="list-style-type: none"> <li>• Assigns the donation type to the unit or requires user input, as needed.</li> <li>• Requires data entry of the "reserved for" patient name and/or patient ID and the disease market testing status for "autologous" and "directed" donation types.</li> </ul> <p><b>NOTES</b> _____</p> <p>When a site is a transfusion-only facility, each unit processed must be restricted to a specific patient regardless of donation type.</p>
<p>11. Click the <b>find</b> button next to the Restricted For Patient field to indicate the patient for whom the unit is reserved, as needed.</p>	<ul style="list-style-type: none"> <li>• Allows the user to search for active VistA patients to associate with the unit.</li> <li>• Requires data entry of a patient name and/or patient ID to associate the unit with that patient when a unit has a "directed" or "autologous" donation type or the receiving facility is a "transfusion-only" facility.</li> <li>• When no match exists, warns the user.</li> <li>• When the unit is autologous, requires a user to indicate the disease marker testing status: <ul style="list-style-type: none"> <li>o NEG: unit fully tested and negative for all disease markers</li> <li>o POS: unit tested and positive for one or more disease markers (biohazard)</li> <li>o NFT: unit not fully tested for one or more disease markers (biohazard)</li> </ul> </li> </ul> <p><b>NOTES</b> _____</p> <p>Displays a message when the user selects a patient with a blank first or last name, or the name is too long. The user is instructed to contact the VBECS Administrator for assistance.</p> <p>The Patient Blood Availability Report does not include restricted units not assigned to a patient. To track such units, the user may enter restricted units' IDs in the Special Instruction field in Special Instructions &amp; Transfusion Requirements: Enter and Remove Special Instructions. When a user restricts a unit to a patient and VBECS does not find that patient in its database, VBECS searches VistA. When it finds a valid patient, VBECS adds the patient to the database.</p>

User Action	VBECS
<p>12. Do one or more:</p> <p>Accept the blank check boxes or click a check box to indicate that a unit is CMV Negative or Sickle Cell Negative.</p> <p>Enter an additional cost.</p> <p>Enter an antigen phenotype for a unit (Figure 71).</p>	<ul style="list-style-type: none"> <li>Displays an option to save or clear unit information.</li> </ul> <p><b>NOTES</b> _____</p> <p>VBECS accommodates data entry of:</p> <ul style="list-style-type: none"> <li>A single special typing cost (the sum of charges from \$0.00 to \$999.00, other than the base unit cost) to associate with a unit.</li> </ul> <p>An antigen type may be positive or negative for any antigen listed as a VBECS corresponding antigen in Appendix B: Table 14: Antibody and Antigen Table: Irregular Antibodies. An antigen may not be positive and negative for the same unit.</p> <p>The shipper of the shipment is used to determine the actual costs of the blood products contained in the shipment. The shipper is the “ship from/ship to” shipper and may or may not be the same as the shipper that collected and/or processed the blood product as indicated on the blood component label. A user can select a shipper from previously activated shipper entries that have been marked as active shipper for a division.</p>
<p>13. Review and click <b>Save</b> to accept the unit.</p>	<ul style="list-style-type: none"> <li>The confirmation message presented when saving a unit record states: “Are you sure you want to add this unit, by clicking Yes, you acknowledge that you inspected this unit and that it is satisfactory.”</li> <li>When the user clicks yes, the unit record is added to the division’s inventory. When the user clicks no, the user is allowed to edit information, clear the unit information or may attempt to save again.</li> <li>Adds the unit to the database.</li> <li>Lists summary information for the accepted unit.</li> <li>Allows the user to begin processing a new unit.</li> </ul> <p><b>NOTES</b> _____</p> <p>If the unit is not satisfactory per the visual inspection, the user must click no. The unit is not entered into VBECS inventory and should be processed according to local procedure and policy.</p> <p>VBECS assigns unit statuses based on a site’s configuration in Configure Division. For a:</p> <ul style="list-style-type: none"> <li>“Full-service” facility, VBECS assigns “Limited” status to blood products that contain red cells and “Available” status to all other components.</li> <li>“Transfusion-only” facility, VBECS assigns “Available” status to the units. ABO/Rh confirmation testing is not required.</li> </ul> <p>Units are available only to users in the division in which the units were processed (separate inventories are maintained in a multidivisional site).</p> <p>To edit incorrect saved information, inactivate the unit through Edit Unit Information, then reenter unit information in Incoming Shipment.</p>



User Action	VBECS
	<p>Existing invoices continue to display historic ISBT 128 labeled units.</p> <p>The user may not save the unit record until the donation type and its requirements are satisfied.</p> <p>The system determines that valid required data exist for individual units before adding to the database. The newly added unit displays in a list. This list includes Unit ID, ABO/Rh, Product Code, and Expiration date.</p> <p>When units are added to the database, the records must be created in a manner that the units are retrievable for other VBECS functionality by scanning the Unit ID and Product Code barcodes.</p>
14. Repeat until all units are entered.	<ul style="list-style-type: none"> <li>• Displays all unit data entries.</li> </ul>
15. Click <b>Close</b> .	

**Figure 71: Process Incoming Shipment**

## Outgoing Shipment

The user processes and ships available blood units from his division to an outside facility, blood center, or hospital.

The user may edit return credits (additional fees or credits may be issued from one site to another) before processing the shipment.

VBECS generates a shipping invoice.

## Assumptions

- Users are authorized to release units from inventory.
- Blood units being shipped:
  - Were entered in the division.
  - Are available for scanning.
  - Are located in the user's division.

## Outcome

- An invoice for a completed shipment is closed and may not be modified.
- An invoice for a shipment that is not confirmed is open and may be edited.
- The units are not available in the division's inventory.
- When a unit is transferred to another division in the database, VBECS displays the history of the unit at both divisions. The unit is active in one division at a time.
- A credit for a returned product may be assigned.

## Limitations and Restrictions

- A user may edit only unconfirmed invoices.
- The number of tests displayed on an invoice is limited to 8,000 characters per unit.
- When the user creates an invoice and cancels before saving it, VBECS excludes the invoice number from use.
- When assigned or restricted, the patient first name and ID are not printed on the outgoing shipment document to maintain patient privacy.
- Cost entry is capped at 999.99.

## Additional Information

- None

## User Roles with Access to This Option

All users

## Outgoing Shipment

A user ships blood units from his division's inventory to a facility that is not a transfusion location of his division. These units were removed from inventory and are available for scanning and packaging.

The user identifies the shipment destination, scans the unit ID and product code for the first unit in the shipment, and repeats this process for each unit added to the shipment. The user completes the data entry


and then reviews and accepts the shipment information. When the shipment is confirmed, the user may print a shipping invoice.

VBECS generates an invoice that includes information for the blood units on that invoice.

A user may edit return credits (additional fees or credits may be issued from one site to another) before processing the shipment.

User Action	VBECS
<p>1. Select <b>Shipments</b> from the main menu.</p> <p><b>Select Outgoing Shipment.</b></p>	<ul style="list-style-type: none"> <li>• Displays options for processing shipments.</li> <li>• Displays options for selecting a new invoice or an existing (recent) invoice.</li> </ul>
<p>2. To process a new invoice, select <b>New Invoice</b>. (To process an existing invoice, continue at Step 3.)</p> <p>Enter the invoice number provided by the outside facility or leave the Invoice Number field blank.</p> <p>Select a shipment destination from the drop-down list.</p> <p>Click the <b>Return to Shipper</b> or <b>Send to Unrelated Shipper</b> radio button to select a shipment type (Figure 72).</p> <p>Continue at Step 4.</p>	<ul style="list-style-type: none"> <li>• Displays fields for entering shipment information.</li> <li>• Generates an invoice number when the Invoice Number field is blank.</li> </ul> <p><b>NOTES</b> _____</p> <p>To select an existing invoice while in the new invoice window, click <b>Use Existing Invoice</b>.</p>
<p>3. To process an existing invoice, select <b>Recent Invoice</b> from the Outgoing Shipment menu.</p> <p>Select an invoice.</p>	<ul style="list-style-type: none"> <li>• Displays a list of open (unconfirmed) invoices.</li> </ul> <p><b>NOTES</b> _____</p> <p>To select a new invoice while in the existing invoice window, click <b>Create New Invoice</b>.</p> <p>A user may edit only saved, not confirmed, outgoing shipment invoices.</p> <p>A user may select shipping destination (ship-to) facilities based on site parameters that define a shipper for a division.</p> <p>A shipper must be marked as an active collection facility for a division before a user may select it as a valid shipper for a blood product for that division. This includes marking the division facility itself as an active collection facility to accommodate the creation of new blood products through modification.</p>
<p>4. Click <b>OK</b> to save the information entered (Figure 72).</p>	<ul style="list-style-type: none"> <li>• Displays fields for entering unit information.</li> </ul>

User Action	VBECS
<p>5. For each unit to be added to the shipping invoice, scan the unit ID and product code barcodes or enter them in the Unit ID and Product fields (Figure 73 and Figure 74).</p> <p>Click <b>OK</b>.</p>	<ul style="list-style-type: none"> <li>• Displays corresponding unit data.</li> </ul> <p><b>NOTES</b> _____</p> <p>When a unit selected for inclusion in a shipment is quarantined, VBECS warns the user and asks the user whether he wishes to continue.</p> <p><b>No</b> clears the unit information and allows entry of a new unit or shipment process. <b>Yes</b> adds the unit to the shipment.</p> <p>VBECS allows the shipment or transfer of units located only in the same division as the user and in the blood bank (for "Limited," "Available," "Assigned," and "Crossmatched" unit statuses only).</p> <p>When a unit is expired, VBECS warns the user.</p> <p>When units selected for shipment are assigned or crossmatched to a patient, VBECS warns the user and asks the user whether to release the unit before shipping.</p> <p><b>Yes</b> releases all patient assignments from the unit prior to adding the unit to the invoice (no assignment information is displayed on the invoice). <b>No</b> indicates that the outgoing shipment invoice must include the unit's patient association details.</p>
<p>6. Respond to warnings, when indicated.</p> <p>Click <b>Select</b> to add the selected unit to the shipping invoice or click <b>Clear</b> to delete a unit (Figure 75).</p>	<ul style="list-style-type: none"> <li>• Adds unit to or removes unit from a temporary list of selected units.</li> </ul>
<p>7. Repeat Steps 5 and 6 until all units for outgoing shipment are entered.</p> <p>Click <b>Save</b> to save the information entered (Figure 76) and <b>Yes</b> to confirm the save.</p> <p>To cancel the shipment, click <b>Cancel Shipment</b>, then <b>Yes</b> to delete the invoice and return to Step 1 to begin again, or click <b>No</b> to return to the invoice (Figure 77).</p>	<ul style="list-style-type: none"> <li>• Displays shipment information.</li> <li>• When a user clicks <b>Cancel Shipment</b>, asks whether the user wants to remove all units from the invoice. <b>Yes</b> deletes the invoice. <b>No</b> returns to the invoice.</li> </ul> <p><b>NOTES</b> _____</p> <ul style="list-style-type: none"> <li>• A user may add or delete units from a temporary list before accepting the information entered.</li> </ul>

User Action	VBECS
<p>8. To change the return credit amount for a unit in a "Return to Shipper" unconfirmed shipment, select a unit and right-click the mouse. Select <b>Return Credit Update</b> (Figure 78).</p> <p>Enter the return credit amount.</p> <p>Click <b>OK</b> to save the information entered or <b>Cancel</b> to return to the invoice.</p>	<ul style="list-style-type: none"> <li>• Prompts the user to accept the new amount.</li> <li>• Returns to the invoice and displays the return credit amount on the invoice.</li> <li>• Displays the unit's financial information and allows the user to edit the return credit amount.</li> </ul> <p><b>NOTES</b> _____</p> <p>VBECS applies a return credit (equal to the return credit percentage of the original cost of the unit) to units processed in an outgoing shipment to the original shipper. Users may edit return credits for individual units.</p> <p>Instead of right-clicking the mouse, a user may press the application key () on the keyboard.</p>
<p>9. To confirm the shipment, click <b>Save</b>, then <b>Yes</b> to save the invoice, or <b>Cancel</b> to return to the invoice (Figure 79).</p> <p>Click <b>Print Invoice</b> to generate and view an invoice.</p> <p>Click <b>Printer</b> to print an invoice to send with the shipment or click <b>Close</b> to return to the invoice.</p> <p>Click <b>Cancel</b> to exit or move to Step 9 to change a return credit amount.</p>	<p><b>NOTES</b> _____</p> <p>When units selected for shipment are currently restricted to a patient and the donation type of the unit is Voluntary Allogeneic, in addition to releasing the patient assignment, VBECS also removes the unit restriction in a Transfusion Only division configuration.</p>
<p>10. Cancel or confirm the shipment as in Step 9.</p>	<ul style="list-style-type: none"> <li>• Displays a "Confirm Shipment" option and stores the open invoice until the user chooses to complete the shipment.</li> </ul> <p><b>NOTES</b> _____</p> <p>When a shipment is saved but not confirmed, VBECS displays "NOT CONFIRMED" as the shipment date on the print version of the invoice.</p> <p>The shipping invoice includes patient assignment, restriction, quarantine indicator, biohazard information, antigen typing (specificity and interpretation only), special testing information, and (when it is in a unit record) the autologous unit testing status.</p>

User Action	VBECs
<p>11. When the shipment is confirmed, select or edit the date and time in the Shipment Date/Time field.</p> <p>Click <b>OK</b> to save the information entered or <b>Cancel</b> to return to the invoice.</p>	<ul style="list-style-type: none"> <li>• Displays shipment information, including the shipment date and time. When a user accepts the entered date and time, displays the invoice to be printed.</li> <li>• Updates the status of each unit in the shipment to show that each unit was transferred to the ship-to facility and assigns the shipment date and time as the date and time of the transaction.</li> </ul> <p><b>NOTES</b> _____</p> <p>When VBECs confirms an outgoing shipment, it:</p> <ul style="list-style-type: none"> <li>• Adds the shipment information to the unit record, changes the status of each unit in the shipment to “Transferred,” and assigns the date and time of the transaction to the shipment.</li> <li>• Saves the technologist ID, the name of the printing technologist, the division, and the date and time the shipment was processed. It also saves the confirmation date and time as the date and time of shipment. The user may accept or edit the current date and time displayed (no future date or time may be entered). The unit status is changed to transferred upon invoice confirmation.</li> </ul>
<p>12. Click <b>Print Invoice</b> to generate and view an invoice as in Step 9, or click <b>Cancel</b> to exit.</p>	<ul style="list-style-type: none"> <li>• Displays the information to be printed on the invoice.</li> <li>• Prints the shipping invoice.</li> </ul> <p><b>NOTES</b> _____</p> <p>When a user prints an invoice and VBECs does not find information to include in the invoice, VBECs notifies the user and asks whether he wants to continue to print. The user may cancel or print the report, which will include “No pending orders found.”</p>

**Figure 72: Process Outgoing Shipment**

Figure 73: Select Units 1

Figure 74: Select Units 2

Unit ID	Product	Code	ABO/Rh	Expires	Donation	Status
0100007A	FFP Thaw CPD	18201	O Pos	05/06/2006 15:04	Volunteer Allogeneic Donor	Available
0100007B	FFP Thaw CPD	18201	O Pos	05/06/2006 15:04	Volunteer Allogeneic Donor	Available
0100009	FFP Thaw CPD	18201	O Pos	05/08/2006 12:19	Volunteer Allogeneic Donor	Available
0100010	FFP Thaw CPD	18201	O Pos	05/09/2006 12:19	Volunteer Allogeneic Donor	Available (Quarantined)

 At the bottom, there are 'OK' and 'Cancel' buttons.

Figure 75: Select Units 3

The screenshot shows the 'Process Outgoing Shipment' window. At the top, the title bar reads 'VBECs - Process Outgoing Shipment'. Below the title bar, the window header is 'Process Outgoing Shipment'. The main interface includes:

- Invoice:** 959
- Ship To:** VAMC North Chicago, IL
- Date Processed:** 08/11/2011 14:46
- Unit Search:** Unit ID\* (W000000000009), Product\* (E0291V00)
- Details:** Donation Type (Volunteer Allogeneic Donor), Date Received (07/27/2011 12:33), Unit Status (Available), Volume (mL) (350)
- Selected Units Table:**

Unit ID	Product	ABO/Rh	Expires	Donation	Status	Date/Time Received	Patient	Biohazardous	Quarantined	Return Credit
- Buttons:** Select, Clear, Cancel Shipment, Confirm Shipment, Print Invoice, Save, Cancel

On the right side, a simulated blood unit label is displayed with the following information:

- Barcode: W000000000009
- ABO/Rh: A
- Donation Type: VOLUNTEER DONOR
- Expiration Date: 06 Aug 2011 12:33
- Product Code: E0291V00
- Barcode: RBC CPD>AS1

Figure 76: Select Units 4

This screenshot shows the same 'Process Outgoing Shipment' window as Figure 75, but with one unit selected in the 'Selected Units' table:

Unit ID	Product	ABO/Rh	Expires	Donation	Status	Date/Time Received
<input checked="" type="checkbox"/>	W000000000009	RBC CPD>AS1	A Neg	08/06/2011	Volunteer Allogeneic Donor	Available 07/27/2011 12:33

The 'Save' button is now highlighted with a floppy disk icon, indicating that the selection has been saved.



Figure 77: Select Units 5

VBECs - Process Outgoing Shipment

Process Outgoing Shipment

Invoice 959 Ship To VAMC North Chicago, IL Date Processed 08/11/2011 14:46

Unit Search  
Unit ID\* [ ] find  
Product\* [ ]

Details  
Donation Type [ ]  
Date Received [ ]  
Unit Status [ ]  
Volume (mL) [ ]

Selected Units

	Unit ID	Product	ABO/Rh	Expires	Donation	Status	Date/Time Received
<input checked="" type="checkbox"/>	VW000000000009	RBC CPD>AS1	A Neg	08/06/2011	Volunteer Allogeneic Donor	Available	07/27/2011 12:33

\* Required Field

Cancel Shipment Confirm Shipment Print Invoice Save Cancel

Figure 78: Return Credit Update

VBECs - Process Outgoing Shipment

Process Outgoing Shipment

Invoice 959 Ship To VAMC North Chicago, IL Date Processed 08/11/2011 14:46

Unit Search  
Unit ID\* [ ] find  
Product\* [ ]

Details  
Donation Type [ ]  
Date Received [ ]  
Unit Status [ ]  
Volume (mL) [ ]

Selected Units

	Unit ID	Product	ABO/Rh	Expires	Donation	Status	Date/Time Received
<input checked="" type="checkbox"/>	VW000000000009	RBC CPD>AS1	A Neg	08/06/2011	Volunteer Allogeneic Donor	Available	07/27/2011 12:33

\* Required Field

Cancel Shipment Confirm Shipment Print Invoice Save Cancel

Remove Unit  
Return Credit Update

Figure 79: Confirm Invoice

VBECs - Process Outgoing Shipment

Process Outgoing Shipment

Invoice: 959    Ship To: VAMC North Chicago, IL    Date Processed: 08/11/2011 14:46

Unit Search  
 Unit ID\*     Find  
 Product\*

Details  
 Donation Type   
 Date Received   
 Unit Status   
 Volume (mL)

Selected Units

	Unit ID	Product	ABO/Rh	Expires	Donation	Status	Date/Time Received
<input checked="" type="checkbox"/>	W000000000009	RBC CPD>AS1	A Neg	08/06/2011	Volunteer Allogeneic Donor	Available	07/27/2011 12:33

\* Required Field                   

Figure 80: Modify Return Credit

VBECs - Return Credit

Modify Return Credit

Unit Financials

Unit ID

Unit Cost \$

Special Testing Cost \$

Return Credit Amount\* \$

\* Required Field

**Figure 81: Confirm Shipment**

VBECs - Process Outgoing Shipment

Process Outgoing Shipment

Invoice 959 Ship To VAMC North Chicago, IL Date Processed 08/11/2011 14:46

Unit Search  
Unit ID\* [ ] [Find]  
Product\* [ ]

Details  
Donation Type [ ]  
Date Received [ ]  
Unit Status [ ]  
Volume (mL) [ ]

Selected Units

	Unit ID	Product	ABO/Rh	Expires	Donation	Status	Date/Time Received
<input checked="" type="checkbox"/>	W000000000009	RBC CPD>AS1	A Neg	08/06/2011	Volunteer Allogeneic Donor	Available	07/27/2011 12:33

\* Required Field

Cancel Shipment Confirm Shipment Print Invoice Save Cancel

**Figure 82: Confirm Shipment Date**

VBECs - Process Outgoing Shipment

Process Outgoing Shipment

Invoice 959 Ship To VAMC North Chicago, IL Date Processed 08/11/2011 14:46

Unit Search  
Unit ID\* [ ] [Find]  
Product\* [ ]

Details  
Donation Type [ ]  
Date Received [ ]  
Unit Status [ ]  
Volume (mL) [ ]

Selected Units

	Unit ID	Product	ABO/Rh	Expires	Donation	Status	Date/Time Received
<input checked="" type="checkbox"/>	W000000000009	RBC CPD>AS1	A Neg	08/06/2011	Volunteer Allogeneic Donor	Available	07/27/2011 12:33

\* Required Field

Cancel Shipment Confirm Shipment Print Invoice Save Cancel

VBECs - Shipment Date Entry


Shipment Date Entry

Confirm Shipment Date

Invoice Number 959  
Ship-To Facility VAMC North Chicago, IL  
City, State North Chicago, IL  
Shipment Date\* 08/11/2011 14:54

OK Cancel

**Figure 83: Confirmation of Invoice**

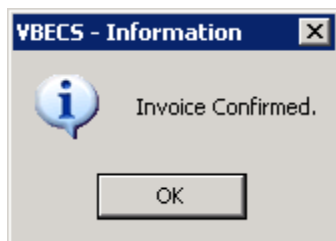

## Edit Invoice Text

VBECS includes a shipping invoice. The user may edit the explanatory text above the signature on the last page of a shipping invoice.

### Assumptions

- Information on the outgoing shipment invoice is generated during Outgoing Shipment.

### Outcome

- A division customized the text displayed on an outgoing shipment invoice.

### Limitations and Restrictions

- None

### Additional Information

- The default explanatory text reads: "I certify that the blood products listed were properly maintained, in accordance with the Code of Federal Regulations, while in storage at this institution. Components were inspected when packed for shipment and found to be satisfactory in color and appearance."

### User Roles with Access to This Option

- Enhanced Supervisor

### Edit Invoice Text

The blood bank user needs to edit the shipping invoice text box to conform to changes in regulations or division requirements. When the edit is complete, the user saves the altered text for future shipping invoices. An audit trail of changes is maintained.

User Action	VBECS
1. Select <b>Shipments</b> from the main menu.  Select <b>Edit Invoice Text</b> .	<ul style="list-style-type: none"> <li>• Displays options for processing blood product shipments.</li> <li>• Displays the current text for outgoing shipment invoice.</li> </ul>
2. Edit the text.	<ul style="list-style-type: none"> <li>• Displays entered data and changes for review.</li> <li>• Prompts the user to save or reject the changes.</li> </ul>

User Action	VBECS
3. Review the changes.  Click <b>OK</b> to save the text.  Click <b>Yes</b> to confirm the save and exit.	<ul style="list-style-type: none"> <li>Saves the new text and makes it available for outgoing shipment invoices.</li> </ul>

## Unit Search Screen

VBECS displays the Unit Search screen (Figure 84) when the user clicks the **find** button to select a unit through some of the Blood Units, Patients, Shipments, and Supervisor main menu items. The title of the screen may vary (e.g., Select Unit for Printing, Unit ABO/Rh Confirmation, Unit Antigen Typing, Edit Unit Information) depending on the option selected. (The user then selects criteria to continue the search.)

User Action	VBECS
1. Select an item from the main menu and the desired option.  Click additional buttons as needed.	<ul style="list-style-type: none"> <li>Displays the Unit Search screen.</li> </ul> <p><b>NOTES</b> _____</p> <p>Although multiple search criteria can be checked, the Unit Search screen performs independent searches not a summary search for all the selected variables.</p> <p>Users can obtain a listing of all available unit specifics by creating a custom Blood Availability Report allowing them to make the appropriate selections without looking at each unit's record.</p>
2. To search for a unit, enter the full or partial unit ID and click the <b>find</b> button, or just click the <b>find</b> button (Figure 84).  Click a check box to select a unit.  Click <b>OK</b> .	<ul style="list-style-type: none"> <li>Lists units that meet the search criteria.</li> <li>Displays unit-associated data.</li> </ul> <p><b>NOTES</b> _____</p> <p>To speed the search, enter a full or partial unit ID instead of just clicking the <b>find</b> button (which causes VBECS to search the entire database and takes longer).</p>

*VBECS checks unit IDs and product codes after the user enters the product code. When there is a discrepancy, VBECS displays the Input Error icon and allows the user to edit the data.*

Figure 84: Click the Find Button


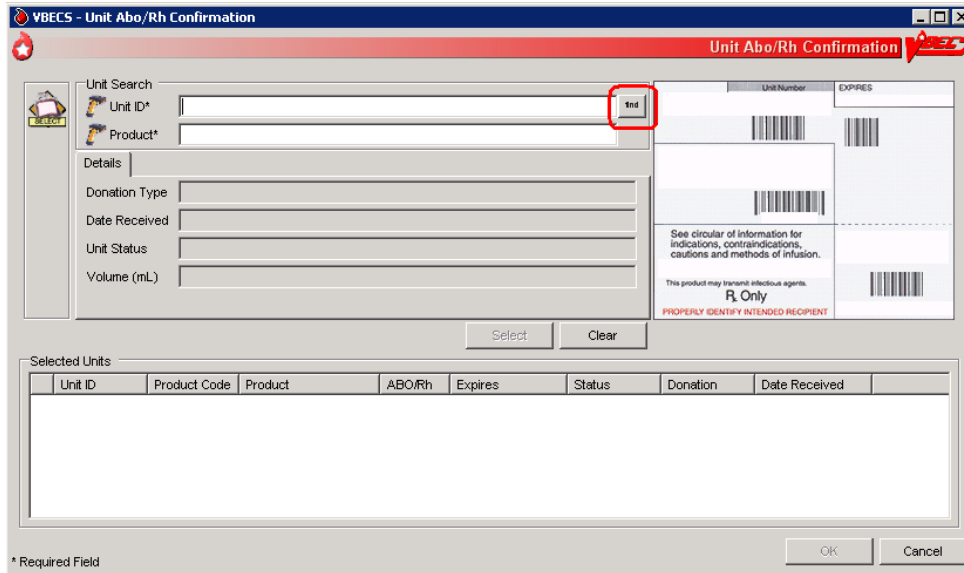


Figure 85: Display Unit Search Screen

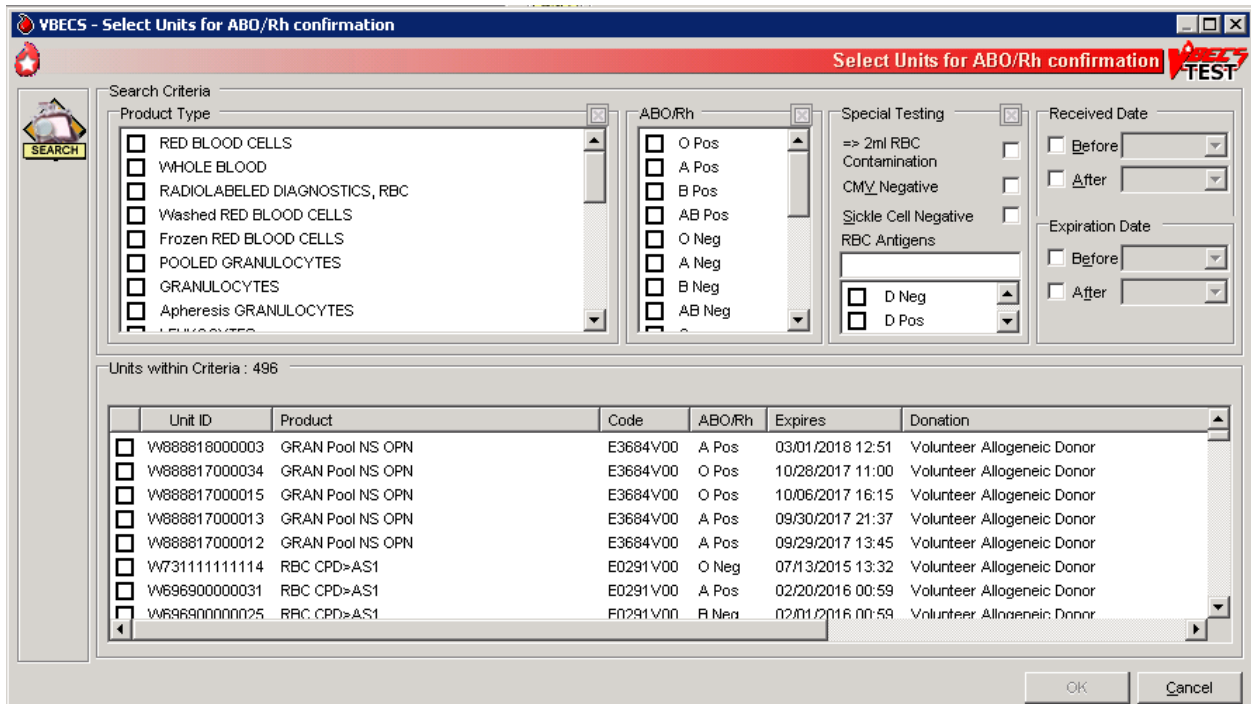

## **Test Units**

### **ABO/Rh Confirmation**

Blood products that contain equal to or greater than ( $= >$ ) of red blood cells (Red Blood Cells, Whole Blood, Granulocytes and Leukocytes) require ABO/Rh confirmation. The user may process ABO/Rh confirmation results by selecting an invoice or individual units of blood received in a shipment. VBECS records the ABO/Rh test results for the list of blood units tested.

### **Assumptions**

- The division is “full service.”
- A unit’s labeled ABO/Rh was entered in the same division as the user during login of an incoming shipment.
- Only invoices processed within a user’s division may be accessed to create an ABO/Rh confirmation worklist.

### **Outcome**

- Unit ABO/Rh observed test results and interpretations are available for selection for patient use.

### **Limitations and Restrictions**

- This option is disabled for transfusion-only facilities.
- When building a worklist by invoice number, only invoiced units not previously tested and confirmed appear on the worklist. The user may deselect or add additional units.
- For batch processing, the user can select incoming shipment invoices from his division, processed within the previous five days.
- VBECS does not display product codes associated with unit IDs in ABO/Rh Confirmation. Therefore, do not create ABO/Rh confirmation worklists containing units with the same unit ID.
- POOLED GRANULOCYTES labeled with an Rh type of “Pooled” is considered an Rh Positive unit which does not require confirmation testing.
- POOLED GRANULOCYTES labeled with an ABO group of “Pooled” cannot be confirmed. A blood component unit received as “POOLED ABO” will have to be transfused using the Supervisory option, Document ABO Incompatible Transfusion.
- Patient assignment is not released automatically when a unit is quarantined due to discrepant ABO/Rh retype testing. The unit is quarantined and cannot be selected for a different patient or issued for the current patient. Quarantined units are clearly marked in VBECS and cannot be issued.

### **Additional Information**

- Within a multidivisional facility when ABO/Rh confirmation is repeated on units from a facility within the same database, both the originating facility ABO/Rh confirmation testing results and the current testing results are stored.

### **User Roles with Access to This Option**

All users

## ABO/Rh Confirmation Testing


To confirm ABO/Rh testing, the user creates a worklist of individual units from an invoice, or creates a custom worklist. The user records observed test reactions and interpretations. VBECS uses truth tables to verify that user-entered interpretations are consistent with observed reactions. VBECS compares ABO and/or Rh interpretations to units labeled “ABO/Rh” during login and displays warnings when there are inconsistencies.




*When a unit’s ABO/Rh confirmation test is repeated after issue to a patient, VBECS does not compare the test results to the patient’s blood type at time of issue. The user must detect discrepancies and notify clinical personnel, in accordance with local policy.*

User Action	VBECS
<p>1. When processing a shipment, select <b>Shipments</b> from the main menu.</p> <p>When processing selected units, select <b>Blood Units</b> from the main menu.</p> <p>Select <b>ABO/Rh Confirmation</b>.</p> <p>Select an invoice or units.</p> <p>Click <b>OK</b>.</p>	<ul style="list-style-type: none"> <li>• Displays options for processing blood product shipments. By default, displays units in the worklist in the order in which they were entered during Incoming Shipment.</li> <li>• Lists invoices processed within the last five days in reverse chronological order.</li> <li>• Displays fields for entering rack information.</li> <li>• Allows units to be deleted from or added to the worklist.</li> </ul> <p><b>NOTES</b> _____</p> <p>The user may select only active units and those not in an inactive or final status.</p> <p>To create a customized list of units:</p> <ul style="list-style-type: none"> <li>• Enter a unit ID or click the <b>find</b> button to search for a unit. VBECS lists units with “limited” status.</li> <li>• Select parameters from the options on the Select Units for ABO/Rh Confirmation screen.</li> <li>• Select <b>Units within Criteria</b> to work with.</li> <li>• Click <b>OK</b>.</li> <li>• Repeat this step until all units are included in the Selected Units list on the Unit ABO/Rh Confirmation screen.</li> <li>• Click <b>OK</b>.</li> </ul>



User Action	VBECS
<p>2. Select a rack.</p> <p>Select a date or accept the default.</p> <p>Select a Tested By name.</p> <p>Click <b>Using Automated Instrument</b>, as appropriate.</p> <p>Click the <b>Perform Anti-D Testing</b> check box, when appropriate.</p> <p>Click <b>OK</b>.</p>	<ul style="list-style-type: none"> <li>• Displays the racks available using the pick list provided.</li> <li>• Displays the date and time testing was performed. The default is the current date and time. The date may be edited using the pick list provided.</li> <li>• Displays the testing technologist's identification. The default is the current user. Select a name from the pick list of valid division users to show a different testing technologist.</li> <li>• Displays warnings and the option to indicate that the user is ready to begin entering results after all warnings are addressed.</li> <li>• Generates an Exception Report entry when the user inactivates or deletes the partially completed result grids and enters a comment.</li> </ul> <p><b>NOTES</b> _____</p> <p>A user may disable the anti-D testing grid column in the ABO/Rh unit confirmation worksheet. When the Anti-D testing grid display is disabled, the user may select only RH Positive labeled units for confirmation testing.</p>
<p>3. Select a unit for which to record test results.</p> <p>Select a cell and enter an observed test result from the legend at the bottom of the screen.</p> <p>Enter a valid interpretation, then press the <b>Enter key</b> to go to the next data grid interpretation or Comment cell.</p> <p>Select enter comments from the Unit Testing for the ABO and Rh interpretations, as needed.</p> <p>Repeat until all test results for the selected unit are entered.</p>	<ul style="list-style-type: none"> <li>• Creates a grid for data entry of results and interpretations, displaying units in the order selected.</li> <li>• Allows the user to select a blood test type and enter a test result for the unit.</li> <li>• Restricts data entry to valid selections for testing results. Displays testing results and interpretations entered.</li> <li>• An interpretation indicates that all appropriate test results were entered for the unit.</li> </ul> <p><b>NOTES</b> _____</p> <p> VBECS emits an audible alert and generates a warning requiring override when the ABO and/or Rh confirmation interpretation does not match the ABO and/or Rh of the unit at initial login. When the user confirms the discrepancy, inventory units whose ABO/Rh confirmation does not match the ABO/Rh at login are marked "Quarantined." VBECS captures details for inclusion in an Exception Report (exception type: Unit ABO/Rh login vs. confirmation do not match, unit quarantined).</p> <p>ABO/Rh confirmation testing data grids accommodate uniform data entry for Forward ABO testing.</p> <p>The cursor moves with a single valid keystroke from the top left cell across the row, returning to the leftmost cell in the next row, and so on down the worksheet grid. The cursor stays in the reaction results cells by default (the technologist may use the mouse to click in the interpretation area) until the last entry is made in the reaction result portion of the grid. The cursor then goes to the first (top) interpretation cell.</p> <p>The user may enter free-text comments or comments from the Unit Testing for the ABO and Rh interpretations (Table 4).</p>

User Action	VBECS												
	<p><b>Table 4: Valid Interpretations</b></p> <table border="1" data-bbox="729 336 1183 512"> <thead> <tr> <th data-bbox="729 336 945 365">ABO</th> <th data-bbox="945 336 1183 365">Rh</th> </tr> </thead> <tbody> <tr> <td data-bbox="729 365 945 394">A</td> <td data-bbox="945 365 1183 394">P, POS (positive)</td> </tr> <tr> <td data-bbox="729 394 945 424">B</td> <td data-bbox="945 394 1183 424">N, NEG (negative)</td> </tr> <tr> <td data-bbox="729 424 945 453">AB</td> <td data-bbox="945 424 1183 453">I, Inconclusive</td> </tr> <tr> <td data-bbox="729 453 945 483">O</td> <td data-bbox="945 453 1183 483"></td> </tr> <tr> <td data-bbox="729 483 945 512">I, Inconclusive</td> <td data-bbox="945 483 1183 512"></td> </tr> </tbody> </table> <p>When a user saves a partially completed ABO/Rh testing worklist, VBECS clears unit entries with incomplete or no results, releases them from the worklist, and makes them available for inclusion in a future worklist.</p> <p>Row validation is established for the interpretation vs. reaction results pattern by system rules.</p> <p>ABO/Rh interpretations are consistent with observed test results based on system rules. Discrepancies are resolved before VBECS can verify ABO/Rh confirmation results and add them to the database. The user may reenter valid results or delete entries and start over.</p>	ABO	Rh	A	P, POS (positive)	B	N, NEG (negative)	AB	I, Inconclusive	O		I, Inconclusive	
ABO	Rh												
A	P, POS (positive)												
B	N, NEG (negative)												
AB	I, Inconclusive												
O													
I, Inconclusive													
<p>4. Confirm that all results were reviewed and are acceptable.</p>	<ul style="list-style-type: none"> <li>Repeats data validation and prompts the user to save.</li> </ul> <p><b>NOTES</b> _____</p> <p>VBECS warns that it doesn't save partially completed unit ABO/Rh confirmation worksheets.</p> <p> When VBECS verifies that the ABO or Rh interpretation is "Inconclusive," it assigns the unit a "Quarantine" indicator, emits an audible alert, and captures details for inclusion in an Exception Report (exception type: ABO/Rh confirmation Inconclusive, unit quarantined). An Exception Report entry is not created when a unit is resulted as weak D positive or weak D inconclusive and is properly quarantined.</p> <p>Upon successful completion of ABO/Rh confirmation, the unit status is updated. Successful completion means that the ABO and Rh (if required) interpretation is consistent with recorded observed results and matches the ABO/Rh at unit login and any repeat ABO/Rh tests on the unit that are not invalidated.</p>												
<p>5. Click <b>OK</b> to save the ABO/Rh confirmation testing.</p>	<ul style="list-style-type: none"> <li>Prompts the user to confirm that all results were reviewed and are acceptable.</li> </ul>												
<p>6. Click <b>OK</b>.</p>													

## Unit Antigen Typing

The user performs antigen typing for one or more of the antigens in a blood unit and records the results in a worklist.

### Assumptions

- The division is “full service”.
- The units and the user are in the same division.
- Antiserum inventory is entered during Log In Reagents.
- Workload codes are assigned through Workload Codes.

### Outcome

- Antigen typing was applied in the unit record for informational purposes in other options.
- VBECS stores the lot number, vial identifier, manufacturer, and expiration date of the positive and negative control cells with the testing record.

### Limitations and Restrictions

- Antigen typing applies only to red blood cell antigens. This option does not address platelet, histocompatibility locus antigens (HLAs), or Immunoglobulin A (IgA) antigenicity.
- Only blood units of the “WHOLE BLOOD” and “RED BLOOD CELLS” ICCBBA component classes can be selected for inclusion in the antigen type worklist.
- Antigen typing is not required on products with  $\geq 2$  mL of RBC contamination that are not Red Blood Cells or Whole Blood.
- VBECS does not check whether the lot and vial numbers selected for the positive control cells differ from those of the negative control cells.
- D antigen typing must not be combined in batch testing of Rh positive units.
- The sort order of the selected units may not be inherited by the testing grid. Organize physical test materials according to the order of units presented on the testing grid.

### Additional Information

- This option is disabled for transfusion-only facilities.
- Clinical significance and availability of typing reagents are not implied when antigens are included in the list of antigen type tests.
- A CPRS patient order is not required to perform antigen typing of units.
- A user may query the database for the presence of antigen-typed units and display or print the query results in Blood Availability.
- A user may retrieve evidence of antigen typing for any date or date range. Regulatory agencies require documentation of the reagent name, lot number, reagent expiration date, and positive and negative control.
- A user is required to complete antigen typing QC in order to save test results. If an antigen testing worklist is started but not saved and a second testing worklist is started for the same antigen, the QC grid will display and require completion because the first worklist has not been saved.

- When invalidating results from Unit Antigen Typing, the Exception Report does not show all invalidated values. View the invalidated tests on the Testing Worklist Report and the Unit History Report.
- To prevent a system error when entering reagent lot numbers, don't use the Backspace key and click "OK" to erase one of multiple entries. Highlighting and retyping the lot number will not cause the error to occur.


## User Roles with Access to This Option

All users

## Unit Antigen Typing



The user performs antigen typing for one or more of the antigens on blood units. VBECS creates a list of available antigen specificities, including weak D, that can be tested. For each antigen tested, the user must select a reagent from available inventory. VBECS will check to see if valid positive and negative controls were tested for the selected reagent on the calendar day of testing. When controls are not tested, VBECS adds tests for the positive and negative controls to the worksheet.


Red cell units prepared for patient transfusion may need to be negative for this antibody specificity based on their setting in the antibody table defined in Antibodies.

 *Deletion of a partially completed antigen typing worklist invalidates any completed tests contained on the worklist regardless of the status of the unit, including transfused units.*

*It is the user's responsibility to verify that the records are being invalidated appropriately.*

User Action	VBECS
1. Select <b>Blood Units</b> from the main menu.  Select <b>Unit Antigen Typing</b> .	<ul style="list-style-type: none"> <li>• Displays options for processing blood units.</li> <li>• Displays options for processing antisera.</li> </ul>
2. Click <b>Create New Worklist</b> (Figure 86, Figure 87).  When partially completed worklists exist, click a check box to select an existing worklist and go to Step 5 (Figure 88).  Click <b>Delete</b> to delete the worklist from the list or <b>OK</b> to continue.	<ul style="list-style-type: none"> <li>• Accommodates selection of a partially completed testing worklist or creation of a new testing worklist.</li> <li>• Displays an option to select or delete a partially completed worksheet.</li> <li>• When selecting a partially completed worklist, the user verifies the lot numbers, manufacturer name, and expiration date of the antiserum and control cells previously selected. These may not be edited. When a discrepancy is found, the user must delete the worklist and start over.</li> </ul> <p><b>NOTES</b> _____</p> <p>VBECS displays the date and time the testing was performed (default: current date and time). These are editable to allow for retrospective entry.</p> <p>VBECS displays the testing technologist ID (default: current user). A pick list of valid division users is available for the user's selection.</p>

User Action	VBECS
<p>3. To create a new testing worklist, select a date or accept the default.</p> <p>Select a "Tested By" name.</p> <p>Click a radio button to select Tube, Gel, or Solid Phase.</p> <p>Click <b>Using Automated Instrument</b>, as appropriate.</p> <p>Click <b>OK</b>.</p>	<p>When an automated instrument was used for testing, the user enters the name of the instrument and may indicate that successful quality control testing was done for the date.</p> <ul style="list-style-type: none"> <li>• Displays an option to select antiserum types.</li> <li>• Displays an option to select a lot number for each antiserum type selected.</li> <li>• Displays the lot number information.</li> <li>• Prompts the user to select the correct antiserum entry for the specificity selected.</li> </ul> <p><b>NOTES</b> _____</p> <p>VBECS lists available antigen type tests from which the user may select and allows the user to enter the specificity to expedite the search. The list of antigen type tests is derived from the Corresponding Antigens field from Appendix B: Table 14: Antibody and Antigen Table: Irregular Antibodies.</p> <p> VBECS displays data fields for the user to select or enter the lot number, and displays the manufacturer and expiration date of each antiserum to be used in this transaction to allow the user to choose the correct antiserum. VBECS displays in-date antisera and allows the selection of outdated antisera, if needed, emits an audible alert, and requires an override and generation of an Exception Report (exception type: expired antisera used). A comment is required.</p> <p> When a user attempts to change the phases of reactivity, VBECS emits an audible alert, warns that the user selected phases that were not used previously for this reagent, and asks whether the user wishes to continue.</p> <p><b>No</b> returns the user to the screen to select the correct testing phases. <b>Yes</b> requires the user to enter a comment explaining why different testing phases are selected, and captures details for inclusion in an Exception Report (exception type: antigen testing phase change).</p> <p>When a user selects a reagent rack for which quality control (QC) testing was never performed in Enter Daily QC Results, VBECS displays a warning message, clears the selection, and allows the user to select a different rack.</p> <p>When the user QC'd the antigen typing reagent lot number and performed patient antigen typing on the same date under different testing phases, VBECS enables the QC grid for the antisera in the phases the user selected.</p>
<p>4. Define the antiserum used.</p> <p>Enter a lot number for each antiserum specificity.</p> <p>Select the testing phase:</p> <ul style="list-style-type: none"> <li>• IS</li> <li>• IS/RT</li> </ul>	<ul style="list-style-type: none"> <li>• Displays an option to select the testing phases for the antiserum specificity by lot number.</li> <li>• Displays a testing grid disabling the phases not to be entered.</li> <li>• Stores the antiserum information and checks whether the positive and negative controls need to be tested with this batch.</li> <li>• Prompts the user to enter the lot number and vial identifier for the positive and negative control used for each antiserum tested.</li> </ul>

User Action	VBECS
<ul style="list-style-type: none"> <li>• IS/37</li> <li>• AHG/CC.</li> </ul> <p>Select the rack, if applicable.</p> <p>Select <b>Add</b>.</p> <p>Select additional antisera, if desired.</p> <p>Respond to warnings and enter comments, when indicated.</p> <p>Select <b>OK</b> (Figure 89).</p>	<p><b>NOTES</b> _____</p> <p>When VBECS displays lot numbers for selection, it checks the inventory system for the reagent name and lot numbers in a rack. The user cannot force the entry of a lot number that VBECS disallowed on the selection list. The user must choose a different reagent. The user may not select a reagent with an “unsatisfactory” value in the Inspection field or with a quantity of zero.</p> <p> VBECS compares the expiration date of each reagent on the selected rack to the testing date, emits an audible alert, and warns when the reagent is expired.</p> <p>VBECS enables the rack identifier field when a user selects the weak D or AHG/CC phase and disables the field when a user selects other phases.</p> <p>VBECS automatically adds the positive and negative control tests to the selected reagent type worklist if they have not been tested on that calendar date for the selected antisera lot number. Reagent lot numbers that are selectable for positive and negative control cells:</p> <ul style="list-style-type: none"> <li>• Reverse ABO Typing Cells (set)</li> <li>• A2 cell (vial)</li> <li>• Screening Cells (set)</li> <li>• Check Cells (vial)</li> <li>• Panel (set)</li> <li>• Other (set)</li> </ul> <p>VBECS requests information about an antigen lot number only the first time it is used on a specific day.</p> <p>If there are multiple test tabs, VBECS displays a message that may not correspond to the selected tab, but will correspond to information required by one of the requested tests.</p>
<p>5. Select one or more blood units to add to the worklist.</p> <p>Enter the unit ID and the product code from the inventory, or click the <b>find</b> button to search for a blood unit and select from the list of available units.</p> <p>Click <b>OK</b> when all units are selected.</p>	<ul style="list-style-type: none"> <li>• Allows the user to scan or enter units to be included in the worklist or select them from a list.</li> <li>• Lists the selected units.</li> </ul> <p><b>NOTES</b> _____</p> <p>Select units in small batches (&lt;38) to prevent process delays and screen distortion. The order of units does change from the selection order and requires verification.</p>
<p>6. Select the lot number for the reagent and the positive and negative control cells. Enter the vial identifiers, if required. Enter “0” when not using panel cells.</p>	<ul style="list-style-type: none"> <li>• Displays the reagent type and lot numbers, including the vial identifier.</li> </ul> <p><b>NOTES</b> _____</p> <p>The vial identifier, comprising one or two digits, identifies the vial contained in a kit of reagent red blood cells. A vial contains</p>

User Action	VBECS
<p>Verify the reagent type and the positive and negative control cell lot numbers selected and vial identifiers entered, if required.</p>	<p>antigen positive or antigen negative blood cells to be used in quality control of the antisera.</p>
<p>7. Respond to warnings and continue with antigen typing.</p> <p>Review the lot number, manufacturer name, and expiration date for the antiserum and control cells. If they are not correct, start over.</p> <p>Select a control cell or unit to begin entering serologic reactions and interpretations in the data entry grid.</p>	<ul style="list-style-type: none"> <li>• Displays each selected unit in a separate tab for each selected specificity with its own grids and quality control.</li> <li>• Displays a direct data entry (DDE) grid to enter serologic reaction results for antigen typing of units. This may already be filled with results from a previously saved partially completed worksheet.</li> <li>• Allows a user to select a unit or control cell and enter a test result.</li> <li>• Allows only valid data entries in the grid reaction result and interpretation cells.</li> <li>• Restricts data entry to valid selections for testing results.</li> <li>• Keeps all testing results and entered interpretations on the screen until the user saves them.</li> </ul> <p><b>NOTES</b> _____</p> <p>VBECS displays the worklist for the grids by antiserum types.</p> <p>Valid observed test results entered by a user:</p> <ul style="list-style-type: none"> <li>• 1 [1+ (positive)]</li> <li>• 2 [2+ (positive)]</li> <li>• 3 [3+ (positive)]</li> <li>• 4 [4+ (positive)]</li> <li>• W [Weak (positive)]</li> <li>• F [Mixed Field (positive)]</li> <li>• M [Microscopic (positive)]</li> <li>• R [rouleaux (negative)]</li> <li>• 0 (zero) [No Agglutination (negative)]</li> <li>• X (Not Tested) (does not indicate a disabled cell)</li> </ul> <p>When the AHG phase is selected for testing, the AHG must be negative and the CC must be at least 1+ positive to consider the test negative for a unit or for the negative control cell.</p> <p>The column heading of the data grid specifies the phases of reactivity that may be used in a test selected for the specificity.</p> <p>The row heading of the data grid specifies the unit ID being tested. These fields cannot be edited.</p> <p>The rows for each antiserum type grid display the positive and negative control cells, then units, in the order in which a user added units to the worklist.</p> <p>A comment can be associated with any antigen test performed. A user may select canned comments required by an override from the drop-down list or select <b>Other</b> to enter free-text comments.</p>

User Action	VBECS
<p>8. Enter results for the control cells and for all selected units. Refer to the legend at the bottom of the screen (Figure 90).</p>	<ul style="list-style-type: none"> <li>• Prompts the user to confirm that all results were reviewed and are acceptable.</li> </ul> <p><b>NOTES</b> _____</p> <p>The positive and/or negative control cell test results on the date of testing must be a valid positive or negative.</p> <p>When a weak D interpretation is positive or inconclusive and is verified for a tested blood unit that was labeled as (Rh) D negative, VBECS warns the user and adds the quarantine indicator to the unit.</p> <p>VBECS compares the antigen typing interpretation a user enters to previously assigned antigen typings for the same antigen (as assigned during Incoming Shipment, Edit Unit Information, or previous testing). When the previous interpretation was not "inconclusive" and the current interpretation does not match the previously identified antigen type, VBECS warns the user and quarantines the unit.</p> <p>System rules establish row validation for the interpretation vs. reaction results pattern [for all user-indicated phase(s) of reactivity] for blood unit and quality control tests.</p>
<p>9. Repeat Steps 4–8 for each antiserum type.</p>	<ul style="list-style-type: none"> <li>• Displays the option to save.</li> </ul>
<p>10. Respond to warnings and save.</p>	<ul style="list-style-type: none"> <li>• Saves the completed QC results with partially or fully completed unit testing data.</li> <li>• Reconfirms that all appropriate test results were entered, repeats all data validation, and saves the information on the worksheet displayed.</li> </ul> <p><b>NOTES</b> _____</p> <p>VBECS saves the testing of one antiserum specificity at a time.</p> <p>The Positive and Negative control cells must be complete and valid prior to the save of a unit antigen typing result. VBECS will display a tooltip message to the user if an attempt to save is made and the QC is not valid.</p> <p>VBECS stores the lot number, vial identifier, manufacturer, and expiration date of the positive and negative control cells with the testing record.</p>
<p>11. Click <b>OK</b> to exit.</p>	



Figure 86: Unit Antigen Typing (Initial screen)

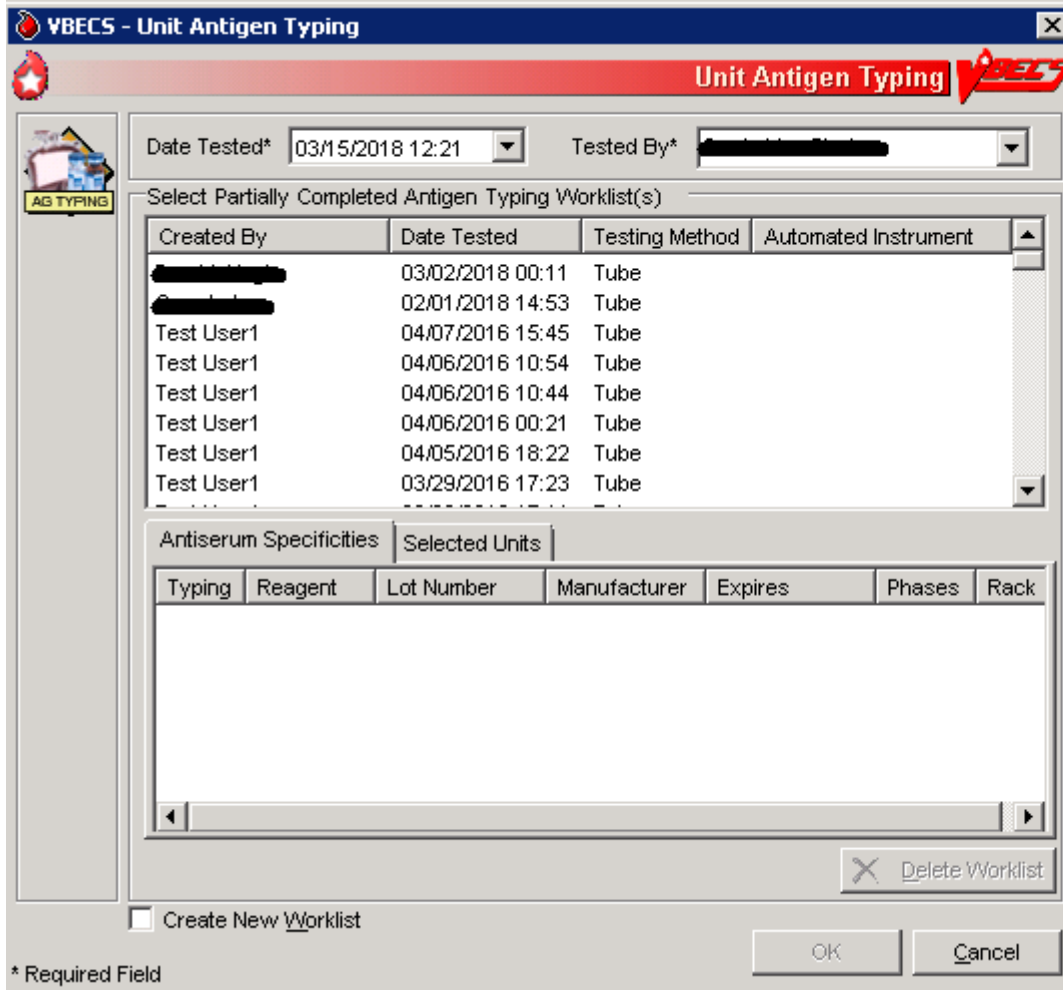


Figure 87: Unit Antigen Typing (New worklist)

**VBECS - Unit Antigen Typing**

**Unit Antigen Typing**

Date Tested\* 03/15/2018 12:19 Tested By\* Carrie Van Stedum

Testing Method\*

Tube

Gel

Solid Phase

Using Automated Instrument

Instrument Name\*

Instrument\_QC'd?

Define Antiserum Specificity

Antigen Typing\*

Lot Number\*

Reagent Type

Manufacturer

Expires

Rack\*

Testing Phases\*

IS

IS/RT

IS/37

AHG/CC

37/AHG/CC

Add Clear

Selected Specificities

Typing	Reagent	Lot Number	Manufacturer	Expires	Phases	Rack

Select Partially Completed Worklist.

OK Cancel

\* Required Field

Figure 88: Unit Antigen Typing (Partially Completed Worklist)

VBECs - Unit Antigen Typing

Unit Antigen Typing

wD

AS TYPING

**Unit Weak D Typing**

Unit ID	Product Name	D 37	Dc 37	D AHG	Dc AHG	D CC	Dc CC	Interp.	Comment
W000077770012	RBC CPD>ASS IR								
1171001	RBC Froz Rej Gly								
W001100000001	RBC CPD>AS1 P								

Lot Number  Manufacturer  Expires

**Key**

- ✓ Valid
- W Warning
- E Error

**Valid Entries**

0 = No Agglutination, R = Rouleaux, 1 = 1+, 2 = 2+, 3 = 3+, 4 = 4+, W = Weak, F = Mixed Field, M = Microscopic, X = Not Tested

\* Required Field

OK Cancel

Figure 89: Unit Antigen Typing

**VBECS - Unit Antigen Typing**

Date Tested\* 02/02/2006 14:00 Tested By\* vhaishvbecs1

Testing Method\*  
 Tube  
 Gel  
 Solid Phase

Using Automated Instrument  
 Instrument Name\*  
 Instrument Q/C'd?

Define Antiserum Specificity

Antigen Typing\*  
 Lot Number\*  
 Reagent Type  
 Manufacturer  
 Expires  
 Rack\*

Testing Phases\*  
 IS  
 IS/RT  
 IS/37  
 AHG/CC  
 37/AHG/CC

Add Clear

Selected Specificities						
	Typing	Reagent	Lot Number	Manufacturer	Expires	Rack
<input checked="" type="checkbox"/>	AGK	Anti-K	20	A.A.A	08/20/2006 23:59	AHG/CC A

\* Required Field

OK Cancel

Figure 90: Unit Antigen Typing Serologic Results

VBECs - Unit Antigen Typing

Unit Antigen Typing

K

AS TYPING

Control	Reagent	Lot Number	Vial	IS	RT	37	AHG	CC	Interp.	Comment
✓ POS	Screening Cells	1	1				2	X	P	
✓ NEG	Screening Cells	1	2				0	2	N	

Unit ID	Product Name	IS	RT	37	AHG	CC	Interp.	Comment
✓ 1941954	RBC AS1, 04210				0	2	N	

Lot Number  Manufacturer  Expires

Key

- ✓ Valid
- ⚠ Warning
- ✖ Error

Valid Entries

Any Text

\* Required Field

OK Cancel

## **Modify Components**

### **Modify Units**

The user modifies a unit, which results in a different blood product (target product) with its own properties. A division's configuration determines the available modification types.

#### **Assumptions**

- Product Modifications was executed.
- The division is the active shipper for target blood products.
- A unit may be modified with or without patient assignment or restriction.
- A unit may be modified with or without a patient order.
- Units selected for modification must be in the same division as the user.
- Product labels and unit numbers for modified products are on hand.
- Supplies needed for the unit modification were entered in VBECS.

#### **Outcome**

- Data generated for an Exception Report are stored for inclusion in the next Exception Report.
- Workload-related data collected as the result of unit modification are stored for inclusion in Workload Reports.
- At the completion of a unit modification, VBECS assigns the original unit a final status of "modified."
- A permanent virtual link exists between the original unit and target units created.
- The shipper of the target unit created is the division in which the modification was performed. The cost assigned to the target unit is based on what was defined in Blood Products.

#### **Limitations and Restrictions**

- VBECS allows modification of units only in limited, assigned, crossmatched, and available statuses.
- VBECS presents the conservative expiration date for target units. VBECS does not know the specific collection details. It is the responsibility of each user to accurately enter and process any one blood unit based on its expiration date/time as labeled by the collection facility.
- Pooling of units that have the attribute form V0013 (irradiated) and attribute V0021005 (psoralen treatment) does not produce a target.
- Weld controls are disabled after the user makes their initial selection.

#### **Additional Information**

- VBECS requires the identification of the transferred-to container when using an open method or a sterile connection device (SCD).
- VBECS requires the identification of the SCD wafer data when using an SCD.
- When a user extends an expiration date beyond the default expiration date, the override message displays the original expiration date.
- Due to the variations in supplies and equipment used for other modification types, VBECS does not require supply or equipment entry, but allows entry of those data as part of the modification record.
- During the modification options (Modify Units: Pool Units; Modify Units: Split a Unit; Modify Units), a user may select an assigned blood unit for modification and transfer one patient assignment to the target units created.

- To reverse other modifications, remove the “modified” status from each unit through Supervisor Menu, Remove Final Status.
- To edit the number of units in an existing pool, use Add/Remove Units from a Pool.
- The user can scan to select the target, but it will only present the product codes available as a target by VBECS.
- The user can change the expiration date and time prior to completing blood unit’s modification.

## User Roles with Access to This Option

All users

### Modify Units

The user selects the units to be modified. The user indicates the type of modification to be performed, then enters (preferably by scanning) the unit ID and product code.


VBECS displays a creation date and time (default: current date and time) for the target units, which the user may change to a date and time in the past and not before the login date and time of the original unit. VBECS calculates a new expiration date for the target unit (based on the creation date and time, modification type, method, and original unit product type) and records the date and time the original unit was modified. Some modification procedures require materials in addition to the new container. VBECS requires the entry of a new container lot number and SCD wafer, if indicated.


The modified target unit is now available for use.


User Action	VBECS
1. Select <b>Blood Units</b> from the main menu.  Select <b>Modify Units</b> .	<ul style="list-style-type: none"> <li>• Displays options for processing blood units.</li> <li>• Displays valid modification options configured for the division and allows the user to select an option.</li> </ul>
2. Enter the date and time of the unit modification.  Enter the name of the user who modifies the unit.	<ul style="list-style-type: none"> <li>• Displays the default (current) date and time as the modification date and time.</li> <li>• Allows the user to change the date and time to a past date and time.</li> </ul> <p><b>NOTES</b> _____</p> <p>The user may not enter a future date and time.</p> <p>When a unit is modified, the processing fee is added to the cost of a modified unit target.            A single unit modification has the processing fee added to the target unit.            When split, the processing fee is divided and a portion is added to each target unit.            When pooling, the processing fee is multiplied for each original unit included in the pool.</p>

User Action	VBECS
<p>3. Click a radio button to select a modification type:</p> <ul style="list-style-type: none"> <li>• Thaw</li> <li>• Pool (See Modify Units: Pool Units.)</li> <li>• Thaw/Pool Cryo (See Modify Units: Pool Units.)</li> <li>• Split/Divide (See Modify Units: Split a Unit.)</li> <li>• Irradiate</li> <li>• Leukoreduce</li> <li>• Volume Reduce</li> <li>• Wash</li> <li>• Rejuvenate</li> <li>• Freeze</li> <li>• Deglycerolize</li> <li>• Add/Remove Units from Pool (See Add/Remove Units from a Pool.)</li> </ul>	<ul style="list-style-type: none"> <li>• Allows the user to enter data based on the modification type selected.</li> <li>• Allows the user to select a different processing technologist.</li> <li>• Displays only modifications enabled for the division.</li> </ul>
<p>4. Click a radio button to select a modification method, if required:</p> <ul style="list-style-type: none"> <li>• Open</li> <li>• Closed</li> <li>• Sterile Connection Device (Figure 91)</li> </ul>	<ul style="list-style-type: none"> <li>• Displays the selected method.</li> </ul> <p><b>NOTES</b> _____</p> <p>VBECS stores the date and time for the original unit when it is modified. The modification date and time are also stored as the creation time for the target unit for splits and pools only.</p> <p>When the modification method is “Open” or “Sterile Connection Device (SCD),” the user must select a container supply type. When the modification method is “Closed,” VBECS does not allow the user to select a container supply type.</p> <p>For an SCD modification method, the user must indicate the completeness of the weld, in accordance with local policy and procedures. VBECS defines weld status as:</p> <ul style="list-style-type: none"> <li>• Complete: the weld is complete and has no leakage or air bubbles.</li> <li>• Incomplete: the weld has a leak or air bubbles are present. VBECS changes the expiration date and time of the unit to those of an open system for that product type. VBECS stores this weld check with the historic record of the target unit created in the modification.</li> </ul> <p>The system stores this weld check with the longitudinal record of the target unit created.</p>
<p>5. Click <b>OK</b> to accept the selected method, or edit it.</p>	<ul style="list-style-type: none"> <li>• Allows the user to scan, enter, or select units to modify.</li> </ul>



User Action	VBECS
<p>6. Scan, enter, or select the ID and product code of the unit to be modified.</p>	<ul style="list-style-type: none"> <li>• Displays current data for the selected unit.</li> <li>• Verifies the appropriateness of the selected unit based on system rules and truth tables and displays error and warning messages based on the modification type selected and the user's security level.</li> <li>• Prompts the user to continue with the selected unit or select a new unit based on the data in the review window.</li> </ul> <p><b>NOTES</b> _____</p> <p>The modification types are presented in the sort order displayed followed by Add/Remove Units from Pool.</p> <p>See Appendix B: Table 22: Allowable Product Modifications by Original Product Type.</p> <p>The user may select only active units and those not in an inactive or final status.</p> <p>When the user selects a status other than "limited," "available," "assigned," or "crossmatched" for an original unit, VBECS warns the user. There is no override.</p> <p>When the selected unit is not appropriate for the modification type selected, VBECS warns the user and instructs him to verify the selected unit and modification type. There is no override. When the unit is expired or quarantined, VBECS displays an error message.</p> <p>The user may select units containing red blood cells chosen for rejuvenation within three full days past their labeled expiration date. (For example, when a unit's expiration date is the current day, VBECS counts this day as "day 0," the next day as "day 1," etc. The last acceptable date and time for selection is day 3 at 23:59). After this three-day period, VBECS considers the unit to be expired.</p> <p>When the modification type is "volume reduce," VBECS records the discarded plasma volume on the unit record. Otherwise, the user may not edit the Discarded Plasma Volume field. The discarded plasma volume may not be "0" (zero).</p> <p> See Table 5 for alerts that may occur during this option.</p>
<p>7. Click <b>OK</b> to accept the unit for modification.</p>	<ul style="list-style-type: none"> <li>• When the modification type is "thaw" or "irradiate," allows the user to select additional units to process or to continue processing the selected units. Otherwise, go to Step 8.</li> </ul> <p><b>NOTES</b> _____</p> <p>The user may process multiple products in a batch when the units are thawed or irradiated. The creation date and time apply to all units in the batch.</p> <p>VBECS records the codes/costs assigned or unassigned with a date/time stamp and the user ID and user name on any reports of the person creating the update.</p>

User Action	VBECS
<p>8. Click the <b>Weld Complete</b> or <b>Weld Incomplete</b> radio button to indicate the integrity of the SCD weld, when indicated.</p> <p>Click the <b>Container, Supplies, Equipment,</b> or <b>SCD Wafer</b> tab and enter another supply and/or equipment lot or ID number (Figure 92).</p>	<ul style="list-style-type: none"> <li>Calculates and displays a default expiration date and time of the target unit based on rules specific to the original blood product, the modification type, and processing conditions (open, closed); prompts the user to accept the date and time; and allows the user to edit them.</li> </ul> <p><b>NOTES</b> _____</p> <p>VBECS maintains lot and ID numbers entered, the new container lot number in the unit modification record, and the historic record of the target unit.</p> <p>VBECS checks the inventory system for the supply name, lot numbers, expiration date, and a quantity greater than “0” (zero). VBECS does not allow the user to select supplies with an “unsatisfactory” value in the Inspection field. When the user enters an unsatisfactory supply lot number, VBECS warns the user. The user must select a different supply item.</p> <p> See Table 5 for alerts that may occur during this option.</p>
<p>9. Select a target product type from the drop-down list in the Product field.</p>	<ul style="list-style-type: none"> <li>Displays target component types for the modification, conditions selected, and the original unit product type, and allows the user to select a single product type.</li> <li>Verifies that an active target component was selected and warns the user.</li> <li>Allows the user to enter lot and ID numbers (with auto-completion) or select multiple items from the supply and equipment lists.</li> <li>Warns that the target unit is not active and allows the user to activate the target component.</li> </ul> <p><b>NOTES</b> _____</p> <p>The user must select a modification method (open, closed, Sterile Connection Device) for modification types, as described in Appendix B: Table 22: Allowable Product Modifications by Original Product Type.</p> <p>VBECS displays available target component types based on ISBT 128 modification rules and site parameters for the division. When no target is found, VBECS indicates this to the user and prevents the modification from being saved.</p> <p>When the user selects a target component type not marked as active, VBECS warns the user, allows him to mark the target component as active, and returns him to this step in the modification process. Otherwise, VBECS returns the user to this step and allows him to select another target component type.</p> <p>When the original unit type did not require ABO/Rh confirmation, the target unit, when created, does not either. The target unit is immediately available for assignment, crossmatch, issue, or transfusion on completion of the modification.</p> <p>When the original (parent) unit type required ABO/Rh confirmation, the target (child) unit, when created, does too.</p>

User Action	VBECS
	<ul style="list-style-type: none"> <li>○ If the target is not inheriting ABORh confirmation testing, then the target unit's status is Limited.</li> <li>○ If the target is inheriting ABORh confirmation testing, then the target unit is immediately available for assignment, crossmatch, issuance, or transfusion upon completion of modification.</li> </ul> <p>When the original unit is assigned or crossmatched to a patient at the time of modification, VBECS notifies the user and asks whether he wishes to automatically assign the modified units to the patient. <b>Yes</b> (default) applies the unit's assignment or crossmatch data to the target component. <b>No</b> does not apply the assignment to the target, and changes the target unit status to "available." The original unit retains patient assignment in both cases.</p> <p>The user may select the target product code from available targets by scanning.</p> <p>When the Blood Group (V0076) attribute group, is present in the original it is required in the target but is not used by VBECS.</p> <p>When a unit is modified, the processing fee as set in Configure Division is added to the cost of a modified unit target. When split, the processing fee is divided and a portion is added to each target unit.</p> <p> See Table 5 for alerts that may occur during this option.</p>
10. Accept or edit the expiration date.	<ul style="list-style-type: none"> <li>● Verifies the validity of the expiration date and warns the user, if indicated.</li> </ul> <p><b>NOTES</b> _____</p> <p>VBECS calculates an expiration date for the target component type and the system integrity (Open or Closed). The system will calculate an expiration date for the target component based on the following. (Due to the nature of the process, the calculated expiration date will greatly exceed the original expiration date of the original unit for Rejuvenate and Freeze, and will supersede the unit's original expiration date.)</p> <p>Expiration Date is the shortest of:</p> <ul style="list-style-type: none"> <li>● Original unit's expiration date/time (For Rejuvenate and Freeze, the Original unit's expiration is not factored in the calculation)</li> <li>● If Expiration hours is &gt; 0, the Modification date/time + Expiration hours**</li> <li>● Modification date/time + Target blood product's maximum storage time (MST)**</li> </ul> <p>**If Expiration hours or Target blood product's MST is greater than (&gt;) 24 hours: divide the hours by 24 to get the number of days (no rounding); add the number of days to the Modification date/time and the time will be 23:59.</p>

User Action	VBECS
<p>11. Review the displayed unit data and click <b>OK</b> to confirm the modification.</p> <p>If the unit has been relabeled, click <b>Yes</b> to verify the label (Figure 93).</p> <p>Click <b>No</b> to cancel and continue at Step 14.</p>	<ul style="list-style-type: none"> <li>When the original unit is currently assigned to a patient, notifies the user and asks whether he wishes to transfer the status to the target unit.</li> </ul>
<p>12. Scan the label barcodes to verify the target labels, based on local practice and labeling type of the modified unit.</p> <p>Enter a comment to complete the verification. (This step is optional.)</p>	<ul style="list-style-type: none"> <li>Displays the label verification window.</li> <li>Indicates whether the scanned barcodes match those in the label.</li> <li>For each target created, system prompts user to indicate readiness to verify the labeling of the modified product.</li> <li>No new labels applied to modified product. Proceed to save the unit.</li> <li>Displays a screen to accommodate data entry of the modified unit's: <ul style="list-style-type: none"> <li>ABO/Rh</li> <li>Product Code</li> <li>Expiration Date</li> </ul> </li> <li>This step repeats until all targets have been labeled and/or are ready for continued processing.</li> </ul> <p><b>NOTES</b> _____</p> <p>The user may select "Verify modified unit label" to verify the modified unit label. When the user does not verify the label, he must enter a comment and then click <b>Cancel</b> to exit.</p> <p>When the user does not select "Verify modified unit label," he may continue at Step 14. VBECS bypasses the label verification and does not create an Exception Report.</p> <p>When a component is modified and a new label is applied, American Association of Blood Banks (AABB) standards require the labeling process to include a method to ensure the correctness of the unit number, ABO/Rh, expiration date, and component label.</p> <p>To verify the label, VBECS compares the entered data with the data of the target unit from the database. The user may scan or enter:</p> <ul style="list-style-type: none"> <li>Unit ABO/Rh Label (scan or manual entry)</li> <li>Unit Product Code (scan or manual entry)</li> <li>Unit Expiration Date (scan or manual entry)</li> </ul> <p>If all comparisons are okay, VBECS displays a message stating that label verification is complete and asks whether the user wishes to continue.</p> <p>If there is not a match, VBECS provides an audible alert and displays an error message stating that &lt;data element&gt; does NOT match &lt;data element&gt; of record and asks the user to verify labeling and reenter, or select <b>Continue</b>.</p>

User Action	VBECS
	<p>If there is a null or no user entry for one of the fields, VBECS displays an error message stating that &lt;data element&gt; was not entered and asks the user to enter a value or select <b>Continue</b>.</p> <p>If checks were either invalid or incomplete, VBECS will not allow unit issue to this or any other patient. VBECS displays a warning message: "Unit is unacceptable for transfusion. Releasing all patient assignments and quarantining the unit." VBECS clears the unit data and allows for entry of a different unit number. When the user selects <b>Continue</b>, a comment is required and details are captured in an exception report. Details in the exception report include:</p> <ul style="list-style-type: none"> <li>• Transaction type: Target Product Label Failed checks</li> <li>• Type of modification: &lt;insert type defined by the transaction&gt;</li> <li>• Target Unit ID</li> <li>• Target Unit ABO/Rh</li> <li>• Target Unit Product Code</li> <li>• Target Unit Product (Short) Name</li> <li>• Target Expiration Date</li> <li>• Date/Time of Modification (transaction)</li> <li>• Data Field User Entered Value or "no entry" if null</li> <li>• Data Field Correct or Expected Value</li> <li>• Required Comment ("unit modification" context)</li> <li>• Modifying Tech ID, print Tech name on report</li> <li>• Login Tech ID, print Tech name on report</li> <li>• Division</li> </ul>
<p>13. Scan or enter each label on the target unit.</p>	<ul style="list-style-type: none"> <li>• Compares the data entered in each field to the database.</li> <li>• Asks whether the user wishes to save the data. When the user confirms that the data are correct, notifies him that the label verification is complete.</li> <li>• Records the modified or restriction-released unit relabeling data as part of the unit record and stores the data for inclusion in the unit history report.</li> <li>• When the user confirms that the comparisons are correct, notifies him and asks whether he wishes to continue.</li> </ul> <p><b>NOTES</b> _____</p> <p>When there is no previous entry for a field, VBECS notifies the user and instructs him to enter a value or select <b>Cancel</b>.</p> <p>VBECS requires a comment when the user selects <b>Cancel</b>, and captures details for inclusion in an Exception Report (exception type: target product label failed checks).</p>
<p>14. Click <b>OK</b> to confirm the save.</p>	<ul style="list-style-type: none"> <li>• Saves the new modified unit data.</li> <li>• Updates the status of the original unit to "modified."</li> <li>• Records the unit ID of the target unit on the record of the original unit.</li> <li>• Prompts the user to modify another unit.</li> </ul> <p><b>NOTES</b> _____</p> <p>The target unit automatically inherits original unit data:</p>

User Action	VBECS
	<ul style="list-style-type: none"> <li>• Additional unit testing (CMV, Sickle Cell, antigen typings, = &gt; 2 mL RBC contamination)</li> <li>• Unit ABO/Rh (from login)</li> <li>• ABO/Rh confirmation testing information (when performed)</li> <li>• Unit ID</li> <li>• Unit Volume (exceptions – target unit created as a result of Split or Volume Reduce does not inherit original unit volume. Target unit volume must be manually entered for volume reduced units)</li> <li>• Donation type</li> <li>• Disease marker testing information, when autologous.</li> <li>• Restricted for patient name, ID.</li> <li>• Current status</li> <li>• Biohazard Indicator</li> <li>• Quarantine Indicator</li> <li>• Division</li> </ul>
15. Repeat Steps 3–14 to modify another unit or exit.	

**Figure 91: Modify Unit(s)**

Figure 92: Split/Divide Unit

**VBECs - Split/Divide Unit** Split/Divide Unit

---

Split\*  Pediatric Split   mL

Modification Method:   Weld\*  Weld Complete  Weld Incomplete  
 Modification Date:

---

**Original Unit**

Unit ID:  Donation Type:   
 Product:  Patient Name:   
 ABO/Rh:  Volume (ml):  Patient ID:   
 Expiration Date:  Disease Marker Testing:   
 Unit Status:  Restricted For:

---

**Target Unit 1** | **Target Unit 2**

Unit ID:  Donation Type:   
 Product:  Patient Name:   
 ABO/Rh:  Volume (ml)\*:  Patient ID:   
 Expiration Date\*:  Disease Marker Testing:   
 Unit Status:  Restricted For:

---

Lot Number	Supply Type	Manufacturer	Expires	<input type="button" value="Add Container*"/>
553466	Clean Containers	Super Supplier	03/02/2006	

Original Container

---

\* Required Field

Figure 93: Verify Printed Label

## Alerts

Table 5: Alerts That May Occur in Modify Units

Step	Alerts
6	<p>VBECS compares the original unit's expiration date and time to those of the modification to determine whether the selected unit was expired at the time of modification. When the user is a Blood Bank Technologist and the unit selected for modification expired, VBECS emits an audible alert, displays an error message, clears the selected unit, and allows the user to select another unit for modification.</p> <p>➤➤ When the user is an Enhanced Technologist and the unit selected for modification is expired, VBECS emits an audible alert, notifies the user, and asks him to verify to continue, or cancel to correct the entry data. <b>Override</b> requires a comment and VBECS captures details for inclusion in an Exception Report (exception type: expires unit modified). <b>Cancel</b> clears the selected unit and allows the user to select another unit for modification.</p>
6	<p>When ABO/Rh confirmation testing is required but not performed, VBECS notifies the user that testing must be completed before issuing a target unit. The user may continue with the modification.</p>
6	<p>When a unit has an indicator of "quarantined," VBECS notifies the user that it must be released before processing may continue, clears the selected unit, and allows the user to select another unit.</p> <p>➤➤ When the user is an Enhanced Technologist and a unit has an indicator of "quarantined," VBECS notifies the user that it must be released before processing may continue and asks whether the user wishes to release the unit. <b>Cancel</b> clears the selected unit and allows the user to select another unit. <b>Override</b> releases the unit through Discard or Quarantine.</p>
6	<p>When the selected unit has a positive biohazard indicator, VBECS will not allow modification unless the user role is Enhanced Technologist, VBECS security levels 2 and above. When the user role is Technologist, VBECS security level 1, the system emits an audible alert and displays a warning</p>



Step	Alerts
	<p>message stating that the user does not have the proper security to perform this function. The system does not allow an override.</p> <p>When the user role is Enhanced Technologist, VBECS security levels 2 and above, VBECS provides an audible alert and displays a warning message: "&lt;Unit ID, Product Name&gt; is Biohazardous. Do you wish to continue with the modification?" If the user responds "No," the unit is cleared from the screen and a new unit can be selected. If the user responds "Yes," a comment (context: Unit modification) is required. Details of this override are captured for inclusion in an exception report.</p>
8	<p>When a supply item is expired, VBECS emits an audible alert, warns the user, and asks whether the user still wishes to use the item. <b>Yes</b> requires a comment and VBECS captures details for inclusion in an Exception Report (exception type: expires supply used in unit modification). <b>No</b> requires the user to select another supply lot number.</p>
10	<p>When the user extends the system-recommended expiration date and time, VBECS warns the user, asks him to reenter the unit expiration date, and redisplay the original entry. The user may enter another expiration date and time for the target unit.</p> <p>➤➤ When an Enhanced Technologist extends the target unit expiration date and time past the system-recommended expiration date and time, VBECS emits an audible alert, warns the user, and asks whether he wishes to continue. <b>Cancel</b> restores the expiration date. <b>Override</b> allows the user to continue with this date, requires a comment, and captures details for inclusion in an Exception Report (exception type: target product outdate extended). Information about this transaction and the comment are queued for an exception report.</p>
13	<p>When there is no match, VBECS emits an audible alert, notifies the user, and instructs him to verify the labeling and reenter label data or select <b>Cancel</b>.</p>

## Modify Units: Split a Unit

The user selects the unit to be split (divided) for transfusion.

### Assumptions

- See Modify Units.

### Outcome

- See Modify Units.

### Limitations and Restrictions

- A unit previously split in VBECS cannot be split again. Only an ISBT unit that was split outside of VBECS can be split again.
- A unit pooled in VBECS may not be split.
- A unit that has a division code present upon receipt in incoming shipment may be split in VBECS.
- Weld controls are disabled after the user makes their initial selection.

### Additional Information

- To reverse other modifications, remove the “modified” status from each unit through Supervisor Menu, Remove Final Status.

### User Roles with Access to This Option

All users

## Modify Units: Split a Unit

The user selects the unit to be split for transfusion. VBECS automatically applies directed and autologous restrictions to aliquots created. The user may or may not maintain patient assignments (usually, these units are crossmatched and available for the patient before the split is made). This section lists only rules unique to the division of a blood unit.

User Action	VBECS
1. Select <b>Blood Units</b> from the main menu.  Select <b>Modify Units</b> .	<ul style="list-style-type: none"><li>• Displays options for processing blood units.</li><li>• Displays valid modification types for selection.</li></ul>
2. Click the <b>Split/Divide</b> radio button.  Click <b>OK</b> .	<ul style="list-style-type: none"><li>• Allows the user to search for a unit.</li></ul>
3. Select a unit and click <b>OK</b> .  Click <b>Select</b> to confirm the unit selected.	<ul style="list-style-type: none"><li>• Displays data for the original and target units.</li></ul>

User Action	VBECS
<p>4. Edit the number of adult splits to be created, if required (Figure 92).</p> <p>Click <b>Split*</b> to verify the number of splits to be created and add the appropriate number of target unit tabs.</p>	<ul style="list-style-type: none"> <li>• Displays the number of splits to be created (default: 2) for verification.</li> <li>• Requires entry of the number of splits to be created.</li> </ul> <p><b>NOTES</b> _____</p> <p>The user may split a unit into 2 to 26 subunits (or however many available alphabetic characters remain) until the volume of the original container is consumed. The system will calculate the patient charge of each split (fraction) as its portion of the original unit cost in U.S. dollars (\$0,000.00).</p> <p>When split is selected, users must indicate the number of split units to be created if other than the default number of 2.</p>
<p>5. Select or confirm the desired target component type.</p> <p>For a Sterile Connection Device modification method, click the <b>Weld Complete</b> or <b>Weld Incomplete</b> radio button to indicate the condition of the weld.</p> <p>Click the <b>Container, Supplies, Equipment,</b> or <b>SCD Wafer</b> tab and enter another supply and/or equipment lot or ID number.</p>	<ul style="list-style-type: none"> <li>• Allows the user to enter lot and ID numbers or select multiple items from the supply and equipment lists.</li> </ul> <p><b>NOTES</b> _____</p> <p>When multiple aliquot/split units are created, the user must associate a container lot number with each unit created. "Original container" may serve as the lot number for one of the aliquots created.</p> <p>The original blood collection facility must keep a record of the original container lot numbers so that, when a lot number is recalled, it is known which patient was exposed to it.</p> <p>The user may select the target product code from available targets by scanning.</p> <p>For THAW modification type:</p> <ul style="list-style-type: none"> <li>• There is no attribute assigned for thaw.</li> <li>• The product class, modifier and Blood Core (V001) change.</li> <li>• The system integrity should not change (V0012)</li> <li>• Maintain original attribute groups to the target; do not add any attribute values.</li> <li>• Maintain the original anticoagulant or NS.</li> <li>• Temperature must change to a non-frozen temperature</li> <li>• Target volume may be XX, or match the volume of the parent</li> </ul>
<p>6. Accept or edit the displayed expiration date.</p>	<ul style="list-style-type: none"> <li>• Displays the expiration date.</li> <li>• Calculates and displays the volume of each split unit.</li> </ul> <p><b>NOTES</b> _____</p> <p>VBECS calculates and displays default volumes for each target split (does not include pediatric aliquots) based on the volume of the original unit and the number of splits being created, consuming the total volume of the original unit. The system will restrict the user from entering a value between 1 and 2000.</p> <p>The system will split the volume evenly among the splits and add the remainder to the first split.</p>

User Action	VBECS
<p>7. Accept or edit the displayed unit volumes.</p>	<ul style="list-style-type: none"> <li>• Displays the original and target unit data for review.</li> </ul> <p><b>NOTES</b> _____</p> <p style="padding-left: 40px;">The user may not edit original unit data.</p> <p><b>For Split modification type:</b></p> <ul style="list-style-type: none"> <li>• Do not allow splitting frozen products.</li> <li>• There is no attribute assigned for split.</li> <li>• Maintain original product type attributes to the target.</li> </ul>
<p>8. Review the displayed data for the target units and confirm the modification type.</p>	<ul style="list-style-type: none"> <li>• Displays the target unit's ID number.</li> </ul> <p><b>NOTES</b> _____</p> <ul style="list-style-type: none"> <li>• ISBT 128: the unit ID, five-digit product description code, and sixth-character donation code remain the same. Characters 7 and 8 of the ISBT 128 product code barcode reflect the aliquot level: when a previously undivided unit is split, character 7 of the unique eight-digit ISBT 128 product code is incremented alphabetically in uppercase letters, resulting in characters 7 and 8: A0, B0, C0, D0 ... Z0.</li> </ul>
<p>9. Click <b>OK</b> to save.</p> <p>Click <b>Yes</b> to verify the label.</p>	<ul style="list-style-type: none"> <li>• Prompts the user to save the data, including patient assignment and restriction information, if any.</li> <li>• Saves aliquots created from the modified unit.</li> <li>• Records the unit ID of the target unit on the record of the original unit.</li> <li>• Saves only the technologist name and the date and time the label was verified. When the user indicates that a new label applied fails to complete the verification process, releases the assignment, quarantines the unit, and captures details for inclusion in an Exception Report (exception type: target product label failed checks).</li> </ul>
<p>10. Click <b>Yes</b> to save.</p> <p>Repeat Steps 2–9 to split additional units, or return to Step 1 to use a different modification method on another unit.</p>	<ul style="list-style-type: none"> <li>• Prompts the user to split or modify another unit.</li> </ul>
<p>11. Exit.</p>	

## Modify Units: Pool Units

The user selects units to be pooled.

### Assumptions

- See Modify Units.

### Outcome

- See Modify Units.

### Limitations and Restrictions

- VBECS allows modification of units only in limited, assigned, crossmatched, and available statuses.
- When a user adds a biohazardous unit to a pool, VBECS does not designate the pool as biohazardous on the Blood Transfusion Record Form (BTRF) or Caution Tag.
- Weld controls are disabled after the user makes their initial selection.

### Additional Information

- When autologous or directed units are pooled, the “restricted for” patient must be the same for all units. VBECS assigns the same donation type and applies the “restricted for” patient data to the pool.
- To reverse the pooling of units:
  - Inactivate the target through Edit Unit Information.
  - Remove the “modified” status from each unit in the pool through Remove Final Status.
- To edit the number of units in an existing pool, use Add/Remove Units from a Pool.

### User Roles with Access to This Option

All users

## Modify Units: Pool Units


When the pooling is in process or almost complete, the user records it in VBECS. Data entry may also be completed retrospectively. The user pools units and enters the ID and product code of the original unit and the IDs and product codes for each unit added to the pool.


VBECS records the date and time the units are pooled, uses the date and time to calculate a new expiration date for the pool, and assigns a unit ID and ABO/Rh, as defined by system rules. When an original unit in a pool was previously assigned to a patient, VBECS asks the user whether the entire pool is to be assigned to that patient. VBECS automatically assigns autologous and directed patient restriction data from an individual unit to the pool. VBECS determines the volume of the pool based on the sum of the volumes of the units in the pool.

The new, pooled unit needs a full set of unit properties, some of which were defined by the original units' properties. VBECS assigns new properties such as volume, date and time created, container lot number, and expiration date to the target.

VBECS uses system rules to check pertinent blood bank requirements associated with pooling and displays appropriate target unit information for review. The user reviews and edits the information, as needed, and completes the transaction. VBECS updates internal records, including the collection of workload and unit statuses, for each unit involved.

The user may now assign the units or print the BTRF and Caution Tag, and notify appropriate personnel that the ordered units are ready for issue.

User Action	VBECS
<p>1. Select <b>Blood Units</b> from the main menu.</p> <p>Select <b>Modify Units</b>.</p>	<ul style="list-style-type: none"> <li>• Displays options for processing blood units.</li> <li>• Displays valid modification types for selection.</li> </ul> <p><b>NOTES</b> _____</p> <p>The &gt;&gt; Enhanced Technologist or above is authorized to modify expired blood products.</p>
<p>2. Click the <b>Pool</b> or <b>Thaw/Pool Cryo</b> radio button.</p> <p>Click <b>OK</b>.</p>	<ul style="list-style-type: none"> <li>• Allows the user to search for a unit.</li> </ul>
<p>3. Scan, enter, or select the ID and product code of the most restricted (e.g., biohazardous, autologous, directed) unit to be pooled.</p>	<ul style="list-style-type: none"> <li>• Displays current data for the selected unit.</li> </ul> <p><b>NOTES</b> _____</p> <p>VBECS applies the restriction of the first unit selected to the pool.</p> <p> Based on the product code, VBECS determines the component class of each selected unit to verify that the component class of the selected unit matches that of the first unit selected for the pool (only units from the same component class may be pooled). When they do not match, VBECS warns the user and clears the entry. The user may enter another unit.</p> <p>If one unit containing = &gt; 2 mL of red blood cell contamination is added to a pool, the target is marked as containing = &gt; 2 mL of red blood cell contamination.</p>
<p>4. Enter the ID and product code of another unit of the same donation type for inclusion in the pool.</p>	<ul style="list-style-type: none"> <li>• May warn the user and asks whether to release the unit from "restricted for patient."</li> </ul> <p><b>NOTES</b> _____</p> <p>When the donation type of a unit selected for inclusion in a pool is restricted, the user may release the directed unit to the general blood supply, if indicated, and complete the pooling process.</p> <p>When the first unit entered for a pool is labeled in ISBT 128, VBECS determines the unit ID of the pool created from ISBT-labeled units using the country and ICCBBA-assigned facility code (division preparing the pool), two-number year, and six-number sequential number of the pool event at the division, and displays it for acceptance only.</p> <p>If one unit containing = &gt; 2 mL of red blood cell contamination is added to a pool, the target is marked as containing = &gt; 2 mL of red blood cell contamination.</p>

User Action	VBECS
<p>5. Accept the previously selected unit and repeat Step 4 to select another unit to add to the pool.</p>	<ul style="list-style-type: none"> <li>• Displays all data for all original units.</li> </ul> <p><b>NOTES</b> _____</p> <p>VBECS tallies the number of units pooled for this transaction. VBECS will not allow the user to edit this number. The user may not edit this number. VBECS calculates the pooled unit's cost and passes that amount to the patient billing system in U.S. currency.</p> <p>VBECS does not allow a unit to be entered twice in a pool. A split unit created in VBECS may not be pooled. A unit that has a division code (7<sup>th</sup>/8<sup>th</sup> characters in the ISBT Product code) present upon receipt in incoming shipment may be pooled.</p>
<p>6. Accept the displayed units' data for the pool.</p>	<ul style="list-style-type: none"> <li>• Displays target component types for the modification, conditions selected, and the original unit product type, and allows the user to select a single modification type.</li> </ul>
<p>7. Select or confirm the desired target component type.</p> <p>Click the <b>Container, Supplies, Equipment,</b> or <b>SCD Wafer</b> tab and enter another supply and/or equipment lot or ID number.</p>	<ul style="list-style-type: none"> <li>• Verifies that an active target component was selected. Requires activation, as needed.</li> <li>• Allows the user to enter lot and ID numbers (with auto-completion) or select multiple items from the supply and equipment lists.</li> </ul>
<p>8. Accept the displayed expiration date or edit it.</p>	<ul style="list-style-type: none"> <li>• Calculates and displays the expiration date of the pooled unit based on rules specific to the participant blood product.</li> <li>• Verifies the validity of the expiration date and warns the user, when indicated.</li> </ul> <p><b>NOTES</b> _____</p> <p>When the method selected is "Thaw/Pool Cryo", The system shall 'thaw' frozen components automatically when they are being pooled.</p> <p>The workload for thawing each unit in the pool is collected in addition to the pool workload.</p> <p>The new expiration date of the thawed unit is determined by the date/time of pooling.</p> <p>VBECS calculates an expiration date for the pool based on the product type, the date and time created, and the expiration dates and times of each pool participant. When one or more participants in the pool have an expiration date that is earlier than the calculated expiration date, VBECS uses the earlier date as the default expiration date for the pool.</p> <p> For each unit in the pool that was previously assigned or crossmatched to a patient, VBECS warns the user and asks whether he wishes to assign the entire pool to that patient. <b>Yes</b> applies the patient-assigned data to the pool. <b>No</b> releases the unit from the patient. VBECS offers the user the ability to carry over any past assignment.BR</p>

User Action	VBECS
<p>9. Click <b>OK</b> to save.</p>	<ul style="list-style-type: none"> <li>• Calculates the volume, total number of participants, ID, and ABO/Rh for the pool. Applies CMV negative status to the pool, when indicated.</li> <li>• Determines whether any pool participants were previously assigned to a patient and queries the user to maintain the association, when indicated.</li> </ul> <p><b>NOTES</b> _____</p> <p>When the ABO/Rh of each unit in a pool is identical, VBECS assigns that ABO/Rh to the pool. When the ABO and/or Rh of the units in a pool are not identical, VBECS assigns “mixed” as the ABO and/or Rh group and type.</p> <p>VBECS determines the volume of the pooled unit based on the sum of the individual volumes of the original units included in the pool and displays that as the default. VBECS will restrict the user to a value between 1 and 2,000 mL.</p> <p>If all of the parent units selected for a pool are CMV negative, the child (pooled unit) will also be labeled CMV negative by the system. If any one unit is not CMV negative, the pool cannot be labeled CMV negative.</p> <p>A pooled unit cannot be labeled as “Pre-treated to Prevent GVHD” unless all of the parent (original) units used in creating that pool have at least one attribute form from attribute group V0013 indicating that the unit is irradiated or the attribute form indicating the unit was psoralen treated, V0021005. There is no override.</p> <p>A pooled unit cannot be labeled as “Leukocyte Reduced” unless all of the parent (original) units used in creating that pool are have at least one of the attribute forms from attribute group V0014, indicating that the unit has an altered “Residual Leukocyte Content”.</p> <p>Example, a pool is created from 5 units, if any one of the units is not irradiated or psoralen related, then the target cannot be labeled as Pre-treated to Prevent GVHD”.</p> <p>Example, a pool is created from 5 units, if any one of the units does not contain at least one of the attribute forms from the attribute group indicating that the unit has an altered “Residual Leukocyte Content” then the target is cannot be labeled as “Leukocyte Reduced”</p> <p>If one biohazardous unit is added to a pool, the target is marked biohazardous.</p>



User Action	VBECS
<p>10. Click <b>Yes</b> to verify the label.</p> <p>Click <b>OK</b> to save.</p>	<ul style="list-style-type: none"> <li>• Saves the new pooled unit.</li> <li>• Updates the status of each participant unit to “modified.”</li> <li>• Records the unit ID of the target unit on the record of the original unit.</li> <li>• Prompts the user to modify another unit.</li> <li>• Saves only the technologist name and the date and time the label was verified. When the user indicates that a new label applied fails to complete the verification process, releases the assignment, quarantines the unit, and captures details for inclusion in an Exception Report (exception type: target product label failed checks).</li> </ul>
<p>11. Repeat Steps 2–10 to modify another unit, or exit.</p>	

## Add and/or Remove Units from a Pool

The user selects a pooled unit to add and/or remove some of the participants.

### Assumptions

- See Modify Units.

### Outcome

- See Modify Units.

### Limitations and Restrictions

- See Modify Units.

### Additional Information


- None.

### User Roles with Access to This Option

- Enhanced Technologist (add/remove prior to patient associations)
- Traditional Supervisor (add/remove with patient associations)

## Add/Remove Units from a Pool

The user adds or deletes units to or from a pool.

User Action	VBECS
1. Select <b>Blood Units</b> from the main menu.  Select <b>Modify Units</b> .	<ul style="list-style-type: none"> <li>• Displays options for processing blood units.</li> <li>• Displays valid modification types and allows the user to select an option.</li> </ul>
2. Click the <b>Add/Remove Units from a Pool</b> radio button.  Click <b>OK</b> .	<ul style="list-style-type: none"> <li>• Allows the user to search for a pooled unit.</li> </ul>
3. Scan, enter, or select the ID and product code for a pooled unit.	<ul style="list-style-type: none"> <li>• Displays current data for the selected unit.</li> </ul> <p><b>NOTES</b> _____</p> <p>The user must select a unit that was previously pooled within the user's division.</p> <p>The user may edit a pooled unit only once.</p> <p> VBECS does not list units (for selection) with a final status. When a user enters a unit with a final status, VBECS alerts the user and clears the screen. The user may select another unit.</p>
4. Review the existing pooled unit data.  Select one or more units to add to the pool.	<ul style="list-style-type: none"> <li>• Allows the user to select one or more units to add to or remove from the pool.</li> </ul> <p><b>NOTES</b> _____</p> <p>The user may not remove all of the original units.</p>


User Action	VBECS
<p>Clear one or more check boxes to remove units from the pool.</p>	<p>The user may add only units with the same product and donation types as those of the previously pooled unit.</p> <p>Properties that may be changed based on the edit of the target pool are:</p> <ul style="list-style-type: none"> <li>• Patient Assignment</li> <li>• ABO/RH</li> <li>• Expiration date</li> <li>• Volume</li> <li>• CMV Negative indicator</li> <li>• SC Negative indicator</li> <li>• Biohazardous indicator</li> </ul> <p>Required crossmatch if = &gt; 2 mL contamination changes</p>
<p>5. Confirm to add the selected units to or remove them from the pool (Figure 94).</p>	<ul style="list-style-type: none"> <li>• Displays the updated pooled unit data.</li> <li>• Requires the user to confirm to continue.</li> <li>• Requires a comment.</li> </ul> <p><b>NOTES</b> _____</p> <p>All units in a pool must be of the same restriction type (allogeneic, autologous, directed). The specific donation type within the restriction type of the first unit selected for the pool is assigned to the target.</p> <p>For each individual unit removed from a pool, the status is available.</p> <p>When the unit's original expiration date and time is in the future, VBECS assigns units removed from the pool a status of "available." When the unit's original expiration date and time is in the past, VBECS assigns units removed from the pool a status of "available."</p> <p> When the user adds a unit to a pool and the unit expires before the pool expires, VBECS emits an audible alert, warns the user, and asks the user to continue or cancel. <b>Override</b> requires a comment and VBECS captures details for inclusion in an Exception Report (exception type: expired unit modified). <b>Cancel</b> clears the selected unit data and allows the user to select another unit to modify.</p>
<p>6. Review the entered data and save the updated pool record.</p>	<ul style="list-style-type: none"> <li>• Saves the modified pooled unit. Updates the status of each added unit to "modified." Updates the status of each removed unit. Records the unit ID of the target unit on the record of the original unit.</li> </ul>
<p>7. Repeat Steps 1–6 to modify another unit or exit.</p>	<ul style="list-style-type: none"> <li>• Allows the user to modify another unit.</li> </ul>

Figure 94: Add/Remove Units from Pool

**VBECS - Add/Remove Units from Pool**

**Add/Remove Unit(s) from Pool**

Pooled Unit

Unit ID: 589A6061    Expiration Date: 02/02/2006 18:36    ABO/Rh: O Neg

Product: 12091 - PLT Pool    Modification Date: 02/02/2006 14:36    Modified By: vhaishvbecs1

Unit Search

Unit ID\* [ ]    Find

Product\* [ ]

Details

Donation Type [ ]

Date Received [ ]

Unit Status [ ]

Volume (mL) [ ]

Select    Clear

Unit Number    EXPIRES

See circular of information for indications, contraindications, cautions and methods of infusion.

This product may transmit infectious agents.

**Rx Only**

PROPERLY IDENTIFY INTENDED RECIPIENT

Selected Units

	Unit ID	Product Code	Product	ABO/Rh	Expires	Status	Donation	Date Received
<input checked="" type="checkbox"/>	4654681	12000	PLT	O Neg	02/04/2006 23:59	Modified	Volunteer Allogeneic Donor	01/31/2006 10:05
<input checked="" type="checkbox"/>	34631818	12000	PLT	O Neg	02/04/2006 23:59	Modified	Volunteer Allogeneic Donor	01/31/2006 10:05

\* Required Field

OK    Cancel

## Maintain Unit Records

### Edit Unit Information

The user may inactivate unit tests, correct a data entry error from login, and add information about the unit that was not recorded during login.

#### Assumptions

- The user is logged into the division that contains the unit record.
- The connection to VistA is active.

#### Outcome

- The unit's new or changed information was saved with its record.

#### Limitations and Restrictions

- VBECS does not limit the discarded plasma volume.
- If errors are found in the unit's product code, division or blood type at login, supplier, or donation type, the user must inactivate the unit and reenter it correctly.
- When unit does not have a patient association, editable fields are available to an >> Enhanced Technologist.
- Only units in a Limited or Available status may be edited.
- Inactivation of the unit record is required when the user needs to change the unit's information recorded at login: unit identification number, blood type, product code, unit expiration date, unit donation type, VA division or supplier.

#### Additional Information

- Units must be in "Limited" or "Available" status to have information inactivated regardless of historic patient associations.
- Crossmatches are considered a patient test and are inactivated in Invalidate Test Results.

#### User Roles with Access to This Option

- >> Enhanced Technologist

### Edit Unit Information

The user records additional unit information normally recorded during incoming shipment (login), i.e., CMV negative, sickle cell negative, and antigen typing provided by the supplier.

*To inactivate a unit, remove the final status, return the unit from issue, and release it from previous assignment. No changes may be made to units in a final status.*

User Action	VBECS
1. Select <b>Blood Units</b> from the main menu.  Select <b>Edit Unit Information</b> .	<ul style="list-style-type: none"><li>• Displays options for processing blood units.</li><li>• Displays options for selecting and editing a unit.</li></ul>

User Action	VBECs
<p>2. Scan or enter a unit ID and product code. May cancel and select a different unit.</p> <p>When the unit entered is in a final status, click <b>Clear</b> to clear the Unit ID and Product fields and enter another unit.</p>	<ul style="list-style-type: none"> <li>• Allows the user to scan or enter a unit ID and product code or select a unit.</li> <li>• Displays the editable unit information based on the user's security level.</li> </ul> <p><b>NOTES</b> _____</p> <p>VBECs does not allow selection of a unit when it:</p> <ul style="list-style-type: none"> <li>• Is currently assigned to a patient.</li> <li>• Has a final unit status.</li> <li>• Has a previous patient assignment record and the user is a Lead or Enhanced Technologist.</li> </ul> <p>There is no override. VBECs clears the screen and allows the user to select a new unit.</p>
<p>3. Review and enter or edit the data (Figure 95),</p> <p>or</p> <p>Go to Inactivate a Unit.</p>	<ul style="list-style-type: none"> <li>• Displays tabs to select and edit: <ul style="list-style-type: none"> <li>o Unit information</li> <li>o Unit ABO/Rh confirmation</li> <li>o Antigen typing</li> </ul> </li> <li>• Displays fields to add or inactivate unit information and an option to inactivate a unit based on the condition of the unit and the user's security level.</li> </ul> <p><b>NOTES</b> _____</p> <p>The user may add data to a field with no previous entry. When the blood product code within the ICCBBA component classes is RED BLOOD CELLS, WHOLE BLOOD, the <b>Technologist</b> and above may add unit information: sickle cell negative, CMV negative, red blood cell contamination of = &gt; 2 mL, and antigen typing to a unit when the unit has no patient assignments.</p> <p>Users may edit unit volume, add special testing and RBC antigens, mark a unit biohazardous, and change the patient restriction.</p> <p>When a blood product code in the RED BLOOD CELLS or WHOLE BLOOD component class is entered, VBECs enables data entry of sickle cell negative and antigen typing. The CMV negative data entry option is always enabled.</p> <p>An antigen type may be positive or negative for any antigen listed as a VBECs corresponding antigen in Appendix B: Table 14: Antibody and Antigen Table: Irregular Antibodies. An antigen may not be positive and negative for the same unit.</p> <p>Only unit antigen type results originally entered through Incoming Shipment are available for invalidation. An Enhanced Technologist must invalidate unit antigen type results originally entered through Unit Antigen Typing by selecting Inactivate Unit Testing.</p> <p>VBECs restricts the editing of disease marker testing to autologous units.</p>

User Action	VBECS
	<p>When changing the disease marker testing of an autologous unit from “NEG” to “POS” or “NFT (not fully tested)”, VBECS automatically enables the biohazard indicator.</p> <p>The user may not clear the biohazard indicator for a unit when:</p> <ul style="list-style-type: none"> <li>• The disease marker testing is POS or NFT,</li> <li>• The ISBT 128 Donation Type codes are “X, For Autologous Use Only, Biohazardous” or “3, For Directed Donor Use Only, Biohazardous,” or 6, Designated Donor Biohazardous”</li> <li>• The unit is quarantined.</li> </ul> <p>When editing the “restricted for” patient name for a unit, the user may change the name of a restricted patient but may not save the transaction without a new patient name for the restriction.</p> <p>When the user restricts a unit to a patient, the patient ID is the primary identifier of the intended recipient. When VBECS searches the VistA PATIENT File (#2) and finds a match, it populates the VBECS database with the selected patient name. When the intended recipient is not in the VistA database, VBECS warns the user, clears the screen, and does not allow the unit to be further processed.</p> <p>➤➤ The Enhanced Technologist may edit items associated with a unit and add the new data to the unit. VBECS displays only the latest entry for these fields to the Enhanced Technologist before the unit has any patient associations:</p> <ul style="list-style-type: none"> <li>• Restrictions</li> <li>• CMV negative</li> <li>• Sickle Cell negative</li> <li>• Biohazard indicator</li> <li>• Disease marker testing</li> <li>• Antigen typing entered at log-in</li> <li>• Unit volume</li> <li>• Discarded plasma volume, if one</li> <li>• Red blood cell contamination of equal to or greater than (= &gt;) 2 mL (non-RBC units)</li> </ul> <p>➤➤➤ The Traditional Supervisor may edit items associated with a unit and add the new data when the unit has a history of patient associations.</p>
<p>4. Select a test for inactivation (Figure 96).</p>	<ul style="list-style-type: none"> <li>• Displays the previously recorded results for the selected test.</li> <li>• Requires a comment to process the inactivation.</li> </ul> <p><b>NOTES</b> _____</p> <p>The user may view, not edit, previously entered serologic results and interpretations.</p> <p>When the user selects the unit antigen typing test, VBECS displays antigen typing reagent types and interpretations available for invalidation at the user’s division. Inherited testing is not available for invalidation.</p>

User Action	VBECS
	<p>When the user selects the ABO/Rh confirmation test, VBECS displays the unit confirmation test interpretations available for invalidation at the user's division.</p> <p>The system will check if the selected blood unit has current, valid ABO/Rh Confirmation or Antigen Typing associated with it. If the unit does have associated current, valid tests, the system does not allow the user to inactivate the unit and displays a warning message to the user: "Unit has current, valid tests, previous patient assignments, or crossmatches that must be invalidated before the unit can be inactivated." There is no override.</p>
<p>5. Verify the test to be inactivated and enter a comment.</p> <p>Click <b>OK</b> to confirm and exit.</p>	<ul style="list-style-type: none"> <li>• Displays "Test Result Invalidated" in the unit record for the selected test.</li> </ul> <p><b>NOTES</b> _____</p> <p>When the last (or only) ABO/Rh confirmation testing is invalidated, VBECS returns the unit to a limited status and requires an ABO/Rh confirmation test for Red Blood Cell contamination of equal to or greater than (<math>\geq</math>) 2 mL indicator.</p> <p>When a unit that requires ABO/Rh confirmation testing no longer has a valid test result in VBECS due to invalidation of previous unit tests, VBECS treats the unit as though no ABO/Rh confirmation testing was performed.</p> <p>VBECS flags invalidated test results as "Test Result Invalidated" and captures details for inclusion in an Exception Report.</p> <p>VBECS does not inactivate workload associated with the invalidated test.</p> <p>Replacement results are entered according to local policy.</p>
<p>6. Return to Step 1 to edit another unit.</p>	
<p>7. Click <b>OK</b> to save.</p>	<ul style="list-style-type: none"> <li>• Updates and saves the data.</li> </ul>



Figure 95: Edit Unit Information

00:00:07.3788946

**Edit Unit Information** TEST

**Unit Search**

Unit ID\*

Product\*

**Unit Information** | ABO/Rh Confirmation | Antigen Typing

**Volume**

Unit Volume (mL)

Discarded Plasma Volume (mL)

**Donation Details**

Donation Type

Disease Marker Testing

**Special Testing**

=>2mL RBC Contamination

CMV Negative

Sickle Cell Negative

**RBC Antigens**

C NEG

C POS

E NEG

E POS

c NEG

Biohazardous?

**Restricted For**

Patient

\* Required Field

Figure 96: Edit Unit Information Invalidate Test

### Inactivate a Unit

User Action	VBECS
<ol style="list-style-type: none"> <li>1. Select <b>Blood Units</b> from the main menu.  Select <b>Edit Unit Information</b>.</li> </ol>	<ul style="list-style-type: none"> <li>• Displays options for processing blood units.</li> <li>• Displays options for selecting and editing a unit.</li> </ul>
<ol style="list-style-type: none"> <li>2. Scan or enter a unit ID and product code. May cancel and select a different unit.</li> </ol>	<ul style="list-style-type: none"> <li>• Allows the user to scan or enter a unit ID and product code or select a unit.</li> <li>• Displays the editable unit information based on the user's security level.</li> </ul> <p><b>NOTES</b> _____</p> <p>VBECS does not allow selection of a unit when it:</p> <ul style="list-style-type: none"> <li>• Is currently assigned to a patient.</li> <li>• Has a final unit status.</li> <li>• Has a previous patient assignment record and the user is a Lead or Enhanced Technologist.</li> </ul> <p>There is no override. VBECS clears the screen and allows the user to select a new unit.</p>

User Action	VBECS
<p>3. Click <b>Inactivate Unit</b>, as desired.</p>	<ul style="list-style-type: none"> <li>• Displays fields to add or inactivate unit information and an option to inactivate a unit, based on the condition of the unit and the user's security level.</li> <li>• Displays unit data.</li> <li>• Requires a comment to process the unit inactivation.</li> </ul> <p><b>NOTES</b> _____</p> <p>➤➤➤➤ When the Traditional Supervisor attempts to inactivate a unit's record, VBECS warns the user. A blood unit's record cannot be destroyed or removed from the database for any reason. The system displays a warning message with no override to all users: "Inactivated unit records cannot be destroyed."</p> <p>An inactivated unit is not accessible. The record is available in the Audit Trail and Unit History Reports and in division statistical and informational reports. When VBECS inactivates a unit, it also inactivates costs previously associated with that unit.</p> <p>The user may inactivate the unit with prior but no current patient assignments. VBECS requires a comment to explain the inactivation.</p> <p>Unit inactivation appears on the Audit Trail Report.</p>
<p>4. Enter a comment.</p> <p>Click <b>OK</b> to save.</p>	<ul style="list-style-type: none"> <li>• Marks the selected unit record "inactive."</li> <li>• Saves the record.</li> </ul>

## Edit Financial Data

The user enters and/or edits charges associated with a unit after it is logged into inventory.

### Assumptions

- The user is logged into the division containing the unit to be edited.

### Outcome

- Unit billing information was changed.

### Limitations and Restrictions

- When a Technologist enters a unit ID and product code for a unit that is not in final status, VBECS does not enable the Special Test Cost field. An Enhanced Technologist or higher access can change the Special Test Cost field in the Edit Financial Data option.
- Cost entry is capped at 999.99.

### Additional Information

- None

### User Roles with Access to This Option

All users

## Edit Financial Data

The user adds and/or edits a unit charge. The user confirms the unit selection, enters the dollar amount of the charge, and reviews and accepts the charge entered.

User Action	VBECS
1. Select <b>Blood Units</b> from the main menu.  Select <b>Edit Financial Data</b> .	<ul style="list-style-type: none"> <li>• Displays options for processing blood units.</li> <li>• Displays fields for selecting a unit and editing its financial data.</li> </ul>
2. Scan or enter a unit ID or product code, or click the <b>find</b> button to select a unit from the drop-down list.	<ul style="list-style-type: none"> <li>• Displays unit data.</li> </ul>
3. Edit the Base Cost, Special Test Cost, and/or Return Credit fields (Figure 97).	<ul style="list-style-type: none"> <li>• Allows the user to edit the fields in the Financial Information area.</li> <li>• Displays the edited data.</li> </ul> <p><b>NOTES</b> _____</p> <p>➤➤ The Enhanced Technologist may edit existing amounts.</p> <p>VBECS allows the user to enter a single special typing cost (the sum of charges from \$1.00 to \$999.00, other than the base unit cost, added to a unit by the shipper) to associate with a unit.</p> <p>When no special typing costs are entered through Incoming Shipment, a user may enter them in this option until the unit is in a final status.</p> <p>➤➤ The Enhanced Technologist may edit a previously entered unit cost (unit base cost, return credit or special testing cost).</p> <p>VBECS displays a return credit in U.S. currency (defined by the return credit percentage multiplied by the base unit cost for the blood product type in Blood Products). A user may enter a return credit amount only when VBECS assigned the return credit percentage through Outgoing Shipment or Discard or Quarantine. Otherwise, VBECS disables the Return Credit field.</p>
4. Review the data, click <b>OK</b> to save, and click <b>OK</b> again to confirm the save.	<ul style="list-style-type: none"> <li>• Requests confirmation to update the database.</li> <li>• Saves and updates the unit record.</li> </ul>
5. Repeat Steps 2–4, as needed, to update additional unit financial data.	
6. Click <b>Cancel</b> to exit.	

Figure 97: Edit Unit Financial Data

Unit Search

Unit ID\* W0000000000005 Find

Product\* E0291V00

Clear

Details Testing

Donation Type Volunteer Allogeneic Donor

Date Received 07/27/2011 12:33

Unit Status Available

Volume (mL) 350

Financial Information

Base Cost \$ 1.00

Special Test Cost \$ 0.00

Return Credit \$ 0.00

OK Cancel

\* Required Field

Properly Identify Intended Recipient  
See Center of Information for indications, contraindications, cautions and methods of storage. This product may contain infectious agents.  
Caution: Federal Law prohibits dispensing without a prescription

W0000000000005

O

Rh NEGATIVE

VOLUNTEER DONOR

E0291V00

RBC CPD>AS1

0112381233

Expiration Date

26 Aug 2011 12:33

## Free Directed Unit For Crossover

The user removes a patient restriction from a directed unit that is eligible for crossover, making that unit available for any patient, according to system rules.

### Assumptions

- The division is “full service.”
- The unit and the user are in a blood bank in the same division.
- The unit is restricted for a patient.
- The unit is not autologous.
- The unit is not in a final status.

### Outcome

- The unit has no restrictions.
- VBECs documents the restriction and its removal in the unit record.

## Limitations and Restrictions

- The unit must not be in a final status to release the restriction.
- This option does not allow the removal of patient restrictions for most of the ISBT donation types that are eligible for crossover.
- This option cannot process multiple directed units simultaneously.

## Additional Information

- This option is disabled for transfusion-only facilities.

## User Roles with Access to This Option

- Enhanced Technologist

## Free Directed Unit For Crossover

The user releases the restriction on a directed donor unit to make the unit available in general inventory.




User Action	VBECS
1. Select <b>Blood Units</b> from the main menu.  Select <b>Free Directed Unit For Crossover</b> .	<ul style="list-style-type: none"> <li>• Displays options for processing units.</li> <li>• Prompts the user to scan or enter the unit ID and product code.</li> </ul>
2. Scan or enter the unit ID and product code for each unit.	<ul style="list-style-type: none"> <li>• Displays the unit information.</li> </ul> <p><b>NOTES</b> _____</p> <p>When a user adds a directed unit to an undirected pool, VBECS queries the user regarding releasing the unit's restriction.</p> <p> The unit must have a donation type of only Volunteer Directed, Eligible for Crossover or Family Reserved. If not, VBECS warns that the unit has an invalid donation type and is in a final status. The unit must not be autologous."</p> <p> The unit must not be assigned to a patient to release the restriction. VBECS warns that the unit is currently assigned and that he must release the assignment before continuing.</p>
3. Enter a date and time and possibly a different user, or accept the defaults (Figure 98).  Enter the required comment.	<ul style="list-style-type: none"> <li>• Displays the unit information.</li> <li>•  Emits an audible alert, requires a comment, and captures details for inclusion in an Exception Report (exception type: released patient restriction).</li> </ul>
4. Click <b>OK</b> to process the unit and add it to available inventory.	<ul style="list-style-type: none"> <li>• Displays the return time, the editable (past date and time) field (default: current date and time), and the list of users for selection.</li> <li>• Saves the information to the database.</li> <li>• Changes the donation type of the unit to "Voluntary Allogeneic" and removes the restriction associated with the unit.</li> </ul>
5. Click <b>Yes</b> to confirm the save.	

Figure 98: Free Directed Unit For Crossover

## Discard or Quarantine

The user discards, quarantines, or releases from quarantine one or more units of blood.

## Assumptions

- The unit of blood is in the same division as the user.

## Outcome

- Discarded units are not available.
- Quarantined units are available for limited processing, not for patient assignment.
- Units released from quarantine are available.

## Limitations and Restrictions

- When a user retroactively updates a unit status through this option, the Unit History Report displays the updated information, but does not display the date the change was made.
- The option does not accommodate the entry of a second user for retrospective data entry.
- Cost entry is capped at 999.99.

## Additional Information

- None

## User Roles with Access to This Option

All users


- Enhanced Technologist (release blood products from quarantine)

## Discard or Quarantine



The user changes the status (discard, quarantine, release from quarantine) of one or more units of blood. The user must indicate the ID and product code of the units to be updated. VBECS also allows batch data entry to process all units from an invoice.

A quarantined unit has limited availability for continued processing. Under certain circumstances, VBECS assigns quarantine status automatically; otherwise, the user determines and enters the status. The user must specify whether a discarded unit is for credit or for waste, which determines whether or not a return credit is assigned to the selected unit.

## Discard or Quarantine Individual Units

User Action	VBECS
1. Select <b>Blood Units</b> from the main menu.  Select <b>Discard or Quarantine</b> .	<ul style="list-style-type: none"> <li>• Displays options for processing blood units.</li> <li>• Displays fields for entering unit information.</li> <li>• Allows the user to discard or quarantine individual units.</li> </ul>
2. Enter an ID and product code for a blood unit.	<ul style="list-style-type: none"> <li>• When a unit is appropriate for processing, displays unit information, including the current status.</li> </ul> <p><b>NOTES</b> _____</p> <p style="text-align: center;">The user may deselect units prior to editing.</p>
3. Click <b>Select</b> to add the selected unit to the worklist, or click <b>Clear</b> to select another unit.	<ul style="list-style-type: none"> <li>• Adds the unit to a temporary list of selected units.</li> </ul>
4. Repeat Step 3, as needed, to process multiple blood units as a single batch.  Click <b>OK</b> to continue.	<ul style="list-style-type: none"> <li>• Displays selected units.</li> </ul>
5. In the Activity area, click the <b>Discard, Quarantine, or Release from Quarantine</b> radio button (Figure 99).	<p><b>NOTES</b> _____</p> <ul style="list-style-type: none"> <li>➤➤ The Enhanced Technologist may release a unit from quarantine.</li> </ul>
6. Select or enter a comment in the Comment field to explain the change in status, when indicated. Enter details, as needed.	<ul style="list-style-type: none"> <li>• Displays entered data.</li> </ul> <p><b>NOTES</b> _____</p> <p>When a status of "Discard" is assigned to a unit, the user must enter a comment.</p> <p> When a blood unit is assigned an indicator of "quarantined," VBECS emits an audible alert, requires a comment, and captures details for inclusion in an Exception Report (exception type: Quarantine, Discard or Release from Quarantine, depending on the user action).</p>



User Action	VBECS
	<p> When a unit is released from quarantine, a new status must be assigned. VBECS assigns the previous status as the default status. When the unit expiration date is in the past, VBECS emits an audible alert, warns the user, and asks whether he wishes to continue.</p> <p><b>Yes</b> accepts the default status, requires a comment, and captures details for inclusion in an Exception Report (exception type: expired unit released from quarantine). <b>No</b> stops the process; the unit remains marked as quarantined.</p> <p>When the user assigns a status of "Discard," he must indicate whether the discard is waste or credit.</p> <p>When VBECS processes a unit as a discard that is waste, it does not apply a return credit to the discarded unit. (There is no refund of monies from the blood shipper.)</p> <p>When VBECS saves a unit as "discard for credit," it applies a default return credit. A user may edit the default return credit applied to the unit before updating the database.</p> <p>When a unit has had a quarantine indicator assigned, VBECS releases any patient assignments. Patient restrictions associated with the unit remain intact.</p> <p>When a user attempts to release a unit from quarantine and the unit was quarantined because of a data conflict, VBECS checks the inconsistency before allowing the unit to be released. VBECS warns the user. There is no override. The user must invalidate the incorrect information or test to resolve the data conflict.</p>
<p>7. Enter the date and time of the status update.</p> <p>Click the <b>Waste</b> or <b>Credit</b> radio button.</p>	<ul style="list-style-type: none"> <li>• VBECS defaults the date and time of the status update to the date and time the user opened the Discard or Quarantine option</li> <li>• Displays entered data.</li> </ul> <p><b>NOTES</b> _____</p> <p>The default date and time are the current date and time.</p> <p>When the user updates the status of a unit and enters a date or time in the past, VBECS compares the retrospective date and time with the last record update for the unit. VBECS allows the user to update a unit record with a date and time between the last record update and the current date and time.</p> <p> &gt;&gt;&gt; When the user attempts to update a unit record prior to the last record update, VBECS emits an audible alert, warns the Enhanced Technologist, and instructs him to check the date and time entered. There is no override.</p> <p>&gt;&gt;&gt;&gt; When the user is a Lead Technologist, VBECS also requires a comment and asks whether the user wishes to continue. <b>Yes</b> accepts the date and time entered, requires a comment, and captures details for inclusion in an Exception</p>

User Action	VBECS
	Report (exception type: retrospective update prior to last record update). <b>No</b> allows the user to correct the entry.
8. Review the unit status update information and click <b>Save</b> to save.	<ul style="list-style-type: none"> <li>Evaluates the unit's current status to determine whether it is eligible for the selected function.</li> <li>Displays appropriate error messages when the unit cannot be processed.</li> <li>Prompts the user to confirm the save.</li> </ul>
9. Click <b>Yes</b> to confirm the save.	<ul style="list-style-type: none"> <li>Saves the data.</li> </ul> <p><b>NOTES</b> _____</p> <p>During batch data entry of "Discard," "Quarantine," and "Release from Quarantine," VBECS saves transaction details, including user-entered comments, to each blood unit included in the batch.</p>

**Figure 99: Discard or Quarantine Units**

**VBECS - Discard or Quarantine Units**

Discard or Quarantine Units

Unit ID	Product Name	Code	Status	Credit	Quarantined?
1941954	RBC AS1	04210	Available	\$10.00	No

**Activity\***

Discard

Quarantine

Release from Quarantine

**Discard Unit(s)**

Comment\*

Details

Discard Date\*

**Reimbursement\***


Waste


Credit

\* Required Field

Save Cancel

## Discard or Quarantine Units by Invoice

User Action	VBECS
<p>1. Select <b>Shipments</b> from the main menu.</p> <p>Select <b>Discard or Quarantine</b>.</p> <p>Select an invoice and click <b>OK</b>.</p>	<ul style="list-style-type: none"> <li>• Displays options for processing blood product shipments.</li> <li>• Displays fields for entering shipment information.</li> <li>• Allows the user to discard or quarantine units by invoice.</li> </ul>
<p>2. Click (clear) one or more check boxes to remove units from the invoice.</p> <p>Click <b>OK</b>.</p>	<ul style="list-style-type: none"> <li>• Allows the user to select individual units.</li> <li>• Displays unit information, including the current status included in the invoice.</li> <li>• Allows the user to deselect units from the batch prior to editing.</li> </ul> <p><b>NOTES</b> _____</p> <p>For batch processing, the user may select incoming shipment invoices from the previous five days processed in the same division as the user.</p> <p>Only incoming shipment invoices processed within the user's division may be accessed for batch processing.</p>
<p>3. In the Activity area, click the <b>Discard, Quarantine, or Release from Quarantine</b> radio button.</p>	<p><b>NOTES</b> _____</p> <p>➤➤ The Enhanced Technologist may release a unit from quarantine.</p>
<p>4. Select or enter a comment in the Comment field to explain the change in status, when indicated. Enter details, as needed.</p>	<ul style="list-style-type: none"> <li>• Displays entered data.</li> </ul> <p><b>NOTES</b> _____</p> <p>When a status of "Discard" is assigned to a unit, the user must enter a comment.</p> <p>When a blood unit is assigned an indicator of "quarantined," VBECS emits an audible alert, requires a comment, and captures details for inclusion in an Exception Report (exception type: Quarantine, Discard or Release from Quarantine, depending on the user action).</p> <p> When a unit is released from quarantine, a new status must be assigned. VBECS assigns the previous status as the default status. When the unit expiration date is in the past, VBECS emits an audible alert, warns the user, and asks whether he wishes to continue.</p> <p><b>Yes</b> accepts the default status, requires a comment, and captures details for inclusion in an Exception Report (exception type: expired unit released from quarantine). <b>No</b> stops the process; the unit remains marked as quarantined.</p> <p>When the user assigns a status of "Discard," he must indicate whether the discard is waste or credit.</p>

User Action	VBECS
	<p>When VBECS processes a unit as a discard that is waste, it does not apply a return credit is not applied to the discarded unit. (There is no refund of monies from the blood shipper.)</p> <p>When VBECS saves a unit as “discard for credit,” it applies a default return credit. A user may edit the default return credit applied to the unit before updating the database.</p> <p>When a unit has had a quarantine indicator assigned, VBECS releases any patient assignments. Patient restrictions associated with the unit remain intact.</p> <p>When a user attempts to release a unit from quarantine and the unit was quarantined because of a data conflict, VBECS checks the inconsistency before allowing the unit to be released. VBECS warns the user. There is no override. The user must invalidate the incorrect information or test to resolve the data conflict.</p>
<p>5. Enter the date and time of the status update.</p> <p>Click the <b>Waste</b> or <b>Credit</b> radio button.</p>	<ul style="list-style-type: none"> <li>• VBECS defaults the date and time of the status update to the date and time the user opened the Discard or Quarantine option</li> <li>• Displays entered data.</li> </ul> <p><b>NOTES</b> _____</p> <p>The default date and time are the current date and time.</p> <p>When the user updates the status of a unit and enters a date or time in the past, VBECS compares the retrospective date and time with the last record update for the unit. VBECS allows the user to update a unit record with a date and time between the last record update and the current date and time.</p> <p> &gt;&gt;&gt; When the user attempts to update a unit record prior to the last record update, VBECS emits an audible alert, warns the Enhanced Technologist, and instructs him to check the date and time entered. There is no override.</p> <p>&gt;&gt;&gt;&gt; When the user is a Lead Technologist, VBECS also requires a comment and asks whether the user wishes to continue. <b>Yes</b> accepts the date and time entered, requires a comment, and captures details for inclusion in an Exception Report (exception type: retrospective update prior to last record update). <b>No</b> allows the user to correct the entry.</p>
<p>6. Review the unit status update information and click <b>Save</b> to save.</p>	<ul style="list-style-type: none"> <li>• Evaluates the unit’s current status to determine whether it is eligible for the selected function.</li> <li>• Displays appropriate error messages when the unit cannot be processed.</li> <li>• Prompts the user to confirm the save.</li> </ul>
<p>7. Click <b>Yes</b> to confirm the save.</p>	<ul style="list-style-type: none"> <li>• Saves the data.</li> </ul> <p><b>NOTES</b> _____</p> <p>During batch data entry of “Discard,” “Quarantine,” and “Release from Quarantine,” VBECS saves transaction details, including user-entered comments, to each blood unit included in the batch.</p>

## Select Units

A user selects a patient component order and assigns one or more products to a patient for possible transfusion.

### Assumptions

- An appropriate current order was accepted through Accept Orders: Accept an Order.
- The division may or may not allow electronic crossmatch (eXM).
- When a division is “transfusion only,” eXM is not available. The user must enter a serologic crossmatch (XM) interpretation.
- Current patient TAS specimen is required for (ABO/Rh and Antibody Screen) required tests for “WHOLE BLOOD” and “RED BLOOD CELLS” component classes and individual “PLATELET” and “OTHER” units that contain equal to or greater than ( $\geq$ ) 2 mL of RBC contamination which requires crossmatch.

### Outcome

- The status of the unit and the associated component order task status are updated.
- VBECS indicates that units are available for issue when units do not require XM or other additional testing.
- VBECS allows the user to print a Blood Transfusion Record Form (BTRF) and a Caution Tag for available blood products.
- When serologic XM is required, VBECS automatically adds units to the XM data entry grid when Patient Testing: Record a Crossmatch is accessed.
- When eXM is enabled for the facility, the patient is eligible for eXM in accordance with system rules (see Table 23: Rules for Electronic and Serologic Crossmatch), and the user selects a unit, and processes the eXM. VBECS assigns the unit to the patient without further testing.

### Limitations and Restrictions

- When a user checks the “D Pos” or “D Neg” check box to modify a search for blood, VBECS returns no units, regardless of the availability of units.
- VBECS displays a component order as “Filled” when the number of units transfused and the number of units ordered are the same and continues to display the component order until the order expires. CPRS displays a component order as “Completed”. Therefore, CPRS displays the order as “Completed” before VBECS displays the order as “Filled.” The associated LAB order will then also be marked as “Completed”.
- A Specimen UID must be accepted in VBECS to associate it with a component order. Any VBECS maintained TAS specimen UID may be associated with a component order by entering the number in the UID field.
- A system error occurs when attempting to associate a specimen with an order when no unit is selected. Associate the specimen UID in Accept Order when possible.
- When associating a specimen with a patient in the “Select Unit for a Patient” window, the “Expires” field within the “Associate with Specimen” panel displays the time in AM/PM format. VBECS normally displays date/time fields in “military” time format. The workaround is to look in Maintain Specimen and check the expiration time military time format.

- When an Rh positive unit is selected for an Rh negative patient with an antigen negative requirement for D, the user receives a message about unit not satisfying patient negative requirements for D instead of a message about unit being positive for D. The proper message is displayed upon unit issue.

### **Additional Information**

- Under certain conditions in a blood bank, more than one user can select units for the same blood component order at the same time.
- When a patient order is selected, the system must search for physically available (in limited, available, or crossmatched unit status and with a future expiration date) autologous and directed units within a multidivisional database regardless of the unit's current division, display the unit information to the user, and allow assignment of the unit if it is in the same division as the user.
- Active component orders and the user must be in the same division.
- Multiple users may select units associated with a component order.
- Product codes associated with a component order are limited to those in that Component Class.
- When a user selects a patient order within a multidivisional database, VBECS searches for available autologous and directed units regardless of the unit's current division, displays the unit information, and allows the user to assign the unit when it is in the user's division.
- A user may select units for only one component order at a time.
- An existing specimen and its associated component order(s) remains available to select additional units, until its expiration date.
- The selection of blood units completes a CPRS order when the number of units are selected that "fill" the order per the number requested.
- Each component task is accepted or cancelled individually.
- When a component order is cancelled or rejected, the order is no longer retrievable.
- When a patient has only Component Requirements, the system does not prohibit the use of electronic crossmatch.
- A user must accept or cancel each component task in its entirety.
- The user reserves autologous and directed units and all units processed in a "transfusion-only" facility, regardless of donation type, for a specific patient. VBECS displays this information to all divisions in a multidivisional database and accommodates immediate selection of the unit for the patient when the user is in the same division.
- When VBECS compares a patient's current and previous ABO/Rh results, it does not evaluate previously recorded ABO/Rh results marked "Test Result Invalidated."
- An ABO/Rh confirmation test interpretation is required when a unit is indicated to contain  $\geq 2$  mL of RBC contamination. The confirmation test must match the unit ABO/Rh at login and repeat tests, when repeat tests are not invalidated. The unit must have a satisfactory confirmation result before VBECS allows the unit to be available for selection for patient use.
- When the CPRS order is WB or RBC, the user may select products from either product class, RBC or WB to fill the order
- When Whole Blood is selected for a WB or RBC CPRS order and the unit is not ABO compatible for the selected patient, the system presents a message to the user stating that the unit is not ABO compatible and releases the unit from selection (assignment).
- Users can view the Lab Order number using the Pending Order List or the Order History Report, if it is needed in areas where it is not already provided.


- In a multidivisional database, restricted units residing in another division will be displayed but are not selectable. The tooltip (mouseover) message wording incorrectly states "Unit is not in division ###". The division shown will be the division where the unit IS located.
- The "Success" message received after electing to print caution tags is a message that the job has queued to the print queue, and does not verify that the printing has actually occurred.


## User Roles with Access to This Option

All users


### Select Units

A user selects a patient's component order and assigns one or more products to a patient. The user may then enter individual units or select units from a list of selectable, potentially compatible units for each component order.


User Action	VBECs
<p>1. Select Patients from the main menu and select Patient <b>Testing</b>, or</p> <p>Select <b>Blood Units</b> from the main menu and select <b>Select Units</b>.</p>	<ul style="list-style-type: none"> <li>• Displays options for processing patient-related functions.</li> <li>• Displays the Pending Task List (PTL) and PTL search parameters in the Diagnostic Tests tab or in the Component Orders tab.</li> </ul>
<p>2. Enter or select PTL search parameters, if appropriate (Figure 100).</p> <p>Click the check boxes in the PTL to select one to four component orders.</p> <p>Click <b>OK</b> to continue (Figure 101).</p>	<ul style="list-style-type: none"> <li>• Displays information for each component order.</li> <li>• When the user selects a blood unit to fill a component order or RED BLOOD CELLS, "PLATELETS" or "OTHER" and has = &gt; 2 mL RBC contamination, and the unit is ABO compatible, VBECs requires a crossmatch test and must require the user to associate a TAS specimen UID to the order to proceed to crossmatch of the unit.</li> <li>• Upon completion of the selection process, evaluates the unit for eXM when enabled at the site or added to the serologic crossmatch data grid as appropriate.</li> <li>• When the user selects blood units to fill a component order of "PLATELETS" or "OTHER" and is indicated as having less than (&lt;) 2 mL RBC contamination, electronic or serologic crossmatch are not required.</li> </ul> <p><b>NOTES</b> _____</p> <p> See Table 6 for alerts that may occur during this option.</p>
<p>3. <b>Steps 3 through 8 are optional. If a specimen is already associated with the order, skip to Step 9.</b> To associate a specimen UID with a unit requiring XM that was not previously associated with a specimen, select (highlight) an issued unit in the Order Group tree view in the left pane (Figure 102).</p>	<ul style="list-style-type: none"> <li>• Enables the Associate Specimen check box.</li> <li>• When an order is accepted without a specimen, the user must associate the specimen prior to performing crossmatch testing. VBECs will not allow an expired specimen to be associated with the order unless the Unit Assigned Date is modified to be earlier than the Specimen Expiration Date (for retrospective data entry). See Figure 102.</li> </ul> <p><b>NOTES</b> _____</p> <p>If eXM is enabled, complete the TAS prior to associating the specimen UID with the emergency-issued units.</p>

User Action	VBECS
	<p>When eXM is enabled at the division and blood units require XM, VBECS determines whether the patient and units are eligible for eXM. When indicated, specific warning messages explaining the reason for the ineligibility are displayed. Users are given the opportunity to accept or reject the selected unit for selection and may or may not process the eXM. All parameters listed in Appendix B: Table 23: Rules for Electronic and Serologic Crossmatch must be met. Units not eligible for eXM must have serologic XM performed.</p> <p>When eXM is not enabled, the user must enter serologic XM results for red blood cell classes or when the unit has = &gt; 2 mL RBC contamination. VBECS does not change the unit status to “crossmatched,” make units available for issue, or allow the Caution Tag or BTRF to be printed until a user enters a valid XM result, unless an emergency-issued unit is being processed.</p>
<p>4. Click the <b>Associate with Specimen</b> check box.</p> <p>Click the <b>find</b> button to select a specimen UID (Figure 102).</p>	<ul style="list-style-type: none"> <li>• Opens the Select Patient Specimen window.</li> </ul>
<p>5. Scan or enter the correct specimen UID and click <b>Search</b>.</p>	<ul style="list-style-type: none"> <li>• Displays the selected patient specimen information.</li> </ul>
<p>6. Click <b>OK</b>.</p>	<ul style="list-style-type: none"> <li>• Saves the specimen information and returns the user to the Select Units window.</li> <li>• Displays the associated specimen UID in the Specimen UID field.</li> </ul>
<p>7. Repeat Steps 3 through 6 to associate the specimen UID with other emergency-issued units, or click <b>OK</b> to save.</p>	<ul style="list-style-type: none"> <li>• When the user clicks OK, displays a message asking the user if he wishes to complete serologic crossmatch when at least one unit requires crossmatch.</li> </ul>
<p>8. Click <b>Yes</b> to complete the serologic crossmatch.</p> <p>Click <b>No</b> to return to the VBECS desktop.</p>	<ul style="list-style-type: none"> <li>• Allows the user to enter results in the Serologic Crossmatch (XM) cells in the patient testing grid.</li> </ul>
<p>9. Click <b>Add Units</b> to add units to the component orders selected.</p>	<ul style="list-style-type: none"> <li>• Searches the database and displays available restricted autologous and directed units that the user must select before selecting allogeneic units of the same component class.</li> <li>• Displays the unit ID, product type, unit ABO/Rh, unit expiration date, specimen UID, if applicable, of units assigned to the patient in the division’s blood bank on a different specimen or order. The user may add and remove the units to the current order until saved.</li> <li>• When eXM was configured for the facility, displays the eXM status (eligible or not eligible) for the patient.</li> <li>• When the component order is for “PLATELETS” or “OTHER”, the units with = &gt; 2 mL of RBC contamination must be presented and will display <i>before</i> products with less than (&lt;) 2 mL of RBC contamination.</li> <li>• VBECS will present all available units when the Product Type is “PLATELETS” or “OTHER” non-plasma components that have less than (&lt;) 2 mL RBC contamination.</li> </ul> <p><b>NOTES</b> _____</p> <p> See Table 6 for alerts that may occur during this option.</p>



User Action	VBECS
	<p>Units that are selectable with an override do not appear on the list of available units and must be scanned or entered by the user.</p> <p>A patient's historical ABO/Rh is displayed, when present. When the current specimen is not yet tested, is normally not required, or must be saved as inconclusive, a unit of blood can be selected based on the patient's historical ABO/Rh.</p> <p>When the patient's historic ABO/Rh is "NR," "inconclusive", from database conversion, or the current specimen has an "emergency issue" order indicator present and/or the required ABO/Rh specimen testing is incomplete, the user may select components as follows, including autologous units:</p> <p>For RBC or WB order:</p> <ul style="list-style-type: none"> <li>• Group O, Rh positive or O Rh negative RBC</li> <li>• WHOLE BLOOD is not selectable</li> </ul> <p>For FFP order:</p> <ul style="list-style-type: none"> <li>• Group AB, Rh positive, Rh negative, or not specified</li> </ul> <p>For PLT order:</p> <ul style="list-style-type: none"> <li>• All units with &lt; 2 mL RBC contamination are available</li> <li>• Units with <math>\geq</math> 2 mL RBC contamination are not available</li> </ul> <p>For CRYO order:</p> <ul style="list-style-type: none"> <li>• All available units are selectable.</li> </ul> <p>For OTHER order:</p> <ul style="list-style-type: none"> <li>• All units with &lt; 2 mL RBC contamination are available</li> <li>• Units with <math>\geq</math> 2 mL RBC contamination are not available</li> </ul>
<p>10. Scan, enter or select *allogeneic units. Units may be selected from the Order Group tree view or by clicking the <b>find</b> button for a list of compatible units.</p> <p>* when typing in a unit ID and Product Code, use the tab key or mouse to leave the field.</p> <p>Enter or select a date in the Selection Date field. Each unit may have its own selection date and time.</p> <p>Click <b>Add Units</b> to display the unit selection form.</p>	<ul style="list-style-type: none"> <li>• Maintains a list of selected units. The user may deselect selected units.</li> <li>• Evaluates the selected unit: <ul style="list-style-type: none"> <li>o For the ABO/Rh compatibility of each unit and warns the user.</li> <li>o Against patient Transfusion Requirements (TRs) and warns the user.</li> <li>o For unit antigen negative compatibility and warns the user.</li> </ul> </li> <li>• Performs additional checks that include days remaining until the unit's expiration date, the current status, assignments to other patients, quarantine and biohazard indicators, when applied, and associated special testing.</li> </ul> <p><b>NOTES</b> _____</p> <p> See Table 6 for alerts that may occur during this option.</p> <p><i>When the user selects a unit in the tree view, he may accept or edit the current date and time displayed; the date and time must be in the past.</i></p> <p>Restricted (autologous or directed) unit selection eligibility criteria:</p> <ul style="list-style-type: none"> <li>• Restricted For Patient ID: must match</li> <li>• Restricted For Patient Name: must match</li> </ul>

User Action	VBECS
	<ul style="list-style-type: none"> <li>• Unit expiration date must be in the future.</li> <li>• Component class (transfusion only requires restriction of all units)</li> </ul> <p>The criteria for selecting or including an individual unit in a pick list include evaluating a unit for:</p> <ul style="list-style-type: none"> <li>• Considered substitutable for the order</li> <li>• Same division as the user</li> <li>• Current location is in the blood bank</li> <li>• Has no quarantine indicator</li> <li>• Is not already assigned or crossmatched to the patient</li> <li>• Is not “restricted” for a different patient</li> <li>• Meets ABO/Rh compatibility requirements specific to the component class</li> <li>• Future expiration date</li> </ul> <p>VBECS lists only ABO and Rh compatible units for selection. The “select units” utility creates a system-generated pick list that is sorted and displayed using the following criteria: first, ABO/Rh identical, then expiration date (oldest units first); then, ABO/Rh compatible.</p> <p>The pick list or individually selected unit field display includes:</p> <ul style="list-style-type: none"> <li>• Unit ID</li> <li>• Product Short Name</li> <li>• ABO/Rh</li> <li>• Expiration Date</li> <li>• Unit Status</li> <li>• Donation Type</li> <li>• Restricted for Patient Name and ID</li> <li>• Patient Assigned (if unit assigned to other patient but with location of blood bank)</li> <li>• Antigen Type Results (if available)</li> </ul> <p>When ABO/Rh results were not created for the specimen associated with the component order and the order is not designated “emergency issue,” VBECS allows the user to select units using historic records, and notifies the user that ABO/Rh testing must be performed at this division on this patient before any selected units can be issued.</p> <p>VBECS displays and allows the user to complete a pending XM associated with a patient order without a final unit status.</p>
11. Repeat Step 10 until all units are selected.	<ul style="list-style-type: none"> <li>• Requests the user to confirm the list of selected blood units.</li> </ul> <p><b>NOTES</b> _____</p> <p>The user may select or deselect additional units.</p>

User Action	VBECS
12. Click <b>OK</b> to confirm the selection of units.	<ul style="list-style-type: none"> <li>• Determines whether the units are available for issue or whether they require additional modification and/or testing based on the component class.</li> <li>• Updates the database, as appropriate.</li> <li>• Sends selected blood units with patient details to the BCE COTS system via an HL7 message when that BCE COTS is available.</li> </ul> <p><b>NOTES</b></p> <p>When the total number of blood units eligible for issue, issued, or transfused is greater or equal to the number of ordered units, VBECS sends an order completion message to CPRS.</p> <p> See Table 6 for alerts that may occur during this option.</p>
13. Select another component order to continue selecting units, or exit.	<ul style="list-style-type: none"> <li>• Displays orders for the next patient.</li> </ul>

**Figure 100: Pending Task List Component Orders**

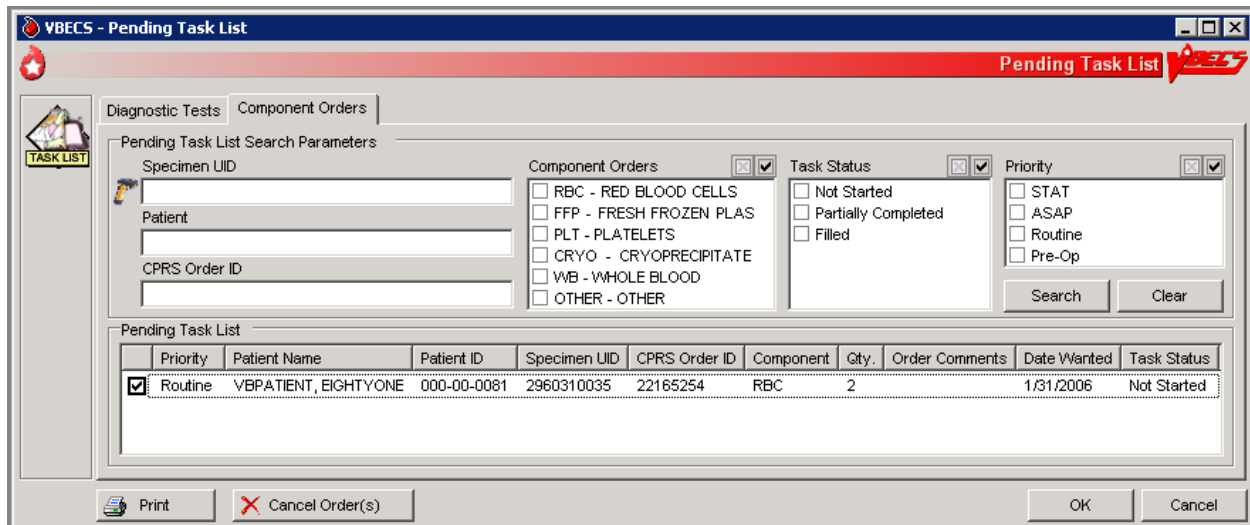


Figure 101: Select Units for a Patient

**VBECS - Select Units for a Patient**

Select Units for a Patient

**Order Group**

- RED BLOOD CELLS

**Patient Specimen**

Name: VBPATIENT    Patient ID: 000-00-0081    ABO/Rh: O Neg

Specimen UID: 2960310035    Expires: 02/03/2006 23:59

**Component Order**

Component Ordered: RED BLOOD CELLS    Quantity: 2

Additional Requirements: [Empty]

Order Comments: [Empty]

Urgency: Routine

Date Received: 01/31/2006 08:31    Date Wanted: 01/31/2006 08:30

Ordered By: USER, ONE

Pre-Op Surgery Name: [Empty]

Reason Requested: [Empty]

Add Units

\* Required Field

OK    Cancel

**Figure 102: Associate a Specimen with an Emergency-Issued Unit**

Alerts

**Table 6: Alerts That May Occur in Select Units**

Step	Alerts
2	VBECs displays  and emits an audible alert when there are Special Instructions (SIs) or TRs for the patient.
2	When the user selects blood units that require XM, and eXM is enabled at the division but the current patient is not eligible for eXMs, VBECs displays a warning that includes the reason for the ineligibility. Selected units must have serologic XM performed. The user may not print the Caution Tag or the BTRF. VBECs does not change the unit status to “crossmatched” until the user enters a valid serologic XM result in Patient Testing: Record a Crossmatch.
2	When the user clicked the Emergency Order check box during specimen acceptance and selects blood units requiring crossmatch, VBECs does not allow the user to perform an electronic or serologic crossmatch.
9	When a current specimen is required for the selected component class based on Component Classes and one does not exist for the patient, the user may not select ABO type-specific blood products. VBECs notifies the user that he is selecting units using the emergency issue compatibility rules.
9	When the patient record is marked as having a previous ABO/Rh discrepancy that was justified, a current specimen is required. The user must select units based on the current specimen ABO/Rh. When the current specimen ABO/Rh was not performed, VBECs warns that the patient had a previous ABO/Rh

Step	Alerts
	discrepancy and instructs the user to perform ABO/Rh on the current specimen to continue using normal rules and policies or follow emergency issue rules and policies.
9	When there is a current specimen and ABO/Rh testing was performed, VBECS compares the ABO/Rh of the current specimen to the historical ABO/Rh. When there is a discrepancy, the ABO type is inconclusive, and there is no previous justification for the discrepancy on the patient record, VBECS notifies the user that the ABO/Rh on the current specimen does not agree with the historical ABO/Rh and instructs the user to resolve the discrepancy to continue using normal rules and policies, and recommends that the user release previously selected units or follow emergency issue rules and local policies.
9	When a repeat ABO/Rh is performed on the patient specimen and the interpretation does not match the original ABO/Rh interpretation.
9	When a user selects a blood unit that is evaluated as ABO incompatible (based on comparison of the patient's current ABO/Rh entered by the user with the ABO/Rh on record), VBECS emits an audible alert and warns the user that the unit is not ABO compatible for this patient and that the user may not select the unit. There is no override; VBECS clears the screen and allows the user to select another unit.
9	When a blood component belongs to a Product Type and is indicated as "Frozen" and is selected for a patient, VBECS emits an audible alert and displays a warning message: "Selected blood unit must be thawed and may require additional processing prior to issue". There is no user response required and the blood unit is not eligible for issue. The status of the units is updated to "assigned" to the indicated patient.
10	The user may override the selection of restricted units. VBECS warns that more restricted units were not selected for the patient and asks whether the user wishes to continue. The override requires a comment and captures details for inclusion in an Exception Report (exception type: less restrictive unit issued).
10	VBECS warns the user when he selects type-specific FFP/PLASMA products that do not require a specimen, according to the division configuration of component classes, for a patient with a history of justified ABO/Rh change. Note: VBECS does not enforce type-specific compatibility rules for product types in the CRYOPRECIPITATE and PLATELETS [with less than (<) 2 mL of RBC contamination] component classes.
10	Once a valid XM interpretation is verified and the unit is compatible, VBECS does not allow a user to select that unit (ID and product code) for the same specimen. The user may select the unit for a XM with the same patient's new specimen after the unit is released from the current assignment. VBECS warns that the unit has a valid current XM with this patient. The user clears the warning and may continue or enter a new unit number for selection.
10	When a selected unit is currently assigned to another patient, VBECS warns the user and asks whether the user wishes to continue to select for this patient. <b>Yes</b> allows the user to continue. <b>No</b> clears the screen and allows the user to select a new unit.
10	When a user is a Technologist and selects an expired unit, VBECS warns him. There is no override.  ➤➤ When the user is an Enhanced Technologist or above, VBECS warns that the unit is expired and asks whether the user wishes to continue to select.  <b>No</b> clears the screen. <b>Yes</b> requires a comment and captures details for inclusion in an Exception Report (exception type: expired unit selected).
10	When a unit is selected for a patient, VBECS displays a tool tip stating that the unit may not be selected or removed.
10	When the user selects a unit and these conditions exist, VBECS warns the user. There is no override. VBECS clears the screen and the user may select another unit: <ul style="list-style-type: none"> <li>• For XM for the same patient, VBECS warns that the unit was already selected for the patient.</li> <li>• That is not substitutable for the order, VBECS warns that the selected unit is not valid for order.</li> <li>• That is quarantined, VBECS warns that the unit is quarantined and may not be selected.</li> <li>• That is on a pending outgoing shipment list, VBECS warns that the unit is currently included on a pending outgoing shipment invoice and must be removed from the shipment invoice before it can be selected.</li> </ul>

Step	Alerts
	<ul style="list-style-type: none"> <li>• That is not in the blood bank or is issued to another patient, VBECS warns that the unit is not in the blood bank and may not be selected.</li> <li>• That is restricted for a different patient, VBECS warns that the unit is restricted for a different patient and may not be selected.</li> <li>• The confirmation is not performed or is incomplete, VBECS emits an audible alert and warns that the unit ABO/Rh confirmation was not performed or is incomplete and that the unit may not be selected.</li> </ul>
10	When the selected unit had an incompatible crossmatch result with the current patient in the past, VBECS warns the user. The user may continue with this unit and proceed with electronic or serologic crossmatch testing.
10	When the user selects an antigen positive (or inconclusive for testing) unit and the patient has the corresponding antigen negative requirement, VBECS warns the user and requires action determined by the user's security level.
10	<p>➤➤ When the user is an Enhanced Technologist or below, VBECS emits an audible alert and warns that the patient has a history of an antibody, that the unit does not have a negative test for the corresponding antigen, and that the user does not have the proper security level to select the unit. There is no override.</p>
10	<p>➤➤➤ When the user is a Lead Technologist or above, VBECS emits an audible alert, warns that the patient has a history of an antibody and that the unit is positive or inconclusive for the corresponding antigen, and asks whether the user wishes to continue to select the unit.</p> <p><b>No</b> allows the user to continue to select units for the patient. <b>Yes</b> requires a comment and captures details for inclusion in an Exception Report (exception type: antigen positive units selected).</p>
10	<p>When a unit was not antigen typed for the corresponding antigen, VBECS emits an audible alert, warns that the unit must be typed and found negative to be available for this patient, and asks whether the user wishes to continue to select this unit. The user may continue to select units for the patient.</p> <p><b>Yes</b> requires a comment and captures details for inclusion in an Exception Report (exception type: antigen positive units selected).</p> <p>See Appendix B: Table 14: Antibody and Antigen Table: Irregular Antibodies.</p>
10	<p>When an Rh negative patient has an active anti-D antibody or a D antigen negative requirement set in Special Instructions and Transfusion Requirements, VBECS verifies the presence or absence of the corresponding antigen based on the ABO/Rh of the selected blood unit. When a user selects an Rh positive blood unit for a patient with anti-D, VBECS warns the user and requires action based on the user's security level.</p> <p>The antigen negative requirement for anti-D is D, not AGwD. AGwD is not a selectable antigen negative requirement in Special Instructions &amp; Transfusion Requirements: Antigen Negative RBCs.</p>
10	<p>For an Rh negative, unknown, or inconclusive patient: when the selection of Rh positive or Rh pooled units from the WHOLE BLOOD, RED BLOOD CELLS, PLATELETS, and OTHER VBECS component classes are individually entered and confirmed as required, VBECS emits an audible alert, warns that the patient is Rh negative and the selected unit is Rh positive, instructs the user to check the patient sex and hospital policy before continuing, and asks whether the user wishes to continue to select the unit.</p> <p><b>No</b> clears the unit data and allows the user to select a different unit. <b>Yes</b> requires additional comment (unit selection canned comment context) and captures details for inclusion in an Exception Report (exception type: Rh positive units selected).</p>
10	When the user selects a blood unit to fill a component order of RED BLOOD CELLS, "PLATELETS" or "OTHER" and has = > 2 mL RBC contamination, the unit will be evaluated for ABO/Rh red cell compatibility. When the unit is not ABO compatible for the selected patient, VBECS presents a message to the user stating that the unit is and releases the unit from selection (assignment).
10	When a patient has TRs other than CMV negative or Sickle Cell negative, VBECS verifies whether the selected unit satisfies the requirement based on system rules TT_3.04 Blood Unit Substitution and the Fulfillment of Transfusion Requirements. When the TR is not satisfied, VBECS emits an audible alert, warns that the unit does not satisfy all of the patient's TRs, and asks whether the user wishes to continue to select the unit.

Step	Alerts
	<b>No</b> clears the screen. <b>Yes</b> allows the user to continue without comment.
10	<p>When a patient has a TR of CMV negative or Sickle Cell negative, VBECS verifies the selected unit for compliance. When the selected unit does not fit the criteria, VBECS emits an audible alert and warns the user:</p> <ul style="list-style-type: none"> <li>• &gt;&gt; Enhanced Technologist or below: VBECS warns that the patient requires CMV negative or Sickle Cell negative blood, that the selected unit is not CMV negative or Sickle Cell negative, and that the user does not have the proper security level to select the unit. There is no override. VBECS clears the screen, and the user must select another unit.</li> <li>• &gt;&gt;&gt; Lead Technologist or above: VBECS warns that the patient requires CMV negative or Sickle Cell negative blood, that the selected unit is not CMV negative or Sickle Cell negative, and asks whether the user wishes to continue to select. <b>No</b> clears the screen and the user must select a new blood unit. <b>Yes</b> requires a comment and captures details for inclusion in an Exception Report (exception type: Transfusion Requirement incompatible unit selected).</li> </ul>
10	<p>When a user selects a unit with a biohazardous indicator for a patient, VBECS emits an audible alert, warns the user, and asks whether the user still wishes to select the unit.</p> <p><b>No</b> clears the entry and allows the user to select a new unit. <b>Yes</b> requires a comment and captures details for inclusion in an Exception Report (exception type: biohazardous unit selected).</p>
10	<p><input type="checkbox"/> When the user selects blood components from the frozen product types for a patient, VBECS emits an audible alert and warns that the selected blood unit must be thawed and may require additional processing prior to issue. VBECS updates the blood unit status to “assigned” and the unit is no longer available for issue.</p>
10	<p>When a user selects a unit requiring XM, eXM is enabled, and the patient is eligible, the selected unit ABO/Rh must be compatible with the patient ABO/Rh. VBECS displays a message based on Appendix B: Table 23: Rules for Electronic and Serologic Crossmatch. If the user continues to select that unit in which VBECS requires a serologic XM, VBECS does not allow the Caution Tag or BTRF to be printed, and does not update the unit status to “crossmatched” under normal system rules and policies. When the selected unit is available for emergency issue to the patient, VBECS allows the Caution Tag and BTRF to be printed.</p>
12	<p>When eXM is enabled and the patient and selected units are eligible, VBECS notifies the user and asks whether he wishes to continue with eXM.</p> <p><b>Yes</b> assigns a XM result of “Compatible: Electronically Crossmatched.” VBECS updates the database with the XM result, date and time of XM, XM technologist, division, specimen UID, specimen phlebotomist, and specimen expiration date and time. VBECS allows a Caution Tag and BTRF to be printed and the unit is ready for issue and transfusion. VBECS changes the unit status to “crossmatched.”</p> <p><b>No</b> keeps the unit assigned to the patient and available for data entry of serologic XM. VBECS does not allow the Caution Tag or BTRF to be printed and does not change the unit status to “crossmatched.” When a division is “transfusion only,” the unit remains assigned to the patient and is available for the user to enter the serologic XM interpretation, VBECS does not allow the Caution Tag or BTRF to be printed, and the unit is not crossmatched.</p>
12	<p>When the user selects a unit requiring XM, the division is defined as “full service,” eXM is enabled, and the patient is not eligible for eXM, VBECS warns the user that the selected units require serologic XM and asks whether he wishes to continue.</p> <p><b>Yes</b> adds the units to the XM data entry grid and transfers the user to Patient Testing: Record Patient Test Results to begin the XM process. <b>No</b> records that a serologic XM test is needed. The user may not print a Caution Tag or Blood Transfusion Record Form; the unit status does not change to “crossmatched.”</p>



## ***Issue Blood Components***

The user issues one or more patient-assigned blood units to a transporter for storage in a remote location or for possible transfusion.

### **Assumptions**

- The facility has a procedure for the emergency issue of blood products.
- The unit and the user are in the same division.
- The unit is assigned and/or crossmatched for the patient currently selected for issue.
- VistA is available for the user to choose a physician and valid hospital locations.
- All issue-to locations must be valid entries in the VistA HOSPITAL LOCATION (#44).

### **Outcome**

- The unit status was updated to “issued.”
- The unit’s current location was updated to the “issue-to” location.

### **Limitations and Restrictions**

- VBECS does not check for restricted units (not assigned to the patient) received between selection and issue.
- All issue-to locations must be valid entries in the VistA hospital location file.
- If VistA link is not available, the user may proceed with the issue process but the current issue-to hospital location is not recorded. All of the associated report records display “VistALink Down” in the Issue To location field.
- ISBT 128 tables include blood product codes labeled as not for manufacturing or transfusion. VBECS does not restrict such products from being entered, issued, or transferred (local policy controls their use) and does not accommodate blood modification for these products.
- When a site has configured subservient VistA institutions to the VBECS division, users can only issue units to locations in the primary VistA. Sites may add locations to VistA to account for the relocations to the subservient VistA institutions.
- BCE information applies ONLY when a BCE COTS product is available for communication and is configured.
- Changing a patient's transfusion requirements (component requirements, antigen negative requirement or antibody identified) after a unit is crossmatched does not provide an alert at Issue Unit regarding the type of crossmatch (electronic or serologic) performed on a unit available for issue.
- VBECS asks for a comment when issuing a unit that is assigned to two different patients. Enter a comment to issue the unit or release one of the patient assignments before issuing. The comment will appear on the Unit History Report in the Issue section.

### **Additional Information**

- When no VistA order is available, the user must process the request manually until VistA provides them to VBECS. The user must print blank Caution Tags and Blood Transfusion Record Forms (BTRFs) to accommodate this rare occurrence. When VistA does not provide a pseudonym or associated pseudo-ID for the patient that allows placement of a VistA order, VBECS cannot create a patient order for emergency issue of blood.

- “Issue with Medical Director Approval” components must be approved prior to this option and do not require special handling by VBECS.
- Emergency issue processing accommodates the issue of blood to patients without completed specimen testing, as required.
- Sites may enter the lot number of blood administration sets and leukoreducing filters when they enter the name of the individual picking up the blood product. VBECS retains this information in the unit history record.
- User must date all signature lines on the BTRF dating of the signatures [i.e., Inspected and issued by, Issued to, First Identifier, Second Identifier, Transfusion Data section completed by, Transfusion Reaction Data section completed by (as needed)].


### User Roles with Access to This Option


All users

*To correct issue time and other errors, return the unit from issue and reissue it with the correct information.*


### Issue Blood Components (Routine)

The user issues assigned or crossmatched blood to a transporter for patient transfusion at valid inpatient and outpatient hospital locations, and records issue details, including the transporter’s name or identifier.

User Action	VBECS
1. Select <b>Patients</b> from the main menu.  Select <b>Issue Blood Components</b> .	<ul style="list-style-type: none"> <li>• Displays options for processing patient-related functions.</li> <li>• Displays an option to scan the Caution Tag or enter a patient name or ID.</li> </ul> <p><b>NOTES</b> _____</p> <p>The user must verify the correct patient selection.</p> <p>The &gt;&gt; Enhanced Technologist or above is authorized to issue expired blood products.</p> <p>The &gt;&gt;&gt; Lead Technologist or above is authorized to issue antigen positive and untyped red blood cells.</p>
2. Select a patient.	<ul style="list-style-type: none"> <li>• Displays patient data, including entries that meet the entered patient criteria, and allows the user to select the correct patient.</li> </ul> <p><b>NOTES</b> _____</p> <p> See Table 7 for alerts that may occur during this option.</p> <p>Not all possible error messages may display when multiple messages may apply to the unit/patient. Correction of any one of the patient/unit problems and subsequent re-attempts to issue the unit to the patient will display another of the possible messages until all errors are corrected.</p>
3. Click <b>OK</b> to verify that the correct patient was selected.	<ul style="list-style-type: none"> <li>• Displays fields for the user to enter data for the issue transaction.</li> </ul>

User Action	VBECS
<p>4. Enter the required data in the transporter name, issue-to location, and date and time fields (Figure 103).</p> <p>Click <b>OK</b>.</p>	<p><b>NOTES</b> _____</p> <p>The user may edit the transfusion-ordering clinician, which does not change the blood-ordering clinician.</p> <p>When the user updates the transfusion-ordering-clinician field, VistALink queries VistA for valid providers only. The user may narrow the search by entering the provider's name.</p> <p>The user may edit the issue date and time (default: current date and time) to allow retrospective data entry; no future issue date or time is allowed. When the issue time is incorrect, the user must undo the issue and reissue the unit with the correct date and time.</p> <p>A remote storage indicator is optional (default: No). The transporter name field allows free text and does not display a selection list. The issue-to location lists current hospital locations provided by VistALink, including the inpatient or outpatient location indicator (VistA refers to this as a "patient class"). This is related to billing reimbursement and is not an enabled field in the issue option.</p> <p>A user could return the unit from issue and reissue it with a hospital location when VistALink is up again.</p> <p>When a remote storage location indicator is checked, the user may select from a list of previously entered locations or enter a new, free-text location. Each location entered and saved is available for future selection as a remote location.</p> <p>When the component order is for PLATELETS or OTHER, the units with = &gt; 2 mL of RBC contamination must be presented and checked for compatibility and will display before products with less than (&lt;) 2 mL of RBC contamination. The system will present all available units when the Product Type is PLATELETS or OTHER non-plasma components that have &lt; 2 mL RBC contamination.</p>
<p>5. Scan the unit ID and product code, or click one or more check boxes to select units for issue to the selected patient (Figure 104).</p>	<ul style="list-style-type: none"> <li>• Displays the option to select units from the Assigned Units or Emergency Issue Units tabs.</li> <li>• Lists the selected units, allows the user to select additional units or deselect units, and prompts the user to indicate that the selections are complete.</li> <li>• Allows the user to select the unsatisfactory unit and edit its issue selections.</li> </ul> <p><b>NOTES</b> _____</p> <p> See Table 7 for alerts that may occur during this option.</p> <p>When the user attempts to issue a unit with an inconclusive crossmatch result, VBECS warns the user that the repeat crossmatch is required before the unit can be issued.</p>

User Action	VBECS
	<p>When the user attempts to issue a unit that is not assigned or crossmatched to the patient, VBECS warns the user that the unit is not assigned to the patient and allows the user to enter another unit number.</p> <p>When the current division is configured as "full service" in UC_09 Maintain VAMC Division, ABO confirmation is required for all blood units contained in the WB or RBC or PLT or OTHER units with = &gt; 2 mL RBC contamination When performed, ABO confirmation must match unit ABO at log-in.</p> <p>The unit must have a satisfactory confirmation result before it is considered available for issue and the Blood Transfusion Record Form and Unit Caution Tag may be generated.</p> <p>Rh confirmation results are required on all blood units from the "WHOLE BLOOD", "RED BLOOD CELLS" component classes or OTHER units with = &gt; 2 mL RBC contamination labeled as Rh negative during shipment receipt. When performed, Rh confirmation must match Rh at log-in.</p> <p>When a current specimen is required, VBECS compares the current and previous ABO/Rh results. It does not evaluate results marked "Test Result Invalidated."</p> <p>When issuing blood components from the Any unit that contains &gt; = 2 mL of RBC contamination, the associated specimen's TAS must be completed and the unit status must be crossmatched to the selected patient to routine issue. Optionally, the unit may be emergency issued.</p> <p>Whole blood is not allowed for the emergency issue process. Completed Type and Screen and compatible crossmatch tests are required for whole blood issuance.</p>
<p>6. Click <b>OK</b> to save and complete the unit issue.</p>	<ul style="list-style-type: none"> <li>• Displays entries for review and editing.</li> </ul> <p><b>NOTES</b> _____</p> <p>When the user selects multiple units for a patient and creates a batch for issue, all units in the batch are issued simultaneously to the same location. VBECS assigns the issue information to all units in the batch. The user may not enter different information, including any override comment, for an individual unit in the batch.</p> <p>If the user clicks Cancel, VBECS closes without displaying a confirmation message that data will be lost.</p>
<p>7. Click <b>Yes</b> when the unit is satisfactory to save, or <b>No</b> when the unit is unsatisfactory.</p>	<ul style="list-style-type: none"> <li>• Saves the data.</li> <li>• Displays the unit inspection indicator (satisfactory or unsatisfactory) listing each selected unit.</li> <li>• Sends updated patient and blood unit information to the BCE COTS system via an HL7 message.</li> <li>• Sends updated patient and blood unit information to the BCE COTS system via an HL7 message. The message contains the date time when VBECS creates the message and not the date time the user selects the unit for issue.</li> </ul>

User Action	VBECS
	<p><b>NOTES</b></p> <p> See Table 7 for alerts that may occur during this option.</p> <p>Sites using BCE COTS must issue blood components in real-time for BCE COTS to be able to perform transfusion verification.</p> <p>The area to mark a unit unsatisfactory for issue extends beyond the checkbox. A user that clicks in that area, but not the checkbox, could inadvertently mark a unit unsatisfactory for issue. A VBECS Confirmation message window appears to verify the user wants to remove patient association and quarantine the unit. The user must click No if he mistakenly indicated that the unit was unsatisfactory for issue.</p>
8. Exit.	

**Figure 103: Issue Blood Components (1)**

**VBECS - Issue Blood Components**

**Issue Blood Components**

**ISSUE**

**Issue Details**

Requesting MD: USER, ONE [Find]

Issue Date\*: 08/12/2011 07:19

Issued By\*: Test User1

**Issue-To**

Transporter\*: Careful Carrier

Location Name\*: 3N CCU-KC

Remote Storage Location

Remote Name: [ ]


\* Required Field



OK Cancel

**Figure 104: Issue Blood Components (2)**

**Issue Blood Components (Emergency)**


The user issues assigned or crossmatched blood to a transporter for patient transfusion that does not meet routine issue conditions at valid inpatient and outpatient hospital locations, and records issue details, including the transporter’s name or identifier.

User Action	VBECS
1. Select <b>Patients</b> from the main menu.  Select <b>Issue Blood Components</b> .	<ul style="list-style-type: none"> <li>Displays options for processing patient-related functions.</li> <li>Displays an option to scan the Caution Tag or enter a patient name or ID.</li> </ul>
2. Select a patient.	<ul style="list-style-type: none"> <li>Displays patient data, including entries that meet the entered patient criteria, and allows the user to select the correct patient.</li> </ul> <p><b>NOTES</b> _____</p> <p> See Table 8 for alerts that may occur during this option.</p>
3. Click <b>OK</b> to verify that the correct patient was selected.	<ul style="list-style-type: none"> <li>Displays fields for the user to enter the data for the issue transaction.</li> </ul>
4. Enter the required data.	<ul style="list-style-type: none"> <li>Displays the option to select units from the Assigned Units or Emergency Issue Units tabs.</li> </ul>

User Action	VBECs
<p>5. Click the <b>Emergency Issue Units</b> tab.</p> <p>Scan or click one or more check boxes to select units for issue to the selected patient (Figure 105).</p> <p>Click <b>OK</b>.</p>	<ul style="list-style-type: none"> <li>Lists the selected units, allows the user to select additional units or deselect units, and prompts the user to indicate that the selections are complete.</li> <li>Allows the user to select the unsatisfactory unit and edit its issue selections.</li> </ul> <p><b>NOTES</b> _____</p> <p> See Table 8 for alerts that may occur during this option.</p>
<p>6. Click <b>OK</b> to accept the entries and complete the unit issue.</p>	<ul style="list-style-type: none"> <li>Displays entries for review and editing.</li> </ul>
<p>7. Click <b>Yes</b> when the unit is satisfactory or <b>No</b> when the unit is unsatisfactory.</p>	<ul style="list-style-type: none"> <li>Saves the data.</li> <li>Displays the unit inspection indicator (satisfactory or unsatisfactory) listing each selected unit.</li> <li>Sends updated patient and blood unit information to the BCE COTS system via an HL7 message.</li> </ul> <p><b>NOTES</b> _____</p> <p> See Table 8 for alerts that may occur during this option.</p>
<p>8. Exit.</p>	

**Figure 105: Issue Blood Components Emergency Issue Units Tab**

**Figure 106: Emergency Issue**

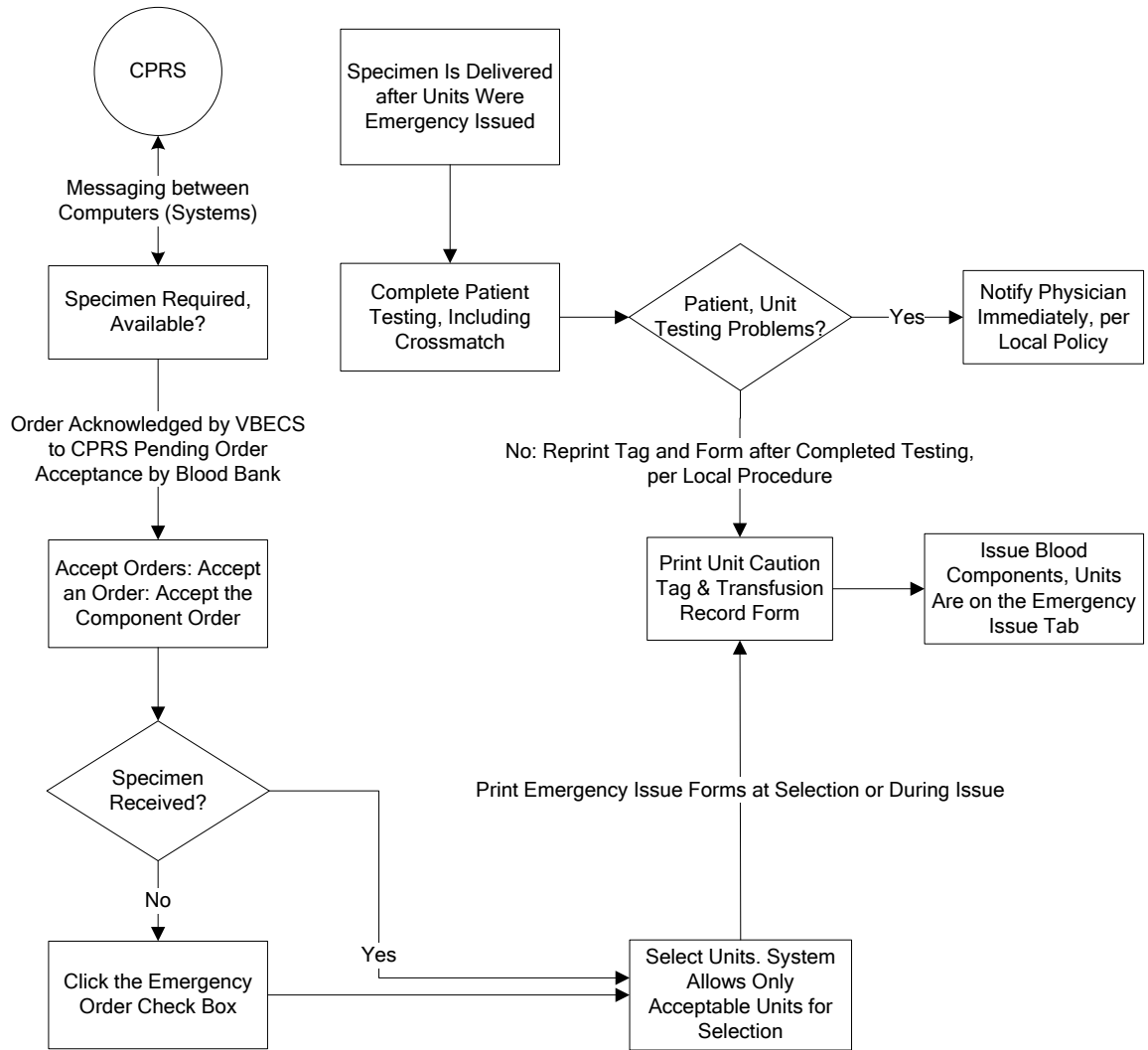


Emergency issue in VBECS requires a CPRS order and a VistA Laboratory specimen UID. VBECS allows a user to bypass this requirement to issue units when a specimen is not received.



## Alerts


**Table 7: Alerts That May Occur in Issue Blood Components (Routine)**

User Action Step	Alerts
2	VBECs displays  and emits an audible alert when there are Special Instructions (SIs) or Transfusion Requirements (TRs) for the patient.
2	When no patient is found, VBECs warns the user and does not allow the user to continue.
2	When the patient has no components available for issue, VBECs warns the user and does not allow the user to continue.
2	When required testing is incomplete at issue, VBECs emits an audible alert, warns the user, and asks whether to emergency issue the unit in accordance with hospital policy.  <b>No</b> stops the user from issuing the unit and requires selection of another unit. <b>Yes</b> creates an Exception Report entry for the issue, indicates on the unit record that the unit was issued emergently, and asks whether the user wishes to print the Caution Tags and Blood Transfusion Record Forms. <b>No</b> allows the user to continue with the emergency issue. <b>Yes</b> prints a Caution Tag and/or Blood Transfusion Record Form and proceeds with the issue.
5	When the user attempts to issue a unit located in a different division, VBECs warns him that the unit cannot be issued and does not allow him to continue.
5	When autologous or directed, or designated or dedicated blood units of the same component class are available in the same division as the user, VBECs requires blood units to be issued in this order: <ol style="list-style-type: none"> <li>1. Autologous units</li> <li>2. Restricted units</li> <li>3. Voluntary allogeneic units</li> </ol> <p>When the user attempts to issue a unit out of order, VBECs warns him that more restrictive units are available for issue and asks whether he wishes to continue. <b>No</b> allows the user to enter another unit number. <b>Yes</b> allows the user to continue and captures transaction details for inclusion in an Exception Report (exception type: less restrictive unit issued).</p>
5	When a user indicates that the inspected unit is unsatisfactory, VBECs does not allow the unit to be issued, releases it from assignment, and warns the user that the unit is unacceptable for transfusion. VBECs allows the user to enter another unit number.
5	When the user attempts to select frozen blood components, VBECs warns the user. Units selected in their frozen state must be thawed before issue.
5	When the patient has Component Requirements (CRs) and Antigen Negative Requirements, the user attempts to issue (including emergency issue) red blood cells that do not satisfy the requirements, and the user is: <ul style="list-style-type: none"> <li>➤➤ An Enhanced Technologist or below, VBECs emits an audible alert and warns the user that the unit is not selectable and he is not authorized to issue the unit. The unsatisfied requirements are not detailed. VBECs does not allow an override.</li> </ul>
5	When the patient has Component Requirements, the user attempts to issue (including emergency issue) red blood cells that do not satisfy the requirement, and the user is: <ul style="list-style-type: none"> <li>• ➤➤ An Enhanced Technologist or below, VBECs emits an audible alert and warns the user that the patient's component requirement is not satisfied and he is not authorized to issue the unit. VBECs does not allow an override.</li> <li>• ➤➤➤ A Lead Technologist or above, VBECs emits an audible alert and warns the user that the patient's component requirement is not satisfied and allows for an override. <b>No</b> clears the screen. <b>Yes</b> requires a comment and captures details for inclusion in an Exception Report (exception type: unit issued with unsatisfied TR).</li> </ul>
5	When a repeat ABO/Rh performed on the patient specimen does not match the original ABO/Rh interpretation and the unit doesn't qualify for emergency issue, VBECs warns the user and instructs him to resolve the discrepancy before issuing the unit. There is no override. Units that qualify for emergency

User Action Step	Alerts
	issue may continue and VBECS will display a message that the ABO/Rh on the current specimen does not agree with the historic ABO/Rh.
5	When a unit has a biohazard indicator, VBECS emits an audible alert, warns the user, and asks whether he still wishes to issue the unit. <b>No</b> clears the entry and allows the user to select a new unit. <b>Yes</b> requires a comment and captures details of the transaction for inclusion in an Exception Report (exception type: biohazardous unit issued).
5	<p>For an Rh negative, unknown, or inconclusive patient: the user may enter and confirm Rh positive or Rh pooled units from the WHOLE BLOOD, RED BLOOD CELLS, PLATELETS, and OTHER VBECS component classes. When VBECS component classes are individually entered and confirmed by the user, VBECS emits an audible alert, warns the user, and asks whether he wishes to issue the unit. <b>No</b> clears the unit data and allows the user to select a different unit. <b>Yes</b> requires additional comment and captures details for inclusion in an Exception Report (exception type: antigen positive/untested units issued).</p> <p>Additional details to be included in the exception report when the user overrides this business rule:</p> <ul style="list-style-type: none"> <li>• Transaction Type: Antigen positive/untested units issued</li> <li>• Patient Antibody Identified</li> <li>• Unit Antigen Phenotype Result</li> </ul>
5	<p>The system will not allow the issue of units assigned/crossmatched or that require an ABO/Rh associated with an expired specimen when compared to the issue date and time unless the user role is Enhanced Technologist, VBECS security levels 2 and above.</p> <p>When the user role is Technologist, VBECS security level 1, the system will display the warning message: Patient specimen is expired. You do not have the proper security to issue this unit. The system will not allow an override.</p> <p>When the user role is Enhanced Technologist, VBECS security levels 2 and above, the system will provide an audible alert and display the warning message: Patient specimen is expired. Continue to issue? Yes/No. If the user responds "No" the system will clear the screen. If the user responds "Yes" the system will require a comment (free text or canned, "unit issue" context) and captures the transaction details for inclusion in an exception report entry for the issue.</p> <p>Additional details to be included in the exception report when the user overrides this business rule:</p> <ul style="list-style-type: none"> <li>• Transaction Type: Unit issued on expired specimen</li> <li>• Specimen UID</li> </ul>
5	<p>When the user attempts to issue an expired unit, VBECS warns him. There is no override.</p> <p>➤➤ When the user is an Enhanced Technologist or above, VBECS emits an audible alert, warns him, and asks whether he wishes to continue to issue. <b>No</b> clears the screen. <b>Yes</b> requires a comment and captures details for inclusion in an Exception Report (exception type: issued expired unit).</p>
5	<p>When the user selects units from the WHOLE BLOOD or RED BLOOD CELLS component classes, VBECS determines when the patient has a calculated or a persistent antigen negative requirement. This does not apply when selecting autologous (ISBT 128 units with an autologous donation type). When the selected unit was antigen typed and is positive for the corresponding antigen or is untested for the antigen, VBECS warns the user and requires action based on his security level.</p> <p>➤➤ When the user is an Enhanced Technologist or below, VBECS warns that he does not have the proper security level to issue the unit. There is no override.</p> <p>➤➤➤ When the user is a Lead Technologist or above, VBECS emits an audible alert, warns him, and asks whether he wishes to continue to issue. <b>No</b> clears the screen. <b>Yes</b> requires a comment and captures details for inclusion in an Exception Report entry (exception type: antigen-positive/untested units issued).</p>

User Action Step	Alerts
	Additional details to be included in the exception report when the user overrides: <ul style="list-style-type: none"> <li>• Transaction Type: Antigen-positive/untested units issued</li> <li>• Patient Antibody(ies) Identified</li> <li>• Unit Antigen Phenotype Result or “untested”</li> </ul>
5	VBECs compares the ABO/Rh of the current specimen to the historic ABO/Rh. When there is a discrepancy, there is no previous justification for the discrepancy on the patient record, as described in Justify ABO/Rh Change, and the unit does not qualify for emergency issue, VBECs warns the user and instructs him to resolve the discrepancy before issuing the unit. There is no override. VBECs exits the option, clears the screen, and returns the user to the point of entry. Units that qualify for emergency issue may continue and VBECs will display a message that the ABO/Rh on the current specimen does not agree with the historic ABO/Rh.
5	When no valid ABO exists for the patient, the user may select components, including autologous units: <ul style="list-style-type: none"> <li>• For RBC or WB order: group O, Rh positive or O Rh negative RBC. WHOLE BLOOD is not selectable.</li> <li>• For FFP order only: group AB, Rh positive or Rh negative.</li> <li>• For PLT, CRYO, or OTHER order, all available units are selectable.</li> </ul>
5	When the division is “transfusion only” and a blood unit is ABO incompatible at issue based on the interpretation of the current patient specimen’s ABO test, VBECs warns the user that the unit is not ABO compatible for the patient’s current specimen and may not be issued. The user must resolve pending ABO discrepancies before issuing the unit.
7	When the division is “full service” and a blood unit is ABO incompatible at issue based on the system interpretation of the current patient specimen’s ABO test result, VBECs emits an audible alert, warns that the unit is not ABO compatible with the patient’s current specimen results, and asks whether he wishes to continue. <b>No</b> clears the screen and allows the user to select another unit. <b>Yes</b> requires a comment and captures details for inclusion in an Exception Report (exception type: ABO discrepant unit issue).

**Table 8: Alerts That May Occur in Issue Blood Components (Emergency)**

User Action Step	Alerts
2	VBECs displays  and emits an audible alert when there are Special Instructions (SIs) or Transfusion Requirements (TRs) for the patient.
2	When no patient is found, VBECs warns the user and does not allow the user to continue.
2	When the patient has no components available for issue, VBECs warns the user and does not allow the user to continue.
2	When required testing is incomplete at issue, VBECs emits an audible alert, warns the user, and asks whether to emergency issue the unit in accordance with hospital policy.  <b>No</b> stops the user from issuing the unit and requires selection of another unit. <b>Yes</b> creates an Exception Report entry for the issue, indicates on the unit record that the unit was issued emergently, and asks whether the user wishes to print the Caution Tags and Blood Transfusion Record Forms. <b>No</b> allows the user to continue with the emergency issue. <b>Yes</b> prints a Caution Tag and/or Blood Transfusion Record Form and proceeds with the issue.
5	When the user attempts to issue a unit located in a different division, VBECs warns him that the unit cannot be issued and does not allow him to continue.
5	When a user indicates that the inspected unit is unsatisfactory, VBECs does not allow the unit to be issued, releases it from assignment, and warns the user that the unit is unacceptable for transfusion. VBECs allows the user to enter another unit number.
5	When the user attempts to select frozen blood components, VBECs emits an audible alert and warns the user. Units selected in their frozen state must be thawed before issue.

User Action Step	Alerts
5	When the patient has Component Requirements and Antigen Negative Requirements, the user attempts to issue (including emergency issue) red blood cells that do not satisfy the requirements, and the user is: >> An Enhanced Technologist or below, VBECS emits an audible alert and warns the user that the unit is not selectable and he is not authorized to issue the unit. The unsatisfied requirements are not detailed. VBECS does not allow an override.
5	When the patient has Component Requirements, the user attempts to issue (including emergency issue) red blood cells that do not satisfy the TR, and the user is: <ul style="list-style-type: none"> <li>• &gt;&gt; An Enhanced Technologist or below, VBECS warns the user that he is not authorized to issue untyped red blood cells (for the clinically significant antibodies in the table). VBECS does not allow an override.</li> <li>• &gt;&gt;&gt; A Lead Technologist or above, VBECS emits an audible alert and warns the user. <b>No</b> clears the screen. <b>Yes</b> requires a comment and captures details for inclusion in an Exception Report (exception type: unit issued with unsatisfied TR).</li> </ul>
5	When a repeat ABO/Rh performed on the patient specimen does not match the original ABO/Rh interpretation and the unit does not qualify for emergency issue, VBECS warns the user and instructs him to resolve the discrepancy before issuing the unit. There is no override. Emergency issue may continue for qualifying units.
5	When a unit has a biohazard indicator, VBECS emits an audible alert, warns the user, and asks whether he still wishes to issue the unit. <b>No</b> clears the entry and allows the user to select a new unit. <b>Yes</b> requires a comment and captures details of the transaction for inclusion in an Exception Report (exception type: biohazardous unit issued).
5	For an Rh negative, unknown, or inconclusive patient, the user may enter and confirm Rh positive and Rh pooled units from the WHOLE BLOOD, RED BLOOD CELLS, PLATELETS, and OTHER VBECS component classes. VBECS emits an audible alert, warns that the patient is Rh negative and the selected unit is Rh positive, instructs the user to check the patient sex and hospital policy before continuing, and asks whether the user wishes to continue to select the unit. <b>No</b> clears the unit data and allows the user to select a different unit. <b>Yes</b> requires additional comment and captures details for inclusion in an Exception Report (exception type: antigen positive/untested units issued).
5	When the user attempts to issue an expired unit, VBECS emits an audible alert and warns him. There is no override.  >> When the user is an Enhanced Technologist or above, VBECS emits an audible alert, warns him, and asks whether he wishes to continue to issue. <b>No</b> clears the screen. <b>Yes</b> requires a comment and captures details for inclusion in an Exception Report (exception type: issued expired unit).
5	When the user selects units from the WHOLE BLOOD or RED BLOOD CELLS component classes, VBECS determines when the patient has a calculated or a persistent antigen negative requirement. This does not apply when selecting autologous (ISBT 128 units with an autologous donation type). When the selected unit was antigen typed and is positive for the corresponding antigen or is untested for the antigen, VBECS warns the user and requires action based on his security level.  >> When the user is an Enhanced Technologist or below, VBECS emits an audible alert and warns that he does not have the proper security level to issue the unit. There is no override.  >>> When the user is a Lead Technologist or above, VBECS emits an audible alert, warns him, and asks whether he wishes to continue to issue. <b>No</b> clears the screen. <b>Yes</b> requires a comment and captures details for inclusion in an Exception Report entry (exception type: antigen-positive/untested units issued). Additional details to be included in the exception report when the user overrides: <ul style="list-style-type: none"> <li>• Transaction Type: Antigen-positive/untested units issued</li> <li>• Patient Antibody(ies) Identified</li> <li>• Unit Antigen Phenotype Result or "untested"</li> </ul>
5	VBECS compares the ABO/Rh of the current specimen to the historic ABO/Rh. When the unit was electronically crossmatched, the current and historic ABO/Rh results must have been recorded in the login user's division. When there is a discrepancy, there is no previous justification for the discrepancy

User Action Step	Alerts						
	<p>on the patient record, as described in Justify ABO/Rh Change, and the unit does not qualify for emergency issue, VBECS warns the user and instructs him to resolve the discrepancy before issuing the unit. There is no override. VBECS exits the option, clears the screen, and returns the user to the point of entry. Emergency issue may continue for qualifying units.</p>						
5	<p>When no valid ABO exists for the patient, the user may select components, including autologous units:</p> <ul style="list-style-type: none"> <li>• For RBC or WB order: group O, Rh positive or O Rh negative RBC. WHOLE BLOOD is not selectable.</li> <li>• For FFP order only: group AB, Rh positive or Rh negative.</li> <li>• For PLT, CRYO, or OTHER order, all available units are selectable.</li> </ul>						
5	<p>□When the user selects blood components from the frozen product types for a patient, VBECS emits an audible alert and warns that the selected blood unit must be thawed and may require additional processing prior to issue. VBECS updates the blood unit status to “assigned” and the unit is no longer available for issue.</p>						
5	<p>When a Caution Tag or BTRF is printed for an emergency-issued unit, “Emergency Issue” is included on the form.</p> <p>The recipient ABO/Rh prints as “Unk” or “Unknown” during emergency issue when no current specimen results are on file; VBECS may not use a historic record blood type (from any source) for the typing printed on the tag/form.</p> <table border="1" data-bbox="310 884 1073 968"> <thead> <tr> <th data-bbox="310 884 691 911">Compatibility Interpretation</th> <th data-bbox="691 884 1073 911">Text on BTRF</th> </tr> </thead> <tbody> <tr> <td data-bbox="310 911 691 938">Crossmatch</td> <td data-bbox="691 911 1073 938">TESTING INCOMPLETE</td> </tr> <tr> <td data-bbox="310 938 691 968">Crossmatch Not Required</td> <td data-bbox="691 938 1073 968">Not Required</td> </tr> </tbody> </table> <p>VBECS prints “Emergency Issue” in the TR section of the form.</p>	Compatibility Interpretation	Text on BTRF	Crossmatch	TESTING INCOMPLETE	Crossmatch Not Required	Not Required
Compatibility Interpretation	Text on BTRF						
Crossmatch	TESTING INCOMPLETE						
Crossmatch Not Required	Not Required						
5	<p>When blood products are emergency issued (units has an emergency issue” order indicator, or the testing is incomplete or inconclusive on the current specimen), the remarks section of the Blood Transfusion Record Form includes “This unit is issued for transfusion BEFORE the completion of required compatibility testing. The blood bank staff will continue to test the compatibility of the unit using the current patient specimen. If you are alerted to a unit’s incompatibility by the blood bank, stop the transfusion immediately.”</p> <p>When “****” prints for the patient ABO/Rh and crossmatch, the form includes “Specimen testing incomplete at time of issue.”</p>						
5	<p>When a unit was issued and a conflicting test result found that indicates a potential unit incompatibility with the patient, such as a patient ABO/Rh discrepancy, the unit XM is incompatible with the patient, the patient ABS is positive, the patient DAT is positive, or the unit is antigen positive where antigen negative is required on the tested specimen, VBECS warns the user and instructs him to notify the physician of this potential problem immediately, according to hospital policy. VBECS warns the user during the execution of the various tests related to the compatibility of a blood unit and captures details for inclusion in an Exception Report (exception type: unit issued, testing problem).</p>						
7	<p>When the division is “full service” and a blood unit is ABO incompatible at issue based on the system interpretation of the current patient specimen’s ABO test result, VBECS emits an audible alert and warns that the unit is not ABO compatible with the patient’s current specimen results and asks whether he wishes to continue. No clears the screen and allows the user to select another unit. Yes requires a comment and captures details for inclusion in an Exception Report (exception type: ABO discrepant unit issue).</p>						

## ***Return Issued Units To Blood Bank***

An issued unit was returned to the blood bank in satisfactory condition after issue to a valid hospital location.

### **Assumptions**

- The user is logged into the division that contains the unit.
- The blood unit is in the “issued” status.
- The connection to VistA is active.

### **Outcome**

- The unit is returned to a hospital location of a blood bank.
- The unit retains its patient assignment at the time of issue when returned in satisfactory condition.
- The user may mark the unit “unsatisfactory” and quarantine the unit to generate an Exception Report, if needed per hospital policy.

### **Limitations and Restrictions**

- None

### **Additional Information**

- None

### **User Roles with Access to This Option**

All users

## Return Issued Units To Blood Bank

An issued unit was returned to the blood bank in satisfactory condition after issue to a valid hospital location. The user updates VBECS to reflect the change in location.

User Action	VBECS
<p>1. Select <b>Blood Units</b> or <b>Patients</b> from the main menu.</p> <p>Select <b>Return Issued Units To Blood Bank</b>.</p>	<ul style="list-style-type: none"> <li>Displays options for processing blood units or for processing patient-related functions.</li> <li>Displays the return date and time and the name of the user who returned the issued unit.</li> </ul> <p><b>NOTES</b> _____</p> <p>The user may edit the return date and time (default: current date and time) to a past date and time.</p> <p>VBECS assigns the same return date and time to all units processed in a batch.</p>
<p>2. Enter or edit the date and time and/or select a user, or click <b>OK</b> to accept the default data.</p>	
<p>3. Enter the patient data.</p>	<ul style="list-style-type: none"> <li>Prompts the user to enter the unit ID and product code or enter patient data to retrieve the records for the units issued.</li> <li>Displays an option to search for or enter a patient or unit based on the original menu option selected.</li> </ul>
<p>4. Select a patient or unit.</p>	<ul style="list-style-type: none"> <li>Lists the units in "issued" status that are assigned to the patient.</li> </ul>
<p>5. Select one or more units to return from issue.</p>	<ul style="list-style-type: none"> <li>Displays the units selected for return for review and allows the user to select or deselect additional units.</li> </ul>
<p>6. Scan the unit ID and product code, or click one or more check boxes to select units to return from issue.</p>	<ul style="list-style-type: none"> <li>Displays the data entered and allows the user to enter additional units and indicate that he completed the entry for this transaction.</li> <li>Displays data for each unit entered.</li> </ul> <p><b>NOTES</b> _____</p> <p>When a selected unit is not in "issued" status, VBECS warns the user, clears the unit field, and allows the user to enter a different unit number.</p> <p>VBECS allows the user to select multiple units to create a worklist for simultaneous processing.</p> <p>VBECS requires the user to enter a "returned-from" location (default: "issue-to" location) and allows the user to change this location to a different valid hospital location. To do this, click the <b>find</b> button and select a hospital location.</p> <p>The system provides an audible alert and displays a warning message to the user for each unit returned after issue to a non-remote storage location: "Unit has been out of controlled storage. Check division policy for unit acceptability." The system will require the user to enter a comment regarding the return ("unit return" context). The system will create an exception report entry for the unit accepted from prolonged issue in uncontrolled storage. Exception report entry details include:</p> <ul style="list-style-type: none"> <li>Transaction Type: Unit out of controlled storage found acceptable</li> </ul>


User Action	VBECS
	<ul style="list-style-type: none"> <li>• Patient Name</li> <li>• Patient ID</li> <li>• Patient ABO/Rh</li> <li>• Returned Unit ID</li> <li>• Returned Unit Product Short Name</li> <li>• Returned Unit ABO/Rh</li> <li>• Returned Unit Expiration Date</li> <li>• Issue-to Location</li> <li>• Visual Inspection Information (Note that although the exception type display on the report is “Unit out of controlled storage found acceptable”, the Visual Inspection Information will display “Unsatisfactory” if entered by the user in the comment field.</li> <li>• Required Comment</li> <li>• Processing Tech Name</li> <li>• Log-in Tech Name</li> <li>• Division</li> <li>• Date/Time of Transaction</li> </ul>
<p>7. Respond to the warnings and messages (Figure 107).</p>	<ul style="list-style-type: none"> <li>• Allows the user to indicate that each unit or batch of units was visually inspected and marked satisfactory or unsatisfactory.</li> </ul> <p><b>NOTES</b> _____</p> <p> When the user indicates that the unit is unsatisfactory, VBECS emits an audible alert and quarantines the unit, relocates it to the hospital location of the blood bank, and captures details for inclusion in an Exception Report (exception type: unit unsatisfactory upon return from issue). Upon quarantining the unit, VBECS releases it from assignment and requires a comment regarding the return.</p> <p>When a unit has had a quarantine indicator assigned, VBECS releases any patient assignments. Patient restrictions associated with the unit remain intact.</p> <p>VBECS places units returned from issue to the blood bank in satisfactory condition in the status they had prior to issue.</p> <p>When the unit returned and/or the specimen used for assignment or crossmatch is expired, VBECS warns the user.</p>
<p>8. Click <b>OK</b> to accept the entries, and complete the unit return.</p> <p>Exit.</p>	<ul style="list-style-type: none"> <li>• Saves the data.</li> <li>• Sends update to the BCE COTS system via an HL7 message to remove the unit from the system.</li> </ul> <p><b>NOTES</b> _____</p> <p>VBECS returns units to the blood bank location within the division.</p>



Figure 107: Return Issued Units

**VBECS - Return Issued Units**

Name: VBPATIENT, EIGHTYONE    Patient ID: 000-00-0081    ABO/Rh: O Neg

Return Date: 2/7/2006 5:49:54 PM    Returned By: yhaisvbecs1

Return?	Unit ID	Product	ABO/Rh	Expires
<input checked="" type="checkbox"/>	6873183	RBC AS1, 04210	O Neg	2/28/2006 11:59:00 PM

**Selected Unit**

**Details**

Donation Type: Volunteer Allogeneic Donor  
 Date Received: 01/31/2006 10:05  
 Unit Status: Issued  
 Volume (mL): 350

**Unit Issue Details**

Issue Date: 2/7/2006 5:24:56 PM  
 Transporter: Careful Carrier  
 Issued To: LE-BH NUTRITION  
 Remote Storage Location: Identity

**Unit Return Details**

Returned From\*: LE-BH NUTRITION  
 Unit Unsatisfactory for Return

\* Required Field    OK    Cancel

## Release Units From Patient Assignment

The user releases crossmatched and assigned blood units not issued after a period determined by regulatory standards and by internal policy.

### Assumptions

- The blood unit is assigned or crossmatched to at least one patient.

### Outcome

- Patient restrictions still apply to a unit even after it is released from assignment and made available.

### Limitations and Restrictions

- This option does not apply to the release of patient restrictions (autologous and directed).
- When a patient, specimen, or blood unit expires, VBECS does not automatically release an assigned blood unit: a user must release the unit.
- Users should select and release units in a timely manner to avoid the units being issued by another user prior to the release process completion.

### Additional Information

- A user may release units from assignment as a group or individually from a patient's list of units.

### User Roles with Access to This Option

All users

## Release Units From Patient Assignment

A user may select one or more blood units for release from patient assignment and return to "available" status. A user may release one of multiple patient assignments without affecting other patient assignments on a unit.

User Action	VBECS
1. To release units by patient, select <b>Patients</b> from the main menu.  To release units by unit, select <b>Blood Units</b> from the main menu.  Select <b>Release Units From Patient Assignment</b> .	<ul style="list-style-type: none"><li>• Displays options for processing patient-related functions or for processing units.</li></ul>

User Action	VBECS
<p>2. Scan or enter a unit ID or select a patient to release units from patient assignment and/or crossmatch (XM).</p>	<ul style="list-style-type: none"> <li>• Allows the user to edit the date and time and the user who released the units.</li> <li>• Lists currently assigned units.</li> <li>• Allows the user to list assigned units by patient, by unit ID, or by unit expiration date.</li> </ul> <p><b>NOTES</b> _____</p> <p>VBECS indicates which specimens and units are expired.</p> <p>VBECS displays a unit's donation type only when it is restricted.</p> <p>VBECS displays all currently assigned/crossmatched units located in the blood bank. When the user enters a unit ID that is not currently in the blood bank location, VBECS warns the user, clears the unit ID entry, and allows the user to enter another unit or patient.</p>
<p>3. Indicate which units to release.</p> <p>Click <b>OK</b>.</p> <p>Indicate a reason for release of units (Figure 108).</p>	<ul style="list-style-type: none"> <li>• Allows the user to enter a reason for releasing for an individual unit.</li> <li>• Allows the user to review the selections, select additional units, and deselect units before saving.</li> </ul> <p><b>NOTES</b> _____</p> <p>VBECS allows the user to select no units or one or more units for each patient on the list.</p>
<p>4. Review the entered information and click <b>OK</b> to save.</p> <p>Click <b>OK</b> to exit.</p>	<ul style="list-style-type: none"> <li>• Updates the patient record, the unit record, and the status.</li> <li>• Sends update to the BCE COTS system via an HL7 message to remove the unit from the system.</li> </ul> <p><b>NOTES</b> _____</p> <p>When a unit has a future expiration date and holds no other assignments, VBECS updates the unit to "available" status. When a unit is assigned to a different patient, it remains assigned to that patient.</p> <p>VBECS alerts the user when the unit is expired and reminds him to remove additional patient assignments and the unit from the available inventory according to local policies. VBECS instructs the user to release the unit from all assignments and remove it from the available inventory per hospital policy.</p>

Figure 108: Confirm Release

The screenshot displays the VBECS software interface for releasing units from a patient assignment. The main window is titled "VBECS - Release Unit(s) from Patient Assignment". It features a search criteria section with fields for Unit ID (1941954), Patient, Unit Expires (Before/After 02/08/2006), and Specimen Expires (Before/After 02/08/2006). Below this is a table titled "Select Assigned Units for Release" with columns for Patient Name, Patient ID, Unit ID, Product, Product Name, ABO/Rh, Unit Expires, and Specimen Expires. A single row is selected, showing Patient Name "VBPAIENT, EIGHTYONE", Patient ID "000-00-0081", Unit ID "1941954", Product "04210", Product Name "RBC AS1", ABO/Rh "O Neg", Unit Expires "02/28/2006 23:59", and Specimen Expires "02/09/2006 23:59".

Overlaid on the main window is a smaller dialog box titled "VBECS - Confirm Release". This dialog box contains the following fields:

- Released By\*: vhaishvbecs2
- Release Date\*: 02/08/2006 11:46
- Enter a comment before releasing the selected unit(s).
- Comment\*: For use by another patient
- Details: (empty text area)

Buttons for "OK" and "Cancel" are present at the bottom of the dialog box. A legend at the bottom left indicates that an asterisk (\*) denotes a required field.

## Print Unit Caution Tag & Transfusion Record Form

The user may print a Caution Tag and/or a Blood Transfusion Record Form (BTRF) when crossmatch compatibility, assignment, or emergency issue are completed or as an independent action.

### Assumptions

- The blood unit, patient order, and user are in the same division.
- There is a current component class or emergency issue order for the patient, and the blood unit printed on the Caution Tag fulfills the patient order requirements.
- When the blood unit belongs to a component class that does not require crossmatch, it has a status of “assigned” (non-emergency issue only).
- For non-emergency issue, the blood unit meets all patient Transfusion Requirements (TRs).

### Outcome

- The barcode generated on the Caution Tag may be used in bedside verification procedures, in accordance with local policy and procedures.
- A user may reprint a BTRF and Caution Tag.

### Limitations and Restrictions

- VBECS cannot reprint a Caution Tag on an expired unit or expired patient order.
- VBECS does not print a unit’s biohazardous status on the BTRF or Caution Tag.
- If a unit has been crossmatched by two different technologists, the Compatibility Interpretation section of the BTRF displays the technologist that performed the earlier XM testing. Handwrite the missing current technologist initials on the printed BTRF.

### Additional Information

- The informational barcode printed on the Caution Tag includes only the patient and unit IDs and the unit product code.
- A user may print multiple Caution Tags and BTRFs for a blood unit when it is assigned to multiple patients.
- Attach the Caution Tag labels to blood bags in accordance with local policy and procedures.

### User Roles with Access to This Option

All users


### Print Backup Caution Tags and Blood Transfusion Record Forms

It is recommended that blank Caution Tags and BTRFs be kept on hand in case the system or printer is not available to generate them.

User Action	VBECS
1. Select <b>Tools</b> from the main menu.  Select <b>Print Backup Forms</b> .  Enter or select the number of blank tags or forms to print (Figure 109).	<ul style="list-style-type: none"> <li>• Displays options for processing administrative functions.</li> <li>• Prompts the user to enter the number of Caution Tags to be printed.</li> <li>• Prints the blank Caution Tags.</li> </ul> <b>NOTES</b> _____

User Action	VBECs
Click <b>OK</b> to print and exit.	<p>VBECs displays an option to print a Caution Tag or a BTRF when the user selects a unit or a patient.</p> <p>The user may print blank Caution Tags and BTRFs to use in case of downtime.</p> <p>VBECs interrupts or cancels a print job to accommodate the printing of an emergency issue tag. VBECs notifies the user of the number of tags printed and instructs the user to enter a new request to print more tags.</p>

**Figure 109: Print Backup Forms**

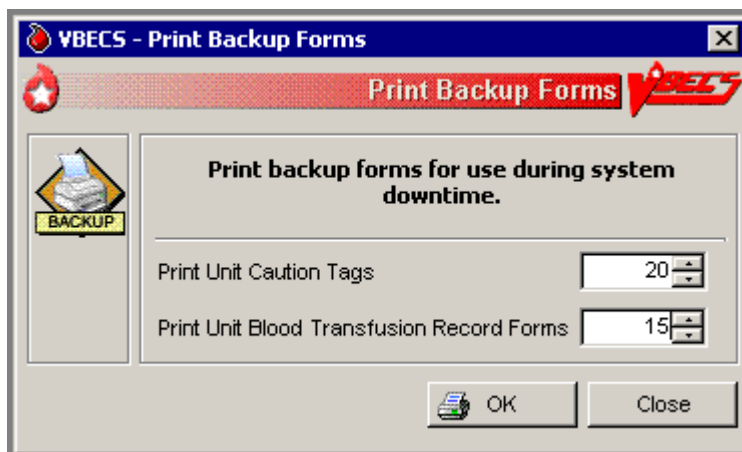


### Print Unit Caution Tag & Transfusion Record Form

The user may need to generate Caution Tags when they were not printed immediately after unit selection and crossmatch data entry, emergency issue, or when there is a problem with the printed Caution Tag. According to local policy, the user selects a current patient order and one or more units to print from the VBECs-generated list of units eligible for immediate transfusion.

VBECs does not allow a user to print a Caution Tag unless the unit is satisfactory for immediate issue.

User Action	VBECs
<p>1. Select <b>Patients</b> or <b>Blood Units</b> from the main menu.</p> <p>Select <b>Print Unit Caution Tag &amp; Transfusion Record Form</b>.</p>	<ul style="list-style-type: none"> <li>Displays options for processing patient-related functions or for processing units.</li> <li>Displays the option to select a patient, if one is not already selected.</li> </ul> <p><b>NOTES</b> _____</p> <p>For patient with an Rh that is not Positive (negative, unknown, or inconclusive): When <b>Rh "Positive" or Rh "Pooled"</b> units from the <b>WHOLE BLOOD, RED BLOOD CELLS, PLATELETS, and OTHER</b> VBECs component classes are individually entered and confirmed by the user, the system provides an audible alert and displays the alert: "Patient is an Rh negative and the selected unit is Rh positive. Check patient sex and hospital policy before continuing. Proceed?" The user will click OK to continue.</p>

User Action	VBECS
<p>2. Enter the patient name or ID and click <b>Search</b> to select a patient, or</p> <p>Click <b>Search by Unit</b> (Figure 110).</p> <p>Enter the unit ID and/or product.</p>	<ul style="list-style-type: none"> <li>• Requests that the user select a Caution Tag and/or a BTRF to print by unit ID or by patient.</li> <li>• Displays an informational message, when appropriate.</li> </ul>
<p>3. Select a patient, or</p> <p>Select or scan a blood unit.</p>	<ul style="list-style-type: none"> <li>• Compiles and displays the information associated with the user's selection.</li> <li>• Displays the patient's name, ID, date of birth, the unit ID, and the long product name of units to appear on the Caution Tags.</li> <li>• Prompts the user to select the units when selecting by patient.</li> <li>• Allows the user to select and deselect blood units to print.</li> <li>•  See Table 9 for alerts that may occur during this option.</li> </ul> <p><b>NOTES</b> _____</p> <p>VBECS retrieves the most recent crossmatch result (original or repeat) to determine unit availability and compatibility status to allow printing the Caution Tag and/or BTRF and to allow the unit to be issued.</p> <p>When the "Emergency Issue" indicator is present and/or the required ABO/Rh specimen testing is incomplete or inconclusive, enforce the following:</p> <ul style="list-style-type: none"> <li>• For RBC or WB order: <ul style="list-style-type: none"> <li>○ Group O, Rh positive or O Rh negative RBC</li> <li>○ WHOLE BLOOD is not selectable</li> </ul> </li> <li>• For FFP order: <ul style="list-style-type: none"> <li>○ Group AB, Rh positive or Rh negative, or not specified.</li> </ul> </li> <li>• For PLT order: <ul style="list-style-type: none"> <li>○ All units with &lt; 2 ML RBC containment are available</li> <li>○ Units with ≥ 2 mL RBC contamination are not available</li> </ul> </li> <li>• For CRYO order: <ul style="list-style-type: none"> <li>○ All available units are selectable</li> </ul> </li> </ul> <p>For OTHER order:</p> <ul style="list-style-type: none"> <li>○ All units with &lt; 2 mL contamination are available</li> <li>○ Units with ≥ 2 mL RBC contamination are not available</li> </ul> <p>When a user selects a Caution Tag and/or BTRF from the Blood Unit Menu and the units are assigned or have been assigned and/or crossmatched to multiple patients, the selected form may be printed for only one patient during that session. The message will display that the form cannot be printed again for another assigned patient. VBECS must be closed and re-opened to print a tag for one of the other patients. If you need to print tags on the same unit for multiple assigned patients, access the option from the Patient Menu.</p> <p>When VBECS searches for units available for a patient name, it makes available for printing only units considered available for issue (standard processing or emergency issue).</p>


User Action	VBECS																
	<p>See Appendix B: Table 23: Rules for Electronic and Serologic Crossmatch . The user may select only active units and those not in an inactive or final status. See Appendix B: Figure 148: Unit Status Flowchart.</p>																
<p>4. Acknowledge the selected patient and blood units for printing.</p> <p>Click <b>Print Unit Caution Tag, Print Transfusion Record Form, or Print Both.</b></p> <p>Click <b>OK</b> to confirm and exit.</p>	<ul style="list-style-type: none"> <li>Displays an option to print the Caution Tag and BTRF for units with a patient assignment.</li> <li>Generates a separate Caution Tag and BTRF for each blood unit.</li> </ul> <p><b>NOTES</b></p> <ul style="list-style-type: none"> <li>If a VBECS system error occurs when a Caution Tag is requested, the printer may not be correctly configured. Configure the printer as detailed in the <i>Vista Blood Establishment Computer Software (VBECS) 2.3.1 Technical Manual-Security Guide</i>, with subsequent testing.</li> </ul> <p>When a Caution Tag or BTRF is printed for an emergency-issued unit, "Emergency Issue" is included on the form.</p> <p>The recipient ABO/Rh prints as "Unk" or "Unknown" during emergency issue when no current specimen results are on file; VBECS may not use a historic record blood type (from any source) for the typing printed on the tag/form.</p> <table border="1" data-bbox="730 932 1401 1045"> <thead> <tr> <th>Compatibility Interpretation</th> <th>Text on BTRF</th> </tr> </thead> <tbody> <tr> <td>Crossmatch</td> <td>TESTING INCOMPLETE</td> </tr> <tr> <td>Crossmatch Not Required</td> <td>Not Required</td> </tr> </tbody> </table> <p>VBECS prints "Emergency Issue" in the TR section of the form.</p> <p>The text that appears in the compatibility section is based on the crossmatch interpretation applied.</p> <table border="1" data-bbox="730 1184 1401 1438"> <thead> <tr> <th>Compatibility Interpretation</th> <th>Text on BTRF</th> </tr> </thead> <tbody> <tr> <td>Crossmatch Compatible</td> <td>Compatible</td> </tr> <tr> <td>Compatible: Electronically Crossmatched</td> <td>Compatible</td> </tr> <tr> <td>Crossmatch Not Required</td> <td>Not Applicable</td> </tr> <tr> <td>Crossmatch Incompatible: Give only with Medical Director Approval</td> <td>Incompatible: Given with Medical Director Approval</td> </tr> </tbody> </table> <p>The printed form has two general areas, one for patient and unit identification, and another to document the transfusion.</p> <p>Information that appears on the form follows. All information is provided by VBECS unless stated. Information matching the caution tag will be printed to look alike for the users on both forms.</p> <ul style="list-style-type: none"> <li>Transfusion Division Name, City, State</li> <li>Patient ID</li> <li>Patient Name (Last, First MI)</li> <li>Patient DOB</li> <li>Unit/Pool ID</li> </ul>	Compatibility Interpretation	Text on BTRF	Crossmatch	TESTING INCOMPLETE	Crossmatch Not Required	Not Required	Compatibility Interpretation	Text on BTRF	Crossmatch Compatible	Compatible	Compatible: Electronically Crossmatched	Compatible	Crossmatch Not Required	Not Applicable	Crossmatch Incompatible: Give only with Medical Director Approval	Incompatible: Given with Medical Director Approval
Compatibility Interpretation	Text on BTRF																
Crossmatch	TESTING INCOMPLETE																
Crossmatch Not Required	Not Required																
Compatibility Interpretation	Text on BTRF																
Crossmatch Compatible	Compatible																
Compatible: Electronically Crossmatched	Compatible																
Crossmatch Not Required	Not Applicable																
Crossmatch Incompatible: Give only with Medical Director Approval	Incompatible: Given with Medical Director Approval																



User Action	VBECS
	<ul style="list-style-type: none"> <li>• Recipient (Patient) Blood Type</li> <li>• Unit Blood Type</li> <li>• Compatibility Interpretation, Tech name, Date/time</li> <li>• Assigned Date/Time</li> <li>• Technologist Initials (This is the Assigning/Crossmatching Blood Bank Technologist Name.)</li> <li>• Blood Component selected (the unit's blood product long name)</li> <li>• Product Code (Full, Eye-readable)</li> <li>• Donation type</li> <li>• Unit Expiration Date/Time</li> <li>• Number of Units in Pool</li> <li>• Specimen UID Number, expiration date and time</li> <li>• Location (order location)</li> </ul> <p>Transfusion documentation section: This section of the form provides prompts/placeholders for information to be entered at the bedside during the transfusion. Each listed item must provide adequate room for a transfusionist to manually record the appropriate, required information:</p> <ul style="list-style-type: none"> <li>• First identifier (signature)</li> <li>• Second identifier (signature)</li> <li>• Date/time transfusion was started</li> <li>• Respiration</li> <li>• Pretransfusion temperature</li> <li>• Pretransfusion pulse</li> <li>• Pretransfusion blood pressure</li> <li>• 15-minute temperature</li> <li>• 15-minute pulse</li> <li>• 15-minute blood pressure</li> <li>• Mid-transfusion temperature</li> <li>• Mid-transfusion pulse</li> <li>• Mid-transfusion blood pressure</li> <li>• Post-transfusion temp</li> <li>• Post-transfusion pulse</li> <li>• Post-transfusion blood pressure</li> <li>• Amount given (mL)</li> <li>• Date/time transfusion was completed or interrupted</li> <li>• Completing transfusionist (place for wet signature)</li> <li>• Reverification of patient area</li> <li>• Transfusion reaction indicator (No reaction/Reaction suspected)</li> <li>• Description of Reaction (includes Urticaria, Chill, Pain, Fever, Dyspnea, Other)</li> <li>• Other difficulties</li> <li>• Signature of person noting the transfusion reaction</li> <li>• Instructions for suspected transfusion reaction immediate response (see sample form for text)</li> <li>• Bedside verification statement (see sample form for text)</li> </ul> <p>Facility Name is the name and address of the compatibility testing facility. If a full-service facility, "Facility Name" is the</p>


User Action	VBECS
	<p>user's current division. If a transfusion-only facility, "Facility Name" is the testing facility entered. For a transfusion-only facility, both the testing facility and the current division must appear on the form.</p> <p>Patient Special Instructions and patient Transfusion Requirements do not print on the Blood Transfusion Record Form when the requirement isn't appropriate (listed as "NA") for the blood product.</p> <p>When blood products are emergency issued (indicator or the testing is incomplete or inconclusive on the current specimen), print in the remarks section of the Blood Transfusion Record Form: "This unit is issued for transfusion BEFORE the completion of required compatibility testing. The blood bank staff will continue to test the compatibility of the unit using the current patient specimen. If you are alerted to a unit's incompatibility by the blood bank, stop the transfusion immediately."</p> <p>When "****" prints for the patient ABO/Rh and crossmatch, include "Specimen testing incomplete at time of issue" in the printout.</p>
5. Print and exit.	


**Figure 110: Print Unit Caution Tag and Transfusion Record Form**

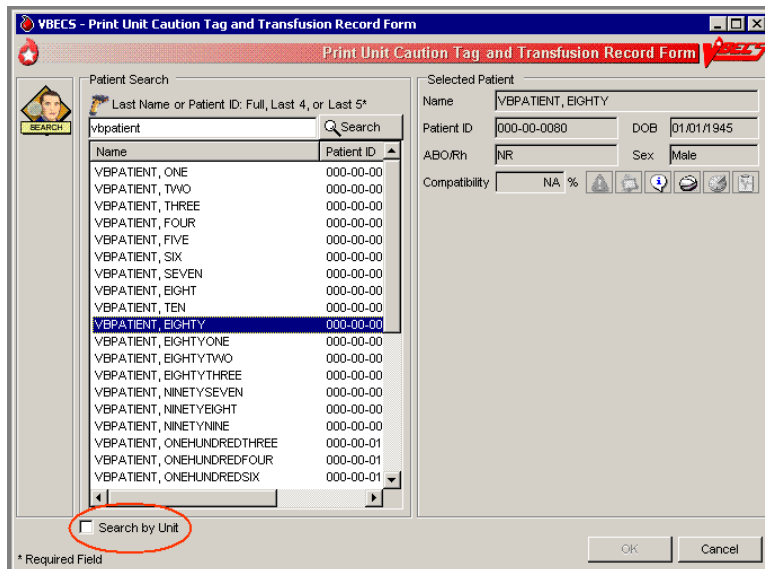

## Alerts

**Table 9: Alerts That May Occur in Print Unit Caution Tag & Transfusion Record Form (Step 3)**


User Action Step 3: When This Condition Exists ...	...  VBECS Emits an Audible Alert and Warns the User:	User Role
<p>VBECS determines whether the patient has a calculated or persistent antigen negative requirement (except when selecting autologous units). The selected unit was antigen typed and is positive for the corresponding antigen or is untested for the antigen.</p>	<p>The patient has a history of a clinically significant antibody, the unit is positive or untested for the corresponding antigen, the user does not have the proper security level. There is no override.</p>	<p>Enhanced Technologist or below</p>
	<p>The patient has a history of an antibody(s) and the unit ID is positive or untested for the corresponding antigen(s). Continue to print Caution Tag or BTRF? <b>No</b> clears the screen. <b>Yes</b> requires a comment and captures details for inclusion in an Exception Report (exception type: Caution Tag or BTRF printed for antigen positive or untested unit).</p>	<p>Lead Technologist</p>
<p>When the patient has TRs, VBECS evaluates the selected unit to see whether a TR was satisfied based on system rules, including emergency issue.</p>	<p>The patient has at least one TR that is not satisfied, the user does not have the proper security level to print the Caution Tag or BTRF for this unit. Further processing is needed and must be documented in VBECS. There is no override.</p>	<p>Enhanced Technologist or below</p>
	<p>The patient has at least one TR that is not satisfied. Continue to print Caution Tag or BTRF? <b>No</b> clears the screen. <b>Yes</b> requires a comment and captures details for inclusion in an Exception Report (exception type: Caution Tag printed with unsatisfied TR).</p>	<p>Lead Technologist</p>
<p>The patient specimen is expired.</p>	<p>The patient specimen is expired. There is no override.</p>	<p>Technologist</p>
	<p>The patient specimen is expired. Continue to print the Caution Tag and/or the BTRF? <b>No</b> clears the screen. <b>Yes</b> requires a comment and captures details for inclusion in an Exception Report (exception type: Caution Tag printed on expired specimen).</p>	<p>Enhanced Technologist</p>
<p>The user attempts to print Caution Tags for emergency issue and:</p> <ul style="list-style-type: none"> <li>• The current specimen has an “Emergency Issue” order.</li> <li>• The patient has no record of ABO/Rh typing.</li> <li>• The required ABO/Rh specimen testing is incomplete.</li> </ul> <p>The user may select:</p> <ul style="list-style-type: none"> <li>• For WB or RBC order only: group O, Rh positive or O Rh negative (WHOLE BLOOD is not available).</li> <li>• For FFP order only: group AB, Rh positive or Rh negative.</li> </ul>	<p>The unit does not meet these requirements. Testing is incomplete and this unit is not eligible for emergency issue. The Caution Tag cannot be printed. There is no override.</p>	<p>All users</p>

User Action Step 3: When This Condition Exists ...	...  VBECS Emits an Audible Alert and Warns the User:	User Role
<ul style="list-style-type: none"> <li>For PLT order: all available units are issuable.</li> </ul>		
<p>VBECS determines whether a blood unit that does require crossmatch (electronic or serologic) is ready for issue based on the following rules:</p> <ul style="list-style-type: none"> <li>Unit is not in a frozen state.</li> <li>The unit is crossmatched to the patient (eXM, or serologic XM).</li> <li>TAS Specimen testing is complete and valid (ABO/Rh and ABS results are entered).</li> </ul> <p>One of these crossmatch interpretations was entered:</p> <ul style="list-style-type: none"> <li>Crossmatch Compatible</li> <li>Compatible: Electronically crossmatched</li> <li>Crossmatch Incompatible: Give only with Medical Director approval</li> </ul>	<p>All of the conditions for the unit are met but an emergency issue order indicator is in place or the testing is incomplete or inconclusive on the current specimen and units were selected. Do you want to print emergency issue tags/forms for the unit/patient? <b>Yes</b> prints the emergency issue tags and forms. <b>No</b> does not allow the user to continue.</p>	All users
<p>For blood units that do not require crossmatch when the unit is assigned to the patient and is not in a frozen state, VBECS determines when a blood unit is ready for issue.</p>	<p>All of the conditions for the unit are not met. The Caution Tag and/or the BTRF cannot be printed at this time and the reason they cannot be printed. Additional patient testing or unit processing must be performed.</p>	All users

**Figure 111: Caution Tag**

Product Code		2D Barcode	
Division Name City, State			
Patient Name:			
Patient ID:		Recipient Blood Type:	
Unit/Pool ID:		Donor Blood Type:	
Compatibility Interpretation:			
<p><b>CAUTION: Before transfusing this blood component, perform bedside verification of the patient's identity</b> by matching the patient identification on this tag with the patient wristband and the Blood Transfusion Record Form.</p>			
Assigned Date/Time:		Assigning Tech/ Crossmatch Tech:	

**Figure 112: Blood Transfusion Record Form**

Blood Transfusion Record Form			
 VA HEARTLAND - WEST, VISN 15, Kansas City, MO		Transfusion Requirements	
Patient Name: VBECSPATIENTC, TWOHUNEIGHT			
Patient ID: 666-00-0208		Patient Blood Type: A Pos	
Unit/ Pool ID: W0930 00 000006		Unit Blood Type: A Pos	
Donation: Volunteer Allogeneic Donor		Component: RED BLOOD CELLS CPD>AS1/450mL/refg	
Compatibility Interpretation: Compatible, 09/30/2017 15:57, Test User		Product code: E0291V00	
Assigned Date/Time: 09/30/2017 15:57		Technologist Initials: KD	
		Component Expiration Date/Time: 10/28/2017 23:59	
		Number of units in pool: 0	
		Additional Patient Information	
		Specimen UID, Expiration: 2972730099, 10/03/2017 23:59	
		Location: KC-1010 NEW PT CLINIC 2	
Remarks			
*This unit is issued for transfusion BEFORE the completion of required compatibility testing. The blood bank staff will continue to test the compatibility of the unit using the current patient specimen. If you are alerted to a unit's incompatibility by the blood bank, stop the transfusion immediately.			
EMERGENCY ISSUE			
Pretransfusion Data			
Inspected and Issued by:		Date and time issued:	
Issued to:		Issued to location:	
I have verified and compared, AT THE BEDSIDE, the transfusion recipient's identity (i.e., wristband), the unit ID tag, the blood component container label, and this form. I verify that all information matches and is consistent ITEM for ITEM. I verify that the intended recipient is the same person named on this form and on the unit ID tag. Furthermore, I verify that there is a current, valid INFORMED CONSENT, and a PHYSICIAN'S ORDER for this transfusion.			
1st identifier (Sign):		2nd identifier (Sign):	

Transfusion Data											
	Date/Time	Temperature	Pulse	Blood Pressure	Respiration						
Start											
15 minutes											
Mid (optional)											
Stop											
Amount given (mL): _____ <input type="checkbox"/> Completed <input type="checkbox"/> Interrupted <input type="checkbox"/> No reaction <input type="checkbox"/> Reaction suspected (Complete Transfusion Reaction Data section)											
Other difficulties (equipment, clots, etc.): <input type="checkbox"/> No <input type="checkbox"/> Yes (specify):			Transfusion Data section completed by (Sign):								
Transfusion Reaction Data											
Patient identity match with unit compatibility tag and this form reverified by (Sign):				MD notified:							
Description of reaction: <input type="checkbox"/> Urticaria <input type="checkbox"/> Chill <input type="checkbox"/> Pain <input type="checkbox"/> Fever <input type="checkbox"/> Dyspnea <input type="checkbox"/> Other (specify):											
Transfusion Reaction Data section completed by (Sign):											
Suspected Transfusion Reaction: Immediate Response											
<p><b>If a reaction is suspected, IMMEDIATELY:</b></p> <table border="0"> <tr> <td>1. Interrupt the transfusion. Give emergency treatment. Keep the intravenous line open. Reverify patient and blood product identification.</td> <td>4. Follow transfusion reaction procedures.</td> </tr> <tr> <td>2. Notify physician and Transfusion Service.</td> <td>5. Do NOT discard the component. Return the blood component bag, filter set, and attached solutions to Transfusion Service. Collect patient specimens as required.</td> </tr> <tr> <td></td> <td>6. Order a Transfusion Reaction Workup.</td> </tr> </table>						1. Interrupt the transfusion. Give emergency treatment. Keep the intravenous line open. Reverify patient and blood product identification.	4. Follow transfusion reaction procedures.	2. Notify physician and Transfusion Service.	5. Do NOT discard the component. Return the blood component bag, filter set, and attached solutions to Transfusion Service. Collect patient specimens as required.		6. Order a Transfusion Reaction Workup.
1. Interrupt the transfusion. Give emergency treatment. Keep the intravenous line open. Reverify patient and blood product identification.	4. Follow transfusion reaction procedures.										
2. Notify physician and Transfusion Service.	5. Do NOT discard the component. Return the blood component bag, filter set, and attached solutions to Transfusion Service. Collect patient specimens as required.										
	6. Order a Transfusion Reaction Workup.										
Patient Name: <u>VBECSPATIENTC, TWOHUNEIGHT</u> Patient ID: <u>666-00-0208</u> Patient DOB: <u>2/3/1980</u> BTRF, Version. 4.0											

## Invalidate Test Results

The user invalidates incorrectly entered patient test results and reenters correct patient test results and interpretations.

### Assumptions

- A completed patient order exists in VBECS.

### Outcome

- Test results are invalidated.
- VBECS may assign “correction” status to a task to allow the user to repeat a test and add it to the Pending Task List (PTL).

### Limitations and Restrictions

- A user may not inactivate a component task in this option.
- This option does not release units from patient assignment. The user must release “assigned” units before invalidating the test results.
- To maintain a clear record and system checks, the user must invalidate both original and repeat ABO/Rh tests.
- Do not invalidate part of the configured test. Doing this will put the specimen into a state where it cannot be processed. Invalidate all tests configured as part of the TAS regardless of which sub test (ABO/Rh, ABS or DAT) is the problem.

### Additional Information

- VBECS invalidates the entire test group during this option. The user must reenter results to replace the invalidated results.
- Workload and billing are inactivated or rescinded from the inactivated testing.
- VBECS invalidates transfusion reaction workups (TRWs) in total and crossmatches (XMs) by unit.
- Invalidation of an antibody identification (ABID) task invalidates all antibody specificities entered in that task.
- VBECS displays results and interpretations of the tests but not any QC performed with the tests.
- VBECS displays “Test Result Invalidated” to indicate invalidated test results.

### User Roles with Access to This Option

- Enhanced Technologist

### Invalidate Test Results

The user may invalidate one or more previously entered test results and replace them with corrected results.

User Action	VBECS
1. Select <b>Patients</b> from the main menu.  Select <b>Invalidate Test Results</b> .	<ul style="list-style-type: none"><li>• Displays options for processing patient-related functions.</li></ul> <b>NOTES</b> _____  Click the <b>Component Orders</b> tab to view its PTL.

User Action	VBECS
<p>2. Enter the patient name and/or specimen UID.</p> <p>Edit the “expires after” date, if required.</p> <p>Click <b>Search</b>, if needed.</p>	<ul style="list-style-type: none"> <li>• Displays an option to enter a specimen UID or a patient for a specific date or for a range of dates.</li> <li>• Displays the complete diagnostic tests associated with the entry.</li> </ul>
<p>3. Select a test result from the Specimen Test Results list.</p>	<ul style="list-style-type: none"> <li>• Displays the original results for the user to review.</li> </ul> <p><b>NOTES</b> _____</p> <p>When the user chooses to invalidate an ABID, VBECS warns that the invalidation of the antibody specificity also invalidates the corresponding calculated antigen negative requirements. No user action is required.</p> <p>➤➤➤➤ When the user is a Traditional Supervisor and the selected specimen is the most recent specimen tested for the selected patient, the user may invalidate a previously justified ABO/Rh test result. When the specimen ABO/Rh test associated with a justification is invalidated, VBECS removes the justification information from the patient record.</p> <p>When the user attempts to invalidate a non-XM test for a patient specimen and blood units are currently associated with the specimen (assigned, electronically or serologically crossmatched, issued or transfused), VBECS warns that blood components are associated to this specimen and that the associated units must be removed before testing can be invalidated. The user may not continue to invalidate the test.</p> <p>When the user attempts to invalidate a XM test for a patient specimen and the associated blood units are currently issued or transfused, VBECS warns the user that the selected test cannot be invalidated because the unit was issued based on the XM result. The user may not continue to invalidate the XM test.</p>
<p>4. Verify the accuracy of the selected patient specimen and patient information (Figure 113).</p> <p>Click <b>OK</b> to accept the results.</p>	<ul style="list-style-type: none"> <li>• Marks these test results as “invalid” and asks the user whether he needs to replace the invalidated results with new test results. <ul style="list-style-type: none"> <li>○ <b>Yes</b> (test result replacement is needed) changes the task status to “correction” to return the task to the PTL for completion.</li> <li>○ <b>No</b> (test replacement is not needed) marks the erroneous test results as “Test Result Invalidated”; the task status does not change.</li> </ul> </li> <li>• When the user selects all the diagnostic tests associated with a specimen, displays an option to mark the specimen unacceptable and invalidate the tests.</li> </ul>
<p>5. Select the test to invalidate from the Patient Test Results list.</p> <p>Click <b>Invalidate Results</b> (Figure 114).</p> <p>Enter a comment.</p>	<ul style="list-style-type: none"> <li>• Asks the user whether he wishes to change the specimen status to “unacceptable.”</li> </ul> <p><b>NOTES</b> _____</p> <p>To maintain a clear record and system checks, invalidate both original and repeat ABO/Rh tests.</p>



User Action	VBECS
Click the <b>Add test to the Pending Task List to allow for replacement results</b> check box, if appropriate.	When the user invalidated all of the diagnostic tests associated with a given specimen, VBECS asks whether the user wishes to change the specimen to "unacceptable." <b>No</b> allows the user to invalidate the test results without making the specimen unacceptable. <b>Yes</b> changes the specimen to "unacceptable" and requires a comment.
Click <b>OK</b> .	
6. Click <b>OK</b> to exit.	

**Figure 113: Invalidate Patient Test Results**

**Specimen Search Criteria\***

Patient: vbpa

Specimen UID: [ ]

Expires After: 02/08/2006 [ ] Search

**Specimen Search Results**

Specimen UID	Expires	Patient Name	Patient ID
2960310037	02/10/2006 23:59	VBPATIENT, EIGHTYTWO	000-00-0082
2960310035	02/09/2006 23:59	VBPATIENT, EIGHTYONE	000-00-0081

Include Unacceptable Specimens

**Selected Patient Specimen**

Specimen UID: 2960310037

Collection Date: 02/07/2006 08:42

Received Date: 02/07/2006 10:58

Expiration Date: 02/10/2006 23:59

Phlebotomist: Kevin

Visual Inspection: Acceptable

Comment: [ ]

**Selected Patient**

Patient Name: VBPATIENT, EIGHTYTWO

Patient ID: 000-00-0082

Patient Sex: Male


Date of Birth: 01/01/1945

ABO/Rh: O Neg

\* Required Field

OK Cancel

Figure 114: Confirm Invalidation

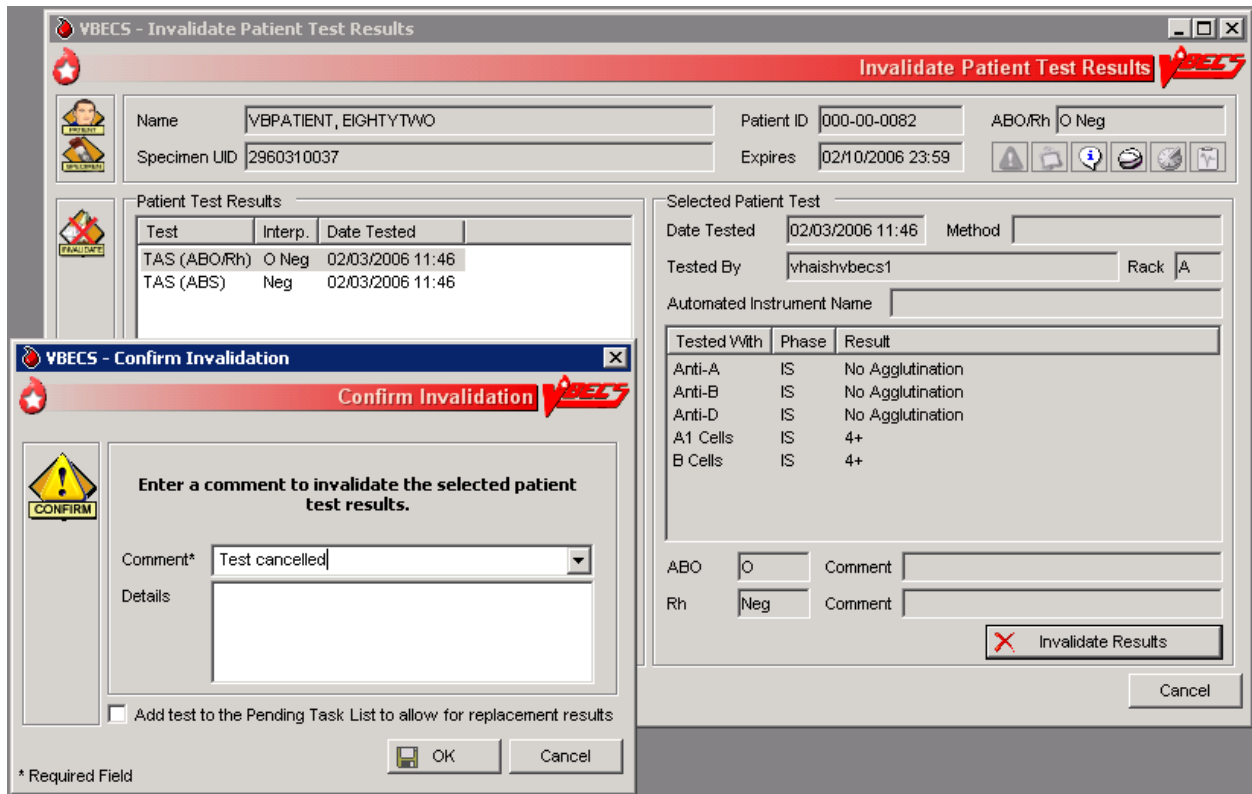

## Remove Final Status

The user removes a final status from a unit. All other statuses are manipulated through normal system rules and practices.

### Assumptions

- The user is logged into the division containing the unit record.
- The unit is in a final (transfused, transferred, modified, discarded) status.
- The user must invalidate the modification status of the split or pooled units, and then the original unit or pool.

### Outcome

- The new information was saved with the unit's historic record.
- The unit is in the status immediately prior to the final status.

### Limitations and Restrictions

- This option applies only to the release of units in final statuses.

### Additional Information


- This option allows a user to restore a unit to inventory from a final status. It may be used to begin undoing a transfusion episode (step by step in reverse order) or correct a unit modification record.
- "Issued" is not a final unit status.
- This option is not used to add units to or remove them from a pool.

### User Roles with Access to This Option

➤➤➤➤ Traditional Supervisor

### Remove Final Status

The user corrects a clerical error: a unit was processed and moved to a final status in the computer incorrectly.

 *Removing final status of a unit reported as transfused by the Transfusion Verification application will create a discrepancy between the systems. Changes or updates to a transfusion record must be made in the Transfusion Verification application first.*

User Action	VBECS
1. Select <b>Supervisor</b> from the main menu.  Select <b>Remove Final Status</b> .	<ul style="list-style-type: none"><li>• Displays options available to supervisors.</li><li>• Displays fields for selecting a unit and removing its final status.</li></ul>

User Action	VBECS
<p>2. Scan or enter a unit ID or product code, or click the <b>find</b> button to select a unit from the drop-down list.</p>	<ul style="list-style-type: none"> <li>• Displays unit data, including the previous and current unit statuses.</li> <li>• Warns the user to proceed with caution.</li> </ul> <p><b>NOTES</b> _____</p> <p>VBECS warns the user when the unit does not have a final status or was not modified (other than pool, thaw/pool, and split). The user may select another unit.</p> <p>When the final status is “modified” (pooled, thaw/pooled, or split), VBECS verifies that the target unit is invalidated. When the target unit was not invalidated, the user may not continue.</p>
<p>3. Verify that the unit selected is the desired unit.</p> <p>Click <b>OK</b>.</p> <p>Enter a comment (Figure 115).</p>	<ul style="list-style-type: none"> <li>• Asks the user if he would like to remove the final status.</li> <li>• Requires the user to enter an explanatory comment.</li> </ul> <p><b>NOTES</b> _____</p> <p>VBECS updates the unit status to the previous status and maintains restrictions on the original unit.</p>
<p>4. Review the data, click <b>OK</b> to save, and click <b>OK</b> again to confirm the save.</p>	<ul style="list-style-type: none"> <li>• Displays the unit data and asks the user to verify the return to the previous status.</li> <li>• Allows the user to review his entries and choices and save the correction to the unit record or exit without saving.</li> <li>• Updates the unit record.</li> </ul> <p><b>NOTES</b> _____</p> <p>When VBECS removes a final status of “discard” or “transfer,” it also removes return credits previously assigned to the unit.</p>

Figure 115: Remove Final Unit Status

**VBECs - Remove Final Unit Status**

**Remove Final Unit Status or Modification**

**WARNING: Removing a final status or a modification may have serious implications. Continue with caution.**

**Unit Search**

Unit ID\*

Product\*


**Current Unit Details**

Product Name

Expiration Date

Volume (mL)

Unit Status



**Previous Unit Details**

Product Name

Expiration Date

Volume (mL)

Unit Status

Comment\*

Details\*

\* Required Field

This page intentionally left blank.

# Processing Orders

## ***Accept Orders: Accept an Order***

The user accepts the receipt of an order.


### **Assumptions**

- Orders were placed in CPRS for a patient and a VBECS order number was generated.
- When a specimen is required, the user located it.
- Inappropriate Transfusion Request parameters are defined for orderable component classes.
- The connection to VistA is active.

### **Outcome**

- Ordered tests may be processed and are available to various worklists.

### **Limitations and Restrictions**

- An order for TAS must be accepted before the component order if the user wants to include the component request to the Inappropriate Transfusion Request Report. Accepting the TAS after the component order is marked inappropriate and accepted clears the user's indication that this is an inappropriate request.
- When no CPRS order exists, the user processes the specimen offline. The emergency order check box is not enabled unless there is a CPRS order and a patient specimen accessioned in VistA. If necessary, process the emergency issue of units manually, per local policy, until the specimen is received and accessioned.
- The user must accept the Type & Screen (TAS) before he accepts other diagnostic tests or component orders.
- The user cannot click  (Restricted or Assigned Units) to access information on autologous or directed units that may be restricted to a patient.
- Orders placed for patients with names that exceed the database standard for File #2 (30 characters) or do not have a last name in VistA are rejected and do not appear in VBECS. The VBECS System Administrator is notified of each rejection via an email to the address entered in VBECS Administrator "Email Alerts" field.
- A VBECS order on a Collection List has to be received with a date and time in VistA to make the specimen available in VBECS. Orders should not be assigned a date/time collected of T@U.
- Orders must include a patient location. A VBECS system error occurs when a user tries to print an order without a patient location.
- The "Display Restricted or Assigned Units" button in Accept Orders is disabled. The Blood Availability Report displays units assigned and restricted for all patients.
- On the Accept Orders screen, clicking the Received and Wanted column headers does not sort on the date and time of the Pending Order List.

### **Additional Information**

- VBECS automatically cancels previously accepted orders the day after they expire.
- To maintain a specimen with a TAS and diagnostic test orders, the TAS must be processed first, before any diagnostic test order (see Maintain Specimen).

- The user may investigate patient clinical information using VistA and CPRS.
- A facility may capture the phlebotomist's signature and the date and time of specimen collection on a printed order from CPRS. [Local IRM personnel may configure CPRS to print the blood bank order: refer to *Setup Guide* (August 2000) in VistA Document Library, Computerized Patient Record System (CPRS)].


## User Roles with Access to This Option


All users

### Accept Orders: Accept an Order



The user reviews the list of pending orders and the specimen, determines that local policy requirements are satisfied, and accepts all or part of a routine order. The user accepts the VBECS order and associates the Laboratory generated specimen UID number with the order.



The user uses the Pending Order List (POL) to view additional details, print outstanding orders, and to select, accept, and add orders to the Pending Task List (PTL). Note that the POL can take several seconds to print, depending on the number of samples processed.

 *Changes made in the VBECS Administrator option mapping orders to another VBECS division do not affect delivered orders. Orders delivered to a VBECS division must be completed, rejected, or canceled in that division. Resubmit orders after mapping is completed to send an order to another VBECS division.*

User Action	VBECS
1. Select <b>Orders</b> from the main menu.  Select <b>Accept Orders</b> .	<ul style="list-style-type: none"> <li>• Displays options for processing orders.</li> <li>• Displays the POL and POL search criteria.</li> </ul>
2. Select an order to accept from the POL (Figure 116), or  In the Pending Order List Search Criteria area, enter the known data, including the date wanted.  Click <b>Search</b> to find an order.	<ul style="list-style-type: none"> <li>• Displays options for selecting, reviewing, and processing patient data and pending orders.</li> <li>• Allows the user to associate a specimen with an order.</li> <li>• Displays the Patient Information Toolbar that allows the user to review.</li> </ul> <p><b>NOTES</b></p> <p>Select an order for TAS before selecting a corresponding order for diagnostic tests or components.</p> <p> VBECS emits an audible alert when there are patient Special Instructions or Transfusion Requirements.</p> <p>VBECS alerts the user when the component class ordered requires a specimen. The status may be REQ (specimen required) or NRQ (no specimen required). The user may not</p>



User Action	VBECS
	<p>continue until specimen information is added.  When modifiers are included in the physician's component order, VBECS provides visual and audible alerts to remind the user to evaluate the request. VBECS does not monitor subsequent action.</p> <p>VBECS alerts the user when units (autologous, directed, or assigned/crossmatched) with a future expiration date are associated with the patient in any division in the database.</p>
<p>3. Scan or enter the specimen UID in the Specimen UID field. (See Maintain Specimen.) (Figure 117)</p> <p>When the component requires a current specimen and a valid sample is not available, click the <b>Emergency Order</b> check box.</p> <p>Click <b>Accept</b> to accept the order without an associated specimen.</p>	<ul style="list-style-type: none"> <li>• Allows the user to enter new specimen information or correct problems with an existing specimen with the current VBECS order.</li> <li>• Allows the user to process an order without an associated specimen for emergency issue and retroactively associate a specimen with the order.</li> </ul> <p><b>NOTES</b> _____</p> <p>Alphabetical characters in specimen UIDs must be capitalized: VBECS recognizes only uppercase letters.</p> <p>A specimen must be processed in Maintain Specimen prior to selecting for a component order.</p> <p>When the order status is "In BloodBank", a specimen was associated with the order; when the status is "Specimen Required", the user must identify a specimen by its UID and enter specimen details according to Maintain Specimen.</p> <p>Specimens collected for diagnostic tests only are to be used for performing the specifically ordered tests and are not available for processing blood component orders.</p> <p>Within an order group containing orders for blood components and the Type &amp; Screen (TAS) test, the specimen associated with the TAS test applies to the TAS test and to all blood component orders contained in the order group.</p> <p>For all orders requiring a specimen, VBECS queries VistALink to verify that the lab order numbers are consistent with the specimen. If they are not consistent, VBECS notifies the user that the specimen's lab number does not match the number associated with the order. VBECS does not accept the order until the user identifies a specimen with a matching lab order number.</p> <p>When an order for a component class requires a specimen (WHOLE BLOOD and RED BLOOD CELLS component classes, and, optionally, based on Setting Component Classes) the user must select this function and replace the existing specimen with a new one. However, when VBECS displays the "emergency issue" order indicator, the user may continue without identifying an associated specimen. VBECS captures details for inclusion in an Exception Report [exception type: Emergency order accepted (no specimen)].</p>
<p>4. Click  to review order details.</p>	<ul style="list-style-type: none"> <li>• Displays order details that may not be edited.</li> </ul>

User Action	VBECS
<p>5. When the urgency is "Pre-Op," click  in the Patient Information Toolbar to review the MSBOS information for the division.</p> <p>View and/or search entries in the MSBOS for possible matches with the patient surgery.</p>	<ul style="list-style-type: none"> <li>Allows the user to review current MSBOS settings.</li> <li>Lists the current MSBOS site parameters, based on generic surgery types.</li> </ul> <p><b>NOTES</b> _____</p> <p>When the pre-op indicator is "Yes," the user may access the MSBOS to determine whether to include details of the order in the <b>Inappropriate Request Report</b>.</p>
<p>6. Based on the review of the MSBOS parameter associated with the surgery type and the details of the current order, click the <b>Include In Inappropriate Transfusion Request Report</b> check box, according to local policy.</p>	<ul style="list-style-type: none"> <li>Adds the details of the order to the <b>Inappropriate Transfusion Request Report</b> when the user completes <b>Accept Orders: Accept an Order</b>.</li> </ul> <p><b>NOTES</b> _____</p> <p>To display a list of available autologous and/or directed units in the system for the selected patient, see <b>Blood Availability</b>.</p> <p>A user must accept the corresponding TAS order before including a component order in <b>Inappropriate Transfusion Request Report</b>.</p>
<p>7. Click  in the Patient Information Toolbar to review patient data, Transfusion Requirements and Special Instructions.</p>	<ul style="list-style-type: none"> <li>Displays read-only TR and SI patient VBECS data.</li> </ul>
<p>8. Click <b>Accept</b> or <b>Reject</b> for each order.</p>	<ul style="list-style-type: none"> <li>For each order accepted, updates the: <ul style="list-style-type: none"> <li>Status to "accepted" and adds it to the PTL for further processing.</li> <li>Existing lab accession number to collect data associated with every order.</li> </ul> </li> </ul> <p><b>NOTES</b> _____</p> <p>When a user rejects an order, VBECS acknowledges the order and cancels it through <b>Accept Orders: Cancel a Pending Order</b>.</p> <p>When no specimen is required, VBECS assigns an order expiration date to three days in the future at 23:59 (the date of acceptance of the component order acceptance is day "0").</p> <p>When a specimen is required, the order expiration date is the same as the associated specimen. VBECS automatically expires the associated component order when the specimen is expired. (See Appendix B: Figure 147: Order Status Flowchart.)</p> <p>The user evaluates each diagnostic test and component order separately.</p> <p>When the pre-op indicator is "No," VBECS compares the Laboratory test values included in the order message from CPRS to the threshold values defined for the ordered component class requirements in <b>Configure ICCBBA Component Classes Inappropriate Request Parameters</b> setting (see <b>Component Classes</b>). When the threshold values were exceeded for one or more ordered component classes, VBECS notifies the user that the Laboratory test value exceeds site parameters for the</p>

User Action	VBECS
	<p>component class orders and asks whether to add the order to the Inappropriate Transfusion Request report. <b>Yes</b> adds the details of the order to the Inappropriate Request Report. <b>No</b> requires a comment to be associated with the order.</p> <p>The user may cancel one or more components of an order through Accept Orders: Cancel a Pending Order to cancel while working in Accept Orders: Accept an Order. When this occurs, VBECS sends a message to CPRS and notifies the VistA Laboratory application that the order was canceled.</p>
<p>9. Repeat Steps 2 through 8 to accept another order, or click <b>Close</b> to exit.</p>	

**Figure 116: Accept Orders**

**VBECS - Accept Orders**

Accept Orders

Pending Order List Search Criteria

Specimen UID  Patient

Date Wanted 02/03/2006 To 02/03/2006 Lab Order No.

Search Clear

Patient Name	Patient ID	Lab Order No.	Urgency	Specimen	Request	Ordered By	Patient Location
VBPATIENT, EIGHTYTWO	000-00-0082	65028	Routine	Required	TAS	USER, ONE	VBEC LEAVENM
VBPATIENT, EIGHTYTHREE	000-00-0083	65060	Routine	Required	TAS	USER, ONE	VBEC LEAVENM

Print

**Order Group**

- VBPATIENT, EIGHTYTWO
  - TAS
  - RED BLOOD CELLS
  - FRESH FROZEN PL.

**Selected Order**

Patient Name  ABO/Rh

Patient ID  Patient Location

Ordered By  Lab Order No.

Request  Order Status

Comments  Requirements

Specimen UID  Quantity

Emergency Order

Accept Reject

Close

\* Required Field

**Figure 117: Maintain Specimen**

**VBECS - Maintain Specimen**

**Maintain Specimen**

Name: VBECSPATIENTC, THIRTYSE Patient ID: 666-00-0037 ABO/Rh: A Pos

**Patient Specimen**

Specimen UID\*: 2912220073

Collection Date\*: 08/10/2011 13:03

Received Date\*: 08/10/2011 13:06

Expiration Date\*: 08/12/2011 23:59

Phlebotomist ID\*:

Processing Tech\*: Test User1

Acceptable Specimen

Unacceptable Specimen

Comment\*:

Details:

**Associated Orders**

Order	Status	Date Wanted

\* Required Field

OK Cancel

## Accept Orders: Pending Order List

The Pending Order List (POL) lists pending blood bank orders.

### Assumptions

- A CPRS order is acknowledged by VBECS.
- An order is printed or an alert is issued per site parameters.
- The connection to VistA is active.

### Outcome

- A list of pending orders displays and may be sorted and searched.
- A list of pending orders may be printed in the default sort presentation.

### Limitations and Restrictions

- Orders placed for patients with names that exceed the database standard for File #2 (30 characters) or do not have a first or last name in VistA are rejected and do not appear in VBECS. The VBECS System Administrator is notified of each rejection via an email to the address entered in VBECS Administrator “Email Alerts” field.
- TAS must be accepted prior to any diagnostic test associated with the same specimen UID to allow the specimen to be used for the TAS.

### Additional Information

- This option is used for diagnostic and blood component test orders.
- Blood derivatives, for example: Octoplas, are processed using CPRS FFP order.

### User Roles with Access to This Option

All users

## Accept Orders: Pending Order List

VBECS displays the POL with search criteria, which allows the user to limit the number of viewable items. The user may review details of an order without taking action on it.

User Action	VBECS
<p>1. Select <b>Orders</b> from the main menu.</p> <p>Select <b>Accept Orders</b>, or</p> <p>Click the <b>Pending Orders alert icon</b>, if available, in the status bar at the bottom of the screen (Figure 116).</p>	<ul style="list-style-type: none"><li>• Displays options for processing orders.</li><li>• Displays the Pending Order List and Pending Order List search criteria.</li><li>• Allows the user to search for specific orders.</li><li>• Provides the criteria to limit the search and subsequent presentation and processing.</li><li>• Allows acceptance of an order.</li></ul> <p><b>NOTES</b></p> <p>VBECS displays orders directed to the division by a user during system configuration.</p> <p>See Appendix B: Figure 147: Order Status Flowchart.</p> <p>When searching for pending orders based on patient name, the system displays all pending and current accepted orders for the</p>

User Action	VBECS
<p>2. Change the selection criteria, as needed, to prepare the list or to accept the default presentation.</p>	<p>patient. A current order is a previously accepted order with a future order/specimen expiration date.</p> <ul style="list-style-type: none"> <li>• Displays the POL requested by user's selection(s).</li> </ul> <p><b>NOTES</b> _____</p> <p>The POL can be sorted by Patient, Procedure, Urgency, Date/time received or date/time wanted.</p> <p>The user may select an entry on the POL to view additional details for the order, including:</p> <ul style="list-style-type: none"> <li>• Patient Name</li> <li>• Patient ID</li> <li>• Additional component class or diagnostic tests included in the same order group as the selected order. (These are displayed whether they are "in progress" or still on the POL. There should be some indication to distinguish those that are already in progress.)</li> <li>• Order comments (additional requirements from CPRS)</li> <li>• Quantity (component class orders only)</li> <li>• Patient hospital location</li> <li>• Lab order #</li> <li>• Patient ABO/Rh (if known)</li> <li>• The CPRS Order ID, Group Order ID for all diagnostic tests and component orders placed in one CPRS order is the CPRS Order ID used by VBECS on the Order History Report and the Pending Task Lists.</li> </ul>
<p>3. To continue processing the POL, review the displayed list, print the list, or go to Accept Orders: Accept an Order.</p>	<ul style="list-style-type: none"> <li>• Allows the user to return to Step 1 to change the presentation of the list, review details of pending orders, print the list, or select an order or proceed to Accept Orders: Accept an Order.</li> </ul> <p><b>NOTES</b> _____</p> <p>When a user requests a POL report and VBECS does not find any information, VBECS notifies the user and asks whether he wishes to continue to print. The user may cancel or print the report, which will include "No pending orders found."</p>
<p>4. Exit.</p>	

## Display Order Alerts

VBECS notifies the user of new and updated patient orders.

### Assumptions

- VBECS receives a new order or an order update message from VistA for the user's division.

### Outcome

- VBECS displays notification icons only to users logged into VBECS.

### Limitations and Restrictions

- VBECS issues no notifications for order cancellations.

### Additional Information



- The user may set VBECS to automatically print an order alert with limited information on the designated printer when VBECS acknowledges the order.
- The alert displays the individual test's (child) CPRS ID. The individual test's (child) CPRS ID is not displayed on the Pending Order List, elsewhere in VBECS or on reports and is not usable in search functions.

### User Roles with Access to This Option

All users

### Display Order Alerts

VBECS notifies the user that order updates occurred and allows him to accept the new orders.

User Action	VBECS
<p>1. Hold the mouse over the icon to view a tool tip, or</p> <p>Click  to view a panel showing new or updated orders and hide the icon.</p> <p>Click  again to hide the panel.</p>	<ul style="list-style-type: none"> <li>• Acknowledges a new or updated patient order.</li> <li>• Displays an order notification icon in the status bar when new orders are received, according to local settings.</li> <li>• Displays information regarding new orders and order updates and allows the user to accept them.</li> </ul> <p><b>NOTES</b> _____</p> <p>When the division is configured to alert the user of orders received by VBECS, VBECS notifies the user of ordered components or ordered tests that were:</p> <ul style="list-style-type: none"> <li>• Directed to the user's division.</li> <li>• Added or updated in VistA since the system last checked.</li> </ul> <p>When the division is configured to alert the user of orders needed in the next specified number of hours, VBECS also notifies the user of ordered components or ordered tests with a required time less than a specified number of hours in the future.</p> <p>The system displays the following information for each entry in the summary:</p> <ul style="list-style-type: none"> <li>• Urgency</li> <li>• Patient Name</li> </ul>

User Action	VBECS
	<ul style="list-style-type: none"> <li>• Patient ID</li> <li>• Ordered Item</li> <li>• Wanted Date</li> <li>• Lab Order ID</li> <li>• CPRS Order ID</li> </ul> <p>If the notification icon was already visible, the system appends new notification information to the summary. If it was not visible, the system displays only the new notification information in the summary.</p>



## **Display Patient Update Alerts**

VBECs notifies the user of patient update, death, and merge events.

### **Assumptions**

- A patient demographic update is received from VistA for the user's division for a VBECs patient with an active order.

### **Outcome**

- The patient demographics are updated in VBECs.

### **Limitations and Restrictions**

- VBECs does not notify users of additional VistA patient updates or merges while the icon is displayed. When a user views an update and there is an unprocessed VistA patient merge, the Patient Alert icon remains on the screen until the Traditional Supervisor updates the patient records and clears the icon.
- Patient updates involving patient's with names that exceed the database standard for File #2 (30 characters) or do not have a first or last name in VistA are rejected and do not appear in VBECs. The VBECs System Administrator is notified of each rejection via an email to the address entered in VBECs Administrator "Email Alerts" field.
- Patient update alerts only display when active component or diagnostic test orders are pending or active<sup>4</sup> at the time the update occurs.

### **Additional Information**

- None

### **User Roles with Access to This Option**



All users

---

<sup>4</sup> Active: VBECs order that is not rejected, canceled or completed. When the patient update is received and the order is 'active', the update is displayed regardless of order cancellation until it is viewed by a VBECs user.

## Display Patient Update Alerts

VBECS notifies the user that patient updates occurred and allows him to view additional details.

User Action	VBECS
<p>1. Hold the mouse over the Patient Alert icon to view a tool tip, or</p> <p>Click  to view a menu and select <b>Updated Patients</b>, <b>Deceased Patients</b>, or <b>Patient Merge</b> to view the updates.</p> <p>Click  again to hide the menu.</p>	<ul style="list-style-type: none"> <li>• Displays a Patient Alert icon in the status bar when it receives a patient update or death event message and the patient has active or pending orders in the user's VBECS division.</li> <li>• Displays a Patient Alert icon in the status bar when it receives a merge event message from VistA.</li> <li>• The system displays a patient update or merge icon alert to all users until all merge events are cleared and all updates, including death notices are viewed.</li> </ul> <p><b>NOTES</b> _____</p> <p>The user may also select <b>Patients</b> from the main menu, <b>Patient Updates</b>, and <b>Updated Patients</b>, <b>Deceased Patients</b>, or <b>Patient Merge</b> to view the updates.</p>

## Display Patient Merge Alerts

VBECs notifies the user of VistA patient merges.

### Assumptions

- A patient demographic merge alert is received from VistA for the user's division for a VBECs patient.

### Outcome

- VBECs notified the user that patient records were merged in VistA and that the user must take action to match the records in VBECs.

### Limitations and Restrictions

- VBECs does not notify users of additional VistA patient updates or merges while the icon is displayed. When a user views an update and there is an unprocessed VistA patient merge, the Patient Alert icon remains on the screen until the Traditional Supervisor updates the patient records and clears the icon.

### Additional Information



- VBECs notifies users of VistA patient merges until a Traditional Supervisor clears the Patient Alert icon. See Patient Merge for instructions.
- VBECs displays an alert to users for a VistA patient merge event if either the merge-from or merge-to patient exists in the VBECs system when the merge message is received.

### User Roles with Access to This Option

All users

## Display Patient Merge Alerts

VBECs notifies the user that patient merges occurred in VistA and allows him to view additional update details.

User Action	VBECs
<p>1. Hold the mouse over the Patient Alert icon to view a tool tip, or</p> <p>Click  to view a menu and select <b>Updated Patients</b>, <b>Deceased Patients</b>, or <b>Patient Merge</b> to view the updates.</p> <p>Click  again to hide the menu.</p>	<ul style="list-style-type: none"><li>• When it receives a patient update or death event message and the patient has active or pending orders in the user's VBECs division, displays a Patient Alert icon in the status bar.</li><li>• When it receives a merge event message from VistA, displays a Patient Alert icon in the status bar.</li></ul> <p><b>NOTES</b> _____</p> <p>The user may also select <b>Patients</b> from the main menu, <b>Patient Updates</b>, and <b>Patient Merge</b> to view the updates.</p> <p>VBECs displays a patient update or merge icon alert to all users until all merge events are cleared and all updates, including death notices are viewed.</p>

## Emergency Transfusion Order Processing

**Question:** How do I Process an Emergency Transfusion Order for red blood cells when no specimen is collected?

**Important Note:** If the patient is not found in VistA, you must process the request manually. When the patient's CPRS order is placed, enter your testing results and unit processing information with correct dates and times recreating the record as retrospective data entry.

**Answer: No Specimen Available:** Below are the steps for processing a component order when no specimen is available but required. This example is for a Red Blood Cell order.

- 1) A CPRS order for a TAS and RBC must be placed to process the order in VBECS.
- 2) Accession only the RBC order in the VistA LAB package. If the TAS is accessioned but not physically available, it will not interfere with the next step (Figure 118).

Figure 118: Example of Red Blood Cells Accession


```
Telnet [redacted]
UBPATIENT, FIVE          000-00-0005   Requesting location: UBO-C0
Date/Time Ordered: 10/19/2009 10:08   By: USER, ONE
-Lab Order # 41917                 Provider: USER, ONE
UBECS - NO SPECIMEN REQUIRED OTHER
RED BLOOD CELLS - LAB
      ROUTINE Requested <SEND PATIENT> for: 10/19/2009
UBPATIENT, FIVE          000-00-0005   Requesting location: UBO-C0
Date/Time Ordered: 10/19/2009 10:08   By: USER, ONE
-Lab Order # 41917                 Provider: USER, ONE
BLOOD
TYPE & SCREEN - LAB
      ROUTINE Requested <SEND PATIENT> for: 10/19/2009
Is this the correct order? Yes// y YES
Collection Date[Time: //n <OCT 19, 2009@10:21:09>
Do you have the entire order? n NO
Do you have the OTHER UBECS - NO SPECIMEN REQUIRED ? No// y <Yes>
Print labels on: LABBARCODE-1-CO// home UCX TELNET SESSION

ACCESSION: CBB 1019 2 <C192920002>
RED BLOOD CELLS - LAB UBECS - NO SPECIMEN REQUIRED OTHER
Do you have the BLOOD 15ML REDTOP? No// <No>

Select Order number: _
```

- 3) Accept the RBC order in VBECS by checking the Emergency Order checkbox [**Orders, Accept Orders** (Figure 119)].


**Figure 119: Example of Emergency Order checkbox**

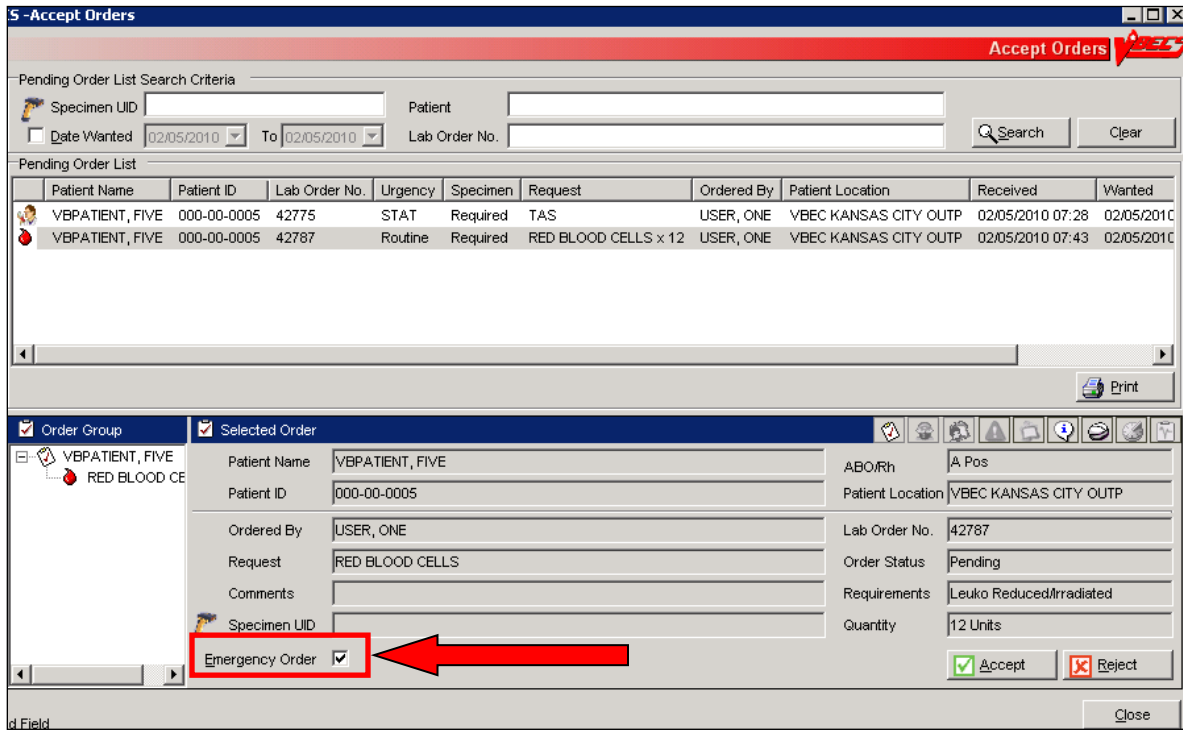

- 4) Select Units and issue from the Emergency Issue tab until a specimen is received.

- 5) Associate the specimen with the RBC order in Select Units by selecting one of the ISSUED units to perform retrospective crossmatch testing. Click the ellipsis button to search for an available specimen UID (Figure 120). See the Select Units section for additional information.

**Figure 120: Example of Specimen UID Ellipsis button**

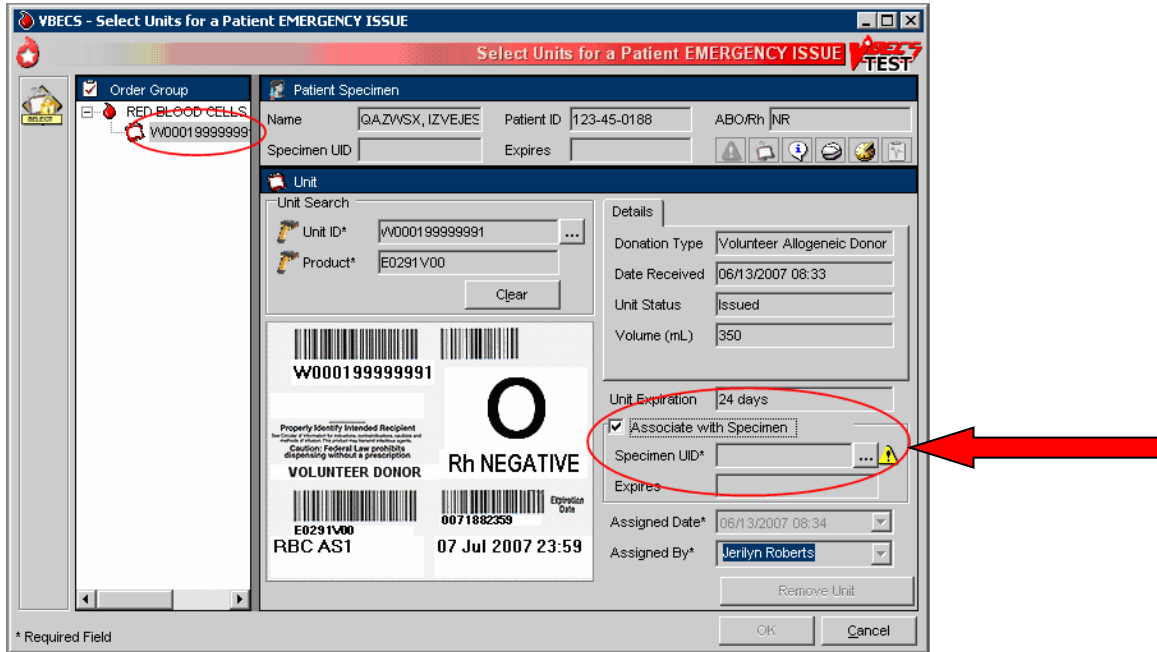

## **Maintain Specimen**

A specimen for testing is received in the blood bank and is processed as part of initial order acceptance, as described in Accept Orders: Accept an Order.

### **Assumptions**

- Pending CPRS orders and active orders require an acceptable specimen.
- A specimen was collected and previously accessioned in the VistA Laboratory system, which generated the specimen's universal ID (UID).

### **Outcome**

- The user accepted or rejected a specimen.

### **Limitations and Restrictions**

- When a search by UID causes VBECS to time out, repeat the search by UID and patient name or by patient name only.
- Specimen recollection requests must be made offline.
- Designating a specimen as "unacceptable" after accepting a CPRS order closes that order and requires a new CPRS order and sample.

### **Additional Information**

- A specimen may be marked unsuitable for use after its acceptance.
- A specimen expiration date may be edited after acceptance.
- A specimen may be recollected without canceling or reordering the VistA order. This practice is dictated by local policy and procedures.
- Specimens marked as originally associated with diagnostic tests only or unacceptable are not considered available specimens when CPRS is queried for specimen availability.

### **User Roles with Access to This Option**

All users

## **Maintain Specimen**

A specimen for testing is received in the blood bank and is processed as part of initial order acceptance, as described in Accept Orders: Accept an Order.


Information about a previously accepted specimen may be updated. The user may verify whether an active specimen is currently in the blood bank or update the information concerning a previously accepted specimen. After the user checks the physician's order against the specimen requirement, the user verifies the specimen's condition, labeling, and accompanying paperwork, according to local policy. The user verifies the specimen information and acceptability and prepares the specimen for testing.

The user may edit specimen information, such as extending the expiration time according to local policy, and marking a specimen as unsatisfactory or expired so that future orders require that a new specimen be collected.

When a specimen is required, the user must find that specimen before acknowledging that order, as described in Accept Orders: Accept an Order.

User Action	VBECS
1. Select <b>Orders</b> from the main menu.  Select <b>Maintain Specimen</b> .	<ul style="list-style-type: none"> <li>• Displays options for processing orders.</li> <li>• Displays fields for entering patient specimen information.</li> </ul>
2. Scan or enter the specimen's UID.	<ul style="list-style-type: none"> <li>• Displays a field to enter the specimen UID and allows the user to search for existing specimens.</li> <li>• Displays previously entered information for the specimen and accommodates data entry of new specimen information.</li> </ul> <p><b>NOTES</b> _____</p> <p>The user may:</p> <ul style="list-style-type: none"> <li>• Create and/or maintain a specimen associated with an order from Accept Orders: Accept an Order.</li> <li>• Update specimen information from a previously received specimen.</li> </ul> <p>Specimens associated with blood bank diagnostic tests are not available in Maintain Specimen. Specimens collected for diagnostic tests only are to be used for performing the specifically ordered tests and are not available for processing blood component orders.</p> <p>The orders and associated specimens are maintained for three days before the order or specimen expires: Type &amp; Screen (TAS), component orders (RBC, FFP, CRYO, PLT, WB, OTHER).</p> <p>The list of orderable and reflex tests below may be added to a specimen maintained for a component order, but may not have a component order added to them: they do not have maintained specimens.</p> <p>These tests have a specimen drawn and used for their ordered test. The specimens are not maintained for component preparation:</p> <ul style="list-style-type: none"> <li>• DAT</li> <li>• ABS</li> <li>• ABO/Rh</li> <li>• Transfusion Reaction Workup (TRW)</li> </ul> <p>VBECS allows the user to search all existing patient specimens associated with a specific patient. The user may enter the patient name or a specimen UID to access the list.</p>
3. Enter new specimen information or edit previously entered information (Figure 117).	<ul style="list-style-type: none"> <li>• Displays uneditable patient demographic information.</li> <li>• Displays an option to edit.</li> </ul> <p><b>NOTES</b> _____</p> <p>When the user extends the specimen expiration date and time, VBECS calculates the number of days by which the specimen was extended (based on the originally calculated expiration date) and the newly entered expiration date. VBECS compares the difference to the site parameter for the maximum number of extension days allowed for a specimen. When the entered</p>



User Action	VBECS
	<p>expiration date exceeds the parameter, VBECS warns the user. There is no override.</p> <p>When a specimen expiration date is extended on patients with transfusions greater than 3 months in the past, it will trigger the warning that patient has been transfused in the past 3 months. Users should complete the override as requested to proceed per local policy and procedure.</p> <p>The default expiration date is calculated as three days in the future with the day of the original specimen collection as "0." For example, a specimen collected on September 16, 2007 (regardless of the time), is day "0" and the calculated expiration date and time are September 19, 2007 @ 23:59 (midnight).</p> <p>When a new or existing specimen is being associated with a pending component order during Accept Orders: Accept an Order, the specimen must be acceptable.</p> <p>When there is a specimen requirement for orders, each order must be evaluated for a valid specimen before it can be accepted.</p> <p>VBECS displays specimen UIDs for selection by CPRS-associated maintainable specimen and then by acceptable and unexpired maintainable specimens, in reverse chronological order.</p> <p>VBECS allows the user to include expired and unacceptable specimens.</p> <p> When the user attempts to save an expiration date past the default of three days, VBECS checks the patient's record for evidence of transfusion within the previous three months and the history of antibody identification, regardless of the division. If either is found, VBECS emits an audible alert, warns that the patient has evidence of transfusion within previous three months and/or antibody formation, and instructs the user to enter a comment to extend the specimen expiration date beyond three days. The user must enter a canned comment (with the specimen extension) and/or free-text comment to continue. VBECS captures details for inclusion in an Exception Report (exception type: expiration date extended).</p> <p>When the specimen is deemed unacceptable, the status of the orders associated with the specimen remains "pending" or "active."</p> <p> When a user attempts to reactivate a previously expired or unacceptable specimen, VBECS emits an audible alert, warns that this is a previously expired or unacceptable specimen, and instructs the user to enter a comment to make it available. A comment is required to continue. VBECS captures details for</p>

User Action	VBECS
	<p>inclusion in an Exception Report (exception type: unacceptable/expired specimen used).</p> <p>VBECS requires the date and time a specimen was collected and the phlebotomist ID.</p> <p>When a specimen is processed, VBECS displays the details of associated orders (read only).</p> <p>The user must enter a comment when marking a specimen as unacceptable.</p>
<p>4. Enter or verify specimen-related information from Step 3.</p>	<ul style="list-style-type: none"> <li>• Allows the user to review and edit previously entered information.</li> </ul>
<p>5. Review the entered information and click <b>OK</b> to save.</p>	<ul style="list-style-type: none"> <li>• Saves the data for use in further specimen order processing and subsequent retrieval.</li> </ul> <p><b>NOTES</b> _____</p> <p>When saving or updating an expired specimen, VBECS warns the user and asks whether he wishes to continue. The user may continue without entering a comment.</p> <p>The UID initially used to identify the specimen becomes the VBECS specimen number, which VBECS retains and uses to process subsequent orders during the life of the specimen, including CPRS, VBECS, and Laboratory orders, and UIDs associated with the specimen. A historical record lists changes made to the specimen and is available for review.</p> <p>A permanent link remains between an exhausted or unacceptable specimen UID and its order, which allows the user to document the recollection of a new specimen, according to local policy.</p>
<p>6. Click <b>OK</b> to exit.</p>	

## ***Patient Testing: Cancel an Active Order***

The user cancels an accepted order on the Pending Task List (PTL) and adds a comment to justify the cancellation.

### **Assumptions**

- The order and the user are in the same division.
- A user accepted an order that originated in CPRS through Accept Orders: Accept an Order, or created a reflex test order through Order Reflex Tests.
- The connection to VistA is active.

### **Outcome**

- The order was removed from the PTL.
- An order update message was sent to CPRS.

### **Limitations and Restrictions**

- Cancellation of one order does not affect other orders placed with it.
- Before a user may cancel a partially completed task, he must invalidate the results for that task, thereby changing its status to “not started.”

### **Additional Information**

- None

### **User Roles with Access to This Option**

All users


## **Patient Testing: Cancel an Active Order**

The user selects one or more tasks to cancel from the PTL, cancels the accepted order from within VBECS, and adds a comment regarding the cancellation.

<b>User Action</b>	<b>VBECS</b>
1. Select <b>Patients</b> from the main menu.  Select <b>Patient Testing</b> .	<ul style="list-style-type: none"><li>• Displays options for processing patient-related functions.</li><li>• Displays the Pending Task List in the Diagnostic Tests tab.</li></ul> <b>NOTES</b> _____  Click the <b>Component Orders</b> tab to view its PTL.

User Action	VBECs
<p>2. Click one to four check boxes in the PTL in the Diagnostic Tests tab or the Component Orders tab to select tasks (Figure 121).</p> <p>Click <b>OK</b> to continue, or click <b>Cancel Order(s)</b> to cancel the selected tasks.</p> <p>Select or enter a comment.</p> <p>Click <b>OK</b> to confirm the cancellation.</p>	<ul style="list-style-type: none"> <li>Deletes the selected tasks from the PTL and sends a message to CPRS with an order status update.</li> </ul> <p><b>NOTES</b></p> <p>Select a canned comment and/or free-text comment</p> <p>VBECs allows a user to cancel only tasks that are not started.</p> <p>See Appendix B: Figure 147: Order Status Flowchart.</p>

**Figure 121: Pending Task List Diagnostic Tests**

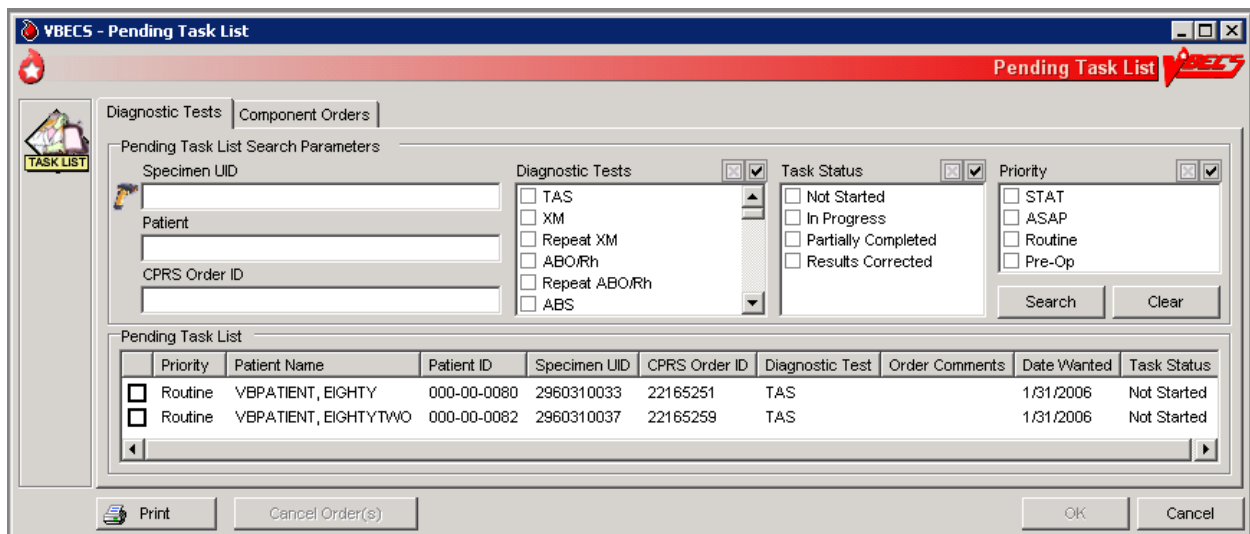

## Patient Testing: Pending Task List

The user accesses a Pending Task List (PTL) to select tasks (accepted orders), organize a batch of tests to be performed in a session, process patient component requests, and view and/or print a list of outstanding blood bank tasks.

### Assumptions

- CPRS orders were accepted or a reflex order was placed within the VBECS division.

### Outcome

- The task is available for further processing.

### Limitations and Restrictions

- This option is for diagnostic, blood component, and reflex test orders placed in or directed to the user's division.
- The PTL does not include daily quality control, equipment maintenance schedules, or pending ABO/Rh confirmation and auxiliary tests on red blood cell units.
- Users may perform testing for no more than four patients in one VBECS session.

### Additional Information

- This is the main entry point for other testing options.
- The PTL does not display completed diagnostic tests.
- When a specimen is not associated with a component order, partially completed and filled component orders remain on the Pending Task List for 10 days from date of order acceptance.
- When a specimen is associated with a diagnostic test or component order, partially completed diagnostic tests and filled component orders remain on the Pending Task List until the specimen expires.
- VBECS cancels accepted orders (tasks) the day after they expire.
- Multiple users are prohibited from performing the same task on the same unit of blood for the same patient.
- The task may be selected and assigned to a technologist, preventing duplicate selection by another technologist.

### User Roles with Access to This Option

All users

### Patient Testing: Pending Task List

The user accesses a list of pending tasks. VBECS displays the PTL with search criteria to allow the user to limit the number of items viewed.

User Action	VBECS
1. Select <b>Patients</b> from the main menu.  Select <b>Patient Testing</b> (Figure 121).	<ul style="list-style-type: none"><li>• Displays options for processing patient-related functions.</li><li>• Displays the PTL and PTL search parameters in the Diagnostic Tests tab or in the Component Orders tab.</li></ul> <p><b>NOTES</b> _____ Entering a random string consisting of letters and digits into the Patient search field returns the full pending task list. The correct</p>

User Action	VBECS
	<p>combinations (full SSN, last 4 digits of SSN, first letter of last name + last 4 digits of SSN) will return a patient list.</p>
<p>2. Click the check boxes in the PTL to select one to four tasks for processing and click <b>OK</b>, or</p> <p>Click <b>Print</b> to print the list.</p>	<ul style="list-style-type: none"> <li>• Allows the user to limit the search and display.</li> <li>• Prompts the user to select a task for continued processing, print a task list, or return to Step 1 to change the display of the list.</li> </ul> <p><b>NOTES</b></p> <p>Additional information is available in Appendix G: Frequently Asked Questions.</p> <p>The user may click <b>Cancel Order(s)</b>, select a comment, and click <b>OK</b> to exit.</p> <p>The user may select one to four tasks for testing at a time. The tasks may be component class order tasks, diagnostic test tasks, or a combination of both.</p> <p>VBECS displays “NRQ” (not required) when the task does not require a specimen for processing.</p> <p>For each order, VBECS displays the specific diagnostic test or component order. The default searches for the current date’s tasks, then yesterday’s tasks, and so on until the task status is “completed.” The display includes the task’s status and a user name, when applicable.</p> <p>VBECS alerts the user when he attempts to select multiple tasks of the same type for a patient, and does not allow the selection of two or more tasks of the same type ordered for a patient during a testing session. Tasks are considered to be of the same type when they need to accommodate the use of the same testing grid in Patient Testing: Record Patient Test Results. For example, a user may not select a TAS task and one of its subtests that is ordered in a testing session. VBECS provides for one testing grid display per patient per test, not per specimen UID. If the user selected a TAS and an ABS, VBECS displays only the first one selected for result entry.</p> <p>Task and order statuses and functions that are displayed on the PTL:</p> <ul style="list-style-type: none"> <li>• <b>Not Started:</b> the order was not selected.</li> <li>• <b>Partially Completed &lt;user ID&gt;:</b> allows selection by another user for a diagnostic test to complete, invalidate, or delete the work on this specimen. <i>Component orders remain in this status until the specimen and order expire.</i></li> <li>• <b>Filled:</b> This order status is assigned when the total number of units ordered is updated to a unit status of “issued” or “transfused.”</li> <li>• <b>Correction:</b> previous results were invalidated; a corrected entry is required.</li> <li>• <b>Instrument Results Pending Review:</b> displayed on the Pending Task List: Diagnostic tests: does not allow the selection by another user for manual testing.</li> </ul>

User Action	VBECs
	<p>Task and order statuses that are not displayed for selection on the PTL:</p> <ul style="list-style-type: none"> <li>• <b>Completed:</b> applies to diagnostic tests</li> <li>• <b>Expired:</b> applies to component orders at expiration of the associated specimen.</li> </ul> <p>See Appendix B: Figure 147: Order Status Flowchart for the transmission of orders between CPRS and VBECs.</p> <p>When a user prints a PTL and no pending tasks are found, VBECs asks whether the user wishes to continue to print. The user may cancel or print the report, which states that no pending orders were found.</p>
3. Click <b>OK</b> or <b>Cancel</b> to exit.	

## Accept Orders: Cancel a Pending Order

The user cancels an unaccepted order on the Pending Order List (POL).

### Assumptions

- An order was not accepted by a user.
- The connection to VistA is active.

### Outcome

- An order update message was sent to CPRS.
- The order was removed from the POL.

### Limitations and Restrictions

- Cancellation of one order does not affect other orders placed with it.

### Additional Information

- Each pending order is managed individually.
- VBECs does not automatically clear or cancel pending or unaccepted orders from the POL.

### User Roles with Access to This Option

All users


## Accept Orders: Cancel a Pending Order

The user selects an unaccepted order for cancellation.

User Action	VBECs
<p>1. Select <b>Orders</b> from the main menu.</p> <p>Select <b>Accept Orders</b>.</p>	<ul style="list-style-type: none"> <li>• Displays options for processing orders.</li> <li>• Displays the Pending Order List and Pending Order List search criteria.</li> </ul>

User Action	VBECs
2. Select an order to cancel from the Pending Order List (Figure 122).  Click <b>Reject</b> .	<ul style="list-style-type: none"> <li>Requires the user to enter a comment.</li> <li>Requires the user to confirm the cancellation of the pending order.</li> </ul>
3. Enter or select a comment in the Comment field.  Enter details in the Details field, if desired.  Click <b>OK</b> and <b>Close</b> to confirm the order cancellation.	<ul style="list-style-type: none"> <li>Removes the order from the Pending Order List and sends a message to CPRS with an order status update.</li> </ul> <p><b>NOTES</b> _____</p> <p>VBECs does not cancel a lab order if the lab order has already been accessioned.</p>

**Figure 122: Accept Orders: Reject an Order**

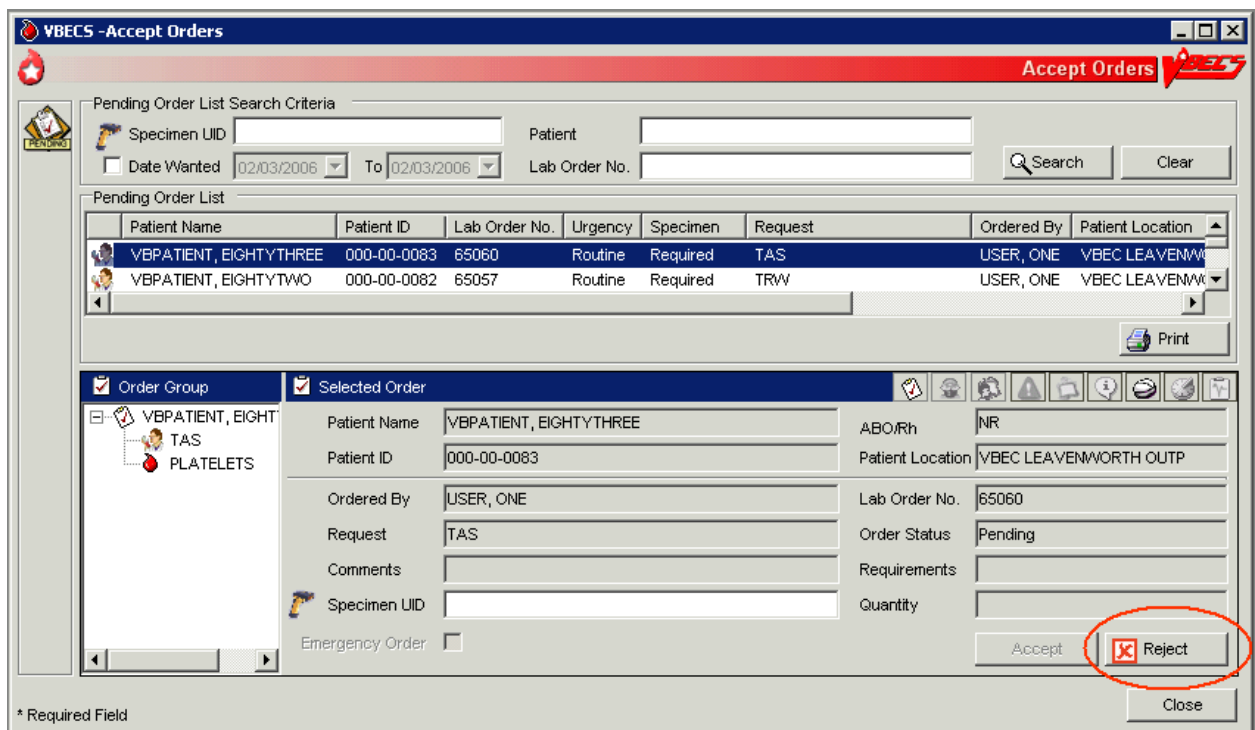



## Order Reflex Tests

The user may order additional blood bank tests for a complete or incomplete accepted patient specimen to add to the Pending Task List (PTL). The ordered test or test group limits the type of reflex and repeat test order that may be associated with a specimen.

### Assumptions

- A patient's test result requires further investigation or repetition of a diagnostic test.

### Outcome

- One or more reflex orders are in the PTL with a "routine" urgency.

### Limitations and Restrictions

- A crossmatch (XM) and a repeat XM cannot be ordered unless they are associated with a specimen with a future expiration date and a phlebotomist is associated with that specimen.
- When a user selects weak D for the antigen typing test and the selected patient is Rh positive, VBECS warns that weak D testing cannot be performed. There is no override and VBECS does not add the test.

### Additional Information

- Unlimited repeat tests may be ordered and completed, but are not available to CPRS or to billing.

### User Roles with Access to This Option

All users

### Order Reflex Tests

The user orders additional blood bank tests for a patient specimen that was already accepted and that may or may not be completed. The ordered reflex test(s) are added to the PTL.

User Action	VBECS
<p>1. Select <b>Orders</b> from the main menu.</p> <p>Select <b>Order Reflex Tests</b>.</p>	<ul style="list-style-type: none"><li>• Displays options for processing orders.</li><li>• Displays specimen search criteria and search results.</li></ul> <p><b>NOTES</b> _____</p> <p>The user may also access Order Reflex Tests by clicking <b>Order Reflex Tests</b> when executing Patient Testing or by right-clicking the testing grid that generates the reflexive order.</p>
<p>2. Enter the patient name and/or the specimen UID.</p> <p>Click <b>Search</b>, if necessary.</p>	<p><b>NOTES</b> _____</p> <p>The user may identify specific specimen for reflex test ordering by scanning or by manual data entry of the UID on the specimen.</p>


User Action	VBECS
<p>3. Select an item from Specimen Search Results.</p> <p>Review the information in the Selected Patient Specimen area and in the Patient Details and Order Details tabs (Figure 123).</p> <p>Click <b>OK</b> to accept the order.</p> <p>Respond to messages, as required.</p>	<ul style="list-style-type: none"> <li>• Displays an option to search for specimens currently available (in-date) for a patient by standard patient search conventions.</li> <li>• Accommodates direct selection of a specimen by scanning or by entering a specimen UID.</li> <li>• Displays available (in-date) patient specimens or orders in reverse chronological order within a date range defined by the site parameter. Only one patient specimen may have reflex tests added in one session. The expired and unacceptable specimens will follow.</li> <li>• Displays a message directing the user to complete Patient Testing when the selected test is not completed. User returns to Patient Testing.</li> <li>• Displays a message when a specimen is expired or unacceptable. User may click <b>Yes</b> to continue with the order or <b>No</b> to select or request another specimen, as appropriate.</li> <li>• Displays the date/time/division of the specimen collection when selecting a specimen for ordering reflex tests.</li> </ul> <p><b>NOTES</b> _____</p> <p> VBECS emits an audible alert, warns that the specimen is expired, and instructs the user to enter a comment to continue. When a user enters or scans an expired specimen. VBECS requires a comment and captures details for inclusion in an Exception Report (exception type: reflex test ordered on an expired specimen).</p>
<p>4. When a specimen is already selected, go to Step 5.</p> <p>Select or enter a patient specimen that requires the additional test.</p>	<ul style="list-style-type: none"> <li>• Displays all orders associated with a specimen that can be used to request reflex tests for an existing diagnostic test order within an order group.</li> <li>• Automatically associates the reflex test with the order when there is only one order associated with the specimen.</li> </ul>
<p>5. Click one or more check boxes to select reflex tests from the Order Reflex Test(s) list.</p>	<ul style="list-style-type: none"> <li>• Displays the selected tests for review.</li> <li>• Allows the user to select or to deselect reflex tests.</li> </ul>
<p>6. Click <b>OK</b> to save and exit.</p> <p>Enter a comment, if required. cancellation.</p>	<ul style="list-style-type: none"> <li>• Adds the ordered patient reflex tests to the PTL.</li> </ul>

Figure 123: Order Reflex Test(s)

**VBECS - Order Reflex Test(s)**

**Order Reflex Test(s)**

Name: VBPATIENT, EIGHTYONE    Patient ID: 000-00-0081    ABO/Rh: O Neg

Specimen UID: 2960310035    Expires: 02/03/2006 23:59

Order	Test Code	Reflex Test Name
<input type="checkbox"/>	Repeat ABO/Rh	Repeat ABO/Rh
<input type="checkbox"/>	Repeat ABS	Repeat ABS
<input checked="" type="checkbox"/>	DAT	DAT
<input type="checkbox"/>	ABID	ABID
<input type="checkbox"/>	AGwD	Weak D Typing
<input type="checkbox"/>	AGC	C Typing
<input type="checkbox"/>	AGE	E Typing
<input type="checkbox"/>	AGc	c Typing
<input type="checkbox"/>	AGe	e Typing
<input type="checkbox"/>	AGK	K Typing
<input type="checkbox"/>	AGk	k Typing
<input type="checkbox"/>	AGKp(a)	Kp(a) Typing
<input type="checkbox"/>	AGKp(b)	Kp(b) Typing
<input type="checkbox"/>	AGJk(a)	Jk(a) Typing
<input type="checkbox"/>	AGJk(b)	Jk(b) Typing

\* Required Field

OK    Cancel

This page intentionally left blank.

# Patient Services

## Patient Search Screen

VBECs displays the Patient Search screen to search for a specific patient in VBECs.

### Assumptions

- The patient record exists in VBECs with the exception of specific searches that return patient information from VistA. For the Edit Unit Information, Incoming Shipment, and Document ABO Incompatible Transfusion options, VBECs searches VistA.

### Outcome

- VBECs presents the user with a list of patients for selection based on their search input.

### Limitations and Restrictions

- VBECs updates information from VistA File #2, not VistA File #20. Name changes made in VistA File #20 will not appear in VBECs. Align File #2 with those changes to update VBECs.

### Additional Information

- The Patient Search Screen is used by options throughout VBECs to select patients.

### User Roles with Access to This Option


All users

### Patient Search Screen

User Action	VBECs
<p>1. To access the Patient Search screen, open one of the options listed in Notes. To search for a patient, enter the patient's full name (last name first) or ID (Figure 124), or</p> <p>Enter the last four digits of the ID, or the initial of the last name and the last four digits of the ID, and click <b>Search</b>.</p>	<ul style="list-style-type: none"> <li>• Displays the Patient Search screen.</li> <li>• Lists patients that meet the search criteria.</li> </ul> <p><b>NOTES</b> _____</p> <p>Options that allow the user to search for a patient:</p> <ul style="list-style-type: none"> <li>• <b>Blood Units</b> from the main menu. Select <b>Edit Unit Information</b> (for restricted units only).</li> <li>• <b>Patients</b> from the main menu. Select:               <ul style="list-style-type: none"> <li>○ Recent Orders</li> <li>○ Blood Availability</li> <li>○ Recent Transfusions/Issued Units</li> <li>○ Patient Testing Worklist Report</li> <li>○ Print Unit Caution Tag &amp; Transfusion Record Form</li> <li>○ Issue Blood Components</li> <li>○ Post-Transfusion Information</li> <li>○ Return Issued Units To Blood Bank and click OK.</li> <li>○ Special Instructions &amp; Transfusion Requirements</li> <li>○ Transfusion Reaction History</li> <li>○ Patient History Report</li> </ul> </li> <li>• <b>Shipments</b> from the main menu. Select <b>Incoming Shipment</b> (for restricted units only).</li> <li>• <b>Orders</b> from the main menu. Select <b>Recent Patient Orders</b>.</li> </ul>

User Action	VBECS
	<ul style="list-style-type: none"> <li>• <b>Reports</b> from the main menu. Select: <ul style="list-style-type: none"> <li>○ <b>Blood Availability</b> (for individual patient only)</li> <li>○ <b>Medication Profile</b></li> <li>○ <b>Patient History Report</b></li> <li>○ <b>Transfusion Effectiveness Report</b> (for individual patient report only)</li> </ul> </li> <li>• <b>Supervisor</b> from the main menu. Select: <ul style="list-style-type: none"> <li>○ Document ABO Incompatible Transfusion</li> <li>○ Justify ABO/Rh Change</li> </ul> </li> </ul> <p>Selecting a patient record that is not found in the VBECS database will insert a new patient record from VistA.</p>
2. Select a patient from the list.	<ul style="list-style-type: none"> <li>• Displays patient-associated data.</li> <li>• Displays the Patient Information Toolbar.</li> <li>• Displays Special Instructions (SIs) and Transfusion Requirements (TRs), when appropriate.</li> </ul> <p><b>NOTES</b> _____</p> <p>Displays a message when the user selects a patient with a blank first or last name, or the name is too long. The user is instructed to contact the VBECS Administrator for assistance.</p>

**Figure 124: Patient Search**

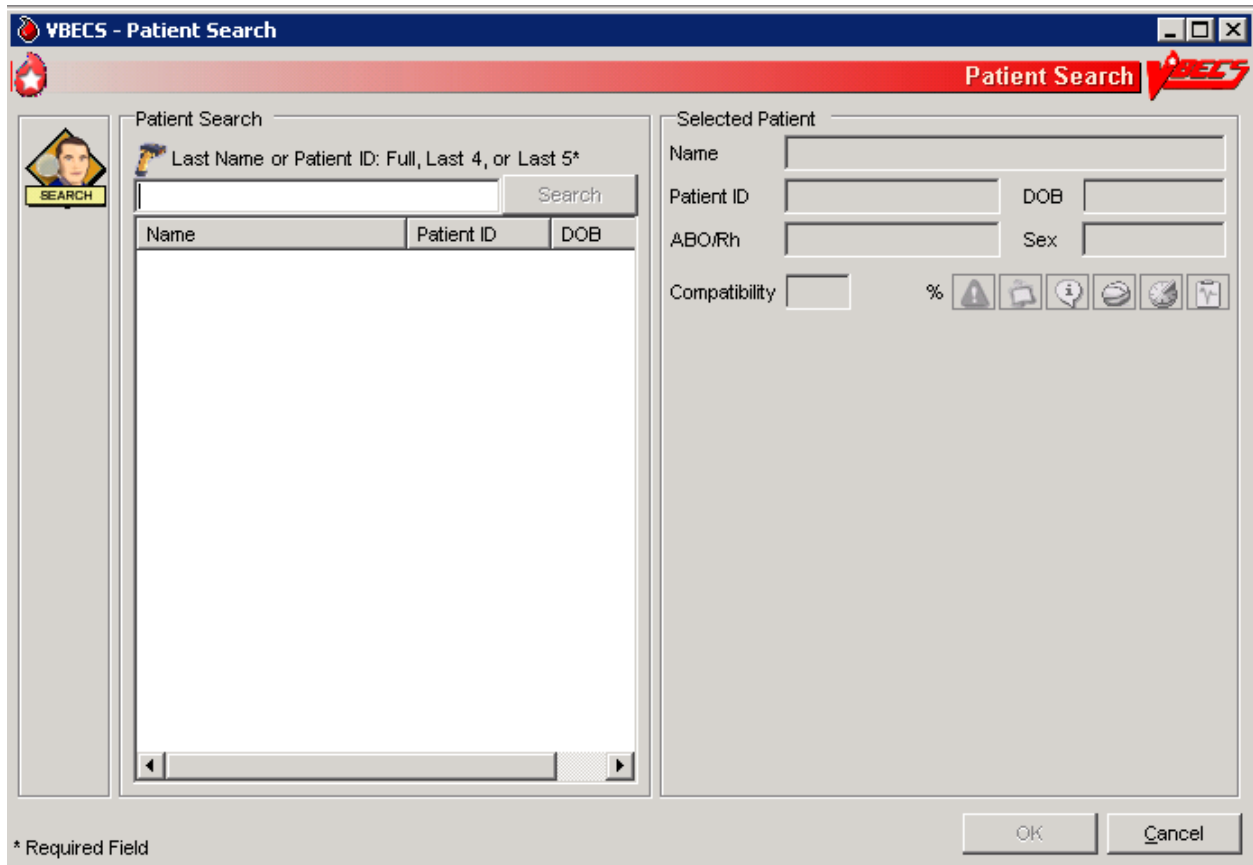

## Patient Merge

VBECS notifies the user of VistA patient merges and displays details for his review.

### Assumptions

- The connection to VistA is active.
- VBECS received the message about a VistA patient merge.
- VBECS accepts merge messages when either or both the merge from or merge to patients exist in VBECS.

### Outcome

- VBECS notified the user that patient records were merged in VistA.

### Limitations and Restrictions

- The Traditional Supervisor or above must update patient records manually.
- VBECS does not notify users of additional VistA patient updates or merges while the icon is displayed. When a user views an update and there is an unprocessed patient merge, the Patient Alert icon remains on the screen until the Traditional Supervisor completes the update and clears the icon.
- Patient merges involving patients with names that exceed the database standard for File #2 (30 characters) or do not have a first or last name in VistA are rejected and do not appear in VBECS. The VBECS System Administrator is notified of each rejection via an email to the address entered in VBECS Administrator “Email Alerts” field.
- VBECS displays duplicate merge events to the user for merge. Users can process one of the merge events manually and clear both merge alerts. The Master Veteran Index team is working on a patch to discontinue sending a duplicate message.

### Additional Information

- If more than two records are merged in VistA, VBECS will receive a corresponding number of merge messages.

### User Roles with Access to This Option

All users

## Patient Merge

The VBECS user examines records that were merged in VistA and decides whether to update them.






*VBECS does not create links for merged records. Users must update patient information and Special Instructions to show the merged-patient relationship.*




*Patient merges that do not change the patient name or social security number (e.g. patient record moved to a new DFN) are displayed to the VBECS user. However, the records displayed in the Merge*

*window will not appear to have any differences. Users are advised to contact customer support if a record merge of this type occurs.*

User Action	VBECS
<p>1. Select <b>Patients</b> from the main menu.</p> <p>Select <b>Patient Updates</b>.</p> <p>Select <b>Patient Merge</b>, or</p> <p>Click  in the Patient Information Toolbar.</p>	<ul style="list-style-type: none"> <li>• Displays  and notifies the user of the VistA patient merge.</li> <li>• Lists the patient records to be updated.</li> <li>• Displays limited merge-from and merge-to patient information for evaluation, including:               <ul style="list-style-type: none"> <li>○ Patient name</li> <li>○ Patient ID, if available</li> <li>○ Date and time the merge message was received from VistA</li> </ul> </li> </ul> <p><b>NOTES</b> _____</p> <p>VBECS displays  at the first instance of a VistA patient update or merge. The icon remains on the screen until the Traditional Supervisor clears the patient merge alert.</p> <p>The list of patient records is sorted in chronological order: the oldest received updated record appears at the top of the list.</p> <p>The user may update both patients' VBECS files in accordance with local policy and procedure.</p>
<p>2. Select a VistA patient merge event from the list and review the data (Figure 125).</p>	<ul style="list-style-type: none"> <li>• Displays data about merged VistA patient records:               <ul style="list-style-type: none"> <li>○ Patient name</li> <li>○ Patient ID, if available</li> <li>○ Patient sex, if available</li> <li>○ Date and time the merge message was received from VistA</li> <li>○ Patient ABO/Rh, if available</li> <li>○ Patient Transfusion Requirements (TRs) and Special Instructions (SIs), if available</li> <li>○ Recent orders and recent transfusion records for the patient (available only by clicking Patient Information Toolbar icons), if available</li> </ul> </li> </ul> <p><b>NOTES</b> _____</p> <p>“Patient has no VBECS records at the receipt of the merge message” displays when the merge-from or merge-to patient is not found in VBECS.</p>
<p>Merge-to Patients</p> <p>3. Update the merge-to patient record to include SIs and TRs from the merge-from patient record.</p> <p>Insert a comment in the Special Instruction field to note that a merge occurred.</p>	<p><b>NOTES</b> _____</p> <p>A user cannot update SIs or TRs if the merge to patient does not have a VBECS record.</p> <p>Local policy must define how to handle records when the merge to patient does not exist in VBECS.</p> <p>The merge-to patient record is the active record in VBECS.</p> <p>Use Special Instructions &amp; Transfusion Requirements: Enter a Transfusion Requirement, and Special Instructions &amp; Transfusion Requirements: Enter and Remove Special</p>



User Action	VBECS
	<p>Instructions to enter, update, or delete relevant component requirements, antibodies, or antigen requirements.</p> <p>Enter an SI comment in the merge-to patient record, for example, "This patient record was merged with &lt;insert merge-from patient name&gt; in VistA and is used to process orders after &lt;insert merge date and time&gt;. Refer to the records of &lt;insert merge-from patient name&gt; for testing and transfusion information prior to the merge event."</p>
<p>4. Resolve ABO/Rh discrepancies among patients.</p> <p>Determine which blood type is correct for use.</p>	<p><b>NOTES</b> _____</p> <p>Use Justify ABO/Rh Change to finalize blood type changes for the merge-to patient.</p>
<p>5. Complete pending orders for the merge-to patient.</p>	
<p>Merge-from Patients</p> <p>6. Update the merge-from patient record to include SIs and TRs from the merge-to patient record</p> <p>Insert a comment in the Special Instruction field to note that a merge occurred.</p>	<p><b>NOTES</b> _____</p> <p>A user cannot update SIs or TRs if the merge from patient does not have a VBECS record.</p> <p>Local policy must define how to handle records when the merge from patient does not exist in VBECS.</p> <p>The merge-from patient record is the inactive record in VBECS.</p> <p>Use Special Instructions &amp; Transfusion Requirements: Enter a Transfusion Requirement, and Special Instructions &amp; Transfusion Requirements: Enter and Remove Special Instructions to enter, update, or delete relevant component requirements, antibodies, or antigen requirements.</p> <p>Enter an SI comment in the merge-from patient record, for example, "This patient record was merged with &lt;insert merge-to patient name&gt; in VistA and must not be used to process orders after &lt;insert merge date and time&gt;. Refer to the records of &lt;insert merge-to patient name&gt; for testing and transfusion information after the merge event."</p>
<p>7. Cancel pending orders on the merge-from patient. Request replacement orders for the merge-to patient, as appropriate.</p>	<p><b>NOTES</b> _____</p> <p>Local policy must define how to handle orders when the merge from patient or merge to patient does not exist in VBECS.</p>
<p>8. To confirm that the merge-from and merge-to patient SIs and TRs match on the Patient Merge display.</p>	<p><b>NOTES</b> _____</p> <p>Local policy must define how to handle records when the merge from patient or merge to patient does not exist in VBECS.</p>
<p>9. Click <b>Clear Patient Merge Alert</b>.</p>	<ul style="list-style-type: none"> <li>• When there are no pending patient updates, clears  (the Patient Alert icon) from the main status bar.</li> <li>• Displays an information message stating that the user viewed the record.</li> </ul> <p><b>NOTES</b> _____</p>

User Action	VBECS
	<p>➤➤➤➤ The Clear Patient Merge Alert function is available only to the Traditional Supervisor and above.</p>
10. Click <b>OK</b> to acknowledge the message.	<ul style="list-style-type: none"> <li>Clears the icon when the user acknowledges the message.</li> </ul>
11. Click <b>Close</b> to exit, or return to Step 2.	

**Figure 125: Patient Merge**

**Patient Merge**

Merge Patient Records

Merge-From Name	Merge-To Name	Merge-From ID	Merge-To ID	Date Updated
VBPATIENT, SEVENHUNDREDEIGHT	VBPATIENT, EIGHTYEIGHT	N/A	000-00-0088	01/09/2009 17:17
VBPATIENT, NINETYFIVE	VBPATIENT, SEVENHUNDREDSEVEN	000-00-0095	000-00-0707	01/09/2009 17:17

Selected Merge Record

Date Merge Notification Received: 01/09/2009 17:17

Merge-From Patient		Merge-To Patient	
Name	VBPATIENT, SEVENHUNDREDEIGHT	Name	VBPATIENT, EIGHTYEIGHT
Patient ID	Not available	Patient ID	000-00-0088
ABO/Rh	Not available	ABO/Rh	NR
Sex	N/A	Sex	Male

Transfusion Requirements

Component Requirements

**03/03/2009 19:25, DA, 589:**  
**Washed RBC products**  
**Leuko-reduced cellular products**  
**Irradiated cellular products**

Antigen Neg. RBC    Patient Antibodies

\* Required Field

Clear Patient Merge Alert    Close

## Updated Patients and Deceased Patients

The user views patient updates and death events for patients with pending or active orders in VBECS.

### Assumptions

- The connection to VistA is active.
- The patient record was updated in VistA.
- VBECS received a message from VistA about a patient record update or death.
- The affected patient's records are in the VBECS database.
- The affected patient has a pending or active order in VBECS for updates and/or death event.

### Outcome

- The user reviews updated patient records.

### Limitations and Restrictions

- Demographic and death event updates do not display information for patients without pending or active orders in VBECS. Updates will occur without user input.
- VBECS does not notify users of additional VistA patient updates or merges while the icon is displayed. When a user views an update and there is an unprocessed patient merge, the Patient Alert icon remains on the screen until the Traditional Supervisor completes the update and clears the icon.
- When VBECS notifies a user of an update to a patient's prefix or suffix, VBECS does not display the updated information: the user must access VistA to view the updated information.
- Suffix data may be outdated but will be refreshed upon a new order for a patient.
- When a VistA patient update removes a patient's middle name, VBECS displays both the Patient Name and the Previous name in the Patient Name column. The Previous Name column displays the previous name correctly. Once the user closes and re-opens the Patient Update window, the subsequent display of the patient information is correct.

### Additional Information

- None


### User Roles with Access to This Option

All users

## Updated Patients and Deceased Patients

The user views updated patient records.

User Action	VBECS
1. Select <b>Patients</b> from the main menu.  Select <b>Patient Updates</b> .  Select <b>Updated Patient</b> or <b>Deceased Patient</b> , or	<ul style="list-style-type: none"><li>• Displays options for processing patient-related functions.</li><li>• Lists updated or deceased patients (also lists these data after the update if they changed), including:<ul style="list-style-type: none"><li>○ Patient name</li><li>○ Patient ID</li><li>○ Patient date of birth</li><li>○ Date and time of the update, or date of death</li></ul></li></ul>

User Action	VBECS
Click  in the Patient Information Toolbar, when it is active.	
2. Click <b>Close</b> to exit, or return to Step 1 to view another patient's data.	

## Patient Testing: General Instructions

The user records a patient's serologic test results.

### Assumptions

- The patient specimen and order were processed and found satisfactory for testing.
- The user selected one to four diagnostic and/or component tasks for testing from the Pending Task List (PTL).
- The patient may or may not have a previous blood bank record. Previously saved VBECS data related to this patient are available for comparison with the user's test results and interpretations.

### Outcome

- The test record was filed, is unique, and is retrievable for the life of VBECS.
- Partially completed testing is available for completion or invalidation but is not available to the patient record or reports.
- Completed, current specimen test results are available for inclusion in various reports.
- VBECS changes the task status to "completed" and triggers the generation of a message to CPRS that the order is complete.
- The test record includes the details of the verified, completed testing.

### Limitations and Restrictions

- This test record is not editable after the user completes a test and saves it to the database. When a correction needs to be filed, a corrected report must be generated, as described in Invalidating Patient Test Results.
- User selection of partially completed testing grids with different phases of testing (All Phases or AHG Only) as previously selected creates a testing discrepancy in VBECS.
- Limited batch testing to partially completed orders with the same testing grid selection, either All Phases or AHG Only.
- VBECS cannot compare test results for a patient when two users are entering test results for the same patient at the same time.
- Cost entry is capped at 999.99.

### Additional Information




- Partially completed tests may be invalidated within the testing option.

### User Roles with Access to This Option

All users

## Patient Testing: General Instructions

A user records a patient's test results: Type & Screen (TAS), ABO/Rh, Antibody Screen Test (ABS), Direct Antiglobulin Test (DAT), and crossmatches (XMs) on units selected for a patient. This option describes the configuration of reflex or repeat testing and patient antigen typing.

User Action	VBECS
<p>1. Select <b>Patients</b> from the main menu.</p> <p>Select <b>Patient Testing</b>.</p>	<ul style="list-style-type: none"> <li>• Displays options for processing patient-related functions.</li> <li>• Displays the Pending Task List (PTL) in the Diagnostic Tests tab.</li> </ul> <p><b>NOTES</b> _____</p> <p>A user may search for specific tasks using search criteria detailed in Pending Task List.</p>
<p>2. Click one to four check boxes in the Pending Task List to select tasks for testing (Figure 121).</p> <p>Click <b>OK</b>.</p>	<ul style="list-style-type: none"> <li>• Displays patient names and associated task information.</li> </ul>
<p>3. Click a check box to select a reagent rack.</p> <p>In the ABS or XM Testing Phases area, click the <b>All Phases</b> or <b>AHG Only</b> radio button (Figure 126).</p> <p>Click <b>OK</b> to continue with testing.</p>	<ul style="list-style-type: none"> <li>• Allows the user to select the All Phases or AHG Only radio buttons only when applicable.</li> <li>• Displays reagent rack names and associated information.</li> <li>• Requires the user to select the reagent rack name. The testing grid configuration for serologic testing defaults to the setting selected in the Configure Division option. The user is allowed to indicate whether an automated instrument was used.</li> <li>• Displays the date and time testing was performed.</li> <li>• Displays the testing technologist name.</li> </ul> <p><b>NOTES</b> _____</p> <p>When the testing method selected is AHG only, VBECS enables specific phases for the ABS and XM data grid boxes for:</p> <ul style="list-style-type: none"> <li>• Antibody Screen (ABS): AHG phase</li> <li>• Serologic crossmatch (XM): IS and AHG phases</li> </ul> <p>Rack configuration of AHG applies only to the antibody screen and crossmatch testing grids.</p> <p>The user may select another user's name from the list, as appropriate. The logged on user's name appears on the Testing Worklist Report as the Processing Tech in addition to the selected user name.</p> <p>The user may indicate that testing was performed at a past date and time.</p> <p> See Table 10 for alerts that may occur during this option.</p>
<p>4. Respond to warnings, which may include entering a comment and details, and click <b>OK</b> to continue.</p>	<ul style="list-style-type: none"> <li>•  Displays  and emits an audible alert when there are SIs or TRs for the patient.</li> <li>• Prepares a direct data entry (DDE) grid for the user to enter the serologic reaction results for testing associated with the selected tasks.</li> <li>• Displays testing results and entered interpretations until the user saves them.</li> </ul>
<p>5. Select a patient to begin entering his serologic reactions (Figure 127).</p>	<ul style="list-style-type: none"> <li>• Allows the user to select one patient at a time and enter a test result for that patient.</li> <li>• Allows only valid data entries in the grid reaction result and interpretation cells.</li> <li>• Allows a user to save partially completed tests.</li> </ul>


User Action	VBECS
<p>Repeat this step for each selected patient until all test results and interpretations are entered in the test grid.</p>	<ul style="list-style-type: none"> <li>When the user enters interpretation data, allows the save with or without warnings and overrides based on the test results entered for the patient and the user's security level.</li> </ul> <p><b>NOTES</b> _____</p> <p>The user may order a reflex test and may update the patient's TRs and SIs.</p> <p>The ABO and Rh tests are considered one test and must be result together before a save is permitted.</p>
<p>6. Respond to warnings for the selected patient and confirm that appropriate test results were entered.</p> <p>Repeat Steps 5–6 for each patient until all patient work is saved.</p>	<ul style="list-style-type: none"> <li>When a user enters a valid interpretation and saves it for a test, updates the order status to "completed."</li> <li>Displays each patient's data for review before allowing the user to save them.</li> <li>Saves patient testing results for retrieval from completed tests.</li> <li>Saves only the patient testing displayed.</li> </ul> <p><b>NOTES</b> _____</p> <p>The user may save part or all of one patient's testing results at a time. When all data for a patient are saved, the patient tab is no longer visible.</p> <p>When a user saves a test with a "correction" task status, VBECS saves the results and interpretations and adds a comment to the Testing Worklist Report.</p> <p>VBECS sends an order completion message to CPRS when the user enters and successfully saves all test results and interpretations for a diagnostic test.</p> <p>If the antibody screen is positive, the system shall emit an audible alert and warning that displays Patient has a POSITIVE Antibody Screen Result. Please perform Antibody Identification on this patient prior to issue of blood components.</p> <p> May display multiple warning and override messages for the user's response.</p>
<p>7. When the selected patient testing is complete, click <b>OK</b> and <b>Yes</b> to confirm changes and continue to the next patient, or exit.</p>	

Figure 126: Testing Details

**VBECS - Testing Details**

**Testing Details**

**TESTING**

Date Tested\* 02/03/2006 11:37

Tested By\* vhaishvbecs2

Reagent Rack\*

Rack	Tested
<input checked="" type="checkbox"/> A	No

ABS or XM Testing Phases\*

All Phases

AHG Only

Using Automated Instrument

Instrument Name\*

Instrument QC'd?

\* Required Field

OK Cancel



**Figure 127: Patient Testing**

## Alerts

**Table 10: Alerts That May Occur in Patient Testing: General Instructions**

User Action Step	Alerts
4	When the division is configured as “full service” in Configure Division, VBECS displays data fields for the user to select or enter the lot number, and displays the manufacturer and expiration date of each antiserum to be used in this transaction to allow the user to choose the correct antiserum. VBECS displays in-date antisera and allows the user to select outdated antisera, if needed, emits an audible alert, and requires an override. VBECS captures details for inclusion in an Exception Report (exception type: expired antisera used).
5	The user may cancel or delete a partially completed grid. VBECS: <ul style="list-style-type: none"> <li>• Emits an audible alert and requires a comment.</li> <li>• Clears previously entered observed results from the data grid and allows the user to enter new data.</li> <li>• Captures details for inclusion in an Exception Report (exception type: previously recorded results invalidated). Note that the time displayed in the "Date/Time results invalidated" field is the time the results were entered.</li> </ul>
6	Warns that the user is saving the first instance of a historic ABO/Rh.

User Action Step	Alerts
6	When the ABO/Rh interpretation matches observed results, is not inconclusive, and does not match the patient's historic ABO/Rh (if any, including from database conversion), VBECS emits an audible alert, warns that the current ABO/Rh does not match previous results, and instructs the user to enter a comment to continue, or click <b>Cancel</b> to clear the ABO/Rh testing results and interpretation from the screen. VBECS captures details for inclusion in an Exception Report (exception type: ABO/Rh discrepancy).
6	When the interpretation of a repeat ABO/Rh does not match the original interpretation, VBECS: <ul style="list-style-type: none"> <li>• Verifies the retest results.</li> <li>• Emits an audible alert and warns the user.</li> <li>• Instructs the user to resolve the discrepancy before the specimen may be used.</li> <li>• Asks whether the user wishes to continue to save. <b>No</b> clears the ABO/Rh testing results and interpretation from the screen. <b>Yes</b> requires a comment and captures details for inclusion in an Exception Report (Exception type: ABO/Rh discrepancy).</li> </ul>
6	When a user documents an ABO/Rh discrepancy, VBECS releases units with "assigned" and "crossmatched" statuses associated with this specimen from the patient.
6	When a unit was issued and a conflicting test result found that indicates a potential unit incompatibility with the patient, such as a patient ABO/Rh discrepancy, the unit XM is incompatible with the patient, the patient ABS is positive, the patient DAT is positive, or the unit is antigen positive where antigen negative is required on the tested specimen, VBECS warns the user and instructs him to notify the physician of this potential problem immediately, according to hospital policy. VBECS warns the user during the execution of the various tests related to the compatibility of a blood unit and captures details for inclusion in an Exception Report (exception type: unit issued, testing problem).
6	If the antibody screen is positive, the system shall emit an audible alert and warning that displays Patient has a POSITIVE Antibody Screen Result. Please perform Antibody Identification on this patient prior to issue of blood components.

## Patient Testing: Record a Patient ABO/Rh

The user records observed results when performing ABO/Rh blood typing.

### Assumptions

- A CPRS order exists that includes an ABO/Rh typing [ABO/Rh Test or Type & Screen (TAS)].
- The user accesses patient ABO/Rh grids through Patient Testing: Record Patient Test Results.

### Outcome

- The ABO/Rh result is used to determine overall blood component compatibility.
- When ABO/Rh is ordered as a stand-alone test, VBECS changes the task status to “completed” and triggers the generation of a message to CPRS that the order is completed.

### Limitations and Restrictions

- VBECS does not allow entry of ABO or Rh subgroups as a 'group' and/or 'type' when the previous blood group/type is "NR", indicating no record of previous testing. See FAQ ABO Subgroup Interpretation and contact the Service Desk for assistance, prior to adding patient transfusion records in VBECS, when possible.
- When the patient has no previous ABO/Rh test record (NR), the ABO/Rh test associated with the Type and Screen test (TAS) must be completed and saved prior to entering and saving a repeat ABO/Rh test on this specimen.
- A user is permitted to select two partially completed tests with different test phases (e.g., AHG only and All Phases) along with a not started test. The testing phase for the not started test will present as AHG only even though results for all phases are required by VBECS. Entering only AHG test results for the specimen that was not started and saving will result in a partially completed test that cannot be completed or invalidated. Contact the Service Desk when you encounter this problem. Specimens that cannot be completed or invalidated require the attention of Tier 3 support. Select partially completed orders that have been previously saved with the same grid test pattern. Optionally, a not started test may be selected with these but must be tested with the same grid pattern to avoid the problem.
- When canceling out of a Discrepant ABO/Rh Typing override, the interpretations are cleared, but the testing results are not cleared. Correct the testing entries or click the red 'X' to clear the testing results.

### Additional Information

- When an ABO/Rh discrepancy exists, VBECS releases assigned and crossmatched units. These units are then available for emergency issue based on system rules.
- VBECS does not enforce local policies that require running a D control for AB positive patients.
- The most recent instance of ABO/Rh testing in the database serves as the historic ABO/Rh record.
- Weak D testing results recorded in Patient Antigen Typing do not interact or change the Rh typing recorded for the patient by an ABO/Rh test.

### User Roles with Access to This Option


All users

## Patient Testing: Record a Patient ABO/Rh

This option describes a test to determine the patient's ABO/Rh typing from a blood specimen. The user enters the serologic reaction results in the data entry grid using the instructions and rules in this option. The grid is used for the Confirm Patient ABO/Rh reflex test as part of a TAS and crossmatch (XM) test battery. VBECS monitors the user's entries and interpretations and maintains the records for patient reports.

These steps describe the entry of data for a patient ABO/Rh test when the test is ordered individually or as part of a TAS.

User Action	VBECS
<p>1. Select <b>Patients</b> from the main menu.</p> <p>Select <b>Patient Testing</b>.</p>	<ul style="list-style-type: none"> <li>Displays options for processing patient-related functions.</li> <li>Displays the Pending Task List (PTL) in the Diagnostic Tests tab.</li> </ul> <p><b>NOTES</b> _____</p> <p>A user may search for specific tasks using search criteria detailed in Pending Task List.</p>
<p>2. Click one to four check boxes in the Pending Task List to select tasks for testing.</p> <p>Click <b>OK</b>.</p>	<ul style="list-style-type: none"> <li>Displays patient names and associated task information.</li> </ul>
<p>3. Enter patient ABO/Rh testing results.</p>	<ul style="list-style-type: none"> <li>Displays a direct data entry (DDE) grid for entering serologic reaction results for ABO/Rh patient testing.</li> </ul> <p><b>NOTES</b> _____</p> <p>The user may reenter valid results or delete entries and start over before saving the results. The user may invalidate, but not edit, data in a grid from a previous session by clicking the red "X" in the upper right corner of the testing grid, as described in Invalidate Test Results.</p> <p>VBECS determines the validity of the user-entered patient ABO/Rh interpretation by comparing the reaction results pattern of the row with the truth table and with the previous ABO/Rh entries for the patient.</p>
<p>4. Select a patient to begin entering his serologic reactions.</p>	<ul style="list-style-type: none"> <li>Allows the user to select one patient at a time and enter a test result for that patient.</li> <li>Allows only valid data entries in the grid reaction result and interpretation cells.</li> </ul> <p><b>NOTES</b> _____</p> <p>When A,B or D control is not part of the daily QC configuration, VBECS disables it in the ABO/Rh test grid.</p> <p>When an AB Positive patient requires the performance of a control cell, the user must order a repeat ABO/Rh test.</p> <p>When A,B is enabled, the user enters a result for Anti-A,B as part of the patient test, it is included in determining the interpretation for that patient test. "X" is an acceptable entry.</p>

User Action	VBECS
	<p>A,B and D control are always enabled for the repeat ABO/Rh test grid, regardless of the QC configuration.</p> <p>Reflex testing grid fields are available for data entry regardless of QC rack configuration.</p> <p>See Table 4: Valid Interpretations. Row validation is established for the interpretation vs. reaction results pattern defined by system rules.</p>
<p>5. Repeat Steps 2–4 until some or all portions of the test grid for the selected patient are entered.</p> <p>Enter the interpretation for the patient ABO/Rh typing, if appropriate.</p>	<ul style="list-style-type: none"> <li>Prompts the user to confirm that all results were reviewed and are acceptable.</li> </ul> <p><b>NOTES</b> _____</p> <p>When the ABO/Rh interpretation matches observed results, is not inconclusive, and does not match the patient’s historic ABO/Rh (if any, including from database conversion), VBECS emits an audible alert, warns that the current ABO/Rh does not match previous results, and instructs the user to enter a comment to continue, or click <b>Cancel</b> to clear the ABO/Rh testing results and interpretation from the screen. VBECS captures details for inclusion in an Exception Report (exception type: ABO/Rh discrepancy).</p> <p> &gt;&gt; VBECS allows the Enhanced Technologist to enter an ABO/Rh interpretation that matches the patient’s previously recorded (historic) ABO/Rh interpretation when the system interpretation is inconclusive. This allows unit selection to continue based on the patient’s ABO/Rh interpretation entered by the user. VBECS requires the user to document the findings of any offline investigation, alerting the reviewer to appropriately follow up, according to local policy. VBECS emits an audible alert and warns the user. VBECS allows an override with comment and captures details for inclusion in an Exception Report (exception type: discrepant ABO/Rh override) that includes all interpretations and historic patient information.</p>
<p>6. When all required fields are complete, click <b>OK</b> to save and continue to the next patient, or exit.</p>	<ul style="list-style-type: none"> <li>Saves the data.</li> </ul> <p><b>NOTES</b> _____</p> <p>In a Transfusion Only ABO/Rh testing grid, free text comments cannot be saved. Create the comment in Canned Comments, Patient Testing context, and then select it. Inactivate the Canned Comment, if it is not to be used again.</p>

## Patient Testing: Record a Patient Antibody Screen

The user records the serologic results when performing the Antibody Screen Test (ABS).

### Assumptions

- The user accesses patient ABO/Rh grids through Patient Testing: Record Patient Test Results.
- A CPRS order exists that includes an ABO/Rh typing [ABO/Rh Test or Type & Screen (TAS)]. An ABS may also be ordered as a reflex patient test.
- Configure Daily QC and Configure Testing are complete.

### Outcome

- When an ABS is ordered as a stand-alone test, VBECS changes the task status to “completed” and triggers the generation of a message to CPRS that the order is complete.

### Limitations and Restrictions

- This option is for patient testing only.

### Additional Information

- The user performs lab tests and enters data simultaneously.

### User Roles with Access to This Option

All users

## Patient Testing: Record a Patient Antibody Screen

The user tests the patient’s specimen for the presence of antibodies-to-irregular red cell antigens. The user may perform this test alone or as part of a battery of tests.

The user accesses patient antibody screen grids through Patient Testing: Record Patient Test Results and enters observations of the serologic reactions in the data entry grid during testing. VBECS makes the interpretation available to CPRS and patient reports.

User Action	VBECS
1. Select <b>Patients</b> from the main menu.  Select <b>Patient Testing</b> .	<ul style="list-style-type: none"><li>• Displays options for processing patient-related functions.</li><li>• Displays the Pending Task List (PTL) in the Diagnostic Tests tab.</li></ul> <p><b>NOTES</b> _____</p> <p>A user may search for specific tasks using search criteria detailed in Pending Task List.</p>
2. Click one to four check boxes in the Pending Task List to select tasks for testing.  Click <b>OK</b> .	<ul style="list-style-type: none"><li>• Displays patient names and associated task information.</li></ul>

User Action	VBECS
<p>3. Click a check box to select a reagent rack.</p> <p>In the ABS or XM Testing Phases area, click the <b>All Phases</b> or <b>AHG Only</b> radio button.</p> <p>Click <b>OK</b> to continue with testing.</p>	<p><b>NOTES</b> _____</p> <p>When the testing method selected is AHG only, VBECS disables the data grid boxes for phases IS, 37, and CC.</p>
<p>4. When processing more than one patient, select a name from the drop-down list in the Name field (or click the right or left arrow) to record testing for that patient.</p>	<ul style="list-style-type: none"> <li>• Allows the user to select a patient and enter a test result for the patient.</li> <li>• Allows only valid data entries in the grid reaction result and interpretation cells.</li> </ul> <p><b>NOTES</b> _____</p> <p>The logic described in Appendix B: Table 16: Antibody Screen Test Interpretation is an integral part of the configuration of the reaction result grid developed for Patient Testing: Record a Patient Antibody Screen.</p> <p>There may be two to five rows of screening cells to be validated as a group (the ABS). The user enters only one interpretation for the combination of cells tested.</p> <p>Valid interpretations of an ABS Test are positive or negative.</p> <p>If the antibody screen is positive, the system emits an audible alert and warning that displays that the Patient has a POSITIVE Antibody Screen Result. Perform Antibody Identification on this patient prior to issue of blood components.</p>
<p>5. Repeat Step 4 until results are entered in some or all portions of the test grid for the selected patient. Enter the interpretation for the patient ABS Test, if desired.</p>	<ul style="list-style-type: none"> <li>• Verifies that appropriate tests are entered.</li> <li>• Prompts the user to confirm that results were reviewed and are acceptable.</li> </ul> <p><b>NOTES</b> _____</p> <p>When the ABS Test interpretation must be consistent with observed test results based on Table 16: Antibody Screen Test Interpretation, discrepancies must be resolved before VBECS verifies the results and adds them to the database. The user may reenter valid results or delete the entries and start over before saving.</p>
<p>6. Enter test results in preparation for updating the database.</p>	<ul style="list-style-type: none"> <li>• Saves the data.</li> </ul>
<p>7. Click <b>OK</b> to save, and <b>OK</b> to confirm and exit.</p>	

## Patient Testing: Record a Direct Antiglobulin Test

The user records observed results when performing a Direct Antiglobulin Test (DAT).

### Assumptions

- A CPRS order exists that includes a DAT, which the user may also order as a reflex patient test.
- The user is testing a patient specimen in Patient Testing: Record Patient Test Results.
- VBECS is configured to accommodate the DAT as part of a Type & Screen (TAS).

### Outcome

- When a DAT is ordered as a stand-alone test, VBECS changes the task status to “completed” and triggers the generation of a message to CPRS that the order is complete.

### Limitations and Restrictions

- VBECS does not check whether the lot and vial numbers selected for the positive control cells are different from those of the negative control cells.
- Comments entered in the DAT POS and NEG control cell rows are not displayed on reports in VBECS or CPRS. Enter comments only in the patient test row.

### Additional Information

- The site settings determine which reagent types and lot numbers are available.
- VBECS checks the rack for the DAT reagents in use. When the rack testing does not include Poly AHG, Anti-IgG, or anti-complement, VBECS displays additional positive and negative control rows with the test grid for the DAT sera.
- The user may select reagent lot numbers for positive and negative control cells: Reverse ABO Typing Cells (Set), A2 cell (vial), Screening Cells (set), Check Cells (vial), Panel (set), and Other (set).
- DAT tests indicated as performed using GEL with valid control cells no longer require the entry of check cell results. DAT tests performed by manual method will also no longer require entry of check cell results. Test results do not have to be entered, but the cell cannot be blank. An “X” can be entered to indicate “Not Tested”, if testing was not performed.

### User Roles with Access to This Option

All users


## Patient Testing: Record a Direct Antiglobulin Test


The user performs a DAT on a patient sample.

The DAT may generate additional reflex test orders such as antibody identification. VBECS saves the reactions and interpretation in the database as part of the patient’s record.

User Action	VBECS
1. Select <b>Patients</b> from the main menu.  Select <b>Patient Testing</b> .	<ul style="list-style-type: none"><li>• Displays options for processing patient-related functions.</li><li>• Displays the Pending Task List (PTL) in the Diagnostic Tests tab.</li></ul> <b>NOTES</b> _____  A user may search for specific tasks using search criteria detailed in Pending Task List.



User Action	VBECS
<p>2. Click one to four check boxes in the Pending Task List to select tasks for testing.</p> <p>Click <b>OK</b>.</p>	<ul style="list-style-type: none"> <li>• Displays patient names and associated task information.</li> </ul>
<p>3. When QC was not performed, enter a lot number for the antiserum.</p>	<ul style="list-style-type: none"> <li>• Displays reaction result grids for DAT tests for each patient.</li> <li>• Lists the AHG antisera related to the test.</li> <li>• Displays information about the lot number, including the manufacturer and expiration date.</li> <li>• Prompts the user to select the correct antisera and possibly the control cell entry.</li> </ul> <p><b>NOTES</b> _____</p> <p>Reagent lot numbers that are selectable for positive and negative control cells:</p> <ul style="list-style-type: none"> <li>• Reverse ABO Typing Cells (set)</li> <li>• A2 cell (vial)</li> <li>• Screening Cells (set)</li> <li>• Check Cells (vial)</li> <li>• Panel (set)</li> <li>• Other (set)</li> </ul> <p>VBECS lists the AHG antisera related to the test from previously entered inventory. When VBECS displays multiple DAT tests or the full battery, the user selects each DAT serum separately by repeating this step until all are selected.</p> <p>Each positive and negative control cell requires the user to select a lot number.</p> <p>VBECS displays options to select the lot number of the antisera and check cells used for QC of the antiglobulin sera. The user must select complement check cells for Anti-Complement AHG tests.</p> <p> VBECS compares the control cell results entered to those of the previous day for the same rack number and alerts the user when there is a decrease in reactivity of 2 or more within the acceptable limits for each reagent type by lot number. VBECS emits an audible alert, warns that there is a decrease in reagent reactivity of two or more, and asks whether the user wishes to continue to use this vial.</p> <p><b>Yes</b> allows the user to continue, with comment. VBECS captures details for inclusion in an Exception Report (exception type: QC decrease reagent reactivity <math>\geq 2</math>).</p> <p><b>No</b> allows the user to reenter the lot number for this reagent type and allows the approval of this same lot number (different vial) or the entry of a new lot number, returning to the testing worksheet where the user left off. (The user may replace an individual vial that is losing its reactivity with one of the same lot number.)</p>

User Action	VBECS
	<p>When the user-entered test results indicate a satisfactory interpretation, VBECS indicates that testing is satisfactory and allows the user to save the results. VBECS considers the affected QC incomplete until it is retested with satisfactory results. VBECS generates warnings through other options when QC testing is incomplete.</p> <p> VBECS compares the expiration date of the reagent on the selected reagent to the testing date. When the reagent is expired, VBECS emits an audible alert, warns the user and asks whether he still wishes to use this reagent. <b>Yes</b> requires a comment and captures details for inclusion in an Exception Report (exception type: expired reagent QC'd).</p> <p><b>No</b> removes the unsatisfactory reagent and requires the user to enter a new one. Unsatisfactory lot numbers are never displayed. VBECS captures details for inclusion in an Exception Report (exception type: Expired reagent QC'd).</p> <p>When the antisera QC is required, the user verifies the lot numbers, manufacturer name, and expiration date and saves them as part of the QC record for the associated test.</p>
4. Review and accept the selected reagent information.	<ul style="list-style-type: none"> <li>• Displays the previously entered information for user review.</li> </ul>
5. Respond to warnings and continue with DAT testing.	<ul style="list-style-type: none"> <li>• Displays a direct data entry (DDE) grid for the user to enter serologic reaction results for DAT testing.</li> </ul>
6. Enter valid test serologic reactions in the data entry grid.	<ul style="list-style-type: none"> <li>• Allows the user to enter valid data and test results in the interpretation cells.</li> </ul> <p><b>NOTES</b> _____</p> <p>Row validation is established for the interpretation vs. reaction results pattern as defined by system rules.</p>
7. Repeat Step 4 until some or all portions of the test grid for the selected patient are entered. When appropriate, enter the interpretation for the patient DAT testing.	<ul style="list-style-type: none"> <li>• Prompts the user to confirm that all results were reviewed and are acceptable.</li> </ul>
8. Enter test results in preparation for updating the database.	
9. Click <b>OK</b> to exit.	

## Patient Testing: Record a Crossmatch

The user records observed results and interpretations when performing a serologic crossmatch (XM) between a patient and one or more units of blood.

### Assumptions

- A CPRS order exists for a blood component that requires a serologic XM. The patient order and the selected units are in the same division as the user.
- A blood unit that contains  $\geq 2$  mL red blood cell contamination, selected in Select Units, is acceptable for testing, according to the patient's current or historic records and ABO/Rh.
- A current specimen was accepted and is in process for ABO/Rh and Antibody Screen Test (ABS), or was previously completed and is not expired.


### Outcome

- When patient Transfusion Requirements (TRs), if any, are fulfilled and a user enters a compatible XM interpretation, VBECS makes units available for generating the Blood Transfusion Record Form (BTRF), for printing the Caution Tag, and for issue to the patient.
- When a user enters an unacceptable XM interpretation, VBECS releases the selected units from the patient.

### Limitations and Restrictions

- The specimen associated with testing must have an expiration date and time that are later than the testing date and time entered in Patient Testing: Record Patient Test Results. When this is not the case, VBECS warns that the patient specimen expired. There is no override.
- This option does not include electronic crossmatch (eXM).

### Additional Information

-  AABB Standards (5.16.1) requires a serological XM to detect ABO incompatibility. A local policy must be in place if the site is not performing a IS AHG as part of their serologic crossmatch test, manual or using an instrument.
- When the user saves a partially completed XM testing worklist, VBECS does not clear patient entries without results or release them from the worklist, which VBECS saves for completion later.
- When VBECS saves a partially completed XM worksheet, it maintains a status of "assigned" for selected units until the serologic XM is complete. When VBECS saves the XM test, including interpretation, to the database, it assigns "crossmatched" status to the unit. The user may correct verified patient data only through Invalidate Test Results.
- Units that require antigen typing and/or modification to fulfill a component requirement and/or Sickle Cell testing may be crossmatched pending the completion of further processing of the unit.
- VBECS does not allow users in a transfusion-only facility to save partially completed worksheets. The user must enter interpretations for all selected units before VBECS enables the OK button.
- XM compatibility is one aspect of blood unit compatibility. VBECS takes into account the known requirements for compatibility is taken into account when the user prints Caution Tags and BTRFs, and includes such issues as antigen negative requirements and component requirements.



## User Roles with Access to This Option


All users


### Patient Testing: Record a Crossmatch



During Select Units, the user selects a blood unit to fill a patient order. If the unit selected contains red blood cells and eXM is not enabled, or the patient or unit does not qualify, VBECS directs the user to enter serologic XM results. (When eXM is enabled, the user may select serologic XM.)

The user performs the serologic XM test and records results in VBECS. VBECS monitors the user's entries and interpretations to determine (with the user) compatibility for each crossmatched unit.

User Action	VBECS
<p>1. Select <b>Patients</b> from the main menu and select <b>Patient Testing</b>, or</p> <p>Select <b>Blood Units</b> from the main menu and select <b>Select Units</b>.</p>	<ul style="list-style-type: none"> <li>Displays options for processing patient-related functions.</li> <li>Displays the Pending Task List (PTL) and PTL search parameters in the Diagnostic Tests tab or in the Component Orders tab.</li> </ul>
<p>2. Enter or select PTL search parameters, if appropriate.</p> <p>Click the check boxes in the PTL to select one to four component orders.</p> <p>Click <b>OK</b> to continue.</p>	<ul style="list-style-type: none"> <li>Displays information for each component order.</li> <li>When the selected order class is RED BLOOD CELLS and the division enabled eXM, evaluates the patient for eXM unit processing.</li> </ul> <p><b>NOTES</b> _____</p> <p> See Table 11 for alerts that may occur during this option.</p>
<p>3. Click <b>Add Units</b> to add units to the component orders selected (Figure 101).</p>	<ul style="list-style-type: none"> <li>Searches the database and displays available restricted autologous and directed units that the user must select before selecting allogeneic units of the same ICCBBA component class.</li> <li>Displays the unit ID, product type, unit ABO/Rh, unit expiration date, specimen UID, if applicable, of units assigned to the patient in the division's blood bank on a different specimen or order. The user may release and reassign the units to the current order.</li> <li>When eXM was configured for the facility, displays the eXM status (eligible or not eligible) for the patient.</li> </ul> <p><b>NOTES</b> _____</p> <p> See Table 11 for alerts that may occur during this option.</p> <p>When no valid ABO exists for the patient, the user may select components, including autologous units:</p> <ul style="list-style-type: none"> <li>For RBC or WB order: <ul style="list-style-type: none"> <li>Group O, Rh positive or O Rh negative RBC</li> <li>WHOLE BLOOD is not selectable</li> </ul> </li> <li>For FFP order: <ul style="list-style-type: none"> <li>Group AB, Rh positive or Rh negative, or not specified.</li> </ul> </li> <li>For PLT order: <ul style="list-style-type: none"> <li>All units with &lt; 2 ML RBC containment are available</li> <li>Units with = &gt; 2 mL RBC contamination are not available</li> </ul> </li> <li>For CRYO order: <ul style="list-style-type: none"> <li>All available units are selectable</li> </ul> </li> <li>For OTHER order: <ul style="list-style-type: none"> <li>All units with &lt; 2 mL contamination are available</li> </ul> </li> </ul>

User Action	VBECS
<p>4. Scan or enter allogeneic units, select compatible units from the Order Group list, or click the <b>find</b> button to select units.</p> <p>Enter or select a date in the Selection Date field. Each unit may have its own selection date and time.</p> <p>Click <b>Add Units</b> to display the unit selection form.</p> <p>To change the Assigned Date and Assigned By fields, click a unit in the Order Group list.</p>	<ul style="list-style-type: none"> <li>○ Units with = &gt; 2 mL RBC contamination are not available</li> </ul> <ul style="list-style-type: none"> <li>● Maintains a list of selected units. The user may deselect selected units.</li> <li>● Evaluates the selected unit: <ul style="list-style-type: none"> <li>○ For the ABO/Rh compatibility of each unit and warns the user.</li> <li>○ Against patient Transfusion Requirements (TRs) and warns the user.</li> <li>○ For unit antigen negative compatibility and warns the user.</li> </ul> </li> <li>● Performs additional checks that include days remaining until the unit's expiration date, the current status, assignments to other patients, quarantine and biohazard indicators, when applied, and associated special testing.</li> </ul> <p><b>NOTES</b></p> <p> See Table 11 for alerts that may occur during this option.</p> <p>The user may accept or edit the current date and time displayed; the date and time must be in the past.</p> <p>Restricted (autologous or directed) unit selection eligibility criteria:</p> <ul style="list-style-type: none"> <li>● Restricted For Patient ID: must match</li> <li>● Restricted For Patient Name: must match</li> <li>● Unit expiration date must be in the future.</li> <li>● Component class (transfusion only requires restriction of all units)</li> </ul> <p>The criteria for selecting or including an individual unit in a pick list include evaluating a unit for:</p> <ul style="list-style-type: none"> <li>● Considered substitutable for the order</li> <li>● Same division as the user</li> <li>● Current location is in the blood bank</li> <li>● Has no quarantine indicator</li> <li>● Is not already assigned or crossmatched to the patient</li> <li>● Is not "restricted" for a different patient</li> <li>● Meets ABO/Rh compatibility requirements specific to the component class</li> <li>● Future expiration date</li> </ul> <p>VBECS lists only ABO/Rh compatible units for selection.</p> <p>When ABO/Rh results were not created for the specimen associated with the component order and the order is not designated "emergency issue," VBECS allows the user to select units using historic records, and notifies the user that ABO/Rh testing must be performed at this division on this patient before any selected units can be issued.</p> <p>VBECS displays and allows the user to complete a pending XM associated with a patient order without a final unit status.</p>


User Action	VBECS
5. Repeat Step 4 until all units are selected.	<ul style="list-style-type: none"> <li>• Requests the user to confirm the list of selected blood units.</li> </ul> <p><b>NOTES</b> _____</p> <p>The user may select or deselect additional units.</p>
6. Click <b>OK</b> to confirm the selection of units.	<ul style="list-style-type: none"> <li>• Determines whether the units are available for issue or whether they require additional modification and/or testing based on the component class.</li> <li>• Updates the database, as appropriate.</li> </ul> <p><b>NOTES</b> _____</p> <p> See Table 11 for alerts that may occur during this option.</p> <p>When eXM is enabled at the division and blood units require XM, VBECS determines whether the patient and units are eligible for eXM. All parameters listed in Appendix B: Table 23: Rules for Electronic and Serologic Crossmatch must be met. Units not eligible for eXM must have serologic XM performed.</p> <p>When eXM is not enabled, the user must enter serologic XM results for red blood cell classes or when a user orders an optional XM for OTHER component classes. VBECS does not change the unit status to “crossmatched,” make units available for issue, or allow the Caution Tag or BTRF to be printed until a user enters a valid XM result, unless an emergency-issued unit is being processed.</p>
7. Select another component order to continue selecting units, or exit.	<ul style="list-style-type: none"> <li>• Displays orders for the next patient.</li> </ul>
8. Select a unit to begin entering serologic reactions.  Enter valid text, then press the <b>Enter key</b> to move to the next data grid comment cell.	<ul style="list-style-type: none"> <li>• Allows only valid data entries in the grid reaction result and interpretation cells.</li> <li>• Restricts data entry to valid selections for testing results. All testing results and entered interpretations remain on the screen.</li> </ul> <p><b>NOTES</b> _____</p> <p>During XM result data entry within a given row, the IS phase result must be entered first. Once the IS phase has been entered, data entry may progress and an observed result or NT entered in each phase before the XM interpretation is validated by VBECS.</p> <p>XM interpretations must be consistent with observed test results based on system rules for the testing method. Discrepancies must be resolved before VBECS can verify results and add them to the database. The user may reenter valid results or delete entries and start over.</p> <p>If the patient has a history of a clinically significant antibody or a persistent antigen negative requirement, VBECS warns that the patient is not eligible for IS XM and instructs the user to perform the antiglobulin transaction XM on the patient. There is no override. The user must complete all phases of the XM test.</p>

User Action	VBECs
<p>9. Repeat Step 3 until the test grid for the selected patient is complete.</p> <p>Enter the interpretation for the patient XM test, if desired.</p> <p>Respond to warnings and enter comments, when indicated.</p>	<ul style="list-style-type: none"> <li>Verifies that all appropriate test results have been entered for the patient and the selected unit(s).</li> <li>Displays various warnings based on rules.</li> <li>Prompts the user to confirm that results were reviewed and are acceptable.</li> </ul> <p><b>NOTES</b> _____</p> <p> When VBECs emits an audible alert and requires a comment, it captures details for inclusion in an Exception Report (exception type: inconclusive crossmatch).</p> <p>A user may enter a "Compatible" XM interpretation only after entering an observed test result for the IS phase (other phases are NT) when the antibody screen on the current specimen is complete and negative <i>and</i> when the patient has no calculated or persistent antigen negative requirement.</p> <p>When observed test result patterns are invalid, VBECs warns that test results are invalid and that the user must repeat tests. VBECs stores these results as documentation and clears the grid for the user to begin again.</p> <p> VBECs emits an audible alert and requires a comment documenting medical director approval, according to site policy. VBECs captures details for inclusion in an Exception Report (exception type: Crossmatch Incompatible: Give Only with Medical Director Approval).</p>
10. Enter test results in preparation for saving.	<ul style="list-style-type: none"> <li>Saves data.</li> <li>Prompts the user to generate a BTRF and a Caution Tag during Patient Testing: Record Patient Test Results when the crossmatched unit is acceptable for issue.</li> </ul>
11. Click <b>OK</b> to exit.	

## Crossmatch Interpretations

In all cases, VBECs saves the crossmatch information to the database.

**Table 11: Crossmatch Interpretations for Print Unit Caution Tag & Transfusion Record Form**

User Entry	Crossmatch Interpretation	VBECs Updates the Unit Status to:	Is Information Available to CPRS?	May User Print the BTRF and Caution Tag?	Other
C	Crossmatch Compatible	Crossmatched	Yes (unit is available)	Yes	
D	Crossmatch Compatible: Don't Transfuse	Available	No	No	
G	Crossmatch Incompatible: Give Only with Medical Director Approval	Crossmatched	Yes (unit is available)	Yes	
I	Crossmatch Incompatible	Available	No	No	

User Entry	Crossmatch Interpretation	VBECS Updates the Unit Status to:	Is Information Available to CPRS?	May User Print the BTRF and Caution Tag?	Other
Z	Inconclusive	Available	No	No	Applies to the manual serologic crossmatch test only



## Patient Testing: Record a Patient Antigen Typing

The user records observed results when performing a patient antigen typing test other than the ABO/Rh test.

### Assumptions

- Patient antigen typing is ordered through Order Reflex Tests.
- The user accesses patient antigen typing grids through Patient Testing: Record Patient Test Results.
- Reagent inventory was entered in Maintain Minimum Levels.

### Outcome

- The testing record includes the patient's antigen typing and the control cell information.

### Limitations and Restrictions

- Antigen typing applies only to red blood cell antigens. (This option does not address platelet, HLA, or IgA antigens.)
- VBECS does not check:
  - The patient antigen typing with the specificity of any patient's antibodies entered.
  - Whether the lot and vial numbers selected for the positive control cells differ from those of the negative control cells.
- The patient antigen typing result is not available to CPRS.

### Additional Information


- Clinical significance and availability of typing reagents are not implied when antigens are included in the list of antigen type tests.
- Unit antigen tests may be added through Incoming Shipment and Edit Unit Information.
- The patient antigen typing test is associated with a CPRS patient order.
- The patient's antigen typing information does not appear as part of the Transfusion Requirements or Special Instructions in the Patient Information Toolbar.
- When VBECS displays lot numbers for selection, it checks the inventory for the reagent name and lot numbers in use in a rack. VBECS does not offer reagents with an "unsatisfactory" value in the Inspection field or with a quantity of zero as a selectable reagent. The user may not enter a lot number that VBECS disallowed on the selection list. The user must choose a different reagent.
- When a user selects a reagent rack that was never QC'd, VBECS warns the user, clears the selection, and allows the user to select a different rack.
- VBECS automatically adds the positive and negative control tests to the selected reagent type worklist if they have not been tested on that calendar date for the selected antisera lot number.
- VBECS stores the lot number, vial identifier, manufacturer, and expiration date of the positive and negative control cells with the testing record.
- The user may not enter "H" (hemolysis) in antigen typing testing.
- Weak D testing results do not interact or change the Rh typing recorded for the patient by an ABO/Rh test.


### User Roles with Access to This Option


All users

## Patient Testing: Record a Patient Antigen Typing

This option describes a test to determine the patient antigen typing from a blood specimen.

User Action	VBECs
<p>1. Select <b>Patients</b> from the main menu.</p> <p>Select <b>Patient Testing</b>.</p>	<ul style="list-style-type: none"> <li>Displays options for processing patient-related functions.</li> <li>Displays the Pending Task List (PTL) in the Diagnostic Tests tab.</li> </ul> <p><b>NOTES</b> _____</p> <p>A user may search for specific tasks using search criteria detailed in Pending Task List.</p>
<p>2. Click one to four check boxes in the Pending Task List to select tasks for testing.</p> <p>Click <b>OK</b>.</p>	<ul style="list-style-type: none"> <li>Displays patient names and associated task information.</li> <li>Displays a direct data entry (DDE) grid for entering serologic reaction results for antigen typing testing. This grid may be partially populated from a previous session.</li> <li>Allows the user to select a unit and enter a test result for the unit.</li> <li>Displays options for finding and selecting tests to process.</li> </ul> <p><b>NOTES</b> _____</p> <p>When the user selects a partially completed worksheet, he must verify the lot numbers, manufacturer name, and expiration date of the antiserum and control cells previously selected. These fields may not be edited. (A user who finds a discrepancy must delete the worksheet and start over.)</p>
<p>3. Confirm the selected patient.</p> <p>When control cells were tested for the antiserum, go to Step 7.</p>	<ul style="list-style-type: none"> <li>Displays an option to select a lot number for each of the antiserum types selected.</li> </ul> <p><b>NOTES</b> _____</p> <p>VBECs displays grids and quality control cells for each antiserum type.</p>
<p>4. Select a lot number for each antiserum specificity.</p>	<ul style="list-style-type: none"> <li> Allows the user to select or enter the lot number and displays the manufacturer and expiration date of each antiserum to be used in this transaction so the user can select the correct antiserum. VBECs displays in-date antisera. When the user selects outdated antisera, VBECs emits an audible alert and requires a comment and an override. VBECs captures details for inclusion in an Exception Report (exception type: expired antisera used).</li> <li>Prompts the user to select the correct antiserum entry for the specificity selected.</li> <li>Displays the lot number information.</li> </ul> <p><b>NOTES</b> _____</p> <p>VBECs requests information about an antigen lot number only the first time it is used on a specific day.</p> <p>If there are multiple test tabs, VBECs displays a message that may not correspond to the selected tab, but will correspond to information required by one of the requested tests.</p>
<p>5. Verify the antiserum lot number selected (Figure 128).</p>	<ul style="list-style-type: none"> <li>Allows the user to select the testing phases for the antiserum specificity by lot number.</li> <li>Displays a testing grid and disables the phases that are not to be completed.</li> </ul>

User Action	VBECS
	<ul style="list-style-type: none"> <li>Stores the antiserum information and checks whether the positive and negative controls are needed.</li> </ul> <p><b>NOTES</b> _____</p> <p>The user must indicate the phases of reactivity for each antiserum specificity:</p> <ul style="list-style-type: none"> <li>IS</li> <li>IS/RT</li> <li>IS/37</li> <li>AHG/CC</li> </ul> <p>VBECS enables the Rack Identifier field when the user selects the weak D or AHG/CC phase. VBECS disables the rack identifier field when the user selects other phases.</p> <p>When the user QC'd the antigen typing reagent lot number and performed patient antigen typing on the same date under different testing phases, VBECS enables the QC grid for the antisera in the phases the user selected.</p> <p> When the user attempts to change the phases of reactivity previously recorded for the lot number, VBECS emits an audible alert, warns that the user selected phases that were not used previously for this reagent, and asks whether the user wishes to continue. <b>No</b> requires the user to select the correct testing phases. <b>Yes</b> requires the user to enter a comment explaining why he is selecting different testing phases. VBECS captures details for inclusion in an Exception Report (exception type: antigen testing phase change). VBECS requires a comment and captures details for inclusion in an Exception Report. Antigen testing phase change</p>
<p>6. Select the lot number for the reagent and the positive and negative control cells. Enter the vial identifiers, if required. Enter "0" when not using panel cells.</p> <p>Verify the reagent type and the positive and negative control cell lot numbers selected and vial identifiers entered, if required (Figure 129).</p>	<ul style="list-style-type: none"> <li>Displays the lot number information, including the vial identifier.</li> </ul> <p><b>NOTES</b> _____</p> <p>Selectable reagent lot numbers for positive and negative control cells are:</p> <ul style="list-style-type: none"> <li>Reverse ABO Typing Cells</li> <li>A2 cell</li> <li>Screening Cells</li> <li>Check Cells</li> <li>Panel</li> <li>Other (set)</li> </ul> <p>The vial identifier, comprising one or two digits, identifies the vial contained in a kit of reagent red blood cells. A vial contains antigen positive or antigen negative blood cells to be used in quality control of the antisera.</p>

User Action	VBECS																																	
<p>7. Respond to warnings and continue with antigen typing.</p>	<ul style="list-style-type: none"> <li>Displays a DDE grid for entering serologic reaction results for antigen typing testing. This grid may be partially populated with results from a previous session.</li> </ul> <p><b>NOTES</b> _____</p> <p>VBECS organizes the worklist for the grids by antiserum types.</p> <p>The row title of the data grid specifies the positive and negative control cells; the column title specifies the phases of reactivity that may be used in the test. VBECS fills in this information; the user may not edit it.</p>																																	
<p>8. Select one control cell or patient to begin entering serologic reactions and interpretations.</p>	<ul style="list-style-type: none"> <li>Allows the user to select a patient or control cell and enter test results.</li> </ul> <p><b>NOTES</b> _____</p> <p>Valid observed test results include:</p> <table border="1" data-bbox="727 741 1328 1115"> <thead> <tr> <th data-bbox="727 741 816 770">Entry</th> <th colspan="2" data-bbox="816 741 1328 770">VBECS Translation</th> </tr> </thead> <tbody> <tr> <td data-bbox="727 770 816 800">1</td> <td data-bbox="816 770 1166 800">1+</td> <td data-bbox="1166 770 1328 800">Positive</td> </tr> <tr> <td data-bbox="727 800 816 829">2</td> <td data-bbox="816 800 1166 829">2+</td> <td data-bbox="1166 800 1328 829">Positive</td> </tr> <tr> <td data-bbox="727 829 816 858">3</td> <td data-bbox="816 829 1166 858">3+</td> <td data-bbox="1166 829 1328 858">Positive</td> </tr> <tr> <td data-bbox="727 858 816 888">4</td> <td data-bbox="816 858 1166 888">4+</td> <td data-bbox="1166 858 1328 888">Positive</td> </tr> <tr> <td data-bbox="727 888 816 917">W</td> <td data-bbox="816 888 1166 917">Weak</td> <td data-bbox="1166 888 1328 917">Positive</td> </tr> <tr> <td data-bbox="727 917 816 947">F</td> <td data-bbox="816 917 1166 947">Mixed Field</td> <td data-bbox="1166 917 1328 947">Positive</td> </tr> <tr> <td data-bbox="727 947 816 976">M</td> <td data-bbox="816 947 1166 976">Microscopic</td> <td data-bbox="1166 947 1328 976">Positive</td> </tr> <tr> <td data-bbox="727 976 816 1056">X</td> <td data-bbox="816 976 1166 1056">Not Tested X (Not Tested) (does not indicate a disabled cell)</td> <td data-bbox="1166 976 1328 1056">Not Tested</td> </tr> <tr> <td data-bbox="727 1056 816 1085">0</td> <td data-bbox="816 1056 1166 1085">No Agglutination</td> <td data-bbox="1166 1056 1328 1085">Negative</td> </tr> <tr> <td data-bbox="727 1085 816 1115">R</td> <td data-bbox="816 1085 1166 1115">Rouleaux, No Agglutination</td> <td data-bbox="1166 1085 1328 1115">Negative</td> </tr> </tbody> </table> <p>System rules establish row validation for the interpretation vs. reaction results pattern [for all user-indicated phase(s) of reactivity] for blood unit and quality control tests.</p> <p>When completing previously saved partially completed testing, the user may not change the antiserum lot number, manufacturer name, or expiration date. The user must invalidate the antiserum testing and start over.</p>	Entry	VBECS Translation		1	1+	Positive	2	2+	Positive	3	3+	Positive	4	4+	Positive	W	Weak	Positive	F	Mixed Field	Positive	M	Microscopic	Positive	X	Not Tested X (Not Tested) (does not indicate a disabled cell)	Not Tested	0	No Agglutination	Negative	R	Rouleaux, No Agglutination	Negative
Entry	VBECS Translation																																	
1	1+	Positive																																
2	2+	Positive																																
3	3+	Positive																																
4	4+	Positive																																
W	Weak	Positive																																
F	Mixed Field	Positive																																
M	Microscopic	Positive																																
X	Not Tested X (Not Tested) (does not indicate a disabled cell)	Not Tested																																
0	No Agglutination	Negative																																
R	Rouleaux, No Agglutination	Negative																																
<p>9. Enter results for selected patients and control cells.</p>	<ul style="list-style-type: none"> <li>Prompts the user to confirm that results were reviewed and are acceptable.</li> </ul> <p><b>NOTES</b> _____</p> <p> VBECS searches the current order and the patient's historical record for previously entered red cell antigen results for each specificity within the user's division and compares the results to prevent a discrepancy with previous test entries. When a discrepancy is found, VBECS emits an audible alert, notifies the user that the current specimen and a previously entered antigen typing on this patient do not match, and asks whether the user wishes to save the current result.</p>																																	

User Action	VBECS
	<p><b>No</b> closes the form and returns to the grid. <b>Yes</b> allows the user to update the conflicting results in the database and/or enter a comment explaining why he is accepting this result. VBECS captures details for inclusion in an Exception Report (exception type: discrepant patient antigen typing).</p>
10. Repeat Steps 5–9 for each antiserum type.	<ul style="list-style-type: none"> <li>Allows the user to save.</li> </ul> <p><b>NOTES</b> _____</p> <p>The positive and/or negative control cell test results on the date of testing must be a valid positive or negative.</p>
11. Respond to warnings and click <b>OK</b> to save the data.	<ul style="list-style-type: none"> <li>Reconfirms that all appropriate test results are entered, repeats all data validation, and saves the data.</li> </ul>
12. Click <b>OK</b> to exit.	<ul style="list-style-type: none"> <li>Saves the data.</li> </ul>

**Figure 128: Patient Testing Antigen Typing (1)**

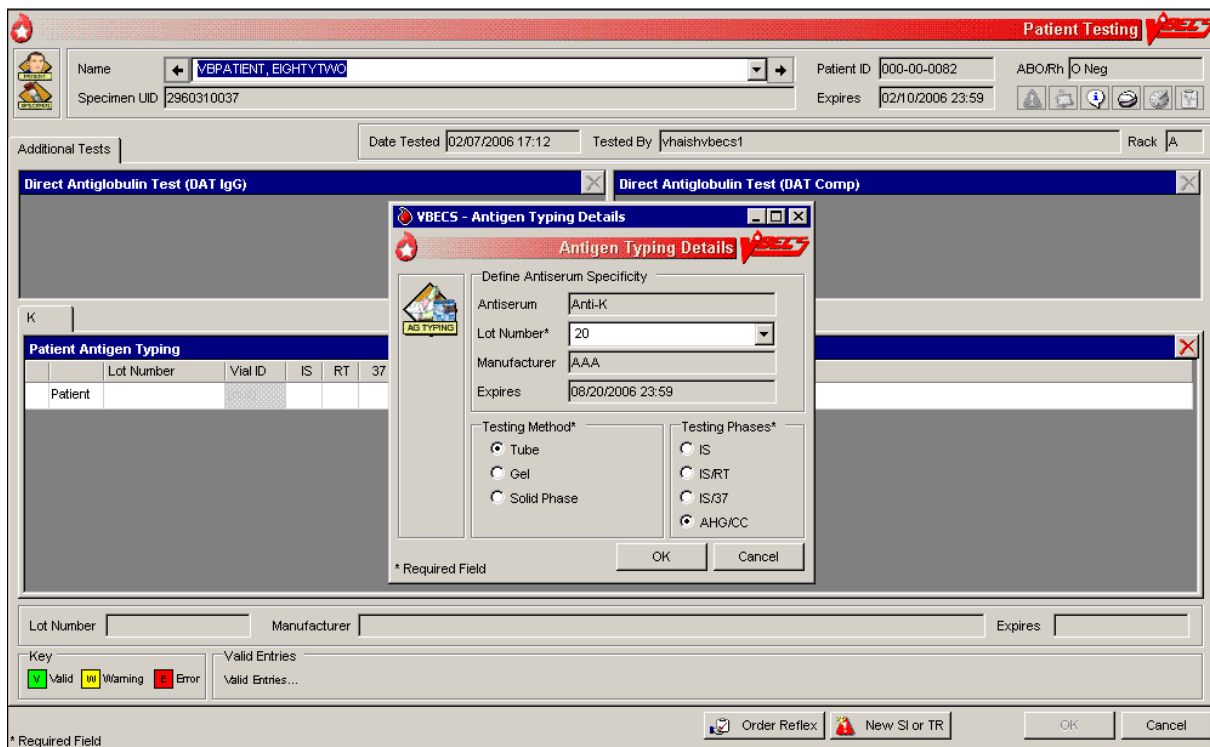


Figure 129: Patient Testing Antigen Typing (2)

**VBECs - Patient Testing** Patient Testing

Name: VBPAIENT, EIGHTYTWO Patient ID: 000-00-0082 ABO/Rh: O Neg  
 Specimen UID: 2960310037 Expires: 02/03/2006 23:59

Additional Tests: Date Tested: 02/03/2006 11:48 Tested By: vhaishvbecs2 Rack: A

Direct Antiglobulin Test (DAT IgG) Direct Antiglobulin Test (DAT Comp)

K

**Patient Antigen Typing**

	Lot Number	Vial ID	IS	RT	37	AHG	CC	Interp.	Comment
POS									
NEG									
Patient	20								

Lot Number: Manufacturer: Expires:

Key: V Valid W Warning E Error Valid Entries: Valid Entries...

\* Required Field Order Reflex New SI or TR OK Cancel

## Patient Testing: Enter Antibody Identification Results

The user records results of reflex tests needed to complete a patient blood bank antibody workup.

### Assumptions

- An order for an antibody identification (ABID) was placed and is opened in the Pending Task List (PTL).

### Outcome

- VBECS displays the ABID in the patient's record, which the user may review by clicking an icon in the Patient Information Toolbar.
- Entry of antibody specificity through this option may automatically set an antigen negative requirement [Transfusion Requirement (TR)] based on the division-specific parameters.

### Limitations and Restrictions

- When an antigen negative requirement is not configured for an antibody, the user may enter the requirement for a patient through Special Instructions & Transfusion Requirements: Enter a Transfusion Requirement.

### Additional Information

- Correction to verified patient data may be accessed only through Invalidate Test Results and does require extensive removal of associated units, even if transfused. As this is the case, optionally record workload and billing here but the antibody specificity identified using Special Instructions Transfusion requirements option to allow for future editing.
- Only one technologist at a time can enter test results for a selected PTL task.
- Patient antigen testing is not included in this option.

### User Roles with Access to This Option

All users

## Patient Testing: Enter Antibody Identification Results

The user may select workload codes for work performed on-site during the ABID workup. When the ABID is performed off-site, the user enters the associated charges.

User Action	VBECS
1. Select <b>Patients</b> from the main menu.  Select <b>Patient Testing</b> .	<ul style="list-style-type: none"><li>• Displays options for processing patient-related functions.</li><li>• Displays the Pending Task List (PTL) in the Diagnostic Tests tab.</li></ul> <b>NOTES</b> _____  A user may search for specific tasks using search criteria detailed in Pending Task List.
2. Click one to four check boxes in the Pending Task List to select tasks for testing.  Click <b>OK</b> .	<ul style="list-style-type: none"><li>• Displays patient names and associated task information.</li></ul>

User Action	VBECS
<p>3. Click a check box to select a reagent rack.</p> <p>Click <b>OK</b> to continue with testing.</p>	<p><b>NOTES</b> _____</p> <p>No serologic testing is recorded for ABID.</p>
<p>4. Select antibodies.</p> <p>Enter associated testing details.</p> <p>Respond to comments and continue.</p> <p>Verify Patient information.</p> <p>Click <b>OK</b>.</p>	<ul style="list-style-type: none"> <li>• Displays the list of antibodies and allows the user to select from the identified specificities and to add antibodies or to indicate that no antibody was identified.</li> </ul> <p><b>NOTES</b> _____</p> <p>Multiple workload codes may apply when work is performed on-site.</p>
<p>5. Review the selections displayed. Return to Step 1 to change selections when necessary.</p> <p>Accept the reviewed entries.</p>	<ul style="list-style-type: none"> <li>• Displays applicable data fields for the ordered ABID.</li> <li>• Allows the user to indicate the testing site and associated billing from outside sources.</li> </ul>
<p>6. Enter the location where testing was performed and cost, when off-site.</p>	<ul style="list-style-type: none"> <li>• Requires the user to indicate whether the ABID was performed at the division and/or off-site and saves that information with the ABID.</li> <li>• Allows the user to enter a charge, but not the workload, when the ABID work was performed off-site.</li> <li>• Allows the user to indicate multiples of any processes when a test was performed on-site.</li> <li>• Allows the user to enter comments.</li> <li>• Do not exceed 50 characters when entering the name of the off-site testing location.</li> </ul>
<p>7. Review the antibody tests entered and comments selected and, when necessary, change them before accepting (Figure 130).</p>	<ul style="list-style-type: none"> <li>• Displays the requirements for antigen negative blood components, if any, based on the antibody entered.</li> </ul>
<p>8. Click <b>OK</b> to save the information to the database.</p>	<ul style="list-style-type: none"> <li>• Saves the data.</li> </ul>
<p>9. Click <b>OK</b> to exit.</p>	



Figure 130: Patient Testing Antibody Identification

**Patient Testing**

Name: VB PATIENT, EIGHTY TWO Patient ID: 000-00-0082 ABO/Rh: O Neg  
 Specimen UID: 2960310037 Expires: 02/10/2006 23:59

Antibody ID: Date Tested: 02/07/2006 17:15 Tested By: yhaishvbecs1 Rack: A

Antibodies Identified\*

<input type="checkbox"/> Anti-C	<input type="checkbox"/> Anti-N	<input type="checkbox"/> Anti-V
<input type="checkbox"/> Anti-E	<input type="checkbox"/> Anti-S	<input type="checkbox"/> Anti-Xg(a)
<input type="checkbox"/> Anti-c	<input type="checkbox"/> Anti-s	<input type="checkbox"/> Anti-M(g)
<input type="checkbox"/> Anti-e	<input type="checkbox"/> Anti-U	<input type="checkbox"/> Anti-Mi(a)
<input type="checkbox"/> Anti-f	<input type="checkbox"/> Anti-P1	<input type="checkbox"/> Anti-P
<input type="checkbox"/> Anti-G	<input type="checkbox"/> Anti-Le(a)	<input type="checkbox"/> Anti-A1
<input checked="" type="checkbox"/> Anti-K	<input type="checkbox"/> Anti-Le(b)	<input type="checkbox"/> Anti-H
<input type="checkbox"/> Anti-k	<input type="checkbox"/> Anti-Js(a)	<input type="checkbox"/> Anti-I
<input type="checkbox"/> Anti-Kp(a)	<input type="checkbox"/> Anti-Js(b)	<input type="checkbox"/> Anti-i
<input type="checkbox"/> Anti-Kp(b)	<input type="checkbox"/> Anti-Lu(a)	<input type="checkbox"/> Warm auto-antibody
<input type="checkbox"/> Anti-Jk(a)	<input type="checkbox"/> Anti-Lu(b)	<input type="checkbox"/> Cold auto-antibody
<input type="checkbox"/> Anti-Jk(b)	<input type="checkbox"/> Anti-Lu3	<input type="checkbox"/> Anti-Le(ab)
<input type="checkbox"/> Anti-Jk3	<input type="checkbox"/> Anti-C(w)	<input type="checkbox"/> Anti-Di(a)
<input type="checkbox"/> Anti-Fy(a)	<input type="checkbox"/> Anti-Ce	<input type="checkbox"/> Anti-Di(b)
<input type="checkbox"/> Anti-Fy(b)	<input type="checkbox"/> Anti-C(x)	<input type="checkbox"/> Anti-Do(a)
<input type="checkbox"/> Anti-M	<input type="checkbox"/> Anti-E(w)	<input type="checkbox"/> Anti-Do(b)

No antibody was identified.

Antibody ID Testing Comments

Comment:

Details:

Test Performed Off-Site  
 Location\*:  Cost\* (\$): 0.00

Test Performed On-Site  
 Workload Process\*: Antibody Identification panel, standa  
 Multiplier\*:  Add

Workload Process	Multiplier

\* Required Field

Order Reflex New SI or TR OK Cancel

## Patient Testing: Record a Transfusion Reaction Workup

The user documents a transfusion reaction workup (TRW).

### Assumptions

- A TRW is ordered and is “not started” or “in progress” on the Pending Task List (PTL).

### Outcome

- The user views or prints a preliminary TRW report of findings.
- Preliminary report findings are available for display in CPRS.

### Limitations and Restrictions

- Adverse reactions to transfusion detected at a later time (disease transmission) are not recorded through this option. Refer to Transfusion Complications Report.
- Each tab is completed and saved and is not editable. To change information on a previously completed tab, the TRW must be invalidated and corrected in its entirety.
- If comments are entered when a TRW is finalized as type "Other," they won't show on the CPRS Blood Bank Report.

### Additional Information

- All implicated units are added to data grids.
- Testing is based on local policies.
- VBECS allows the user to invalidate previously entered and saved information. When the user invalidates specimen data entered, VBECS invalidates the TRW.
- A delayed hemolytic transfusion reaction is typically identified about 7 to 10 days post-transfusion. It may be addressed in this option or as an antibody identification. (See Patient Testing: Enter Antibody Identification Results.)
- This test record is not editable after the user saves it to the database. When a correction needs to be filed, a corrected report must be generated, as described in Invalidate Test Results.
- See the printed chart copy for the medical director's signature and TRW interpretation, as described in Finalize/Print TRW.
- A user may order a TRW prior to the completion of a unit's transfusion record.
- The user must complete the unit's transfusion record prior to implicating a unit in a TRW.
- These data remain encapsulated as part of the TRW and are not saved, compared to the patient record, or associated with other pre-transfusion grid testing.
- TRW ABO/Rh typing results are not saved as historical ABO/Rh typing results. These test records are maintained as part of the TRW only.
- A system error occurs when a user attempts to access a locked TRW.


### User Roles with Access to This Option

All users

## Patient Testing: Record a Transfusion Reaction Workup

The user accepts the TRW order, inspects the sample for hemolysis, and performs implicated clerical checks. The TRW may require entry of patient data, including vital signs and serologic test results. A signed, printed TRW report must be filed with the patient's chart.

Transfusion reactions may be worked up in stages depending on patient symptoms and serologic findings. The first stage includes the initial clerical and specimen checks. The second stage includes serologic test interpretations, including pre- and post-patient testing, ABO/Rh, ABS, XM, and DAT.

User Action	VBECs
<p>1. Select <b>Patients</b> from the main menu.</p> <p>Select <b>Patient Testing</b>.</p>	<ul style="list-style-type: none"> <li>Displays options for processing patient-related functions.</li> <li>Displays the Pending Task List (PTL) in the Diagnostic Tests tab.</li> </ul> <p><b>NOTES</b> _____</p> <p>A user may search for specific tasks using search criteria detailed in Pending Task List.</p>
<p>2. Click a check box in the PTL to select a task.</p> <p>Click <b>OK</b>.</p>	<ul style="list-style-type: none"> <li>Displays TRW patient order information and allows the user to indicate which portion of the TRW to perform (initial clerical and specimen checks).</li> </ul> <p><b>NOTES</b> _____</p> <p>The TRW patient name and patient ID are visible at all times.</p> <p>Initial clerical and specimen checks must be performed before TRW serologic testing is available.</p>
<p>3. Enter the date the transfusion reaction was noted and the date it was investigated.</p>	<ul style="list-style-type: none"> <li>Allows the user to document observations about the pre- and post-transfusion specimens.</li> </ul>
<p>4. Click a radio button in the Specimen Checks tab to indicate whether pre- and post-transfusion specimens are available (Figure 131).</p> <p>Scan or enter specimen UIDs in the Specimen UID fields and enter comments, as required.</p> <p>Select an item from the drop-down menu in the Hemolysis Inspection field.</p>	<ul style="list-style-type: none"> <li>Allows the user to enter the date and time the transfusion reaction was noted (provided by the clinician) and the date and time of the investigation (default: current date and time).</li> </ul> <p><b>NOTES</b> _____</p> <p>When the pre- and/or post-transfusion samples are available to perform the clerical check, VBECs requires the user to indicate whether the clerical check is okay.</p> <p>VBECs allows the user to scan or enter the pre- and post-transfusion specimen UIDs. VBECs compares the patient name and ID associated with each specimen UID to those associated with the TRW workup.</p> <p> When the patient name does not match VBECs data for the TRW patient, VBECs emits an audible alert, warns the user, and instructs him to notify the physician immediately. VBECs allows the user to reenter the specimen UID or verify that the data are correct. VBECs requires a comment when there is no match and captures details for inclusion in an Exception Report (exception type: Transfusion Reaction Workup Specimen).</p> <p>The user may enter "icteric," "absent," "1+," "2+," "3+," or "4+" in the Hemolysis Inspection field to identify the presence or</p>

User Action	VBECS
	absence of hemolysis in the pre- and post-transfusion patient specimens or blood container, when available.
5. Click the <b>Clerical Check OK</b> or <b>Clerical Check Failed</b> radio button to perform initial clerical and identification checks (Figure 132).	
6. Click <b>OK</b> to save, and <b>Yes</b> to confirm the save.	<p><b>NOTES</b> _____</p> <p>When the user exits and saves the data, VBECS changes the task status of the TRW to “partially completed.”</p> <p>A partially completed TRW may be corrected only by invalidating and reentering the TRW.</p>
7. Enter the date and time of the transfusion reaction noted by the clinician and the date and time of the reaction investigation.	<p><b>NOTES</b> _____</p> <p>VBECS allows the user to enter the date and time of the reaction noted by the clinician.</p> <p>When the implicated unit is unknown, VBECS allows the user to continue without identifying a unit or entering implicated unit information.</p> <p>VBECS requires the user to enter the date and time of the TRW investigation (default: current date and time), including a date and time in the past for retrospective data entry. The user may not enter a future date and time.</p> <p>If the user indicates that implicated units are unknown, he continues at Step 10.</p>
<p>8. In the Implicated Units tab, click the <b>Implicated Unit(s) Identified</b> radio button to select an implicated unit.</p> <p>Scan or enter the unit ID and product code.</p> <p>Click the <b>Blood Bag Returned</b> or <b>Blood Bank Not Returned</b> radio button.</p> <p>Select an item from the drop-down menu in the Hemolysis Inspection field.</p> <p>Click the <b>Clerical Check OK</b> or <b>Clerical Check Failed</b> radio button to perform initial clerical and identification checks.</p> <p>Enter a comment and details, as required (Figure 133).</p>	<ul style="list-style-type: none"> <li>• Allows the user to: <ul style="list-style-type: none"> <li>○ Enter implicated unit information.</li> <li>○ Indicate whether the bag and infusion set were returned.</li> <li>○ Review and/or edit the entered information.</li> </ul> </li> </ul> <p><b>NOTES</b> _____</p> <p>When the user indicates that the blood bag was not returned, VBECS disables the hemolysis and clerical checks.</p> <p>All implicated units entered are available for TRW ABO/Rh and TRW crossmatching. Actual data entry of test results for any entries is optional, based on a facility’s local policies and procedures.</p> <p>The user may enter a free-text comment to document site-specific information such as other solutions or additives, urinary hemoglobin checks, and whether the product was warmed during transfusion.</p>

User Action	VBECS
<p>Click <b>Add</b> to add the unit.</p> <p>or</p> <p>Click the <b>No Implicated Units</b> radio button.</p> <p>Click <b>OK</b> to save, and <b>Yes</b> to confirm the save.</p>	
<p>9. Repeat Step 8 for each implicated unit.</p>	<ul style="list-style-type: none"> <li>Allows the user to enter the patient's symptoms.</li> </ul> <p><b>NOTES</b> _____</p> <p>VBECS accepts data for multiple implicated units and stores and reports them as part of the clerical check. When the user enters implicated unit data, VBECS adds the unit to TRW serologic testing.</p>
<p>10. In the Symptoms tab, click the <b>Patient Symptoms Identified</b> radio button.</p> <p>Click the check boxes to select one or more patient symptoms (Figure 134),</p> <p>or</p> <p>Click the <b>No Patient Symptoms Identified</b> radio button.</p> <p>Click <b>OK</b> to save, and <b>Yes</b> to confirm the save.</p>	<ul style="list-style-type: none"> <li>Allows the user to review and/or edit the entered data before it saves the record.</li> <li>Allows the user to indicate that the TRW is complete.</li> </ul>
<p>11. In the Serologic Tests tab, click a check box to select which tests to perform.</p> <p>Select a rack from the drop-down list in the Rack field.</p> <p>Click the <b>Using Automated Instrument</b> check box, as required.</p> <p>Enter the name of the instrument in the Name field.</p> <p>Click the <b>Instrument QC'd?</b> check box, as appropriate.</p> <p>Enter interpretations for TRW testing (Figure 135).</p>	<ul style="list-style-type: none"> <li>Allows the user to enter one or more additional TRW test interpretations: <ul style="list-style-type: none"> <li>TRW ABO/Rh testing</li> <li>TRW ABS testing</li> <li>TRW DAT testing</li> <li>TRW XM testing</li> </ul> </li> <li>Requires the user to enter the rack name and test method to be applied to the specimens selected in this testing group for serologic testing. The user must also indicate whether an automated instrument was used in testing.</li> <li>Displays the date and time the testing was performed (default: current date and time), which the user may edit retrospectively.</li> <li>Displays the testing technologist's identification (default: current user). Lists valid division users for the user's selection.</li> </ul> <p><b>NOTES</b> _____</p> <p>When no units were implicated or no pre- or post-transfusion specimens are available, VBECS disables those portions of the grid.</p> <p>VBECS displays the standard TRW tests for the user to select to perform some, none or all.</p>


User Action	VBECS
	<p>When a facility is “full service” and an automated instrument was used for testing, a user must enter the name of the instrument and may indicate that successful quality control testing was done for the testing date.</p> <p>For implicated units, the ABO/Rh TRW, ABS TRW, DAT TRW, and Crossmatch TRW testing data grids are pre-populated with pre- and post-specimen data. Data entry of test results is optional, based on a facility’s local policies and procedures.</p>
<p>12. Review the entries.</p> <p>Click <b>OK</b> to save, and <b>Yes</b> to confirm the save.</p>	<ul style="list-style-type: none"> <li>• Stores data for retrieval.</li> <li>• Makes the findings viewable through a CPRS query.</li> </ul> <p><b>NOTES</b> _____</p> <p>VBECS allows the user to indicate that the TRW is complete, then updates the TRW task status to “complete.”</p> <p>Once the TRW is completed, the summary of results and observations includes the comment, “Final report to follow in hard copy with medical director evaluation of the transfusion reaction workup.”</p> <p>VBECS indicates on the implicated unit’s patient transfusion record that it was implicated in a transfusion reaction and notes the patient’s name and ID and the date and time.</p> <p> VBECS saves the record and one or more clerical checks are marked “failed,” VBECS emits an audible alert, requires a comment, and captures details for inclusion in an Exception Report (exception type: Transfusion Reaction Workup).</p>
<p>13. Click <b>Finalize/Print TRW</b> and continue at Finalize/Print TRW to finalize the TRW (Figure 136), or</p> <p>Click <b>Cancel</b> to complete the TRW and exit.</p>	<p><b>NOTES</b> _____</p> <p>A completed TRW indicates that serologic testing was completed. A finalized TRW indicates that the medical director reviewed the TRW, classified the reaction, and may make recommendations for future transfusions.</p>

Figure 131: Transfusion Reaction Workup: Specimen Check

The screenshot shows the 'Specimen Checks' section of the VBECS Transfusion Reaction Workup interface. The patient information at the top includes Name: VBPAIENT, NINETY, Patient ID: 000-00-0090, and ABO/Rh: O Neg. The dates for the reaction are 02/23/2006 16:10. The 'Pre-Transfusion Specimen Availability\*' section has two radio buttons: 'Pre-Transfusion Specimen Available' (selected and circled in red) and 'Pre-Transfusion Specimen Not Available'. The 'Post-Transfusion Specimen Availability\*' section has two radio buttons: 'Post-Transfusion Specimen Available' (selected and circled in red) and 'Post-Transfusion Specimen Not Available'. Below these are fields for 'Specimen UID\*', 'Hemolysis Inspection\*', and 'Clerical Check\*' (with 'Clerical Check OK' and 'Clerical Check Failed' radio buttons). There are also 'Comment' and 'Details' text areas. At the bottom, there are buttons for 'Finalize/Print TRW', 'Invalidate TRW', and 'Cancel', along with an 'OK' button.

Figure 132: Transfusion Reaction Workup: Clerical Check

The screenshot shows the 'Specimen Checks' section of the VBECS Transfusion Reaction Workup interface, focusing on the 'Clerical Check' section. The patient information and dates are the same as in Figure 131. The 'Pre-Transfusion Specimen Availability\*' and 'Post-Transfusion Specimen Availability\*' radio buttons are now 'Pre-Transfusion Specimen Available' and 'Post-Transfusion Specimen Available' respectively. The 'Pre-Transfusion Specimen Checks' section has 'Specimen UID\*' set to 2960540153 and 'Hemolysis Inspection\*' set to 'Absent'. The 'Clerical Check\*' section has 'Clerical Check OK' selected and circled in red. The 'Post-Transfusion Specimen Checks' section has 'Specimen UID\*' set to 2960540156 and 'Hemolysis Inspection\*' set to 'Absent'. The 'Clerical Check\*' section has 'Clerical Check OK' selected and circled in red. There are also 'Comment' and 'Details' text areas. At the bottom, there are buttons for 'Finalize/Print TRW', 'Invalidate TRW', and 'Cancel', along with an 'OK' button.

**Figure 133: Transfusion Reaction Workup: Implicated Units**

VBECs - Transfusion Reaction Workup

Name: VBPATIENT, NINETY Patient ID: 000-00-0090 ABO/Rh: O Neg

Date Transfusion Reaction Noted: 02/23/2006 16:10 Date Transfusion Reaction Investigated: 02/23/2006 16:10

Specimen Checks: Implicated Units

Implicated Units?\*

Implicated Unit(s) Identified

No Implicated Units Identified

Implicated Unit Checks

Unit ID\* 6167616

Product\* 04210

Blood Bag?\*

Blood Bag Returned

Blood Bag Not Returned

Blood Bag Checks

Hemolysis Inspection\* Absent

Clerical Check\*  Clerical Check OK

Clerical Check Failed

Comment: [Empty]

Details: [Empty]

↓ Add Clear

Implicated Unit(s)

Unit ID	Product	ABO/Rh	Expires	Hemolysis

OK

\* Required Field

Finalize/Print TRW Invalidate TRW Cancel

**Figure 134: Transfusion Reaction Workup: Symptoms**

VBECs - Transfusion Reaction Workup

Name: VBPATIENT, NINETY Patient ID: 000-00-0090 ABO/Rh: O Neg

Date Transfusion Reaction Noted: 02/23/2006 16:10 Date Transfusion Reaction Investigated: 02/23/2006 16:10

Specimen Checks: Implicated Units Symptoms

Symptoms Identified?\*

Patient Symptoms Identified

No Patient Symptoms Identified

Patient Symptoms\*

Urticaria (Rash)

Pruritis (itching)

Wheezing, shortness of breath

Dyspnea (difficulty breathing)

Tachypnea (rapid respiration)

Hypoxemia

Chills (rigors)

Fever (>1C or 2F increase)

Flushing

Pain at infusion site

Pain in Flanks

Pain in Chest

Pain in Abdomen

Localized edema (angioedema)

Oozing or bleeding associated with consumptive coagul

Nausea, with or without vomiting

Darkened urine

Bleeding or other manifestations of consumption coagul

Anxiety

Circulatory shock

Other

Other Details: [Empty]

OK

\* Required Field

Finalize/Print TRW Invalidate TRW Cancel



Figure 135: Transfusion Reaction Workup: Serologic Tests

**VBECS - Transfusion Reaction Workup**

Name: VBPAIENT, NINETY Patient ID: 000-00-0090 ABO/Rh: O Neg

Date Transfusion Reaction Noted: 02/23/2006 16:10 Date Transfusion Reaction Investigated: 02/23/2006 16:10

Specimen Checks | Implicated Units | Symptoms | Serologic Tests

Date Tested\*: 02/23/2006 16:19  Using Automated Instrument Comment:   
 Tested By\*: Test User1 Name\*: SeroSerious X422 Details:   
 Rack\*: A Instrument QC'd?:

ABO/Rh

	ABO Interp.	Rh Interp.
Pre (2960540153)	O	Neg
Post (2960540156)	O	Neg
6167616	O	Neg

XM

	Pre Interp.	Post Interp.
6167616	Compatible - Ss	Compatible - Ss

ABS

	Interp.
Pre (2960540153)	Neg
Post (2960540156)	Neg

DAT

	Interp.
Pre (2960540153)	
Post (2960540156)	

Finalize/Print TRW Invalide TRW OK Cancel

\* Required Field

Figure 136: Transfusion Reaction Workup Report

**VBECS - Transfusion Reaction Workup Report**

LEAVENWORTH VAMC(589A6)  
 4101 S. Fourth St.  
 Leavenworth, KS 66048-5055

Date Printed: 02/23/2006 16:22  
 Printed By: Test User1

**Transfusion Reaction Workup Report**

Patient Name: VBPAIENT, NINETY Patient ID: 000-00-0090  
 Reaction Type: Undetermined

Specimen Checks	Specimen UID	Hemolysis	Checks OK?	Comments
Pre-Transfusion	2960540153	Absent	Yes	
Post-Transfusion	2960540156	Absent	Yes	

Reported Symptom:  
 Bleeding, shortness of breath  
 Flushing  
 Anxiety

Serologic Test Name	Specimen UID	Unit ID	Product	Interp.	Comments
Pre ABO Interp	2960540153			O	
Pre Rh Interp	2960540153			neg	
Pre Antibody Screen Interp	2960540153			neg	
Post ABO Interp	2960540156			O	
Post Rh Interp	2960540156			neg	

Current Page No: 1 Total Page No: 1 Zoom Factor: Page Width

Finalize Transfusion Reaction Workup Report

Schedule Print Print Now Close

\* Required Field

## Finalize/Print TRW

The user finalizes and views and/or prints the Transfusion Reaction Workup Report.

### Assumptions

- Patient Testing: Record a Transfusion Reaction Workup was completed.

### Outcome

- The Transfusion Reaction Workup Report is available for printing and signing.
- Final report findings are available for display in CPRS.

### Limitations and Restrictions

- The user cannot invalidate or change a finalized TRW. Contact the Service Desk if a TRW needs to be updated after finalization.
- Transfusion reaction reports are not available to clinicians through CPRS: they must be printed and filed with patient charts.
- VBECS displays these default values for transfusion reactions converted from VistA to VBECS. They do not reflect actual interpretations or results:

Converted Transfusion Reaction	Default
Pre-Transfusion Specimen Checks, Hemolysis Checks OK?	Yes
Post-Transfusion Specimen Checks, Hemolysis Checks OK?	Yes
Symptoms	No symptoms identified.
Implicated units	No units implicated in reaction.
Serologic tests	No serologic tests performed.

### Additional Information

- The medical director's signature is on the patient's printed chart.
- If a report needs to be replaced, the user may retrieve this report after it is complete. The final report is a printed document with the medical director's signature.
- Do not use the barcode scanner to enter a transfusion reaction specimen ID.
- To avoid a system error, once testing is entered, do not uncheck the testing box and recheck it.
- Limit Finalize TRW comments to <350 characters to allow saving and printing.
- A system error occurs if a user has a TRW open and then attempts to open a second instance of the same TRW. Restart the VBECS session and open one TRW to result and finalize.

### User Roles with Access to This Option

All users

## Finalize Transfusion Reaction Workup Report

➤➤➤ The Lead Technologist reviews the TRW details, identifies the type of reaction, and adds comments, according to the medical director's instructions. The user prints a copy for the medical director's signature and the Lead Technologist finalizes the TRW, generating a preliminary report. The final printed report requires the signature of the medical director.

User Action	VBECs
<p>1. Select <b>Reports</b> from the main menu.</p> <p>Select <b>Finalize/Print TRW</b>.</p> <p>Click the <b>Include Non-Finalized TRW Reports</b> check box (Figure 137).</p> <p>Select a completed TRW to finalize.</p> <p>Click <b>OK</b>.</p>	<ul style="list-style-type: none"> <li>Lists report names.</li> <li>Lists the completed TRWs for selection.</li> </ul> <p><b>NOTES</b> _____</p> <p>When the user enters and saves the portion recorded in Patient Testing: Record a Transfusion Reaction Workup, the TRW status is "completed." When the Transfusion Reaction Workup Report is complete, the TRW status is "finalized."</p>
<p>2. Click the <b>Finalize Transfusion Reaction Workup Report</b> check box.</p>	<ul style="list-style-type: none"> <li>Prepares the completed report for review.</li> <li>Displays the name of the blood bank medical director to sign the final report.</li> <li>VBECs recreates the final report for viewing or printing in the future.</li> <li>Displays the selected Transfusion Reaction Workup Report data.</li> </ul>
<p>3. Select the Transfusion Reaction Type and complete the Transfusion Reaction Comments required fields (Figure 138).</p>	<p><b>Requires the user to select a transfusion reaction type listed in</b></p> <ul style="list-style-type: none"> <li>Table 12: Transfusion Reaction Types.</li> <li>Allows the user to enter a TRW comment (maximum 1,000 characters).</li> <li>Allows the user to save the data entered.</li> </ul> <p><b>NOTES</b> _____</p> <p>The user must indicate one transfusion reaction type. Entry of a comment is optional for all types except O (other), which requires details to be entered.</p>
<p>4. Click <b>OK</b> and <b>Yes</b> to save.</p>	<ul style="list-style-type: none"> <li>Updates the transfusion reaction to "finalized" in the database and prints a report for the medical director's signature.</li> </ul> <p><b>NOTES</b> _____</p> <p>VBECs notifies the user when DSS updates cannot be sent because VistALink is not active.</p> <p>VBECs will attempt to resend the DSS information when VistALink is available after any user logs into the VBECs application's VistALink connection.</p>
<p>5. Select <b>Schedule Print</b> or <b>Print Now</b> and follow the instructions to print.</p>	<ul style="list-style-type: none"> <li>Prints the report on the selected printer.</li> </ul>
<p>6. Click <b>Close</b> to exit.</p>	

Figure 137: Select Transfusion Reaction Workup

Patient Name	Patient ID	Pre-Specimen UID	Post-Specimen UID	Date Completed
VBPAIENT, SEVENTYTWO	000-00-0072	2960450031	2960450036	02/15/2006 14:59

Figure 138: Finalize TRW Report

Medical Director: Dr. Vbprovider One

Transfusion Reaction Type

- Acute Hemolysis (ABO Incompatibility)
- Acute Hemolysis (non ABO)
- Anaphylaxis
- Citrate Toxicity
- Delayed Hemolytic
- Febrile Nonhemolytic
- Graft vs. Host disease
- Post-Transfusion: Associated Disease
- Transfusion Associated Circulatory Overload (TACO)
- Transfusion Related Acute Lung Injury (TRALI)
- Transfusion Related Microbial Infection
- Unrelated to Transfusion
- Urticaria
- Other

Transfusion Reaction Comments

Comment: [Dropdown]

Details: [Text Area]

## View/Print a Finalized Transfusion Reaction Workup Report

The user views and/or prints the finalized Transfusion Reaction Workup Report.

User Action	VBECS
<p>1. Select <b>Reports</b> from the main menu.</p> <p>Select <b>Finalize/Print TRW</b>.</p> <p>Select a completed TRW (Figure 136).</p>	<ul style="list-style-type: none"> <li>• Lists report names.</li> <li>• Displays the selected Transfusion Reaction Workup Report data.</li> <li>• Displays an option to view and/or print a finalized TRW by selecting a patient.</li> </ul> <p><b>NOTES</b> _____</p> <p>The user finalizing the report may not edit previously entered TRW data.</p>
<p>2. Select a patient.</p>	<ul style="list-style-type: none"> <li>• Compiles the report and displays an option to print the report or exit.</li> </ul> <p><b>NOTES</b> _____</p> <p>Details included in the finalized TRW report:</p> <ul style="list-style-type: none"> <li>• Patient name</li> <li>• Patient ID</li> <li>• Specimen UID</li> <li>• Reported symptoms</li> <li>• Specimen clerical checks (pre-, post-, and unit)</li> <li>• Serologic testing (if performed, interpretations only)</li> <li>• Transfusion reaction type</li> <li>• Comment</li> <li>• Implicated units (include from Patient Testing: Record a Transfusion Reaction Workup)</li> <li>• TRW report completed by:               <ul style="list-style-type: none"> <li>○ Date and time the report was completed</li> <li>○ Signature line for the medical director</li> <li>○ Line for the date and time the medical director completed the review</li> <li>○ Finalized by</li> <li>○ Finalized date and time</li> </ul> </li> </ul>
<p>3. Select <b>Schedule Print</b> or <b>Print Now</b> and follow the instructions to print.</p>	<ul style="list-style-type: none"> <li>• Prints the report on the selected printer.</li> </ul>
<p>4. Click <b>Close</b> to exit.</p>	

**Table 12: Transfusion Reaction Types**

<b>Transfusion Reaction Type Text</b>
Anaphylaxis
Citrate Toxicity
Delayed Hemolytic
Febrile Nonhemolytic
Graft vs. Host Disease
Acute Hemolysis (non ABO)
Acute Hemolysis (ABO Incompatibility)
Transfusion Associated Circulatory Overload (TACO)
Transfusion Related Acute Lung Injury (TRALI)
Transfusion Related Microbial Infection
Post-Transfusion: Associated Disease
Unrelated to Transfusion
Urticaria
Other: free-text data required on new entries

## **Maintain Patient Records**

### **Special Instructions & Transfusion Requirements: Enter a Transfusion Requirement**

The user enters or inactivates the Transfusion Requirements (TRs) in the patient's record by division.

#### **Assumptions**

- The patient exists in the VBECS database.

#### **Outcome**

- VBECS displays TRs to all divisions within a multidivisional database. The division in which they were entered maintains and controls them.
- VBECS maintains a historic record of changes to the patient's TRs.

#### **Limitations and Restrictions**

- Inactivation is defined and restricted by security level.
- VistA Special Instructions (SIs) from database conversion do not become TRs: they remain SIs. SIs are information only and are not enforced unless entered in the Component Requirements tab in Special Instructions & Transfusion Requirements.

#### **Additional Information**

- No current patient specimen or order is required to access this option.
- When TRs are not met, VBECS prohibits the printing of the Caution Tag and Blood Transfusion Record Form (BTRF) and prevents issue of the blood unit.
- User-entered antigen negative requirements are persistent. VBECS enforces them even when they are in conflict with the division's antibody table settings.
- Patient antibody identifications from VistA are included in the database conversion.
- By default, inactivation of an antibody removes requirements for antigen negative blood derived from the antibody entry.

#### **User Roles with Access to This Option**

All users

### **Special Instructions & Transfusion Requirements: Enter a Transfusion Requirement**

The user assigns patient blood component requirements or indicates the need for antigen negative blood to be applied during blood component selection and issue.


VBECS displays TR data and applies requirements during blood unit selection.

All items in these TR categories indicate the need for pre-transfusion processing of a unit before the unit may be issued to the patient. All users may activate entries. >>>> Only a Traditional Supervisor may inactivate a component requirement for a patient. A patient may have one or more TRs:

- Pre-treated to Prevent GVHD
- Leuko-reduce cellular products\*

- Washed RBC products
- Washed PLT products
- Sickle Cell Negative RBC products
- CMV negative cellular products\*

\*All product types in OTHER are evaluated for the requirement including those that are non-cellular.

User Action	VBECS
<p>1. Select <b>Patients</b> from the main menu.</p> <p>Select <b>Special Instructions &amp; Transfusion Requirements</b>, or</p> <p>Click  in the Patient Information Toolbar, when it is active.</p>	<ul style="list-style-type: none"> <li>• Displays options for processing patient-related functions.</li> <li>• Displays the option to select a patient, if one is not already selected.</li> </ul> <p><b>NOTES</b> _____</p> <p>The user may also click <b>New SI or TR</b> in Patient Testing to access this option.</p>
<p>2. Select the <b>Antigen Negative RBCs, Component Requirements</b>, or <b>Antibodies Identified</b> tab (Figure 139).</p>	<ul style="list-style-type: none"> <li>• Displays the patient's TRs.</li> <li>• Allows the user to add or inactivate information based on his security level.</li> </ul> <p><b>NOTES</b> _____</p> <p>When a user accesses antigen negative requirements, VBECS displays and enforces the antigen negative requirement for the patient, regardless of the division setting for the antibody.</p> <p>When a user accesses the Antibodies Identified tab, VBECS compares the patient's antibodies with the division's antibody table setting. When the antibody is set to require antigen negative blood at issue, VBECS displays and enforces the antigen negative requirement.</p>
<p>3. Select a TR from the pick list, or enter an antigen or antibody, as the tab allows.</p> <p>Click <b>Add</b> to move the selection to the patient's record.</p> <p>Click <b>OK</b> when all TRs are selected to save them to the patient's record.</p> <p>Click <b>Yes</b> to confirm changes and exit.</p>	<ul style="list-style-type: none"> <li>• Allows multiple selections from the Component Requirements, Antibodies Identified, and Antigen Negative tabs.</li> </ul> <p><b>NOTES</b> _____</p> <p>VBECS applies the TR to component classes, as indicated.</p> <p>The user may not add duplicate SIs or TRs in the same session, but may duplicate them in another session.</p> <p>The division code and date of entry are associated with the entered component or antigen negative requirement, or antibody specificity so that other divisions will to know whom to contact for further information, when needed. The original entries may have been made through Patient Testing (associated with an antibody identification) or TRs from any division within a multidivisional database.</p>
<p>4. Select the TR to be inactivated (Figure 140).</p> <p>Select and/or enter an explanatory comment and click <b>OK</b>.</p>	<ul style="list-style-type: none"> <li>• Allows the user to enter a comment.</li> </ul> <p><b>NOTES</b> _____</p> <p>When a Transfusion Requirement is inactivated in any division in a multidivisional configuration, a comment (free text or from a canned comment with the context: Transfusion Requirement</p>



User Action	VBECS
	<p>Inactivation) must be added that will be saved and printed on the audit trail of originating division alerting the originating division of the change to the patient's TR setting if they were not consulted regarding that change.</p> <p>Note: It is best practice that inactivating a TR in the non-originating division involves a consultation with the originating division and allowing the originating to make the change with any locally required documentation.</p> <p>For inactivation of an item:</p> <ul style="list-style-type: none"> <li>• &gt;&gt;&gt;&gt; A Traditional Supervisor from any division may inactivate TRs and persistent antigen negative requirements in a multidivisional database regardless of the source of the TR.</li> <li>• When an antibody specificity is inactivated, VBECS inactivates the corresponding calculated (VBECS-generated) antigen negative requirement.</li> <li>• For antibodies identified, VBECS displays a summary of antibodies currently associated with the patient and identifies their source. Additional antibodies may be added and previously identified antibodies may be inactivated.</li> </ul>
5. Return to Step 2 to continue editing the patient's TRs.	
6. Accept entries and exit, or  Return to Step 1 to make entries for a different patient.	

Figure 139: Enter Patient Special Instructions and Transfusion Requirements

VBECs - Enter Patient Special Instructions and Transfusion Requirements

Enter Special Instructions

Name VBPAIENT, EIGHTYC Patient ID 000-00-0081 ABO/Rh O Neg

Special Instructions | Antigen Negative RBCs | Component Requirements | Antibodies Identified

New Special Instruction

Special Instruction\* Split units for transfusion today

Add

Existing Special Instructions


Active	Special Instruction	Creation Date, User
<input checked="" type="checkbox"/>	Split units for transfusion today	02/07/2006 17:57 by vhaishvbecs1

Show Inactive Special Instructions

\* Required Field

OK Cancel

**Figure 140: Inactivate Component Requirements**

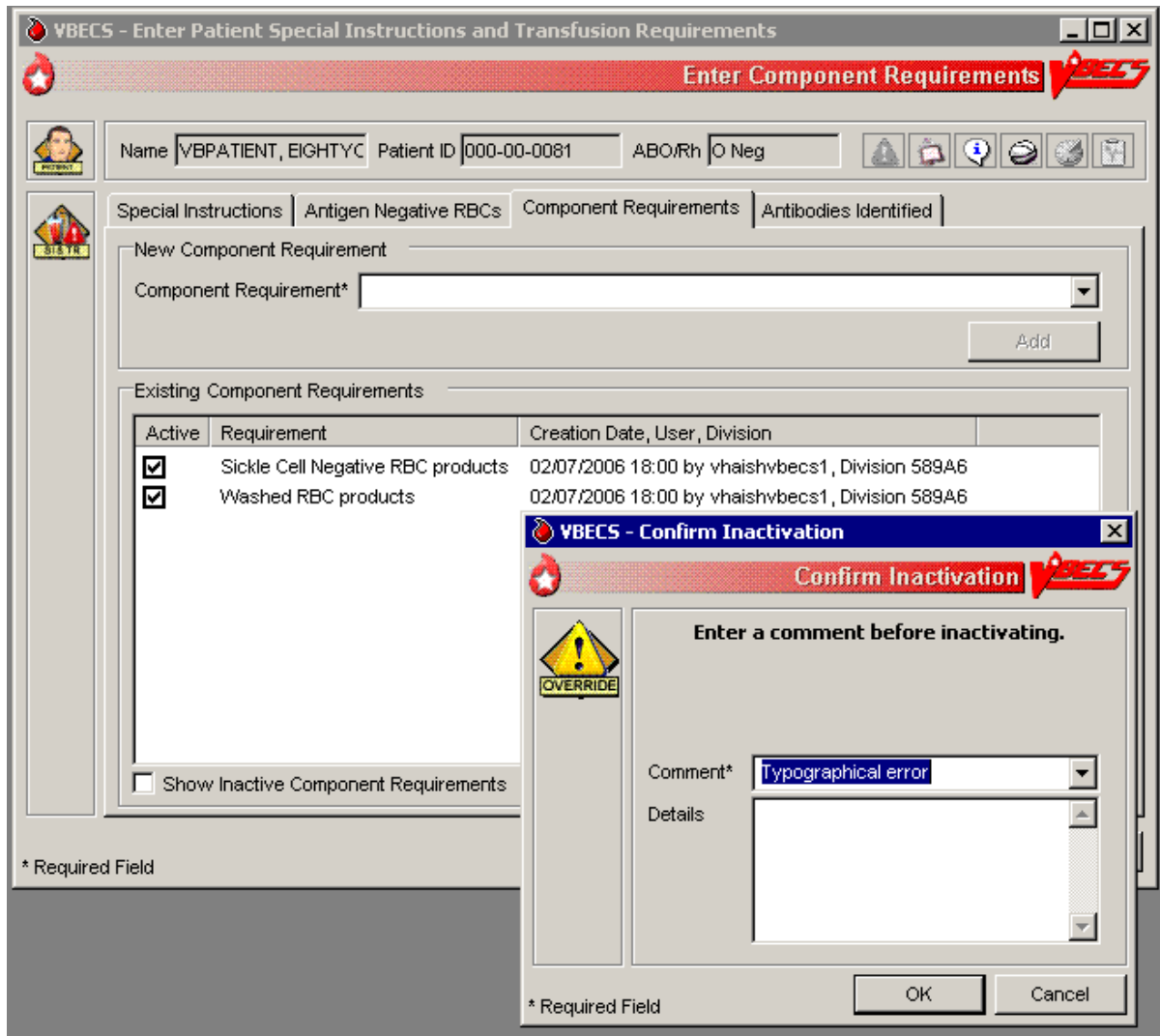


## Special Instructions & Transfusion Requirements: Enter and Remove Special Instructions

The user enters comments in a patient's file that are not enforced by VBECS.

### Assumptions

- Fields containing Special Instructions (SIs) may be populated during database conversion.

### Outcome

- When the user searches for a patient or when the Patient Information Toolbar is displayed, VBECS displays  to alert the user to view the patient's SI entries.
- When a user retrieves a patient record and is in a multidivisional setting, he may view SIs created in all divisions within the database.

### Limitations and Restrictions

- VBECS does not enforce Transfusion Requirements (TRs) or antigen negative requirements entered through this option.
- SIs moved to VBECS during the database conversion may contain information that must be entered as TRs in VBECS. Inactivate the old SI and create a TR and new updated SI: the database conversion software cannot automatically translate the SI from database conversion into a rule-based TR.
- VBECS will truncate SI text from the VistA database conversion.

### Additional Information

- Inactivation of patient SI entries is allowed only in the division of entry.
- Only currently active SIs at a division will automatically display via the database to all divisions.
- Inactive SIs are not deleted from the VBECS database. They may not be reactivated but may be viewed in the maintenance fields when requested.
- Canned comments in the Special Instructions context must be entered by the division before they can be selected for use.
- Previously saved Special Instructions that require editing due to context or spelling errors must be inactivated and reentered in total.
- Database conversion entries are labeled as such.
- SIs from database conversion may include component or antigen negative requirements that must be entered in TRs so that VBECS can enforce them.

### User Roles with Access to This Option

All users

## Special Instructions & Transfusion Requirements: Enter a Special Instruction

A user may add to and inactivate SI comments in a patient's blood bank record. Users in all divisions in a multidivisional database may view these comments.


User Action	VBECS
<p>1. Select <b>Patients</b> from the main menu.</p> <p>Select <b>Special Instructions &amp; Transfusion Requirements</b>, or</p> <p>Click  in the Patient Information Toolbar, when displayed.</p>	<ul style="list-style-type: none"> <li>• Displays options for processing patient-related functions.</li> <li>• Displays the option to select a patient, if one is not already selected.</li> </ul> <p><b>NOTES</b> _____</p> <p>The user may also click <b>New SI or TR</b> in Patient Testing to access this option.</p>
<p>2. Select a patient.</p> <p>Click <b>OK</b>.</p>	<ul style="list-style-type: none"> <li>• Displays options to select a patient.</li> <li>• Displays the option to enter a new comment or remove an existing comment in the division.</li> <li>• Displays the active SI entries in reverse chronological order with all active SIs first, along with date, division code, and text of the entry.</li> </ul> <p><b>NOTES</b> _____</p> <p>The need for SIs is usually temporary and may not be required for all transfusions.</p> <p>The user may choose to display inactivated SI comments.</p>
<p>3. Select the <b>Special Instructions</b> tab (Figure 141).</p> <p>Click the <b>Show inactive special instructions</b> check box to display inactivated SI entries.</p>	<ul style="list-style-type: none"> <li>• Allows the user to enter or inactivate comments.</li> <li>• Displays inactive Special Instructions but does not allow the user to reactivate them.</li> </ul> <p><b>NOTES</b> _____</p> <p>A user may add or inactivate a patient-specific comment at any time in the division in which it was entered.</p> <p>➤➤➤➤ A Traditional Supervisor may inactivate SIs from database conversion.</p>
<p>4. Confirm the activation of the entry and enter a comment.</p> <p>Click <b>OK</b>.</p> <p>Return to Step 2 to add more SIs for this patient.</p> <p>Select another patient and return to Step 1, if desired.</p> <p>Exit.</p>	<ul style="list-style-type: none"> <li>• Saves the entry in the database.</li> </ul>

Figure 141: Enter Patient Special Instructions and Transfusion Requirements

The screenshot shows the 'VBECS - Enter Patient Special Instructions and Transfusion Requirements' window. At the top, there is a title bar and a red banner with the text 'Enter Patient Special Instructions and Transfusion Requirements'. Below this, there are input fields for 'Name' (VBPATIENT, EIGHTYONE), 'Patient ID' (000-00-0081), and 'ABO/Rh' (O Neg). A toolbar with various icons is located to the right of these fields. Below the input fields, there are four tabs: 'Special Instructions', 'Antigen Negative RBCs', 'Component Requirements', and 'Antibodies Identified'. The 'Special Instructions' tab is active. Under this tab, there is a section for 'New Special Instruction' with a text input field containing 'Check the patient's blood counts before selecting units, use often exceeds original order' and an 'Add' button. Below this is a section for 'Existing Special Instructions' which contains a table with the following headers: 'Active', 'Special Instruction', 'Creation Date, User', 'Inactivation Date, User', and 'Inactivation Comment'. The table is currently empty. At the bottom left of the window, there is a checkbox labeled 'Show Inactive Special Instructions' which is unchecked. At the bottom right, there are 'OK' and 'Cancel' buttons. A legend at the bottom left indicates '\* Required Field'.

Name: VBPATIENT, EIGHTYONE    Patient ID: 000-00-0081    ABO/Rh: O Neg

Special Instructions | Antigen Negative RBCs | Component Requirements | Antibodies Identified

New Special Instruction

Special Instruction\*  Add

Existing Special Instructions

Active	Special Instruction	Creation Date, User	Inactivation Date, User	Inactivation Comment
--------	---------------------	---------------------	-------------------------	----------------------

Show Inactive Special Instructions

\* Required Field    OK    Cancel

## ***Post-Transfusion Information***

The user enters post-transfusion details from the Blood Transfusion Record Form (BTRF) or VBECS saves post-transfusion details received from the Transfusion Verification System.

### **Assumptions**

- The BTRF is returned and completed for each unit.
- The unit status is “issued” or “presumed transfused” in VBECS.
- Interface connections to the BCE COTS system are enabled to receive transfusion update messages.

### **Outcome**

- The unit status is updated to “transfused.”
- The VBECS patient record includes the blood transfusion information.

### **Limitations and Restrictions**

- VBECS cannot update a unit record to presumed transfused when a user has that unit record open (locked) in Enter Post Transfusion Information. VBECS will attempt to update the unit the next time the presumed transfused background routing runs if the unit remains in an issued status.
- If post-transfusion information is not transmitted to VBECS by the BCE COTS interface, the user will need to enter that information to update the record from presumed transfused to transfused.
- The transfusion comments and volume transfused entered in the Transfusion Verification application as an update to a completed transfusion are not sent to VBECS.
- Transfusion reaction symptoms are not provided to VBECS by the Transfusion Verification application unless a transfusion reaction has also been indicated for that event.

### **Additional Information**


- A user with the appropriate security level may enter or edit bedside verification information unless it is received from BCE COTS interface.
- The user may indicate that a transfusion reaction and symptoms were noted on the BTRF as free text. Noting a transfusion reaction does not create an order for a transfusion reaction workup (TRW).
- Remove a “transfused” status through Remove Final Status unless that unit was transfused by the Transfusion Verification application. See limitations in Remove Final Status.

### **User Roles with Access to This Option**

All users

## Post-Transfusion Information

The user verifies the date and time transfused with a returned BTRF, adds other transfusion details, initiates a system check of the bedside verification information, enters that information, and indicates whether a reaction occurred during or immediately after the transfusion, as documented on the BTRF. VBECS does not require entry of post-transfusion information: once a day, without user input, VBECS moves units 48 hours after issue to “presumed transfused” status.

User Action	VBECS
<p>1. Select <b>Patients</b> from the main menu.</p> <p>Select <b>Post-Transfusion Information</b>.</p>	<ul style="list-style-type: none"> <li>• Displays options for processing patient-related functions.</li> <li>• Displays an option to scan the Caution Tag or enter a patient name or ID.</li> </ul>
<p>2. Select a patient and click <b>OK</b>.</p>	<ul style="list-style-type: none"> <li>• Lists the units issued to the selected patient.</li> <li>• Displays the issue date and time.</li> <li>• Displays the units one at a time for update in the order in which the user selected them.</li> </ul> <p><b>NOTES</b> _____</p> <p>➤➤➤➤ VBECS displays an option to obtain a list of transfused (those not received by BCE COTS) units for the patient when the user is a Traditional Supervisor or above.</p> <p>When post-transfusion information is not entered into VBECS within 48 hours from the date/time the unit was recorded as “issued”, VBECS will change the unit status to “transfused” with a presumed indicator after the execution of a scheduled background job. The unit will then be considered transfused by the system regarding report generation. If post-transfusion information is entered later, the presumed indicator is deactivated.</p>
<p>3. Click one or more check boxes to select units (Figure 142).</p> <p>Click <b>OK</b>.</p>	<ul style="list-style-type: none"> <li>• Allows the user to select units for which to enter post-transfusion information.</li> </ul> <p><b>NOTES</b> _____</p> <p> VBECS does not list units (for selection) with a final status. When a user enters a unit with a final status, VBECS alerts the user and clears the screen. The user may select another unit.</p>
<p>4. Enter transfusion details for a blood unit.</p> <p>Verify the unit selection.</p>	<ul style="list-style-type: none"> <li>• Displays the unit information entered by the bedside verification check and allows the user to enter to edit the information.</li> <li>• Displays an option to enter or edit unit data.</li> </ul> <p><b>NOTES</b> _____</p> <p>➤➤➤➤ A user with a security setting of Traditional Supervisor may edit user entered bedside verification, transfusion reaction, or post-transfusion detail information.</p> <p>VBECS displays an option to enter information for each unit. This information is available for patient record retrieval.</p>



User Action	VBECS
	<p>To prevent rejection of transfusion information exchanged between VBECS and the Transfusion Verification system, a variance of +/- five (5) minutes is permitted for time of issue and transfusion start and end times. The variance allows the two systems to communicate even when their internal time is not precisely synced to the minute.</p>
<p>5. Select one or more units. Enter or edit the bedside verification check information with the proper security level.</p>	<ul style="list-style-type: none"> <li>• Displays the entries for review, including the comment, until all details are entered.</li> <li>• Displays an option to save the data and exit.</li> </ul> <p><b>NOTES</b> _____</p> <p>Upon data entry and saving of the post-transfusion information, VBECS assigns a status of “transfused” for the unit of blood assigned to the indicated patient.</p>
<p>6. Enter post-transfusion details, including:</p> <ul style="list-style-type: none"> <li>• Transfusion start and end dates and times</li> <li>• Transfusionist ID</li> <li>• Location of transfusion</li> <li>• Whether the transfusion was completed or interrupted</li> <li>• If interrupted, the amount transfused</li> <li>• Whether there was a transfusion reaction</li> <li>• If there was a reaction, symptoms (Figure 143)</li> </ul> <p>Click <b>OK</b> to save.</p>	<ul style="list-style-type: none"> <li>• Displays an option to enter unit data.</li> <li>• Warns the user or allows the user to continue to the next selected unit.</li> </ul> <p><b>NOTES</b> _____</p> <p>When a user indicates that a transfusion reaction occurred, VBECS does not automatically order a TRW.</p> <p>VBECS notifies the user when the transfusion end date and time is earlier than the start or issue time. The user may correct the entry.</p> <p>When the unit was not issued to a remote storage location, VBECS compares the date and time the unit was issued to the start transfusion time entered.</p> <p>When the difference is greater than 30 minutes, VBECS notifies the user that this is a delayed start time and allows the user to edit the transfusion start date and time. <b>No</b> allows the user to continue. <b>Yes</b> allows the user to edit the date and time the transfusion ended. When the user edits the date and time transfused, VBECS repeats this process until the user stops editing and chooses to continue.</p> <p>When the unit was not issued to a remote storage location, VBECS compares the date and time recorded as the transfusion end time to the start date, calculates the transfusion time in minutes, and compares this period to the maximum transfusion time for the component class. When the calculated transfusion time for this unit exceeds the defined maximum transfusion time for the component class, VBECS notifies the user that this is a prolonged transfusion and allows the user to edit the transfusion end date and time. <b>No</b> allows the user to continue. <b>Yes</b> allows the user to edit the date and time the transfusion ended. When the user edits the date and time transfused, VBECS repeats this process until the user stops editing and chooses to continue.</p>
<p>7. Respond to warnings or continue to enter data for the next unit.</p>	<ul style="list-style-type: none"> <li>• Allows the user to repeat for subsequent selected units until all details are entered.</li> <li>• Displays an option to save the data and exit.</li> </ul>

User Action	VBECS NOTES
8. Exit.	<p>VBECS saves all transfusion information for future retrieval and reporting as part of the patient and unit historic records.</p> <p>VBECS notifies the user when DSS updates cannot be sent because VistALink is not active.</p> <p>VBECS will attempt to resend the DSS information when VistALink is available after any user logs into the VBECS application's VistALink connection</p>

**Figure 142: Post-Transfusion Information**

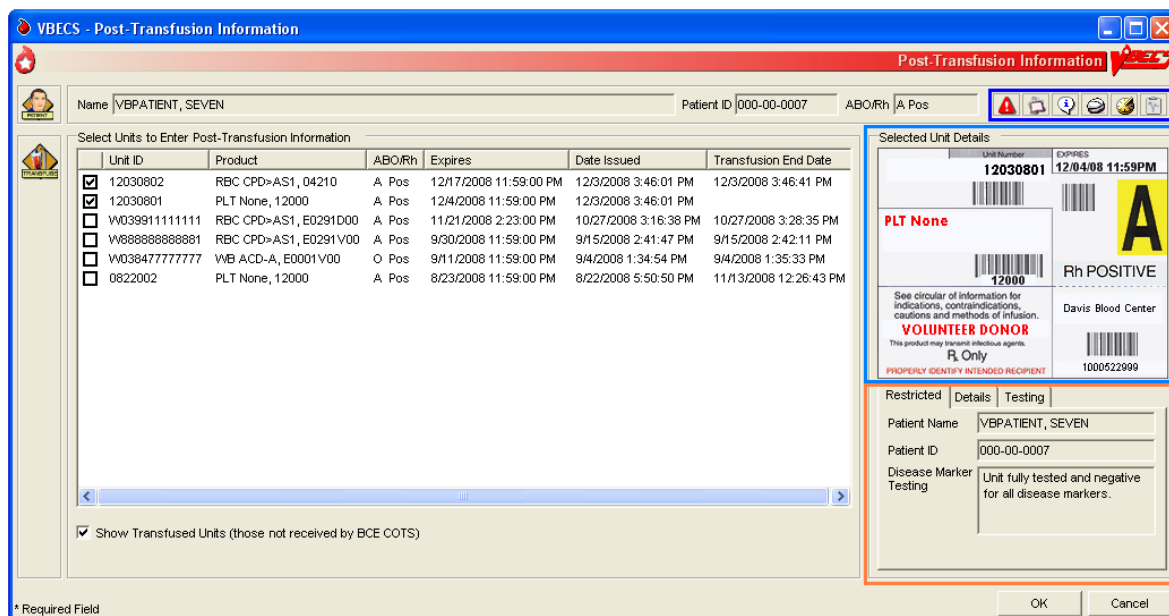


Figure 143: Post-Transfusion Information Details

**VBECS - Post-Transfusion Information**

**Post-Transfusion Information**

Name: VBPATIENT, EIGHTYONE    Patient ID: 000-00-0081    ABO/Rh: O Neg

6873183, RBC AS1

**Issue Details**

Date Issued: 02/09/2006 10:45  
Issued By: vhaishvbecs1

**Bedside Verification Check**

Verification Date:

Unit ID: 6873183  
Product Code: 04210V

Patient Name: VBPATIENT, EIGHTYONE  
Patient ID: 000-00-0081

Transfusion Start Date: 02/09/2006 10:45  
Transfusionist ID #1: Cupid  
Transfusionist ID #2:

**Post-Transfusion Information**

Transfusion End Date: 02/13/2006 16:22  
Patient Location at Transfusion: A2MED IM-LE

Transfusion Completed?\*

Transfusion Completed  
 Transfusion Interrupted

Amount Transfused\*  1/4  1/2  3/4  
Volume:  mL

Transfusion Reaction Noted?\*

No Reaction  
 Reaction Noted

Symptoms: goofy smile

Comments:

OK    Cancel

\* Required Field

## Automated Entry of Post-Transfusion Details from BCE COTS Application

VBECS receives and saves the details of a unit transfusion episode as recorded in the BCE COTS system.

User Action	VBECS
<p>1. BCE COTS system user completes a transfusion for a patient which triggers a HL7 message containing transfusion information to VBECS for a blood unit.</p>	<ul style="list-style-type: none"> <li>• VBECS rejects a BCE COTS HL7 message when data elements are not acceptable per the data entry rules for manual entry.</li> </ul> <p><b>NOTES</b> _____</p> <p>Units that are not updated by a BCE COTS message will be marked presumed transfused 48 hours after issue by a VBECS background process.</p> <p>Transfusion information is available in the Unit History Report.</p>
<p>2. VBECS updates the patient transfusion record with the details received in the successfully acknowledged HL7 message.</p>	<ul style="list-style-type: none"> <li>• VBECS generates an exception report entry: Electronic Transfusion Record Update when a BCE COTS update to a previously saved manually or electronically entered transfusion record is received.</li> <li>• When an update to a previously saved transfusion record is made by BCE COTS or a user, VBECS will display the original entry and the updated details in the Patient History and Unit History Reports.</li> </ul> <p><b>NOTES</b> _____</p> <p>The BCE COTS system delivers these patient and unit specific transfusion data elements to VBECS for update</p> <ul style="list-style-type: none"> <li>• Bedside verification date and time</li> <li>• Transfusion verification manual Unit ID entry reason</li> <li>• Transfusion Start date and time</li> <li>• Transfusionist ID #1</li> <li>• Transfusionist ID #2 (may be blank if so configured in BCE COTS application)</li> <li>• Transfusion End Date and time</li> <li>• Patient location at Transfusion</li> <li>• Transfusion interrupted indicator</li> <li>• Transfused volume, (ml not in fractions by the BCE COTS)</li> <li>• Transfusion Reaction noted</li> <li>• Transfusion Reactions symptoms entered, if any</li> <li>• Transfusion comments, if any</li> </ul>

## Document ABO Incompatible Transfusions

The user enters post-transfusion details from the Blood Transfusion Record Form (BTRF) to document ABO incompatible and other inadvertent transfusions.

### Assumptions

- A unit was transfused to a patient other than the intended recipient.
- The connection to VistA is active.

### Outcome

- The unit status is updated to “transfused.”
- The patient record includes the blood transfusion information.

### Limitations and Restrictions

- VBECS does not enforce compatibility checks between the patient and the unit.
- VBECS does not update CPRS component order status when the Document ABO Incompatible Transfusion option is used.

### Additional Information


- FDA guidance govern when to file blood product deviation reports.
- Follow local policy and procedures for additional reporting requirements.
- VBECS requires a specimen and an order placed through CPRS to process transfusions. When a patient was transfused and no specimen was collected or order placed for blood components, the user may record the transfusion using this option.

### User Roles with Access to This Option



- Traditional Supervisor

## Document ABO Incompatible Transfusions

When a unit is transfused to the incorrect patient, the user enters transfusion details for that unit.

 *VBECS does not enforce compatibility checks to allow consistent and accurate online availability of a unit's final disposition. Use of this option is restricted to supervisory blood bank personnel who have thoroughly investigated the transfusion and documented the inability to use normal VBECS business processes to enter the transfusion documentation.*

User Action	VBECS
1. Select <b>Supervisor</b> from the main menu.  Select <b>Document ABO Incompatible Transfusion</b> .	<ul style="list-style-type: none"><li>• Displays options for processing patient-related functions.</li><li>• Displays an option to scan the Caution Tag or enter a patient name or ID.</li></ul>

User Action	VBECS
2. Select a patient and click <b>OK</b> .	<p>Displays an option to select a patient that may not be available from the VBECS files.</p> <p><b>NOTES</b> _____</p> <p>Displays a message when the user selects a patient with a blank first or last name, or the name is too long. The user is instructed to contact the VBECS Administrator for assistance.</p>
3. Select a unit to document a transfusion that does not follow system rules.	<ul style="list-style-type: none"> <li>• Displays an option to enter a unit ID and product code or search for one.</li> </ul> <p><b>NOTES</b> _____</p> <p>VBECS presents all units in the system for selection when the search button is used. This search will require time to complete as the number of units to display increases. VBECS does not enforce compatibility checks in this option.</p> <p> When a user enters a unit with a final status, VBECS alerts the user and clears the screen. The user may select another unit.</p>
4. Enter post-transfusion details and a comment.	<ul style="list-style-type: none"> <li>• Displays an option to enter or edit unit data. See Post-Transfusion Information.</li> <li>•  VBECS emits an audible alert, requires a comment, and captures the transaction details for inclusion in an Exception Report (exception type: ABO incompatible unit transfused).</li> <li>• Displays the entries for review, including the comment, until all details are entered.</li> </ul>
5. Click <b>OK</b> and <b>Yes</b> to save.	<ul style="list-style-type: none"> <li>• Saves the data.</li> <li>• Allows the user to enter comments and details.</li> </ul>
6. Enter or select a comment and details, in accordance with local policy and procedures.  Click <b>OK</b> .	<p><b>NOTES</b> _____</p> <p>VBECS saves transfusion information for future retrieval and reporting as part of the patient and unit historic records.</p> <p>Upon data entry and saving of the post-transfusion information, VBECS assigns a status of “transfused” to the unit of blood for the indicated patient.</p> <p>VBECS captures details of this override in an Exception Report.</p>
7. Exit.	

## Justify ABO/Rh Change

The user changes a patient's historical blood type only when there is a clinical change in blood type, not to correct an error. (See Invalidate Test Results.)

### Assumptions

- The patient has a previous ABO/Rh on record from the user's current division, from a different division within the database, or from the database conversion.
- The patient has a current specimen test result with an ABO/Rh interpretation that does not match the historical record. Both previous and current specimen test results are valid.

### Outcome

- VBECS permanently marks the patient record and the specimen test to indicate that the patient has a justified ABO/Rh change, regardless of the user's division.
- The justified ABO/Rh on the patient record forces the user's selection to be used as the new historical record, replacing the current test result as the historic record. VBECS does not check for ABO/Rh discrepancies with tests entered for this patient prior to the justified test result.
- VBECS permanently disallows electronic crossmatch (eXM) for this patient.

### Limitations and Restrictions

- VBECS does not allow the selection or issue of certain blood components that may appear ABO/Rh compatible when discrepant ABO/Rh results are not justified.

### Additional Information

- When the discrepant ABO/Rh tests are on the same specimen, the user may not justify the patient's ABO/Rh.
- Do not use this option for ABO/Rh discrepancies that occur because the patient had a recent transfusion of a component of a type other than his own.
- ABO/Rh discrepancies that occur because the previously tested or current specimen was tested in error are handled by inactivating the previously recorded test results through Invalidate Test Results.
- Subsequent specimens may need to be justified in the presence of continued discrepant ABO results.
- This option is used in rare clinical situations such as following bone marrow transplant.

### User Roles with Access to This Option

- Traditional Supervisor

## Justify ABO/Rh Change

The user determines that the patient's current ABO/Rh testing results are valid, although they differ from previously recorded results, and that the patient requires a justification indicator on his record.

This option is used only when both the previous specimen testing and the current specimen testing are considered valid. There is a valid explanation for why the ABO/Rh interpretations do not match and invalidation of the test result is not appropriate. Once the current patient ABO/Rh result change is justified, VBECS marks the patient's record so that special handling for component orders for the patient apply from this point forward and remain in effect until the patient record is inactivated by the recording of his death in VBECS or by removing the justification.

User Action	VBECS
1. Select <b>Supervisor</b> from the main menu.  Select <b>Justify ABO/Rh Change</b> .	<ul style="list-style-type: none"> <li>• Displays options available to supervisors.</li> </ul>
2. Enter a patient name in the Patient field and click <b>Search</b> to select a patient.  Select a patient.  Click <b>OK</b> .	<ul style="list-style-type: none"> <li>• Displays a screen for patient selection.</li> <li>• Disallows the justification or allows the user to continue.</li> </ul> <p><b>NOTES</b> _____</p> <p>VBECS compares the results of the current ABO/Rh with the previous ABO/Rh results. When there is no discrepancy, VBECS warns that no justification is required. There is no override. VBECS clears the screen and the user may select another patient.</p> <p>VBECS warns the user to use this option only when both the current and previous specimen tests are valid, otherwise, use Invalidate Patient Test Results.</p> <p>When initially displayed, the patient name field may not display the expanded field. Maximize the window to display the expanded (full name) field for long patient names.</p>
3. Select an ABO/Rh interpretation as the patient's justified historical record (Figure 144).	<ul style="list-style-type: none"> <li>• Displays details of the historic and current ABO/Rh testing for comparison and a list of available ABO and Rh types for user selection.</li> <li>• Requires a comment to continue.</li> </ul> <p><b>NOTES</b> _____</p> <p>The user may select one ABO group and one Rh type for the patient to serve as the justified ABO/Rh typing in the historic record. VBECS displays the current ABO and Rh interpretations as the defaults.</p> <p>VBECS requires a comment to continue.</p>
4. Confirm the update of the database with the justified ABO/Rh.  Exit.	<ul style="list-style-type: none"> <li>• Updates the patient record.</li> </ul>



Figure 144: Justify ABO/Rh Change

**VBECS - Justify ABO/Rh Change**

**Justify ABO/Rh Change**

Name: VBPATIENT, NINETYE Patient ID: 000-00-0098 ABO/Rh: O Neg

**Historical ABO/Rh**

Date Tested: 02/24/2006 10:08  
Division Tested: LEAVENWORTH VAMC  
ABO: O  
Rh: Neg  
ABO Comment:  
Rh Comment:

**Current Specimen ABO/Rh**

Date Tested: 02/24/2006 10:10  
Division Tested: LEAVENWORTH VAMC  
ABO: A  
Rh: Neg  
ABO Comment:  
Rh Comment:

**Justified ABO/Rh Change**

**WARNING: Use this option only when both the current and previous specimen tests are valid.**

Justified ABO\*: A  
Justified Rh\*: Neg

Comments:  
Comment\*  
Details

\* Required Field

OK Cancel

This page intentionally left blank.

# Automated Testing

## Automated Testing Interface

The user sends test results to VBECS from an automated instrument.

For details about interface configuration please refer to the *Vista Blood Establishment Computer Software (VBECS) 2.3.1 Technical Manual-Security Guide*.

### Assumptions

- Test Results have been reviewed and released on the instrument to be transmitted to VBECS.
- A patient testing order must be accepted in VBECS and pending test results.
- A blood unit must be received in the VBECS division.
- The testing instrument is fully functional and available for use: The instrument QC is completed and valid, no maintenance is required and test results are available for transmission to VBECS.
- Interface Control Parameters have been completed and are enabled for the automated instrument interface in the VBECS Administrator.
- Data Innovations Instrument Manager message interface is properly and completely configured.


### Outcome

- Test results are received by VBECS.
- Task status is updated for tests pending completion.

### Limitations and Restrictions

- VBECS has only been tested with one automated instrument connected.
- Not available for transmission to Transfusion only facilities from a blood center due to constraints of VA's security software.
- The results of Antibody Panels (ABID) will not be accepted by the VBECS Automated Interface.
- The results of QC information from automated instruments will not be accepted by the VBECS Automated Interface.
- Time zone will not be included in HL7 message date time from the broadcasting application (Instrument Manager). The time zone is determined from the division code contained in the message header (MSH) segment of the message.

### Additional Information

- VBECS will accommodate antigen typing test results where antigen typing tests are available in VBECS.
-  AABB Standards (5.16.1) requires a serological XM to detect ABO incompatibility. A local policy must be in place if the site is not performing a IS AHG as part of their serologic crossmatch test, manual or using an instrument.

## User Roles with Access to This Option

All users

### Automated Testing Interface

This feature allows the interface of automated blood banking instruments with VBECS through Data Innovations Instrument Manager. It eliminates the potential for human error associated with the manual entry of results from an automated instrument into VBECS. It also eliminates the potential for transcription errors that can lead to a patient safety issue of ABO incompatible transfusion. In addition to safety, these enhancements will also reduce the time spent entering results manually.

User Action	VBECS
<p>1. User sends test results from the Automated Instrument to VBECS.</p>	<ul style="list-style-type: none"> <li>• Updates the task status on the Pending Task List (PTL) upon successful receipt of testing.</li> <li>• Stores received results for review, processing and acceptance by a user.</li> </ul> <p><b>NOTES</b> _____</p> <p>If the blood unit record is not found, the message is rejected and results are not available for review.</p> <p>If the patient specimen is not associated with an active patient order, the message is rejected and results are not available for review.</p> <p>If the cross-matched units are not assigned to a patient order, the message containing crossmatch results is rejected and results are not available for review.</p> <p>Patient specimen blood test results pending review must be rejected by the user in VBECS before replacement results can be received.</p> <p>If test results currently exist for a patient order (entered manually or by the Automated Instrument), the replacement test results will not be accepted into VBECS until the user invalidates the existing test results.</p> <p>When patient test results are sent to VBECS and multiple orders exist for the same test type and the same specimen UID on the VBECS PTL, the message is rejected. For example: When ABS test results and Repeat ABS test results for the same specimen UID are sent to VBECS at the same time, because the Automated Instrument does not indicate which test is primary and which test is reflex, the message must be rejected.</p> <p>When the testing instrument for the patient ABO/Rh message contains results for Anti-D1 and Anti-D2 which do not conflict (both are positive or both are negative), the Anti-D1 result will be stored in VBECS.</p> <p>When the testing instrument for the patient ABO/Rh message contains conflicting results for Anti-D1 and Anti-D2 (one is positive and the other negative), the message is rejected and results are not available for review.</p>

User Action	VBECS
	<p>The message is processed by the VBECS system. If the message is accepted (per the approved HL7 message profile specification), a success reply is sent; otherwise, a rejection reply is sent to the broadcasting application and results are not available for review.</p> <p>The system associates interpretation with test results received from an Automated Instrument as follows:</p> <ul style="list-style-type: none"> <li>• 1 is a Positive</li> <li>• 2 is a Positive</li> <li>• 3 is a Positive</li> <li>• 4 is a Positive</li> <li>• 0 is a Negative</li> </ul> <p>See Table 13: HL7 Tests Accepted by VBECS.</p> <p>VBECS stores the following information:</p> <ul style="list-style-type: none"> <li>• Test Results</li> <li>• Test Interpretations</li> <li>• Testing Tech ID</li> <li>• Automated Instrument identifier</li> <li>• Date and Time Tested</li> <li>• Automated Instrument Test Comments</li> </ul>
<p>2. The user logs into VBECS to review and approve messages.</p> <p>Refer to Automated Testing Review for Blood Units or Automated Testing Review for Patients.</p>	

**Table 13: HL7 Tests Accepted by VBECS**

The first result received is considered the primary test. Once the primary test is accepted into VBECS, a second message with the same testing result will fill the repeat test.

Row ID	Tests
1.	Patient Type and Screen (TAS)
2.	Patient ABO/Rh
3.	Repeat Patient ABO/Rh
4.	Patient ABS
5.	Repeat Patient ABS
6.	Patient AHG DAT
7.	Repeat Patient AHG DAT
8.	Crossmatch
9.	Repeat Crossmatch
10.	Patient IgG DAT AHG
11.	Repeat Patient IgG DAT AHG
12.	Patient C3 DAT AHG
13.	Repeat Patient C3 DAT AHG
14.	Patient Antigen Typing
15.	Repeat Patient Antigen Typing
16.	Unit ABO/Rh
17.	Unit Antigen Typing

## **Automated Testing Review for Blood Units**

Allows the user to review and approve testing results completed on an automated instrument and transmitted to VBECS.


### **Assumptions**

- Division is defined as full service.
- The testing instrument is fully functional and available for use. The instrument QC is completed and valid, no maintenance is required. No QC information is transmitted to VBECS in the HL7 message.
- Message is received successfully as defined in Automated Testing Interface.
- The blood unit records are in the VBECS database for ABO/Rh confirmation or Unit Antigen Typing.
- Each transmitted test must include its results and interpretation.

### **Outcome**

- The test record will capture the details of the verified testing. Completed, current unit and specimen test results will be available for the life of the system for inclusion in various reports generated to document patient and unit testing.
- Unit(s) will have ABO/Rh and antigen test results recorded in the database.
- The system updates the workload reporting in VBECS.

### **Limitations and Restrictions**

- VBECS does not accept inconclusive ABO/Rh test results.
- Pooled Product Types labeled with an Rh type of “Pooled” is considered an Rh Positive unit which does not require confirmation testing.
- Pooled Product Types that require confirmation, labeled with an ABO group of “Pooled” cannot be confirmed. A blood component unit received as “POOLED ABO” will have to be transfused using the Supervisory option, Document ABO Incompatible Transfusion.
-  Users can select and assign multiple units with the same DIN and different product codes for crossmatch. Users can only process one DIN (e.g., W123400345345) on the Automated Instrument at a time. It is incumbent on the user to verify that the crossmatch result from the Automated Instrument matches the product code (e.g., E0134V00) associated with the patient in VBECS. To decrease the risk of a mislabeled unit at issue, local practice may want to have only one on the units selected in VBECS at a time when crossmatch testing is being performed on the automated instrument.
- When a user chooses to reject a transmitted test, a comment is required. The comment is stored but is not available for review on a report at this time. It is recommended that the test is reviewed in the instrument and not sent to VBECS as a result for rejection.

### **Additional Information**

- When Workload is not configured per Workload Codes, then Workload cannot be collected.

### **User Roles with Access to This Option**

All users

## Automated Testing Review

This feature allows the interface of automated blood banking instruments with VBECS through Data Innovations Instrument Manager. It eliminates the potential for human error associated with the manual entry of results from an automated instrument into VBECS. It also eliminates the potential for transcription errors that can lead to a patient safety issue of ABO incompatible transfusion. In addition to safety, these enhancements will also reduce the time spent entering results manually.


User Action	VBECS
<p>1. User logs into VBECS and selects <b>Blood Units, Automated Testing</b> from the main menu.</p> <p>Or</p> <p>User logs into VBECS and clicks on the <b>Automated Testing Toolbar Icon</b> (blood drop with lightning bolt).</p>	<ul style="list-style-type: none"> <li>• Displays the Automated Testing – Pending Blood Units window.</li> <li>• Displays message when no pending blood units were found to review.</li> </ul> <p><b>NOTES</b> _____</p> <p>The functionality is not accessible if division is defined as a Transfusion Only facility.</p> <p>Only data within the user's division can be accessed for any type of processing.</p>
<p>2. User selects from the testing tabs that are enabled on the Automated Testing – Pending Blood Units window (Figure 145).</p> <p><b>Unit ABO-Rh tab</b></p> <p><b>Unit Antigen Typing tab.</b></p>	<ul style="list-style-type: none"> <li>• Determines that all appropriate test results were entered for the blood unit. Row validation and test validation are completed.</li> </ul> <p><b>NOTES</b> _____</p> <p>The unit ID and product code are used to identify a specific unit from inventory. The pick list of Units for ABO/Rh or Antigen Typing will only display the product codes of units in an Active status and per the typing requirements.</p> <p>When a weak D antigen typing test is received on a blood unit that is Rh positive, the test is marked as in error (E) the system provides a warning message:</p> <p>“Cannot perform weak D testing on Rh positive unit.”</p> <p>The test cannot be saved in VBECS and must be rejected.</p> <p>The list view of test results on the Blood Unit windows is sortable by unit ID or date/time of test completion. Default sort order is on unit ID.</p> <p>When testing is received for an inactivated unit or unit in final status, the system provides a warning message: “Cannot perform testing on an inactivated unit or a unit in final status.” The test cannot be saved in VBECS and must be rejected.</p>
<p>3. User selects a unit for review by clicking the check box next to a unit.</p>	<ul style="list-style-type: none"> <li>• Allows selection of blood unit test results.</li> </ul> <p><b>NOTES</b> _____</p> <p>Selection of multiple tests with a Valid indicator is permitted.</p> <p>Tests with a Warning or Error indicator can only be selected and processed one at a time (no batch selection).</p>



User Action	VBECS
	<p>If the unit id transferred from the instrument does not uniquely identify the unit In VBECS, then the system will prompt the user to select the correct product code for the unit available in VBECS.</p> <p>Selection of blood products is based on product codes that require ABO/Rh confirmation.</p>
<p>4. User <b>Accepts</b> or <b>Rejects</b> the selected unit test results.</p>	<ul style="list-style-type: none"> <li>• Processes any applicable overrides.</li> </ul> <p><b>NOTES</b> _____</p> <p>When the test is rejected, the system will:</p> <ul style="list-style-type: none"> <li>• Prompt user to confirm they want to reject.</li> <li>• Prompt user to enter a comment reason for rejection.</li> <li>• Remove the test from the list for acceptance.</li> <li>• No rejection message is sent to the instrument.</li> <li>• The Diagnostic test returns to the previous status and ready for testing.</li> </ul> <p>When an interpretation of “Inconclusive” is verified for either the ABO or Rh interpretation, the unit is given an indicator of “Quarantine” and the details of the testing are captured for inclusion in an exception report. The system provides an audible alert.</p> <p>The system provides an audible alert and a warning message requiring override is generated if the ABO and/or Rh confirmation interpretation does not match the ABO and/or Rh of the unit at initial login. Upon user confirmation of the discrepancy (override), inventory units whose ABO/Rh confirmation does not match the ABO/Rh at login are given an indicator of “Quarantine” and the details of the transaction are captured for inclusion in an exception report.</p> <p>The system displays all data input during the session and accommodates editing prior to the user electing to update the database. The system will warn the user that if he chooses to cancel using the cancel or “X” button or by leaving/refreshing the current page, the system will not save any data changes entered during the session.</p> <p>The unit has had a quarantine indicator assigned. The system will release any patient assignment. Any patient restriction associated with the unit remains intact (e.g., Directed or Autologous).</p> <p>The system compares the antigen typing interpretation entered by the technologist to any previously assigned antigen typing for the same antigen (Incoming Shipment, Edit Unit Information, or antigen typing test other than weak D). If the current interpretation does not match the previously identified antigen type, the system displays a warning message and quarantines the unit: Current antigen type does not match previously</p>

User Action	VBECS
	<p>recorded antigen type (login, update, or test). This must be resolved before this unit can be used. Unit has been quarantined.</p> <p>When a weak D interpretation is positive or inconclusive and is verified for a tested blood unit that was labeled as (Rh) D negative, display a warning message and add the quarantine indicator to the unit: "Unit is weak D positive or inconclusive: unit will be quarantined." No user action is required.</p> <p>When a unit was issued and a conflicting test result found that indicates a potential unit incompatibility with the patient, such as a patient ABO/Rh discrepancy, or the unit is antigen positive where antigen negative is required on the tested specimen, VBECS displays a warning message such as: Unit has been issued. Notify the physician of this potential problem IMMEDIATELY, per hospital policy! The warning message displays during the execution of the various tests related to the compatibility of a blood unit. An exception report entry is generated at the save.</p>
<p>5. User clicks <b>Yes</b> to acknowledge acceptance or rejection of test results.</p>	<ul style="list-style-type: none"> <li>• Saves user input.</li> </ul> <p><b>NOTES</b> _____</p> <p>Upon successful completion of ABO/Rh confirmation and/or Antigen Typing completion, the unit status is updated. Successful completion means that the ABO and Rh (if required) interpretation is consistent with recorded observed results and matches the ABO/Rh at unit login and any repeat ABO/Rh tests on the unit that are not invalidated</p> <p>When the test is rejected, the system will:</p> <ul style="list-style-type: none"> <li>• Prompt user to confirm they want to reject.</li> <li>• Remove the test from the list for acceptance.</li> <li>• No rejection message is sent to the instrument.</li> <li>• The Diagnostic test returns to the previous status and ready for testing.</li> </ul>

**Figure 145: Automated Testing – Pending Blood Units Window**

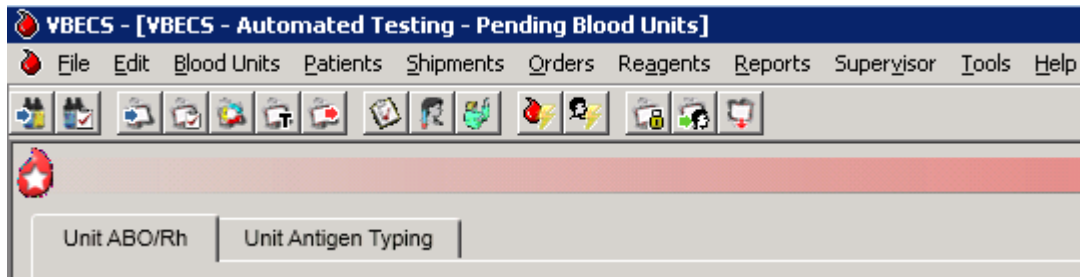

## ***Automated Testing Review for Patients***

Allows the user to review and approve testing results completed on an automated instrument and transmitted to VBECS.

### **Assumptions**

- Division is defined as full service.
- The testing instrument is fully functional and available for use. The instrument QC is completed and valid, no maintenance is required. No QC information is transmitted to VBECS in the HL7 message.
- Message is received successfully as defined in Automated Testing Interface.
- VBECS database contains an active patient order for diagnostic tests or blood components including reflex tests that have been accepted and appear on the Pending Task List.
- Blood Units are selected for a patient for serologic crossmatch testing prior to result transmission from an automated instrument.
- Each transmitted test must include its results and interpretation.


### **Outcome**

- The test record will capture the details of the verified testing. Completed, current unit and specimen test results will be available for the life of the system for inclusion in various reports generated to document patient and unit testing.
- The system updates the task status, the workload reporting in VBECS and the order status in VBECS and VistA.

### **Limitations and Restrictions**

- VBECS does not accept inconclusive ABO/Rh test results.
- VBECS does not accept inconclusive interpretations for antibody screen tests.
- Immediate spin crossmatch is not allowed via automated test entry.
- Patient Antigen typing applies only to patient specimens that contain red blood cells.
- The system does not check the specificity of a patient's antibody against an existing patient antigen typing.
- The patient antigen typing result is not reported to the CPRS record.
- Order Rejection comment entry is required but is not visible on a report.

### **Additional Information**

-  AABB Standards (5.16.1) requires a serological XM to detect ABO incompatibility. A local policy must be in place if the site is not performing a IS AHG as part of their serologic crossmatch test, manual or using an instrument.
- The patient's antigen typing information will display in Display Patient VBECS Data, but will not appear as part of the Transfusion Requirements or Special Instructions that display in Display Patient Transfusion Requirements and Special Instructions.
- When the Antibody Screen test is configured with a Direct Antiglobulin Test (DAT) for manual testing, that DAT test must be executed as a separate test when using an automated instrument for testing and displays on the DAT tab, not the TAS tab. Attempts to send the DAT to VBECS without a separate

standalone DAT order (either a DAT ordered in CPRS or a Reflex DAT entered in VBECS) will result in a rejection of the DAT result message. Users must have a separate DAT test pending in order to send test results from the Automated Instrument.

- When Workload is not configured per Workload Codes, then Workload cannot be collected.
- Review comment entered for ABO/Rh test results from an automated instrument is associated with both ABO and Rh interpretation and will be displayed for both on reports.

## User Roles with Access to This Option

All users

## Automated Testing Review

This feature allows the interface of automated blood banking instruments with VBECS through Data Innovations Instrument Manager. It eliminates the potential for human error associated with the manual entry of results from an automated instrument into VBECS. It also eliminates the potential for transcription errors that can lead to a patient safety issue of ABO incompatible transfusion. In addition to safety, these enhancements will also reduce the time spent entering results manually.

User Action	VBECS
<p>1. User logs into VBECS and selects <b>Patients, Automated Testing</b> from the main menu.</p> <p>Or</p> <p>User logs into VBECS and clicks on the <b>Automated Testing Toolbar Icon</b> (patient face with lightning bolt)</p>	<ul style="list-style-type: none"> <li>• Displays the Automated Testing – Pending Patient Orders window.</li> <li>• Allows a user to select a patient.</li> <li>• Displays message when no pending patient specimens were found to review.</li> </ul> <p><b>NOTES</b> _____</p> <p>The functionality is not accessible if division is defined as a Transfusion Only facility.</p> <p>The list displayed contains only those patients associated with pending results from an instrument message.</p> <p>System displays a list of patients who have pending test results from Auto Instrument including the following information:</p> <ul style="list-style-type: none"> <li>• Patient name</li> <li>• Patient ID</li> <li>• Patient Historic ABO/Rh</li> <li>• Specimen UIDs</li> <li>• List of pending auto instrument tests for patient</li> </ul>
<p>2. User selects the patient to review from the list of pending results and clicks <b>OK</b>.</p>	<ul style="list-style-type: none"> <li>• Displays information about the patient selected.</li> </ul> <p><b>NOTES</b> _____</p> <p>The list displayed contains only those patients associated with pending results from an instrument message.</p> <p>System displays a list of patients who have pending test results from Auto Instrument including the following information:</p> <ul style="list-style-type: none"> <li>• Patient name</li> <li>• Patient ID</li> <li>• Patient Historic ABO/Rh</li> <li>• Specimen UIDs</li> <li>• List of pending auto instrument tests for patient</li> </ul>


User Action	VBECS
	<p>When patient is selected the system displays the following information:</p> <ul style="list-style-type: none"> <li>• Patient name</li> <li>• Patient ID</li> <li>• Patient DOB</li> <li>• Patient Historic ABO/Rh</li> <li>• Patient Sex</li> <li>• Patient location</li> </ul>
<p>3. User selects from the testing tabs that are enabled on the Automated Testing – Pending Specimen Tests window (Figure 146).</p> <p><b>TAS</b> tab</p> <p><b>ABO/Rh</b> tab</p> <p><b>ABS</b> tab.</p> <p><b>DAT</b> tab.</p> <p><b>Antigen Typing</b> tab</p> <p>See Step 7 for process when the User selects <b>XMatch</b> tab.</p>	<ul style="list-style-type: none"> <li>• Displays the first tab from the left side of the screen that has test results to review.</li> <li>• Determines that all appropriate test results were entered for the patient.</li> <li>• Validates test results and displays Warning or Error indicator when results do not match expected pattern.</li> <li>• Displays comments from the Automated Instrument.</li> </ul> <p><b>NOTES</b> _____</p> <p>The system provides a visual and audible indication to the user when there are patient Special Instructions or Transfusion Requirements to be viewed. When the user has unsaved changes and attempts to close the Automated Testing window, the application will warn the user that the changes will not be saved. Clicking <b>Yes</b> closes the form; clicking <b>No</b> returns the user to the Automated Testing – Pending Specimen Tests window.</p>
<p>4. User clicks on the checkbox next to the result to be accepted or rejected.</p> <p>Clicks on the comment balloon to enter <b>Review Comments</b> as needed.</p>	<ul style="list-style-type: none"> <li>• Presents appropriate warning messages to the user.</li> <li>• Allows user selection of multiple tests with a Valid indicator.</li> <li>• Restricts the processing of tests with a Warning or Error indicator to one at a time.</li> </ul> <p><b>NOTES</b> _____</p> <p>When the user selects an expired task for processing, the system will provide an audible alert and generate a warning message: "This order is expired. Do you want to continue processing it?" When the user responds "No", the task is cleared and a new task can be entered. When the user responds "Yes", he must enter a comment and the system will generate an exception report entry.</p> <p>When row validation for a patient ABO or Rh test results matches the instrument interpretation and does not match the previous ABO or Rh interpretation on record, the system emits an audible alert and displays a warning message that the current ABO/Rh does not match previous interpretations. The user is required to enter a comment (free text or canned from the "patient testing" context) to continue. The comment and the details of the transaction are captured for inclusion in an exception report.</p>

User Action	VBECS
	<p>The system checks if a repeat ABO/Rh was performed on the patient specimen. When performed, the repeat ABO/Rh interpretation must match the original ABO/Rh interpretation. If they do not match, the system will provide an audible warning and display a warning message: "Original and repeat ABO/Rh interpretation does not match. You must resolve the discrepancy before this specimen may be used." Continue to save?" A "No" response clears the ABO/Rh testing results and interpretation from the screen. A "Yes" response requires a comment (free text or canned from the "patient testing" context) and the details of the transaction are captured for inclusion in an exception report.</p> <p>If the antibody screen is positive, the system emits an audible alert and warning that displays that the Patient has a POSITIVE Antibody Screen Result. Perform Antibody Identification on this patient prior to issue of blood components.</p>
<p>5. User <b>Accepts</b> or <b>Rejects</b> the selected patient test results.</p>	<ul style="list-style-type: none"> <li>• Verifies the results for save.</li> <li>• Gathers comments from user for rejected results prior to saving.</li> </ul> <p><b>NOTES</b> _____</p> <p>When the test is accepted, the system will save the following:</p> <ul style="list-style-type: none"> <li>• Reviewing Tech ID (Name)</li> <li>• Testing Tech ID (Name)</li> <li>• Interfaced instrument id (Name) as transmitted from instrument,</li> <li>• Date and time tested</li> <li>• Date and time reviewed</li> <li>• Test information: test results, test interpretations, and test comments.</li> </ul> <p>When the test is rejected, the system will:</p> <ul style="list-style-type: none"> <li>• Prompt user to confirm they want to reject.</li> <li>• Remove the test from the list for acceptance.</li> <li>• No rejection message is sent to the instrument.</li> <li>• The Diagnostic test returns to the previous status and is ready for testing.</li> </ul> <p>Changes test Task status Completed for all tests except for Type and Screen which must have ABO/Rh and ABS saved before the TAS goes to a task status of Completed.</p> <p>If the user saves only the ABO/Rh and rejects ABS or only saves the ABS and rejects the ABO-Rh, the TAS goes to a task status of Partially Completed and is then selectable from the Pending Task List.</p> <p>The system will update the order status when a valid interpretation is entered and saved for a test, the status of the ordered test is updated to "completed," and an order status update message is sent to CPRS.</p> <p>When a unit was issued and a conflicting test result found that indicates a potential unit incompatibility with the patient, such as a patient ABO/Rh discrepancy, the unit crossmatch is</p>

User Action	VBECS
	<p>incompatible with the patient, the patient antibody screen test is positive, the patient DAT is positive, or the unit is antigen positive where antigen negative is required on the tested specimen, VBECS displays a warning message such as: Unit has been emergency issued. Notify the physician of this potential problem IMMEDIATELY, per hospital policy! The warning message displays during the execution of the various tests related to the compatibility of a blood unit. An exception report entry is generated at the save.</p> <p>A comment appears on CPRS reports for patients with a positive Antibody Screen or Direct Antiglobulin Test: Preparation of red cell components for transfusion may be delayed due to serologic problems. Contact transfusion service for information on potential clinical significance and availability of blood components.</p>
<p>6. User clicks <b>Yes</b> to confirm the acceptance or rejection of the selected test(s).</p>	<ul style="list-style-type: none"> <li>• Closes the Automated Testing – Pending Specimen Tests window and displays the desktop when there are no more pending tests to review.</li> <li>• Closes the Automated Testing – Pending Specimen Tests window and displays the Automated Testing – Pending Patient Orders window when there are pending tests to review.</li> </ul>
<p><b>CROSSMATCH</b></p> <p>7. User selects <b>XMatch</b> from the testing tabs that are enabled on the Automated Testing – Pending Specimen Tests window.</p>	<ul style="list-style-type: none"> <li>• Determines that all appropriate test results were entered for the patient.</li> <li>• Validates test results and displays Warning or Error indicator when results do not match expected pattern.</li> <li>• Displays comments from the Automated Instrument.</li> </ul> <p><b>NOTES</b> _____</p> <p>Units for crossmatch must be selected for the patient prior to results transmission from the Automated Instrument for the results to appear in the Automated Testing – Pending Specimen Tests window on the XMatch tab for review.</p>
<p>8. User clicks on the checkbox next to the Unit crossmatch result to be accepted or rejected.</p> <p>Clicks on the comment balloon to enter <b>Review Comments</b> as needed.</p>	<ul style="list-style-type: none"> <li>• Presents appropriate warning messages to the user.</li> <li>• Requires the user to select the product code for the unit crossmatched when there are more than one unit selected for the patient with the same unit ID.</li> <li>• Allows selection of multiple tests with a Valid indicator.</li> <li>• Restricts the processing of tests with a Warning or Error indicator to one at a time.</li> </ul> <p><b>NOTES</b> _____</p> <p>When the user selects an expired task for processing, the system will provide an audible alert and generate a warning message: "This order is expired. Do you want to continue processing it?" When the user responds "No", the task is cleared and a new task can be entered. When the user responds "Yes", he must enter a comment and the system will generate an exception report entry.</p>
<p>9. User can change the XMatch Interp by using the drop-down arrow to select another interpretation.</p>	<ul style="list-style-type: none"> <li>• Displays the crossmatch results from the Automated Instrument and allows the user to edit that interpretation before saving.</li> </ul> <p><b>NOTES</b> _____</p>

User Action	VBECS
	<p>The system allows a user to change a final crossmatch interpretation of <b>Crossmatch Compatible</b> (C) to <i>Crossmatch Compatible: don't transfuse</i> (D) prior to saving. The system re-evaluates changes made to the test interpretation and allows saving.</p> <p>The system allows a user to change a final crossmatch interpretation of <b>Crossmatch Incompatible</b> (I) to <i>Crossmatch Incompatible: give only with Medical Director approval</i> (G) prior to saving. The system re-evaluates changes made to the test interpretation and allows the user to save with appropriate warning messages and overrides.</p> <p>Refer to Table 11: Crossmatch Interpretations for Print Unit Caution Tag &amp; Transfusion Record Form.</p>
10. Refer to Step 5 and Step 6 for process to save the crossmatch results.	

**Figure 146: Automated Testing - Pending Specimen Tests Window**

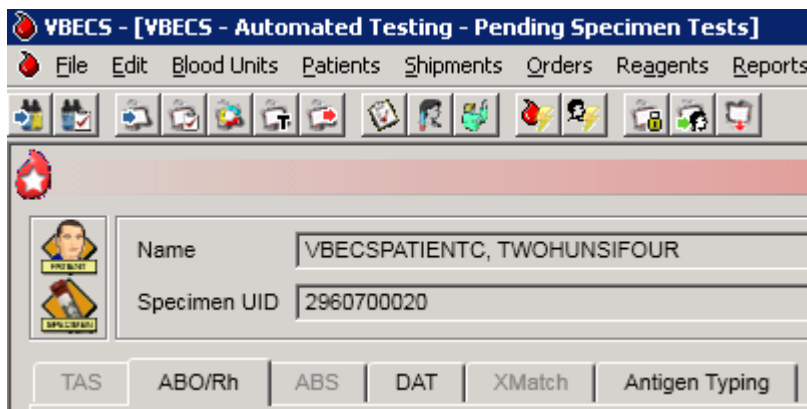



# Reports

The report information that is presented on-screen, not the print preview, can be sorted by clicking on each column header. The sort changes made do not apply to the printed reports. Reports print in a default sort order.

**Note 1:** Switching to Normal layout view from Print layout view so the user can search the report may cause the error message “**Error. Thread was being aborted.**” to appear for a large report. The report proceeds to load in the different layout view chosen by the user. This is a normal behavior of a report engine and no harm is done.

**Note 2:** The transfusion count total may indicate a site needs to complete a National Blood Collection and Utilization Survey if the total transfusions is greater than or equal to 100.

**Note 3:** The report search in VBECS is limited to 2000 characters. Requesting a report that exceeds this limit will result in an incomplete report.

## ***Administrative Data Report***

The user views and/or prints the Administrative Data Report.

### **Assumptions**

- Data are available for inclusion in the report.

### **Outcome**

- The user views and/or prints the report.

### **Limitations and Restrictions**

- None

### **Additional Information**

- The report is available as a single-division or multidivisional (all divisions) report.

### **User Roles with Access to This Option**

All users

## **Administrative Data Report**

The user views and/or prints the Administrative Data Report, which includes blood product transactions for a given date range.

<b>User Action</b>	<b>VBECS</b>
1. Select <b>Reports</b> from the main menu.  Select <b>Administrative Data Report</b> .	<ul style="list-style-type: none"><li>• Lists report names.</li><li>• Displays report compilation criteria.</li><li>• Displays a date range for selection.</li></ul>
2. Enter or select start and end dates.	<ul style="list-style-type: none"><li>• Allows the user to select one or all divisions.</li></ul>

User Action	VBECS
<p>Click the <b>Current Divisions</b> or <b>All Divisions</b> radio button.</p>	<p><b>NOTES</b></p> <hr/> <p>For each component class and its subdivisions, VBECS tallies:</p> <ul style="list-style-type: none"> <li>• Products received</li> <li>• Products modified/prepared</li> <li>• Units transfused</li> <li>• Units shipped</li> <li>• Outdated units</li> </ul> <p>Tallies are presented for total units received as well as subtotals for donation type categories, allogeneic, directed, and autologous.</p> <p>Tallies are presented for total units received as well as subtotals for donation type categories, allogeneic, directed, and autologous.</p> <p>All categories are calculated within the selected date range, not for a total in the system.</p> <p>If there is no calculation for a given field, the system will not display any text.</p> <p>Identify split/divided units using the unit product code in ISBT 128; labeling contains the unit division code in the seventh and eighth characters or may be a separate product code, indicated as 1st container, 2nd container, and so on.</p> <p>For Pooled totals PREPARED in VBECS, include two numbers, the first being the total number of units included in those pools. The second number is number of pools created, for example, (2) 1.</p>
<p>3. In the Report Compilation area, click the <b>Create Report Now</b> radio button. Click <b>OK</b> to view the report, or</p> <p>Click the <b>Schedule Print Job</b> radio button and <b>OK</b>, then enter or select a print date and printer name. Click <b>OK</b> to view and print the report.</p>	<ul style="list-style-type: none"> <li>• Compiles and displays the report.</li> </ul> <p><b>NOTES</b></p> <hr/> <p>When the date range requested for a section of the report does not provide any data, print on the report that no entries are found for the requested criteria.</p> <p>The user may choose to schedule a report to run at a future date and time and print on a designated printer. The date range requested might include a future date that will occur before the date/time of printing. This applies only when a date range is supplied for reports. The report scheduled to run at the future date/time will start processing the data at the time the report is scheduled to print (future date/time). If the date range spans more than 30 days, the system displays the warning message: "You are requesting more than 30 days of data to be printed. Continue?"</p> <p>When the user chooses to a view report, the system always displays the report. The user may elect to print a hard copy of the report on a designated printer or exit.</p>

User Action	VBECS
4. Select <b>Print</b> to print the report, <b>Export</b> to export the report (See Report Export section), or click <b>Close</b> to exit without printing.	<ul style="list-style-type: none"> <li>• If Print is selected, prints the report on the designated printer.</li> <li>• If Export is selected, exports the report to the designated file.</li> <li>• If Close is selected, exits the report preview.</li> </ul>

## Audit Trail

The user views and/or prints an Audit Trail Report.

### Assumptions

- Data are available for inclusion in the report.
- The user must be logged into the division in which the report is to be generated.

### Outcome

- The user views and/or prints the report.

### Limitations and Restrictions

- VBECS does not accommodate online review (verification, signature) of reports.

### Additional Information

- Audit trail entries reflect the actual date and time the change was made.
- VBECS displays only the ISBT 128 five-digit product code in the sub-header sections of the report. The full eight-digit product code is available in the Unit History Report.

### User Roles with Access to This Option

All users

### Audit Trail

The user views and/or prints an Audit Trail Report, which lists changes to specific patient, unit, and division configuration settings. It is a regulatory requirement to maintain documentation proving that these changes were reviewed. Modify local record retention policy to instruct users to print, review, document, and save these reports. The Audit Trail Report contains only data changes generated in the user's login division.

User Action	VBECS
1. Select <b>Reports</b> from the main menu.  Select <b>Audit Trail</b> .	<ul style="list-style-type: none"><li>• Lists report names.</li><li>• Displays report compilation criteria.</li><li>• Displays a date range for selection.</li></ul>
2. Enter or select start and end dates.	<p><b>NOTES</b> _____</p> <p>The parameter table and Transfusion Requirements Report sections include changes from no previous entry to an entry and from an entry to no entry.</p> <p>Each entry includes an area for supervisor acknowledgement (signature or initials and date of review) and comment.</p>

User Action	VBECS
<p>3. In the Report Compilation area, click the <b>Create Report Now</b> radio button. Click <b>OK</b> to view the report, or</p> <p>Click the <b>Schedule Print Job</b> radio button and <b>OK</b>, then enter or select a print date and printer name. Click <b>OK</b> to view and print the report.</p>	<ul style="list-style-type: none"> <li>• Compiles and displays the report.</li> </ul>
<p>4. Select <b>Print</b> to print the report, <b>Export</b> to export the report (See Report Export section), or click <b>Close</b> to exit without printing.</p>	<ul style="list-style-type: none"> <li>• If Print is selected, prints the report on the designated printer.</li> <li>• If Export is selected, exports the report to the designated file.</li> <li>• If Close is selected, exits the report preview.</li> <li>• See Appendix B: Table 20: Details in Audit Trail Report.</li> </ul>

## Blood Availability

The user views and/or prints the Blood Availability Report.

### Assumptions

- A blood unit is logged into a division in the database.

### Outcome

- VBECS users may view available blood unit data in several formats.

### Limitations and Restrictions

- None.

### Additional Information

- VBECS displays the defined reports and allows the user to view an individual Patient Blood Availability Report or to create a custom report.
- The report is available as a single-division or multidivisional (all divisions) report.

### User Roles with Access to This Option

All users

## Blood Availability

The user may check the blood availability for a specific patient and view and/or print a variety of Blood Inventory Reports from the user's and other divisions, as available. Blood Availability Reports include the product's unit status, indicators, patient information for units in the "assigned" and "issued" statuses, restrictions, and special testing or handling of a unit.

User Action	VBECS
<p>1. For a non-individual report, select <b>Reports</b> from the main menu.</p> <p>Select <b>Blood Availability</b>, or</p> <p>For a patient-specific report, select <b>Patients</b> from the main menu.</p> <p>Select <b>Blood Availability</b>.</p>	<ul style="list-style-type: none"><li>• Displays options for generating, viewing, and printing blood reports.</li><li>• Displays non-individual report type and compilation criteria.</li><li>• Displays options for generating a patient-specific report.</li></ul>

User Action	VBECS
<p>2. Click a radio button to select a report format:</p> <ul style="list-style-type: none"> <li>• <b>Daily Inventory (All Units)</b></li> <li>• <b>Daily Inventory (Available Units)</b></li> <li>• <b>Antigen Typed Inventory (Available Units)</b></li> <li>• <b>Units with No Disposition (Available Expired)</b></li> <li>• <b>Units with Final Disposition (Not Transfused)</b></li> <li>• <b>Custom Report</b></li> <li>• <b>Individual Patient Blood Availability</b></li> </ul>	<ul style="list-style-type: none"> <li>• Displays a print preview of the report.</li> </ul> <p><b>NOTES</b> _____</p> <p>The user may click the <b>Custom Report</b> radio button and <b>OK</b>, and then click the appropriate check boxes in the Component Class, Testing, Donation Type, Unit Status, Date Range, ABO/Rh, Product Attributes, and Division tabs to customize the report.</p> <p>When a user selects the Individual Patient Blood Availability report for his division only, the report includes all units available (unit statuses: assigned, crossmatched, and issued) in his division for the selected patient.</p> <p>When a user selects Units with Final Disposition, the report includes units with a status of “discarded,” “modified,” or “transferred.”</p> <p>When a user selects Units with No Disposition, the report includes expired units with a status of “limited” or “available.”</p>
<p>3. In the Report Compilation area, click the <b>Create Report Now</b> radio button. Click <b>OK</b> to view the report, or</p> <p>Click the <b>Schedule Print Job</b> radio button and <b>OK</b>, then enter or select a print date and printer name. Click <b>OK</b> to view and print the report.</p>	<ul style="list-style-type: none"> <li>• Compiles and displays the report.</li> </ul>
<p>4. Select <b>Print</b> to print the report, <b>Export</b> to export the report (See Report Export section), or click <b>Close</b> to exit without printing.</p>	<ul style="list-style-type: none"> <li>• If Print is selected, prints the report on the designated printer.</li> <li>• If Export is selected, exports the report to the designated file.</li> <li>• If Close is selected, exits the report preview.</li> </ul>

## Cost Accounting Report

The user views and/or prints the Cost Accounting Report.

### Assumptions

- Data for the report were saved in VBECS.

### Outcome

- The report is viewable or printed.

### Limitations and Restrictions

- None

### Additional Information

- The report is for a single division.
- A unit with an edited return credit in final status will not revert to the original return credit amount when the unit's final status is removed. Select Edit Financial Data and correct the return credit amount.

### User Roles with Access to This Option

All users

## Cost Accounting Report

The user views and/or prints the Cost Accounting Report for a division. The Cost Accounting Report itemizes incoming and outgoing shipments, modifications of blood products, and patient testing costs for a given period. The report includes base cost, special testing, modifications, and return credits for units, as applicable.

User Action	VBECS
1. Select Reports from the main menu.  Select <b>Cost Accounting Report</b> .	<ul style="list-style-type: none"> <li>• Lists report names.</li> <li>• Displays report compilation criteria.</li> <li>• Displays a date range for selection.</li> </ul>
2. Enter or select start and end dates.	<ul style="list-style-type: none"> <li>• Compiles and displays the report.</li> </ul> <p><b>NOTES</b> _____</p> <p>Final calculations appear at the end of the report:</p> <ul style="list-style-type: none"> <li>• Subtotal of base cost per invoice (Incoming Shipment section only)</li> <li>• Subtotal of special testing cost (Incoming Shipment section only)</li> <li>• Subtotal of return credit for an invoice (Outgoing Shipment section only)</li> <li>• Total base cost of all units</li> <li>• Total return credits of all units</li> <li>• Total special testing cost of all units</li> <li>• Total off-site antibody identification testing cost</li> <li>• Total number of units received</li> <li>• Total number of units transferred</li> <li>• Total number of units modified</li> </ul>



User Action	VBECS
	<ul style="list-style-type: none"> <li>• Total cost of modifications</li> <li>• Total number of units discarded</li> </ul>
<p>3. In the Report Compilation area, click the <b>Create Report Now</b> radio button. Click <b>OK</b> to view the report, or</p> <p>Click the <b>Schedule Print Job</b> radio button and <b>OK</b>, then enter or select a print date and printer name. Click <b>OK</b> to view and print the report.</p>	<ul style="list-style-type: none"> <li>• Compiles and displays the report.</li> </ul>
<p>4. Select <b>Print</b> to print the report, <b>Export</b> to export the report (See Report Export section), or click <b>Close</b> to exit without printing.</p>	<ul style="list-style-type: none"> <li>• If Print is selected, prints the report on the designated printer.</li> <li>• If Export is selected, exports the report to the designated file.</li> <li>• If Close is selected, exits the report preview.</li> </ul>

## C:T Ratio Report

The user views and/or prints the C:T Ratio Report.

### Assumptions

- Data for the report are available.

### Outcome

- The report is viewed or printed.

### Limitations and Restrictions

- None

### Additional Information

- This calculation does not include component classes that do not require XMs [FRESH FROZEN PLASMA, CRYOPRECIPITATE, PLATELETS (< 2 mL RBC contamination and component types in the OTHER component class)].
- The report is available as a single-division or multidivisional (all divisions) report.

### User Roles with Access to This Option

All users

## C:T Ratio Report

This option compiles and calculates the crossmatch-to-transfusion ratio. The user may select one of five report formats.

User Action	VBECS
1. Select <b>Reports</b> from the main menu.  Select <b>C:T Ratio Report</b> .	<ul style="list-style-type: none"> <li>• Lists report names.</li> <li>• Displays report type and compilation criteria.</li> <li>• Displays a crossmatched date range for selection.</li> </ul>
2. Enter a crossmatched date range.	<ul style="list-style-type: none"> <li>• Displays an option to compile the report for one or all divisions in the database.</li> </ul>
3. Click the <b>Current Division</b> or <b>All Divisions</b> radio button.	<ul style="list-style-type: none"> <li>• Allows the user to select a report format.</li> </ul> <p><b>NOTES</b> _____</p> <p>The user may request the Detailed Report, Physician Summary Report, Treating Specialty Summary Report, Detailed Report with Physician Summary, or Detailed Report with Treating Specialty Summary.</p>
4. Click a radio button to select a report format: <ul style="list-style-type: none"> <li>• <b>Detailed Report</b></li> <li>• <b>Physician Summary Report</b></li> <li>• <b>Treating Specialty Summary Report</b></li> <li>• <b>Detailed Report with Physician Summary</b></li> </ul>	<ul style="list-style-type: none"> <li>• Compiles the report and displays an option to print the report or exit.</li> </ul> <p><b>NOTES</b> _____</p> <p>The physician included in this report is the ordering physician. VBECS compiles the C:T ratio report as a summary or detailed report. All versions include a single C:T ratio for the division.</p>

User Action	VBECS
<ul style="list-style-type: none"> <li>• <b>Detailed Report with Treating Specialty Summary</b></li> </ul>	<p>VBECS determines the C:T ratio for a division by dividing the number of units crossmatched (electronically or serologic) during the period selected by the number of units transfused at the division.</p> <p>This calculation does not include units with a XM result of "Compatible - Don't Transfuse" or "Incompatible - Unsafe to Transfuse."</p> <p>The detailed C:T ratio report includes a list of units crossmatched (electronically or serologic) for a patient during the selected date range. The report includes the unit ID, physician name, treating specialty, XM interpretation, date and time the XM was completed, product short name, and an indication as to whether the unit was transfused or presumed transfused.</p> <p>VBECS determines the C:T ratio for a physician by dividing the number of units that were crossmatched (electronically or serologic) during the period selected by the number of transfused units.</p> <p>VBECS determines the C:T ratio for a treating specialty by dividing the number of units crossmatched (electronically or serologic) for all patients listed in that treating specialty by the number of units transfused to all patients listed for that specialty.</p>
<p>5. In the Report Compilation area, click the <b>Create Report Now</b> radio button. Click <b>OK</b> to view the report, or</p> <p>Click the <b>Schedule Print Job</b> radio button and <b>OK</b>, then enter or select a print date and printer name. Click <b>OK</b> to view and print the report.</p>	<ul style="list-style-type: none"> <li>• Compiles and displays the report.</li> </ul>
<p>6. Select <b>Print</b> to print the report, <b>Export</b> to export the report (See Report Export section), or click <b>Close</b> to exit without printing.</p>	<ul style="list-style-type: none"> <li>• If Print is selected, prints the report on the designated printer.</li> <li>• If Export is selected, exports the report to the designated file.</li> <li>• If Close is selected, exits the report preview.</li> </ul>

## Division Transfusion Report

The user views and/or prints the Division Transfusion Report.

### Assumptions

- Transfused unit data for the report were saved in VBECS.

### Outcome

- The user views and/or prints the report.

### Limitations and Restrictions

- None

### Additional Information

- VBECS compiles data for the report from the user's login division.

### User Roles with Access to This Option

All users

## Division Transfusion Report

The user views and/or prints a Division Transfusion Report, which lists details of units transfused during a selected period.

User Action	VBECS
<p>1. Select <b>Reports</b> from the main menu.</p> <p>Select <b>Division Transfusion Report</b>.</p>	<ul style="list-style-type: none"> <li>• Lists report names.</li> <li>• Displays fields for entering a report date range.</li> </ul>
<p>2. Enter or select start and end dates.</p> <p>In the Report Type area, select the <b>Issuing Physician Report</b>, <b>Treating Specialty Report</b>, or <b>Issued-To Location Report</b> radio button.</p>	<ul style="list-style-type: none"> <li>• Allows the user to compile the report by issuing physician, by treating specialty, or by issued-to location.</li> <li>• Allows the user to include some or all physicians, treating specialties, or issued-to locations.</li> </ul> <p><b>NOTES</b> _____</p>
<p>3. In the Report Compilation area, click the <b>Create Report Now</b> radio button. Click <b>OK</b> to review the report, or</p> <p>Click the <b>Schedule Print Job</b> radio button and <b>OK</b>, then enter or select a print date and printer name. Click <b>OK</b> to view and print the report.</p>	<ul style="list-style-type: none"> <li>• Compiles and displays the report.</li> </ul> <p><b>NOTES</b> _____</p> <p>The report displays tallies of units transfused by component class within the date range, by:</p> <ul style="list-style-type: none"> <li>• Physician name</li> <li>• Treating specialty</li> <li>• Issued-to location</li> <li>• Number of units transfused</li> <li>• Number of units presumed transfused</li> <li>• Total number of patients transfused</li> <li>• Total number of units transfused and presumed transfused</li> </ul>

User Action	VBECS
	Reports include: <ul style="list-style-type: none"> <li>• Patient name</li> <li>• Patient ID</li> <li>• Unit ID</li> <li>• Product short name</li> <li>• If pooled, number of participants in pool</li> <li>• Date and time Issued</li> <li>• Issued-to location</li> <li>• Physician</li> <li>• Treating specialty</li> <li>• Transfused or presumed transfused indicator, if present</li> <li>• Transfusion comments, if any</li> </ul>
4. Select <b>Print</b> to print the report, <b>Export</b> to export the report (See Report Export section), or click <b>Close</b> to exit without printing.	<ul style="list-style-type: none"> <li>• If Print is selected, prints the report on the designated printer.</li> <li>• If Export is selected, exports the report to the designated file.</li> <li>• If Close is selected, exits the report preview.</li> </ul>

## Division Workload Report

The user views and/or prints a workload report.

### Assumptions

- VistA workload codes were assigned to VBECS processes through Workload Codes.
- Data for the report were saved in VBECS.

### Outcome

- The user views and/or prints the report.

### Limitations and Restrictions

- A report printed with an end date of today or greater does not print “Preliminary” in the header. Check the subsequent pages of the report and write “PRELIMINARY” on the front page of the report prior to filing. The information included in the report is accurate to the time printed. There is a possibility of a gap of information should the user not overlap the print requests. Generally, this report is printed for the day prior or a time period that does not include the current day.

### Additional Information

- The report is available as a single-division or multidivisional (all divisions) report.
- The report will not print as “Preliminary” when the report contains the current day in the range of data requested.

### User Roles with Access to This Option

All users

## Division Workload Report

The user views and/or prints a workload report, which includes totals of workload types, for one or all divisions.

User Action	VBECS
1. Select <b>Reports</b> from the main menu.  Select <b>Division Workload Report</b> .	<ul style="list-style-type: none"><li>• Lists report names.</li><li>• Displays report division and compilation criteria.</li><li>• Displays a date range for selection.</li></ul>
2. Click the <b>Current Divisions</b> or <b>All Divisions</b> radio button.	<ul style="list-style-type: none"><li>• Allows the user to select one or all divisions.</li><li>• Displays a date range for selection.</li></ul>
3. Enter or select start and end dates.	<ul style="list-style-type: none"><li>• Compiles and displays the report.</li><li>• Allows the user to print the report.</li></ul> <p><b>NOTES</b> _____</p> <p>A multiple-division workload report includes totals for each type of workload for each division requested in the report.</p> <p>VBECS searches its workload file for entries within the selected date range.</p>

User Action	VBECS
<p>4. In the Report Compilation area, click the <b>Create Report Now</b> radio button. Click <b>OK</b> to view the report, or</p> <p>Click the <b>Schedule Print Job</b> radio button and <b>OK</b>, then enter or select a print date and printer name. Click <b>OK</b> to view and print the report.</p>	<ul style="list-style-type: none"> <li>• Compiles and displays the report.</li> </ul>
<p>5. Select <b>Print</b> to print the report, <b>Export</b> to export the report (See Report Export section), or click <b>Close</b> to exit without printing.</p>	<ul style="list-style-type: none"> <li>• If Print is selected, prints the report on the designated printer.</li> <li>• If Export is selected, exports the report to the designated file.</li> <li>• If Close is selected, exits the report preview.</li> </ul>

## **Exception Report**

The user views and/or prints an Exception Report.

### **Assumptions**

- Data are available for inclusion in the report.
- The user must be logged into the division in which the report is to be generated.

### **Outcome**

- The user views and/or prints the report.

### **Limitations and Restrictions**

- VBECS does not accommodate online review (verification, signature) of reports.
- Only the first twenty patient antigens will print on the report when an exception is generated for a unit issued with unsatisfied antigen negative requirements.
- In some areas of the report, the user ID is displayed instead of the user name.
- The Exception Date displayed is not consistently recorded to data records. It will either be the time the user entered the exception comment or the time the user saved the data record.
- Some exceptions do not display full (eight-digit) product codes for ISBT 128 labeled units. The sixth, seventh, and eighth digits are not included.

### **Additional Information**

- A supervisor may use a printed report to document the review.
- Exception Report entries reflect the actual date and time an exception occurred.
- Exception report entries associated with testing do not include additional information about method, manual or automated instrument. See the *Testing Worklist Report, Patient History Report or Unit History Report* for test method information.
- In the Emergency Issue section of the report, VBECS displays ISBT unit ID and product short name, but not the product code. See the Unit History report for the missing data saved in other options.
- Issue Blood Component exceptions that include the Testing Status field displays numeric values 0, 1, 2 or 3. Those numbers represent the unit's disease marker testing.
  - (0) – N/A for non-autologous units,
  - (1) – NEG – Unit fully tested and negative for all disease markers,
  - (2) – POS – Unit tested and positive for one or more disease markers, Biohazard,
  - (3) – NFT – Unit not fully tested for one or more disease markers, Biohazard.

### **User Roles with Access to This Option**

All users

### **Exception Report**

The user views and/or prints an Exception Report. Exception Reports list details of transactions that involve overrides to warnings and selected transactions of interest to the supervisor (for example, quarantined units).



User Action	VBECS
1. Select <b>Reports</b> from the main menu.  Select <b>Exception Report</b> .	<ul style="list-style-type: none"> <li>• Lists report names.</li> <li>• Displays report compilation criteria.</li> <li>• Displays a date range for selection.</li> </ul>
2. Enter or select start and end dates.	<p><b>NOTES</b> _____</p> <p>The report is sorted by exception type. Exception types are in chronological order.</p>
3. In the Report Compilation area, click the <b>Create Report Now</b> radio button. Click <b>OK</b> to view the report, or  Click the <b>Schedule Print Job</b> radio button and <b>OK</b> , then enter or select a print date and printer name. Click <b>OK</b> to view and print the report.	<ul style="list-style-type: none"> <li>• Compiles and displays the report.</li> </ul>
4. Select <b>Print</b> to print the report, <b>Export</b> to export the report (See Report Export section), or click <b>Close</b> to exit without printing.	<ul style="list-style-type: none"> <li>• If Print is selected, prints the report on the designated printer.</li> <li>• If Export is selected, exports the report to the designated file.</li> <li>• If Close is selected, exits the report preview.</li> <li>• See Appendix B: Table 21: Details in Exception Report.</li> </ul>

## ***Finalize/Print TRW (See Patient Services, Finalize/Print TRW)***

### ***Inappropriate Transfusion Request Report***

The user views and/or prints the Inappropriate Transfusion Request Report.

#### **Assumptions**

- The order is flagged as an inappropriate transfusion request during Accepting an Order.
- Setting Component Requirements was processed and there are inappropriate test indicators associated with the component classes in the login division.
- Data for the report were saved in VBECS.

#### **Outcome**

- The user view or prints the report.

#### **Limitations and Restrictions**

- None

#### **Additional Information**

- None

#### **User Roles with Access to This Option**

All users

### **Inappropriate Transfusion Request Report**

The user views and/or prints the Inappropriate Transfusion Request Report, which includes inappropriate transfusion orders determined by the division's configuration of each component class.

<b>User Action</b>	<b>VBECS</b>
1. Select <b>Reports</b> from the main menu.  Select <b>Inappropriate Transfusion Request Report</b> .	<ul style="list-style-type: none"><li>• Lists report names.</li><li>• Displays report compilation criteria.</li><li>• Displays a date range for selection.</li></ul>
2. Enter or select start and end dates.	<ul style="list-style-type: none"><li>• Compiles and displays the report.</li></ul>
3. In the Report Compilation area, click the <b>Create Report Now</b> radio button. Click <b>OK</b> to view the report, or  Click the <b>Schedule Print Job</b> radio button and <b>OK</b> , then enter or select a print date and printer name. Click <b>OK</b> to view and print the report.	<ul style="list-style-type: none"><li>• Compiles and displays the report.</li></ul>
4. Select <b>Print</b> to print the report, <b>Export</b> to export the report (See Report Export section), or click <b>Close</b> to exit without printing.	<ul style="list-style-type: none"><li>• If Print is selected, prints the report on the designated printer.</li><li>• If Export is selected, exports the report to the designated file.</li><li>• If Close is selected, exits the report preview.</li></ul>

## Issued/Returned Unit Report

The user views and/or prints the Issued/Returned Unit Report.

### Assumptions

- Data for the report were saved in the VBECS database.

### Outcome

- The user views and/or prints the report.

### Limitations and Restrictions

- None

### Additional Information

- The report is compiled for one division.
- Pooled units include the number of participants in the pool.

### User Roles with Access to This Option

All users

## Issued/Returned Unit Report

The user views and/or prints a report of units issued and/or returned to patients within a specified date range for a division.

User Action	VBECS
1. Select <b>Reports</b> from the main menu.  Select <b>Issued/Returned Report</b> .	<ul style="list-style-type: none"> <li>• Lists report names.</li> <li>• Displays fields for entering a report date range.</li> </ul>
2. Enter or select start and end dates.	<ul style="list-style-type: none"> <li>• Allows the user to sort the report by issued date range or by patient name.</li> </ul>
3. In the Report Sort Order area, select the <b>Issue/Return Date</b> or <b>Patient Name</b> radio button.	<ul style="list-style-type: none"> <li>• Compiles the report and, for each unit issued and when units are returned, includes return information.</li> </ul> <p><b>NOTES</b> _____</p> <p>The report includes a unit that is issued or returned within the entered date range. When a unit is identified as issued or returned during the entered date range, the report also includes unit return and issuance data, respectively, even when they fall outside the date range of the report.</p> <p>If a unit is entered into Incoming Shipment as one product code and then issued, and after issue modified to a new product code, the report will not show the original product code. Only the new modified product code displays. A user can determine the original product code by viewing the Unit History Report.</p> <p>The report does not include the selected date/ time or processing date/ time fields. This information is displayed on the Unit History Report in the Issue Information section.</p>

User Action	VBECS
<p>4. In the Report Compilation area, click the <b>Create Report Now</b> radio button. Click <b>OK</b> to view the report, or</p> <p>Click the <b>Schedule Print Job</b> radio button and <b>OK</b>, then enter or select a print date and printer name. Click <b>OK</b> to view and print the report.</p>	<ul style="list-style-type: none"> <li>• Compiles and displays the report.</li> </ul>
<p>5. Select <b>Print</b> to print the report, <b>Export</b> to export the report (See Report Export section), or click <b>Close</b> to exit without printing.</p>	<ul style="list-style-type: none"> <li>• If Print is selected, prints the report on the designated printer.</li> <li>• If Export is selected, exports the report to the designated file.</li> <li>• If Close is selected, exits the report preview.</li> </ul>

## Medication Profile (See Patient Information Toolbar)

### Order History Report

The users views and/or prints a Single-Order History Report or an Order Summary Report.

#### Assumptions

- Data for the report are available.

#### Outcome

- The user may view and/or print a report.

#### Limitations and Restrictions

- VBECS does not display the identity of the CPRS user who discontinues a signed order in CPRS. The CPRS order details option contains all pertinent order information.
- Canceled orders are included on the Order Summary Report without a column/data to help identify them as canceled. The Single Order History Report includes the canceled order information. When the order is also canceled in VistA, which would be the norm, there is a lab report that can be referenced.

#### Additional Information

- Test counts and workload information for other VBECS processes are in the Workload Report.

#### User Roles with Access to This Option

All users

### Order History Report

The user views and/or prints an Order History Report. The user may retrieve a report for a single order or a summary report for all VBECS orders within a date range. The user may sort the Order Summary Report by treating specialty or by patient location.

User Action	VBECS
1. Select <b>Reports</b> from the main menu.  Select <b>Order History Report</b> .	<ul style="list-style-type: none"><li>• Lists report names.</li><li>• Displays options for generating, viewing, and printing reports.</li><li>• Displays report type and compilation criteria.</li></ul>
2. Click the <b>Order Summary Report</b> or <b>Single-Order History Report</b> radio button to select a report type.	
3. For an Order Summary Report, create a report date range:  Select or edit the dates and times in the Start Date and End Date fields.  In the <b>Report Sort Order</b> area, click the <b>Treating Specialty</b> or	<ul style="list-style-type: none"><li>• Displays the option to enter a date range for the report based on the date and time VBECS acknowledged the order.</li><li>• Displays an option to sort the report by treating specialty or by ordering location. The report tallies orders in a given date range by ordering location and treating specialty.</li></ul>

User Action	VBECS
<p><b>Ordering Location</b> radio button and continue at Step 5.</p>	
<p>4. For a Single-Order History Report:</p> <p>Enter the search criteria and/or click the <b>find</b> button to select an order from the list.</p>	<ul style="list-style-type: none"> <li>• Displays the Select Patient Order window.</li> </ul> <p><b>NOTES</b> _____</p> <p>The user enters a date or a specimen UID, or selects a patient to locate an order. VBECS displays the following information based on the selected search option:</p> <ul style="list-style-type: none"> <li>• When a user enters a date, VBECS displays orders accepted or canceled by VBECS on that date.</li> <li>• When the specimen UID is entered, VBECS displays only orders associated with that specimen.</li> <li>• When a patient name is selected, VBECS displays the patient's orders listed in reverse chronological order for the last 90 days with an option to display all of the patient's accepted and canceled orders.</li> </ul>
<p>5. In the Report Compilation area, click the <b>Create Report Now</b> radio button and <b>OK</b> to view the report, or</p> <p>Click the <b>Schedule Print Job</b> radio button and <b>OK</b>, then enter or select a print date and printer name, then click <b>OK</b> to view and print the report.</p>	<ul style="list-style-type: none"> <li>• Compiles and displays the report.</li> </ul> <p><b>NOTES</b> _____</p> <p>The Order Summary Report includes a section for each treating specialty or ordering location, including accepted and canceled VBECS orders and the Diagnostic Test or Component Orders for the selected date range. The report lists individual orders by order type, including the Patient Name, ID, CPRS and Lab Order Numbers, and Date.</p> <p>The Single-Order History Report contains details of an individual order group, including:</p> <ul style="list-style-type: none"> <li>• Patient Name</li> <li>• Patient ID</li> <li>• CPRS Order Name</li> <li>• CPRS and Lab Order Numbers</li> <li>• Order Urgency</li> <li>• CPRS MODIFIER(S)</li> <li>• CPRS Reason for Request</li> <li>• CPRS Order Comment(s)</li> <li>• Specimen Accession Number</li> <li>• Specimen UID, if any</li> <li>• Specimen unacceptable comment, if any</li> <li>• Received Date, original and any updates</li> <li>• Expiration Date, original and any updates</li> <li>• Accepting Technologist Name</li> <li>• Acceptance Date and Time</li> <li>• Cancellation Date, if any</li> <li>• Cancellation Division</li> <li>• Cancellation Tech ID, if any</li> <li>• Cancellation Comment, if any</li> <li>• Ordering Provider</li> <li>• Ordering Location</li> <li>• Treating Specialty (patient at time of order)</li> <li>• Component or Diagnostic Test originally ordered</li> <li>• All Added Reflex Tests, Date(s), and Requesting Tech Name</li> </ul>

User Action	VBECS
	<ul style="list-style-type: none"> <li>• Inappropriate Request Information, if any</li> <li>• Workload name and VistA (LMIP) code after completion</li> <li>• Exception report section message: Exception report details are available as part of the patient's record. Refer to the specimen section of the patient's record.</li> </ul>
<p>6. Select <b>Print</b> to print the report, <b>Export</b> to export the report (See Report Export section), or click <b>Close</b> to exit without printing.</p>	<ul style="list-style-type: none"> <li>• If Print is selected, prints the report on the designated printer.</li> <li>• If Export is selected, exports the report to the designated file.</li> <li>• If Close is selected, exits the report preview.</li> </ul>

## Patient History Report

The user views and/or prints the Patient History Report.

### Assumptions

- VBECS displays the report header even when there is no information in the report.

### Outcome

- The user views and prints a patient's VBECS record.

### Limitations and Restrictions

- When units are restricted for a patient during Incoming Shipment, VBECS does not display them on the Patient History Report.
- VBECS displays these default values for transfusion reactions converted from VistA to VBECS. They do not reflect actual interpretations or results:

Converted Transfusion Reaction	Default
Pre-Transfusion Specimen Checks, Hemolysis Checks OK?	Yes
Post-Transfusion Specimen Checks, Hemolysis Checks OK?	Yes
Symptoms	No symptoms identified.
Implicated units	No units implicated in reaction.
Serologic tests	No serologic tests performed.

### Additional Information

- The patient record report is divided into sections for report retrieval.
- This report includes:
  - Post-transfusion details.
  - Data from all divisions within the database.

### User Roles with Access to This Option

All users

### Patient History Report

The user views and/or prints the patient's VBECS record.

User Action	VBECS
1. Select <b>Patients</b> or <b>Reports</b> from the main menu.  Select <b>Patient History Report</b> .	<ul style="list-style-type: none"><li>• Displays options for processing patient-related functions, or for generating, viewing, and printing reports.</li><li>• Displays report sections and compilation criteria.</li></ul>



User Action	VBECS
<p>2. Enter the patient name and/or ID, or click the <b>find</b> button to search for patient data.</p> <p>Click <b>OK</b>.</p>	<ul style="list-style-type: none"> <li>• Allows the entry or selection of a patient.</li> <li>• Displays the patient report section types for selection: <ul style="list-style-type: none"> <li>○ Patient Demographic History</li> <li>○ Transfusion Requirements</li> <li>○ Testing Details</li> <li>○ Unit Assignment History</li> <li>○ Transfusion History</li> <li>○ Specimen Details</li> <li>○ Special Instructions</li> <li>○ Transfusion Reaction Details</li> <li>○ Exception Report Entries</li> <li>○ Assignments/Releases</li> <li>○ Audit Trail Report</li> </ul> </li> </ul>
<p>3. Select one or more sections to include in the report.</p>	
<p>4. Click the <b>Schedule Print Job</b> radio button and <b>OK</b>, then enter or select a print date and printer name. Click <b>OK</b> to view and print the report.</p>	<ul style="list-style-type: none"> <li>• Compiles and displays the report.</li> <li>• Allows the user to print the report.</li> </ul> <p><b>NOTES</b> _____</p> <p>When the patient has an ABO/Rh as the result of a conversion, VBECS displays a message in the Transfusion History section instructing the user to check the legacy records for additional information.</p>
<p>5. Select <b>Print</b> to print the report, <b>Export</b> to export the report (See Report Export section), or click <b>Close</b> to exit without printing.</p>	<ul style="list-style-type: none"> <li>• If Print is selected, prints the report on the designated printer.</li> <li>• If Export is selected, exports the report to the designated file.</li> <li>• If Close is selected, exits the report preview.</li> </ul>

## Prolonged Transfusion Time Report

The user views and/or prints the Prolonged Transfusion Time Report.

### Assumptions

- Data for the report were saved in VBECS.

### Outcome

- The user views and prints the report.

### Limitations and Restrictions

- This report does not include presumed transfused units or units issued to a remote storage location.
- Units issued to a remote storage location will display on the report when the transfusion start time is greater than 30 minutes after issue time.
- Units issued to a remote storage location should not display to have a prolonged issue time as this setting indicates that the unit is issued to a monitored location maintained at the proper temperature for the product type.
- The Delayed (minutes) column on the report is not subtracting 30 minutes when the Transfusion Start time is greater than 30 minutes from the time of issue. Handwrite the correct Delayed (minutes) on the printed report.

### Additional Information

None

### User Roles with Access to This Option

All users

### Prolonged Transfusion Time Report

The user views and/or prints the Prolonged Transfusion Time Report, which shows transfusions started more than 30 minutes after issue and units transfused for more than the maximum transfusion time set by each division.

User Action	VBECS
1. Select <b>Reports</b> from the main menu.  Select <b>Prolonged Transfusion Time Report</b> .	<ul style="list-style-type: none"> <li>• Displays options for generating, viewing, and printing reports.</li> <li>• Displays a date range for selection.</li> <li>• Displays report sort order and compilation criteria.</li> </ul>
2. Enter a date range.  Click the <b>Transfusion Date (Reverse Chronological)</b> or <b>Issue-To Location</b> radio button to select a sort order.	
3. In the Report Compilation area, click the <b>Create Report Now</b> radio button. Click <b>OK</b> to view the report, or	<ul style="list-style-type: none"> <li>• Compiles and displays the report.</li> </ul> <p><b>NOTES</b> _____</p> <p>The report includes:</p>

User Action	VBECS
<p>Click the <b>Schedule Print Job</b> radio button and <b>OK</b>, then enter or select a print date and printer name. Click <b>OK</b> to view and print the report.</p>	<ul style="list-style-type: none"> <li>• Issue-to location</li> <li>• Unit ID</li> <li>• Product short name</li> <li>• Issue Date/time</li> <li>• Start Date/time</li> <li>• End Date/time</li> <li>• Number of minutes elapsed between unit issue and start time, when exceeds 30 minutes</li> <li>• Number of minutes elapsed between the unit start and end dates and times, when they exceed the maximum transfusion setting for the component class.</li> <li>• Patient Name</li> <li>• Patient ID</li> <li>• Treating Specialty</li> </ul> <p>The report includes instances of a prolonged start time and/or the prolonged transfusion time.</p> <p>A prolonged transfusion time is the number of minutes elapsed between the transfusion start and end times and the maximum transfusion time setting for the component class. When the number of minutes exceeds the maximum transfusion time setting for the component class, the unit and its transfusion data are included in the report.</p> <p>A delayed start of transfusion is the number of minutes elapsed between the issue time and the transfusion start time. When the start time of the unit's transfusion is greater than 30 minutes from the unit's issue time, the unit and its transfusion data are included in the report. When no start time is entered, VBECS records the issue time as the start time; it does not calculate a delayed start time.</p> <p>The total transfusion time is the number of minutes elapsed between the issue and end times.</p> <p>The report includes:</p> <ul style="list-style-type: none"> <li>• Issue-To Location</li> <li>• Unit ID</li> <li>• Product Short Name</li> <li>• Issue Date/Time</li> <li>• Start Date/Time</li> <li>• End Date/Time</li> <li>• Delayed start interval (the number of minutes elapsed between unit issue and start times, when it exceeds 30 minutes)</li> <li>• Prolonged transfusion interval (the number of minutes elapsed between unit start and end dates and times, when they exceed the maximum transfusion setting for the component class)</li> <li>• Patient Name</li> <li>• Patient ID</li> <li>• Treating Specialty</li> </ul>

User Action	VBECS
4. Select <b>Print</b> to print the report, <b>Export</b> to export the report (See Report Export section), or click <b>Close</b> to exit without printing.	<ul style="list-style-type: none"> <li>• If Print is selected, prints the report on the designated printer.</li> <li>• If Export is selected, exports the report to the designated file.</li> <li>• If Close is selected, exits the report preview.</li> </ul>

## ***Patient Testing Worklist and Testing Worklist Reports***

The user views and/or prints the Patient Testing Worklist and Testing Worklist Reports.

### **Assumptions**

- The user must be logged into the division in which the report is to be generated.

### **Outcome**

- The user views and/or prints a report, or schedules a report for future compilation and printing.

### **Limitations and Restrictions**

- Reports do not include Transfusion Reaction Workup (TRW) test results.
- VBECS does not accommodate online review (verification, signature) of reports.
- The Rack QC Testing Worklist Report section of the Testing Worklist Report does not contain a Processed By column. Maintain the original testing records for the QC that was performed offline with the testing tech information. Do not change the testing tech name when entering results in VBECS so that the logged in tech that enters the results is included on the Testing Worklist report.

### **Additional Information**

- In a transfusion-only facility, VBECS does not populate the test cell or phase results sections.
- VBECS collates complete, incomplete, and invalidated unit test results entered in the user's division when the date tested falls within the supplied date range for the unit testing, reagent QC, and patient testing sections of the report.
- VBECS includes test results on all units, regardless of status, for the unit testing section. CMV and Sickle Cell test results are not included in the report.
- VBECS includes the results of the daily rack QC and patient and unit control testing for the reagent QC section.
- For the patient testing section:
  - VBECS collates as above for the selected patient specimens within a specified date range.
  - The report does not include TRW test results.
  - VBECS includes XMs only in the patient testing section.
  - VBECS displays individual tests within the TAS and a statement in the Additional Information Column to this effect.
  - When Antibody Identification (ABID) testing was performed at an outside facility, VBECS prints the name of the outside facility in the Additional Information column.
- If a rack has only been tested offline (never tested within VBECS) and the QC has expired, the message "Rack was never QC'd and cannot be used." is displayed.
- The Testing Worklist Report does not display any information for Rack QC when QC Documentation Offline has been checked for that date.

### **User Roles with Access to This Option**

All users

## Patient Testing Worklist and Testing Worklist Reports

The user views and/or prints a Patient Testing Worklist Report for a specific patient or a Testing Worklist Report for a division. The reports contain both partially completed and completed testing data entries on patients, units, and reagents created in the same division as the user generating the report. Crossmatch test results are sequenced by date and time on the report. Corrected crossmatch results obtained after an invalidated test are not identified as such. The Interp. Column of the Patient Testing Worklist Report displays "Inc", which can mean "Incompatible" or "Inconclusive", depending on the test it is displayed with.

User Action	VBECS
<p>1. For a Patient Testing Worklist Report (for a single patient, select <b>Patients</b> from the main menu.</p> <p>Select <b>Patient Testing Worklist Report</b>.</p> <p>For a Testing Worklist Report for a division, go to Step 5.</p>	<ul style="list-style-type: none"> <li>• Displays options for processing patient-related functions.</li> <li>• Displays patient search criteria.</li> </ul>
<p>2. Enter the patient name or identification number.</p> <p>Click <b>Search</b>, as required.</p> <p>Select a patient and click <b>OK</b>.</p>	<ul style="list-style-type: none"> <li>• Requires the user to select a patient.</li> <li>• Displays specimen search criteria.</li> </ul>
<p>3. Select or edit the dates and times in the Specimen Received Start Date and End Date fields.</p> <p>Click <b>Search</b>, as required.</p>	<ul style="list-style-type: none"> <li>• Allows the user to indicate a date range and patient specimens to include in the report.</li> <li>• Displays specimens with associated testing for the selected patient.</li> </ul> <p><b>NOTES</b> _____</p> <p>The individual Patient Testing Worklist Report includes VBECS specimen UIDs from the last 90 days, by default. The user may remove one or more specimen UIDs and/or edit the default date range.</p> <p>When the patient record is not found, VBECS notifies the user that no matching patients were found.</p> <p>When the selected patient has no specimen on file within the division, VBECS warns the user and asks the user to select a different patient.</p> <p>Requires the user to indicate the format of the report:</p> <ul style="list-style-type: none"> <li>• Patient Testing</li> <li>• Unit Testing</li> <li>• Daily QC</li> <li>• Miscellaneous Reagent QC</li> </ul>
<p>4. Click one or more check boxes to select specimens.</p> <p>Click <b>OK</b>.</p> <p>Go to Step 8.</p>	<ul style="list-style-type: none"> <li>• Displays the report.</li> <li>• Allows the user to print the report.</li> </ul>

User Action	VBECS
<p>5. For a Testing Worklist Report for a division, select <b>Reports</b> from the main menu.</p> <p>Select <b>Testing Worklist Report</b>.</p>	<ul style="list-style-type: none"> <li>• Lists report names.</li> <li>• Displays the report date range, sections (Patient Testing, Unit Testing, Rack QC, and Miscellaneous Reagent QC), and compilation criteria.</li> </ul>
<p>6. Select or edit the dates and times in the <b>Start Date</b> and <b>End Date</b> fields.</p>	<ul style="list-style-type: none"> <li>• Displays the option to enter a date range for the report based on the date and time VBECS acknowledged the order.</li> </ul>
<p>7. In the Report Compilation area, click the <b>Create Report Now</b> radio button. Click <b>OK</b> to view the report, or</p> <p>Click the <b>Schedule Print Job</b> radio button and <b>OK</b>, then enter or select a print date and printer name. Click <b>OK</b> to view and print the report.</p>	<ul style="list-style-type: none"> <li>• Compiles and displays the report.</li> </ul>
<p>8. Select <b>Print</b> to print the report, <b>Export</b> to export the report (See Report Export section), or click <b>Close</b> to exit without printing.</p>	<ul style="list-style-type: none"> <li>• If Print is selected, prints the report on the designated printer.</li> <li>• If Export is selected, exports the report to the designated file.</li> <li>• If Close is selected, exits the report preview.</li> </ul>

## Transfusion Complications Report

The user views and/or prints the Transfusion Complications Report.

### Assumptions

- VistA tests are defined in Transfusion Complications.
- The connection to VistA is active.
- Transfusion Complications was processed and post-transfusion complication tests are defined.
- Units have a status of presumed transfused or transfused for the patient.

### Outcome

- The user views and/or prints the report.

### Limitations and Restrictions

- The report does not include units in an issued status.
- The report cannot be scheduled.

### Additional Information

- The report searches for evidence of a transfusion in VBECS only. If the user enters a start date that requires a search of VistA records to check back a full 12 months, the report includes a message indicating that only transfusions documented in VBECS are included in the report and that the user needs to run a supplementary report in VistA.
- The report searches for VistA tests defined in Transfusion Complications when the report is requested.
- If threshold values are words, they are case insensitive.
- A patient's test results and transfusions are not directly linked by date, episode, or admission event.
- Lab test data retrieved for the report are not saved in VBECS.
- The report is not saved in VBECS.

### User Roles with Access to This Option

All users

### Transfusion Complications Report

The user views and/or prints the Transfusion Complications Report, which includes patients transfused in his division with a test result that exceeds the threshold set in Transfusion Complications for that test. Tests associated with possible post-transfusion complications are gathered by searching for results in a user-defined date range for the report. The report also includes the documented patient transfusions from 12 months prior to the start date of the report through the end date of the report.

User Action	VBECS
1. Select <b>Reports</b> from the main menu.  Select <b>Transfusion Complications Report</b> .	<ul style="list-style-type: none"><li>• Lists report names.</li><li>• Displays fields for entering a report date range.</li></ul>



User Action	VBECS
<p>2. Enter or select start and end dates.</p>	<ul style="list-style-type: none"> <li>• Compiles and displays the report and allows the user to print it.</li> </ul> <p><b>NOTES</b> _____</p> <p>VBECS searches the Laboratory files (VistALink) and gathers instances of tests that exceed the threshold result of the tests set in Transfusion Complications within the specified date range. Retrieved test results are then filtered so that the report includes results only for patients who have transfusions documented in VBECS within the last 12 months prior to the start date of the report through the end date.</p> <p>VBECS includes the evaluated tests and their threshold values in the report criteria section.</p> <p>Details of the report include:</p> <ul style="list-style-type: none"> <li>• Patient name</li> <li>• Patient ID</li> <li>• Current hospital location</li> <li>• Date of death (in lieu of location if patient died)</li> <li>• Unit ID</li> <li>• Unit short name</li> <li>• End date and time of the unit's transfusion, if available</li> </ul> <p>From VistA for each patient's tests:</p> <ul style="list-style-type: none"> <li>• Test name</li> <li>• Associated test result</li> <li>• Date and time recorded</li> </ul>
<p>3. In the Report Compilation area, click the <b>Create Report Now</b> radio button. Click <b>OK</b> to view the report, or</p> <p>Click the <b>Schedule Print Job</b> radio button and <b>OK</b>, then enter or select a print date and printer name. Click <b>OK</b> to view and print the report.</p>	<ul style="list-style-type: none"> <li>• Compiles and displays the report.</li> </ul> <p><b>NOTES</b> _____</p> <ul style="list-style-type: none"> <li>• This report requires a logged on VistA user to retrieve the lab results. It cannot be scheduled to print.</li> </ul>
<p>4. Select <b>Print</b> to print the report, <b>Export</b> to export the report (See Report Export section), or click <b>Close</b> to exit without printing.</p>	<ul style="list-style-type: none"> <li>• If Print is selected, prints the report on the designated printer.</li> <li>• If Export is selected, exports the report to the designated file.</li> <li>• If Close is selected, exits the report preview.</li> </ul>

## Transfusion Effectiveness Report

The user views and/or prints a Transfusion Effectiveness Report.

### Assumptions

- VistA tests are defined in Transfusion Effectiveness.
- The connection to VistA is active.
- Units have a status of presumed transfused or transfused for the patient.
- Data for the report were saved in VBECS and in VistA.

### Outcome

- The user views and/or prints the report.

### Limitations and Restrictions

- The report does not include VBECS diagnostic tests.

### Additional Information

- Only verified Laboratory test results are available from VistA.
- Laboratory test data retrieved for the report from VistA are not saved in VBECS.
- A new page is started for each patient in the report.
- Test results in the report do not include units of measure or normal ranges.

### User Roles with Access to This Option

All users

### Transfusion Effectiveness Report

The user views and/or prints the Transfusion Effectiveness Report, which contains transfusion episodes and Laboratory results for patients for a selected period.

User Action	VBECS
1. Select <b>Reports</b> from the main menu.  Select <b>Transfusion Effectiveness Report</b> .	<ul style="list-style-type: none"><li>• Lists report names.</li><li>• Displays report compilation criteria.</li><li>• Displays a date range for selection.</li></ul>
2. Enter or select start and end dates.	
3. In the Report Type area, select the <b>All Patients</b> radio button, or  Click the <b>Individual Patient Report</b> radio button and the <b>find</b> button to search for a patient.	<ul style="list-style-type: none"><li>• Compiles and displays the report.</li></ul>

User Action	VBECS
<p>4. In the Report Compilation area, click the <b>Create Report Now</b> radio button. Click <b>OK</b> to view the report, or</p> <p>Click the <b>Schedule Print Job</b> radio button and <b>OK</b>, then enter or select a print date and printer name. Click <b>OK</b> to view and print the report.</p>	<ul style="list-style-type: none"> <li>• Compiles and displays the report.</li> </ul> <p><b>NOTES</b> _____</p> <ul style="list-style-type: none"> <li>• This report requires a logged on VistA user to retrieve the lab results. It cannot be scheduled to print.</li> </ul>
<p>5. Select <b>Print</b> to print the report, <b>Export</b> to export the report (See Report Export section), or click <b>Close</b> to exit without printing.</p>	<ul style="list-style-type: none"> <li>• If Print is selected, prints the report on the designated printer.</li> <li>• If Export is selected, exports the report to the designated file.</li> <li>• If Close is selected, exits the report preview.</li> </ul>

## Transfusion Reaction Count Report

The user views and/or prints a Transfusion Reaction Count Report.

### Assumptions

- Data for the report were saved in VBECS.

### Outcome

- The user views and/or prints the report.

### Limitations and Restrictions

- Transfusion Reactions are only included on the report if the data were entered/updated during the selected date range. This means that retrospective entry of transfusion reactions may not display in the correct date range. The recorded date of the reaction is not considered in the selection criteria. Rerun the report for a date range that includes the date the data was entered or updated. For example, if a transfusion reaction that occurred on February 28 is retrospectively entered on March 1, it will only display on a report that includes March 1 in its date range.

### Additional Information

- The report is available as a single-division or multidivisional (all divisions) report.
- The report only displays text entered in the “details” field of the Finalize Transfusion Reaction Workup screen. The canned comments entered are suppressed on the report. Enter all comments in the “details” field for them to print on the report.

### User Roles with Access to This Option

All users

### Transfusion Reaction Count Report

The user views and/or prints the Transfusion Reaction Count Report for a single division or for all divisions in the database.

User Action	VBECS
1. Select <b>Reports</b> from the main menu.  Select <b>Transfusion Reaction Count Report</b> .	<ul style="list-style-type: none"><li>• Lists report names.</li><li>• Displays report division, format, and compilation criteria.</li></ul> <b>NOTES</b> _____  The report includes transfusion reactions that are ordered, pending, completed, or finalized, and those indicated during Post-Transfusion Information.
2. Enter or select start and end dates.	<ul style="list-style-type: none"><li>• Compiles and displays the report.</li><li>• Allows the user to print the report.</li></ul>

User Action	VBECS
<p>3. Click the <b>Current Divisions</b> or <b>All Divisions</b> radio button.</p>	<ul style="list-style-type: none"> <li>• Allows the user to select one or all divisions.</li> <li>• Displays a date range for selection.</li> </ul> <p><b>NOTES</b> _____</p> <p>The Transfusion Reaction Count Report by type includes specific patient information indicated by the user. The default report setting excludes patient names.</p>
<p>4. Click the <b>Detailed Report</b> or <b>Summary Report</b> radio button.</p>	<ul style="list-style-type: none"> <li>• Displays an option to compile the report as detailed or summary.</li> <li>• Compiles and displays the report.</li> <li>• Allows the user to print the report.</li> </ul> <p><b>NOTES</b> _____</p> <p>The Transfusion Reaction Count Report prints the implicated unit numbers or “no implicated units,” as appropriate.</p> <p>VBECS reports the total number of patient transfusion reactions in the selected date range.</p>
<p>5. In the Report Compilation area, click the <b>Create Report Now</b> radio button. Click <b>OK</b> to view the report, or</p> <p>Click the <b>Schedule Print Job</b> radio button and <b>OK</b>, then enter or select a print date and printer name. Click <b>OK</b> to view and print the report, or click <b>Close</b> to exit without printing.</p>	<ul style="list-style-type: none"> <li>• Compiles and displays the report.</li> </ul>
<p>6. Select <b>Print</b> to print the report, <b>Export</b> to export the report (See Report Export section), or click <b>Close</b> to exit without printing.</p>	<ul style="list-style-type: none"> <li>• If Print is selected, prints the report on the designated printer.</li> <li>• If Export is selected, exports the report to the designated file.</li> <li>• If Close is selected, exits the report preview.</li> </ul>

## Transfusion Requirements Report

The user views and/or prints a Transfusion Requirements Report.

### Assumptions

- The database conversion of VistA patient files is complete.
- Data for the report were saved in VBECS.

### Outcome

- The user views and/or prints the report.

### Limitations and Restrictions

- Deceased patient records do not appear on the report when they are generated after the patient has been marked as deceased in the VBECS database via a VistA Death Notification Update.
- The cumulative report is not displaying all patients on file. A patient with ABO/Rh testing and no other requirement appears on the partial Transfusion Requirements report, but not on the cumulative report.
- A report printed with an end date of today or greater does not print “Partial” in the header.

### Additional Information

- A Transfusion Requirements report is compiled from all VBECS divisions defined in the VBECS database.
- VBECS does not allow a report to be printed when no data are available.
- When the date range includes a future date and time, VBECS displays “Partial” in the report header on each page of the report.
- The “Partial” format of the report does not allow scheduling to print in the future; a full report will print instead.
- Each page containing patient information includes the patient name.
- Divisions in a multidivisional database may view each other’s Special Instructions (SI) comments.
- Important transfusion related information for a patient may appear on more than one page. Always check the next page for additional patient information.

### User Roles with Access to This Option

All users

### Transfusion Requirements Report

The user views and/or prints the Transfusion Requirements Report for a division. This option describes the continuous compilation of the complete file of patients in the VBECS database for historic record checking during downtime.

User Action	VBECS
1. Select <b>Reports</b> from the main menu.  Select <b>Transfusion Requirements Report</b> .	<ul style="list-style-type: none"><li>• Lists report names.</li><li>• Displays report compilation criteria.</li><li>• Displays a date range for selection.</li></ul>

User Action	VBECS
<p>2. In the <b>Report Type</b> area, select the <b>Cumulative Report</b> radio button, or</p> <p>Click the <b>Partial Report</b> radio button and enter or select start and end dates.</p>	<ul style="list-style-type: none"> <li>• Compiles and displays the report.</li> </ul> <p><b>NOTES</b> _____</p> <p>Patients who do not have a blood type, SI, TR, or Transfusion Reaction in their VBECS record do not display on the Transfusion Requirements Report in either the Cumulative or Partial format.</p> <p>The Cumulative Report displays patients that have a blood type, SI, TR, or Transfusion Reaction.</p> <p>The Partial Report displays patients that have a change to their VBECS record within the date range of the requested report. Changes are defined as:</p> <ul style="list-style-type: none"> <li>• New order</li> <li>• Record of testing</li> <li>• Patient Update event</li> <li>• New SI entered</li> <li>• New TR entered</li> <li>• New Transfusion Reaction entered</li> </ul>
<p>3. In the Report Compilation area, click the <b>Create Report Now</b> radio button. Click <b>OK</b> to view the report, or</p> <p>Click the <b>Schedule Print Job</b> radio button and <b>OK</b>, then enter or select a print date and printer name. Click <b>OK</b> to view and print the report.</p>	<ul style="list-style-type: none"> <li>• Compiles and displays the report.</li> </ul>
<p>4. Select <b>Print</b> to print the report, <b>Export</b> to export the report (See Report Export section), or click <b>Close</b> to exit without printing.</p>	<ul style="list-style-type: none"> <li>• If Print is selected, prints the report on the designated printer.</li> <li>• If Export is selected, exports the report to the designated file.</li> <li>• If Close is selected, exits the report preview.</li> </ul>

## Unit History Report

The user views and/or prints a Unit History Report.

### Assumptions

- The unit exists in the user's division.

### Outcome

- The user views and/or prints the report.

### Limitations and Restrictions

- The report is not multidivisional.
- When a user retroactively updates a unit status through Discard or Quarantine, the Unit History Report displays the updated information, but does not display the date the change was made.
- The report will not indicate if a unit was returned to shipper, or shipped to another facility. This information can be found in the Cost Accounting Report.
- The report will display incorrect information in the Cost column for Splits/Divides and Pools. See the Cost Accounting Report for this information.
- The report does not display the Antigen Testing Phase Change Exception. See the Exception Report for this information.
- When a user retroactively updates a unit status through the Discard or Quarantine option, the Unit History Report displays the updated information, but does not display the date the change was made.

### Additional Information

- There may be multiple instances of the same unit number and product code in the same division. Inactivated unit records may have a unit ID and product code that duplicate an active unit ID and product code.
- VBECS always displays the current unit status when the user selects a blood unit, so there is no need for a separate report option, as is possible in VistA.
- Unit testing details are included in Testing Worklist Report.
- The report header for RBC and WB products does not display the  $\geq 2$  mL RBC contamination information.

### User Roles with Access to This Option

All users

### Unit History Report

The user views and/or prints the complete record for a blood unit, from incoming shipment through final status, including unit activities saved in the VBECS database. VBECS compiles data for that unit for one division.

User Action	VBECS
1. Select <b>Reports</b> from the main menu.  Select <b>Unit History Report</b> .	<ul style="list-style-type: none"><li>• Lists report names.</li><li>• Allows the user to scan or enter a unit ID and product code.</li><li>• Displays compilation criteria.</li></ul>



User Action	VBECS
<p>2. Scan or enter the unit ID and product code, or click the <b>find</b> button to search for and select a unit.</p>	<p><b>NOTES</b> _____</p> <p>The user may select any unit (active or inactive) in a division. VBECS marks the units so that the active ones are distinct from the inactive ones.</p> <p>When processing an Autologous unit, the user must indicate the testing status of that unit. There are three valid choices: NEG - Unit fully tested and negative for all disease markers POS - Unit tested and positive for one or more disease markers, Biohazard NFT - Unit not fully tested for one or more disease markers, Biohazard</p> <p>When the component order is for PLATELETS or OTHER the units with = &gt; 2 mL of RBC contamination must be presented and checked for compatibility during Issue Blood Component and will display before products with less than (&lt; ) 2 mL of RBC contamination.</p> <p>The system will present all available units when the Product Type is PLATELETS or OTHER non-plasma components that have &lt; 2 mL RBC contamination.</p>
<p>3. In the <b>Include Report Sections</b> area, click one or more check boxes to select report sections.</p>	<ul style="list-style-type: none"> <li>• Displays the unit report section types for selection: <ul style="list-style-type: none"> <li>○ Incoming Shipment</li> <li>○ Outgoing Shipment</li> <li>○ Unit Quarantine Information</li> <li>○ Unit Financials</li> <li>○ Patient Associations History</li> <li>○ Restricted Patient</li> <li>○ Issue Information</li> <li>○ Testing</li> <li>○ Antigen Typing</li> <li>○ Crossmatch</li> <li>○ Modification</li> <li>○ Transfused To</li> <li>○ Exception Report</li> </ul> </li> </ul>

User Action	VBECS
<p>4. In the Report Compilation area, click the <b>Create Report Now</b> radio button. Click <b>OK</b> to view the report, or</p> <p>Click the <b>Schedule Print Job</b> radio button and <b>OK</b>, then enter or select a print date and printer name. Click <b>OK</b> to view and print the report.</p>	<ul style="list-style-type: none"> <li>• Compiles and displays the report for the user's login division.</li> </ul> <p><b>NOTES</b> _____</p> <p>The page header includes the standard VBECS header, the unit ID, product, donation type, expiration date, restricted for patient, status, ABO/Rh, division, product long name, and user.</p> <p>VBECS displays entries from only the related reports (Audit Trail and Exception Reports) specific to the selected unit or patient.</p> <p>Audit Trail and Exception Reports that record only unit override information are linked to the unit's historic record. Those that record only patient override information are linked to the patient's historic record. Those that record both are accessible in unit and patient historic records.</p> <p>See Appendix B: Table 20: Details in Audit Trail Report and Table 21: Details in Exception Report.</p> <p>The report displays the product long name.</p> <p>The report displays "CMV neg", "SC neg" and/or = &gt; 2 mL RBC contamination (which will display only for non-WB and non-RBC blood products) in the "Special Testing" column when a unit qualifies.</p> <p>The report displays the unit's quarantine indicator in the "Quarantined" column: blank when never quarantined, "Yes" when quarantined, and "No" when released from quarantine.</p> <p>The report displays the unit's biohazardous indicator in the "Biohazard" column: blank when never marked as biohazardous, "Yes" when biohazardous, and "No" when the biohazard indicator was removed.</p>
<p>5. Select <b>Print</b> to print the report, <b>Export</b> to export the report (See Report Export section), or click <b>Close</b> to exit without printing.</p>	<ul style="list-style-type: none"> <li>• If Print is selected, prints the report on the designated printer.</li> <li>• If Export is selected, exports the report to the designated file.</li> <li>• If Close is selected, exits the report preview.</li> </ul>

## References

- Association for the Advancement of Medical Instrumentation, Arlington, Virginia. *The Quality System Compendium: GMP Requirements & Industry Practice*, 1998.
- Association for the Advancement of Medical Instrumentation. *Medical device software—Software life cycle processes (ANSI/AAMI SW68:2001)*, 2001.
- Brandt, Gwyneth. ClinicalTrialJobs.com. “Preparing for FDA Inspections.” July 18, 2002.
- *Clinical Laboratory Technical Procedure Manuals; Approved Guideline—Fourth Edition*, NCCLS document GP2-A4 (Wayne, PA: National Committee for Clinical Laboratory Standards, 2002).
- CMMI Product Development Team, Carnegie Mellon Software Engineering Institute. *CMMI<sup>SM</sup>-SE/SW, V1.0, Capability Maturity Model®—Integrated for Systems Engineering/Software Engineering, Version 1.0* (Pittsburgh, PA: Carnegie Mellon Software Engineering Institute, August 2000).
- Commission on Laboratory Accreditation, Laboratory Accreditation Program. *Laboratory General Checklist* (Northfield, IL: College of American Pathologists, 2017).
- Commission on Laboratory Accreditation, Laboratory Accreditation Program. *Transfusion Medicine Checklist* (Northfield, IL: College of American Pathologists, 2017).
- Joint Commission on Accreditation of Healthcare Organizations. *Comprehensive Accreditation Manual for Hospitals: The Official Handbook (CAMH)*. (Oakbrook Terrace, Illinois: Joint Commission on Accreditation of Healthcare Organizations, 2013).
- Department of Veterans Affairs, Software Engineering Process Group (SEPG) and Software Quality Assurance (SQA), *National Documentation Standards* (October 2005).
- Department of Veterans Affairs, Veterans Health Administration. *VHA Directive 99-053, VBECS Blood Bank Software*, November 8, 2007.
- Galloway, Dianne. *Mapping Work Processes*. (Milwaukee, Wisconsin: ASQ Quality Press, 1994).
- *General Principles of Software Validation; Final Guidance for Industry and FDA Staff* (U.S. Department Of Health and Human Services, Food and Drug Administration, Center for Devices and Radiological Health, Center for Biologics Evaluation and Research, January 11, 2002).
- *Guidance for Industry and FDA Premarket and Design Control Reviewers, Medical Device Use-Safety: Incorporating Human Factors Engineering into Risk Management* (U.S. Department of Health and Human Services, Food and Drug Administration, Center for Devices and Radiological Health, Division of Device User Programs and Systems Analysis, Office of Health and Industry Programs, July 18, 2000).
- *Guidance for Industry, FDA Reviewers and Compliance on Off-The-Shelf Software Use in Medical Devices* (U.S. Department Of Health And Human Services, Food and Drug Administration, Center for Devices and Radiological Health, Office of Device Evaluation, September 9, 1999).
- *Guidelines for Implementing the Electronic Crossmatch* (Bethesda, MD: American Association of Blood Banks, 2003).
- *Guidelines for the Labeling of Specimens for Compatibility Testing*. Ed. James P. AuBuchon, MD. (Bethesda, MD: American Association of Blood Banks, 2002).
- *Guidelines for the Laboratory Evaluation of Transfusion Reactions* (Bethesda, MD: American Association of Blood Banks, 2003).
- *IEEE Standard for Software Configuration Management Plans* (IEEE Std 828-1998) (New York: Institute of Electrical and Electronic Engineers, Inc., 1998).
- *ISBT 128 Bar Code Symbology and Application Specification for Labeling of Whole Blood and Blood Components*, Version Number 1.4.0 [York, PA: International Council for Commonality in Blood Banking Automation (ICCBBA), June 2001].

- Jacobson, Ivar, Grady Booch, and James Rumbaugh. *The Unified Software Development Process* (Boston: Addison-Wesley, 1999).
- *Medical devices—Application of risk management to medical devices* (ANSI/AAMI/ISO 14971:2000), 2001. Association for the Advancement of Medical Instrumentation.
- Nevalainen, D. E. and M. F. Callery. *Applying Self-Assessment and Process Analysis to Create a Quality System* (Basel, Switzerland: S. Karger Publishing, 1995).
- Nozick, Robin F. “Demystifying the Information Systems Validation Process,” *AABB News* 7:6 (2005): 50–51.
- *Object-Oriented Analysis and Design Using the UML: Student Manual, Book 1, Version 4.2* (Cupertino, CA: Rational Corporation, 1999).
- *Rational Team Concert. Version 6.0.5*, Rational Software Corporation.
- *Deciding When to Submit a 510(k) for a Software Change to an Existing Device*, October 25, 2017.
- *Department of Veterans Affairs, Office of Information and Technology (OI&T) Documentation Style Guide*, October 2016.
- *Standards for Blood Banks and Transfusion Services*. Twenty-ninth edition (Bethesda, MD: American Association of Blood Banks, 2014).
- *Technical Manual. Eighteenth edition* (Bethesda, MD: American Association of Blood Banks, 2014).
- *The Chicago Manual of Style*, 16th ed., (Chicago: The University of Chicago Press, 2014).
- *Title 21 Code of Federal Regulations*, Part 11, Electronic Records; Electronic Signatures.
- *Title 21 Code of Federal Regulations*, Part 803, Medical Device Reporting.
- *Title 21 Code of Federal Regulations*, Part 806, Medical Devices; Reports of Corrections and Removals.
- *Title 21 Code of Federal Regulations*, Part 807, Establishment Registration and Device Listing for Manufacturers and Initial Importers of Devices.
- *Title 21 Code of Federal Regulations*, Part 810, Medical Device Recall Authority.
- *Title 21 Code of Federal Regulations*, Part 820, Quality System Regulation.
- *Title 21 Code of Federal Regulations*, Parts 200 to 299, Food and Drugs.
- *Title 21 Code of Federal Regulations*, Parts 600 to 799, Food and Drugs.
- *Title 21 Code of Federal Regulations*, Parts 800 to 1299, Food and Drugs.
- Trautman, Kimberly A. *The FDA and Worldwide Quality System Requirements Guidebook for Medical Devices* (Milwaukee, Wisconsin: ASQ Quality Press, 1997).
- U.S. Department Of Health and Human Services, Food and Drug Administration, Center for Devices and Radiological Health. *Design Control Guidance for Medical Device Manufacturers*, March 11, 1997.
- U.S. Department Of Health and Human Services, Food and Drug Administration, Center for Devices and Radiological Health. *Inspection of Medical Device Manufacturers, Final Guidance for Industry and FDA*, February 7, 2001.
- U.S. Department Of Health and Human Services, Food and Drug Administration, Center for Devices and Radiological Health. *Medical Device Quality Systems Manual: A Small Entity Compliance Guide*. First edition, December 1996.
- U.S. Department Of Health And Human Services, Food and Drug Administration, Center for Devices and Radiological Health, Office of Device Evaluation. *Guidance for the Content of Premarket Submissions for Software Contained in Medical Devices*, May 29, 1998.
- U.S. Department Of Health And Human Services, Food and Drug Administration, Center for Devices and Radiological Health, Office of Regulatory Affairs. *Glossary of Computerized System and Software Development Terminology*, August 1995.

- U.S. Department Of Health and Human Services, Food and Drug Administration, Center for Devices and Radiological Health. *Premarket Approval Manual*. January 1998.
- U.S. Food and Drug Administration, Office of Regulatory Affairs and the Center for Devices and Radiological Health. *Guide to Inspections of Medical Device Manufacturers*, December 1997.
- U.S. Food and Drug Administration, ORA Inspectional References, Quality Inspection System Technique. *Guide to Inspections of Quality Systems*, August 1999.
- *United States Industry Consensus Standard for the Uniform Labeling of Blood and Blood Components Using ISBT 128, Version 1.2.0*, prepared by the International Council for Commonality in Blood Banking Automation, Inc. (U.S. Department of Health and Human Services, Food and Drug Administration, Center for Biologics Evaluation and Research, November 1999).
- *Office of Enterprise Development Documentation Standards, Version 1.4*, June 2009.
- Wilson, Paul F., Larry D. Dell, and Gaylord F. Anderson. *Root Cause Analysis: A Tool for Total Quality Management* (Milwaukee, Wisconsin: ASQ Quality Press, 1993).
- *ZPL II Programming Guide*, Customer order # 46530L, Manufacturer part # 46530LB (Zebra Technologies Corporation, November 18, 2016).

This page intentionally left blank.

# Glossary

Acronym, Term	Definition
<b>AABB</b>	American Association of Blood Banks.
<b>ABID</b>	Antibody identification.
<b>ABO</b>	A group for classifying human blood, based on the presence or absence of specific antigens in the blood, which contains four blood types: A, B, AB, and O. The ABO group is the most critical of the human blood systems. It is used to determine general compatibility of donor units to a recipient.
<b>ABS</b>	Antibody screen, Antibody Screen Test.
<b>Access Code</b>	A field in the VistA New Person file used to uniquely identify a user on the VistA system.
<b>Accessioned</b>	Describes an order when a specimen has been collected or has been placed on a current collection list by the VistA Laboratory software. <i>A mechanism for accessioning orders for blood components that do not require a specimen needs to be designed.</i>
<b>Active order</b>	A patient order for blood product(s) that is not completed or expired. Depending on the component class, an accompanying specimen may be required.
<b>Active user</b>	A user with a setting in VistA that allows him to access VBECS. Inactive users may no longer access VBECS.
<b>Activities</b>	Business processes that the product supports.
<b>Activity</b>	An identified verification and validation task.
<b>ADPAC</b>	Automated Data Processing Application Coordinator.
<b>AHG</b>	Anti-Human Globulin.
<b>Aliquot</b>	A sample/portion divided off or separated as a part of a whole.
<b>Allogeneic</b>	See "Homologous."
<b>Antibody</b>	A protein in the serum of some people that will react to a specific antigen on the blood cells. <i>In the case of a red blood cell transfusion, a patient with a clinically significant (leading to a transfusion reaction and possible patient harm) identified antibody must receive only red cells that are typed and found negative for the associated antigen. Once a clinically significant antibody is identified, antigen-negative cells must always be transfused even if a current specimen no longer shows the presence of the antibody. The antibody table in VBECS addresses the antibodies associated with red cell antigens. Though many antibodies are considered clinically significant, some are more of a nuisance to testing than a problem for transfusion. Many antibodies have documentation that defines their clinical significance; that information is used to define the VBECS table system warnings and overrides.</i>
<b>Antigen</b>	A substance on the surface of a red cell that stimulates an immune response (formation of an antibody).
<b>Antigen negative requirement</b>	A prerequisite that specified blood product categories must not contain stated characteristics.

Acronym, Term	Definition
<b>Antigen typing</b>	Testing for the presence or absence of an antigen on a red blood cell.
<b>Antiglobulin</b>	An antibody that combines with a specific protein on a different type of antibody.
<b>Anti-Human Globulin</b>	A commercial reagent that binds to immunoglobulin molecules attached to the surface of a red blood cell.
<b>Antisera</b>	Plural of “antiserum.”
<b>Antiserum</b>	Blood serum (human, animal, monoclonal) containing antibodies that are specific for one or more antigens.
<b>Assigned</b>	Describes a unit of blood in a state that can be issued for transfusion and assigned to a patient.
<b>Attribute</b>	ISBT 128 term that describes a property of a collected blood product. <i>Examples include “Irradiated” and “Residual Leukocyte Content” (Leukopoor).</i>
<b>Audit Trail Report</b>	The report of changes made to previously verified data. <i>Regulations require that a mechanism be in place to prevent “electronic white-out.” An audit report contains details of the change, before and after data, date, and who made the change.</i>
<b>Autocontrol</b>	A test using a patient’s serum (plasma) and cells under the same conditions as the accompanying test.
<b>Autodiscriminate</b>	Describes the ability of a barcode scanner to identify and interpret more than one barcode symbology without having to adjust any software or hardware parameters.
<b>Autologous</b>	Describes blood donated in advance for use by the donor in the future, often for elective surgery. <i>Regulations allow for processing patients with diseases that otherwise would preclude them from donating blood to donate for themselves autologously. It is imperative that software guarantee that such blood is never available for use by anyone other than the intended recipient.</i>
<b>Barcode scanner</b>	An electronic device that uses a finely focused beam of light to sense and transmit the data in a barcode image.
<b>BCE-PPI</b>	Bar Code Enhancement – Positive Patient Identification.
<b>Billable</b>	A characteristic of an LMIP workload code that determines if statistics gathered as a result of that workload code should be included in the monthly report to the Austin Automation Center.
<b>Blood product</b>	A specific example of a blood component class (e.g., <i>AS-1 Red Blood Cells in the component class Red Blood Cells. (BloodProduct in VBECS model).</i>
<b>Blood Transfusion Record Form (BTRF)</b>	A form printed in the blood bank used to record the actual events of a transfusion. <i>Examples: start/stop time, transfusionists, vitals, reaction information. Formerly “SF 518 equivalent” (Vista nomenclature).</i>
<b>Blood type (blood group)</b>	The ABO and Rh type of blood determined by testing.



Acronym, Term	Definition
<b>Canned comment</b>	Defined comments, entered in abbreviated form and displayed as expanded text, used to enter supporting information at a variety of data entry points.
<b>Category type</b>	When used in relation to the file of canned comments, the category of the comment that determines in which functionality the comment may be selected and associated with other data.
<b>Caution Tag</b>	A blood bank-generated form attached to a unit of blood when the unit is assigned or crossmatched.
<b>CC</b>	Check Cell.
<b>Check Cell (CC)</b>	A type of reagent red cell used in the blood bank to verify the reactivity of antiglobulin antisera when the results of an antiglobulin test are negative. <i>Also referred to as "Coombs Control cells."</i>
<b>Check character</b>	A character used to check the integrity of a barcode symbol.
<b>Clinically significant antibodies</b>	If there is a chance of an adverse reaction to a transfusion, the antibodies associated with the antigens in that unit are considered clinically significant. <i>Only clinically significant antibodies require that the patient receive red cells that have been phenotyped as negative for the associated antigen.</i>
<b>CMS</b>	Centers for Medicare and Medicaid Services. The federal agency responsible for administering the Medicare and Medicaid programs as well as certain other healthcare entitlement programs. "CMS" is the new name for HCFA (Health Care Financing Administration) under the Bush administration.
<b>CMV</b>	Cytomegalovirus. Some patients should receive only blood units that have been tested and found negative for CMV.
<b>COTS</b>	Commercial-Off-The-Shelf.
<b>Compatibility testing</b>	Tests performed in the blood bank to determine the suitability of a unit of blood for a patient. <i>Depending on the component class, compatibility testing can be as limited as comparing donor and patient ABO/Rh or as complex as requiring crossmatch.</i>
<b>Completed TRW</b>	A completed TRW indicates that serologic testing was completed.
<b>Component</b>	Any raw material, substance, piece, part, software, firmware, labeling, or assembly that is intended to be included as part of the finished, packaged, and labeled device.
<b>Component class</b>	The general form of a specific product produced by a blood supplier. <i>Example: Red Blood Cells, Fresh Frozen Plasma, and Platelets. (Product Type in VBECS model)</i>
<b>Coombs test</b>	See "Anti-Human Globulin."
<b>CPRS</b>	Computerized Patient Record System. A VistA software application that provides an integrated patient record system for use by clinicians, managers, quality assurance staff, and researchers.

Acronym, Term	Definition
<b>CPT</b>	Current Procedural Terminology. A proprietary terminology system owned by the AMA that enumerates medical and surgical services typically provided by physicians. <i>Although CPT is proprietary, CMS (formerly HCFA) includes it in HCPCS and requires its use for outpatient claims submitted for Medicare and Medicaid beneficiaries.</i>
<b>Crossmatch</b>	A procedure that demonstrates ABO compatibility and the absence of clinically significant antibody reactions between a patient and a unit of blood. <i>Traditionally, a crossmatch consists of a serological test between the patient's blood and a sample of the blood unit to be transfused. New regulations allow for what is called "computer/electronic crossmatch" in defined circumstances.</i>
<b>Crossover</b>	A process by which an autologous or directed donor unit is released from the intended recipient and made available for general transfusion. <i>VBECs supports only directed unit crossover.</i>
<b>D</b>	The major antigen of the Rh system. Also "Rh <sub>0</sub> ." <i>The presence of this antigen on red blood cells determines if a patient or donor is Rh Pos or Rh Neg. A weak expression of the D antigen, referred to as "Du" or "weak D," may require additional testing to determine its presence. "D negative" is another term for "Rh negative"; "D control" is another term for "Rh control." The convention for VBECs GUIs is D control.</i>
<b>Data entry grid</b>	A grid that contains observations and interpretations of observations.
<b>Database</b>	A collection of data arranged for ease and speed of retrieval that may or may not contain data from multiple VBECs divisions.
<b>Day of use</b>	The calendar date times (0000–2359) a reagent is used.
<b>DDE</b>	Direct data entry.
<b>Delete</b>	Remove a VBECs record from user access and keep it in the database.
<b>Direct Antiglobulin Test (DAT)</b>	A laboratory test performed in a blood bank in conjunction with pre-transfusion testing or as a diagnostic test. <i>A positive DAT indicates that an antibody-antigen reaction may have occurred in vivo, which may be related to medications or previous transfusions. "Direct Coombs" and "DAT" are used interchangeably.</i>
<b>Direct data entry (DDE)</b>	The process of entering test results in a computer rather than recording them on paper.
<b>Directed</b>	Blood that is donated in advance to be set aside for a specific patient. <i>Directed donation units must meet all of the requirements of any regular unit of donated blood (all disease marker testing must be negative). If not needed by the intended recipient, a directed unit may be released for general use.</i>
<b>Disposition</b>	The recorded final state of a blood unit. <i>A blood unit with a disposition is no longer available for routine processing.</i>
<b>Edit</b>	To enter or change previously saved information. <i>Vista overwrites the data and does not create an audit trail. VBECs overlays the data and creates an audit trail.</i>

Acronym, Term	Definition
<b>Elution</b>	A procedure performed by a blood bank technologist to investigate a positive DAT. <i>A positive eluate is further tested to determine the specificity of the antibody, which has been eluted from the patient's red cells.</i>
<b>Emergency issue</b>	The release of blood prior to pre-transfusion testing and crossmatch.
<b>Exception Report</b>	A report that details a user deviation from a standard process. <i>Example: a user overrides a warning and is required to enter a justification for continuing.</i>
<b>eXM</b>	Electronic crossmatch.
<b>Expiration date</b>	The last date during which a blood component can be considered safe for transfusion. <i>If the expiration date does not include a time, the time is assumed to be midnight.</i>
<b>Facility (see “Site”)</b>	A blood bank laboratory where a blood unit is located.
<b>FDA</b>	Food and Drug Administration.
<b>FDA UDI</b>	FDA Unique Device Identification.
<b>File</b>	Source code, data, and documentation.
<b>Finalized TRW</b>	A finalized TRW indicates that the medical director reviewed the TRW, classified the reaction, and may make recommendations for future transfusions.
<b>FOIA</b>	Freedom of Information Act.
<b>HCPCS</b>	Healthcare Common Procedure Coding System. Codes used to enumerate medical and surgical services and supplies in the outpatient setting. <i>CMS (formerly HCFA) uses HCPCS for Medicare and Medicaid reimbursement. HCPCS are the required code set for medical and surgical services and supplies for the outpatient setting under the final code set standards for HIPAA. There are three levels of HCPCS codes: level I are CPT codes from AMA; level II are additional codes developed by HCFA for services and supplies not covered by CPT; level III are local codes assigned by individual state Medicare carriers. Level III codes are scheduled to be phased out under HIPAA.</i>
<b>HIPAA</b>	Health Insurance Portability and Accountability Act of 1996.
<b>Historical ABO/Rh</b>	The most recent valid entry entered in the same division as the current order/treatment record, not from the database conversion.
<b>Historical record</b>	Cumulative, redundant data, referenced continually throughout VBECS.
<b>HL7</b>	Health Level 7. An international messaging standard used predominantly in the healthcare industry.
<b>HLA (histocompatibility locus antigens)</b>	Antigens associated with cellular immunity.
<b>Homologous (also “allogeneic”)</b>	The standard type of blood donation: a volunteer donor reports to a blood collection facility and donates a unit of blood. The blood is tested and released for general use.

Acronym, Term	Definition
<b>IAM</b>	Identity and Access Management
<b>ICCBBA</b>	International Council for Commonality in Blood Banking Automation, Inc. The governing body that developed and maintained the database that governs the ISBT 128 blood labeling system.
<b>IEN</b>	Internal Entry Number.
<b>IgA (Immunoglobulin A)</b>	Immune protein usually associated with tissue response (hypersensitivity).
<b>In-date</b>	Describes the state of a unit of blood when the expiration date/time of the unit is in the future.
<b>Interpretation</b>	The evaluation of serologic test reactions.
<b>IRM</b>	Information Resource Management.
<b>Irradiate</b>	To expose blood products to radiation to damage white cells and avoid graft-versus-host disease.
<b>ISBT</b>	International Society of Blood Transfusion.
<b>ISBT 128</b>	New standardized method for labeling blood units, that has replaced Codabar <i>It includes much more information within the barcodes than is possible with Codabar. Its rules are much more structured than Codabar. It is not backwards compatible with Codabar.</i>
<b>Leukopoor</b>	See "leukoreduce."
<b>Leukoreduce</b>	To remove white cells from a cellular product through filtration. The resulting product is considered "leukoreduced" or "leukopoor."
<b>ListServ</b>	An email distribution list to manage discussion groups.
<b>LMIP</b>	Laboratory Management Index Program (LMIP) is the name given to the functionality in Veterans Health Information and Technology Architecture (VistA) software designed to capture and tabulate both technical and professional workload as it relates to the performance of ordered/billable tests in VA laboratories.
<b>Managerial Cost Accounting</b>	MCA means the process of accumulating, measuring, analyzing, interpreting and reporting cost information useful to both internal and external groups concerned with the way the organization uses, accounts for, safeguards and controls its resources to meet its objectives. MCA assists budgeting, financial accounting and reporting and provides useful information to leadership.
<b>Medical device</b>	An instrument, apparatus, implement, machine, contrivance, implant, in vitro reagent, or other similar or related article, including a component part, or accessory that is intended for use in the diagnosis of disease or other conditions, or in the cure, mitigation, treatment, or prevention of disease.
<b>Modifier</b>	An ISBT 128 term that describes a characteristic of a donated unit of blood that distinguishes it from other members of the component class. <i>The VBECS application is concerned with the modifiers "Washed," "Thawed," "Liquid," "Frozen," "Rejuvenated," and "Deglycerolized."</i>

Acronym, Term	Definition
<b>Modifier (order)</b>	Component requirements associated with a specific component order. <i>Example: a component order where red blood cells must be leukoreduced. When a component order is accompanied by special requirements, units issued for transfusion must meet specific criteria that translate into “Modifiers” or “Attributes” in ISBT 128.</i>
<b>MSBOS</b>	Maximum Surgical Blood Ordering Schedule. The appropriate number of specific blood components to be ordered for selected surgeries at a facility. <i>Preoperative blood component orders are compared with this list and details of orders falling outside of these established criteria are recorded and reportable.</i>
<b>Multidivisional database</b>	More than one separate transfusion services share a database.
<b>Negative control</b>	The negative control cell is known to lack the antigen the antiserum type is to detect. This cell should demonstrate negative serologic reaction, and, therefore, have a valid negative interpretation.  The positive and negative control cells must demonstrate a valid positive or negative set of serologic reactions and interpretations for the test of the unit or patient to be considered valid, regardless of the row validation of the unit test itself. (See “Positive control.”)
<b>NFT</b>	Not fully tested.
<b>NLT</b>	National Laboratory Test code. Coding schema used by the Vista Laboratory software to assign numeric identifier code to specific tests. Also known as Workload Code (WKLD CODE).
<b>Order</b>	Request for a service. <i>The request may specify when the order is to be completed, e.g., immediately (STAT). VBECS acts on the order when it has been “accessioned.”</i>
<b>Original unit</b>	A unit before modification.
<b>Outdated</b>	Describes the state of a unit of blood that has passed the expiration date/time.
<b>Patch</b>	Software developed and released to correct a defect, or change or enhance existing functionality; usually a modified subset or small addition to existing software.
<b>Patient care encounter (PCE)</b>	A contact with a patient documented in the patient’s record.
<b>Pending Order List (POL)</b>	A list of orders transmitted from CPRS waiting to be processed.
<b>Pending Task List (PTL)</b>	A list of tasks to be processed in the blood bank.
<b>Phenotype</b>	Describes the presence or absence of specific antigens.
<b>Phenotyping</b>	Testing for the presence or absence of specific antigens. See “Antigen typing.”
<b>Pick list</b>	A system-generated list that allows a user to select one or more entries.
<b>Pool</b>	A blood product from many donors stored for later use.

Acronym, Term	Definition
<b>Positive control</b>	The positive control cell is known to have the antigen that the reagent antiserum type is to detect. This cell should demonstrate positive serologic reaction and, therefore, have a valid positive interpretation.  The positive and negative control cells must demonstrate a valid positive or negative set of serologic reactions and interpretations for the test of the unit or patient to be considered valid, regardless of the row validation of the unit test itself. (See “Negative control.”)
<b>Potential hazard</b>	An event that compromises the safety of the patient or blood supply.
<b>Pre-transfusion testing</b>	A group of tests, including ABO/Rh and an antibody screen, that must be performed on a sample of blood prior to crossmatching red cells for transfusion.
<b>Product</b>	Components, manufacturing materials, in-process devices, finished devices, and returned devices.
<b>Product type</b>	See “Component class.”
<b>Presumed transfused</b>	Unit status that VBECS assigns 48 hours after issue of a blood product.
<b>QC</b>	Quality control. Testing routinely performed to ensure the proper functioning of materials, equipment, and methods.
<b>Quality</b>	The totality of features and characteristics that affect the ability of a device to satisfy fitness-for-use, including safety and performance.
<b>Reagent rack (also “rack”)</b>	A group of reagents used in routine blood bank testing.
<b>Reagent red blood cells</b>	Non-patient testing materials, usually from a commercial source.
<b>Reflex order</b>	An order for additional testing to be performed resulting from observations of previous testing.
<b>Reflex test</b>	A laboratory test not ordered by a physician but performed in response to results of other laboratory tests.
<b>Rh (also “Rh factor”)</b>	Rhesus factor. The major antigen of the Rh system. Also “D.” <i>The presence of this antigen on red blood cells determines if a patient or donor is Rh Pos or Rh Neg. A weak expression of the D antigen, referred to as “Du” or “weak D,” may require additional testing to determine its presence.</i>
<b>SCD</b>	Sterile connection device.
<b>Screening cell (SC)</b>	Antigenically defined cells used to detect alloantibodies.
<b>SD</b>	Service Desk (formerly National Help Desk).
<b>Selected</b>	Status of a unit of red cells when the blood bank technician selects it from available inventory to be crossmatched with a patient specimen. <i>A unit in “selected” state is not available to be issued for transfusion. By entering a crossmatch result of “compatible,” the state changes from “selected” to “assigned.”</i>
<b>Serologic crossmatch</b>	Performance of a laboratory test to predict post-transfusion survival of transfused blood.
<b>Serum</b>	The liquid portion of blood used to detect antibodies.

Acronym, Term	Definition
<b>Server</b>	A computer system in a network shared by multiple users.
<b>SF 518 equivalent</b>	See “Blood Transfusion Reaction Form.”
<b>Site (see also “Facility”)</b>	A blood bank laboratory where a blood unit is located.
<b>SME</b>	Subject matter expert.
<b>Software</b>	Computer programs and associated documentation pertaining to the operation of a system.
<b>Special Instructions (SIs)</b>	Free-text or canned comments entered by the Blood Bank Technologist to communicate problems, restrictions, and considerations not otherwise documented in a patient’s record.
<b>Status</b>	The disposition of a unit of blood.
<b>Sterile connection device</b>	An instrument that connects two plastic tubes while maintaining sterility.
<b>SSRS</b>	SQL Server Reporting Services.
<b>Target unit</b>	A unit formed as a result of a pool or split.
<b>TAS</b>	Type & Screen.
<b>TCP/IP</b>	Transmission Control Protocol/Internet Protocol.
<b>Test (in general and laboratory contexts)</b>	A means of examination to determine a property. In the context of the VBECS database, the contents of a single grid entry (test cell).
<b>Test cell</b>	In the context of the VBECS database: A grid space. In a laboratory: Red cells, in a reagent vial, used in a test.
<b>Testing phase</b>	1. The portion of a laboratory test in which an interpretation is recorded. 2. The phase of the development process in which the functionality is available for testing.
<b>Transfusion Requirements (TRs)</b>	Mandatory blood product attributes applied to every transfusion.
<b>Truth table</b>	A table, specific to a test or action, listing all possible combinations of user inputs and the corresponding acceptable results and/or interpretations. It allows the system to verify user input based on established parameters.
<b>TRW</b>	Transfusion reaction workup.
<b>UID</b>	Universal ID. An eye-readable and barcode-scannable coded representation of the Accession Number assigned by the laboratory package. <i>UID is unique to the generating information system where the specimen was received.</i>
<b>Unit</b>	A single example of a blood component type. Example: a single unit of red cells, a unit of plasma, a unit of platelets.
<b>Unit ID</b>	The label identifier, consisting of a string of characters, of the unit of blood, barcoded on the blood bag label and scanned when a blood unit is received in the blood bank in a shipment.

Acronym, Term	Definition
<b>Unit status</b>	<p>The state of a unit of blood at any given time.</p> <ul style="list-style-type: none"> <li>• Available: Required processing is complete.</li> <li>• Crossmatched/Assigned: The unit is immediately available for transfusion. All required compatibility testing, including crossmatch if indicated, is performed.</li> <li>• Discarded: The unit has been destroyed and is no longer available.</li> <li>• Issued: The unit is not currently located in the blood bank.</li> <li>• Limited: Required processing is complete, except for ABO/Rh confirmation.</li> <li>• Quarantine: The unit cannot be selected.</li> <li>• Selected: The unit is reserved for one or more patients. Additional compatibility testing is required, such as antigen phenotype, crossmatch, and unit ABO/Rh confirmation.</li> <li>• Transferred: The unit has been returned to the original supplier or transferred to a different facility.</li> <li>• Transfused: The unit has been infused into a patient and is no longer available.</li> </ul>
<b>Validation</b>	Establishment by objective evidence that a specific process or device specification conforms to user needs and intended use.
<b>VDL</b>	VistA Documentation Library.
<b>Verification</b>	Confirmation by examination and provision of objective evidence that requirements were fulfilled.
<b>Verify Code</b>	A field in the VistA New Person file used to verify the identity of a user associated with an Access Code.
<b>Vial identifier</b>	A one- or two-character number that identifies a specific vial contained within a kit of reagents. <i>A vial contains antigen positive or antigen negative blood cells to be used in quality control of antisera. Also referred to as "cell identifier."</i>
<b>VISN</b>	Veterans Integrated Services Network.
<b>VistA</b>	Veterans Health Information Systems and Technology Architecture.
<b>VistALink</b>	A real-time communication link that provides connectivity and utilities for data exchange between VistA and VBECS.
<b>WAN</b>	Wide Area Network.
<b>Worklist</b>	A list detailing work to be performed.
<b>Workload</b>	Statistical data used to calculate productivity.
<b>XM</b>	Crossmatch.
<b>XML</b>	Extensible Markup Language.



# Appendices

## ***Appendix A: Downtime Forms and Instructions***

Each site establishes its own policies, procedures, and processes to address unexpected VBECS downtime issues such as:

- The interface is unavailable. It is recommended that blood bank staff notify clinicians when messaging between CPRS and VBECS fails during order completion: clinicians may need to know that blood units are ready for pickup and transfusion.
- Test order handling and completion when the serologic testing is performed off-site or in a non-VA facility. The CPRS test order must be processed and completed even when testing is done off-site.
- The physical handling of a blood unit when it cannot be logged into inventory because VistALink is down and the patient is not in the VBECS database. The unit does not exist in VBECS until it is logged in, but is physically available.
- Manual blood labeling, verification of labels, and tracking documentation of the downtime labeling.
- Some Special Instructions (SIs) that were moved to VBECS during the database conversion must be entered as Transfusion Requirements (TRs) in VBECS. Inactivate the old SI and create a TR and new updated SI: the database conversion software cannot automatically translate the SI from database conversion into a rule-based TR.
- Creating the Audit Trail, Exception, and Testing Worklist Reports.
- Mitigation of the risk of patient identification conflicts due to patient merge issues (changing patient name or ID). It is recommended that staff submit the patient name and ID in writing to the blood bank when picking up blood products.
- VBECS is unavailable. To ensure accurate entry of information when VBECS is running, users must record the unit number and ID on their downtime form.

It is recommended that each site have available:

- Antibody Workup Forms
- Antigen Typing Forms (for patient)
- Antigen Typing Forms (for unit)
- Blood Transfusion Record Forms
- Caution Tags
- Daily QC Forms
- Emergency Issue Forms: developed by each site to reflect site-specific emergency issue policies and procedures
- Reagent Receipt Log
- Supply Receipt Log
- Transfusion Reaction Forms
- Transfusion Requirements Report: current printed version must be available at all times for users to check patient historic records and Transfusion Requirements. (See Appendix I: Recommended Report Usage: Table 37: Recommended Report Usage.)

Each site may use and modify (included in this appendix):

- Patient Testing Forms
- Unit ABO/Rh Confirmation Forms

- Unit Issue and Inspection Logs
- Unit Modification Forms

The information on these forms matches information to be input in VBECS.

## Patient Testing Form

<b>Facility Name:</b>						
<b>Address:</b>						
<b>City, State, Zip Code:</b>						
Patient Name	Patient Identification Number	Location	CPRS Order Number	Lab Order Number	Urgency	Appropriate? (Yes, No)
Specimen UID	Date and Time Collected	Phlebotomist Name	Date and Time Received	Processed By:	Acceptable? (Yes, No)	
<b>Ordered Tests:</b>						
<b>Comments:</b>				<b>Entered By:</b>	<b>Date and Time:</b>	

Reagent Rack	Date and Time		Tech Name					
Historic ABO/Rh Check	Forward ABO Grouping			Rh Typing		Reverse ABO Grouping		ABO/Rh Interpretation
	Anti-A	Anti-B	Anti-A,B	Anti-D	Rh Control	A1 Cell	B Cell	

Antibody Screening																
Screen Cell 1				Screen Cell 2				Screen Cell 3				Screen Cell 4				Screen Interpretation
RT/IS <sup>5</sup>	37°	AHG <sup>6</sup>	CC <sup>7</sup>	RT/IS	37°	AHG	CC	RT/IS	37°	AHG	CC	RT/IS	37°	AHG	CC	
<b>Comments:</b>																

Patient Name		Patient Identification Number			Location		Specimen UID		
Compatibility Testing									
Unit Identification Number		Product Code	ABO/Rh	RT/IS	37°	AHG	CC	Interpretation	Tech Name

<sup>5</sup> Room Temperature/Immediate Spin

<sup>6</sup> Anti-Human Globulin

<sup>7</sup> Coated cell

Direct Antiglobulin Test					
Reagent Antiserum	Lot Number	AHG	CC	Interpretation	Tech Name
Polyspecific					
Anti-IgG					
Anti-C3d					

Direct Antiglobulin QC									
Reagent Antiserum	Lot Number	Expiration Date	Reagent Red Cells	Lot Number	Expiration Date	AHG	CC	Interpretation	Tech Name
Polyspecific									
Anti-IgG									
Anti-C3d									

Test Results: Positive, Graded = M, F, W, 1+, 2+, 3+, 4+; Negative = 0; NT = Not Tested









Unit(s) to Be Modified				
Unit Identification Number	Product Code	ABO/Rh	Container Lot Number	Expiration Date and Time

Unit Assigned? (Yes, No)	Patient Name	Patient Identification Number	Specimen UID

Modified Unit(s)				
Unit Identification Number	Product Code	ABO/Rh	Container Lot Number	Expiration Date and Time

This page intentionally left blank.

## ***Appendix B: Database Table Information***

### **Antibody and Antigen Tables**

#### **Antibody and Antigen Table: Irregular Antibodies**

VBECs creates a division-specific copy in each division in the database. Setting Antibody Parameters allows the division to reset the calculated antigen negative requirement of antibodies to its medical director's policy. Each orderable test name has a corresponding repeat test.

Clinical significance of antibody specificity is set using the 14th edition of the AABB Technical Manual. A specific instance of an antibody for certain patients or different treatment conditions may behave quite differently, requiring antigen negative or crossmatch-compatible blood for the patient. The user may edit these settings for an individual patient.

The antigen negative required settings apply to all instances of the antibody. Individual hospital policy may indicate that a setting of "OPT" should be made "REQ." A policy of a global OPT may remain in the event of a specific clinically significant instance of a specificity in a particular patient. The individual patient's antigen negative requirement may be set for the specificity in Entering and Removing Transfusion Requirements. In the Antigen Negative Requirement column:

- "REQ" sets an antigen negative requirement that is not editable because these are generally considered to require the provision of antigen negative blood for patient transfusion.
- "NO" does not set an antigen negative requirement and is not editable.
- "OPT" in the Antibody sets an antigen negative requirement is editable and may be reset repeatedly on the division's copy of the table.

**Table 14: Antibody and Antigen Table: Irregular Antibodies**

Antibody Name		Clinically Significant?§	Does Antibody Set an Antigen Negative Requirement?	Associated Antigen		Antigen Negative Compatibility Percentage	VBECS Antigen Typing Orderable Test
VBECS*	VistA†			VBECS	VistA†		
Anti-D	Anti-D	Yes	Yes	D	D	13	AGD
N/A	N/A	N/A	N/A	N/A	N/A	N/A	AGwD
Anti-C	Anti-C	Yes	Yes	C	C	53	AGC
Anti-E	Anti-E	Yes	Yes	E	E	75	AGE
Anti-c	Anti-c	Yes	Yes	c	c	12	AGc
Anti-e	Anti-e	Yes	Yes	e	e	2	AGe
Anti-f	Anti-f	Yes	Yes	c and e	f	N/A	AGc AGe
Anti-G	Anti-G	Yes	Yes	C and D	G	N/A	AGD AGC
Anti-K	Anti-K	Yes	Yes	K	K	91	AGK
Anti-k	Anti-k	Yes	Yes	k	k	1	AGk
Anti-Kp(a)	Anti-Kp(a)	Yes	Optional	Kp(a)	Kp(a)	98	AGKp(a)
Anti-Kp(b)	Anti-Kp(b)	Yes	Optional	Kp(b)	Kp(b)	2	AGKp(b)
Anti-Jk(a)	Anti-Jk(a)	Yes	Yes	Jk(a)	Jk(a)	15	AGJk(a)
Anti-Jk(b)	Anti-Jk(b)	Yes	Yes	Jk(b)	Jk(b)	42	AGJk(b)
Anti-Jk3	Anti-Jk(a)Jk(b)	Yes	Yes	Jk(a) and Jk(b)	None	N/A	AGJk(a) AGJk(b)
Anti-Fy(a)	Anti-Fy(a)	Yes	Yes	Fy(a)	Fy(a)	63	AGFy(a)
Anti-Fy(b)	Anti-Fy(b)	Yes	Yes	Fy(b)	Fy(b)	47	AGFy(b)
Anti-M	Anti-M	No	Optional	M	M	26	AGM
Anti-N	Anti-N	No	Optional	N	N	27	AGN
Anti-S	Anti-S	Yes	Yes	S	S	58	AGS
Anti-s	Anti-s	Yes	Yes	s	s	7	AGs
Anti-U	Anti-U	Yes	Optional	U	U	0	AGU
Anti-P1	Anti-P1	No	Optional	P1	P1	21	AGP1
Anti-Le(a)	Anti-Le(a)	No	Optional	Le(a)	Le(a)	78	AGLe(a)
Anti-Le(b)	Anti-Le(b)	No	Optional	Le(b)	Le(b)	37	AGLe(b)
Anti-Js(a)	Anti-Js(a)	Yes	Optional	Js(a)	Js(a)	90	AGJs(a)
Anti-Js(b)	Anti-Js(b)	Yes	Optional	Js(b)	Js(b)	10	AGJs(b)
Anti-Lu(a)	Anti-Lu(a)	No	Optional	Lu(a)	Lu(a)	92	AGLu(a)
Anti-Lu(b)	Anti-Lu(b)	No	Optional	Lu(b)	Lu(b)	2	AGLu(b)
Anti-Lu3	Anti-Lu(a)Lu(b)	No	Optional	Lu (a) and Lu(b)	None	N/A	AGLu(a) AGLu(b)
Anti-C(w)	Anti-Cw	Yes	Optional	C(w)	Cw	99	AGC(w)
Anti-Ce	Anti-Ce	Yes	Yes	C and e	Ce	N/A	AGC AGe
Anti-C(x)	Anti-Cx	Yes	Optional	C(x)	Cx	99	AGC(x)
Anti-E(w)	Anti-E(w)	Yes	Optional	E(w)	E(w)	99	AGE(w)
Anti-V	Anti-V	Yes	Optional	V	V	85	AGV
Anti-Xg(a)	Anti-Xg(a)	No	Optional	Xg(a)	Xg(a)	25	AGXg(a)
Anti-M(g)	Anti-M(g)	No	Optional	M(g)	M(g)	99	AGM(g)
Anti-Mi(a)	Anti-Mi(a)	No	Optional	Mi(a)	Mi(a)	99	AGMi(a)
Anti-P	Anti-P	No	Optional	P	P (not otherwise specified)	25	AGP
Anti-A1	Anti-A-1	No	Optional	A1	A-1	10	AGA1

Antibody Name		Clinically Significant?	Does Antibody Set an Antigen Negative Requirement?	Associated Antigen		Antigen Negative Compatibility Percentage	VBECS Antigen Typing Orderable Test
VBECS*	VistA†			VBECS	VistA†		
Anti-H	Anti-H	No	Optional	H	H	0	AGH
Anti-I	Anti-I	No	No	None	None	0	None
Anti-i	Anti-i	No	No	None	None	99	
Warm autoantibody	Warm autoantibody	Unknown§	No	None	None	0	
Cold autoantibody	Cold autoantibody	No	No	None	None	0	
Anti-Le(ab)	Anti-Le(x)	No	Optional	Le(a) and Le(b)	None	N/A	
Anti-Di(a)	Anti-Di(a)	No	Optional	Di(a)	Di(a)	0	AGDi(a)
Anti-Di(b)	Anti-Di(b)	No	Optional	Di(b)	Di(b)	99	AGDi(b)
Anti-Do(a)	Anti-Do(a)	No	Optional	Do(a)	Do(a)	33	AGDo(a)
Anti-Do(b)	None	No	Optional	Do(b)	None	17	AGDo(b)
Anti-Co(a)	None	No	Optional	Co(a)	None	33	AGCo(a)
Anti-Co(b)	None	No	Optional	Co(b)	None	17	AGCo(b)
Anti-LW(a)	None	No	Optional	LW(a)	None	0	AG LW(a)
Anti-LW(b)	None	No	Optional	LW(b)	None	99	AG LW(b)
Anti-Sc1	None	No	Optional	Sc1	None	0	AGSc1
Anti-Sc2	None	No	Optional	Sc2	None	99	AGSc2
Anti-Sd(a)	None	No	Optional	Sd(a)	None	1	AGSd(a)
Anti-Au(a)	Anti-Au(a)	No	Optional	Au(a)	Au(a)	20	AGAu(a)
Anti-Au(b)	None	No	Optional	Au(b)	None	50	AGAu(b)
Anti-Ch	None	No	Optional	Ch	None	2	AGCh
Anti-Rg	None	No	Optional	Rg	None	3	AGRg
Anti-Cs(a)	None	No	Optional	Cs(a)	None	4	AGCs(a)
Anti-Yk(a)	None	No	Optional	Yk(a)	None	4	AGYk(a)
Anti-Kn(a)	None	No	Optional	Kn(a)	None	1	AGKn(a)
Anti-McC(a)	None	No	Optional	McC(a)	None	1	AGMcC(a)
Anti-JMH	None	No	Optional	JMH	None	1	AGJMH
Anti-Yt(a)	None	No	Optional	Yt(a)	None	1	AGYt(a)
Anti-Yt(b)	None	No	Optional	Yt(b)	None	92	AGYt(b)
HTLA (probable)	None	No	No	None	None	0	None
Antibody, No Specificity Identified	Antibody (not otherwise specified)	Unknown§	No	None	None	0	
Antibody to Low-Incidence Antigen	Antibody, private (not otherwise specified)	Unknown§	No	None	None	99	

Antibody Name		Clinically Significant?§	Does Antibody Set an Antigen Negative Requirement?	Associated Antigen		Antigen Negative Compatibility Percentage	VBECS Antigen Typing Orderable Test
VBECS*	VistA†			VBECS	VistA†		
Antibody to High-Incidence Antigen	Antibody, public (not otherwise specified)	Unknown§	No	None	None	0	
Serologic Crossmatch Test Required	N/A	No	No	None	None	N/A	None

\*Specificities written as superscript or subscript letters in blood bank literature are shown as letters enclosed in parentheses in VBECS (the GUI cannot display superscript or subscript letters). Specificities written as subscript characters in blood bank literature are shown as numbers or letters not enclosed in parentheses.

†Examples of common VistA antibody and antigen terminology. Individual sites may vary. Refer to division-specific mapping documents.

‡The user will order the tests for each antigen separately; VBECS does not display a duplicate antigen typing for this combination antibody specificity.

§When the clinical significance is unknown, it is displayed as "No" due to the limitations of the system.

## Antibody and Antigen Table: Database Conversion

Users cannot set antigen negative requirements for these antibodies in VBECS (Table 15). Clinical significance of antibody specificity is set using the 14th edition of the AABB Technical Manual.

**Table 15: Antibody and Antigen Table: Database Conversion**

Antibody Name		Associated Antigen	
VBECS*	VistA†	VBECS*	VistA†
Anti-P1+P+P(k)	Anti-PP1	P1+P+P(k)	PP1
Anti-Pk	Anti-p(k)	P(k)	p(k)
Anti -M, other	Anti-M, other	None	None
Anti-LW	Anti-LW	None	LW
Anti-Lu	Anti-Lu	None	Lu
Anti-Le, other	Anti-Le, other	None	None
Anti-N, other	Anti-N, other	None	None
Anti-I(int)	Anti-I(int)	None	I(int)
Anti-rhesus (not otherwise specified)	Anti-rhesus (not otherwise specified)	None	None

\*Specificities written as superscript or subscript letters in blood bank literature are shown as letters enclosed in parentheses in VBECS (the GUI cannot display superscript or subscript letters). Specificities written as subscript characters in blood bank literature are shown as numbers or letters not enclosed in parentheses.

†Examples of common VistA antibody and antigen terminology. Individual sites may vary. Refer to division-specific mapping documents.

## Antibody Screen Test Interpretation

For an Antibody Screen Test (ABS) to be correctly interpreted, the system must take the row validation (Table 16) for each screening cell and test method and determine whether the screening in total is positive or negative. This table is the input from the Blood Bank Technologist. Interpreting the ABS: The Blood Bank Technologist does not enter an interpretation for each row/cell of the antibody screen. The technologist enters an interpretation for the combination of screening cells including the autocontrol when tested. The system must correlate the information from the rows in the test, as described in system rules , to determine the proper interpretation for the ABS. Valid interpretations are positive or negative. A technologist may enter an antibody screen comment with the interpretation.

- SR 1: Valid interpretation. Update database.
- SR 2: Warning message. No override. Do not update database at this time.
- SR 3: Valid interpretation. Update database. Collect comment and exception.
- EM 1: “Interpretation does not match your results.”
- EM 2: “You are attempting to file an invalid antibody screen. This is not allowed.”

**Table 16: Antibody Screen Test Interpretation**

Antibody Screen Test Interpretation					
Antibody Screen Interpretation: includes all cells tested; might be SC 1 and SC 2 only, or SC 1, 2, 3, or SC 1, 2, 3, 4, as indicated by site parameter in Configure Daily QC. Method of Tube, Gel, or Solid Phase	SC 1, SC 2, SC 3, SC 4, Autocontrol Interpretation	Tech Entry	System Interpretation of Screening Cell Test	System Response	Error Message
	System interpretation if all individual row validations are negative.	Negative	Negative (N)	SR 1	N/A
		Positive	Negative (N)	SR 2	EM 1
	System interpretation if all individual row validations are negative and indicates that a comment is required.	Negative	Negative (N)	SR 3	N/A
	System interpretation if one, several, or all rows are positive (P).	Positive	Positive (P)	SR 1	N/A
		Negative	Positive (P)	SR 2	EM 1
	System interpretation if one, several, or all rows are positive (P) and indicates that a comment is required.	Positive	Positive (P)	SR 3	N/A
	System interpretation if one, several, or all rows are invalid (I).	Positive or Negative	Invalid (I)	SR 2	EM2

## Canned Comment Category Types and Text

Canned comments are defined comments that can be inserted in a record during execution of a variety of functions. The canned comments available in a function are restricted to the category type associated with that function. Table 17 lists the category types available when defining canned comments.

**Table 17: Canned Comment Category Types and Message Text**

Canned Comment Category Type	Message Text
ABO Incompatible Issue	Blood shortage Issued to ER Massive transfusion Trauma Other
Automated Testing Review	Control NEG Control POS Post-transfusion reaction Remote Data Record reviewed, history Remote Data Record reviewed, no history Results confirmed with tube testing Rh (D) control negative Other
Crossmatch	Other
Incoming Shipment	Prolonged storage outside of monitored storage area Spiked or damaged Unit returned unused Visual inspection unsatisfactory Other
Justify ABO/Rh Change	Other
Justify ABO/Rh Discrepancy	ABO inconclusive Massive transfusion Other
Order Cancellation	Inappropriate blood component ordered Inappropriate test ordered Max time for active orders exceeded Patient deceased Patient discharged or transferred Patient refuses treatment Patient unavailable Other
Order Processing	Verbal order Other
Parameter Setting Change	No longer applies Resetting due to policy change Other
Patient Testing	Lab equipment failure New lot numbers in use New specimen required Specimen ruined during test process Test cancelled Test complete, results inconclusive Test compromised Testing inconclusive Testing interrupted Other



Canned Comment Category Type	Message Text
Quarantine/Discard	ABO inconclusive Disposition for research Expired Incinerated Incomplete donor screening tests Positive donor screening test(s) Prolonged storage outside of monitored storage area Punctured Ruptured during thaw Visual inspection unsatisfactory Other
Reagent and Supply	Broken upon receipt Contaminated Improperly stored Product shortage Spilled Unsatisfactory, return to supplier Updated inventory Other
Release from Quarantine	Other
Reason for Release	24-hour post-op Biohazardous Counts or levels exceed transfusion trigger For use by another patient Order cancelled Patient deceased Patient discharged or transferred Specimen expired Transfusion reaction Unit expired Other
Special Instruction Inactivation	No longer applies Other
Special Instructions	Check the patient's blood counts before selecting units, use often exceeds original order Patient has rouleaux; saline replacement crossmatches are needed Split units for transfusion today Volume reduce platelet pools over 300 mL
Specimen Extension	Pre-op Workup Surgery rescheduled Other
Specimen General	Patient eligible for specimen extension Specimen recollected Other
Specimen Processing	Patient eligible for specimen extension Specimen misplaced Specimen recollected Other

<b>Canned Comment Category Type</b>	<b>Message Text</b>
Specimen Unacceptable	Broken during processing Incomplete label Incomplete or unreadable Phlebotomist ID Incorrectly labeled No phlebotomist ID QNS (quantity not sufficient) Specimen and paperwork do not match Specimen hemolyzed Specimen misplaced Specimen not received Wrong specimen type collected Other
Transfusion Processing	Other
Transfusion Reaction Workup	Other
Specimen Required	Specimen and paperwork do not match Other
Pre-Transfusion Specimen Checks	Incomplete label Incorrectly labeled Specimen and paperwork do not match Specimen misplaced Other
Post-Transfusion Specimen Checks	Incomplete label Incorrectly labeled Specimen and paperwork do not match Specimen misplaced Other
Implicated Units	Biohazardous No longer applies Transfusion reaction Other
Symptom Details	Other
Transfusion Blood Bag Not Returned	Other
Transfusion Clerical Checks Failed	Other
Transfusion Requirement Inactivation	Clerical error No longer applied Order cancelled Typographical error Other
Unit Inactivation	Clerical error Typographical error Other
Unit Data Invalidation	Clerical error Typographical error Other
Unit Issue	Blood shortage Critical need Incomplete donor screening tests Issued to ER Massive transfusion Positive donor screening test(s) Remote storage cooler A Trauma Visual inspection unsatisfactory Other

<b>Canned Comment Category Type</b>	<b>Message Text</b>
Unit Modification	Contaminated Punctured Ruptured during thaw Other
Unit Return	Blood shortage Broken during processing Broken upon receipt Expired Improperly stored Massive transfusion Patient deceased Patient discharged or transferred Patient refuses treatment Patient unavailable Punctured Ruptured during thaw Spiked or damaged Surgery rescheduled Trauma Unsatisfactory, return to supplier Other
Unit Restriction	No longer applied Other
Unit Selection	Blood shortage Critical need Massive transfusion Rh mismatch selection necessary Trauma Other
Unit Status Removal	Other
Unit Testing	Incomplete donor screening tests Lab equipment failure Positive donor screening test(s) Unit test cancelled Other

Table 18 is a partial list of where comments appear in the options based on the requirements documents. The comment list that appears in the “Reason for Change” field in Update Reagent Inventory is a fixed list and is not locally editable.

**Table 18: Canned Comment Category vs Option**

Canned Comment Category	Option Appears In
ABO Incompatible Issue	ABO Incompatible Unit Transfused override during Issue Unit
Automated Testing Review	Automated Testing
Crossmatch	“G” (Give with Medical Director Approval) compatibility Override
Incoming Shipment	Expired Unit Received override
Justify ABO/Rh Change	No use identified at this time.
Justify ABO/Rh Discrepancy	Override to accept a real blood type when the testing is Inconclusive with a previous blood type on file.
Order Cancellation	Accept Order, Patient Testing Task List
Order Processing	Accept Order, Patient Testing Task List
Parameter Setting Change	All the configure division options
Patient Testing	All patient test grid comment boxes including crossmatch test and ABID comment, TRW Serologic test entry. Invalidate patient testing in grid and option. Discrepant Repeat ABO/Rh Typing override.
Quarantine/Discard	Discard, Quarantine
Reagent and Supply	Enter Reagents, when unacceptable, when unacceptable, QC overrides
Release from Quarantine	Release from Quarantine.
Reason for Release	Release unit from assignment
Special Instruction Inactivation	SI when inactivating an SI entry.
Special Instructions	SI actual message is considered a canned comment.
Specimen Extension	Maintain specimen
Specimen General	Accept Order
Specimen Processing	Accept Order
Specimen Unacceptable	Accept Order
Transfusion Processing	Post transfusion data
Transfusion Reaction Workup	TRW Finalization Canned Comment
Specimen Required	No use identified at this time.
Pre-Transfusion Specimen Checks	TRW when pre is marked not available
Post-Transfusion Specimen Checks	TRW when post is marked not available
Implicated Units	No use identified at this time.
Symptom Details	No use identified at this time.
Transfusion Blood Bag Not Returned	TRW when an implicated unit is entered and marked as not returned.
Transfusion Clerical Checks Failed	TRW when specimen is marked available TRW when an implicated unit is entered and marked as returned
Transfusion Requirement Inactivation	TR tabs, including antigen negative requirement, Transfusion requirement and antibody identified tabs.
Unit Inactivation	Edit Unit Information.
Unit Data Invalidation	No use identified at this time.
Unit Issue	Unit Issue Override for TR not satisfied.
Unit Modification	Any modification override
Unit Return	Return unit from Issue when unit is marked unsatisfactory
Unit Restriction	No use identified at this time.
Unit Selection	Select Unit
Unit Status Removal	Remove Final Status, Inactivate unit
Unit Testing	ABO/RH confirmation and Unit Antigen Typing Testing Edit Unit Information (Invalidate test results and unit inactivation)

## CPRS Orderable Blood Components (Component Classes) Mapped to ICCBBA Component Classes

Table 19 is used to map the CPRS/VistA orderables available to clinicians to the ICCBBA component classes.

**Table 19: CPRS Orderable Blood Components (Component Classes) Mapped to ICCBBA Component Classes**

CPRS Orderable Item	Blood Product Component Class
WHOLE BLOOD	RED BLOOD CELLS
WHOLE BLOOD	WHOLE BLOOD
RED BLOOD CELLS	RED BLOOD CELLS
RED BLOOD CELLS	WHOLE BLOOD
FRESH FROZEN PLASMA	FRESH FROZEN PLASMA
PLATELETS	PLATELETS
CRYOPRECIPITATE	CRYOPRECIPITATE
OTHER	OTHER

## Details in Audit Trail Report

Changes include the date and time of the change, ID of the user who made the change, and required comments (Table 20).

**Table 20: Details in Audit Trail Report**

Report Section	Generating Options	GUI Fields Monitored	Accessible in:	
			Unit History Report?	Patient History Report?
Blood Unit Changes	Modify Units: Pool Units (when a previously pooled unit is edited)	For each pooled unit edited (unit ID, long name, product code): <ul style="list-style-type: none"> <li>Added or Removed Unit IDs and Product Codes</li> <li>Comment</li> </ul>	Yes	No
	Editing A Unit	For each unit edited (unit ID, long name, product code): <ul style="list-style-type: none"> <li>Unit Antigen Information</li> <li>Special Testing Information</li> <li>Discarded Plasma Volume</li> <li>Unit Volume</li> <li>Biohazardous Indicator</li> <li>Unit ABO/Rh Confirmation Test, Invalidation Indicator</li> <li>Unit Antigen Typing Test, Invalidation Indicator</li> <li>Unit Record Inactivation Indicator</li> <li>Restricted For Patient Name and Patient ID</li> <li>Comment</li> </ul>	Yes	No
Blood Unit Financial Changes	Edit Financial Data	For each unit edited (unit ID, long name, product code): <ul style="list-style-type: none"> <li>Unit Base Cost</li> <li>Special Test Cost</li> </ul>	Yes	No

Report Section	Generating Options	GUI Fields Monitored	Accessible in:	
			Unit History Report?	Patient History Report?
		<ul style="list-style-type: none"> <li>Return Credit</li> </ul>		
Patient Testing Changes	Invalidate Test Results	For each patient (name, ID): <ul style="list-style-type: none"> <li>Ordered Test Original Results, Invalidation Indicator</li> <li>Comment</li> </ul>	No	Yes
	Justify ABO/Rh Change	For each patient (name, ID): <ul style="list-style-type: none"> <li>Patient ABO/Rh</li> <li>Comment</li> </ul>	No	Yes
Patient Transfusion Requirements	Special Instructions & Transfusion Requirements: Enter a Transfusion Requirement Patient Testing: Record Patient Test Results (new TR created from ABID data entry)	For each Transfusion Requirement category or Special Instruction: <ul style="list-style-type: none"> <li>Status Indicator</li> <li>Comment</li> </ul>	No	Yes
Maintenance: Local Facilities	Local Facilities	For each facility name and FDA Registration Number: <ul style="list-style-type: none"> <li>Eye-readable prefix</li> <li>ICCBBA Registration Number</li> <li>Alpha Characters Indicator</li> <li>Active Facility Indicator</li> <li>Collection Facility Indicator</li> <li>Testing Facility Indicator</li> <li>Address Line 1</li> <li>Address Line2</li> <li>Address Line 3</li> <li>City</li> <li>State</li> <li>Zip</li> <li>Phone</li> <li>Fax</li> </ul>	No	No
Maintenance: Blood Product	Blood Products	For each blood product (name, code): <ul style="list-style-type: none"> <li>HCPCS Code and Text</li> </ul> For each shipper of this product code (name, ID): <ul style="list-style-type: none"> <li>Cost</li> <li>Return Credit Percentage</li> <li>Status Indicator</li> </ul>	No	No
Maintenance: Division Configuration	Configure Division	For each Division (Name and Code): <ul style="list-style-type: none"> <li>Full Service, Transfusion Only Indicator</li> <li>Facility Name</li> <li>Active Status</li> <li>Full-Face-Label Printer Status</li> <li>Full-Face-Label Printer Port Number</li> <li>Full-Face-Label Printer IP address</li> <li>Full-Face-Label Printer COM</li> <li>Report Printer Name</li> </ul>	No	No

Report Section	Generating Options	GUI Fields Monitored	Accessible in:	
			Unit History Report?	Patient History Report?
		<ul style="list-style-type: none"> <li>• Accession Area Name</li> <li>• Lock Inactivity Timeout Setting</li> <li>• Time Zone Setting</li> <li>• Daylight savings time Setting</li> <li>• Daylight savings time start date</li> <li>• Daylight savings time end date</li> </ul> For each Associated Institution (Division Code) <ul style="list-style-type: none"> <li>• Status indicator</li> </ul>		
Maintenance: Division Configuration	Configure Division	For each division (name, code): <ul style="list-style-type: none"> <li>• Electronic Crossmatch Enabled Indicator</li> <li>• ICCBBA Registration Number</li> <li>• Maximum Specimen Expiration Days</li> <li>• Blood Bank MD</li> </ul>	No	No
Maintenance: Division Configuration	Edit Invoice Text	For each division (name, code): <ul style="list-style-type: none"> <li>• Invoice Text</li> </ul>	No	No
Maintenance: Division Configuration	Configure Testing	For each division (name, code): <ul style="list-style-type: none"> <li>• Antibody Screen Indicator</li> <li>• ABS and XM Testing Phases</li> </ul>	No	No
Maintenance: Division Configuration	Order Alerts	For each division (name, code): <ul style="list-style-type: none"> <li>• Login Message</li> </ul>	No	No
Maintenance: Modifications Configuration	Product Modifications	For each modification type: <ul style="list-style-type: none"> <li>• Enabled or Disabled Indicator</li> <li>• Cost</li> </ul>	No	No
Maintenance: User Role Changes	Update User Roles	For each user: <ul style="list-style-type: none"> <li>• User Role Security Setting</li> </ul>	No	No
Maintenance: User Role Changes	Configure Users	For each NT user (name, ID): <ul style="list-style-type: none"> <li>• VistA DUZ</li> <li>• VistA User Name</li> <li>• Email Address</li> <li>• User Initials</li> <li>• Active VBECS User Indicator</li> <li>• Active Division Indicator</li> </ul>	No	No
Maintenance: Antibody Configuration	Antibodies	For each antibody specificity: <ul style="list-style-type: none"> <li>• Compatibility Percentage</li> <li>• Higher Level Override Required Indicator</li> </ul>	No	No
Maintenance: Workload Configuration	Workload Codes	For each workload process and for each LMIP procedure and LMIP/NLT code: <ul style="list-style-type: none"> <li>• Start Date</li> <li>• Stop Date</li> <li>• Status Indicator</li> </ul>	No	No
Maintenance: MSBOS Configuration	MSBOS	For each surgery name: <ul style="list-style-type: none"> <li>• Surgery Status Indicator</li> </ul> For each MSBOS recommendation: <ul style="list-style-type: none"> <li>• TAS Indicator</li> <li>• No Blood Required Indicator</li> </ul> For each selected component class:	No	No

Report Section	Generating Options	GUI Fields Monitored	Accessible in:	
			Unit History Report?	Patient History Report?
		<ul style="list-style-type: none"> <li>• Number of Units</li> <li>• Recommendation Status Indicator</li> </ul>		
Maintenance: Transfusion Effectiveness Changes	Transfusion Effectiveness	For each Laboratory test name, for each VistA Laboratory test Internal Entry Number (IEN), and for each specimen type: <ul style="list-style-type: none"> <li>• Status Indicator</li> </ul>	No	No
Maintenance: Transfusion Complications Changes	Transfusion Complications	For each Laboratory test name, for each VistA Laboratory test IEN, and for each specimen type: <ul style="list-style-type: none"> <li>• Threshold Result</li> <li>• Threshold Setting Status</li> <li>• Status indicator</li> </ul>	No	No
Maintenance: Component Class	Component Classes	For each component class configured: <ul style="list-style-type: none"> <li>• Current ABO/Rh Specimen Results Required Indicator</li> <li>• Maximum Transfusion Time</li> <li>• For each Laboratory Test Name, for each VistA Laboratory Test Internal Entry Number (IEN), and for each Specimen Type: Lab Test Status</li> <li>• Threshold Result</li> </ul>	No	No
Maintenance: Daily QC Configuration	Configure Daily QC	For each daily QC setup changed: <ul style="list-style-type: none"> <li>• Commercial or Non-Commercial QC Indicator</li> <li>• Rack Name Indicator (A through Z or 1 through 26)</li> <li>• Number of Racks to Test</li> <li>• Daily Alert Time</li> <li>• Primary Enhancement Media</li> <li>• Secondary Enhancement Media</li> <li>• Comment</li> </ul> For each reagent type and test with combination: <ul style="list-style-type: none"> <li>• Status Indicator</li> <li>• Minimum Reaction Entry</li> </ul>	No	No
Maintenance: Minimum Reagent Inventory Levels	Maintain Minimum Levels	For each reagent type: <ul style="list-style-type: none"> <li>• Minimum Inventory Level</li> </ul>	No	No
Maintenance: User Alert Configuration	Order Alerts	For each division (name, code): <ul style="list-style-type: none"> <li>• Printer Alerts On</li> <li>• Printer Name</li> <li>• (Printer) Order Alert Type</li> <li>• (Printer) Orders Needed In</li> <li>• Icon Alerts On</li> <li>• Refresh Rate</li> <li>• (Icon) Order Alert Type</li> <li>• (Icon) Orders Needed In</li> </ul>	No	No



Report Section	Generating Options	GUI Fields Monitored	Accessible in:	
			Unit History Report?	Patient History Report?
Maintenance: Interface Control	Configure Interfaces	<ul style="list-style-type: none"> <li>• VistA IP Address or Server Name</li> <li>• VistA Port Number</li> <li>• VBECS VistALink Service IP Address or Server Name</li> <li>• VBECS VistALink Service Port Number</li> <li>• Broker IP Address or Server Name</li> <li>• Broker Port Number</li> </ul>	No	No
Maintenance: Interface Control	Configure Interfaces	<p>For each interface defined:</p> <ul style="list-style-type: none"> <li>• Interface IP Address or Server Name</li> <li>• Interface Port Number</li> <li>• Interface Facility ID</li> <li>• Interface Status</li> </ul> <p>Common Interface settings</p> <ul style="list-style-type: none"> <li>• VBECS IP Address or Server Name</li> <li>• VBECS Port Number</li> <li>• Ack Timeout</li> <li>• Retransmit Attempts</li> <li>• Interface Administrator Email</li> </ul>	No	No

## Details in Exception Report

Table 21 lists exception types and the reports in which they appear.

**Table 21: Details in Exception Report**

Exception Type	Available to Role	Available in Option	Accessible in:	
			Unit History Report?	Patient History Report?
ABO discrepant unit issue	All	Issue Unit	No	Yes
ABO incompatible unit selected	None	Select Unit	No	Yes
ABO Incompatible Unit Transfused	≥ Traditional Supervisor	Document ABO Incompatible Transfusion	No	Yes
Antigen positive unit crossmatched	≥ Lead Tech	Select Unit (when crossmatched)	No	Yes
Antigen positive unit selected	≥ Lead Tech	Select Unit	No	Yes
Antigen testing phase change	All	Unit or Patient Antigen typing	Yes (Antigen Typing)	Yes (Patient Testing: Record a Patient Antigen Typing)
Antigen-positive/untested units issued	≥ Lead Tech	Issue Blood	No	Yes
Biohazardous unit issued	All	Issue Blood	No	Yes
Biohazardous unit modified	All	Modification	Yes	No
Biohazardous unit selected	All	Select Unit	No	Yes
Caution Tag or Blood Transfusion Record Form printed for antigen positive or untested unit	≥ Lead Tech		No	Yes
Caution Tag or Blood Transfusion Record Form printed on expired specimen	≥ Enhanced Tech		No	Yes
Caution Tag or Blood Transfusion Record Form printed on expired unit	≥ Enhanced Tech		No	Yes
Caution Tag or Blood Transfusion Record Form printed with unsatisfied Transfusion Requirement	≥ Lead Tech		No	Yes
Crossmatch incompatible: Give only with MD approval	All		No	Yes
Deletion of partially completed QC	All	Enter Daily QC	No	No
Discrepant ABO/Rh override	≥ Enhanced Tech	Patient Testing	No	Yes
Discrepant patient antigen typing	All	Patient Testing	No	Yes
Electronic Transfusion Record Update	BCE COTS system	Post –Transfusion Information	No	No
Emergency issue request (no specimen)	All	Accept Order	No	Yes
Emergency issue, require testing incomplete	All	Patient Testing	No	Yes

Exception Type	Available to Role	Available in Option	Accessible in:	
			Unit History Report?	Patient History Report?
Expired antisera used	All	Enter Rack QC, Patient and Unit Antigen typing	Yes (Antigen Typing)	Yes (Patient Testing: Record a Patient Antigen Typing)
Expired reagent QC'd	All	Enter Daily QC	No	No
Expired supply used in modification	All	Modify Units	No	No
Expired task processed	All	Pending Task List	No	Yes
Expired unit modified	≥ Enhanced Tech	Modify Units	Yes	No
Expired unit received	All	Incoming Shipment	Yes	No
Expired unit released from quarantine	≥ Enhanced Tech	Release from Quarantine	Yes	No
Expired unit selected	≥ Enhanced Tech	Select Unit	No	Yes
Final unit status or modification removed	≥ Traditional Supervisor	Remove Final Status	Yes	No
Inconclusive crossmatch	All	Patient Testing, serologic crossmatch grid	No	Yes
Issued expired unit	≥ Enhanced Tech	Issue Blood	No	Yes
Less restrictive unit issued	All	Issue Blood	No	Yes
Less restrictive unit selected	All	Select Unit	No	Yes
Non-standard test procedure	All	Patient Testing, Antibody screen, serologic crossmatch	No	Yes
(Patient) ABO/Rh discrepancy	All	Patient testing, ABO/Rh test	No	Yes
Previously recorded results inactivated	All	All testing grids where the red x is used to clear a grid.	No	Yes
Previously recorded results invalidated	All	Invalidate Patient Testing	Yes	No
QC decrease reagent reactivity ≥2	All	All testing where Rack is selected	No	No
QC not performed on rack used for testing	All	All testing where Rack is selected	No	Yes
Reflex test ordered on expired specimen	All	Order Reflex Test	No	Yes
Released patient restriction	≥ Enhanced Tech	Free Directed Donor	No	Yes
Retrospective update prior to last record update	≥ Lead Tech	Discard/Quarantine	Yes	No
Rh positive units selected	All	Select Units	No	Yes
Specimen expiration date extended	All	Maintain Specimen	No	Yes
Target product label failed checks	All	All modification processes	Yes	No
Target product outdate extended	≥ Enhanced Tech	All modification processes	Yes	No
Transfusion Reaction Workup	All	Transfusion Reaction Workup	No	Yes
Transfusion Requirement incompatible unit selected	≥ Lead Tech	Select Unit	No	Yes

Exception Type	Available to Role	Available in Option	Accessible in:	
			Unit History Report?	Patient History Report?
Unacceptable/expired specimen used	All	Patient Testing	No	Yes
Unit ABO/Rh confirmation inconclusive, unit quarantined	All	ABO/Rh Confirmation	Yes	No
Unit ABO/Rh log-in vs. confirmation do not match, unit quarantined	All	ABO/Rh Confirmation	Yes	No
Unit discarded	All	Discard	Yes	No
Unit ER issued, testing problem	All	Patient testing, unit antigen typing	No	Yes
Unit issued on expired specimen	≥ Enhanced Tech	Issue Unit	No	Yes
Unit issued with unsatisfied Transfusion Requirement	≥ Lead Tech	Issue Unit	No	Yes
Unit quarantined	All	Quarantine	Yes	No
Unit released from quarantine	≥ Enhanced Tech	Release Unit from Quarantine	Yes	No
Unit unsatisfactory upon return from issue	All	Return Issued Units to Blood Bank	Yes (quarantined)	No
			Yes (quarantined with patient assignment)	Yes
Unit out of controlled storage found acceptable	All	Return Issued Units to Blood Bank	Yes	No

## Allowable Product Modifications by Original Product Type

ICCBBA provides a comprehensive database with every known blood product type that includes a five-digit product code, Product Type classification, and the Product Attribute for each blood product.

### Calculation of the Expiration Date

VBECS calculates the expiration date of the new unit based on system rules specific to the blood product and modification types, and on whether the system is open or closed. The user may accept or edit this expiration date.

Table 22 shows which kind of modifications are allowed for a specific product type. The first column contains product types; the second set of columns contains allowable modifications. Blank cells indicate that a modification type cannot be performed on an original product type. Modifications are not allowed for product types not listed.

**Table 22: Allowable Product Modifications by Original Product Type**

Product Type Name	Split/Divide	Thaw	Pool	Thaw/Pool	Irradiate	Leuko-reduce	Volume Reduce	Wash	Rejuvenate	Freeze	Deglycerolize
WHOLE BLOOD	X				X	X	X	X	X	X	
RED BLOOD CELLS	X				X	X	X	X	X	X	
Washed RED BLOOD CELLS	X				X	X			X	X	
Frozen RED BLOOD CELLS											X
Frozen Rejuvenated RED BLOOD CELLS											X
Deglycerolized RED BLOOD CELLS	X				X	X					
Deglycerolized Rejuvenated RED BLOOD CELLS	X				X	X					
Rejuvenated RED BLOOD CELLS	X				X	X				X	
APHERESIS RED BLOOD CELLS	X				X	X		X	X	X	
FRESH FROZEN PLASMA		X									
Thawed FRESH FROZEN PLASMA	X				X	X					

Product Type Name	Split/Divide	Thaw	Pool	Thaw/Pool	Irradiate	Leuko-reduce	Volume Reduce	Wash	Rejuvenate	Freeze	Deglycerolize
APHERESIS FRESH FROZEN PLASMA		X		X							
Thawed APHERESIS FRESH FROZEN PLASMA	X		X		X	X					
APHERESIS PLASMA		X		X							
Thawed APHERESIS PLASMA	X		X		X	X					
Liquid PLASMA	X				X	X					
PLASMA		X									
Thawed PLASMA	X				X	X					
PLATELET-RICH PLASMA	X										
PLATELETS	X		X		X	X		X			
Washed PLATELETS	X		X		X	X					
POOLED PLATELETS			X		X	X		X			
Washed POOLED PLATELETS			X		X	X					
APHERESIS PLATELETS	X				X	X		X			
Frozen APHERESIS PLATELETS		X									
Thawed APHERESIS PLATELETS	X				X	X					
Washed APHERESIS PLATELETS	X				X	X					
CRYOPRECIPITATE		X		X							
Thawed CRYOPRECIPITATE	X		X		X						
POOLED CRYOPRECIPITATE		X									

Product Type Name	Split/Divide	Thaw	Pool	Thaw/Pool	Irradiate	Leuko-reduce	Volume Reduce	Wash	Rejuvenate	Freeze	Deglycerolize
Thawed POOLED CRYOPRECIPITATE			X		X						
APHERESIS CRYOPRECIPITATE		X									
Thawed APHERESIS CRYOPRECIPITATE	X				X						
GRANULOCYTES	X		X		X						
APHERESIS GRANULOCYTES	X				X						
POOLED GRANULOCYTES			X		X						
APHERESIS GRANULOCYTES – PLATELETS	X				X						
LEUKOCYTES	X				X						
APHERESIS LEUKOCYTES	X				X						
POOLED PLASMA		X			X						
Liquid APHERESIS PLASMA	X				X						
PLATELET - RICH BUFFY COAT	X				X						
POOLED PLATELET - RICH BUFFY COAT					X						
APHERESIS LYMPHOCYTES	X				X						
APHERESIS MONOCYTES*	X				X						
SERUM											
POOLED SERUM**											

Product Type Name	Split/Divide	Thaw	Pool	Thaw/Pool	Irradiate	Leuko-reduce	Volume Reduce	Wash	Rejuvenate	Freeze	Deglycerolize
Frozen POOLED SERUM											
Washed APHERESIS RED BLOOD CELLS	X				X	X			X		
Frozen APHERESIS RED BLOOD CELLS											X
Deglycerolized APHERESIS RED BLOOD CELLS	X				X	X			X		
Rejuvenated APHERESIS RED BLOOD CELLS	X				X	X				X	
Frozen Rejuvenated APHERESIS RED BLOOD CELLS											X
Deglycerolized Rejuvenated APHERESIS RED BLOOD CELLS	X				X	X					
Thawed POOLED PLASMA					X	X					
Thawed POOLED FRESH FROZEN PLASMA					X	X					
Washed GRANULOCYTES	X				X						
Liquid POOLED PLASMA					X	X					
POOLED FRESH FROZEN PLASMA		X									
Frozen POOLED PLATELETS		X									




Product Type Name	Split/Divide	Thaw	Pool	Thaw/Pool	Irradiate	Leuko-reduce	Volume Reduce	Wash	Rejuvenate	Freeze	Deglycerolize
Thawed POOLED PLATELETS					X	X					
Thawed Apheresis POOLED FRESH FROZEN PLASMA					X	X					
Thawed Apheresis POOLED PLASMA					X	X					
Washed Thawed POOLED PLATELETS											
Washed Thawed Apheresis PLATELETS											
Lyophilized FRESH FROZEN PLASMA											
Reconstituted FRESH FROZEN PLASMA											
Washed Apheresis POOLED PLATELETS											
Frozen PLATELETS											
Apheresis POOLED CRYOPRECIPITATE											
Thawed Apheresis POOLED CRYOPRECIPITATE											
CONVALESCENT PLASMA											
PLATELET LYSATE											
Lyophilized POOLED PLASMA											

Product Type Name	Split/Divide	Thaw	Pool	Thaw/Pool	Irradiate	Leuko-reduce	Volume Reduce	Wash	Rejuvenate	Freeze	Deglycerolize
Lyophilized PLATELET-RICH PLASMA											
Frozen SERUM											
Lyophilized Apheresis FRESH FROZEN PLASMA											
IMMUNE PLASMA											
Apheresis IMMUNE PLASMA											
Apheresis CONVALESCENT PLASMA											
Reconstituted POOLED FRESH FROZEN PLASMA											
SOLVENT DETERGENT POOLED PLASMA											
Thawed SOLVENT DETERGENT POOLED PLASMA											
RADIOLABELLED DIAGNOSTICS, LEUKOCYTES											
RADIOLABELLED DIAGNOSTICS, PLATELETS											
RADIOLABELLED DIAGNOSTICS, RBC											

\*\*This product type has no ICCBBA assigned product codes.

# Order Status Flowchart

Figure 147: Order Status Flowchart

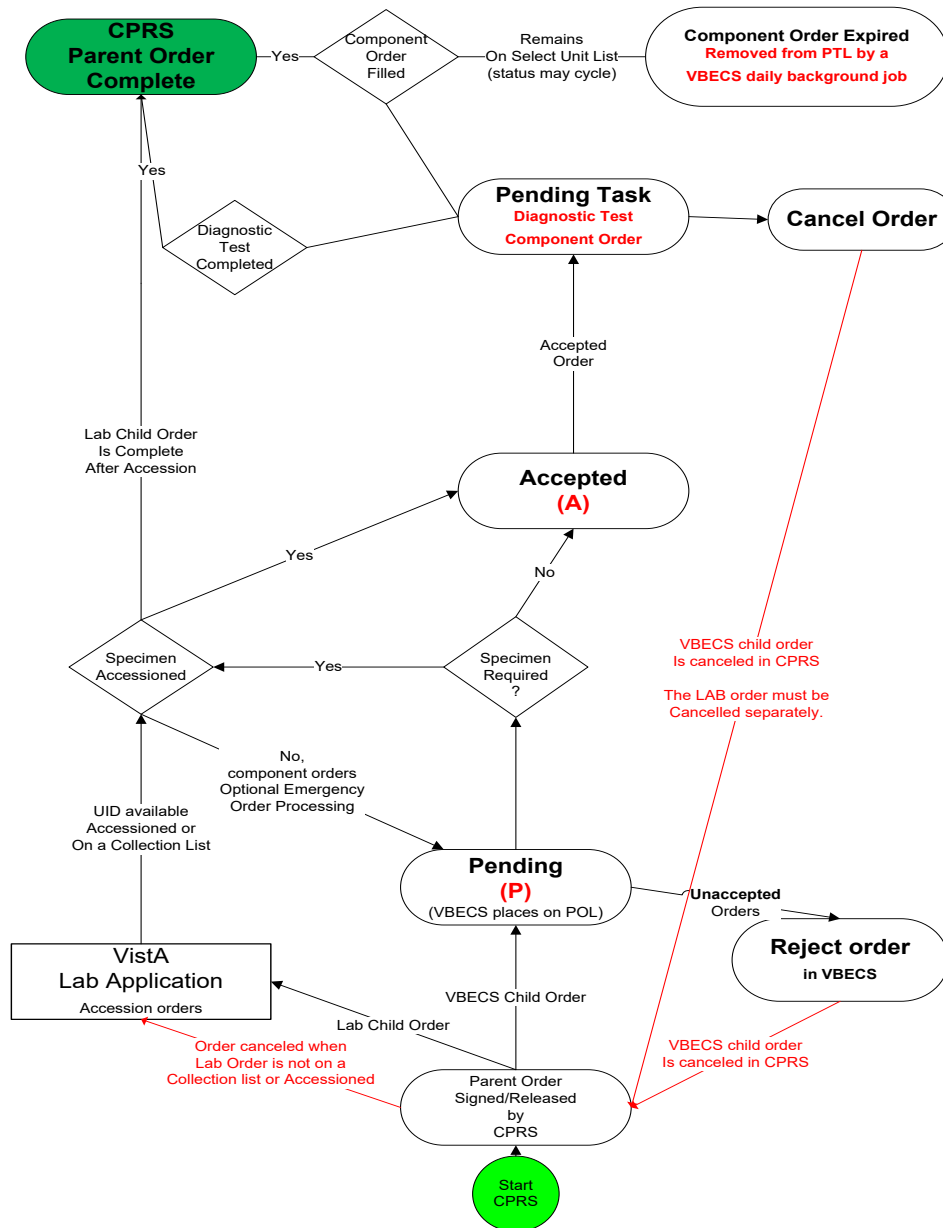


## Rules for Electronic and Serologic Crossmatch

Component Classes with greater than (>) 2 mL RBC contamination require crossmatch evaluation, serologic or electronic (when enabled). For component classes in the component class “OTHER”, a reflex crossmatch test has been ordered per the technologist’s evaluation of the unit itself. Each collection is different and the tech must evaluate the red blood cell content of the unit at the time of selection.

When electronic crossmatch is enabled at a division and units that require crossmatch (Component Classes “WHOLE BLOOD”, “RED BLOOD CELLS”, or a reflex crossmatch test has been ordered) are selected, VBECS performs a comprehensive set of data validation, which does not include inactivated patient or units tests. This patient and unit data validation is to ensure that the combination of patient and blood unit testing allows the unit to be eligible for electronic crossmatch. To assist the user, when the combination does not allow for electronic crossmatch, a warning message is displayed with a short explanation as to why the electronic crossmatch will not be applied.

Table 23 provides the mapping for the actual parameters verified and associated error messages. All parameters must comply in order for the unit to be eligible for electronic crossmatch: there are no exceptions or overrides. The parameters are checked in the order listed and once a parameter fails, VBECS displays the appropriate error message, no further checking occurs, and the selected unit must have serologic crossmatch performed.

 *To use the electronic crossmatch function, sites must perform additional validation of electronic crossmatch to meet regulatory requirements. See Guidelines for Implementing the Electronic Crossmatch (Bethesda, MD: American Association of Blood Banks, 2003).*

**Table 23: Rules for Electronic and Serologic Crossmatch**

Parameter Checked	Error Message When Parameter Fails
Current specimen ABO/Rh and Antibody Screen testing must be complete.	Patient not eligible for eXM. Current specimen not fully tested.
Current specimen Antibody Screen interpretation must be negative. VBECS checks only the last Antibody Screen entered.	Patient not eligible for eXM due to positive Antibody Screen.
Patient must have no history of previously identified antibodies, regardless of clinical significance based on system rules or division (or from conversion) of the antibody ID.	Patient not eligible for eXM due to previous antibody history.
Patient cannot have a persistent antigen negative requirement regardless of the division of the requirement entry.	Patient not eligible for eXM due to transfusion antigen negative requirement.
The patient current ABO/Rh must match the historical record when there is a historical record, and there must be no previous justifications. An ABO/Rh marked as “entered in error” is not considered part of the patient’s record.	Patient not eligible for eXM due to ABO/Rh discrepancy from previous record.
The patient cannot have a documented instance of current or previous Inconclusive ABO/Rh typing.	Patient not eligible for eXM due to ABO/Rh typing difficulty.
There must be at least two valid instances of ABO/Rh typing performed in the same division for the patient. One instance must be a valid ABO/Rh on the current specimen and not “unknown” or “inconclusive”; the other may be from a previous typing episode or a repeat ABO/Rh performed on the current specimen. The patient ABO/Rh interpretations must match in all instances. The user must enter valid forward and reverse typing for the system to use as the second test performed in the division.	Patient not eligible for eXM: he must have two instances of ABO/Rh typing at the division for eXM, including the current specimen.
Selected unit must be ABO compatible based on system rules.	Unit not eligible for eXM. Unit is ABO incompatible with the patient.

## Unit Status Flowchart


### Types of Unit Statuses

- Temporary: awaiting ABO/Rh confirmation (lime green on the flowchart)
- Adjustable: may move in and out of the status as part of normal processing
- Issued: pending an update to transfusion or return (bright green in the flowchart)
- Final: the unit is no longer available for most processes in the division; the unit record remains active (pink and stop signs in the flowchart)

### Indicators Associated with Units

- Quarantined: attaches to the unit record and status when the unit is quarantined. Removed by releasing the unit from quarantined.
- Presumed: attaches to the unit record only when the unit is in an issued unit status for more than 48 hours. Remains on the unit record if no transfusion information is entered through Post-Transfusion Information and Document ABO Incompatible Transfusion.

Figure 148: Unit Status Flowchart

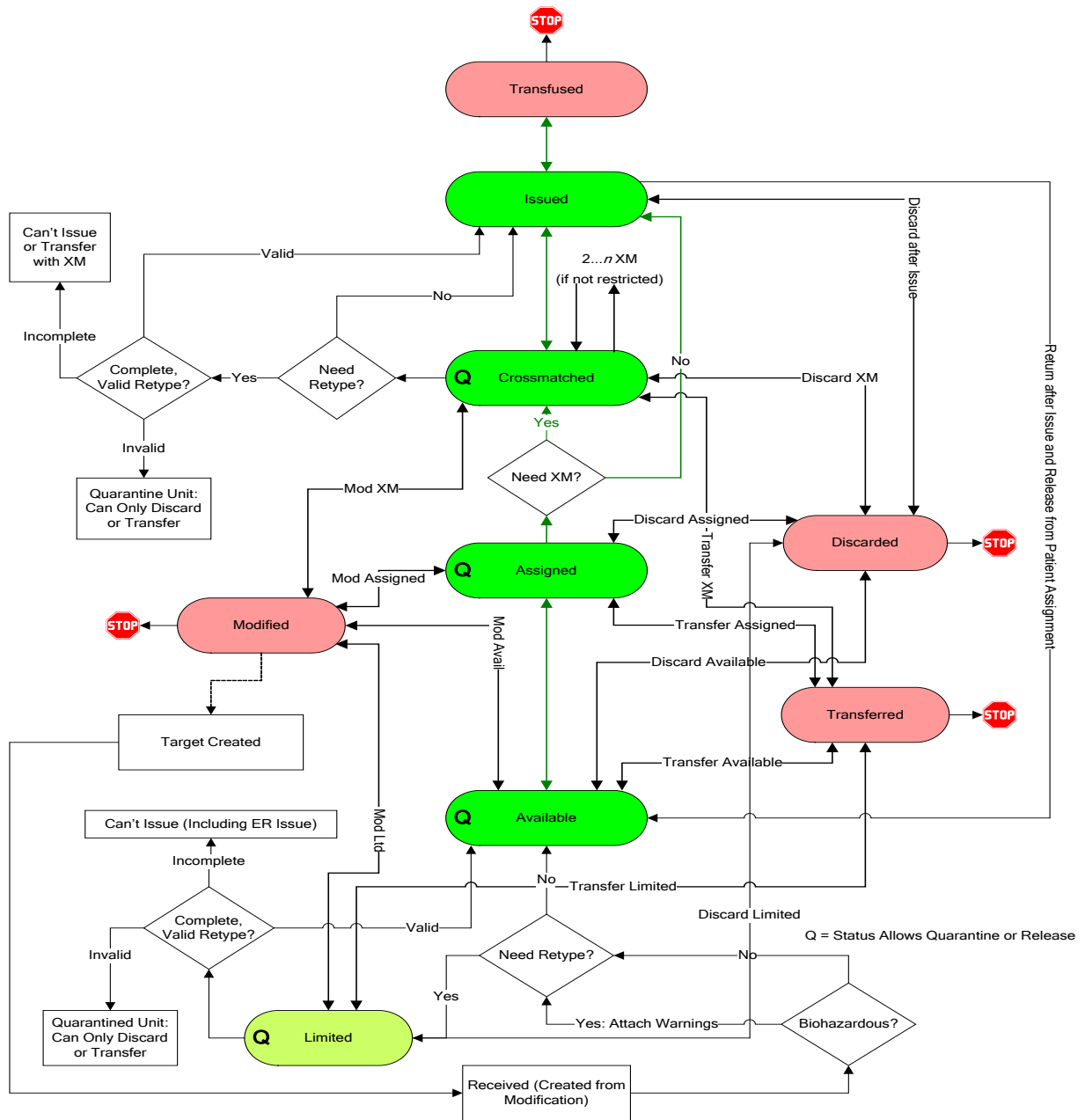

## VBECs Maximum Surgical Blood Order Schedule

Table 24: VBECs Maximum Surgical Blood Order Schedule (MSBOS)

VBECs Maximum Surgical Blood Order Schedule (MSBOS)	
Abdominal-Perineal resection	Keratectomy
Adrenalectomy	Hernia repair
Amputation (A/K, B/K)	Hip Replacement, total
Aneurysm, aortic	Hip Replacement, revision
Aneurysm, abdominal	Knee Replacement, total
Aneurysm, thoracic	Laminectomy, cervical, thoracic, or lumbar
Aneurysm, cranial	Laminectomy, disk repair
Aorto-Femoral Bypass	Laparotomy, exploratory
Appendectomy	Laryngectomy
Coronary Bypass Graft (CABG)	Lobectomy, pulmonary
Coronary Bypass Graft (CABG), redo	Mandibulectomy
Carotid Endarterectomy	Mastectomy, radical
Cervical Discectomy	Maxillectomy
Cervical Fusion	Nephrectomy
Cholecystectomy	Pancreatectomy
Cholostomy	Parathyroidectomy
Colectomy	Portocaval Shunt
Colon Resection	Prostatectomy
Craniotomy	Prostatectomy, total
Cystectomy	Radical Neck Dissection
Cystolithotomy	Renal Transplant
Cystoscopy	Skin Flap
Embolectomy	Skin Graft
Esophageal Resection	Splenectomy
Esophogectomy	Thyroidectomy
Femoral-Popliteal Bypass	Tracheostomy
Fracture, open reduction	Transurethral Resection (TURP)
Fusion, cervical or lumbar	Vagotomy
Gastrectomy	Valve replacement, aortic or mitral
Gastric Bypass	Other*
Glossectomy	
Hematoma Evacuation, sub- or epidural	

\*Other surgeries entered in the Surgery Name field during order entry. (Local facilities may add surgeries to this list.) Entries made during order entry will not have MSBOS component order recommendations. VBECs does not display “Other” in MSBOS.

## BCE COTS Message Updates

**Table 25: BCE COTS Message Updates**

This table identifies the specific subsequent processing of an assigned unit for one patient unit association that may or may not result in a change to the unit status or other quality and how that may or may not trigger an status change that requires update to the BCE COTS product.

**Unit status code legend:**

<b>A</b>	Available	<b>S</b>	Assigned
<b>C</b>	Crossmatched	<b>T</b>	Transfused
<b>D</b>	Discarded	<b>X</b>	Transferred
<b>I</b>	Issued	<b>DS</b>	Dispense Unit (Issued)
<b>L</b>	Limited	<b>RE</b>	Release Unit
<b>M</b>	Modified	<b>RS</b>	Reserve Unit

BCE COTS Message Updates					
Row #	Initial Unit Status	Processing Option	Additional Information About the Processing Option	Unit Status Exiting the Option	Unit Status Reported to BCE COTS
1	A	Select Unit for Patient	Normal flow	S	RS
2	A	Select Unit for Patient	Emergency issue flow	S	RS
3	A	Select Unit for Patient	Electronic Crossmatch processed	C	RS
4	C	Process Outgoing Shipment	Maintain Assignment	X	RE upon confirmation
5	C	Process Outgoing Shipment	Release Assignment upon confirmation	A (then X when confirmed)	RE sent when release processed
6	S	Process Outgoing Shipment	Maintain Assignment	X	RE upon confirmation
7	S	Process Outgoing Shipment	Release Assignment upon confirmation	A (then X when confirmed)	RE sent when release processed
8	D	Remove Final Unit Status	Unit was previously Crossmatched	C	RS
9	D	Remove Final Unit Status	Unit was previously Assigned	S	RS
10	S	Remove Final Unit Status	Remove Modification	A	No message resent
11	X	Remove Final Unit Status	Unit was previously Crossmatched	C	RS
12	X	Remove Final Unit Status	Unit was previously Assigned	S	RS
13	T	Remove Final Unit Status	Normal flow	I	DS
14	S	Pool Units	Maintain Assignment	M	RE-original
15	S	Pool Units	Maintain Assignment	S	RS-target
16	S	Pool Units	Release Assignment	M	RE-original



BCE COTS Message Updates					
Row #	Initial Unit Status	Processing Option	Additional Information About the Processing Option	Unit Status Exiting the Option	Unit Status Reported to BCE COTS
17	S	Pool Units	Release Assignment	A	Target, no message sent
18	C**	Pool Units	Maintain Assignment	M	RE-original(s), may be multiple messages.
19	C**	Pool Units	Maintain Assignment	C	RS-target
20	C**	Pool Units	Release Assignment	M	RE-original
21	C**	Pool Units	Release Assignment	A	Target, no message sent
22	S	Split Units	Maintain Assignment	M	RE-original
23	S	Split Units	Maintain Assignment	S	RS-targets, one message for each target created.
24	S	Split Units	Release Assignment	M	RE-original
25	S	Split Units	Release Assignment	A	Target, no message sent
26	C	Split Units	Maintain Assignment	M	RE-original
27	C	Split Units	Maintain Assignment	C	RS-target
28	C	Split Units	Release Assignment	M	RE-original
29	C	Split Units	Release Assignment	A	Target, no message sent
30	S	Modify Units	Maintain Assignment	M	RE-original (when processing in batch, multiple messages are sent)
31	S	Modify Units	Maintain Assignment	S	RS-target (when processing in batch, multiple messages are sent)
32	S	Modify Units	Release Assignment	M	RE-original (when processing in batch, multiple messages are sent)
33	S	Modify Units	Release Assignment	A	Target, no message sent
34	C	Modify Units	Maintain Assignment	M	RE-original (when processing in batch, multiple messages are sent)
35	C	Modify Units	Maintain Assignment	C	RS-target (when processing in batch, multiple messages are sent)
36	C	Modify Units	Release Assignment	M	RE-original (when processing in batch, multiple messages are sent)
37	C	Modify Units	Release Assignment	A	Target, no message sent
38	C	Discard or Quarantine Unit	Discarded or Wasted	D	RE

BCE COTS Message Updates					
Row #	Initial Unit Status	Processing Option	Additional Information About the Processing Option	Unit Status Exiting the Option	Unit Status Reported to BCE COTS
39	S	Discard or Quarantine Unit	Discarded or Wasted	D	RE
40	S	Discard or Quarantine Unit	Quarantine	A	RE
41	C	Discard or Quarantine Unit	Quarantine	A	RE
42	S	Record Testing when saving Crossmatch Results	Compatible (C)	C	No message is sent
43	S	Record Testing when saving Crossmatch Results	Incompatible (I)	A	RE
44	S	Record Testing when saving Crossmatch Results	Crossmatch Compatible: Don't Transfuse (D)	A	RE
45	S	Record Testing when saving Crossmatch Results	Compatible-Give with MD approval (G)	C	No message is sent
46	S	Record Phenotype Results	Unit is antigen positive for a patient's antigen negative requirement	S	No message is sent
47	C	Record Phenotype Results	Unit is antigen positive for a patient's antigen negative requirement	C	No message is sent
48	S	Record Phenotype Results	Antigen typing result creates a discrepant antigen typing.	A	RE
49	C	Record Phenotype Results	Antigen typing result creates a discrepant antigen typing.	A	RE
50	S	Release Unit from Patient Assignment	Normal flow, or any overrides processed	A	RE
51	C	Release Unit from Patient Assignment	Normal flow, or any overrides processed	A	RE
52	I	Return Issued Units to Blood Bank	Unit without a crossmatch	S	RS
53	I	Return Issued Units to Blood Bank	Crossmatched unit	C	RS
54	I	Return Issued Units to Blood Bank	Unit without a crossmatch, unit is unsatisfactory	A	RE
55	I	Return Issued Units to Blood Bank	Unit crossmatched, unit is unsatisfactory	A	RE

BCE COTS Message Updates					
Row #	Initial Unit Status	Processing Option	Additional Information About the Processing Option	Unit Status Exiting the Option	Unit Status Reported to BCE COTS
56	S	Issue Blood Components	Normal flow with or without overrides	I	DS
57	C	Issue Blood Components	Normal flow with or without overrides	I	DS
58	S	Issue Blood Components	Unit is marked as unsatisfactory for issue	A	RE
59	C	Issue Blood Components	Unit is marked as unsatisfactory for issue	A	RE
60	C	Invalidate Patient Test Results	Invalidate Crossmatch results for the unit-patient association	S	No message is sent as the unit has not been released from assignment from the patient.

\*\*Pooling of products that also require crossmatch is not currently possible.

## Workload Process to Use Case Mapping Table

Table 26: Workload Process to Use Case Mapping Table

Workload Process to Use Case Mapping Table				
VBECS Process	Record Saved in This Use Case	Positive or Negative Workload Accrued	Transaction Type*	Explanation
ABO Forward and reverse typing (patient)	Transfusion Reaction Workup	Positive	P	An ABO/Rh test for pre or post is enabled and a valid interpretation other than Not Tested is selected. A workload event is accrued separately for Pre and Post entries.
	Patient Testing: ABO/Rh	Positive	P	Accrue workload when an ABO/Rh test performed from the ordered test ABO/Rh only (Record Patient ABO-Rh is completed and saved by Record Testing).
	Invalidate Patient Testing: Patient ABO/Rh	<b>Negative</b>	P	Accrue workload when a completed ABO/Rh test is invalidated.
ABO Forward and reverse typing (patient) Repeat Test	Patient Testing: ABO/Rh	Positive	M	Accrue workload when a reflex or repeat ABO/Rh test is performed, completed and saved via Record Testing. Note: ABO/Rh ordered in Order Reflex Test from main menu or via use of Order Reflex Test from Record Testing
	Invalidate Patient Testing: Patient ABO/Rh	<b>Negative</b>	M	Accrue workload when a reflex or repeat ABO/Rh test is invalidated.
ABO forward typing (unit)	Unit ABO/RH confirmation test	Positive	U	An ABO confirmation test is performed. When multiple units are selected in a batch, each unit in the batch accrues a workload event.  Note: Workload generated when Anti-D testing is not included in the unit's confirmation test. Workload is <b>not</b> accrued when an ABO or Rh discrepancy override is processed and VBECS releases all patient assignments. Workload is <b>not</b> accrued when VBECS quarantines the unit due to a discrepancy. No special handling for workload collection for additional confirmation tests on a unit.
	Invalidate Unit ABO/RH Confirmation test	<b>Negative</b>	U	Accrue workload when an ABO confirmation test is invalidated

Workload Process to Use Case Mapping Table				
VBECs Process	Record Saved in This Use Case	Positive or Negative Workload Accrued	Transaction Type*	Explanation
ABO/Rh forward typing (unit)	Unit ABO/RH confirmation test	Positive	U	An ABO/Rh confirmation test is performed. When multiple units are selected in a batch, each unit in the batch accrues a workload event.  Note: Workload generated when Ant-D is part of the unit's confirmation test. Workload is <b>not</b> accrued when an ABO or Rh discrepancy override is processed and VBECs releases all patient assignments. Workload is <b>not</b> accrued when VBECs quarantines the unit due to a discrepancy. Any unit successfully confirmed accrues workload. For modification Split: Workload is not inherited by split units. A split unit that requires confirmation accrues confirmation workload at the time of testing. No special handling for workload collection for additional confirmation tests on a unit.
	Invalidate Unit ABO/RH Confirmation test	<b>Negative</b>	U	Accrue workload when an ABO/Rh confirmation test is invalidated
Accept Order	Accept Order	Positive	M	Accrue workload when an order is accepted. When a multiple orders are selected, each order accrues workload.
Antibody identification Work-Up	ABID Test	Positive	P	User enters additional workload associated with the individual reflex ordered ABID. The selected VBECs multiplier will multiply against the VistA multiplier and display the (multiplication) product on the report in Division Workload Report.
	Invalidate ABID Test	<b>Negative</b>	P	Accrue workload when the ABID is invalidated. All workload processes associated in Enter Reflex Test Results, including the selected VBECs multiplier, are invalidated
Antibody Screen (patient)	Transfusion reaction workup	Positive	P	An ABS test for pre or post is enabled and a valid interpretation other than Not Tested is selected. A workload event is accrued separately for Pre and Post entries.
	Patient Testing ABS test	Positive	P	Accrue workload when an ordered ABS test performed from the CPRS ordered test ABS only (Record Patient Antibody Screen is completed and saved by Record Testing. The workload accrues when the ABS is completed and saved via Record Testing no matter how many cells (rows) are required by the division configuration.
	Invalidate Patient Testing ABS test	<b>Negative</b>	P	Accrue workload when a completed ABS test is invalidated.

Workload Process to Use Case Mapping Table				
VBECS Process	Record Saved in This Use Case	Positive or Negative Workload Accrued	Transaction Type*	Explanation
Antibody Screen (patient) Repeat Test	Patient Testing ABS test	Positive	M	Accrue workload when a reflex or repeat ABS test is performed, completed and saved via Record Testing. Note: ABS ordered in Order Reflex Test from main menu or via use of Order Reflex Test from Record Testing.
	Invalidate Patient Testing ABS test	Negative	M	Accrue workload when a reflex or repeat ABS test is invalidated.
Antigen phenotyping, Single Test phase (QC)	QC for unit or patient antigen typing	Positive	M	Accrue workload when Antiserum QC is included in Record Phenotype Results or Record Patient Antigen Types includes the testing of both the positive and negative control cells, per specificity by lot number, when only one phase of reactivity is chosen for the test grid, IS or AHG. One workload event is collected per completed tab for regular or repeat antigen tests.
Antigen phenotyping, Multiple Test phases (QC)	QC for unit or patient antigen typing	Positive	M	Accrue workload when Antiserum QC is included in Record Phenotype Results or Record Patient Antigen Types includes the testing of both the positive and negative control cells, per specificity by lot number, when only multiple phases of reactivity is chosen for the test grid, IS/RT, RT/37 or weak D. One workload event is collected per completed tab for regular or repeat antigen tests. When weak D is the selected test, QC may not be accrued for the rack selection. QC is accrued when positive and negative cells must be tested for the lot number.
Cancel Order	Accept Order, Reject Order	Positive	M	Accrue workload when an order on the pending order list is canceled. When a multiple orders are canceled, each order accrues workload.
Cancel Order	Patient Testing List, Cancel order	Positive	M	Accrue workload when an order on the pending task list is canceled. When multiple orders are canceled, each order accrues workload.
Crossmatch unit, electronic	Select Unit	Positive	P	This process is invoked when an individual unit is selected for patient assignment and the unit is electronically crossmatched. When multiple units are selected, each unit accrues workload.
Crossmatch unit, serologic immediate spin	Patient Testing, serologic crossmatch	Positive	P	Accrue workload when an individual unit (row) crossmatch, selected to include only the IS phase, is completed and is saved by Record Testing. When multiple units are selected, each unit (row) accrues workload.

Workload Process to Use Case Mapping Table				
VBECS Process	Record Saved in This Use Case	Positive or Negative Workload Accrued	Transaction Type*	Explanation
	Invalidate Patient Testing, serologic crossmatch	<b>Negative</b>	P	Accrue workload when a completed crossmatch test is invalidated. The workload originally saved with the test, serologic immediate spin.
Crossmatch unit, serological Coombs	Transfusion Reaction Workup, serologic crossmatch	Positive	P	A crossmatch test for pre or post is enabled and a valid interpretation other than Not Tested is selected. A workload event is accrued separately for Pre and Post entries. When multiple units are selected, each unit accrues workload.
	Patient Testing, serologic crossmatch	Positive	P	Accrue workload when an individual unit (row) crossmatch, selected to include all phases or only AHG phase, is completed and is saved by Record Testing. When multiple units are selected, each unit (row) accrues workload.
	Invalidate Patient Testing, serologic crossmatch	<b>Negative</b>	P	Accrue workload when a completed crossmatch test is invalidated. The workload originally saved with the test, serological Coombs.
Crossmatch, Repeat Test	Patient Testing, serologic crossmatch	Positive	M	Accrue workload when an individual unit (row) crossmatch, selected to include all phases or IS or only AHG phase, is completed and is saved by Record Testing. When multiple units are selected, each unit (row) accrues workload. Note: ordered via Order Reflex Test from main menu or the use of Order Reflex Test by Record Testing.
	Invalidate Patient Testing, serologic crossmatch	<b>Negative</b>	M	Accrue workload when an individual unit crossmatch is invalidated.
Daily Rack Quality Control (QC)	Enter Daily Reagent Rack QC	Positive	M	Accrue workload when Daily QC rack completed for one individual rack includes all rows in configured QC. When multiple racks are tested, each completed and saved tab accrues a workload event.
DAT (QC)	Enter Daily Reagent Rack QC or DAT QC	Positive	M	Accrue workload when Reagent QC completed or in Record Patient Direct Antiglobulin Test includes the testing of both the positive and negative control cells, per specificity per lot number, when only one phase of reactivity is chosen for the test grid, IS or AHG. One workload event is collected per completed tab for regular or repeat antiglobulin tests (PS, IgG, Comp) tests.

Workload Process to Use Case Mapping Table				
VBECS Process	Record Saved in This Use Case	Positive or Negative Workload Accrued	Transaction Type*	Explanation
Direct Antiglobulin Test (DAT)	Transfusion Reaction Workup, DAT test	Positive	P	A DAT test for pre or post is enabled and a valid interpretation other than Not Tested is selected. A workload event is accrued separately for Pre and Post entries.
	Patient Testing, DAT	Positive	P	Accrue workload when a DAT completed and saved (Record Patient Direct Antiglobulin Test is saved by Record Testing as part of a TAS configuration or individual test ordered. This count is used for all antiglobulin tests (PS, IgG, Comp) when ordered from CPRS or Reflex testing.
	Invalidate Patient Testing, DAT	<b>Negative</b>	P	Accrue workload when a completed DAT, PS, IgG, or Comp is invalidated.
Direct Antiglobulin Test (DAT) Repeat test	Patient Testing, DAT	Positive	M	Accrue workload when a reflex or repeat DAT test is performed, completed and saved via Record Testing. This applies to all repeat antiglobulin tests (PS, IgG, Comp) Note: Any DAT ordered in Order Reflex Test from main menu or via use of Order Reflex Test from Record Testing.
	Invalidate Patient Testing, DAT	<b>Negative</b>	M	Accrue workload when a completed Repeat DAT, PS, IgG, or Comp is invalidated.
Deglycerolize unit	Modify Unit, Deglycerolize	Positive	U	Accrue workload when an individual blood unit is processed by the modification type: Deglycerolize. <i>Note: Workload is not accrued when a patient assignment is processed and VBECS releases all other patient assignments. Workload is not accrued when VBECS is required to quarantine the unit.</i>
	Invalidate Unit Status, Deglycerolize	<b>Negative</b>	U	An individual blood unit's status is invalidated when the original modification process was Deglycerolize.
Discard unit	Discard Unit	Positive	U	Accrue workload when an individual blood unit's status is invalidated. When a batch of units is selected, each unit accrues workload.
	Invalidate Unit Status, Discard	<b>Negative</b>	U	Accrue workload when a unit is discarded for waste or credit. When a batch of units is selected, each unit accrues workload.



Workload Process to Use Case Mapping Table				
VBECS Process	Record Saved in This Use Case	Positive or Negative Workload Accrued	Transaction Type*	Explanation
Freeze unit	Modify Unit, Freeze	Positive	U	Accrue workload when an individual blood unit is processed by the modification type: Freeze. <i>Note: Workload is not accrued when a patient assignment is processed and VBECS releases all other patient assignments. Workload is not accrued when VBECS is required to quarantine the unit.</i>
	Invalidate Unit Status, Freeze	<b>Negative</b>	U	An individual blood unit's status is invalidated when the original modification process was Freeze.
Irradiate unit	Modify Unit, Irradiate	Positive	U	Accrue workload when an individual blood unit is processed by the modification type: Irradiate. When a batch of units is irradiated, each unit accrues workload. <i>Note: Workload is not accrued when a patient assignment is processed and VBECS releases all other patient assignments. Workload is not accrued when VBECS is required to quarantine the unit.</i>
	Invalidate Unit Status, Irradiate	<b>Negative</b>	U	An individual blood unit's status is invalidated when the original modification process was Irradiate.
Leukoreduce unit	Modify Unit, Leukoreduce	Positive	U	Accrue workload when an individual blood unit is processed by the modification type: Leukoreduce. <i>Note: Workload is not accrued when a patient assignment is processed and VBECS releases all other patient assignments. Workload is not accrued when VBECS is required to quarantine the unit.</i>
	Invalidate Unit Status, Leukoreduce	<b>Negative</b>	U	An individual blood unit's status is invalidated when the original modification process was Leukoreduce.
Split unit	Modify Unit, Split	Positive	U	Accrue workload when a unit modification of Split and a single workload event is recorded regardless of the number of units created by the modification. <i>Note: Workload is not accrued when a patient assignment is processed and VBECS releases all other patient assignments. Workload is not accrued when VBECS is required to quarantine the unit. Workload accrued by splitting is invalidated in Remove Final Unit Status with the inactivation of the modification</i>
	Invalidate Unit Status, Split	<b>Negative</b>	U	A Split Unit has its unit status invalidated. A single workload event is recorded regardless of the number of units originally created by the modification.

Workload Process to Use Case Mapping Table				
VBECS Process	Record Saved in This Use Case	Positive or Negative Workload Accrued	Transaction Type*	Explanation
Rejuvenate unit	Modify Unit, Rejuvenate	Positive	U	Accrue workload when an individual blood unit is processed by the modification type: Rejuvenate. <i>Note: Workload is not accrued when a patient assignment is processed and VBECS releases all other patient assignments. Workload is not accrued when VBECS is required to quarantine the unit.</i>
	Invalidate Unit Status, Rejuvenate	<b>Negative</b>	U	An individual blood unit's status is invalidated when the original modification process was Rejuvenate.
Thaw	Modify Unit, Thaw	Positive	U	Accrue workload when an individual blood unit is processed by the modification type: Thaw. When a batch of units is thawed, each unit accrues workload. This applies to Thaw FFP and Thaw Cryo. <i>Note: Workload is not accrued when a patient assignment is processed and VBECS releases all other patient assignments. Workload is not accrued when VBECS is required to quarantine the unit.</i>
	Invalidate Unit Status, Thaw	<b>Negative</b>	U	An individual blood unit's status is invalidated when the original modification process was Thaw. This modification type is applicable to Thaw FFP and Thaw Cryo.
Wash unit	Modify Unit, Wash	Positive	U	Accrue workload when an individual blood unit is processed by the modification type: Wash. <i>Note: Workload is not accrued when a patient assignment is processed and VBECS releases all other patient assignments. Workload is not accrued when VBECS is required to quarantine the unit.</i>
	Invalidate Unit Status, Wash	<b>Negative</b>	U	An individual blood unit's status is invalidated when the original modification process was Wash.
Volume Reduce	Modify Unit, Volume Reduce	Positive	U	Accrue workload when an individual blood unit is processed by the modification type: Volume Reduce. <i>Note: Workload is not accrued when a patient assignment is processed and VBECS releases all other patient assignments. Workload is not accrued when VBECS is required to quarantine the unit.</i>
	Invalidate Unit Status, Volume Reduce	<b>Negative</b>	U	An individual blood unit's status is invalidated when the original modification process was Volume Reduce.
Issue unit	Issue Blood Unit	Positive	P	Accrue workload when a unit (row) is issued or is returned from issue to a patient. When a batch of units is processed, each unit invokes one workload process.

Workload Process to Use Case Mapping Table				
VBECS Process	Record Saved in This Use Case	Positive or Negative Workload Accrued	Transaction Type*	Explanation
Justification	Justify ABO/Rh Change	Positive	M	Workload is accrued when a patient's ABO or Rh typing is justified. One workload event per patient justification.
Login Equipment	Equipment, Login	Positive	M	Accrue workload when a lot number of any type of equipment is logged into the system.
Login Reagent	Reagent, Login	Positive	M	Accrue workload when a lot number of any type of reagent is logged into the system. When multiple lot numbers are processed in a batch, each lot number's workload is counted.
Login Supply	Supply, Login	Positive	M	Accrue workload when a lot number of any type of supply is logged into the system. When multiple lot numbers are processed in a batch, each lot number's workload is counted.
Maintain Specimen	Maintain Specimen	Positive	M	Accrue workload when a specimen is maintained during order acceptance when a specimen is requirement for the order. Note: This event is accrued when the specimen is maintained in Maintain Specimen. This is collected in addition to the accept order workload accrued by Accept Order. Marking a specimen unacceptable does not create a negative workload event.
Patient antigen phenotype (multiple phases)	Patient Antigen typing	Positive	P	Accrue workload when a patient antigen phenotype test [specificity (tab)] with IS/RT or IS/37 phases is completed and saved (when completed and saved via Record Testing). One workload event is collected per completed tab for repeat or regular antigen tests.
	Invalidate, Patient Antigen typing	<b>Negative</b>	P	Accrue workload when a patient antigen phenotype test as defined by the antiserum specificity tested with any phases is invalidated.
Patient antigen phenotype (single phase)	Patient Antigen typing	Positive	P	Accrue workload when a patient antigen phenotype test [specificity (tab)] with AHG or IS phase is completed and saved (when completed and saved via Record Testing). One workload event is collected per completed tab for repeat or regular antigen tests.
	Invalidate, Patient Antigen typing	<b>Negative</b>	P	Accrue workload when a patient antigen phenotype test as defined by the antiserum specificity tested with a single phase is invalidated.

Workload Process to Use Case Mapping Table				
VBECS Process	Record Saved in This Use Case	Positive or Negative Workload Accrued	Transaction Type*	Explanation
Pool unit	Modify, Pool	Positive	U	Accrue workload when a pooled unit is created and a single workload event is recorded regardless of the number of units included in the pooled unit. This applies to modification type: Pool. Add/Remove unit from a pool does not accrue any workload. <i>Note: Workload is not accrued when a patient assignment is processed and VBECS releases all other patient assignments. Workload is not accrued when VBECS is required to quarantine the unit.</i> <i>Workload accrued by Pooling is invalidated in Edit Unit Processing Information with the inactivation of the pooled unit.</i>
	Edit Unit, Inactive Unit (that was in a Pool)	<b>Negative</b>	U	Accrue workload when a unit is inactivated (unit record inactivated) when the pooled unit was created in VBECS.
	Invalidate unit status, Pool	<b>None</b>	N/A	No effect on workload accrual when a unit is removed from a modified status that was included in a pool.
Quarantine unit	Discard/Quarantine, Quarantine unit	Positive	U	Accrue workload when a unit is marked for quarantine. When a batch of units is selected, each unit accrues workload.
Release directed unit	Free Directed Unit	Positive	U	Accrue workload when an individual blood unit with the restriction type of directed is released for use as an allogeneic unit.
Release unit from patient back to inventory	Release Unit from Patient Assignment	Positive	U	Accrue workload when an individual unit is released from patient assignment. When multiple units are selected, each unit accrues workload.
Release unit from Quarantine	Discard/Quarantine, Release Unit from Quarantine	Positive	U	Accrue workload when a unit is released from quarantine. When a batch of units is selected, each unit accrues workload.
Return Issued unit	Return Issued unit to Inventory	Positive	U	Accrue workload when a unit is returned from issue status.
See Serologic Crossmatch, Patient ABO/Rh, Patient ABS, Patient DAT, Patient Antigen Typing	Patient Testing	Positive	Various	<i>Patient Testing encapsulates Serologic Crossmatch, Patient ABO/Rh, Patient ABS, Patient DAT, Patient Antigen Typing. Patient Testing does not in and of itself accrue workload, but accrues workload for the various testing grids for which it controls the data save. See the various options in this list for additional information.</i>

Workload Process to Use Case Mapping Table				
VBECS Process	Record Saved in This Use Case	Positive or Negative Workload Accrued	Transaction Type*	Explanation
Thaw/pool Cryo	Modify Unit, Thaw/Pool	Positive	U	Accrue workload when an individual unit modification of Thaw/Pool Cryo. A single workload event is recorded regardless of the number of units included in the pooled unit. <i>Note: Workload is not accrued when a patient assignment is processed and VBECS releases all other patient assignments. Workload is not accrued when VBECS is required to quarantine the unit.</i>
	Edit, Unit, Inactivate Unit (that was in a Thaw/Pool)	<b>Negative</b>	U	Accrue workload when a unit is inactivated (unit record inactivated) when the pooled unit was created in VBECS.
	Invalidate Unit status, Thaw/Pool	<b>None</b>	N/A	No effect on workload accrual when a unit is removed from a modified status that was included in a Thaw/pool Cryo pool.
Transfuse Unit	Enter Post Transfusion Data	Positive	U	Accrue workload when an individual blood unit's status is updated to transfused by Enter Post-Transfusion Details basic flow or Alt Flow A.
	Invalidate Unit Status, Transfused	<b>Negative</b>	U	An individual blood unit's status is invalidated when the unit was in a status of transfused.
Transfusion Reaction Investigation	Patient Testing, TRW	Positive	P	Accrue workload when a transfusion reaction investigation is saved by Record Transfusion Reaction Workup. <i>This does not include workload accrued by the optional TRW serologic testing.</i> Note: The Division Workload Report workload report reports the sum of Workload Events recorded in Record Transfusion Reaction Workup for an individual TRW.
	Invalidate Patient Testing, TRW	<b>Negative</b>	P	Accrue workload when a transfusion reaction investigation previously saved by Record Transfusion Reaction Workup is invalidated. The workload for the transfusion reaction, as well as all of the workload accrued by the optional TRW serologic testing (pre- and post-ABO/Rh, pre- and post-ABS, pre- and post-DAT) are invalidated, as included in selected TRW (Record Transfusion Reaction Workup). <i>Note: Any pre- or post-crossmatch tests performed as part of the TRW must be invalidated individually.</i>
Unit Antigen phenotyping, Multiple Test phases	Unit Antigen typing	Positive	U	Accrue workload when a unit antigen phenotype test [specificity (tab)] with IS/RT or IS/37 phases is selected and completed for an individual blood unit (row). No special handling for workload collection for additional repeat antigen typing tests on a unit.

Workload Process to Use Case Mapping Table				
VBECs Process	Record Saved in This Use Case	Positive or Negative Workload Accrued	Transaction Type*	Explanation
	Edit Unit, Invalidate Unit Antigen Typing	<b>Negative</b>	U	Accrue workload when a unit antigen phenotype test (specificity) with Multiple Test phases is invalidated for an individual blood unit.
Unit Antigen phenotyping, Single Test phase	Unit Antigen typing	Positive	U	A unit (row) antigen phenotype test [specificity (tab)] with AHG or IS phase is selected and completed for an individual blood unit (row). No special handling for workload collection for additional repeat antigen typing tests on a unit.
	Edit Unit, Invalidate Unit Antigen Typing	<b>Negative</b>	U	Accrue workload when a unit antigen phenotype test (specificity) with Single Test phase is invalidated for an individual blood unit.
Unit login	Incoming Shipment	Positive	U	An individual unit record is activated as saved to an incoming shipment invoice. When multiple units are entered, each unit added to the database accrues workload.
	Edit Unit, Invalidate Unit Antigen Typing	<b>Negative</b>	U	Accrue workload when a unit is inactivated (unit record inactivated) when logged in thru incoming shipment or is a pooled unit created in VBECs. When the unit was created by split modification no workload is invalidated in this option.
Unit logout	Outgoing Shipment	Positive	U	An individual unit's status is updated to <i>transferred</i> on a confirmed outgoing shipment invoice. When multiple units are selected, each unit accrues workload. Accrue workload upon the confirmation of the invoice, not the addition of a unit to a temporary outgoing shipment invoice as an invoice may be confirmed only once.
	Invalidate Unit Status, Transferred	<b>Negative</b>	U	An individual unit status is invalidated when the unit had a previous unit status of transferred.
Update Equipment Record	Equipment, Update	Positive	M	Accrue workload when a lot number of any type of equipment is updated in the system.
Update Reagent Inventory	Reagents, Update	Positive	M	Accrue workload when a lot number of any type of reagent is updated in the system. When multiple lot numbers are processed in a batch, each lot number's workload is counted.
Update Supply Inventory	Supplies, Update	Positive	M	Accrue workload when a lot number of any type of supply is updated in the system. When multiple lot numbers are processed in a batch, each lot number's workload is counted.
Patient ABO Automated	Automated Interface	Positive	P	Accrue workload when an ABO/Rh test, primary or reflex, is reviewed, and saved.

Workload Process to Use Case Mapping Table				
VBECS Process	Record Saved in This Use Case	Positive or Negative Workload Accrued	Transaction Type*	Explanation
	Invalidate patient testing, ABO/Rh	<b>Negative</b>	P	Accrue workload when a reviewed ABO/Rh test is invalidated.
Antibody Screen/Indirect Coombs Automated	Automated Interface	Positive	P	Accrue workload when an ordered ABS test, primary or reflex, is reviewed, and saved. Apply to primary or reflex tests.
	Invalidate patient testing, Antibody Screen	<b>Negative</b>	P	Accrue workload when a completed ABS test is invalidated.
Serologic Crossmatch, Major, Automated	Automated Interface	Positive	P	Accrue workload when an individual unit (row) crossmatch, selected to include all phases or only AHG phase, primary or reflex, is reviewed, and saved. When multiple units are selected, each unit (row) accrues workload.
	Invalidate patient testing, crossmatch	<b>Negative</b>	P	Accrue workload when a reviewed crossmatch test is invalidated. The workload originally saved with the test, serological Coombs.
Direct Antiglobulin Test Automated	Automated Interface	Positive	P	Accrue workload when a DAT, primary or reflex, is reviewed, and saved.
	Invalidate patient testing, Direct Antiglobulin test	<b>Negative</b>	P	Accrue workload when a reviewed DAT is invalidated.
Patient Antigen Typing Automated	Automated Interface	Positive	P	Accrue workload when a patient antigen phenotype test (tab) is reviewed and saved.
	Invalidate patient testing, patient antigen typing	<b>Negative</b>	P	Accrue workload when a reviewed patient antigen test (phenotype) is invalidated.
Blood Unit ABO Confirmation Test Automated	Automated Interface	Positive	U	<p>An ABO confirmation test, primary or repeat, is reviewed, and saved. When multiple units are selected in a batch, each unit in the batch accrues a workload event.</p> <ul style="list-style-type: none"> <li>Note: Workload generated when Anti-D testing is not included in the unit's confirmation test. Workload is <b>not</b> accrued when VBECS quarantines the unit due to a discrepancy. No special handling for workload collection for additional confirmation tests on a unit.</li> </ul>

Workload Process to Use Case Mapping Table				
VBECS Process	Record Saved in This Use Case	Positive or Negative Workload Accrued	Transaction Type*	Explanation
	Invalidate unit testing, ABO Test	<b>Negative</b>	U	Accrue workload when an ABO confirmation test is invalidated
Blood Unit ABO/Rh Confirmation Test Automated	Automated Interface	Positive	U	An ABO/Rh confirmation test, primary or repeat, is reviewed, and saved. When multiple units are selected in a batch, each unit in the batch accrues a workload event.  Any unit successfully confirmed accrues workload. <ul style="list-style-type: none"> <li>Note: Workload is <b>not</b> accrued when VBECS quarantines the unit due to a discrepancy.</li> </ul>
	Invalidate unit testing, ABO/Rh	<b>Negative</b>	U	Accrue workload when an ABO/Rh confirmation test is invalidated
Blood Unit Antigen Typing Automated	Automated Interface	Positive	U	A unit (row) antigen phenotype test, primary or repeat, is reviewed, and saved for an individual blood unit (row).  No special handling for workload collection for additional repeat antigen typing tests on a unit.
	Invalidate unit testing, Antigen Typing	<b>Negative</b>	U	Accrue workload when a unit antigen test is invalidated for an individual blood unit.

\*Transaction type: Patient (P), Unit (U), Misc (M), (N/A) No donor workload types in VBECS at this time.



## ***Appendix C: VBECS Work Flow***

Each of the six user roles is associated with a security level (see Table 1). Privileges accumulate as the security level increases. For example, a Lead Technologist's privileges include those of a Blood Bank Technologist and Enhanced Technologist.

**Figure 149: VBECS Work Flow (left)**


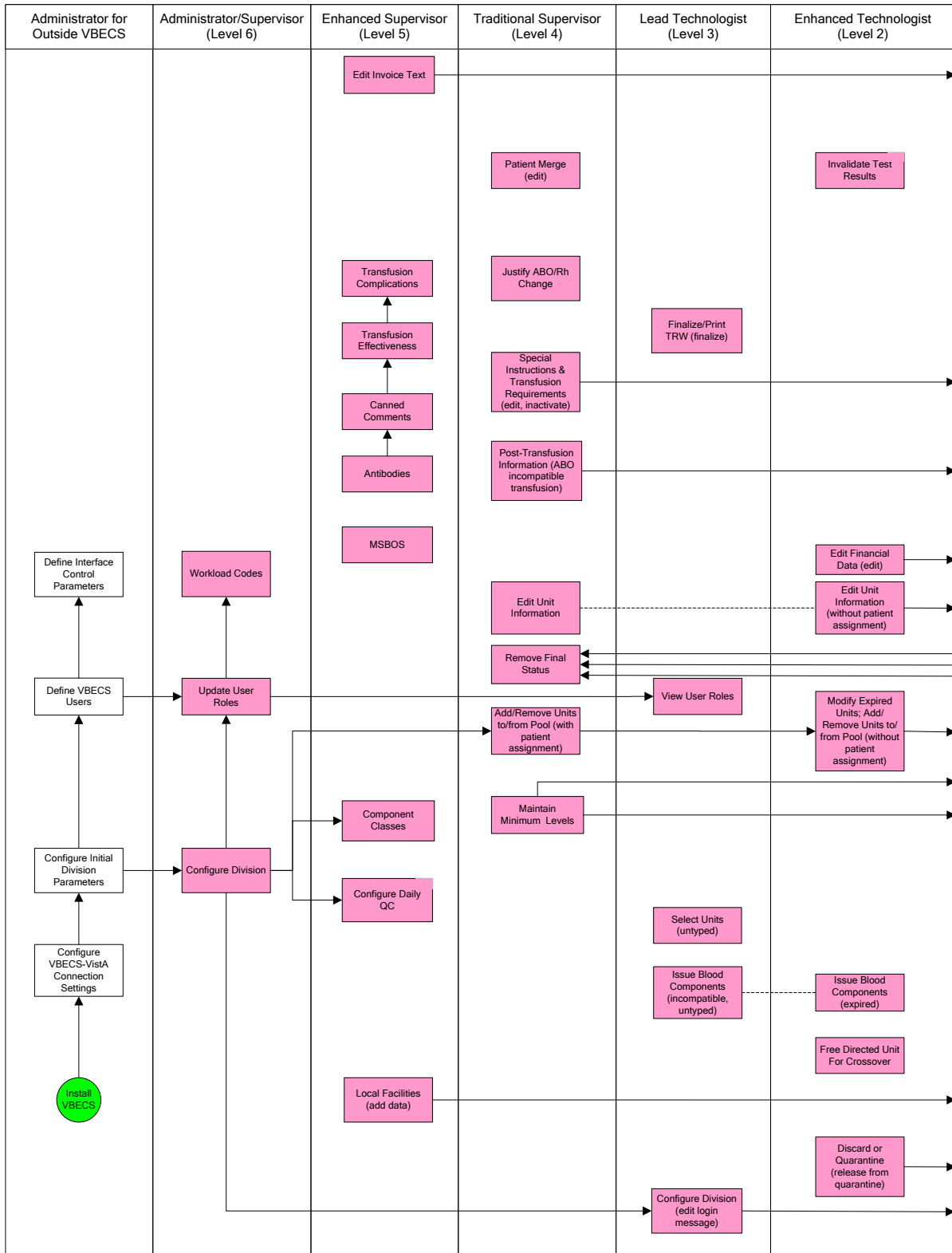


Figure 150: VBECS Work Flow (right)

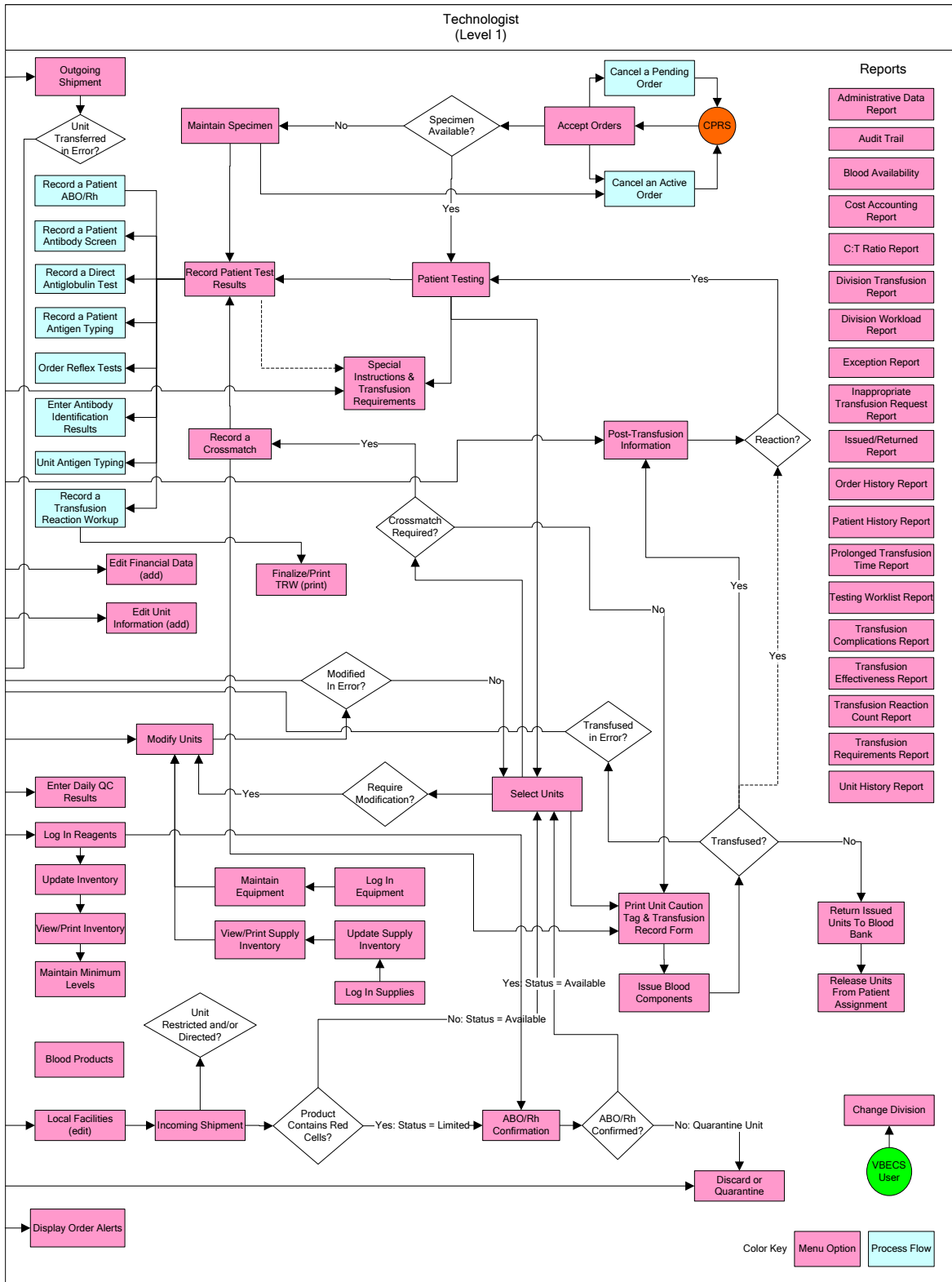

This page intentionally left blank.

## Appendix D: Limitations and Restrictions

Limitations and restrictions are listed by the options in which they occur.


**Table 27: Limitations and Restrictions**

<b>Functions Not Supported</b>
Blood donor collection activities (donor module)
Direct data entry of antibody identification
Direct data entry of transfusion reaction workups
Patient testing, transfusions, and blood components recorded in Legacy VistA are not available in VBECS
Pediatric blood product preparation
Printing of blood product labels
User defined customized queries or reports
User editing blood product, antigen, antibody, or transfusion reaction tables
Validation record storage

<b>Throughout VBECS</b>
After entering a canned or free-text comment in the testing comment field of the testing grid, the user must press <b>Enter</b> to store the comment.
Although the copy-and-paste function is enabled, do not use it to enter patient identification information. It is highly recommended that the user scan this information when possible; otherwise, enter it .
Data retrieval from VBECS is available only by approved database integration agreement.
Information may not be transferred from one window to another without refreshing the screen. When a window is partially displayed or freezes; or when you move from one window to another, enter information, return to the original window, and the entered information is not available, minimize then maximize the window to refresh it.
Initial blood orders must be placed in CPRS. Tests and products may be added to existing orders in VBECS.
Maximize option windows to display safety-critical information in full.
The date and time displayed in VBECS depend on the date and time set on the server. Sites are not notified when an administrator changes the date or time on the server. Users must contact their System Administrator when the date or time is incorrect.
VBECS depends on a connection to VistA and CPRS to retrieve patient information and orders, respectively.
When a message window requires a user response and VBECS displays it behind the active window, the user must minimize the active window or click and drag dialog boxes to expose the message window.
When the default screen resolution is changed, division or VistALink information may not be entirely visible.
When the user does not enter a full or partial unit ID and clicks the <b>find</b> button in the Unit Search Screen, response time will vary based on the size of the database.

<b>ABO/Rh Confirmation</b>
This option is disabled for transfusion-only facilities.
When building a worklist by invoice number, only invoiced units not previously tested and confirmed appear on the worklist. The user may deselect or add additional units.
For batch processing, the user can select incoming shipment invoices from his division, processed within the previous five days.
VBECS does not display product codes associated with unit IDs in ABO/Rh Confirmation. Therefore, do not create ABO/Rh confirmation worklists containing units with the same unit ID.
POOLED GRANULOCYTES labeled with an Rh type of "Pooled" is considered an Rh Positive unit which does not require confirmation testing.
POOLED GRANULOCYTES labeled with an ABO group of "Pooled" cannot be confirmed. A blood component unit received as "POOLED ABO" will have to be transfused using the Supervisory option, Document ABO Incompatible Transfusion.
Patient assignment is not released automatically when a unit is quarantined due to discrepant ABO/Rh retype testing. The unit is quarantined and cannot be selected for a different patient or issued for the current patient. Quarantined units are clearly marked in VBECS and cannot be issued.

<b>Accept Orders: Accept an Order</b>
An order for TAS must be accepted before the component order if the user wants to include the component request to the Inappropriate Transfusion Request Report. Accepting the TAS after the component order is marked inappropriate and accepted clears the user's indication that this is an inappropriate request.
When no CPRS order exists, the user must process the specimen offline. The emergency order check box is not enabled unless there is a CPRS order and a patient specimen accessioned in VistA. If necessary, process the emergency issue of units manually, per local policy, until the specimen is received and accessioned.
The user must accept the Type & Screen (TAS) before he accepts other diagnostic tests or component orders.
The user cannot click  (Restricted or Assigned Units) to access information on autologous or directed units that may be restricted to a patient.
Orders placed for patients with names that exceed the database standard for File #2 (30 characters) or do not have a first or last name in VistA are rejected and do not appear in VBECS. The VBECS System Administrator is notified of each rejection via an email to the address entered in VBECS Administrator "Email Alerts" field.
A VBECS order on a Collection List has to be received with a date and time in VistA to make the specimen available in VBECS. Orders should not be assigned a date/time collected of T@U.
Orders must include a patient location. A VBECS system error occurs when a user tries to print an order without a patient location.
The "Display Restricted or Assigned Units" button in Accept Orders was disabled. The Blood Availability Report displays units assigned and restricted for all patients.
On the Accept Orders screen, clicking the Received and Wanted column headers does not sort on the date and time of the Pending Order List.

<b>Accept Orders: Cancel a Pending Order</b>
Cancellation of one order does not affect other orders placed with it.

### Accept Orders: Pending Order List

Orders placed for patient's with names that exceed the database standard for File #2 (30 characters) or do not have a first or last name in VistA are rejected and do not appear in VBECS. The VBECS System Administrator is notified of each rejection via an email to the address entered in VBECS Administrator "Email Alerts" field.

### Accessing the System

VBECS automatically logs a user with access to only one division into that division.

Connectivity by IAM services access approval (PIV Card) is not possible when IAM services are not available.

Connectivity by IAM services access approval (PIV Card) is not required when a site has a PIV exemption or when IAM services are not available.

Patient testing, transfusions, and blood components recorded in Legacy VistA are not available in VBECS.

Connectivity is not possible when VistA is not available.

The user may log into multiple terminals simultaneously, but accessing VBECS in multiple sessions is no longer possible due to the way the PIV card works.

The Windows Security screen may be hidden when a user attempts to restore a VistALink connection. For the Windows Security screen to appear, the user must drag it from behind another window in order to select a certificate and log in.

### Add/Remove Units from a Pool

See Modify Units.

### Audit Trail

VBECS does not accommodate online review (verification, signature) of reports.

### Automated Testing Interface

VBECS has only been tested with one automated instrument connected.

Not available for transmission to Transfusion only facilities from a blood center due to constraints of VA's security software.

The results of Antibody Panels (ABID) will not be accepted by the VBECS Automated Interface.

The results of QC information from automated instruments will not be accepted by the VBECS Automated Interface.

Time zone will not be included in HL7 message date time from the broadcasting application (Instrument Manager). The time zone is determined from the division code contained in the message header (MSH) segment of the message.

### Automated Testing Review for Blood Units

VBECS does not accept inconclusive ABO/Rh test results.

Pooled Product Types labeled with an Rh type of "Pooled" is considered an Rh Positive unit which does not require confirmation testing.

Pooled Product Types that require confirmation, labeled with an ABO group of "Pooled" cannot be confirmed. A blood component unit received as "POOLED ABO" will have to be transfused using the Supervisory option, Document ABO Incompatible Transfusion.

### Automated Testing Review for Blood Units

Users can select and assign multiple units with the same DIN and different product codes for crossmatch. Users can only process one DIN (e.g., W123400345345) on the Automated Instrument at a time. It is incumbent on the user to verify that the crossmatch result from the Automated Instrument matches the product code (e.g., E0134V00) associated with the patient in VBECS. To decrease the risk of a mislabeled unit at issue, local practice may want to have only one on the units selected in VBECS at a time when crossmatch testing is being performed on the automated instrument.

Order Rejection comment entry is required but is not visible on a report.

### Automated Testing Review for Patients

VBECS does not accept inconclusive ABO/Rh test results.

VBECS does not accept inconclusive interpretations for antibody screen tests.

Immediate spin crossmatch is not allowed via automated test entry.

Patient Antigen typing applies only to patient specimens that contain red blood cells.

The system does not check the specificity of a patient's antibody against an existing patient antigen typing.

The patient antigen typing result is not reported to the CPRS record.

Order Rejection comment entry is required but is not visible on a report.

### Blood Products

Blood product records distributed through VBECS are national standards.

The division must have activated the valid target blood products for modification.

### Canned Comments

The user may not edit comments provided with VBECS, but may create additional comments. Although comments can also be removed, there should remain at least one comment available for selection (e.g., "Other").

Site personnel must enter comments in accordance with local policy and procedures.

To avoid a system error when entering canned comments, the user who configured the division must be added to the users list in VBECS.

### Component Classes

The user may not change the specimen requirement for a component class when there are incomplete orders for that component class.

There is no retrospective data entry or user selection for this option.

### Component Processing

An ISBT 128 Product Code's first characters may be E or X.

VBECS does not check whether blood products with the same unit ID and different product codes are the same blood type.

VBECS does not compare the ABO/Rh of an autologous unit or display a message when the ABO/Rh is incompatible with the restricted for patient.

The user may not save the unit record until the donation type and its requirements are satisfied.



### **Component Processing**

No longer supports the entry of blood products labeled in Codabar label type to new or existing invoices.

The unit expiration [Maximum Storage Time (MST)] is set to the longest possible storage time based on product type, core condition, attributes, and available references as a guide. The MST is set at a higher level than an individual collection product code. VBECS does not know the specific collection details. It is the responsibility of each user to accurately enter and process any one blood unit based on its expiration date/time as labeled by the collection facility.

When blood products intended for quarantine are scanned, VBECS does not automatically quarantine them: the user must do so.

VBECS does not recognize scanned labels for para-Bombay or Bombay ISBT units.

VBECS does not allow duplicate IDs for the same product code.

When a user scans a product code or expiration date barcode in the Unit ID field, VBECS does not display the entered information.

When a user scans an ISBT blood product code ID with "0" as the donation type (sixth character), VBECS indicates that the donation type is not specified. When the user selects a donation type from the drop-down menu, VBECS replaces "0" with a recognizable character in the product code text box and stores it as such. When the user rescans the product ID, he must manually replace "0" or search using the unit ID alone, or the unit ID and the first five characters of the product ID. When scanning a blood product derivative, this does not apply.

Blood product code entries in the blood product table include those from ICCBBA for ISBT 128. Product codes created after the release of VBECS cannot be entered until the updated tables are incorporated through patches to VBECS. Refuse acceptance of the unit, if possible. Users may not edit the blood product table.

ISBT 128 tables include blood product codes labeled as not for manufacturing or transfusion. VBECS does not restrict such products from being entered, issued, or transferred (local policy controls their use), but does not accommodate blood modification for these products.

No entry of an ICCBBA Product Description Code starting with A, B, C, or D aka National or Local Codes A0000-D9999. *If use of such a code is necessary for patient care, contact Service Desk and file a request for the code to be added to VBECS.*

### **Configure Daily QC**

This option pertains only to routine reagents.

When there are partially or currently QC'd racks in the system, the user may not change the QC template.

### **Configure Division**

Changes made to the division setting are not applied to a user until they re-connect to VBECS after the change is made. It is highly recommended that all users log off VBECS prior to any configuration setting change.

### **Discard or Quarantine**

When a user retroactively updates a unit status through this option, the Unit History Report displays the updated information, but does not display the date the change was made.

The option does not accommodate the entry of a second user for retrospective data entry.

Cost entry is capped at 999.99.

### **Display Order Alerts**

VBECS issues no notifications for order cancellations.

### Display Patient Merge Alerts

VBECs does not notify users of additional VistA patient updates or merges while the icon is displayed. When a user views an update and there is an unprocessed VistA patient merge, the Patient Alert icon remains on the screen until the Traditional Supervisor updates the patient records and clears the icon.

### Display Patient Update Alerts

VBECs does not notify users of additional VistA patient updates or merges while the icon is displayed. When a user views an update and there is an unprocessed VistA patient merge, the Patient Alert icon remains on the screen until the Traditional Supervisor updates the patient records and clears the icon.

Patient updates involving patient's with names that exceed the database standard for File #2 (30 characters) or do not have a first or last name in VistA are rejected and do not appear in VBECs. The VBECs System Administrator is notified of each rejection via an email to the address entered in VBECs Administrator "Email Alerts" field.

Patient update alerts only display when active component or diagnostic test orders are pending or active<sup>8</sup> at the time the update occurs.

### Division Workload Report

A report printed with an end date of today or greater does not print "Preliminary" in the header. Check the subsequent pages of the report and write "PRELIMINARY" on the front page of the report prior to filing. The information included in the report is accurate to the time printed. There is a possibility of a gap of information should the user not overlap the print requests. Generally, this report is printed for the day prior or a time period that does not include the current day.

### Document ABO Incompatible Transfusions

VBECs does not enforce compatibility checks between the patient and the unit.

VBECs does not update CPRS component order status when the Document ABO Incompatible Transfusion option is used.

### Edit Financial Data

When a Technologist enters a unit ID and product code for a unit that is not in final status, VBECs does not enable the Special Test Cost field. An Enhanced Technologist or higher access can change the Special Test Cost field in the Edit Financial Data option.

Cost entry is capped at 999.99.

### Edit Unit Information

VBECs does not limit the discarded plasma volume.

If errors are found in the unit's product code, division or blood type at login, supplier, or donation type, the user must inactivate the unit and reenter it correctly.

When a unit does not have a patient association, editable fields are available to an >> Enhanced Technologist.

Only units in a Limited or Available status may be edited.

Inactivation of the unit record is required when the user needs to change the unit's information recorded at login: unit identification number, blood type, product code, unit expiration date, unit donation type, VA division or supplier.

<sup>8</sup> Active: VBECs order that is not rejected, canceled or completed. When the patient update is received and the order is 'active', the update is displayed regardless of order cancelation until it is viewed by a VBECs user.

**Enter Daily QC Results**

Additional user-defined reagent types not available in Configure Daily QC or Enter Daily QC Results are not part of a reagent rack's daily QC testing.

**Equipment**

VBECs does not check for duplicate entries of equipment names and identifiers.

There is no retrospective data entry or user selection for this option.

**Exception Report**

VBECs does not accommodate online review (verification, signature) of reports.

Only the first twenty patient antigens will print on the report when an exception is generated for a unit issued with unsatisfied antigen negative requirements.

In some areas of the report, the user ID is displayed instead of the user name.

Not all exception types use the date and time of the save as the date and time of the exception.

Modification exceptions do not display full (eight-digit) product codes for ISBT 128 labeled units. The sixth, seventh, and eighth digits are not included. Obtain the full ISBT 128 product code from the Unit History Report, as necessary.

**Finalize/Print TRW**

The user cannot invalidate or change a finalized TRW. Contact the Service Desk if a TRW needs to be updated after finalization.

Transfusion reaction reports are not available to clinicians through CPRS: they must be printed and filed with patient charts.

VBECs displays these default values for transfusion reactions converted from VistA to VBECs. They do not reflect actual interpretations or results:

Converted Transfusion Reaction	Default
Pre-Transfusion Specimen Checks, Hemolysis Checks OK?	Yes
Post-Transfusion Specimen Checks, Hemolysis Checks OK?	Yes
Symptoms	No symptoms identified.
Implicated units	No units implicated in reaction.
Serologic tests	No serologic tests performed.

**Free Directed Unit For Crossover**

The unit must not be in a final status to release the restriction.

This option does not allow the removal of patient restrictions for most of the ISBT donation types that are eligible for crossover.

This option cannot process multiple directed units simultaneously.

**Incoming Shipment**

VBECs does not compare the ABO/Rh of an autologous unit or display a message when the ABO/Rh is incompatible with the restricted for patient.

Incoming Shipment
When a product type, code, or local code is not in VBECS or a blood supplier creates a product code, contact the Service Desk. See Customer Support for contact information.
The user may not save the unit record until the donation type and its requirements are satisfied.
No longer supports the entry of blood products labeled in Codabar label type to new or existing invoices.
The unit expiration [Maximum Storage Time (MST)] is set to the longest possible storage time based on product type, core condition, attributes, and available references as a guide. The MST is set at a higher level than an individual collection product code. VBECS does not know the specific collection details. It is the responsibility of each user to accurately enter and process any one blood unit based on its expiration date/time as labeled by the collection facility.
When a user inadvertently enters a unit ID that is longer than 13 characters for an ISBT 128 unit, VBECS accepts the entry but will not associate it with the scanned unit ID.
The display of Special Testing for a unit is limited to 253 characters in Incoming Shipment. All of the special testing is saved in VBECS and is available on reports.
The display label that is built when logging a blood product into VBECS does not change from volunteer donor to autologous when the donation type is changed.
When blood products intended for quarantine are scanned, VBECS does not automatically quarantine them: the user must do so.
VBECS does not accept scanned information for some modified ISBT 128 blood units. VBECS accepts entered information in these instances.
VBECS does not recognize scanned labels for para-Bombay or Bombay ISBT units.
VBECS does not check whether blood products with the same unit ID and different product codes are the same blood type.
A site may receive duplicate unit IDs. VBECS cannot ensure that these IDs represent products from the same donor.
When a user scans a product code or expiration date barcode in the Unit ID field, VBECS does not display the entered information or an error icon.
When a user scans an ISBT product ID with "0" as the donation type (sixth character), VBECS indicates that the donation type is not specified. When the user selects a donation type from the drop-down menu, VBECS replaces "0" with a recognizable character in the product code text box and stores it as such. When the user rescans the product ID, he must manually replace "0" or search using the unit ID alone, or the unit ID and the first five characters of the product ID.
Blood product code entries in the blood product table include those from ICCBBA for ISBT 128. Product codes created after the release of VBECS cannot be entered until the updated tables are incorporated through patches to VBECS. Refuse acceptance of the unit, if possible. Users may not edit the blood product table.
ISBT 128 tables include blood product codes labeled as not for manufacturing or transfusion. VBECS does not restrict such products from being entered, issued, or transferred (local policy controls their use) and does not accommodate blood modification for these products.
A user cannot bring in pooled ABO or pooled Rh types for plasma or FFP. Cryoprecipitate is <b>not</b> affected.
No entry of an ICCBBA Product Description Code starting with A, B, C, or D aka National or Local Codes A0000-D9999. <i>If use of such a code is necessary for patient care, contact Service Desk and file a request for the code to be added to VBECS.</i>

<b>Invalidate Test Results</b>
A user may not inactivate a component task in this option.
This option does not release units from patient assignment. The user must release “assigned” units before invalidating the test results.
To maintain a clear record and system checks, the user must invalidate both original and repeat ABO/Rh tests.
Do not invalidate part of the configured test. Doing this will put the specimen into a state where it cannot be processed; invalidate all tests configured as part of the TAS regardless of which sub test (ABO/Rh, ABS or DAT) is the problem.

<b>Issue Blood Components</b>
VBECS does not check for restricted units (not assigned to the patient) received between selection and issue.
All issue-to locations must be valid entries in the VistA hospital location file.
If VistA link is not available, the user may proceed with the issue process but the current issue-to hospital location is not recorded. All of the associated report records display “VistALink Down” in the Issue To location field.
ISBT 128 tables include blood product codes labeled as not for manufacturing or transfusion. VBECS does not restrict such products from being entered, issued, or transferred (local policy controls their use), but does not accommodate blood modification for these products.
When a site has configured subservient VistA institutions to the VBECS division, users can only issue units to locations in the primary VistA. Sites may add locations to VistA to account for the relocations to the subservient VistA institutions.
BCE information applies ONLY when a BCE COTS product is available for communication and is configured.
Changing a patient's transfusion requirements (component requirements, antigen negative requirement or antibody identified) after a unit is crossmatched does not provide an alert at Issue Unit regarding the type of crossmatch (electronic or serologic) performed on a unit available for issue.
VBECS asks for a comment when issuing a unit that is assigned to two different patients. Enter a comment to issue the unit or release one of the patient assignments before issuing. The comment will appear on the Unit History Report in the Issue section.

<b>Justify ABO/Rh Change</b>
VBECS does not allow the selection or issue of certain blood components that may appear ABO/Rh compatible when discrepant ABO/Rh results are not justified.

<b>Local Facilities</b>
A user may: <ul style="list-style-type: none"> <li>• Activate and edit collection facility records distributed with VBECS.</li> <li>• Add and locally edit active collection facilities.</li> <li>• Activate and/or deactivate local collection facilities.</li> <li>• Edit a division name, which may result in a mismatch between VistA and VBECS division names.</li> <li>• The Local Facility Address maximum length is 45 characters including spaces to allow the address to display on VBECS reports. The third line of the facility address is not displayed on VBECS reports.</li> </ul>

<b>Log In Reagents, Update Inventory, View/Print Inventory, Maintain Minimum Levels</b>
VBECS does not display a reagent report by invoice number.
The unsatisfactory portion of a shipment must be recorded individually.
There is no retrospective data entry or user selection for these options.
Each site must set a standard for recording quantity: it may use the number of cases or the number of units within the cases.
When a user enters a lot number in the Lot # field and clicks the <b>magnifying glass</b> , VBECS displays only the first instance of a duplicate lot number.

<b>Log into VBECS and VistA</b>
VBECS automatically logs a user with access to only one division into that division.
Connectivity by IAM services access approval (PIV Card) is not possible when IAM services are not available.
An Access and Verify code is required to connect to VistA when the IAM services are not available.
Connectivity is not possible when VistA is not available.
The user may log into multiple terminals simultaneously, but accessing VBECS in multiple sessions is no longer possible due to the way the PIV card works.
The Windows Security screen may be hidden when a user attempts to restore a VistALink connection. For the Windows Security screen to appear, the user must drag it from behind another window in order to select a certificate and log in.

<b>Maintain Specimen</b>
When a search by UID causes VBECS to time out, repeat the search by UID and patient name or by patient name only.
Specimen recollection requests must be made offline.
Designating a specimen as "unacceptable" after accepting a CPRS order closes that order and requires a new CPRS order and sample.

<b>Maintain Unit Records</b>
VBECS does not limit the discarded plasma volume.
If errors are found in the unit's product code, division or blood type at login, supplier, or donation type, the user must inactivate the unit and reenter it correctly.
When a unit does not have a patient association, editable fields are available to an ►► Enhanced Technologist.
Only units in a limited or available status may be edited.
Inactivation of the unit record is required when the user needs to change the unit's information recorded at login: unit identification number, blood type at login, product code, unit expiration date, unit donation type, VA division at login or supplier.

<b>Modify Units</b>
VBECS allows modification of units only in limited, assigned, crossmatched, and available statuses.
VBECS presents the conservative expiration date for target units. VBECS does not know the specific collection details. It is the responsibility of each user to accurately enter and process any one blood unit based on its expiration date/time as labeled by the collection facility.
Pooling of units that have the attribute from V0013 (irradiated) and attribute V0021005 (psoralen treatment) does not produce a target.
Weld controls are disabled after the user makes their initial selection.

<b>Modify Units: Pool Units</b>
VBECS allows modification of units only in limited, assigned, crossmatched, and available statuses.
When a user adds a biohazardous unit to a pool, VBECS does not designate the pool as biohazardous on the Blood Transfusion Record Form (BTRF) or Caution Tag.
Weld controls are disabled after the user makes their initial selection.

<b>Modify Units: Split a Unit</b>
A unit previously split in VBECS cannot be split again. Only an ISBT unit that was split outside of VBECS can be split again.
A unit pooled in VBECS may not be split.
A unit that has a division code present upon receipt in incoming shipment may be split in VBECS.
Weld controls are disabled after the user makes their initial selection.

<b>Order Alerts</b>
This is an alert only and contains limited information from the CPRS order.
This option does not create a mechanism to accept or reject orders or specimens or define reports for viewing orders.
The user may select the option to display printer alerts only when the designated printer is configured and available.
The printer must be configured by the System Administrator.

<b>Order History Report</b>
VBECS does not display the identity of the CPRS user who discontinues a signed order in CPRS. The CPRS order details option contains all pertinent order information.
Canceled orders are included on the Order Summary Report without a column/data to help identify them as canceled. The Single Order History Report includes the canceled order information. When the order is also canceled in VistA, which would be the norm, there is a lab report that can be referenced.

<b>Order Reflex Tests</b>
A crossmatch (XM) and a repeat XM cannot be ordered unless they are associated with a specimen with a future expiration date and a phlebotomist is associated with that specimen.
When a user selects weak D for the antigen typing test and the selected patient is Rh positive, VBECS warns that weak D testing cannot be performed. There is no override and VBECS does not add the test.

<b>Outgoing Shipment</b>
A user may edit only unconfirmed invoices.
The number of tests displayed on an invoice is limited to 8,000 characters per unit.
When the user creates an invoice and cancels before saving it, VBECS excludes the invoice number from use.
When assigned or restricted, the patient first name and ID are not printed on the outgoing shipment document to maintain patient privacy.
Cost entry is capped at 999.99.

<b>Patient Information Toolbar</b>
Recent Orders is showing Order Status values like “Not Started” and “Filled”, which are actually Task Statuses.

<b>Patient History Report</b>												
When units are restricted for a patient during Incoming Shipment, VBECS does not display them on the Patient History Report.												
VBECS displays these default values for transfusion reactions converted from VistA to VBECS. They do not reflect actual interpretations or results:												
<table border="1"> <thead> <tr> <th><b>Converted Transfusion Reaction</b></th> <th><b>Default</b></th> </tr> </thead> <tbody> <tr> <td>Pre-Transfusion Specimen Checks, Hemolysis Checks OK?</td> <td>Yes</td> </tr> <tr> <td>Post-Transfusion Specimen Checks, Hemolysis Checks OK?</td> <td>Yes</td> </tr> <tr> <td>Symptoms</td> <td>No symptoms identified.</td> </tr> <tr> <td>Implicated units</td> <td>No units implicated in reaction.</td> </tr> <tr> <td>Serologic tests</td> <td>No serologic tests performed.</td> </tr> </tbody> </table>	<b>Converted Transfusion Reaction</b>	<b>Default</b>	Pre-Transfusion Specimen Checks, Hemolysis Checks OK?	Yes	Post-Transfusion Specimen Checks, Hemolysis Checks OK?	Yes	Symptoms	No symptoms identified.	Implicated units	No units implicated in reaction.	Serologic tests	No serologic tests performed.
<b>Converted Transfusion Reaction</b>	<b>Default</b>											
Pre-Transfusion Specimen Checks, Hemolysis Checks OK?	Yes											
Post-Transfusion Specimen Checks, Hemolysis Checks OK?	Yes											
Symptoms	No symptoms identified.											
Implicated units	No units implicated in reaction.											
Serologic tests	No serologic tests performed.											

<b>Patient Search Screen</b>
VBECS updates information from VistA File #2, not VistA File #20. Name changes made in VistA File #20 will not appear in VBECS. Align File #2 with those changes to update VBECS.

<b>Patient Testing: Record a Patient ABO/Rh</b>
VBECS does not allow entry of ABO or Rh subgroups as a 'group' and/or 'type' when the previous blood group/type is "NR", indicating no record of previous testing. See FAQ ABO Subgroup Interpretation and contact the Service Desk for assistance, prior to adding patient transfusion records in VBECS, when possible.
When the patient has no previous ABO/Rh test record (NR), the ABO/Rh test associated with the Type and Screen test (TAS) must be completed and saved prior to entering and saving a repeat ABO/Rh test on this specimen.
A user is permitted to select two partially completed tests with different test phases (e.g., AHG only and All Phases) along with a not started test. The testing phase for the not started test will present as AHG only even though results for all phases are required by VBECS. Entering only AHG test results for the specimen that was not started and saving will result in a partially completed test that cannot be completed or invalidated. Contact the Service Desk when you encounter this problem. Specimens that cannot be completed or invalidated require the attention of Tier 3 support. Select partially completed orders that have been previously saved with the same grid test pattern. Optionally, a not started test may be selected with these but must be tested with the same grid pattern to avoid the problem.
When canceling out of a Discrepant ABO/Rh Typing override, the interpretations are cleared, but the testing results are not cleared. Correct the testing entries or click the red 'X' to clear the testing results.



<b>Patient Merge</b>
The Traditional Supervisor or above must update patient records manually.
VBECs does not notify users of additional VistA patient updates or merges while the icon is displayed. When a user views an update and there is an unprocessed patient merge, the Patient Alert icon remains on the screen until the Traditional Supervisor completes the update and clears the icon.
Patient merges involving patient's with names that exceed the database standard for File #2 (30 characters) or do not have a first or last name in VistA are rejected and do not appear in VBECs. The VBECs System Administrator is notified of each rejection via an email to the address entered in VBECs Administrator "Email Alerts" field.
VBECs displays duplicate merge events to the user for merge. Users can process one of the merge events manually and clear both merge alerts. The Master Veteran Index team is working on a patch to discontinue sending a duplicate message.

<b>Patient Testing: Cancel an Active Order</b>
Cancellation of one order does not affect other orders placed with it.
Before a user may cancel a partially completed task, he must invalidate the results for that task, thereby changing its status to "not started."

<b>Patient Testing: Enter Antibody Identification Results</b>
When an antigen negative requirement is not configured for an antibody, the user may enter the requirement for a patient through Special Instructions & Transfusion Requirements: Enter a Transfusion Requirement.

<b>Patient Testing: General Instructions</b>
This test record is not editable after the user completes a test and saves it to the database. When a correction needs to be filed, a corrected report must be generated, as described in Invalidating Patient Test Results.
User selection of partially completed testing grids with different phases of testing creates a testing discrepancy in VBECs. Batch testing is limited to testing at the same phases.
Limited batch testing to partially completed orders with the same testing grid selection, either All Phases or AHG Only.
VBECs cannot compare test results for a patient when two users are entering test results for the same patient at the same time.
Cost entry is capped at 999.99.

<b>Patient Testing: Pending Task List</b>
This option is for diagnostic, blood component, and reflex test orders placed in or directed to the user's division.
The PTL does not include daily quality control, equipment maintenance schedules, or pending ABO/Rh confirmation and auxiliary tests on red blood cell units.
Users may perform testing for no more than four patients in one VBECs session.

<b>Patient Testing: Record a Crossmatch</b>
The specimen associated with testing must have an expiration date and time that are later than the testing date and time entered in Patient Testing: Record Patient Test Results. When this is not the case, VBECs warns that the patient specimen expired. There is no override.
This option does not include electronic crossmatch (eXM).

**Patient Testing: Record a Direct Antiglobulin Test**

VBECs does not check whether the lot and vial numbers selected for the positive control cells are different from those of the negative control cells.

Comments entered in the DAT POS and NEG control cell rows are not displayed on reports in VBECs or CPRS. Enter comments only in the patient test row.

**Patient Testing: Record a Patient Antibody ABO/Rh**

VBECs does not allow entry of ABO or Rh subgroups as a 'group' and/or 'type' when the previous blood group/type is "NR", indicating no record of previous testing. See FAQ ABO Subgroup Interpretation and contact the Service Desk for assistance, prior to adding patient transfusion records in VBECs, when possible.

When the patient has no previous ABO/Rh test record (NR), the ABO/Rh test associated with the Type and Screen test (TAS) must be completed and saved prior to entering and saving a repeat ABO/Rh test on this specimen.

**Patient Testing: Record a Patient Antibody Screen**

This option is for patient testing only.

**Patient Testing: Record a Patient Antigen Typing**

Antigen typing applies only to red blood cell antigens. [This option does not address platelet, histocompatibility locus antigen (HLA), or Immunoglobulin A (IgA) antigens.]

VBECs does not check:

- The patient antigen typing with the specificity of any patient's antibodies entered.
- Whether the lot and vial numbers selected for the positive control cells differ from those of the negative control cells.

The patient antigen typing result is not available to CPRS.

**Patient Testing: Record a Transfusion Reaction Workup**

Adverse reactions to transfusion detected at a later time (disease transmission) are not recorded through this option. Refer to Transfusion Complications Report.

Each tab is completed and saved and is not editable. To change information on a previously completed tab, the TRW must be invalidated and corrected in its entirety.

If comments are entered when a TRW is finalized as type "Other," they won't show on the CPRS Blood Bank Report.

**Patient Testing Worklist and Testing Worklist Reports**

Reports do not include Transfusion Reaction Workup (TRW) test results.

VBECs does not accommodate online review (verification, signature) of reports.

The Rack QC Testing Worklist Report section of the Testing Worklist Report does not contain a Processed By column. Maintain the original testing records for the QC that was performed offline with the testing tech information. Do not change the testing tech name when entering results in VBECs so that the logged in tech that enters the results is included on the Testing Worklist report.

<b>Post-Transfusion Information</b>
VBECS cannot update a unit record to presumed transfused when a user has that unit record open (locked) in Enter Post Transfusion Information. VBECS will attempt to update the unit the next time the presumed transfused background routing runs if the unit remains in an issued status.
If post-transfusion information is not transmitted to VBECS, the user will need to enter that information to update the record from presumed transfused to transfused.
The transfusion comments and volume transfused entered in the Transfusion Verification application as an update to a completed transfusion are not sent to VBECS.
Transfusion reaction symptoms are not sent to VBECS by the Transfusion Verification application unless a transfusion reaction has also been indicated for that event.

<b>Print Unit Caution Tag &amp; Transfusion Record Form</b>
VBECS cannot reprint a Caution Tag on an expired unit or expired patient order.
VBECS does not print a unit's biohazardous status on the BTRF or Caution Tag.
If a unit has been crossmatched by two different technologists, the Compatibility Interpretation section of the BTRF displays the technologist that performed the earlier XM testing. Handwrite the missing current technologist initials on the printed BTRF.

<b>Prolonged Transfusion Time Report</b>
This report does not include presumed transfused units or units issued to a remote storage location.
Units issued to a remote storage location will display on the report when the transfusion start time is greater than 30 minutes after issue time.
Units issued to a remote storage location should not display to have a prolonged issue time as this setting indicates that the unit is issued to a monitored location maintained at the proper temperature for the product type.
The Delayed (minutes) column on the report is not subtracting 30 minutes when the Transfusion Start time is greater than 30 minutes from the time of issue. Handwrite the correct Delayed (minutes) on the printed report.

<b>Reagents</b>
VBECS does not display a reagent report by invoice number.
The unsatisfactory portion of a shipment must be recorded individually.
There is no retrospective data entry or user selection for these options.
Each site must set a standard for recording quantity: it may use the number of cases or the number of units within the cases.
When a user enters a lot number in the Lot # field and clicks the magnifying glass, VBECS displays only the first instance of a duplicate lot number.
Free text details can be added only when "Other" is selected as the canned comment.
When reagents with the same Manufacturer Reagent Name and Lot Number are entered from multiple invoices, only one of the reagents is visible for update.
Reagents that have had their Number of Vials Available set to zero (0) are no longer visible on the Update Reagent Inventory screen or the Reagent Inventory Report. Print a copy of the Reagent Inventory Report prior to setting the Number of Vials Available to zero (0).

<b>Release Units From Patient Assignment</b>
This option does not apply to the release of patient restrictions (autologous and directed).
When a patient, specimen, or blood unit expires, VBECS does not automatically release an assigned blood unit: a user must release the unit.
Users should select and release units in a timely manner to avoid the units being issued by another user prior to the release process completion.

<b>Remove Final Status</b>
This option applies only to the release of units in final statuses.

<b>Screen Settings</b>
VBECS was designed to operate with the settings described in this section to display complete screens. Failure to maintain these settings may result in partial display of safety-critical information.

<b>Select Units</b>
When a user checks the “D Pos” or “D Neg” check box to modify a search for blood, VBECS returns no units, regardless of the availability of units.
VBECS displays a component order as “Filled” when the number of units transfused and the number of units ordered are the same and continues to display the component order until the order expires. CPRS displays a component order as “Completed”. Therefore, CPRS displays the order as “Completed” before VBECS displays the order as “Filled.” The associated LAB order will then also be marked as “Completed”.
A Specimen UID must be accepted in VBECS to associate it with a component order. Any VBECS maintained TAS specimen UID may be associated with a component order by entering the number in the UID field.
A system error occurs when attempting to associate a specimen with an order when no unit is selected. Associate the specimen UID in Accept Order when possible.
When associating a specimen with a patient in the “Select Unit for a Patient” window, the “Expires” field within the “Associate with Specimen” panel displays the time in AM/PM format. VBECS normally displays date/time fields in “military” time format. The workaround is to look in Maintain Specimen and check the expiration time military time format.
When an Rh positive unit is selected for an Rh negative patient with an antigen negative requirement for D, the user receives a message about unit not satisfying patient negative requirements for D instead of a message about unit being positive for D. The proper message is displayed upon unit issue.

<b>Special Instructions &amp; Transfusion Requirements: Enter a Transfusion Requirement</b>
Inactivation is defined and restricted by security level.
VistA Special Instructions (SIs) from database conversion do not become Transfusion Requirements (TRs): they remain SIs. SIs are information only and are not enforced unless entered in the Component Requirements tab in Special Instructions & Transfusion Requirements.

<b>Special Instructions &amp; Transfusion Requirements: Enter and Remove Special Instructions</b>
VBECS does not enforce Transfusion Requirements (TRs) or antigen negative requirements entered through this option.
SIs moved to VBECS during the database conversion may contain information that must be entered as TRs in VBECS. Inactivate the old SI and create a TR and new updated SI: the database conversion software cannot automatically translate the SI from database conversion into a rule-based TR.
VBECS will truncate SI text longer than 255 characters from the VistA database conversion.

<b>Supplies</b>
The division's modification processes may allow or require the user to select previously entered supplies for association with a blood unit's modification. An inventory of containers (transfer bags) and sterile connection device (SCD) wafers is <i>required</i> for various modification processes.
There is no retrospective data entry or user selection for this option.
The unsatisfactory portion of a shipment must be recorded individually.
Each site must set a standard for recording quantity: it may use the number of cases or the number of units within the cases.

<b>Transfusion Complications Report</b>
The report does not include units in an issued status.
The report cannot be scheduled.

<b>Transfusion Effectiveness Report</b>
The report does not include VBECS diagnostic tests.

<b>Transfusion Reaction Count Report</b>
Transfusion Reactions are only included on the report if the data were entered/updated during the selected date range. This means that retrospective entry of transfusion reactions may not display in the correct date range. The recorded date of the reaction is not considered in the selection criteria. Rerun the report for a date range that includes the date the data was entered or updated. For example, if a transfusion reaction that occurred on February 28 is retrospectively entered on March 1, it will only display on a report that includes March 1 in its date range.

<b>Transfusion Requirements Report</b>
Deceased patient records do not appear on the report when they are generated after the patient has been marked as deceased in the VBECS database via a VistA Death Notification Update.
The cumulative report is not displaying all patients on file. A patient with ABO/Rh testing and no other requirement appears on the partial Transfusion Requirements report, but not on the cumulative report.
A report printed with an end date of today or greater does not print "Partial" in the header.

<b>Unit Antigen Typing</b>
Antigen typing applies only to red blood cell antigens. This option does not address platelet, histocompatibility locus antigens (HLAs), or Immunoglobulin (IgA) antigenicity.
Only blood units of the "WHOLE BLOOD" and "RED BLOOD CELLS" ICCBBA component classes can be selected for inclusion in the antigen type worklist.
Antigen typing is not required on products with = > 2 mL of RBC contamination that are not Red Blood Cells or Whole Blood.
VBECs does not check whether the lot and vial numbers selected for the positive control cells differ from those of the negative control cells.
D antigen typing must not be combined in batch testing of Rh positive units.
The sort order of the selected units may not be inherited by the testing grid. Organize physical test materials according to the order of units presented on the testing grid.


<b>Unit History Report</b>
The report is not multidivisional.
When a user retroactively updates a unit status through Discard or Quarantine, the Unit History Report displays the updated information, but does not display the date the change was made.
The report will not indicate if a unit was returned to shipper, or shipped to another facility. This information can be found in the Cost Accounting Report.
The report will display incorrect information in the Cost column for Splits/Divides and Pools. See the Cost Accounting Report for this information.
The report does not display the Antigen Testing Phase Change Exception. See the Exception Report for this information.
When a user retroactively updates a unit status through the Discard or Quarantine option, the Unit History Report displays the updated information, but does not display the date the change was made.

<b>Update User Roles</b>
One role at a time may be assigned to a user at a division. The role may be reset.
VBECs does not allow users to edit or delete the sole Administrator/Supervisor.

<b>Updated Patients and Deceased Patients</b>
Demographic and death event updates do not display information for patients without pending or active orders in VBECs. Updates will occur without user input.
VBECs does not notify users of additional VistA patient updates or merges while the icon is displayed. When a user views an update and there is an unprocessed patient merge, the Patient Alert icon remains on the screen until the Traditional Supervisor completes the update and clears the icon.
When VBECs notifies a user of an update to a patient's prefix or suffix, VBECs does not display the updated information: the user must access VistA to view the updated information.
Suffix data may be outdated but will be refreshed upon a new order for a patient.
When a VistA patient update removes a patient's middle name, VBECs displays both the Patient Name and the Previous name in the Patient Name column. The Previous Name column displays the previous name correctly. Once the user closes and re-opens the Patient Update window, the subsequent display of the patient information is correct.

### View Special Instructions and Transfusion Requirements

Changing a patient's antigen negative requirement or antibody identified after a unit is crossmatched does not provide an alert at Issue Unit regarding the type of crossmatch (electronic or serologic) performed on a unit available for issue.

This icon  will be disabled if the icon is displayed within the patient search window, because special instructions and transfusion requirements are already displayed automatically. This icon will also be disabled if the icon is displayed within the patient's SIs and TRs dialog. This icon will always be disabled if the selected patient does not have SIs or TRs to display.

### Workload Codes

Transfusion-only sites must not map workload codes to processes such as ABO/Rh testing not performed on site.

Retrieved LMIP/NLT codes from VistA may have multiple CPT codes. Inactivating any of the associated CPT codes in VBECS will inactivate the LMIP/NLT code. Editing of the LMIP/NLT code associations must be done in VistA.

Invalidating split units will result in extra workload equal to the number of splits created applied to the total number of units logged in. This occurs when split units are incorrectly processed in VBECS.

This page intentionally left blank.



## ***Appendix E: Known Defects and Anomalies***

Copies of *VBECS 2.3.1 Known Defects and Anomalies* may be obtained from the VA Software Document Library Web site (<https://www.va.gov/vdl/application.asp?appid=182>).

This page intentionally left blank.

## **Appendix F: System Validation**

### **Introduction**

Each facility is responsible for validating the VBECS installation, operation, and performance before using VBECS in a live (production) environment. This includes evaluating local intended uses of the software and changing settings to conform to the facility's procedures, processes, and policies. Use internal operating procedures and policies to validate that the software performs in accordance with your site's parameters. Determine the level of validation required for changes to hardware and software.

*This section describes only basic system validation. Sample test scripts are available from the National Training & Education Office (NT&EO).*

### **Validation Plan**

Write a validation plan with clearly stated goals and strategies to demonstrate that the system hardware, application, new features, and software patches function as expected in your facility's environment. Include descriptions of documents, hardware and software Verification & Validation (V&V) tools, techniques, methods, and operating and test environments to be used. Determine and document when an option is not applicable to your facility and does not require testing.

You may separate this plan into three major sections:

#### **Installation Qualification**

The Installation Qualification (IQ) section includes system identification, definition of physical properties, interfaces and peripheral devices, and general functionality. Determine what types of testing must be done. Create test cases in accordance with your internal procedures and intended use.

Apply review, analysis, and testing techniques to determine whether VBECS hardware and peripheral components comply with requirements, including functionality and attributes such as consistency, safety, and security.

#### **Operational Qualification**

The Operational Qualification (OQ) section includes testing the software (the primary validation activity). Use your OQ to challenge VBECS and the connection to VistA. Base the number and types of tests on the criticality of the software function. The focus of this validation is to test the limits of what the system allows you to do, not how the system will be used. Each facility will have a different level of validation based on local practices.

Analyze risks to ensure that the scope and degree of validation are appropriate. (A system error that may result in direct and severe patient harm requires more detailed and rigorous testing than one that may have minor consequences.)

Create a testing environment as much like the production environment as possible. Design test cases to:

- Address critical software functions.
- Detect failures and flaws.
- Verify and challenge system functions and limitations, and user documentation.

Verify data from VistA displayed in VBECS and data from VBECS displayed in VistA. (Results will vary based on your VBECS settings.) Evaluate the output of the testing and determine whether the test passed or failed.

Maintain evidence of software testing, as part of the validation record, in accordance with standards established by accrediting agencies associated with the blood bank and laboratory. Include, at minimum, your inputs, outputs, and evidence that tests performed were satisfactory or unsatisfactory. (You may record input manually, make copies of unit labels or patient information, capture screen shots<sup>9</sup> from VistA and VBECS, print various reports, etc.)

## Performance Qualification

The Performance Qualification (PQ) section focuses on the integration of the application and processes into your facility. This begins with the revision of Standard Operation Procedures (SOPs) and policies to include the proper use of the new system. Once the SOPs are in place and validated, train the users.

Ensure that training materials and records are complete before allowing users to operate the system.

Document the validation, including:

- Personnel participating in validation activities
- Hardware and software change controls
- Testing results
- SOP reviews, training plans, installation and implementation
- Evaluation and acceptance
- Review and approval authority and dates

## Prerequisites

Perform these tasks for test and production environments in VBECS before you test the application:

- Install required patches in your VistA test and production environments.
- Install VBECS.
- Complete the database conversion.
- Record system settings.

---

<sup>9</sup> To make a copy of the active window, press **Alt+Print Screen**. To copy the entire screen as it appears on your monitor, press **Print Screen**. To paste the image into a file in an Office program, open the file you want to paste the image into, click **Edit**, and then click **Paste**.

## Record System Settings

These sections describe how to capture system settings in preparation for validation.

### Configure Division

A user defines site parameters that affect how VBECS behaves at a specific division. VBECS may be configured to function differently at each division within a consolidated database. Site parameters may be redefined. For more information, see Appendix F: Work Process Flowcharts.

#### User Roles with Access to This Option

➤➤➤➤➤ Administrator/Supervisor

#### Preparation for Testing

- 1) Select **Configure Division**.
- 2) Record:
  - ICCBBA registration number: \_\_\_\_\_
  - Blood Bank Medical Director's name as you want it to appear on reports: \_\_\_\_\_
  - Latest expiration date (default: three days) for specimens, in accordance with local policies and procedures: \_\_\_\_\_
- 3) Is your division full service (FS) or transfusion only (TO)? \_\_\_\_\_
- 4) If your site does not perform on-site testing, continue at Order Alerts.
- 5) If your division is full service, do you want to enable electronic crossmatch? \_\_\_\_\_
- 6) Print, sign, date, and save an Audit Trail Report for your validation record.

### Configure Testing: Full Service

The user sets testing preferences in accordance with a division's medical policies. For more information, see Appendix F: Work Process Flowcharts.

#### User Roles with Access to This Option

➤➤➤➤➤ Administrator/Supervisor

#### Preparation for Testing

- 1) Set testing preferences.
- 2) Capture and save a screen shot of each piece of information.
- 3) Print, sign, date, and save an Audit Trail Report for your validation record.

## Product Modifications: Full Service

The user enables the types of modifications made at a division. For more information, see Appendix F: Work Process Flowcharts.

### User Roles with Access to This Option

➤➤➤➤➤ Administrator/Supervisor

### Preparation for Testing

Modification processes are disabled by default. You must activate modification types performed at your facility. You may select none, some, or all. Enter a processing cost for each modification type, as applicable. You may update these settings later.

**Table 28: Modification Types and Costs**

Modification Type	Modification Processes		Processing Cost
	Enabled	Disabled	
Thaw			
Pool			
Thaw/Pool Cryo			
Split/Divide			
Irradiate			
Leukoreduce			
Volume Reduce			
Wash			
Rejuvenate			
Freeze			
Deglycerolize			

- 1) Capture and save a screen shot of each piece of information.
- 2) Print, sign, date, and save an Audit Trail Report for your validation record.

## **Order Alerts**

The user configures diagnostic test or component order alerts that appear at login, and turns them on or off. The alerts notify users when new orders are entered, updated, or canceled in VistA. For more information, see Appendix F: Work Process Flowcharts.

### **User Roles with Access to This Option**

➤➤➤➤➤ Administrator/Supervisor

### **Preparation for Testing**

- 1) Capture and save a screen shot of each piece of information.
- 2) Print, sign, date, and save an Audit Trail Report for your validation record.

## Configure Daily QC: Full Service

The user sets up the routine reagent racks and QC template for the division before performing daily reagent QC. For more information, see Appendix F: Work Process Flowcharts.

### User Roles with Access to This Option

➤➤➤➤➤ Enhanced Supervisor

### Preparation for Testing

- 1) If you perform daily reagent rack quality control using a commercial kit, select the commercial kit template. If not, select the non-commercial kit template. Template selected: \_\_\_\_\_
- 2) Daily alert time at which users will be prompted to perform QC: \_\_\_\_\_
- 3) Do you want to use letters or numbers as rack names? \_\_\_\_\_
- 4) Maximum number of reagent racks for daily use: \_\_\_\_\_
- 5) Primary enhancement medium: \_\_\_\_\_
- 6) Secondary enhancement medium: \_\_\_\_\_

The template contains the minimum number of tests required to test the reagents. Select additional tests that are part of your daily reagent rack QC, in accordance with local policies and procedures.

- 7) Capture and save a screen shot of each piece of information.
- 8) Print, sign, date, and save an Audit Trail Report for your validation record.



## **Maintain Minimum Levels: Full Service**

The user sets minimum stock levels for reagent types used in the division. For more information, see Appendix F: Work Process Flowcharts.

### **User Roles with Access to This Option**

➤➤➤➤ Traditional Supervisor

### **Preparation for Testing**

You may enter the minimum number of vials for each reagent type that your blood bank stocks to alert staff when it reaches a critical level.

- 1) Capture and save a screen shot of each piece of information.
- 2) Print, sign, date, and save an Audit Trail Report for your validation record.

## **Log In Reagents: Full Service**

A user records the receipt of specific reagents by type. The reagent record includes the date the shipment was received, reagent name, lot number, expiration date, manufacturer, and number of vials received in a shipment. For more information, see Appendix F: Work Process Flowcharts.

### **User Roles with Access to This Option**

All users

### **Preparation for Testing**

- 1) Enter reagent types and quantities used. (For QC validation, enter a lot number that was received and found unsatisfactory, and a lot number that expires at 23:59 on the day that you test the reagent rack QC.)
- 2) Capture and save a screen shot of all entries.
- 3) Print, sign, date, and save the View/Print Inventory (Reagent) Report for your validation record.

## Blood Products

The user activates, deactivates, and edits a limited number of variables for an existing blood product site parameter. VBECS provides an ISBT 128 Blood Product reference table to facilitate these and other actions. For more information, see Appendix F: Work Process Flowcharts.

### User Roles with Access to This Option

All users

### Preparation for Testing

Activate blood products here or during incoming shipment. It may be helpful to set up your most common products before entering blood inventory in the system.

Healthcare Common Procedure Coding System (HCPCS) codes are provided by your hospital's VistA file. Select the code that is associated with the blood product.

- 1) Use the sample worksheet to record information.
- 2) Capture and save a screen shot of each piece of information before continuing.
- 3) Print, sign, date, and save an Audit Trail Report for your validation record.

**Table 29: Sample Blood Products Worksheet**

Blood Product		HCPCS	Product Shippers			
Product Code	Short Product Name	HCPCS Code	Shipper Name	FDA Reg. #	Cost (\$)	Return Credit %
04020	RBC ACD-A	P2028	Abington Memorial Hospital	2577690	\$150.00	0

**Table 30: Blood Products Worksheet**

Blood Product		HCPCS	Product Shippers			
Product Code	Short Product Name	HCPCS Code	Shipper Name	FDA Reg. #	Cost (\$)	Return Credit %

## Local Facilities

The user customizes collection facility information. For more information, see Appendix F: Work Process Flowcharts.

### User Roles with Access to This Option

- All users (activate a facility)
- Enhanced Supervisor (add a facility)

### Preparation for Testing

- 1) Use the sample worksheet to record information.
- 2) Capture and save a screen shot of each piece of information.
- 3) Print, sign, date, and save an Audit Trail Report for your validation record.

**Table 31: Sample Local Facilities Worksheet**

FDA Reg. No.	Facility Name	ICCBBA Reg. No.	Facility Type		Active Facility?
			Collection Facility?	Testing Facility?	
2577690	Abington Memorial Hospital	W0002	✓		✓

**Table 32: Local Facilities Worksheet**

FDA Reg. No.	Facility Name	ICCBBA Reg. No.	Facility Type		Active Facility?
			Collection Facility?	Testing Facility?	

## Component Classes

The user configures the component class parameters for the selected division. For more information, see Appendix F: Work Process Flowcharts.

### User Roles with Access to This Option

➤➤➤➤➤ Enhanced Supervisor

### Preparation for Testing

- 1) Use the sample worksheet to record information.
- 2) Capture and save a screen shot of each piece of information.
- 3) Print, sign, date, and save an Audit Trail Report for your validation record.

**Table 33: Component Class Requirements Worksheet**

Component Class	Require Current Specimen ABO/Rh Results?	Maximum Transfusion Time <sup>10</sup>	Laboratory Test Name	Specimen Type	Threshold Result
RED BLOOD CELLS	✓ (uneditable)				
FRESH FROZEN PLASMA					
PLATELETS					
CRYOPRECIPITATE					
WHOLE BLOOD	✓ (uneditable)				
OTHER					

---

<sup>10</sup> Default is 270 minutes (4 hours, 30 minutes).

## Transfusion Complications

The user defines Laboratory tests and threshold values used to generate a report listing patients transfused within a specified period who may have developed a transfusion-associated disease. For more information, see Appendix F: Work Process Flowcharts.

### User Roles with Access to This Option

➤➤➤➤➤ Enhanced Supervisor

### Preparation for Testing

- 1) Use the sample worksheet to record information.
- 2) Capture and save a screen shot of all entries.
- 3) Print, sign, date, and save an Audit Trail Report for your validation record.

**Table 34: Transfusion Complications Worksheet**

Laboratory Test Name	Specimen Type	Threshold Result

## Transfusion Effectiveness

The user defines Laboratory tests to monitor the effectiveness of transfusions. For more information, see Appendix F: Work Process Flowcharts.

### User Roles with Access to This Option

➤➤➤➤➤ Enhanced Supervisor

### Preparation for Testing

- 1) Use the sample worksheet to record information.
- 2) Capture and save a screen shot of all entries.
- 3) Print, sign, date, and save an Audit Trail Report for your validation record.

**Table 35: Transfusion Effectiveness Worksheet**

Laboratory Test Name	Specimen Type	Threshold Result

## **Workload Codes**

The user assigns workload Laboratory Management Index Program/National Laboratory Test (LMIP/NLT) tests to processes performed through VBECS. VBECS test names are associated with VistA workload codes, used when compiling and storing data to provide workload-related reports to VistA. For more information, see Appendix F: Work Process Flowcharts.

### **User Roles with Access to This Option**

➤➤➤➤➤ Administrator/Supervisor

### **Preparation for Testing**

Print, sign, date, and save an Audit Trail Report for your validation record.

## **Canned Comments**

The user defines canned comments, by category, for a division. For more information, see Appendix F: Work Process Flowcharts.

### **User Roles with Access to This Option**

➤➤➤➤➤ Enhanced Supervisor

### **Preparation for Testing**

Print, sign, date, and save an Audit Trail Report for your validation record.

## **MSBOS**

The user defines parameters for comparing blood component orders placed by a clinician with the Maximum Surgical Blood Ordering Schedule (MSBOS). For more information, see Appendix F: Work Process Flowcharts.

### **User Roles with Access to This Option**

➤➤➤➤➤ Enhanced Supervisor

### **Preparation for Testing**

Print, sign, date, and save an Audit Trail Report for your validation record.

## **Edit Invoice Text**

The user may edit the explanatory text above the signature on the last page of a shipping invoice.

### **User Roles with Access to This Option**

➤➤➤➤➤ Enhanced Supervisor

### **Preparation for Testing**

Print, sign, date, and save an Audit Trail Report for your validation record.



## Antibodies

The user may edit certain fields in the supplied antibody table (Table 14). Use Table 36 (extracted from Table 14) to document local decisions on antigen-negative settings. For more information, see Appendix F: Work Process Flowcharts.

**Table 36: User-Defined Antigen Negative Requirements**

Antibody Name		Clinically Significant?	VBECS Antigen Negative Setting Default	Local comments and decisions on antigen-negative settings.
VBECS	Vista			
Anti-Kp(a)	Anti-Kp(a)	Yes	No	
Anti-Kp(b)	Anti-Kp(b)	Yes	No	
Anti-M	Anti-M	No	No	
Anti-N	Anti-N	No	No	
Anti-U	Anti-U	Yes	No	
Anti-P1	Anti-P1	No	No	
Anti-Le(a)	Anti-Le(a)	No	No	
Anti-Le(b)	Anti-Le(b)	No	No	
Anti-Js(a)	Anti-Js(a)	Yes	No	
Anti-Js(b)	Anti-Js(b)	Yes	No	
Anti-Lu(a)	Anti-Lu(a)	No	No	
Anti-Lu(b)	Anti-Lu(b)	No	No	
Anti-Lu3	Anti-Lu(a)Lu(b)	No	No	
Anti-C(w)	Anti-Cw	Yes	No	
Anti-C(x)	Anti-Cx	Yes	No	
Anti-E(w)	Anti-E(w)	Yes	No	
Anti-V	Anti-V	Yes	No	
Anti-Xg(a)	Anti-Xg(a)	No	No	
Anti-M(g)	Anti-M(g)	No	No	
Anti-Mi(a)	Anti-Mi(a)	No	No	
Anti-P	Anti-P	No	No	
Anti-A1	Anti-A-1	No	No	
Anti-H	Anti-H	No	No	
Anti-Le(ab)	Anti-Le(x)	No	No	
Anti-Di(a)	Anti-Di(a)	No	No	
Anti-Di(b)	Anti-Di(b)	No	No	
Anti-Do(a)	Anti-Do(a)	No	No	
Anti-Do(b)	None	No	No	
Anti-Co(a)	None	No	No	
Anti-Co(b)	None	No	No	
Anti-LW(a)	None	No	No	
Anti-LW(b)	None	No	No	
Anti-Sc1	None	No	No	
Anti-Sc2	None	No	No	
Anti-Sd(a)	None	No	No	
Anti-Au(a)	Anti-Au(a)	No	No	
Anti-Au(b)	None	No	No	
Anti-Ch	None	No	No	
Anti-Rg	None	No	No	
Anti-Cs(a)	None	No	No	
Anti-Yk(a)	None	No	No	
Anti-Kn(a)	None	No	No	
Anti-McC(a)	None	No	No	
Anti-JMH	None	No	No	
Anti-Yt(a)	None	No	No	

Antibody Name		Clinically Significant?	VBECS Antigen Negative Setting Default	Local comments and decisions on antigen-negative settings.
VBECS	VistA			
Anti-Yt(b)	None	No	No	

### User Roles with Access to This Option

➤➤➤➤➤ Enhanced Supervisor

### Preparation for Testing

- 1) Change the antigen negative requirement setting on those currently not invoked. Change the percent compatible for your donor population.
- 2) Print, sign, date, and save an Audit Trail Report for your validation record.

### Update User Roles

The user reviews and changes the user roles and associated security levels that determine which options and functions are available to a user in the division. For additional information, see Appendix F: Work Process Flowcharts.

### User Roles with Access to This Option

➤➤➤➤➤ Administrator/Supervisor (change user roles)  
 ➤➤➤ Lead Technologist (view user roles)

### Preparation for Testing

- 1) It is recommended to assign one Lead Technologist on each shift to respond to system overrides.
- 2) Print, sign, date, and save an Audit Trail Report for your validation record.

### Testing

Sample test scripts will be provided during the implementation of VBECS. Each site must evaluate the intended uses of the software and make appropriate changes (additions or deletions) to the facility's procedures, processes, policies, and test scripts.

*You are required to validate your process and how you use the software. You must also validate the software itself. Training users is an integral part of the validation process.*

## **VBECS Forms and Reports**


Print and review these reports and forms at the end of validation testing.

- |  |   |
|--|---|
| <input type="checkbox"/> Administrative Data               | <input type="checkbox"/> Patient History                                  |
| <input type="checkbox"/> Audit Trail                       | <input type="checkbox"/> Print Unit Caution Tag & Transfusion Record Form |
| <input type="checkbox"/> Blood Availability                | <input type="checkbox"/> Prolonged Transfusion Time                       |
| <input type="checkbox"/> Cost Accounting                   | <input type="checkbox"/> Testing Worklist                                 |
| <input type="checkbox"/> C:T Ratio                         | <input type="checkbox"/> Transfusion Complications                        |
| <input type="checkbox"/> Division Transfusion              | <input type="checkbox"/> Transfusion Effectiveness                        |
| <input type="checkbox"/> Division Workload                 | <input type="checkbox"/> Transfusion Reaction Count                       |
| <input type="checkbox"/> Exception                         | <input type="checkbox"/> Transfusion Requirements                         |
| <input type="checkbox"/> Finalize/Print TRW                | <input type="checkbox"/> Unit History                                     |
| <input type="checkbox"/> Inappropriate Transfusion Request | <input type="checkbox"/> View/Print Equipment Inventory                   |
| <input type="checkbox"/> Issued/Returned                   | <input type="checkbox"/> View/Print Inventory (Reagent)                   |
| <input type="checkbox"/> Maintain Equipment (Print)        | <input type="checkbox"/> View/Print Supply Inventory                      |
| <input type="checkbox"/> Order History                     |   |


This page intentionally left blank.

# Appendix G: Work Process Flowcharts


Figure 151: Configure Division Settings: Full-Service Divisions

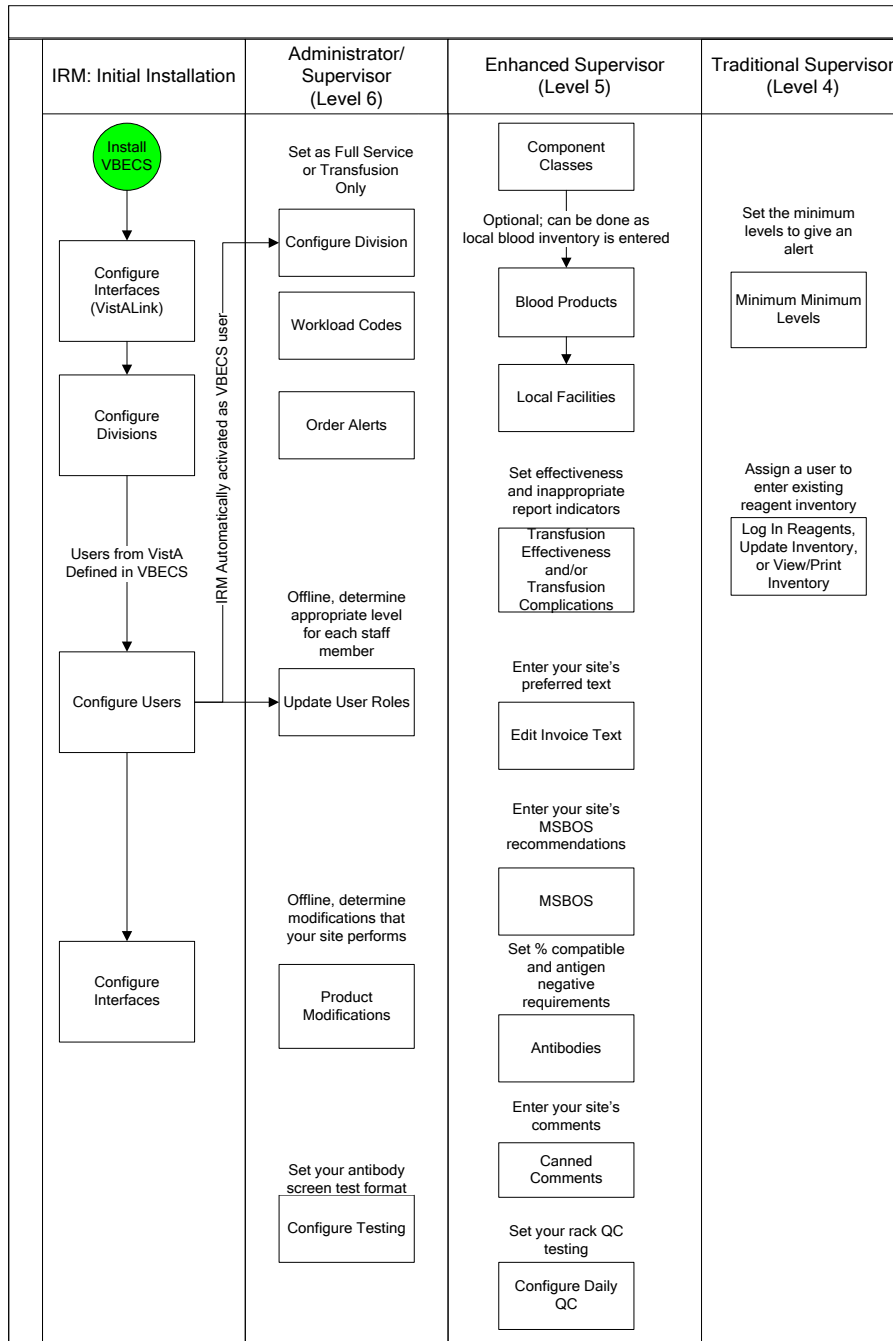


**Figure 152: Configure Division Settings: Transfusion-Only Divisions**

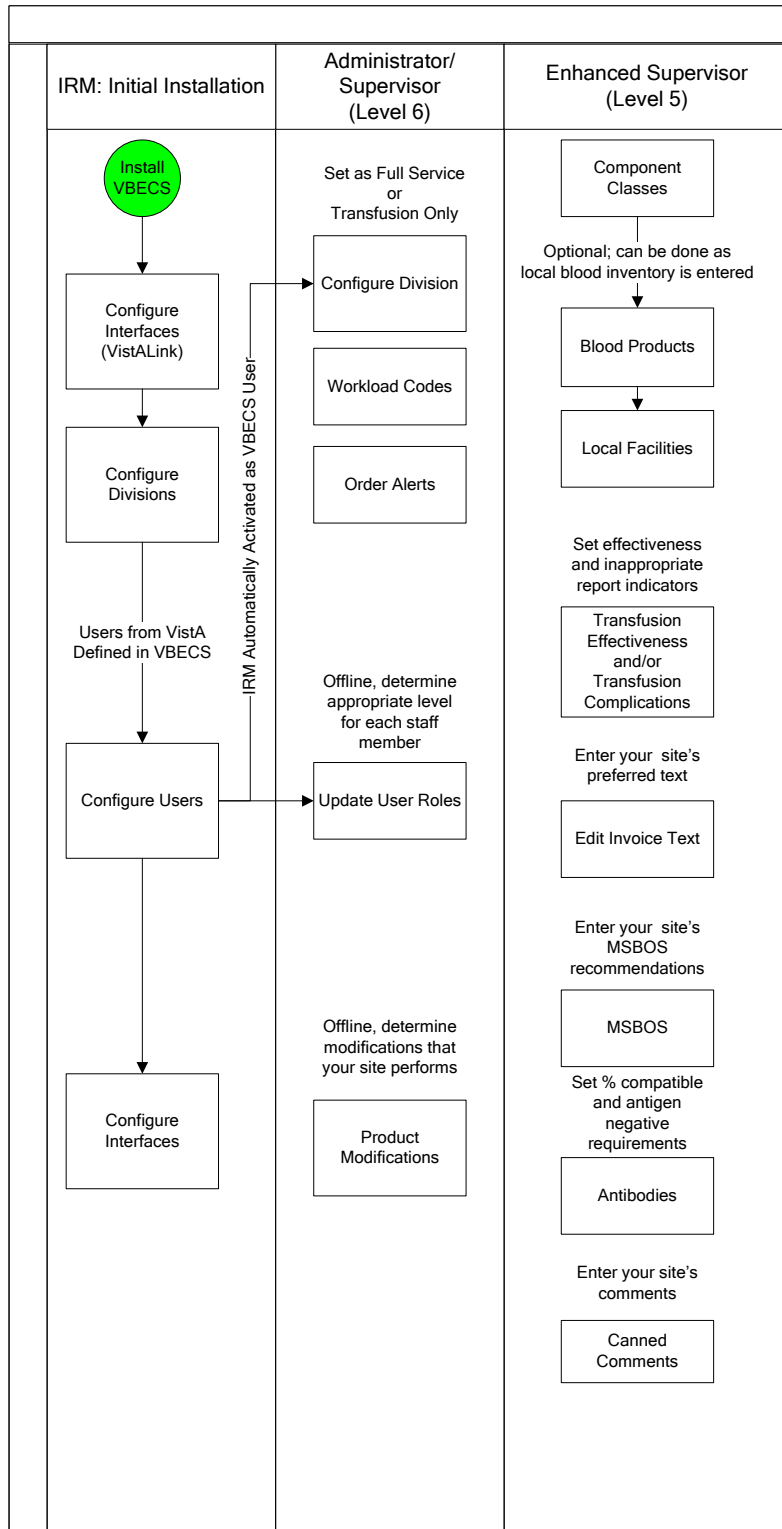

**Figure 153: Maintain Reagents and Daily QC: Full Service Divisions**

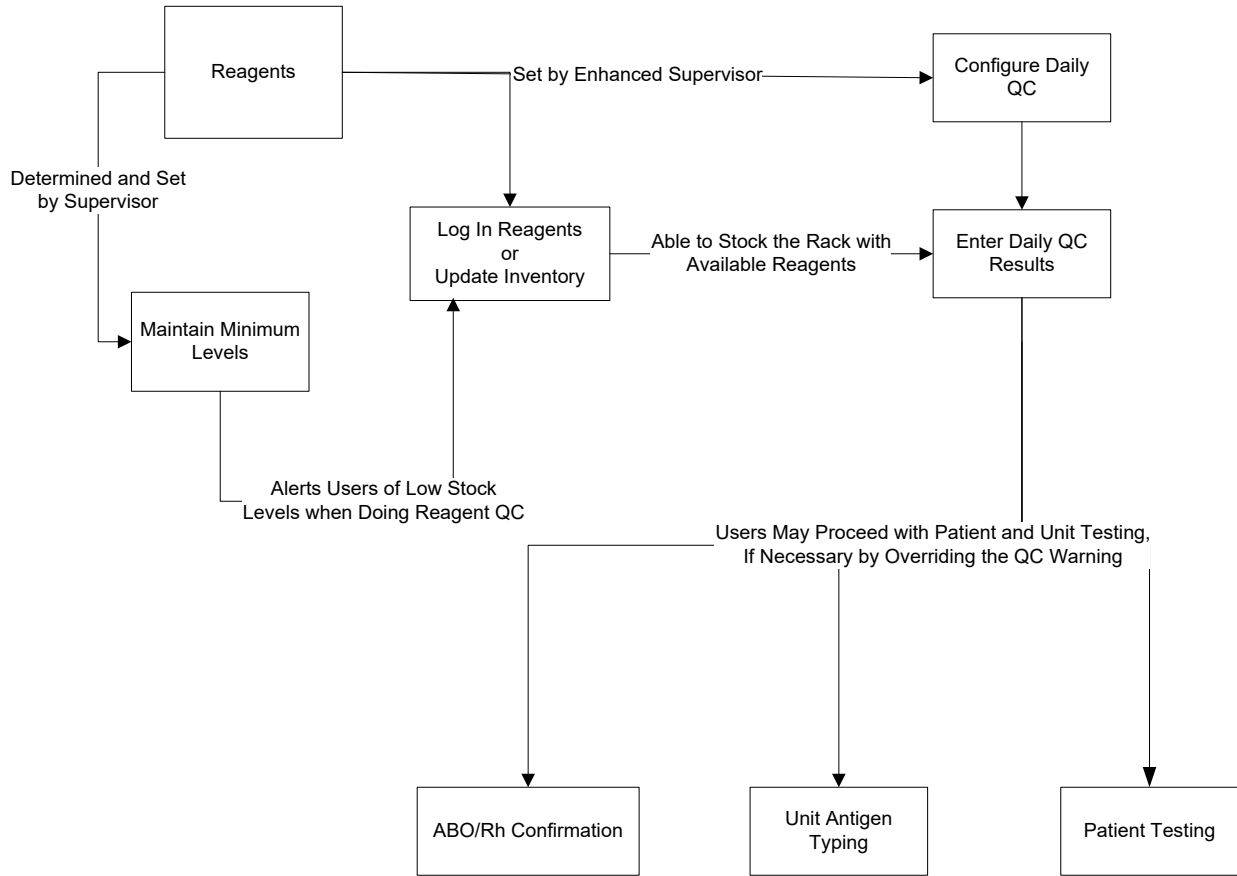


**Figure 154: Process Incoming Shipment: Full-Service Divisions**

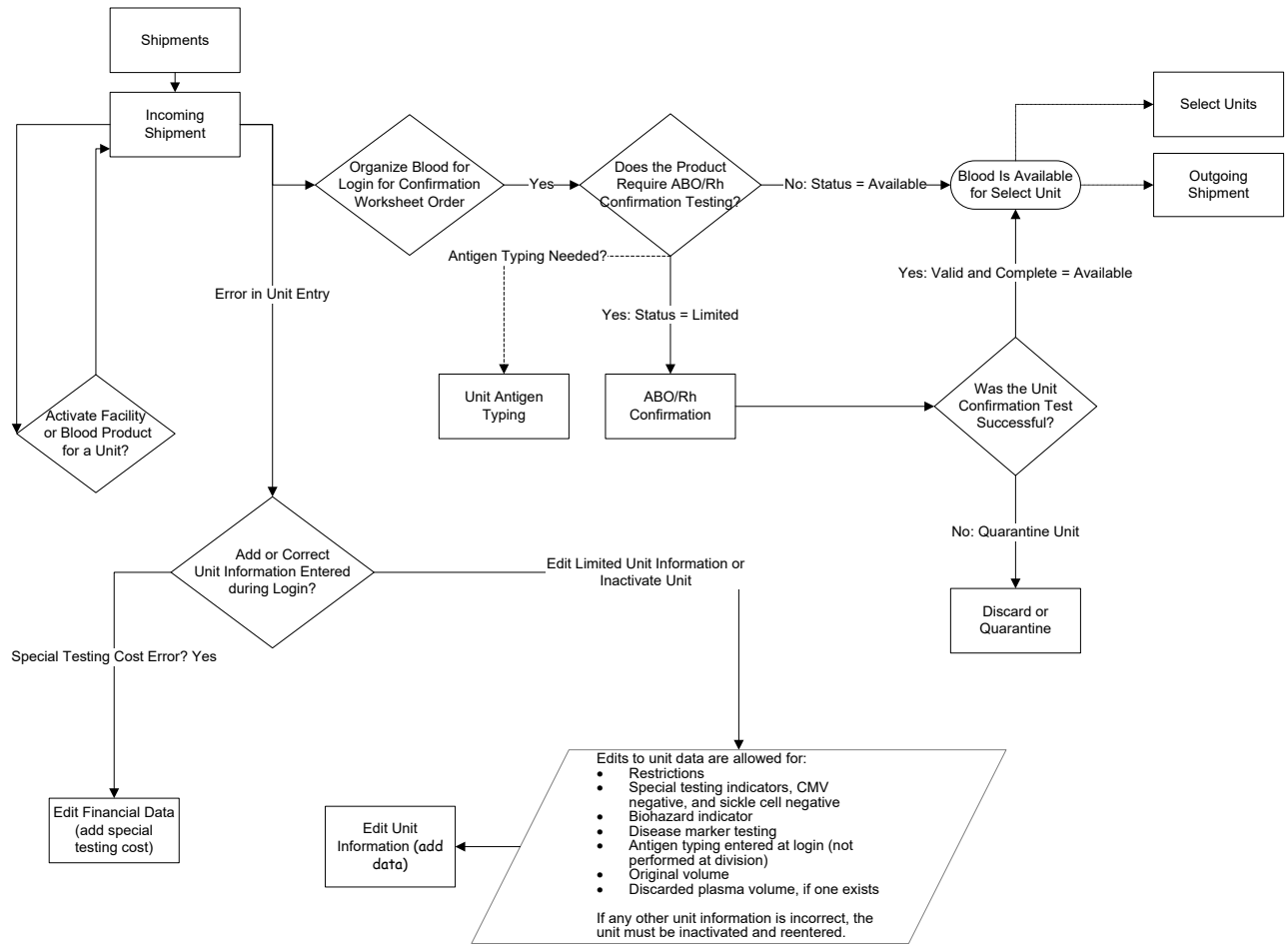





**Figure 155: Process Incoming Shipment: Transfusion-Only Divisions**

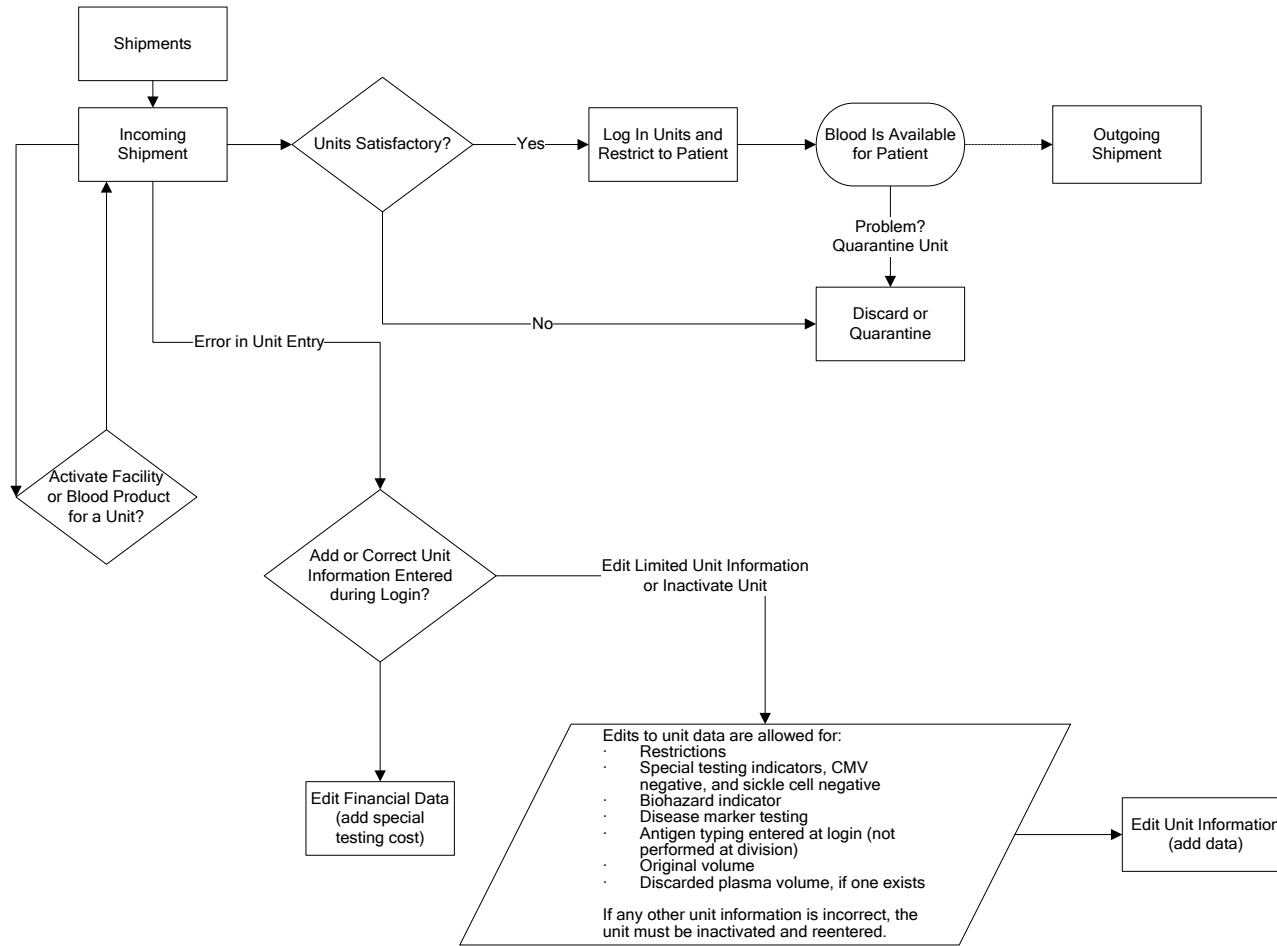


**Figure 156: Modify Units**

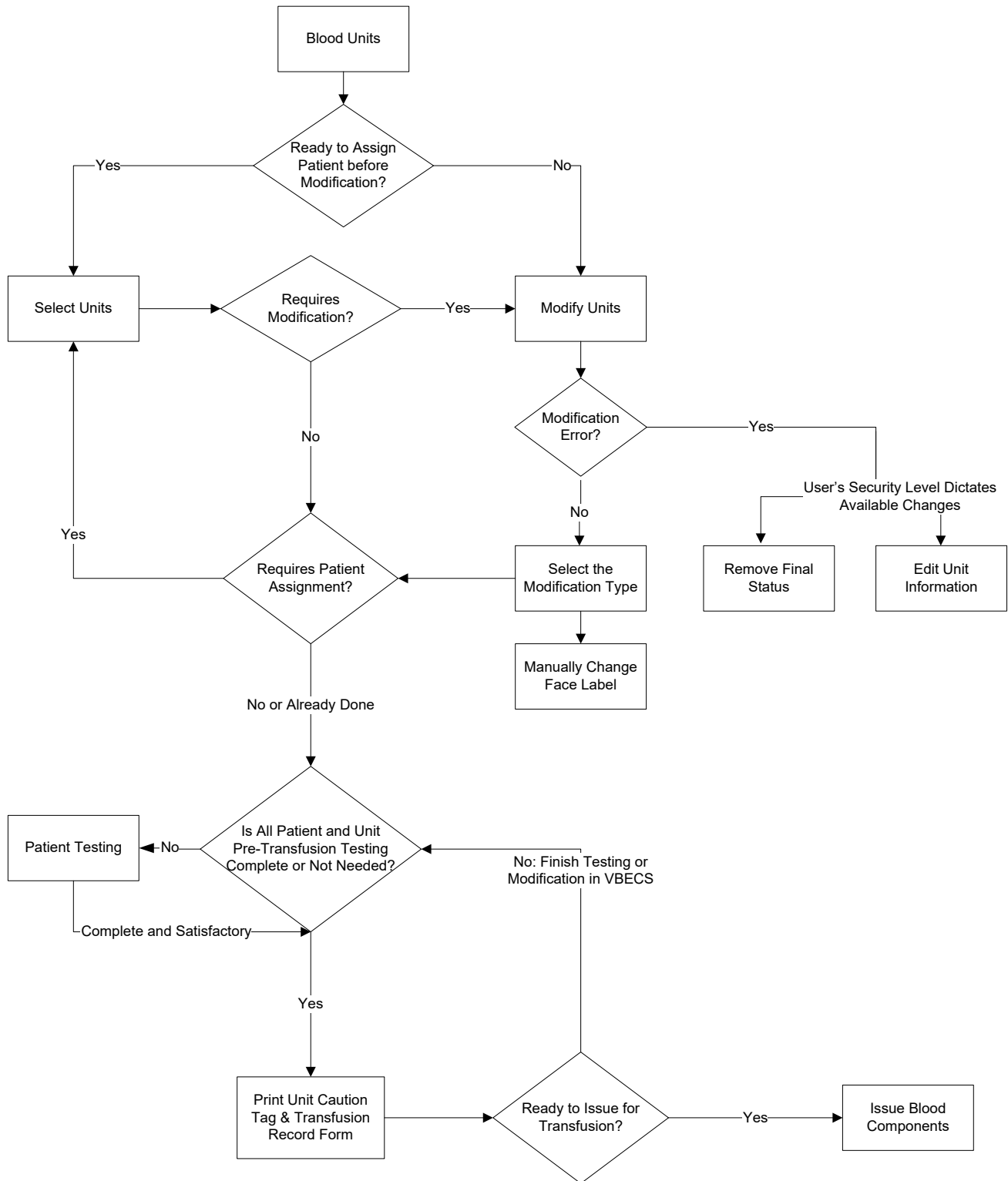


**Figure 157: Issue, Return or Transfuse Units: Full Service Divisions**

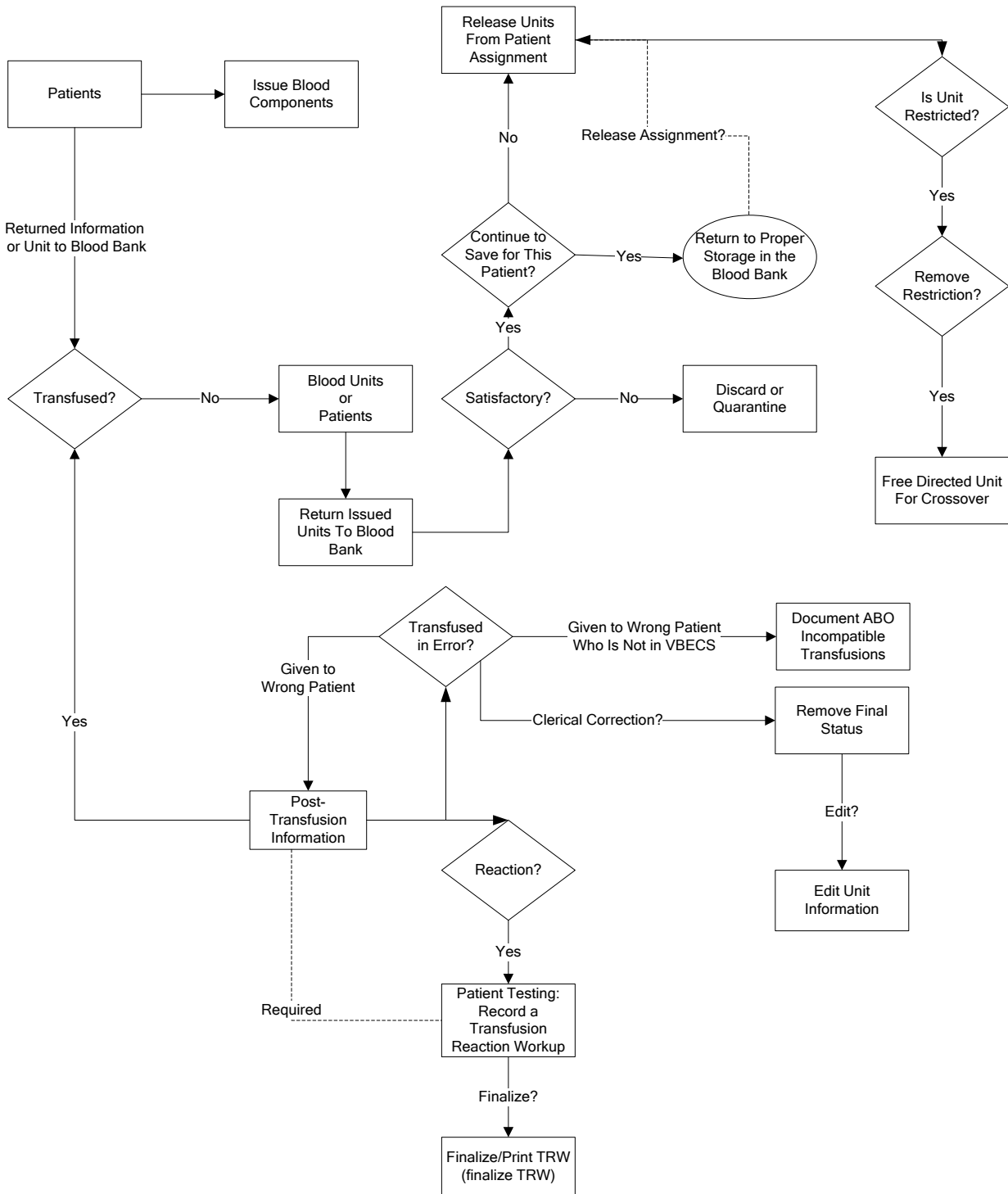


**Figure 158: Issue, or Transfuse: Transfusion-Only Divisions**

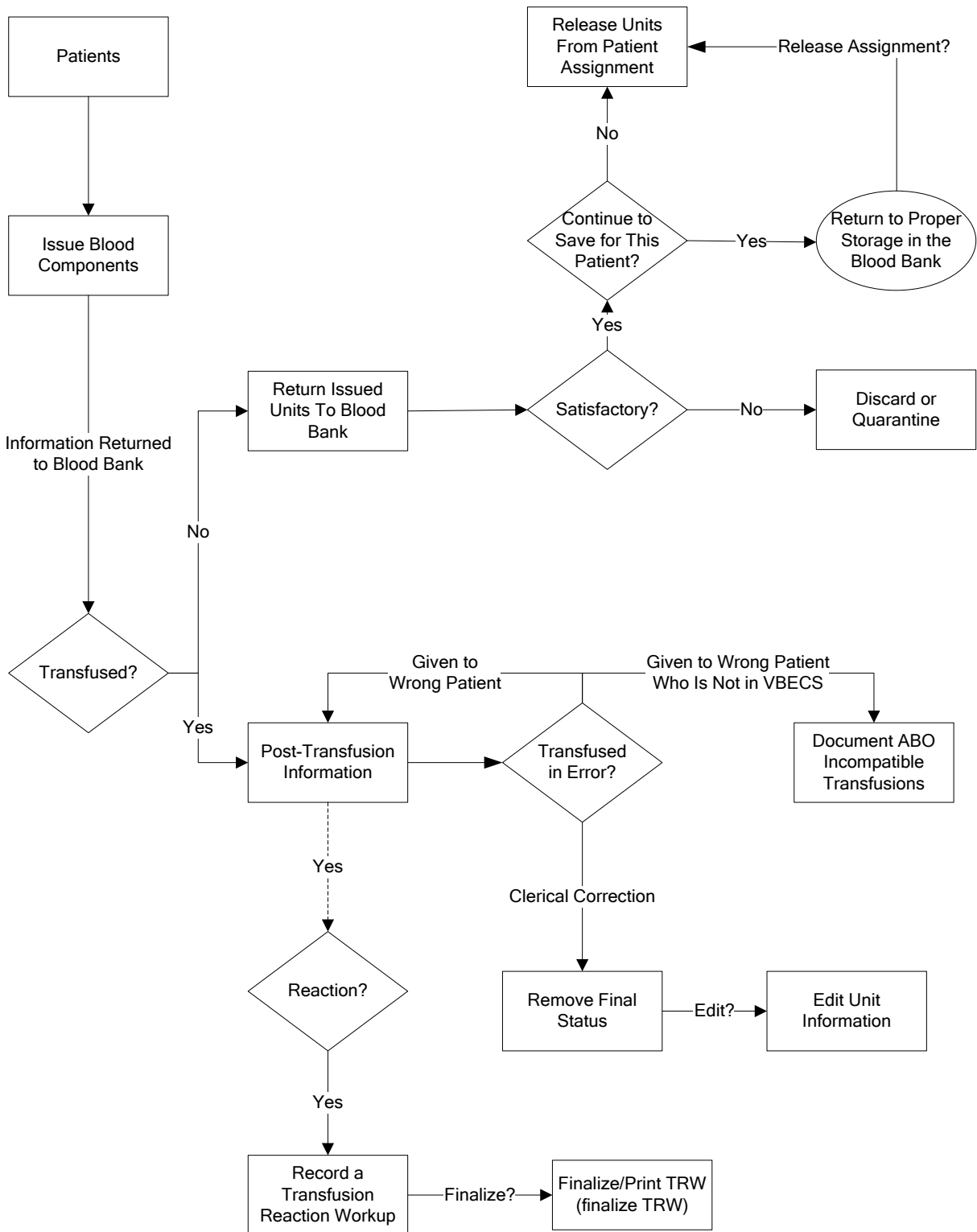

**Figure 159: Process a Patient Order**

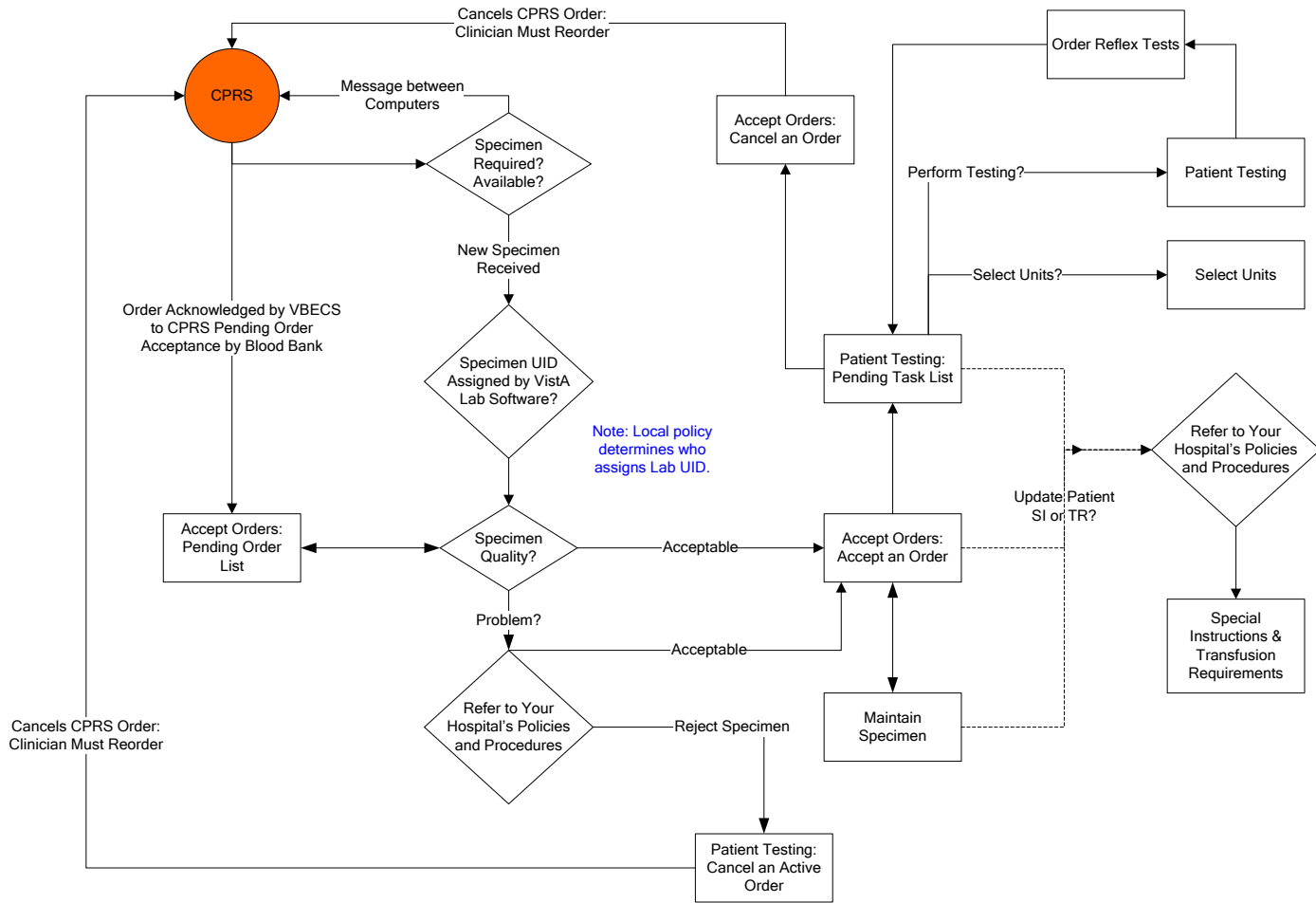

**Figure 160: Patient Testing**

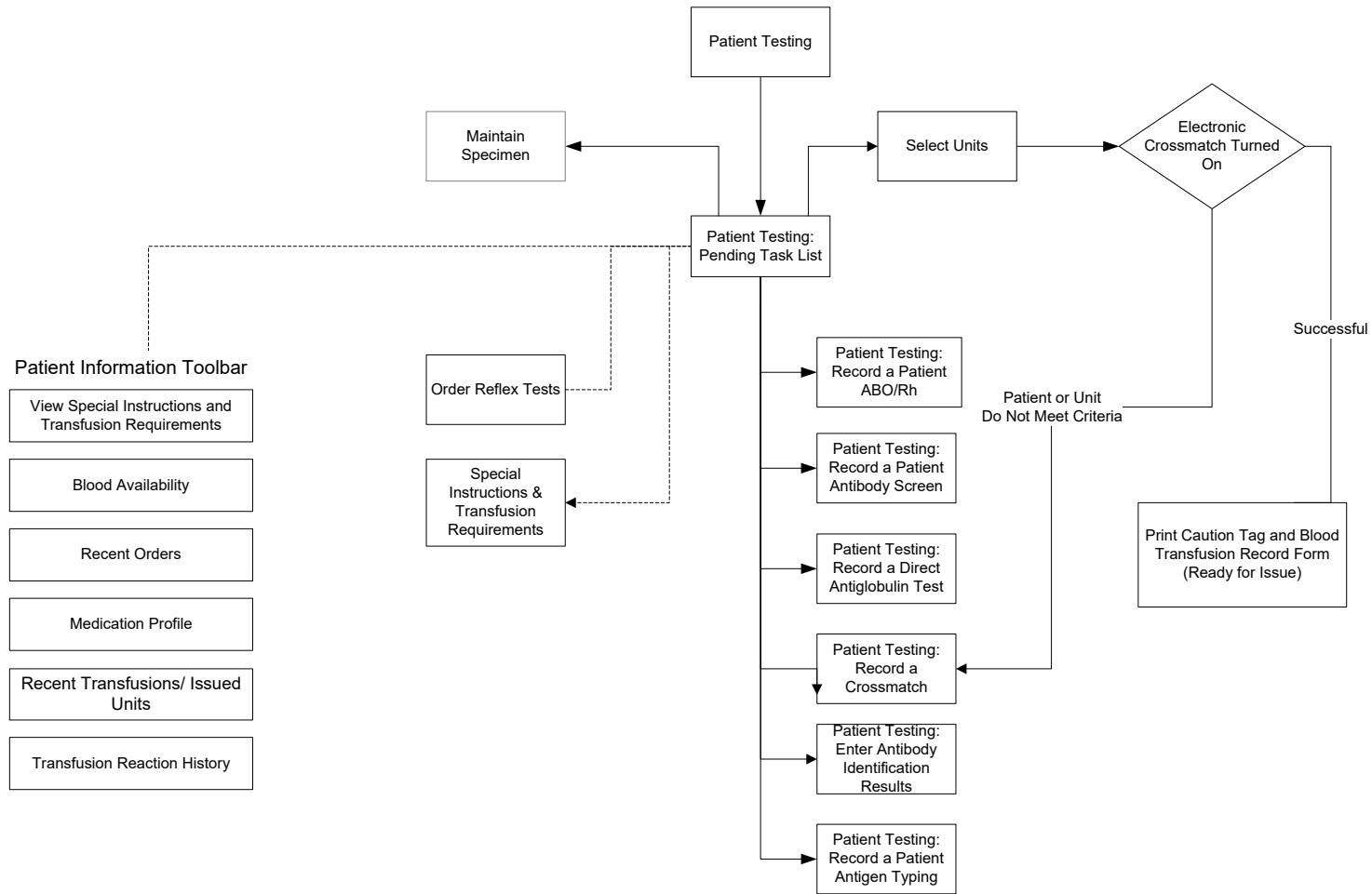

## ***Appendix H: Frequently Asked Questions***

Frequently Asked Questions (FAQs) are released as standalone documents. Topics are explained in a question and answer format. FAQs are available at the following SharePoint site and are version specific:

<https://dvagov.sharepoint.com/sites/oitepmovbecs/default.aspx>

This page intentionally left blank.



## **Appendix I: Common Error Corrections**

### **Correct Patient Test Results**

#### **Define the problem:**

Specifically what data needs correction?

Is it a patient test?

Are any units assigned to the specimen UID associated with this patient order?

The path to transfusion is linear and follows the path from assignment to issue to transfusion. VBECS requires the unit-patient connection to be severed in order to change patient testing used to evaluate compatibility.

**All the units must be released from assignment, no matter when the error occurred and needs correction.** VBECS requires undoing all of the unit associations so the computer can re-evaluate the corrected information entered and present any warnings or overrides based on that new information.

Information to gather before you start trying to correct any test results:

- Actual dates/times of processing for retrospective data entry
  - Testing Worklist Report: Test result entry information details from the testing grids, reaction results, tech who did the work, any comments they entered
  - Unit History Report: Unit modification, Issue and Transfusion details
  - Patient History Report: Other patient information
  - Order History report: See the search option only for the CPRS Order Number if you have to retrieve an expired order. You do not need to actually open the report.
  - Any offline results that may need to be entered.
  - Tracking system numbers from the Known Anomalies and Defects document related to this problem. Have the user enter this number in the comment details field for all overrides processed.
  - Any local incident number that they may need to enter as a comment.

### **Invalidate Test Results**

#### **Important nuances about invalidating tests:**

- When a Type and Screen (TAS) test needs to be invalidated, all the tests in the TAS test configuration must be invalidated to be able to select a test on the Pending Task List (PTL) (It is a loop of a partially completed test).
- When an Antibody Identification (ABID) test is invalidated:
  - The antibody specificities will be removed from the Special Instructions/Transfusion requirements Antibodies Identified at the same time. This also clears any antigen negative requirements that are created by the antibody identified.
  - If separate antigen negative requirements for each of these were entered in SI/TR, they will require update separately.

- **If you need to enter corrected results for the ABID, be sure you check the box to add the order for the PTL. If not, you will have to reorder the reflex test.**

A finalized Transfusion Reaction Workup (TRW) cannot be invalidated. Invalidation requires a Service Desk ticket to have the development team intervene and make that correction. A TRW that has not been finalized can be invalidated and re-entered as any other VBECS test.

**Where to start...**

**1. Are the units transfused?**

Go to Supervisor, Remove Final Status to get the unit back to an issued status.

**2. Are the units issued?**

Go to Return Unit from Issue by either the patient or blood unit menu.  
Return the unit from issue at the current date and time.

**3. Are the units selected for a patient?**

Go to Release Units from Assignment by either the patient or blood unit menu.

When the units are no longer assigned to the patient, go into Invalidate Patient Testing by the Patient Menu and invalidate the patient test.

**STOP and make sure the user checks the box in the lower left corner if they want the test to appear on the PTL for correction.**

## **UN-Pooling a Pooled Unit (in the computer only)**

This may happen when a pool is created in the computer and the physical pool was not created after all. If there is a patient assignment, issue or transfusion, that must be removed first as described above in the **Where to start** section.

1. Go to Blood Unit, Edit Unit Information and INACTIVATE the Pooled Unit.
2. Go to Supervisor, Remove Final Status, select each original unit removing the MODIFIED status and making each available.
3. Repeat Step 2 for each original unit included in the pool.

## **UN-Modifying a Unit**

Go to Supervisor, Remove Final Status.

If there is a patient assignment, issue or transfusion, that must be removed first as described above in the **Where to start** section.

## Appendix J: Recommended Report Usage

Table 37 lists reports, how often to run them, and how to use them. These are recommendations for meeting regulatory requirements. Reports, including the non-individual Blood Availability Report, are available through the Reports menu item. The patient-specific Blood Availability Report is available through the Patients menu item.

**Table 37: Recommended Report Usage**

Run This Report:	Report Option	Purpose	To Meet Regulatory Requirements:
Daily	Audit Trail	Quality assurance (QA) report details data changes in system.	Print, review, sign, date, and save hard copy.
	Blood Availability	Inventory report	This report may not be needed by transfusion-only or other sites.
	Exception Report	QA report details overrides of system rules processed by users.	Print, review, sign, date, and save hard copy.
	Testing Worklist Report	QA report displays all testing within a division.	Print Patient Testing, Unit testing, Rack QC and miscellaneous reagent QC. Print, review, sign, date, and save hard copy.
Daily or Weekly	Issued/Returned Report	QA report displays all relocation transactions.	Review for anomalies and take corrective action.
Daily, Weekly, and Monthly	Transfusion Requirements Report	A compendium of patient blood types, SIs, and TRs are printed for use during downtime.	<ul style="list-style-type: none"> <li>Days 1–6: print report daily.</li> <li>Day 7: print report for previous week; discard daily reports.</li> <li>Start of month: print report for previous month; discard weekly reports.</li> <li>Start of year: print report for previous year; discard monthly reports.</li> </ul> <p>Generate cumulative reports starting with the date of first data entry through the current date to include all patient records. Store reports so that they are easily accessible.</p>
Weekly	Prolonged Transfusion Time Report	Details units that were not infused within the required time frame.	Generate these reports for blood utilization committee and other quality management activities.
	Transfusion Effectiveness Report	Compares transfusions with user-configured lab data indicators to monitor appropriate blood usage.	
Weekly or Monthly	C:T Ratio Report	Details crossmatch-to-transfusion ratio by physician or treating specialty.	
	Transfusion Complications Report	Compares transfusions with user configured lab data indicators to monitor adverse effects from transfusion.	

<b>Run This Report:</b>	<b>Report Option</b>	<b>Purpose</b>	<b>To Meet Regulatory Requirements:</b>
	Inappropriate Transfusion Request Report	Details requests for blood products that did not meet the facility's transfusion criteria.	
Monthly	Administrative Data Report	Management report details products received, transfused, and returned.	Generate these reports for administrative purposes.
	Cost Accounting Report	Management report of expenditure by invoice number.	
	Division Transfusion Report	Details of units transfused during a selected period.	
	Division Workload Report	Totals workload types for one or all divisions.	
	Transfusion Reaction Count Report	Details transfusion reactions that are ordered, pending, completed, or finalized for one or all divisions.	Generate this report for blood utilization committee and other quality management activities.
As needed	Finalize/Print TRW	Patient Transfusion Reaction Report details a transfusion reaction episode.	Print, review, sign, date, and save hard copy.

## **Appendix K: Barcode Scanner Configuration and Troubleshooting**

Make sure the cord is plugged into the scanner and the computer. Point the scanner at a piece of white paper and squeeze the trigger to verify that the laser works.

*For correct barcode scanning results, the Caps Lock key must not be engaged.*



*Never look directly at the laser. Never direct the laser beam toward yourself or another person. Avoid mirror-like surfaces such as glass, metal, watches, jewelry, and other highly reflective materials when directing the laser beam.*

To configure a barcode scanner, scan the barcode in Figure 161. Repeat this process if scanned data are incorrectly displayed on the terminal. Consult the product reference guide or call the Service Desk for further assistance.

If your site uses barcode symbology for specimens not covered in these instructions, use the configuration barcodes in the manual included with the scanner.

**Figure 161: Barcode Scanner Configuration**



This page intentionally left blank.

## Appendix L: System Responses to Active Transfusion Requirements in Select Unit and Issue Blood Component

The following series of tables, Figures 158 through 169, detail VBECS response to an active transfusion requirement, component class of the unit selected and the user's VBECS role at unit selection and blood component issue.

- Figure 162: Irradiate Cellular Products Only and Select Blood Unit
- Figure 163: Irradiate Cellular Products Only and Issue Blood Component
- Figure 164: Leuko-reduce Cellular Products and Select Blood Unit
- Figure 165: Leuko-reduce Cellular Products and Issue Blood Component
- Figure 166: Washed Red Blood Cell (RBC) Products and Select Blood Unit
- Figure 167: Washed Red Blood Cell (RBC) Products and Issue Blood Component
- Figure 168: Washed PLATELET (PLT) Products and Select Blood Unit
- Figure 169: Washed PLATELET (PLT) Products and Issue Blood Component
- Figure 170: Sickle Cell Negative RBC Products and Select Blood Unit
- Figure 171: Sickle Cell Negative RBC Products and Issue Blood Component
- Figure 172: CMV Negative Cellular Products\* and Select Blood Unit
- Figure 173: CMV Negative Cellular Products\* and Issue Blood Component

**Figure 162: Irradiate Cellular Products Only and Select Blood Unit**

Select Unit: IRRADIATED				
Patient Active Transfusion Requirement	Ordered Component Class*	Selected Unit is	User Role	VBECS Response
<b>Irradiate Cellular Products Only</b>	RED BLOOD CELLS WHOLE BLOOD PLATELETS OTHER	Irradiated	<b>ALL VBECS Users</b>	No warning message will display.
	RED BLOOD CELLS WHOLE BLOOD PLATELETS OTHER	<b>Not Irradiated</b>	<b>ALL VBECS Users</b>	The Transfusion Requirement is not satisfied; the system emits an audible alert and displays a warning message: <i>This unit does not currently satisfy all of the patient's Transfusion Requirements. Further processing is needed. Continue to select this unit?</i> If the user clicks "No," the screen is cleared. If the user clicks "Yes," selection can continue to select the unit.

**Figure 163: Irradiate Cellular Products Only and Issue Blood Component**

Issue Blood Component: IRRADIATED				
Patient Active Transfusion Requirement	Ordered Component Class*	Selected Unit is	User Role	VBECS Response
<b>Irradiate Cellular Products Only</b>	RED BLOOD CELLS WHOLE BLOOD PLATELETS OTHER	Irradiated	ALL VBECS Users	No warning message will display.
	RED BLOOD CELLS WHOLE BLOOD PLATELETS OTHER	<b>Not Irradiated</b>	Technologist Enhanced Technologist	The system displays the warning message: <i>Patient's Transfusion Requirement is not satisfied. You do not have the proper security to issue this unit. Further processing is needed and must be documented in VBECS.</i> The tech may not issue this unit in the computer.
	RED BLOOD CELLS WHOLE BLOOD PLATELETS OTHER	<b>Not Irradiated</b>	Lead Technologist Traditional Supervisor Enhanced Supervisor Administrator/Supervisor	The system displays the warning message: <i>Patient's Irradiate Cellular Products Only Transfusion Requirement is not satisfied</i> and prompts the user whether to proceed. If the user responds "No", the system will clear the selected unit. If the user responds "Yes", the system will require a comment (free text or canned, "unit issue" context) and capture the transaction details for inclusion in an Exception Report entry: <i>Unit issued with unsatisfied Transfusion Requirement.</i>



**Figure 164: Leuko-reduce Cellular Products and Select Blood Unit**

Select Unit: Leuko-reduce Cellular Products				
Patient Active Transfusion Requirement	Ordered Component Class*	Selected Unit is	User Role	VBECS Response
<b>Leuko-reduce Cellular Products</b>	RED BLOOD CELLS WHOLE BLOOD PLATELETS	Leuko-reduced	ALL VBECS Users	No warning message will display.
	RED BLOOD CELLS WHOLE BLOOD PLATELETS	<b>Not Leuko-reduced</b>	ALL VBECS Users	The Transfusion Requirement is not satisfied; the system emits an audible alert and displays a warning message: <i>This unit does not currently satisfy all of the patient's Transfusion Requirements. Further processing is needed. Continue to select this unit?</i> If the user clicks "No," the screen is cleared. If the user clicks "Yes," selection can continue to select the unit.

**Figure 165: Leuko-reduce Cellular Products and Issue Blood Component**

Issue Blood Component: Leuko-reduce Cellular Products				
Patient Active Transfusion Requirement	Ordered Component Class*	Selected Unit is	Selected User Role	Expected System Response
<b>Leuko-reduce Cellular Products</b>	RED BLOOD CELLS WHOLE BLOOD PLATELETS OTHER	Leuko-reduced	ALL VBECS Users	No warning message will display.
	RED BLOOD CELLS WHOLE BLOOD PLATELETS	<b>Not Leuko-reduced</b>	Technologist Enhanced Technologist	The system displays the warning message: <i>Patient's Transfusion Requirement is not satisfied. You do not have the proper security to issue this unit. Further processing is needed and must be documented in VBECS.</i> The tech may not issue this unit in the computer.
	RED BLOOD CELLS WHOLE BLOOD PLATELETS	<b>Not Leuko-reduced</b>	Lead Technologist Traditional Supervisor Enhanced Supervisor Administrator/Supervisor	The system displays the warning message: <i>Patient's Leuko-reduce Cellular Products Transfusion Requirement is not satisfied and prompts the user whether to proceed.</i> If the user responds "No", the system will clear the selected unit. If the user responds "Yes", the system will require a comment (free text or canned, "unit issue" context) and capture the transaction details for inclusion in an Exception Report entry: <i>Unit issued with unsatisfied Transfusion Requirement.</i>

**Figure 166: Washed Red Blood Cell (RBC) Products and Select Blood Unit**

Select Unit: Washed Red Blood Cell (RBC) products				
Patient Active Transfusion Requirement	Ordered Component Class*	Selected Unit is	User Role	VBECS Response
<b>Washed RBC products</b>	RED BLOOD CELLS WHOLE BLOOD	Washed	ALL VBECS Users	No warning message will display.
	RED BLOOD CELLS WHOLE BLOOD	<b>Not Washed</b>	ALL VBECS Users	The Transfusion Requirement is not satisfied; the system emits an audible alert and displays a warning message: <i>This unit does not currently satisfy all of the patient's Transfusion Requirements. Further processing is needed. Continue to select this unit?</i> If the user clicks "No," the screen is cleared. If the user clicks "Yes," selection can continue to select the unit.

**Figure 167: Washed Red Blood Cell (RBC) Products and Issue Blood Component**

Issue Blood Component: Washed RBC products				
Patient Active Transfusion Requirement	Ordered Component Class*	Selected Unit is	Selected User Role	Expected System Response
<b>Washed RBC products</b>	RED BLOOD CELLS WHOLE BLOOD	Washed	ALL VBECS Users	No warning message will display.
	RED BLOOD CELLS WHOLE BLOOD	<b>Not Washed</b>	Technologist Enhanced Technologist	The system displays the warning message: <i>Patient's Transfusion Requirement is not satisfied. You do not have the proper security to issue this unit. Further processing is needed and must be documented in VBECS.</i> The tech may not issue this unit in the computer.
	RED BLOOD CELLS WHOLE BLOOD	<b>Not Washed</b>	Lead Technologist Traditional Supervisor Enhanced Supervisor Administrator/Supervisor	The system displays the warning message: <i>Patient's Washed RBC Products Transfusion Requirement is not satisfied</i> and prompts the user whether to proceed. If the user responds "No", the system will clear the selected unit. If the user responds "Yes", the system will require a comment (free text or canned, "unit issue" context) and capture the transaction details for inclusion in an Exception Report entry: <i>Unit issued with unsatisfied Transfusion Requirement.</i>

**Figure 168: Washed PLATELET (PLT) Products and Select Blood Unit**

Select Unit: Washed PLT products				
Patient Active Transfusion Requirement	Ordered Component Class*	Selected Unit is	User Role	VBECS Response
<b>Washed PLT products</b>	RED BLOOD CELLS WHOLE BLOOD PLATELETS OTHER	Washed	ALL VBECS Users	No warning message will display.
	RED BLOOD CELLS WHOLE BLOOD PLATELETS OTHER	<b>Not Washed</b>	ALL VBECS Users	The Transfusion Requirement is not satisfied; the system emits an audible alert and displays a warning message: <i>This unit does not currently satisfy all of the patient's Transfusion Requirements. Further processing is needed. Continue to select this unit?</i> If the user clicks "No," the screen is cleared. If the user clicks "Yes," selection can continue to select the unit.

**Figure 169: Washed PLATELET (PLT) Products and Issue Blood Component**

Issue Blood Component: Washed PLT products				
Patient Active Transfusion Requirement	Ordered Component Class*	Selected Platelet Unit is	Selected User Role	Expected System Response
<b>Washed PLT products</b>	PLATELETS	Washed	ALL VBECS Users	No warning message will display.
	PLATELETS	<b>Not Washed</b>	Technologist Enhanced Technologist	The system displays the warning message: <i>Patient's Transfusion Requirement is not satisfied. You do not have the proper security to issue this unit. Further processing is needed and must be documented in VBECS.</i> The tech may not issue this unit in the computer.
	PLATELETS	<b>Not Washed</b>	Lead Technologist Traditional Supervisor Enhanced Supervisor Administrator/Supervisor	The system displays the warning message: <i>Patient's Washed PLT products Transfusion Requirement is not satisfied</i> and prompts the user whether to proceed. If the user responds "No", the system will clear the selected unit. If the user responds "Yes", the system will require a comment (free text or canned, "unit issue" context) and capture the transaction details for inclusion in an Exception Report entry: <i>Unit issued with unsatisfied Transfusion Requirement.</i>

**Figure 170: Sickle Cell Negative RBC Products and Select Blood Unit**

<b>Select Unit: Sickle Cell Negative RBC products</b>				
<b>Patient Active Transfusion Requirement</b>	<b>Ordered Component Class*</b>	<b>Selected Unit is</b>	<b>User Role</b>	<b>VBECs Response</b>
<b>Sickle Cell Negative RBC products</b>	RED BLOOD CELLS WHOLE BLOOD	Sickle Cell Negative	ALL VBECs Users	No warning message will display.
	RED BLOOD CELLS WHOLE BLOOD	<b>Not Sickle Cell Negative</b>	Technologist Enhanced Technologist	The Transfusion Requirement is not satisfied; the system emits an audible alert and displays a warning message: <i>This unit does not currently satisfy all of the patient's Transfusion Requirements. Further processing is needed. Continue to select this unit?</i> If the user clicks "No," the screen is cleared. If the user clicks "Yes," selection can continue to select the unit.
	RED BLOOD CELLS WHOLE BLOOD	<b>Not Sickle Cell Negative</b>	Lead Technologist Traditional Supervisor Enhanced Supervisor Administrator/Supervisor	The following warning(s) is informational: <i>This unit does not currently satisfy the patient's Sickle Cell Negative Transfusion Requirements. Further processing is needed. Continue to select this unit?</i> <i>Do you wish to proceed with the selection of unit &lt;insert unit ID number&gt;:?</i> If "No," is selected, the screen is cleared and a new blood unit must be selected. If "Yes," is selected, a comment is required and the details of the transaction are captured for later inclusion in an exception report entry: <i>Transfusion Requirement incompatible unit selected.</i>

**Figure 171: Sickle Cell Negative RBC Products and Issue Blood Component**

<b>Issue Blood Component: Sickle Cell Negative RBC Products</b>				
<b>Patient Active Transfusion Requirement</b>	<b>Ordered Component Class*</b>	<b>Selected Unit is</b>	<b>Selected User Role</b>	<b>Expected System Response</b>
<b>Sickle Cell Negative RBC products</b>	RED BLOOD CELLS WHOLE BLOOD	Sickle Cell negative	ALL VBECS Users	No warning message will display.
	RED BLOOD CELLS WHOLE BLOOD	<b>NOT Sickle Cell Negative</b>	Technologist Enhanced Technologist	The system displays the warning message: <i>Patient's Transfusion Requirement is not satisfied. You do not have the proper security to issue this unit. Further processing is needed and must be documented in VBECS.</i> The tech may not issue this unit in the computer.
	RED BLOOD CELLS WHOLE BLOOD	<b>NOT Sickle Cell Negative</b>	Lead Technologist Traditional Supervisor Enhanced Supervisor Administrator/Supervisor	The system displays the warning message: <i>Patient's Sickle Cell Negative RBC products Transfusion Requirement is not satisfied</i> and prompts the user whether to proceed. If the user responds "No", the system will clear the selected unit. If the user responds "Yes", the system will require a comment (free text or canned, "unit issue" context) and capture the transaction details for inclusion in an Exception Report entry: <i>Unit issued with unsatisfied Transfusion Requirement.</i>

**Figure 172: CMV Negative Cellular Products\* and Select Blood Unit**

Select Unit: CMV Negative Cellular Products				
Patient Active Transfusion Requirement	Ordered Component Class*	Selected Unit is	User Role	VBECS Response
<b>CMV negative cellular products*</b>	RED BLOOD CELLS WHOLE BLOOD PLATELETS OTHER	CMV Negative*	ALL VBECS Users	No warning message will display.
	RED BLOOD CELLS WHOLE BLOOD PLATELETS OTHER*	<b>Not CMV negative</b>	Technologist Enhanced Technologist	The Transfusion Requirement is not satisfied; the system emits an audible alert and displays a warning message: <i>This unit does not currently satisfy all of the patient's Transfusion Requirements. Further processing is needed. Continue to select this unit?</i> If the user clicks "No," the screen is cleared. If the user clicks "Yes," selection can continue to select the unit.
	RED BLOOD CELLS WHOLE BLOOD PLATELETS OTHER*	<b>Not CMV negative</b>	Lead Technologist Traditional Supervisor Enhanced Supervisor Administrator/Supervisor	The system displays the warning message: <i>Patient's CMV Negative cellular products Transfusion Requirement is not satisfied</i> and prompts the user whether to proceed. If the user responds "No", the system will clear the selected unit. If the user responds "Yes", the system will require a comment (free text or canned, "unit issue" context) and capture the transaction details for inclusion in an Exception Report entry: <i>Unit issued with unsatisfied Transfusion Requirement.</i>

\*VistA test results are not evaluated by VBECS. CMV Negative status is recorded during receipt at Incoming Shipment or added to the unit record in Edit Unit Information. All product types in OTHER are evaluated for the requirement including those that are non-cellular.

**Figure 173: CMV Negative Cellular Products\* and Issue Blood Component**

Issue Blood Component: CMV Negative Cellular Products				
Patient Active Transfusion Requirement	Ordered Component Class*	Selected Unit is	Selected User Role	Expected System Response
<b>CMV negative cellular products*</b>	RED BLOOD CELLS WHOLE BLOOD PLATELETS OTHER	CMV Negative*	ALL VBECS Users	No warning message will display.
	RED BLOOD CELLS WHOLE BLOOD PLATELETS OTHER*	<b>Not CMV negative</b>	Technologist Enhanced Technologist	The system displays the warning message: <i>Patient's Transfusion Requirement is not satisfied. You do not have the proper security to issue this unit. Further processing is needed and must be documented in VBECS.</i> The tech may not issue this unit in the computer.
	RED BLOOD CELLS WHOLE BLOOD PLATELETS OTHER*	<b>Not CMV negative</b>	Lead Technologist Traditional Supervisor Enhanced Supervisor Administrator/Supervisor	The system displays the warning message: <i>Patient's CMV Negative Cellular Products Transfusion Requirement is not satisfied</i> and prompts the user whether to proceed. If the user responds "No", the system will clear the selected unit. If the user responds "Yes", the system will require a comment (free text or canned, "unit issue" context) and capture the transaction details for inclusion in an Exception Report entry: <i>Unit issued with unsatisfied Transfusion Requirement.</i>

\*VistA test results are not evaluated by VBECS. CMV Negative status is recorded during receipt at Incoming Shipment or added to the unit record in Edit Unit Information. All product types in OTHER are evaluated for the requirement including those that are non-cellular.

This page intentionally left blank.



## Appendix M: Workload Process Mapping to Application Option Table

Table 38 associates record saves with workload processes. The data fields identified for transmission at the completion of a Workload Event are based on current VistA workload-related files and fields. VBECS will transmit information to a new flat file. There are no donor workload types in VBECS.

**Table 38: Workload Process Mapping to Application Option**

Record Save Option	VBECS Process	Transaction Type [P (Patient), U (Unit), M (Miscellaneous)]	Explanation
Record a Transfusion Reaction Workup	ABO Forward and reverse typing (patient)	P	An ABO/Rh test for "pre" or "post" is enabled and a valid interpretation other than Not Tested is selected. A workload event is accrued separately for "Pre" and "Post" entries.
Record Patient ABO/Rh		P	Accrue workload when a CPRS-ordered ABO/Rh test is performed.
Invalidate Patient Test Results*		P	Accrue workload when a completed ABO/Rh test is invalidated.
Record Patient ABO/Rh	ABO Forward and reverse typing (patient) Repeat Test	M	Accrue workload when a reflex or repeat ABO/Rh test is performed, completed, and saved.
Invalidate Patient Test Results*		M	Accrue workload when a reflex or repeat ABO/Rh test is invalidated.
ABO/Rh Confirmation	ABO forward typing (unit)	U	An ABO confirmation test is performed. When multiple units are selected in a batch, each unit in the batch accrues a workload event. Note: Workload generated during Anti-D testing is not included in the unit's confirmation test. Workload is not accrued when an ABO or Rh discrepancy override is processed and VBECS releases all patient assignments. Workload is not accrued when VBECS quarantines the unit due to a discrepancy. There is no special handling for workload collection for additional confirmation tests on a unit.
		U	Accrue workload when an ABO confirmation test is invalidated.
ABO/Rh Confirmation	ABO/Rh forward typing (unit)	U	An ABO/Rh confirmation test is performed. When multiple units are selected in a batch, each unit in the batch accrues a workload event. Note: Workload generated during Anti-D is part of the unit's confirmation test. Workload is not accrued when an ABO or Rh discrepancy override is processed and VBECS releases all patient assignments. Workload is not accrued when VBECS quarantines the unit due to a discrepancy. Any unit successfully confirmed accrues workload. For split modifications: workload is not inherited by split units. A split unit that requires confirmation accrues confirmation workload at the time of testing. There is no special handling for workload collection for additional confirmation tests on a unit.
		U	Accrue workload when an ABO/Rh confirmation test is invalidated.

<b>Record Save Option</b>	<b>VBECS Process</b>	<b>Transaction Type [P (Patient), U (Unit), M (Miscellaneous)]</b>	<b>Explanation</b>
Accept Order	Accept Order	M	Accrue workload when an order is accepted. When multiple orders are selected, each order accrues workload.
Enter Antibody Identification Results	Antibody identification Work-Up	P	User enters additional workload associated with the individual reflex-ordered ABID. The selected VBECS multiplier will multiply against the VistA multiplier and display the (multiplication) product on the Division Workload Report.
Invalidate Patient Test Results*		P	Accrue workload when the ABID is invalidated.
Record a Transfusion Reaction Workup	Antibody Screen (patient)	P	An ABS test for "pre" or "post" is enabled and a valid interpretation other than Not Tested is selected. A workload event is accrued separately for "Pre" and "Post" entries.
Record Patient Antibody Screen		P	Accrue workload when an ordered ABS test is performed.
Invalidate Patient Test Results*		P	Accrue workload when a completed ABS test is invalidated.
Record Patient Antibody Screen	Antibody Screen (patient) Repeat Test	M	Accrue workload when a reflex or repeat ABS test is performed, completed, and saved.
Invalidate Patient Test Results*		M	Accrue workload when a reflex or repeat ABS test is invalidated.
Unit Antigen Typing / Patient Antigen Typing	Antigen phenotyping, Single Test phase (QC)	M	Accrue workload when Antiserum QC in Unit or Patient Antigen Typing includes the testing of both the positive and negative control cells, per specificity by lot number, when only one phase of reactivity is chosen for the test grid (IS or AHG). One workload event is collected per completed tab for regular or repeat antigen tests.
Unit Antigen Typing / Patient Antigen Typing	Antigen phenotyping, Multiple Test phases (QC)	M	Accrue workload when Antiserum QC in Unit or Patient Antigen Typing includes the testing of both the positive and negative control cells, per specificity by lot number, when only multiple phases of reactivity are chosen for the test grid, IS/RT, RT/37, or weak D. One workload event is collected per completed tab for regular or repeat antigen tests. When weak D is the selected test, QC may not be accrued for the rack selection. QC is accrued when positive and negative cells must be tested for the lot number.
Cancel Pending Order	Cancel Order	M	Accrue workload when an order on the pending order list is canceled. When multiple orders are canceled, each order accrues workload.
Cancel Active Order	Cancel Order	M	Accrue workload when an order on the pending task list is canceled. When multiple orders are canceled, each order accrues workload.
Select Units for Crossmatch	Crossmatch unit, electronic	P	This process is invoked when an individual unit is selected for patient assignment and the unit is electronically crossmatched. When multiple units are selected, each unit accrues workload.

Record Save Option	VBECs Process	Transaction Type [P (Patient), U (Unit), M (Miscellaneous)]	Explanation
Enter Crossmatch Results	Crossmatch unit, serologic immediate spin	P	Accrue workload when an individual unit crossmatch is selected to include only the IS phase, is completed, and is saved. When multiple units are selected, each unit accrues workload.
Invalidate Patient Test Results*		P	Accrue workload when a completed crossmatch test is invalidated. This applies to the workload originally saved with the serologic immediate spin test.
Record a Transfusion Reaction Workup	Crossmatch unit, serological Coombs	P	A crossmatch test for "pre" or "post" is enabled and a valid interpretation other than Not Tested is selected. A workload event is accrued separately for "Pre" and "Post" entries. When multiple units are selected, each unit accrues workload.
Enter Crossmatch Results		P	Accrue workload when an individual unit crossmatch is selected to include all phases or only the AHG phase, is completed, and is saved. When multiple units are selected, each unit accrues workload.
Invalidate Patient Test Results*		P	Accrue workload when a completed crossmatch test is invalidated. This applies to the workload originally saved with the test, serological Coombs.
Enter Crossmatch Results	Crossmatch, Repeat Test	M	Accrue workload when an individual unit crossmatch is selected to include all phases or IS or only the AHG phase, is completed, and is saved. When multiple units are selected, each unit accrues workload.
Invalidate Patient Test Results*		M	Accrue workload when an individual unit crossmatch is invalidated.
Enter Daily QC Results	Daily Rack Quality Control (QC)	M	Accrue workload when Daily QC rack completed for one individual rack includes all rows in configured QC. When multiple racks are tested, each completed and saved tab accrues a workload event.
Record Patient Direct Antiglobulin Test	DAT (QC)	M	Accrue workload when Reagent QC completed in Patient DAT testing includes the testing of both the positive and negative control cells, per specificity per lot number, when only one phase of reactivity is chosen for the test grid (IS or AHG). One workload event is collected per completed tab for regular or repeat antiglobulin tests (PS, IgG, Comp).
Record a Transfusion Reaction Workup	Direct Antiglobulin Test (DAT)	P	A DAT test for "pre" or "post" is enabled and a valid interpretation other than Not Tested is selected. A workload event is accrued separately for "Pre" and "Post" entries.
Record Patient Direct Antiglobulin Test		P	Accrue workload when a DAT is completed and saved. This count is used for all antiglobulin tests (PS, IgG, Comp) when ordered from CPRS or Reflex testing.
Invalidate Patient Test Results*		P	Accrue workload when a completed DAT, PS, IgG, or Comp is invalidated.
Record Patient Direct Antiglobulin Test	Direct Antiglobulin	M	Accrue workload when a reflex or repeat DAT test is performed, completed, and saved. This applies to all repeat antiglobulin tests (PS, IgG, Comp).

Record Save Option	VBECs Process	Transaction Type [P (Patient), U (Unit), M (Miscellaneous)]	Explanation
Invalidate Patient Test Results*	Test (DAT) Repeat test	M	Accrue workload when a completed Repeat DAT, PS, IgG, or Comp is invalidated.
Modify Units	Deglycerolize unit	U	Accrue workload when an individual blood unit is processed by the Deglycerolize modification type. Note: Workload is not accrued when a patient assignment is processed and VBECs releases all other patient assignments. Workload is not accrued when VBECs is required to quarantine the unit.
Remove Final Status*		U	An individual blood unit's status is invalidated when the original modification process was "Deglycerolize."
Discard or Quarantine Unit	Discard unit	U	Accrue workload when a unit is discarded for waste or credit. When a batch of units is selected, each unit accrues workload.
Remove Final Status*		U	Accrue workload when an individual blood unit's status is invalidated. When a batch of units is selected, each unit accrues workload.
Modify Units	Freeze unit	U	Accrue workload when an individual blood unit is processed by the Freeze modification type. Note: Workload is not accrued when a patient assignment is processed and VBECs releases all other patient assignments. Workload is not accrued when VBECs is required to quarantine the unit.
Remove Final Status*		U	An individual blood unit's status is invalidated when the original modification process was "Freeze."
Modify Units	Irradiate unit	U	Accrue workload when an individual blood unit is processed by the Irradiate modification type. When a batch of units is irradiated, each unit accrues workload. Note: Workload is not accrued when a patient assignment is processed and VBECs releases all other patient assignments. Workload is not accrued when VBECs is required to quarantine the unit.
Remove Final Status*		U	An individual blood unit's status is invalidated when the original modification process was "Irradiate."
Modify Units	Leukoreduce unit	U	Accrue workload when an individual blood unit is processed by the Leukoreduce modification type. Note: Workload is not accrued when a patient assignment is processed and VBECs releases all other patient assignments. Workload is not accrued when VBECs is required to quarantine the unit.
Remove Final Status*		U	An individual blood unit's status is invalidated when the original modification process was "Leukoreduce."
Split a Unit	Split unit	U	Accrue workload when a unit modification of Split and a single workload event is recorded regardless of the number of units created by the modification. Note: Workload is not accrued when a patient assignment is processed and VBECs releases all other patient assignments. Workload is not accrued when VBECs is required to quarantine the unit.

Record Save Option	VBECs Process	Transaction Type [P (Patient), U (Unit), M (Miscellaneous)]	Explanation
Remove Final Status*		U	A Split Unit has its unit status invalidated. A single workload event is recorded regardless of the number of units originally created by the modification.
Modify Units	Rejuvenate unit	U	Accrue workload when an individual blood unit is processed by the Rejuvenate modification type. Note: Workload is not accrued when a patient assignment is processed and VBECs releases all other patient assignments. Workload is not accrued when VBECs is required to quarantine the unit.
Remove Final Status*		U	An individual blood unit's status is invalidated when the original modification process was "Rejuvenate."
Modify Units	Thaw	U	Accrue workload when an individual blood unit is processed by the Thaw modification type. When a batch of units is thawed, each unit accrues workload. This applies to Thaw FFP and Thaw Cryo. Note: Workload is not accrued when a patient assignment is processed and VBECs releases all other patient assignments. Workload is not accrued when VBECs is required to quarantine the unit.
Remove Final Status*		U	An individual blood unit's status is invalidated when the original modification process was "Thaw." This modification type is applicable to Thaw FFP and Thaw Cryo.
Modify Units	Wash unit	U	Accrue workload when an individual blood unit is processed by the Wash modification type. Note: Workload is not accrued when a patient assignment is processed and VBECs releases all other patient assignments. Workload is not accrued when VBECs is required to quarantine the unit.
Remove Final Status*		U	An individual blood unit's status is invalidated when the original modification process was "Wash."
Modify Units	Volume Reduce	U	Accrue workload when an individual blood unit is processed by the Volume Reduce modification type. Note: Workload is not accrued when a patient assignment is processed and VBECs releases all other patient assignments. Workload is not accrued when VBECs is required to quarantine the unit.
Remove Final Status*		U	An individual blood unit's status is invalidated when the original modification process was Volume Reduce.
Issue Blood Components	Issue unit	P	Accrue workload when a unit is issued to a patient. When a batch of units is processed, each unit invokes one workload process.
Justify Patient ABO/Rh Change	Justification	M	Workload is accrued when a patient's ABO or Rh typing is justified. One workload event is accrued per patient justification.
Login Equipment	Login Equipment	M	Accrue workload when a lot number of any type of equipment is logged into the system. When multiple lot numbers are processed in a batch, each lot number's workload is counted.

<b>Record Save Option</b>	<b>VBECs Process</b>	<b>Transaction Type [P (Patient), U (Unit), M (Miscellaneous)]</b>	<b>Explanation</b>
Login Reagent	Login Reagent	M	Accrue workload when a lot number of any type of reagent is logged into the system. When multiple lot numbers are processed in a batch, each lot number's workload is counted.
Login Supply	Login Supply	M	Accrue workload when a lot number of any type of supply is logged into the system. When multiple lot numbers are processed in a batch, each lot number's workload is counted.
Maintain Specimen	Maintain Specimen	M	Accrue workload when a specimen is maintained during order acceptance and is required for acceptance of the order. Note: This is collected in addition to the accept order workload accrued by accepting an order. Marking a specimen unacceptable does not create a negative workload event.
Patient antigen phenotype	Patient antigen phenotype (multiple phases)	P	Accrue workload when a patient antigen phenotype test with IS/RT or IS/37 phases is completed and saved. One workload event is collected per completed tab for repeat or regular antigen tests.
Invalidate Patient Test Results*		P	Accrue workload when a patient antigen phenotype test as defined by the antiserum specificity tested with any phases is invalidated.
Patient antigen phenotype	Patient antigen phenotype (single phase)	P	Accrue workload when a patient antigen phenotype test with AHG or IS phase is completed and saved. One workload event is collected per completed tab for repeat or regular antigen tests.
Invalidate Patient Test Results*		P	Accrue workload when a patient antigen phenotype test as defined by the antiserum specificity tested with a single phase is invalidated.
Pool Units	Pool unit	U	Accrue workload when a pooled unit is created and a single workload event is recorded regardless of the number of units included in the pooled unit. This applies to the Pool modification type. Add/Remove unit from a pool does not accrue any workload. Note: Workload is not accrued when a patient assignment is processed and VBECs releases all other patient assignments. Workload is not accrued when VBECs is required to quarantine the unit.
Edit Unit Information*		U	Accrue workload when a unit is inactivated if the pooled unit was created in VBECs.
Remove Final Status		N/A	No effect on workload accrual when a unit is removed from a modified status that was included in a pool.
Discard or Quarantine Unit	Quarantine unit	U	Accrue workload when a unit is marked for quarantine. When a batch of units is selected, each unit accrues workload.
Free Directed Unit For Crossover	Release directed unit	U	Accrue workload when an individual blood unit with the restriction type of "directed" is released for use as an allogeneic unit.

Record Save Option	VBECs Process	Transaction Type [P (Patient), U (Unit), M (Miscellaneous)]	Explanation
Release Unit from Patient Assignment	Release unit from patient back to inventory	U	Accrue workload when an individual unit is released from patient assignment. When multiple units are selected, each unit accrues workload.
Discard or Quarantine Unit	Release unit from Quarantine	U	Accrue workload when a unit is released from quarantine. When a batch of units is selected, each unit accrues workload.
Return Issued Unit	Return Issued unit	U	Accrue workload when a unit is returned from issue status.
Modify Units	Thaw/pool Cryo	U	Accrue workload when an individual unit has a modification of Thaw/Pool Cryo. A single workload event is recorded regardless of the number of units included in the pooled unit. Note: Workload is not accrued when a patient assignment is processed and VBECs releases all other patient assignments. Workload is not accrued when VBECs is required to quarantine the unit.
Edit Unit Information*		U	Accrue workload when a unit is inactivated (unit record inactivated) when the pooled unit was created in VBECs.
Remove Final Status		N/A	There is no effect on workload accrual when a unit is removed from a modified status that was included in a Thaw/pool Cryo pool.
Enter Post-Transfusion Details	Transfuse Unit	U	Accrue workload when an individual blood unit's status is updated to "transfused."
Remove Final Status*		U	An individual blood unit's status is invalidated when the unit was in a status of "transfused."
Record a Transfusion Reaction Workup	Transfusion Reaction Investigation	P	Accrue workload when a transfusion reaction investigation is saved. This does not include workload accrued by the optional TRW serologic testing.
Invalidate Patient Test Results*		P	Accrue workload when a transfusion reaction investigation previously saved is invalidated.
Unit Antigen Typing	Unit Antigen phenotyping, Multiple Test phases	U	Accrue workload when a unit antigen phenotype test with IS/RT or IS/37 phases is selected and completed for an individual blood unit. There is no special handling for workload collection for additional repeat antigen typing tests on a unit.
Edit Unit Information*		U	Accrue workload when a unit antigen phenotype test with Multiple Test phases is invalidated for an individual blood unit.
Unit Antigen Typing	Unit Antigen phenotyping, Single Test phase	U	A unit antigen phenotype test with AHG or IS phase is selected and completed for an individual blood unit. There is no special handling for workload collection for additional repeat antigen typing tests on a unit.
Edit Unit Information*		U	Accrue workload when a unit antigen phenotype test with Single Test phase is invalidated for an individual blood unit.

Record Save Option	VBECs Process	Transaction Type [P (Patient), U (Unit), M (Miscellaneous)]	Explanation
Incoming Shipment	Unit login	U	An individual unit record is activated as "saved" to an incoming shipment invoice. When multiple units are entered, each unit added to the database accrues workload.
Edit Unit Information*		U	Accrue workload when a unit is inactivated and logged in through incoming shipment or is a pooled unit created in VBECs. When the unit was created by split modification, no workload is invalidated in this option.
Outgoing Shipment	Unit logout	U	An individual unit's status is updated to "transferred" on a confirmed outgoing shipment invoice. When multiple units are selected, each unit accrues workload. Accrue workload on confirmation of the invoice, not the addition of a unit to a temporary outgoing shipment invoice: an invoice may be confirmed only once.
Remove Final Status*		U	An individual unit status is invalidated when the unit had a previous unit status of "transferred."
Update Equipment Record	Update Equipment Record	M	Accrue workload when a lot number of any type of equipment is updated in the system.
Update Reagent Inventory	Update Reagent Inventory	M	Accrue workload when a lot number of any type of reagent is updated in the system. When multiple lot numbers are processed in a batch, each lot number's workload is counted.
Update Supply Inventory	Update Supply Inventory	M	Accrue workload when a lot number of any type of supply is updated in the system. When multiple lot numbers are processed in a batch, each lot number's workload is counted.

\*Accumulates negative workload when it is associated with inactivation of a unit or removal of a final status.



# Index

## A

Accept Orders: Accept an Order.....	263
Accessing the System .....	37
Administrative Data Report .....	377
Appendices .....	4
Application Architecture.....	6
Assumptions .....	174
Audit Trail .....	380
Automated Entry of Post-Transfusion Details from BCE COTS Application .....	356
Automated Testing Interface.....	363

## B

Background and Basic Knowledge .....	3
Barcode Scanner Configuration and Troubleshooting .....	549
Blood Availability .....	382

## C

C:T Ratio Report.....	386
check character .....	136
Collecting Workload.....	67
Common Error Corrections.....	545
Commonly Used System Rules .....	15
Component Processing .....	135
Configure Division .....	78
Configuring Site Parameters .....	69
Cost Accounting Report.....	384
Creating and Viewing Reports.....	21
Crossmatch Interpretations .....	319
Customer Support .....	11

## D

Database Table Information.....	443
Date and Time .....	15
Discard or Quarantine Individual Units .....	208
Discard or Quarantine Units by Invoice.....	211
Display Order Alerts.....	271
Display Patient Merge Alerts.....	275
Display Patient Update Alerts.....	273
Division Transfusion Report.....	388
Division Workload Report.....	390
Document ABO Incompatible Transfusions .....	357
Downtime Forms and Instructions.....	433
Drop-Down Menus .....	58

## E

Enter Daily QC Results.....	108
-----------------------------	-----

Equipment.....	130
Equipment: Log In Equipment.....	130
Equipment: Maintain Equipment.....	133
Exception Report .....	392

## F

Figures	
Barcode Scanner Configuration.....	549
Blood Transfusion Record Form .....	253
Caution Tag.....	252
Order Status Flowchart.....	467
Unit Status Flowchart .....	469
VBECs Work Flow.....	489
Figures and Tables.....	3
Finalize Transfusion Reaction Workup Report.....	339
Finalize/Print TRW.....	338
Forms	
Patient Testing Form .....	435
Unit ABO/Rh Confirmation Form.....	438
Unit Issue and Inspection Log.....	439
Unit Modification Form.....	440
Freeware Disclaimer.....	2
Frequently Asked Questions .....	543

## G

Glossary .....	423
----------------	-----

## H

Hardware and Infrastructure Architecture.....	7
How This User Guide Is Organized.....	3

## I

Icons and Buttons .....	58
Inactivate a Unit.....	202
Inappropriate Transfusion Request Report .....	394
Introduction .....	1
Issue Blood Components .....	225
Issue Blood Components (Emergency).....	230
Issue Blood Components (Routine) .....	226
Issued/Returned Unit Report.....	395

## J

Justify ABO/Rh Change .....	359
-----------------------------	-----

## K

Keyboard Shortcuts for Windows.....	57
Known Defects and Anomalies.....	513

## L

Limitations and Restrictions .....	493
Local Facilities.....	88
Local Machine Screen Resolution .....	45
Locking and Time-Outs .....	15
Log in Reagents.....	117
Log in Supplies.....	125
Login Message.....	86

## M

Main Status Bar .....	66
Main Toolbar .....	59
Maintain Minimum Levels .....	123
Maintain Patient Records.....	343
Maintain Specimen .....	279
Maintain Unit Records.....	197
Medication Profile .....	61
Modify Component.....	174
MSBOS.....	102

## O

Online Help F1 .....	49
Options .....	3
Order Alerts .....	84
Order History Report .....	397
Order Reflex Tests.....	289
Orientation.....	3
Other VBECS Functions.....	67
Outgoing Shipment.....	146

## P

Patient Blood Availability.....	60
Patient History Report .....	400
Patient Information Toolbar.....	59
Patient Merge.....	295
Patient Search Screen.....	293
Patient Services.....	293
Patient Testing Form.....	435
Patient Testing Worklist and Testing Worklist Reports.....	405
Patient Testing: Cancel an Active Order.....	283
Patient Testing: Enter Antibody Identification Results.....	327
Patient Testing: General Instructions.....	301
Patient Testing: Pending Task List .....	285
Patient Testing: Record a Crossmatch .....	315
Patient Testing: Record a Direct Antiglobulin Test .....	312
Patient Testing: Record a Patient ABO/Rh.....	307
Patient Testing: Record a Patient Antibody Screen .....	310
Patient Testing: Record a Patient Antigen Typing.....	321
Patient Testing: Record a Transfusion Reaction Workup .....	330
Post-Transfusion Information .....	351
Print Backup Caution Tags and Blood Transfusion Record Forms .....	245
Problems? .....	11

<i>Processing an Emergency Transfusion Order</i> .....	276
Processing Orders .....	263
Product Modifications.....	82
Prolonged Transfusion Time Report.....	402

## Q

Quick K Calculator .....	136
--------------------------	-----

## R

Reagents .....	116
Reagents and Supplies .....	108
Recommended Report Usage.....	547
References .....	419
Related Manuals and Materials.....	2
Remote Desktop Connections Screen Size .....	47
Restore VistA Connection .....	43

## S

Screen Display .....	49
Screen Settings .....	45
Search Tab .....	33
Searching the Database.....	16
Security.....	4
<b>Service Desk</b> .....	11
Setting Transfusion Parameters .....	98
Shipments .....	135
Software Basics .....	13
Sound.....	50
Special Instructions & Transfusion Requirements: Enter and Remove Special Instructions .....	348
Supplemental Information .....	4
Supplies .....	125
System Responses to Active Transfusion Requirements in Select Unit and Issue Blood Component.....	551
System Validation	
Antibodies .....	529
Blood Products .....	523
Canned Comments.....	528
Component Classes .....	525
Configure Daily QC: Full Service .....	520
Configure Division.....	517
Configure Testing: Full Service.....	517
Edit Invoice Text.....	528
Installation Qualification .....	515
Introduction .....	515
Local Facilities .....	524
Log In Reagents: Full Service .....	522
Maintain Minimum Levels: Full Service.....	521
MSBOS .....	528
Operational Qualification .....	515
Order Alerts.....	519
Performance Qualification.....	516
Prerequisites .....	516
Product Modifications: Full Service.....	518
Record System Settings.....	517

Testing.....	530
Transfusion Complications.....	526
Transfusion Effectiveness.....	527
Update User Roles.....	530
Validation Plan.....	515
VBECS Forms and Reports.....	531
Workload Codes.....	528
System Validation.....	515

## T

### Tables

Alerts That May Occur in Issue Blood Components (Emergency).....	235
Alerts That May Occur in Issue Blood Components (Routine).....	233
Alerts That May Occur in Patient Testing	
General Instructions.....	305
Alerts That May Occur in Print Unit Caution Tag & Transfusion Record Form.....	251
Alerts That May Occur in Select Units.....	221
Allowable Product Modifications by Original Product Type.....	461
Antibody and Antigen Table, Database Conversion.....	446
Antibody and Antigen Table, Irregular Antibodies.....	443
Antibody Screen Test Interpretation.....	447
BCE COTS Message Updates.....	472
Blood Products Worksheet.....	523
Canned Comment Category Types and Text.....	448
Component Class Requirements Worksheet.....	525
CPRS Orderable Blood Components (Component Classes) Mapped to ICCBBA Component Classes.....	453
Crossmatch Interpretations for Print Unit Caution Tag & Transfusion Record Form.....	319
Details in Audit Trail Report.....	453
Details in Exception Report.....	458
Local Facilities Worksheet.....	524
Modification Types and Costs.....	518
Rules for Electronic and Serologic Crossmatch.....	468
Sample Blood Products Worksheet.....	523
Sample Local Facilities Worksheet.....	524
Transfusion Complications Worksheet.....	526
Transfusion Effectiveness Worksheet.....	527
Transfusion Reaction Types.....	342
User Roles and Functions.....	5
Valid Interpretations.....	162
Valid Observed Test Results Entries.....	19
VBECS Maximum Surgical Blood Order Schedule (MSBOS).....	471
Terms.....	3
Test Units.....	159
Testing.....	17
Throughout VBECS.....	66
Tool Tips.....	58
Toolbars.....	58
Transfusion Complications Report.....	408
Transfusion Effectiveness.....	100
Transfusion Effectiveness Report.....	410
Transfusion Reaction Count Report.....	412
Transfusion Requirements Report.....	414

## U

Unit ABO/Rh Confirmation Form .....	438
Unit History Report .....	416
Unit Issue and Inspection Log .....	439
Unit Modification Form.....	440
Unit Search Screen .....	157
Update Reagent Inventory .....	119
Update Supply Inventory .....	127
Updated Patients and Deceased Patients.....	299
Using Online Help .....	30
Using the Software .....	13
Using VBECS.....	30

## V

VBECS User Report.....	71
VBECS Work Flow .....	489
View/Print a Finalized Transfusion Reaction Workup Report.....	341
View/Print Reagent Inventory .....	121
View/Print Supply Inventory .....	129
Viewing the VBECS Version Number .....	33
VistA Records in VBECS.....	36
Volume .....	56

## W

Work Process Flowcharts .....	533
Working with Data .....	15
Workload Codes .....	104
Workload Process Mapping to Application Option Table .....	561
Workload Process to Use Case Mapping Table .....	476

## Revision History

Date	Revision	Description	Author
3/28/19	1.0	Modified VistA Blood Establishment Computer Software (VBECS) 2.3.0 User Guide, Version 3.0 to create the VistA Blood Establishment Computer Software (VBECS) 2.3.1 User Guide, Version 1.0.	BBM Team
6/6/19	2.0	(VBECS 2.3.1.1 Rev B) Searching the Database section: Revised first bullet sub-bullets. Added second bullet and sub-bullets. Added Note and bullets. Edit Unit Information section, Step 3: Removed last three notes.	BBM Team

This is the last page of *Vista Blood Establishment Computer Software (VBECS) 2.3.1 User Guide*.